

Spare Parts Catalogue

Version: 2010/08/30b EN



küschall[®] Compact

General Information

For any order of spare parts the following information is required:

- Type of wheelchair
- Serial Number
- Part-Number
- Part-Description

Abbreviations

SB Seat width

ST Seat depth

UL Knee-to-heel height



Left side (in direction of movement)



Right side (in direction of movement)



An assembly consisting of a right and a left part, referred to as a 'pair'.

Exploded view drawings and code numbers are not contractual and subject to modification, in relation to our continuous product improvement policy to meet end users needs.

TABLE OF CONTENTS

INFORMATION	5
Frame colours	6
CPS colours	7
CHASSIS	8
Front frames standard	9
Front frames abducted	17
Cross brace single, SB28-42	25
Cross brace double, SB44-50	45
Rear frames	56
Rear wheel fixation	59
SEAT	60
Seat cover standard	61
Seat cover with 1 Velcro band (soft band)	62
Seat cover with 2 Velcro bands (rough band)	65
Seat cushion standard	68
Seat cushion Flo-tech	73
BACKREST	74
Push handles for backrest standard	75
Push handles for backrest angle adjustable	76
Push handles for backrest foldable and angle adjustable	77
Push handles height adjustable	78
Push handles height adjustable for backrest angle adjustable	79
Push handles height adjustable for backrest foldable and angle adjustable	80
Push handles, inclined	82
Backrest stabilization bar	83
Back cover, standard	85
Bands to adjustable backrest	88
Cover to adjustable backrest	89
Sport back	91
LEGRESTS	92
Legrest hanger	93
FOOTREST	96
Footrest 1-piece	97
Footrest 2-piece	99
Footrest 2-pieces for swing away 70° on front frames standard	100
Footrest 2-pieces for swing away 80° on front frames standard	102
Footrest 2-pieces for swing away 90° on front frames standard	104
Footrest 2-pieces for swing away 70° on front frames abducted	106
Footrest 2-pieces for swing away 80° on front frames abducted	108
Footrest 2-pieces for swing away 90° on front frames abducted	110
Footrest 2-piece, angle adjustable	112
Footrest carbon 2-pieces	114
Footrest carbon 2-pieces for swing away 70° on front frames standard	115
Footrest carbon 2-pieces for swing away 80° on front frames standard	117
Footrest carbon 2-pieces for swing away 90° on front frames standard	119
Footrest carbon 2-pieces for swing away 70° on front frames abducted	121
Footrest carbon 2-pieces for swing away 80° on front frames abducted	123
Footrest carbon 2-pieces for swing away 90° on front frames abducted	125
SIDERESTS	127
Clothes-guards	128
Mudguards fixe	129
Mudguards removable	130

TABLE OF CONTENTS


Siderest height adjustable	131
Siderest in colours, flip-back and height adjustable	132
Siderest with desk function	136
Armrest Hemi	137
Tubular armrest	138
FRONT WHEELS	139
Front wheel supporter	140
Front wheels	142
REAR WHEELS	143
22" Kuschall rear wheels complete	144
22" Kuschall rear wheel accessories	146
24" Kuschall rear wheels complete	147
24" Kuschall rear wheel accessories	149
26" Kuschall rear wheels complete	151
26" Kuschall rear wheel accessories	152
High Performance wheel	153
Mountain bike wheel	155
Spider wheel	156
Spinergy wheel	157
Drumbrake wheel	158
BRAKES	160
Brakes	161
Drum brake	162
OPTIONS & ACCESSORIES	163
Tipper aid	164
Antitipper	165
Cane holder	166
Transit wheels	167
Therapy table	168
Chair accessories	169
Winter accessories	170

Pos.	Part number	Description	Valid from → until	Qty
1	-	Frame colours		1
2	-	CPS colours		1

Frame colours




Pos.	Part number	Description	Valid from → until	Qty
-041 1	-	Color code for black matt frames	20100701 →	0
-041 9	-	Color code for silver grey frames	20091001 →	0
-044 2	-	Color code for night shimmer frames	20091001 →	0
-044 5	-	Color code for ivory white frames	20100701 →	0
-044 6	-	Color code for purple fire frames	20091001 →	0
-044 7	-	Color code for emerald green frames	20091001 →	0
-044 8	-	Color code for midnight blue frames	20091001 →	0
-044 9	-	Color code for umber glaze frames	20091001 →	0
-045 0	-	Color code for smokey grey frames	20091001 →	0
-352 4	-	Color code for royal red frames	20091001 →	0
-596 9	-	Color code for royal blue frames	20091001 →	0
-900 5	-	Color code for glossy black frames	20091001 →	0
-911 9	-	Color code for black bronze frames	20091001 → 20100630	0

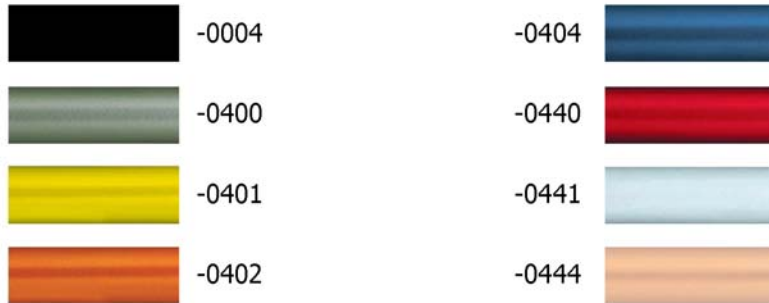


Pos.	Part number	Description	Valid from → until	Qty
-000 4	-	Color code for black anodized CPS		0
-040 0	-	Color code for techno-anthrazit CPS		0
-040 1	-	Color code for lemon CPS		0
-040 2	-	Color code for apricot CPS		0
-040 4	-	Color code for b-blue CPS		0
-044 0	-	Color code for comet red CPS		0
-044 1	-	Color code for arctic blue CPS		0
-044 4	-	Color code for copper glaze CPS		0

CHASSIS

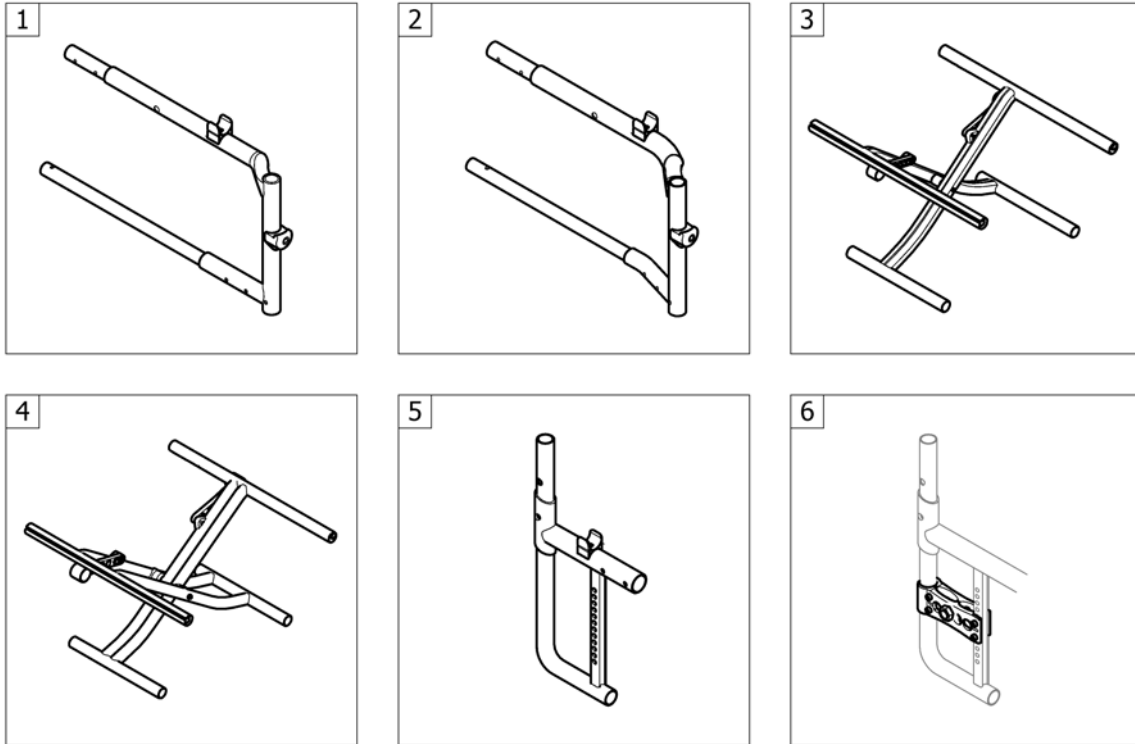

Pos.	Part number	Description	Valid from → until	Qty
1	-	Front frames standard		1
2	-	Front frames abducted		1
3	-	Cross brace single, SB28-42		1
4	-	Cross brace double, SB44-50		1
5	-	Rear frames		1
6	-	Rear wheel fixation		1

Front frames standard

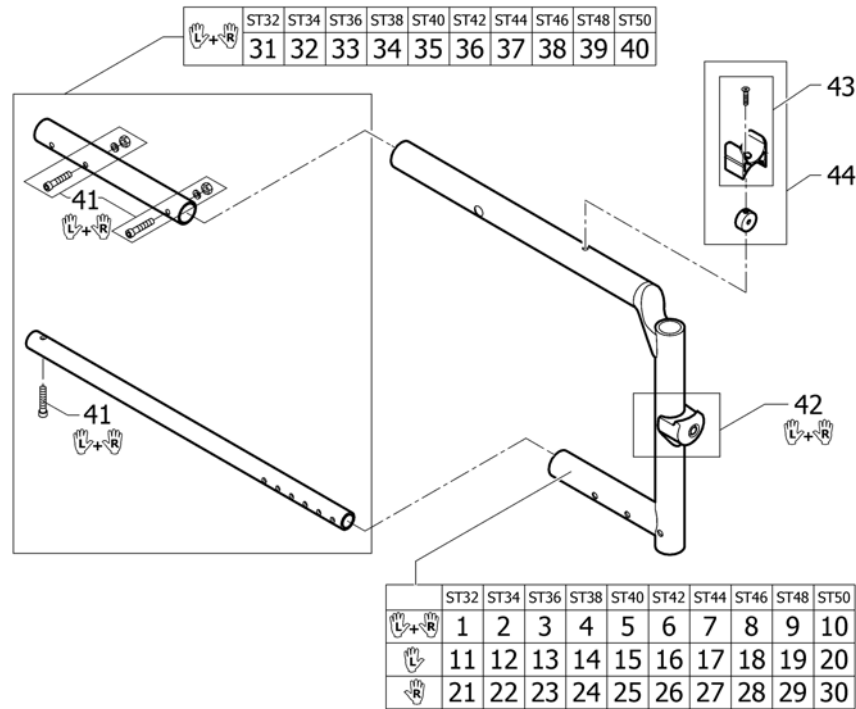

Pos.	Part number	Description	Valid from → until	Qty
1	1539902-0411	Front frame ST32, black, matt, pair	20100701 →	1
1	1539902-0419	Front frame ST32, silver grey, pair		1
1	1539902-0442	Front frame ST32, night shimmer, pair		1
1	1539902-0445	Front frame ST32, ivory white, pair	20100701 →	1
1	1539902-0446	Front frame ST32, purple fire, pair		1
1	1539902-0447	Front frame ST32, emerald green, pair		1
1	1539902-0448	Front frame ST32, midnight blue, pair		1
1	1539902-0449	Front frame ST32, umber glaze, pair		1
1	1539902-0450	Front frame ST32, smokey grey, pair		1
1	1539902-3524	Front frame ST32, royal red, pair		1
1	1539902-5969	Front frame ST32, royal blue, pair		1
1	1539902-9005	Front frame ST32, black, glossy, pair		1
1	1539902-9119	Front frame ST32, black, bronze, pair	20091001 → 20100630	1
2	1539903-0411	Front frame ST34, black, matt, pair	20100701 →	1
2	1539903-0419	Front frame ST34, silver grey, pair		1
2	1539903-0442	Front frame ST34, night shimmer, pair		1
2	1539903-0445	Front frame ST34, ivory white, pair	20100701 →	1
2	1539903-0446	Front frame ST34, purple fire, pair		1
2	1539903-0447	Front frame ST34, emerald green, pair		1
2	1539903-0448	Front frame ST34, midnight blue, pair		1
2	1539903-0449	Front frame ST34, umber glaze, pair		1
2	1539903-0450	Front frame ST34, smokey grey, pair		1
2	1539903-3524	Front frame ST34, royal red, pair		1
2	1539903-5969	Front frame ST34, royal blue, pair		1
2	1539903-9005	Front frame ST34, black, glossy, pair		1
2	1539903-9119	Front frame ST34, black, bronze, pair	20091001 → 20100630	1
3	1539904-0411	Front frame ST36, black, matt, pair	20100701 →	1
3	1539904-0419	Front frame ST36, silver grey, pair		1
3	1539904-0442	Front frame ST36, night shimmer, pair		1
3	1539904-0445	Front frame ST36, ivory white, pair	20100701 →	1
3	1539904-0446	Front frame ST36, purple fire, pair		1

Front frames standard

Pos.	Part number	Description	Valid from → until	Qty
3	1539904-0447	Front frame ST36, emerald green, pair		1
3	1539904-0448	Front frame ST36, midnight blue, pair		1
3	1539904-0449	Front frame ST36, umber glaze, pair		1
3	1539904-0450	Front frame ST36, smokey grey, pair		1
3	1539904-3524	Front frame ST36, royal red, pair		1
3	1539904-5969	Front frame ST36, royal blue, pair		1
3	1539904-9005	Front frame ST36, black, glossy, pair		1
3	1539904-9119	Front frame ST36, black, bronze, pair	20091001 → 20100630	1
4	1539905-0411	Front frame ST38, black, matt, pair	20100701 →	1
4	1539905-0419	Front frame ST38, silver grey, pair		1
4	1539905-0442	Front frame ST38, night shimmer, pair		1
4	1539905-0445	Front frame ST38, ivory white, pair	20100701 →	1
4	1539905-0446	Front frame ST38, purple fire, pair		1
4	1539905-0447	Front frame ST38, emerald green, pair		1
4	1539905-0448	Front frame ST38, midnight blue, pair		1
4	1539905-0449	Front frame ST38, umber glaze, pair		1
4	1539905-0450	Front frame ST38, smokey grey, pair		1
4	1539905-3524	Front frame ST38, royal red, pair		1
4	1539905-5969	Front frame ST38, royal blue, pair		1
4	1539905-9005	Front frame ST38, black, glossy, pair		1
4	1539905-9119	Front frame ST38, black, bronze, pair	20091001 → 20100630	1
5	1539906-0411	Front frame ST40, black, matt, pair	20100701 →	1
5	1539906-0419	Front frame ST40, silver grey, pair		1
5	1539906-0442	Front frame ST40, night shimmer, pair		1
5	1539906-0445	Front frame ST40, ivory white, pair	20100701 →	1
5	1539906-0446	Front frame ST40, purple fire, pair		1
5	1539906-0447	Front frame ST40, emerald green, pair		1
5	1539906-0448	Front frame ST40, midnight blue, pair		1
5	1539906-0449	Front frame ST40, umber glaze, pair		1
5	1539906-0450	Front frame ST40, smokey grey, pair		1
5	1539906-3524	Front frame ST40, royal red, pair		1
5	1539906-5969	Front frame ST40, royal blue, pair		1
5	1539906-9005	Front frame ST40, black, glossy, pair		1
5	1539906-9119	Front frame ST40, black, bronze, pair	20091001 → 20100630	1
6	1539907-0411	Front frame ST42, black, matt, pair	20100701 →	1
6	1539907-0419	Front frame ST42, silver grey, pair		1
6	1539907-0442	Front frame ST42, night shimmer, pair		1
6	1539907-0445	Front frame ST42, ivory white, pair	20100701 →	1
6	1539907-0446	Front frame ST42, purple fire, pair		1
6	1539907-0447	Front frame ST42, emerald green, pair		1
6	1539907-0448	Front frame ST42, midnight blue, pair		1
6	1539907-0449	Front frame ST42, umber glaze, pair		1
6	1539907-0450	Front frame ST42, smokey grey, pair		1
6	1539907-3524	Front frame ST42, royal red, pair		1
6	1539907-5969	Front frame ST42, royal blue, pair		1
6	1539907-9005	Front frame ST42, black, glossy, pair		1
6	1539907-9119	Front frame ST42, black, bronze, pair	20091001 → 20100630	1
7	1539908-0411	Front frame ST44, black, matt, pair	20100701 →	1
7	1539908-0419	Front frame ST44, silver grey, pair		1
7	1539908-0442	Front frame ST44, night shimmer, pair		1
7	1539908-0445	Front frame ST44, ivory white, pair	20100701 →	1
7	1539908-0446	Front frame ST44, purple fire, pair		1
7	1539908-0447	Front frame ST44, emerald green, pair		1
7	1539908-0448	Front frame ST44, midnight blue, pair		1
7	1539908-0449	Front frame ST44, umber glaze, pair		1

Front frames standard

Pos.	Part number	Description	Valid from → until	Qty
7	1539908-0450	Front frame ST44, smokey grey, pair		1
7	1539908-3524	Front frame ST44, royal red, pair		1
7	1539908-5969	Front frame ST44, royal blue, pair		1
7	1539908-9005	Front frame ST44, black, glossy, pair		1
7	1539908-9119	Front frame ST44, black, bronze, pair	20091001 → 20100630	1
8	1539909-0411	Front frame ST46, black, matt, pair	20100701 →	1
8	1539909-0419	Front frame ST46, silver grey, pair		1
8	1539909-0442	Front frame ST46, night shimmer, pair		1
8	1539909-0445	Front frame ST46, ivory white, pair	20100701 →	1
8	1539909-0446	Front frame ST46, purple fire, pair		1
8	1539909-0447	Front frame ST46, emerald green, pair		1
8	1539909-0448	Front frame ST46, midnight blue, pair		1
8	1539909-0449	Front frame ST46, umber glaze, pair		1
8	1539909-0450	Front frame ST46, smokey grey, pair		1
8	1539909-3524	Front frame ST46, royal red, pair		1
8	1539909-5969	Front frame ST46, royal blue, pair		1
8	1539909-9005	Front frame ST46, black, glossy, pair		1
8	1539909-9119	Front frame ST46, black, bronze, pair	20091001 → 20100630	1
9	1539910-0411	Front frame ST48, black, matt, pair	20100701 →	1
9	1539910-0419	Front frame ST48, silver grey, pair		1
9	1539910-0442	Front frame ST48, night shimmer, pair		1
9	1539910-0445	Front frame ST48, ivory white, pair	20100701 →	1
9	1539910-0446	Front frame ST48, purple fire, pair		1
9	1539910-0447	Front frame ST48, emerald green, pair		1
9	1539910-0448	Front frame ST48, midnight blue, pair		1
9	1539910-0449	Front frame ST48, umber glaze, pair		1
9	1539910-0450	Front frame ST48, smokey grey, pair		1
9	1539910-3524	Front frame ST48, royal red, pair		1
9	1539910-5969	Front frame ST48, royal blue, pair		1
9	1539910-9005	Front frame ST48, black, glossy, pair		1
9	1539910-9119	Front frame ST48, black, bronze, pair	20091001 → 20100630	1
10	1539911-0411	Front frame ST50, black, matt, pair	20100701 →	1
10	1539911-0419	Front frame ST50, silver grey, pair		1
10	1539911-0442	Front frame ST50, night shimmer, pair		1
10	1539911-0445	Front frame ST50, ivory white, pair	20100701 →	1
10	1539911-0446	Front frame ST50, purple fire, pair		1
10	1539911-0447	Front frame ST50, emerald green, pair		1
10	1539911-0448	Front frame ST50, midnight blue, pair		1
10	1539911-0449	Front frame ST50, umber glaze, pair		1
10	1539911-0450	Front frame ST50, smokey grey, pair		1
10	1539911-3524	Front frame ST50, royal red, pair		1
10	1539911-5969	Front frame ST50, royal blue, pair		1
10	1539911-9005	Front frame ST50, black, glossy, pair		1
10	1539911-9119	Front frame ST50, black, bronze, pair	20091001 → 20100630	1
11	1539912-0411	Front frame left ST32, black, matt	20100701 →	1
11	1539912-0419	Front frame left ST32, silver grey		1
11	1539912-0442	Front frame left ST32, night shimmer		1
11	1539912-0445	Front frame left ST32, ivory white	20100701 →	1
11	1539912-0446	Front frame left ST32, purple fire		1
11	1539912-0447	Front frame left ST32, emerald green		1
11	1539912-0448	Front frame left ST32, midnight blue		1
11	1539912-0449	Front frame left ST32, umber glaze		1
11	1539912-0450	Front frame left ST32, smokey grey		1
11	1539912-3524	Front frame left ST32, royal red		1
11	1539912-5969	Front frame left ST32, royal blue		1

Front frames standard

Pos.	Part number	Description	Valid from → until	Qty
11	1539912-9005	Front frame left ST32, black, glossy		1
11	1539912-9119	Front frame left ST32, black, bronze	20091001 → 20100630	1
12	1539913-0411	Front frame left ST34, black, matt	20100701 →	1
12	1539913-0419	Front frame left ST34, silver grey		1
12	1539913-0442	Front frame left ST34, night shimmer		1
12	1539913-0445	Front frame left ST34, ivory white	20100701 →	1
12	1539913-0446	Front frame left ST34, purple fire		1
12	1539913-0447	Front frame left ST34, emerald green		1
12	1539913-0448	Front frame left ST34, midnight blue		1
12	1539913-0449	Front frame left ST34, umber glaze		1
12	1539913-0450	Front frame left ST34, smokey grey		1
12	1539913-3524	Front frame left ST34, royal red		1
12	1539913-5969	Front frame left ST34, royal blue		1
12	1539913-9005	Front frame left ST34, black, glossy		1
12	1539913-9119	Front frame left ST34, black, bronze	20091001 → 20100630	1
13	1539914-0411	Front frame left ST36, black, matt	20100701 →	1
13	1539914-0419	Front frame left ST36, silver grey		1
13	1539914-0442	Front frame left ST36, night shimmer		1
13	1539914-0445	Front frame left ST36, ivory white	20100701 →	1
13	1539914-0446	Front frame left ST36, purple fire		1
13	1539914-0447	Front frame left ST36, emerald green		1
13	1539914-0448	Front frame left ST36, midnight blue		1
13	1539914-0449	Front frame left ST36, umber glaze		1
13	1539914-0450	Front frame left ST36, smokey grey		1
13	1539914-3524	Front frame left ST36, royal red		1
13	1539914-5969	Front frame left ST36, royal blue		1
13	1539914-9005	Front frame left ST36, black, glossy		1
13	1539914-9119	Front frame left ST36, black, bronze	20091001 → 20100630	1
14	1539915-0411	Front frame left ST38, black, matt	20100701 →	1
14	1539915-0419	Front frame left ST38, silver grey		1
14	1539915-0442	Front frame left ST38, night shimmer		1
14	1539915-0445	Front frame left ST38, ivory white	20100701 →	1
14	1539915-0446	Front frame left ST38, purple fire		1
14	1539915-0447	Front frame left ST38, emerald green		1
14	1539915-0448	Front frame left ST38, midnight blue		1
14	1539915-0449	Front frame left ST38, umber glaze		1
14	1539915-0450	Front frame left ST38, smokey grey		1
14	1539915-3524	Front frame left ST38, royal red		1
14	1539915-5969	Front frame left ST38, royal blue		1
14	1539915-9005	Front frame left ST38, black, glossy		1
14	1539915-9119	Front frame left ST38, black, bronze	20091001 → 20100630	1
15	1539916-0411	Front frame left ST40, black, matt	20100701 →	1
15	1539916-0419	Front frame left ST40, silver grey		1
15	1539916-0442	Front frame left ST40, night shimmer		1
15	1539916-0445	Front frame left ST40, ivory white	20100701 →	1
15	1539916-0446	Front frame left ST40, purple fire		1
15	1539916-0447	Front frame left ST40, emerald green		1
15	1539916-0448	Front frame left ST40, midnight blue		1
15	1539916-0449	Front frame left ST40, umber glaze		1
15	1539916-0450	Front frame left ST40, smokey grey		1
15	1539916-3524	Front frame left ST40, royal red		1
15	1539916-5969	Front frame left ST40, royal blue		1
15	1539916-9005	Front frame left ST40, black, glossy		1
15	1539916-9119	Front frame left ST40, black, bronze	20091001 → 20100630	1
16	1539917-0411	Front frame left ST42, black, matt	20100701 →	1

Front frames standard

Pos.	Part number	Description	Valid from → until	Qty
16	1539917-0419	Front frame left ST42, silver grey		1
16	1539917-0442	Front frame left ST42, night shimmer		1
16	1539917-0445	Front frame left ST42, ivory white	20100701 →	1
16	1539917-0446	Front frame left ST42, purple fire		1
16	1539917-0447	Front frame left ST42, emerald green		1
16	1539917-0448	Front frame left ST42, midnight blue		1
16	1539917-0449	Front frame left ST42, umber glaze		1
16	1539917-0450	Front frame left ST42, smokey grey		1
16	1539917-3524	Front frame left ST42, royal red		1
16	1539917-5969	Front frame left ST42, royal blue		1
16	1539917-9005	Front frame left ST42, black, glossy		1
16	1539917-9119	Front frame left ST42, black, bronze	20091001 → 20100630	1
17	1539918-0411	Front frame left ST44, black, matt	20100701 →	1
17	1539918-0419	Front frame left ST44, silver grey		1
17	1539918-0442	Front frame left ST44, night shimmer		1
17	1539918-0445	Front frame left ST44, ivory white	20100701 →	1
17	1539918-0446	Front frame left ST44, purple fire		1
17	1539918-0447	Front frame left ST44, emerald green		1
17	1539918-0448	Front frame left ST44, midnight blue		1
17	1539918-0449	Front frame left ST44, umber glaze		1
17	1539918-0450	Front frame left ST44, smokey grey		1
17	1539918-3524	Front frame left ST44, royal red		1
17	1539918-5969	Front frame left ST44, royal blue		1
17	1539918-9005	Front frame left ST44, black, glossy		1
17	1539918-9119	Front frame left ST44, black, bronze	20091001 → 20100630	1
18	1539919-0411	Front frame left ST46, black, matt	20100701 →	1
18	1539919-0419	Front frame left ST46, silver grey		1
18	1539919-0442	Front frame left ST46, night shimmer		1
18	1539919-0445	Front frame left ST46, ivory white	20100701 →	1
18	1539919-0446	Front frame left ST46, purple fire		1
18	1539919-0447	Front frame left ST46, emerald green		1
18	1539919-0448	Front frame left ST46, midnight blue		1
18	1539919-0449	Front frame left ST46, umber glaze		1
18	1539919-0450	Front frame left ST46, smokey grey		1
18	1539919-3524	Front frame left ST46, royal red		1
18	1539919-5969	Front frame left ST46, royal blue		1
18	1539919-9005	Front frame left ST46, black, glossy		1
18	1539919-9119	Front frame left ST46, black, bronze	20091001 → 20100630	1
19	1539920-0411	Front frame left ST48, black, matt	20100701 →	1
19	1539920-0419	Front frame left ST48, silver grey		1
19	1539920-0442	Front frame left ST48, night shimmer		1
19	1539920-0445	Front frame left ST48, ivory white	20100701 →	1
19	1539920-0446	Front frame left ST48, purple fire		1
19	1539920-0447	Front frame left ST48, emerald green		1
19	1539920-0448	Front frame left ST48, midnight blue		1
19	1539920-0449	Front frame left ST48, umber glaze		1
19	1539920-0450	Front frame left ST48, smokey grey		1
19	1539920-3524	Front frame left ST48, royal red		1
19	1539920-5969	Front frame left ST48, royal blue		1
19	1539920-9005	Front frame left ST48, black, glossy		1
19	1539920-9119	Front frame left ST48, black, bronze	20091001 → 20100630	1
20	1539921-0411	Front frame left ST50, black, matt	20100701 →	1
20	1539921-0419	Front frame left ST50, silver grey		1
20	1539921-0442	Front frame left ST50, night shimmer		1
20	1539921-0445	Front frame left ST50, ivory white	20100701 →	1

Front frames standard

Pos.	Part number	Description	Valid from → until	Qty
20	1539921-0446	Front frame left ST50, purple fire		1
20	1539921-0447	Front frame left ST50, emerald green		1
20	1539921-0448	Front frame left ST50, midnight blue		1
20	1539921-0449	Front frame left ST50, umber glaze		1
20	1539921-0450	Front frame left ST50, smokey grey		1
20	1539921-3524	Front frame left ST50, royal red		1
20	1539921-5969	Front frame left ST50, royal blue		1
20	1539921-9005	Front frame left ST50, black, glossy		1
20	1539921-9119	Front frame left ST50, black, bronze	20091001 → 20100630	1
21	1539922-0411	Front frame right ST32, black, matt	20100701 →	1
21	1539922-0419	Front frame right ST32, silver grey		1
21	1539922-0442	Front frame right ST32, night shimmer		1
21	1539922-0445	Front frame right ST32, ivory white	20100701 →	1
21	1539922-0446	Front frame right ST32, purple fire		1
21	1539922-0447	Front frame right ST32, emerald green		1
21	1539922-0448	Front frame right ST32, midnight blue		1
21	1539922-0449	Front frame right ST32, umber glaze		1
21	1539922-0450	Front frame right ST32, smokey grey		1
21	1539922-3524	Front frame right ST32, royal red		1
21	1539922-5969	Front frame right ST32, royal blue		1
21	1539922-9005	Front frame right ST32, black, glossy		1
21	1539922-9119	Front frame right ST32, black, bronze	20091001 → 20100630	1
22	1539923-0411	Front frame right ST34, black, matt	20100701 →	1
22	1539923-0419	Front frame right ST34, silver grey		1
22	1539923-0442	Front frame right ST34, night shimmer		1
22	1539923-0445	Front frame right ST34, ivory white	20100701 →	1
22	1539923-0446	Front frame right ST34, purple fire		1
22	1539923-0447	Front frame right ST34, emerald green		1
22	1539923-0448	Front frame right ST34, midnight blue		1
22	1539923-0449	Front frame right ST34, umber glaze		1
22	1539923-0450	Front frame right ST34, smokey grey		1
22	1539923-3524	Front frame right ST34, royal red		1
22	1539923-5969	Front frame right ST34, royal blue		1
22	1539923-9005	Front frame right ST34, black, glossy		1
22	1539923-9119	Front frame right ST34, black, bronze	20091001 → 20100630	1
23	1539924-0411	Front frame right ST36, black, matt	20100701 →	1
23	1539924-0419	Front frame right ST36, silver grey		1
23	1539924-0442	Front frame right ST36, night shimmer		1
23	1539924-0445	Front frame right ST36, ivory white	20100701 →	1
23	1539924-0446	Front frame right ST36, purple fire		1
23	1539924-0447	Front frame right ST36, emerald green		1
23	1539924-0448	Front frame right ST36, midnight blue		1
23	1539924-0449	Front frame right ST36, umber glaze		1
23	1539924-0450	Front frame right ST36, smokey grey		1
23	1539924-3524	Front frame right ST36, royal red		1
23	1539924-5969	Front frame right ST36, royal blue		1
23	1539924-9005	Front frame right ST36, black, glossy		1
23	1539924-9119	Front frame right ST36, black, bronze	20091001 → 20100630	1
24	1539925-0411	Front frame right ST38, black, matt	20100701 →	1
24	1539925-0419	Front frame right ST38, silver grey		1
24	1539925-0442	Front frame right ST38, night shimmer		1
24	1539925-0445	Front frame right ST38, ivory white	20100701 →	1
24	1539925-0446	Front frame right ST38, purple fire		1
24	1539925-0447	Front frame right ST38, emerald green		1
24	1539925-0448	Front frame right ST38, midnight blue		1


Front frames standard

Pos.	Part number	Description	Valid from → until	Qty
24	1539925-0449	Front frame right ST38, umber glaze		1
24	1539925-0450	Front frame right ST38, smokey grey		1
24	1539925-3524	Front frame right ST38, royal red		1
24	1539925-5969	Front frame right ST38, royal blue		1
24	1539925-9005	Front frame right ST38, black, glossy		1
24	1539925-9119	Front frame right ST38, black, bronze	20091001 → 20100630	1
25	1539926-0411	Front frame right ST40, black, matt	20100701 →	1
25	1539926-0419	Front frame right ST40, silver grey		1
25	1539926-0442	Front frame right ST40, night shimmer		1
25	1539926-0445	Front frame right ST40, ivory white	20100701 →	1
25	1539926-0446	Front frame right ST40, purple fire		1
25	1539926-0447	Front frame right ST40, emerald green		1
25	1539926-0448	Front frame right ST40, midnight blue		1
25	1539926-0449	Front frame right ST40, umber glaze		1
25	1539926-0450	Front frame right ST40, smokey grey		1
25	1539926-3524	Front frame right ST40, royal red		1
25	1539926-5969	Front frame right ST40, royal blue		1
25	1539926-9005	Front frame right ST40, black, glossy		1
25	1539926-9119	Front frame right ST40, black, bronze	20091001 → 20100630	1
26	1539927-0411	Front frame right ST42, black, matt	20100701 →	1
26	1539927-0419	Front frame right ST42, silver grey		1
26	1539927-0442	Front frame right ST42, night shimmer		1
26	1539927-0445	Front frame right ST42, ivory white	20100701 →	1
26	1539927-0446	Front frame right ST42, purple fire		1
26	1539927-0447	Front frame right ST42, emerald green		1
26	1539927-0448	Front frame right ST42, midnight blue		1
26	1539927-0449	Front frame right ST42, umber glaze		1
26	1539927-0450	Front frame right ST42, smokey grey		1
26	1539927-3524	Front frame right ST42, royal red		1
26	1539927-5969	Front frame right ST42, royal blue		1
26	1539927-9005	Front frame right ST42, black, glossy		1
26	1539927-9119	Front frame right ST42, black, bronze	20091001 → 20100630	1
27	1539928-0411	Front frame right ST44, black, matt	20100701 →	1
27	1539928-0419	Front frame right ST44, silver grey		1
27	1539928-0442	Front frame right ST44, night shimmer		1
27	1539928-0445	Front frame right ST44, ivory white	20100701 →	1
27	1539928-0446	Front frame right ST44, purple fire		1
27	1539928-0447	Front frame right ST44, emerald green		1
27	1539928-0448	Front frame right ST44, midnight blue		1
27	1539928-0449	Front frame right ST44, umber glaze		1
27	1539928-0450	Front frame right ST44, smokey grey		1
27	1539928-3524	Front frame right ST44, royal red		1
27	1539928-5969	Front frame right ST44, royal blue		1
27	1539928-9005	Front frame right ST44, black, glossy		1
27	1539928-9119	Front frame right ST44, black, bronze	20091001 → 20100630	1
28	1539929-0411	Front frame right ST46, black, matt	20100701 →	1
28	1539929-0419	Front frame right ST46, silver grey		1
28	1539929-0442	Front frame right ST46, night shimmer		1
28	1539929-0445	Front frame right ST46, ivory white	20100701 →	1
28	1539929-0446	Front frame right ST46, purple fire		1
28	1539929-0447	Front frame right ST46, emerald green		1
28	1539929-0448	Front frame right ST46, midnight blue		1
28	1539929-0449	Front frame right ST46, umber glaze		1
28	1539929-0450	Front frame right ST46, smokey grey		1
28	1539929-3524	Front frame right ST46, royal red		1

Front frames standard

Pos.	Part number	Description	Valid from → until	Qty
28	1539929-5969	Front frame right ST46, royal blue		1
28	1539929-9005	Front frame right ST46, black, glossy		1
28	1539929-9119	Front frame right ST46, black, bronze	20091001 → 20100630	1
29	1539930-0411	Front frame right ST48, black, matt	20100701 →	1
29	1539930-0419	Front frame right ST48, silver grey		1
29	1539930-0442	Front frame right ST48, night shimmer		1
29	1539930-0445	Front frame right ST48, ivory white	20100701 →	1
29	1539930-0446	Front frame right ST48, purple fire		1
29	1539930-0447	Front frame right ST48, emerald green		1
29	1539930-0448	Front frame right ST48, midnight blue		1
29	1539930-0449	Front frame right ST48, umber glaze		1
29	1539930-0450	Front frame right ST48, smokey grey		1
29	1539930-3524	Front frame right ST48, royal red		1
29	1539930-5969	Front frame right ST48, royal blue		1
29	1539930-9005	Front frame right ST48, black, glossy		1
29	1539930-9119	Front frame right ST48, black, bronze	20091001 → 20100630	1
30	1539931-0411	Front frame right ST50, black, matt	20100701 →	1
30	1539931-0419	Front frame right ST50, silver grey		1
30	1539931-0442	Front frame right ST50, night shimmer		1
30	1539931-0445	Front frame right ST50, ivory white	20100701 →	1
30	1539931-0446	Front frame right ST50, purple fire		1
30	1539931-0447	Front frame right ST50, emerald green		1
30	1539931-0448	Front frame right ST50, midnight blue		1
30	1539931-0449	Front frame right ST50, umber glaze		1
30	1539931-0450	Front frame right ST50, smokey grey		1
30	1539931-3524	Front frame right ST50, royal red		1
30	1539931-5969	Front frame right ST50, royal blue		1
30	1539931-9005	Front frame right ST50, black, glossy		1
30	1539931-9119	Front frame right ST50, black, bronze	20091001 → 20100630	1
31	1539963	Frames connecting tubes kit, ST32		1
32	1539964	Frames connecting tubes kit, ST34		1
33	1539965	Frames connecting tubes kit, ST36		1
34	1539966	Frames connecting tubes kit, ST38		1
35	1539967	Frames connecting tubes kit, ST40		1
36	1539968	Frames connecting tubes kit, ST42		1
37	1539969	Frames connecting tubes kit, ST44		1
38	1539970	Frames connecting tubes kit, ST46		1
39	1539971	Frames connecting tubes kit, ST48		1
40	1539972	Frames connecting tubes kit, ST50		1
41	1539973	Screws kit to frames connecting tubes		1
42	1539977	Position mechanism for swing-away, kit, pair		1
43	1537686	Seat lock kit		1
44	1537688	Seat lock kit with sleeves		1

Front frames abducted

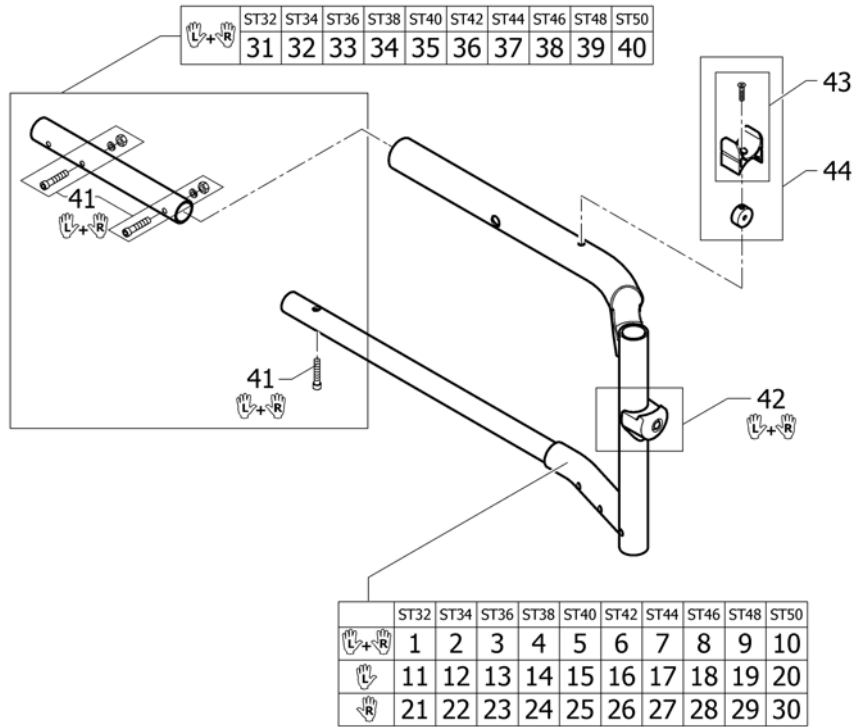

Pos.	Part number	Description	Valid from → until	Qty
1	1539932-0411	Front frame abducted ST32, black, matt, pair	20100701 →	1
1	1539932-0419	Front frame abducted ST32, silver grey, pair		1
1	1539932-0442	Front frame abducted ST32, night shimmer, pair		1
1	1539932-0445	Front frame abducted ST32, ivory white, pair	20100701 →	1
1	1539932-0446	Front frame abducted ST32, purple fire, pair		1
1	1539932-0447	Front frame abducted ST32, emerald green, pair		1
1	1539932-0448	Front frame abducted ST32, midnight blue, pair		1
1	1539932-0449	Front frame abducted ST32, umber glaze, pair		1
1	1539932-0450	Front frame abducted ST32, smokey grey, pair		1
1	1539932-3524	Front frame abducted ST32, royal red, pair		1
1	1539932-5969	Front frame abducted ST32, royal blue, pair		1
1	1539932-9005	Front frame abducted ST32, black, glossy, pair		1
1	1539932-9119	Front frame abducted ST32, black, bronze, pair	20091001 → 20100630	1
2	1539933-0411	Front frame abducted ST34, black, matt, pair	20100701 →	1
2	1539933-0419	Front frame abducted ST34, silver grey, pair		1
2	1539933-0442	Front frame abducted ST34, night shimmer, pair		1
2	1539933-0445	Front frame abducted ST34, ivory white, pair	20100701 →	1
2	1539933-0446	Front frame abducted ST34, purple fire, pair		1
2	1539933-0447	Front frame abducted ST34, emerald green, pair		1
2	1539933-0448	Front frame abducted ST34, midnight blue, pair		1
2	1539933-0449	Front frame abducted ST34, umber glaze, pair		1
2	1539933-0450	Front frame abducted ST34, smokey grey, pair		1
2	1539933-3524	Front frame abducted ST34, royal red, pair		1
2	1539933-5969	Front frame abducted ST34, royal blue, pair		1
2	1539933-9005	Front frame abducted ST34, black, glossy, pair		1
2	1539933-9119	Front frame abducted ST34, black, bronze, pair	20091001 → 20100630	1
3	1539934-0411	Front frame abducted ST36, black, matt, pair	20100701 →	1
3	1539934-0419	Front frame abducted ST36, silver grey, pair		1
3	1539934-0442	Front frame abducted ST36, night shimmer, pair		1
3	1539934-0445	Front frame abducted ST36, ivory white, pair	20100701 →	1
3	1539934-0446	Front frame abducted ST36, purple fire, pair		1

Front frames abducted

Pos.	Part number	Description	Valid from → until	Qty
3	1539934-0447	Front frame abducted ST36, emerald green, pair		1
3	1539934-0448	Front frame abducted ST36, midnight blue, pair		1
3	1539934-0449	Front frame abducted ST36, umber glaze, pair		1
3	1539934-0450	Front frame abducted ST36, smokey grey, pair		1
3	1539934-3524	Front frame abducted ST36, royal red, pair		1
3	1539934-5969	Front frame abducted ST36, royal blue, pair		1
3	1539934-9005	Front frame abducted ST36, black, glossy, pair		1
3	1539934-9119	Front frame abducted ST36, black, bronze, pair	20091001 → 20100630	1
4	1539935-0411	Front frame abducted ST38, black, matt, pair	20100701 →	1
4	1539935-0419	Front frame abducted ST38, silver grey, pair		1
4	1539935-0442	Front frame abducted ST38, night shimmer, pair		1
4	1539935-0445	Front frame abducted ST38, ivory white, pair	20100701 →	1
4	1539935-0446	Front frame abducted ST38, purple fire, pair		1
4	1539935-0447	Front frame abducted ST38, emerald green, pair		1
4	1539935-0448	Front frame abducted ST38, midnight blue, pair		1
4	1539935-0449	Front frame abducted ST38, umber glaze, pair		1
4	1539935-0450	Front frame abducted ST38, smokey grey, pair		1
4	1539935-3524	Front frame abducted ST38, royal red, pair		1
4	1539935-5969	Front frame abducted ST38, royal blue, pair		1
4	1539935-9005	Front frame abducted ST38, black, glossy, pair		1
4	1539935-9119	Front frame abducted ST38, black, bronze, pair	20091001 → 20100630	1
5	1539936-0411	Front frame abducted ST40, black, matt, pair	20100701 →	1
5	1539936-0419	Front frame abducted ST40, silver grey, pair		1
5	1539936-0442	Front frame abducted ST40, night shimmer, pair		1
5	1539936-0445	Front frame abducted ST40, ivory white, pair	20100701 →	1
5	1539936-0446	Front frame abducted ST40, purple fire, pair		1
5	1539936-0447	Front frame abducted ST40, emerald green, pair		1
5	1539936-0448	Front frame abducted ST40, midnight blue, pair		1
5	1539936-0449	Front frame abducted ST40, umber glaze, pair		1
5	1539936-0450	Front frame abducted ST40, smokey grey, pair		1
5	1539936-3524	Front frame abducted ST40, royal red, pair		1
5	1539936-5969	Front frame abducted ST40, royal blue, pair		1
5	1539936-9005	Front frame abducted ST40, black, glossy, pair		1
5	1539936-9119	Front frame abducted ST40, black, bronze, pair	20091001 → 20100630	1
6	1539937-0411	Front frame abducted ST42, black, matt, pair	20100701 →	1
6	1539937-0419	Front frame abducted ST42, silver grey, pair		1
6	1539937-0442	Front frame abducted ST42, night shimmer, pair		1
6	1539937-0445	Front frame abducted ST42, ivory white, pair	20100701 →	1
6	1539937-0446	Front frame abducted ST42, purple fire, pair		1
6	1539937-0447	Front frame abducted ST42, emerald green, pair		1
6	1539937-0448	Front frame abducted ST42, midnight blue, pair		1
6	1539937-0449	Front frame abducted ST42, umber glaze, pair		1
6	1539937-0450	Front frame abducted ST42, smokey grey, pair		1
6	1539937-3524	Front frame abducted ST42, royal red, pair		1
6	1539937-5969	Front frame abducted ST42, royal blue, pair		1
6	1539937-9005	Front frame abducted ST42, black, glossy, pair		1
6	1539937-9119	Front frame abducted ST42, black, bronze, pair	20091001 → 20100630	1
7	1539938-0411	Front frame abducted ST44, black, matt, pair	20100701 →	1
7	1539938-0419	Front frame abducted ST44, silver grey, pair		1
7	1539938-0442	Front frame abducted ST44, night shimmer, pair		1
7	1539938-0445	Front frame abducted ST44, ivory white, pair	20100701 →	1
7	1539938-0446	Front frame abducted ST44, purple fire, pair		1
7	1539938-0447	Front frame abducted ST44, emerald green, pair		1
7	1539938-0448	Front frame abducted ST44, midnight blue, pair		1
7	1539938-0449	Front frame abducted ST44, umber glaze, pair		1

Front frames abducted

Pos.	Part number	Description	Valid from → until	Qty
7	1539938-0450	Front frame abducted ST44, smokey grey, pair		1
7	1539938-3524	Front frame abducted ST44, royal red, pair		1
7	1539938-5969	Front frame abducted ST44, royal blue, pair		1
7	1539938-9005	Front frame abducted ST44, black, glossy, pair		1
7	1539938-9119	Front frame abducted ST44, black, bronze, pair	20091001 → 20100630	1
8	1539939-0411	Front frame abducted ST46, black, matt, pair	20100701 →	1
8	1539939-0419	Front frame abducted ST46, silver grey, pair		1
8	1539939-0442	Front frame abducted ST46, night shimmer, pair		1
8	1539939-0445	Front frame abducted ST46, ivory white, pair	20100701 →	1
8	1539939-0446	Front frame abducted ST46, purple fire, pair		1
8	1539939-0447	Front frame abducted ST46, emerald green, pair		1
8	1539939-0448	Front frame abducted ST46, midnight blue, pair		1
8	1539939-0449	Front frame abducted ST46, umber glaze, pair		1
8	1539939-0450	Front frame abducted ST46, smokey grey, pair		1
8	1539939-3524	Front frame abducted ST46, royal red, pair		1
8	1539939-5969	Front frame abducted ST46, royal blue, pair		1
8	1539939-9005	Front frame abducted ST46, black, glossy, pair		1
8	1539939-9119	Front frame abducted ST46, black, bronze, pair	20091001 → 20100630	1
9	1539940-0411	Front frame abducted ST48, black, matt, pair	20100701 →	1
9	1539940-0419	Front frame abducted ST48, silver grey, pair		1
9	1539940-0442	Front frame abducted ST48, night shimmer, pair		1
9	1539940-0445	Front frame abducted ST48, ivory white, pair	20100701 →	1
9	1539940-0446	Front frame abducted ST48, purple fire, pair		1
9	1539940-0447	Front frame abducted ST48, emerald green, pair		1
9	1539940-0448	Front frame abducted ST48, midnight blue, pair		1
9	1539940-0449	Front frame abducted ST48, umber glaze, pair		1
9	1539940-0450	Front frame abducted ST48, smokey grey, pair		1
9	1539940-3524	Front frame abducted ST48, royal red, pair		1
9	1539940-5969	Front frame abducted ST48, royal blue, pair		1
9	1539940-9005	Front frame abducted ST48, black, glossy, pair		1
9	1539940-9119	Front frame abducted ST48, black, bronze, pair	20091001 → 20100630	1
10	1539941-0411	Front frame abducted ST50, black, matt, pair	20100701 →	1
10	1539941-0419	Front frame abducted ST50, silver grey, pair		1
10	1539941-0442	Front frame abducted ST50, night shimmer, pair		1
10	1539941-0445	Front frame abducted ST50, ivory white, pair	20100701 →	1
10	1539941-0446	Front frame abducted ST50, purple fire, pair		1
10	1539941-0447	Front frame abducted ST50, emerald green, pair		1
10	1539941-0448	Front frame abducted ST50, midnight blue, pair		1
10	1539941-0449	Front frame abducted ST50, umber glaze, pair		1
10	1539941-0450	Front frame abducted ST50, smokey grey, pair		1
10	1539941-3524	Front frame abducted ST50, royal red, pair		1
10	1539941-5969	Front frame abducted ST50, royal blue, pair		1
10	1539941-9005	Front frame abducted ST50, black, glossy, pair		1
10	1539941-9119	Front frame abducted ST50, black, bronze, pair	20091001 → 20100630	1
11	1539942-0411	Front frame abducted left ST32, black, matt	20100701 →	1
11	1539942-0419	Front frame abducted left ST32, silver grey		1
11	1539942-0442	Front frame abducted left ST32, night shimmer		1
11	1539942-0445	Front frame abducted left ST32, ivory white	20100701 →	1
11	1539942-0446	Front frame abducted left ST32, purple fire		1
11	1539942-0447	Front frame abducted left ST32, emerald green		1
11	1539942-0448	Front frame abducted left ST32, midnight blue		1
11	1539942-0449	Front frame abducted left ST32, umber glaze		1
11	1539942-0450	Front frame abducted left ST32, smokey grey		1
11	1539942-3524	Front frame abducted left ST32, royal red		1
11	1539942-5969	Front frame abducted left ST32, royal blue		1

Front frames abducted

Pos.	Part number	Description	Valid from → until	Qty
11	1539942-9005	Front frame abducted left ST32, black, glossy		1
11	1539942-9119	Front frame abducted left ST32, black, bronze	20091001 → 20100630	1
12	1539943-0411	Front frame abducted left ST34, black, matt	20100701 →	1
12	1539943-0419	Front frame abducted left ST34, silver grey		1
12	1539943-0442	Front frame abducted left ST34, night shimmer		1
12	1539943-0445	Front frame abducted left ST34, ivory white	20100701 →	1
12	1539943-0446	Front frame abducted left ST34, purple fire		1
12	1539943-0447	Front frame abducted left ST34, emerald green		1
12	1539943-0448	Front frame abducted left ST34, midnight blue		1
12	1539943-0449	Front frame abducted left ST34, umber glaze		1
12	1539943-0450	Front frame abducted left ST34, smokey grey		1
12	1539943-3524	Front frame abducted left ST34, royal red		1
12	1539943-5969	Front frame abducted left ST34, royal blue		1
12	1539943-9005	Front frame abducted left ST34, black, glossy		1
12	1539943-9119	Front frame abducted left ST34, black, bronze	20091001 → 20100630	1
13	1539944-0411	Front frame abducted left ST36, black, matt	20100701 →	1
13	1539944-0419	Front frame abducted left ST36, silver grey		1
13	1539944-0442	Front frame abducted left ST36, night shimmer		1
13	1539944-0445	Front frame abducted left ST36, ivory white	20100701 →	1
13	1539944-0446	Front frame abducted left ST36, purple fire		1
13	1539944-0447	Front frame abducted left ST36, emerald green		1
13	1539944-0448	Front frame abducted left ST36, midnight blue		1
13	1539944-0449	Front frame abducted left ST36, umber glaze		1
13	1539944-0450	Front frame abducted left ST36, smokey grey		1
13	1539944-3524	Front frame abducted left ST36, royal red		1
13	1539944-5969	Front frame abducted left ST36, royal blue		1
13	1539944-9005	Front frame abducted left ST36, black, glossy		1
13	1539944-9119	Front frame abducted left ST36, black, bronze	20091001 → 20100630	1
14	1539945-0411	Front frame abducted left ST38, black, matt	20100701 →	1
14	1539945-0419	Front frame abducted left ST38, silver grey		1
14	1539945-0442	Front frame abducted left ST38, night shimmer		1
14	1539945-0445	Front frame abducted left ST38, ivory white	20100701 →	1
14	1539945-0446	Front frame abducted left ST38, purple fire		1
14	1539945-0447	Front frame abducted left ST38, emerald green		1
14	1539945-0448	Front frame abducted left ST38, midnight blue		1
14	1539945-0449	Front frame abducted left ST38, umber glaze		1
14	1539945-0450	Front frame abducted left ST38, smokey grey		1
14	1539945-3524	Front frame abducted left ST38, royal red		1
14	1539945-5969	Front frame abducted left ST38, royal blue		1
14	1539945-9005	Front frame abducted left ST38, black, glossy		1
14	1539945-9119	Front frame abducted left ST38, black, bronze	20091001 → 20100630	1
15	1539946-0411	Front frame abducted left ST40, black, matt	20100701 →	1
15	1539946-0419	Front frame abducted left ST40, silver grey		1
15	1539946-0442	Front frame abducted left ST40, night shimmer		1
15	1539946-0445	Front frame abducted left ST40, ivory white	20100701 →	1
15	1539946-0446	Front frame abducted left ST40, purple fire		1
15	1539946-0447	Front frame abducted left ST40, emerald green		1
15	1539946-0448	Front frame abducted left ST40, midnight blue		1
15	1539946-0449	Front frame abducted left ST40, umber glaze		1
15	1539946-0450	Front frame abducted left ST40, smokey grey		1
15	1539946-3524	Front frame abducted left ST40, royal red		1
15	1539946-5969	Front frame abducted left ST40, royal blue		1
15	1539946-9005	Front frame abducted left ST40, black, glossy		1
15	1539946-9119	Front frame abducted left ST40, black, bronze	20091001 → 20100630	1
16	1539947-0411	Front frame abducted left ST42, black, matt	20100701 →	1

Front frames abducted

Pos.	Part number	Description	Valid from → until	Qty
16	1539947-0419	Front frame abducted left ST42, silver grey		1
16	1539947-0442	Front frame abducted left ST42, night shimmer		1
16	1539947-0445	Front frame abducted left ST42, ivory white	20100701 →	1
16	1539947-0446	Front frame abducted left ST42, purple fire		1
16	1539947-0447	Front frame abducted left ST42, emerald green		1
16	1539947-0448	Front frame abducted left ST42, midnight blue		1
16	1539947-0449	Front frame abducted left ST42, umber glaze		1
16	1539947-0450	Front frame abducted left ST42, smokey grey		1
16	1539947-3524	Front frame abducted left ST42, royal red		1
16	1539947-5969	Front frame abducted left ST42, royal blue		1
16	1539947-9005	Front frame abducted left ST42, black, glossy		1
16	1539947-9119	Front frame abducted left ST42, black, bronze	20091001 → 20100630	1
17	1539948-0411	Front frame abducted left ST44, black, matt	20100701 →	1
17	1539948-0419	Front frame abducted left ST44, silver grey		1
17	1539948-0442	Front frame abducted left ST44, night shimmer		1
17	1539948-0445	Front frame abducted left ST44, ivory white	20100701 →	1
17	1539948-0446	Front frame abducted left ST44, purple fire		1
17	1539948-0447	Front frame abducted left ST44, emerald green		1
17	1539948-0448	Front frame abducted left ST44, midnight blue		1
17	1539948-0449	Front frame abducted left ST44, umber glaze		1
17	1539948-0450	Front frame abducted left ST44, smokey grey		1
17	1539948-3524	Front frame abducted left ST44, royal red		1
17	1539948-5969	Front frame abducted left ST44, royal blue		1
17	1539948-9005	Front frame abducted left ST44, black, glossy		1
17	1539948-9119	Front frame abducted left ST44, black, bronze	20091001 → 20100630	1
18	1539949-0411	Front frame abducted left ST46, black, matt	20100701 →	1
18	1539949-0419	Front frame abducted left ST46, silver grey		1
18	1539949-0442	Front frame abducted left ST46, night shimmer		1
18	1539949-0445	Front frame abducted left ST46, ivory white	20100701 →	1
18	1539949-0446	Front frame abducted left ST46, purple fire		1
18	1539949-0447	Front frame abducted left ST46, emerald green		1
18	1539949-0448	Front frame abducted left ST46, midnight blue		1
18	1539949-0449	Front frame abducted left ST46, umber glaze		1
18	1539949-0450	Front frame abducted left ST46, smokey grey		1
18	1539949-3524	Front frame abducted left ST46, royal red		1
18	1539949-5969	Front frame abducted left ST46, royal blue		1
18	1539949-9005	Front frame abducted left ST46, black, glossy		1
18	1539949-9119	Front frame abducted left ST46, black, bronze	20091001 → 20100630	1
19	1539950-0411	Front frame abducted left ST48, black, matt	20100701 →	1
19	1539950-0419	Front frame abducted left ST48, silver grey		1
19	1539950-0442	Front frame abducted left ST48, night shimmer		1
19	1539950-0445	Front frame abducted left ST48, ivory white	20100701 →	1
19	1539950-0446	Front frame abducted left ST48, purple fire		1
19	1539950-0447	Front frame abducted left ST48, emerald green		1
19	1539950-0448	Front frame abducted left ST48, midnight blue		1
19	1539950-0449	Front frame abducted left ST48, umber glaze		1
19	1539950-0450	Front frame abducted left ST48, smokey grey		1
19	1539950-3524	Front frame abducted left ST48, royal red		1
19	1539950-5969	Front frame abducted left ST48, royal blue		1
19	1539950-9005	Front frame abducted left ST48, black, glossy		1
19	1539950-9119	Front frame abducted left ST48, black, bronze	20091001 → 20100630	1
20	1539951-0411	Front frame abducted left ST50, black, matt	20100701 →	1
20	1539951-0419	Front frame abducted left ST50, silver grey		1
20	1539951-0442	Front frame abducted left ST50, night shimmer		1
20	1539951-0445	Front frame abducted left ST50, ivory white	20100701 →	1

Front frames abducted

Pos.	Part number	Description	Valid from → until	Qty
20	1539951-0446	Front frame abducted left ST50, purple fire		1
20	1539951-0447	Front frame abducted left ST50, emerald green		1
20	1539951-0448	Front frame abducted left ST50, midnight blue		1
20	1539951-0449	Front frame abducted left ST50, umber glaze		1
20	1539951-0450	Front frame abducted left ST50, smokey grey		1
20	1539951-3524	Front frame abducted left ST50, royal red		1
20	1539951-5969	Front frame abducted left ST50, royal blue		1
20	1539951-9005	Front frame abducted left ST50, black, glossy		1
20	1539951-9119	Front frame abducted left ST50, black, bronze	20091001 → 20100630	1
21	1539952-0411	Front frame abducted right ST32, black, matt	20100701 →	1
21	1539952-0419	Front frame abducted right ST32, silver grey		1
21	1539952-0442	Front frame abducted right ST32, night shimmer		1
21	1539952-0445	Front frame abducted right ST32, ivory white	20100701 →	1
21	1539952-0446	Front frame abducted right ST32, purple fire		1
21	1539952-0447	Front frame abducted right ST32, emerald green		1
21	1539952-0448	Front frame abducted right ST32, midnight blue		1
21	1539952-0449	Front frame abducted right ST32, umber glaze		1
21	1539952-0450	Front frame abducted right ST32, smokey grey		1
21	1539952-3524	Front frame abducted right ST32, royal red		1
21	1539952-5969	Front frame abducted right ST32, royal blue		1
21	1539952-9005	Front frame abducted right ST32, black, glossy		1
21	1539952-9119	Front frame abducted right ST32, black, bronze	20091001 → 20100630	1
22	1539953-0411	Front frame abducted right ST34, black, matt	20100701 →	1
22	1539953-0419	Front frame abducted right ST34, silver grey		1
22	1539953-0442	Front frame abducted right ST34, night shimmer		1
22	1539953-0445	Front frame abducted right ST34, ivory white	20100701 →	1
22	1539953-0446	Front frame abducted right ST34, purple fire		1
22	1539953-0447	Front frame abducted right ST34, emerald green		1
22	1539953-0448	Front frame abducted right ST34, midnight blue		1
22	1539953-0449	Front frame abducted right ST34, umber glaze		1
22	1539953-0450	Front frame abducted right ST34, smokey grey		1
22	1539953-3524	Front frame abducted right ST34, royal red		1
22	1539953-5969	Front frame abducted right ST34, royal blue		1
22	1539953-9005	Front frame abducted right ST34, black, glossy		1
22	1539953-9119	Front frame abducted right ST34, black, bronze	20091001 → 20100630	1
23	1539954-0411	Front frame abducted right ST36, black, matt	20100701 →	1
23	1539954-0419	Front frame abducted right ST36, silver grey		1
23	1539954-0442	Front frame abducted right ST36, night shimmer		1
23	1539954-0445	Front frame abducted right ST36, ivory white	20100701 →	1
23	1539954-0446	Front frame abducted right ST36, purple fire		1
23	1539954-0447	Front frame abducted right ST36, emerald green		1
23	1539954-0448	Front frame abducted right ST36, midnight blue		1
23	1539954-0449	Front frame abducted right ST36, umber glaze		1
23	1539954-0450	Front frame abducted right ST36, smokey grey		1
23	1539954-3524	Front frame abducted right ST36, royal red		1
23	1539954-5969	Front frame abducted right ST36, royal blue		1
23	1539954-9005	Front frame abducted right ST36, black, glossy		1
23	1539954-9119	Front frame abducted right ST36, black, bronze	20091001 → 20100630	1
24	1539955-0411	Front frame abducted right ST38, black, matt	20100701 →	1
24	1539955-0419	Front frame abducted right ST38, silver grey		1
24	1539955-0442	Front frame abducted right ST38, night shimmer		1
24	1539955-0445	Front frame abducted right ST38, ivory white	20100701 →	1
24	1539955-0446	Front frame abducted right ST38, purple fire		1
24	1539955-0447	Front frame abducted right ST38, emerald green		1
24	1539955-0448	Front frame abducted right ST38, midnight blue		1


Front frames abducted

Pos.	Part number	Description	Valid from → until	Qty
24	1539955-0449	Front frame abducted right ST38, umber glaze		1
24	1539955-0450	Front frame abducted right ST38, smokey grey		1
24	1539955-3524	Front frame abducted right ST38, royal red		1
24	1539955-5969	Front frame abducted right ST38, royal blue		1
24	1539955-9005	Front frame abducted right ST38, black, glossy		1
24	1539955-9119	Front frame abducted right ST38, black, bronze	20091001 → 20100630	1
25	1539956-0411	Front frame abducted right ST40, black, matt	20100701 →	1
25	1539956-0419	Front frame abducted right ST40, silver grey		1
25	1539956-0442	Front frame abducted right ST40, night shimmer		1
25	1539956-0445	Front frame abducted right ST40, ivory white	20100701 →	1
25	1539956-0446	Front frame abducted right ST40, purple fire		1
25	1539956-0447	Front frame abducted right ST40, emerald green		1
25	1539956-0448	Front frame abducted right ST40, midnight blue		1
25	1539956-0449	Front frame abducted right ST40, umber glaze		1
25	1539956-0450	Front frame abducted right ST40, smokey grey		1
25	1539956-3524	Front frame abducted right ST40, royal red		1
25	1539956-5969	Front frame abducted right ST40, royal blue		1
25	1539956-9005	Front frame abducted right ST40, black, glossy		1
25	1539956-9119	Front frame abducted right ST40, black, bronze	20091001 → 20100630	1
26	1539957-0411	Front frame abducted right ST42, black, matt	20100701 →	1
26	1539957-0419	Front frame abducted right ST42, silver grey		1
26	1539957-0442	Front frame abducted right ST42, night shimmer		1
26	1539957-0445	Front frame abducted right ST42, ivory white	20100701 →	1
26	1539957-0446	Front frame abducted right ST42, purple fire		1
26	1539957-0447	Front frame abducted right ST42, emerald green		1
26	1539957-0448	Front frame abducted right ST42, midnight blue		1
26	1539957-0449	Front frame abducted right ST42, umber glaze		1
26	1539957-0450	Front frame abducted right ST42, smokey grey		1
26	1539957-3524	Front frame abducted right ST42, royal red		1
26	1539957-5969	Front frame abducted right ST42, royal blue		1
26	1539957-9005	Front frame abducted right ST42, black, glossy		1
26	1539957-9119	Front frame abducted right ST42, black, bronze	20091001 → 20100630	1
27	1539958-0411	Front frame abducted right ST44, black, matt	20100701 →	1
27	1539958-0419	Front frame abducted right ST44, silver grey		1
27	1539958-0442	Front frame abducted right ST44, night shimmer		1
27	1539958-0445	Front frame abducted right ST44, ivory white	20100701 →	1
27	1539958-0446	Front frame abducted right ST44, purple fire		1
27	1539958-0447	Front frame abducted right ST44, emerald green		1
27	1539958-0448	Front frame abducted right ST44, midnight blue		1
27	1539958-0449	Front frame abducted right ST44, umber glaze		1
27	1539958-0450	Front frame abducted right ST44, smokey grey		1
27	1539958-3524	Front frame abducted right ST44, royal red		1
27	1539958-5969	Front frame abducted right ST44, royal blue		1
27	1539958-9005	Front frame abducted right ST44, black, glossy		1
27	1539958-9119	Front frame abducted right ST44, black, bronze	20091001 → 20100630	1
28	1539959-0411	Front frame abducted right ST46, black, matt	20100701 →	1
28	1539959-0419	Front frame abducted right ST46, silver grey		1
28	1539959-0442	Front frame abducted right ST46, night shimmer		1
28	1539959-0445	Front frame abducted right ST46, ivory white	20100701 →	1
28	1539959-0446	Front frame abducted right ST46, purple fire		1
28	1539959-0447	Front frame abducted right ST46, emerald green		1
28	1539959-0448	Front frame abducted right ST46, midnight blue		1
28	1539959-0449	Front frame abducted right ST46, umber glaze		1
28	1539959-0450	Front frame abducted right ST46, smokey grey		1
28	1539959-3524	Front frame abducted right ST46, royal red		1

Front frames abducted

Pos.	Part number	Description	Valid from → until	Qty
28	1539959-5969	Front frame abducted right ST46, royal blue		1
28	1539959-9005	Front frame abducted right ST46, black, glossy		1
28	1539959-9119	Front frame abducted right ST46, black, bronze	20091001 → 20100630	1
29	1539960-0411	Front frame abducted right ST48, black, matt	20100701 →	1
29	1539960-0419	Front frame abducted right ST48, silver grey		1
29	1539960-0442	Front frame abducted right ST48, night shimmer		1
29	1539960-0445	Front frame abducted right ST48, ivory white	20100701 →	1
29	1539960-0446	Front frame abducted right ST48, purple fire		1
29	1539960-0447	Front frame abducted right ST48, emerald green		1
29	1539960-0448	Front frame abducted right ST48, midnight blue		1
29	1539960-0449	Front frame abducted right ST48, umber glaze		1
29	1539960-0450	Front frame abducted right ST48, smokey grey		1
29	1539960-3524	Front frame abducted right ST48, royal red		1
29	1539960-5969	Front frame abducted right ST48, royal blue		1
29	1539960-9005	Front frame abducted right ST48, black, glossy		1
29	1539960-9119	Front frame abducted right ST48, black, bronze	20091001 → 20100630	1
30	1539961-0411	Front frame abducted right ST50, black, matt	20100701 →	1
30	1539961-0419	Front frame abducted right ST50, silver grey		1
30	1539961-0442	Front frame abducted right ST50, night shimmer		1
30	1539961-0445	Front frame abducted right ST50, ivory white	20100701 →	1
30	1539961-0446	Front frame abducted right ST50, purple fire		1
30	1539961-0447	Front frame abducted right ST50, emerald green		1
30	1539961-0448	Front frame abducted right ST50, midnight blue		1
30	1539961-0449	Front frame abducted right ST50, umber glaze		1
30	1539961-0450	Front frame abducted right ST50, smokey grey		1
30	1539961-3524	Front frame abducted right ST50, royal red		1
30	1539961-5969	Front frame abducted right ST50, royal blue		1
30	1539961-9005	Front frame abducted right ST50, black, glossy		1
30	1539961-9119	Front frame abducted right ST50, black, bronze	20091001 → 20100630	1
31	1539991	Frames abducted connecting tubes kit, ST32		1
32	1539992	Frames abducted connecting tubes kit, ST34		1
33	1539993	Frames abducted connecting tubes kit, ST36		1
34	1539994	Frames abducted connecting tubes kit, ST38		1
35	1539995	Frames abducted connecting tubes kit, ST40		1
36	1539996	Frames abducted connecting tubes kit, ST42		1
37	1539997	Frames abducted connecting tubes kit, ST44		1
38	1539998	Frames abducted connecting tubes kit, ST46		1
39	1539999	Frames abducted connecting tubes kit, ST48		1
40	1540000	Frames abducted connecting tubes kit, ST50		1
41	1539973	Screws kit to frames connecting tubes		1
42	1539977	Position mechanism for swing-away, kit, pair		1
43	1537686	Seat lock kit		1
44	1537688	Seat lock kit with sleeves		1

Cross brace single, SB28-42

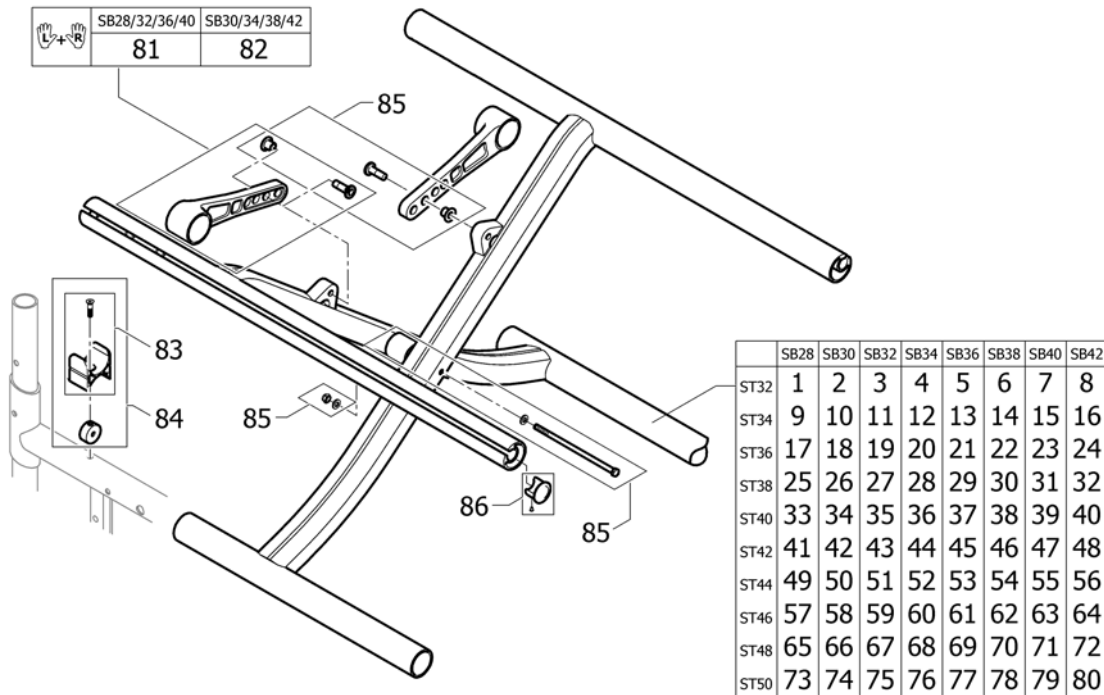

Pos.	Part number	Description	Valid from → until	Qty
1	1536959-0411	Cross-brace single, SB28 ST32, black, matt	20100701 →	1
1	1536959-0419	Cross-brace single, SB28 ST32, silver grey		1
1	1536959-0442	Cross-brace single, SB28 ST32, night shimmer		1
1	1536959-0445	Cross-brace single, SB28 ST32, ivory white	20100701 →	1
1	1536959-0446	Cross-brace single, SB28 ST32, purple fire		1
1	1536959-0447	Cross-brace single, SB28 ST32, emerald green		1
1	1536959-0448	Cross-brace single, SB28 ST32, midnight blue		1
1	1536959-0449	Cross-brace single, SB28 ST32, umber glaze		1
1	1536959-0450	Cross-brace single, SB28 ST32, smokey grey		1
1	1536959-3524	Cross-brace single, SB28 ST32, royal red		1
1	1536959-5969	Cross-brace single, SB28 ST32, royal blue		1
1	1536959-9005	Cross-brace single, SB28 ST32, black, glossy		1
1	1536959-9119	Cross-brace single, SB28 ST32, black, bronze	20091001 → 20100630	1
2	1536960-0411	Cross-brace single, SB30 ST32, black, matt	20100701 →	1
2	1536960-0419	Cross-brace single, SB30 ST32, silver grey		1
2	1536960-0442	Cross-brace single, SB30 ST32, night shimmer		1
2	1536960-0445	Cross-brace single, SB30 ST32, ivory white	20100701 →	1
2	1536960-0446	Cross-brace single, SB30 ST32, purple fire		1
2	1536960-0447	Cross-brace single, SB30 ST32, emerald green		1
2	1536960-0448	Cross-brace single, SB30 ST32, midnight blue		1
2	1536960-0449	Cross-brace single, SB30 ST32, umber glaze		1
2	1536960-0450	Cross-brace single, SB30 ST32, smokey grey		1
2	1536960-3524	Cross-brace single, SB30 ST32, royal red		1
2	1536960-5969	Cross-brace single, SB30 ST32, royal blue		1
2	1536960-9005	Cross-brace single, SB30 ST32, black, glossy		1
2	1536960-9119	Cross-brace single, SB30 ST32, black, bronze	20091001 → 20100630	1
3	1536961-0411	Cross-brace single, SB32 ST32, black, matt	20100701 →	1
3	1536961-0419	Cross-brace single, SB32 ST32, silver grey		1
3	1536961-0442	Cross-brace single, SB32 ST32, night shimmer		1
3	1536961-0445	Cross-brace single, SB32 ST32, ivory white	20100701 →	1
3	1536961-0446	Cross-brace single, SB32 ST32, purple fire		1

Cross brace single, SB28-42

Pos.	Part number	Description	Valid from → until	Qty
3	1536961-0447	Cross-brace single, SB32 ST32, emerald green		1
3	1536961-0448	Cross-brace single, SB32 ST32, midnight blue		1
3	1536961-0449	Cross-brace single, SB32 ST32, umber glaze		1
3	1536961-0450	Cross-brace single, SB32 ST32, smokey grey		1
3	1536961-3524	Cross-brace single, SB32 ST32, royal red		1
3	1536961-5969	Cross-brace single, SB32 ST32, royal blue		1
3	1536961-9005	Cross-brace single, SB32 ST32, black, glossy		1
3	1536961-9119	Cross-brace single, SB32 ST32, black, bronze	20091001 → 20100630	1
4	1536962-0411	Cross-brace single, SB34 ST32, black, matt	20100701 →	1
4	1536962-0419	Cross-brace single, SB34 ST32, silver grey		1
4	1536962-0442	Cross-brace single, SB34 ST32, night shimmer		1
4	1536962-0445	Cross-brace single, SB34 ST32, ivory white	20100701 →	1
4	1536962-0446	Cross-brace single, SB34 ST32, purple fire		1
4	1536962-0447	Cross-brace single, SB34 ST32, emerald green		1
4	1536962-0448	Cross-brace single, SB34 ST32, midnight blue		1
4	1536962-0449	Cross-brace single, SB34 ST32, umber glaze		1
4	1536962-0450	Cross-brace single, SB34 ST32, smokey grey		1
4	1536962-3524	Cross-brace single, SB34 ST32, royal red		1
4	1536962-5969	Cross-brace single, SB34 ST32, royal blue		1
4	1536962-9005	Cross-brace single, SB34 ST32, black, glossy		1
4	1536962-9119	Cross-brace single, SB34 ST32, black, bronze	20091001 → 20100630	1
5	1536963-0411	Cross-brace single, SB36 ST32, black, matt	20100701 →	1
5	1536963-0419	Cross-brace single, SB36 ST32, silver grey		1
5	1536963-0442	Cross-brace single, SB36 ST32, night shimmer		1
5	1536963-0445	Cross-brace single, SB36 ST32, ivory white	20100701 →	1
5	1536963-0446	Cross-brace single, SB36 ST32, purple fire		1
5	1536963-0447	Cross-brace single, SB36 ST32, emerald green		1
5	1536963-0448	Cross-brace single, SB36 ST32, midnight blue		1
5	1536963-0449	Cross-brace single, SB36 ST32, umber glaze		1
5	1536963-0450	Cross-brace single, SB36 ST32, smokey grey		1
5	1536963-3524	Cross-brace single, SB36 ST32, royal red		1
5	1536963-5969	Cross-brace single, SB36 ST32, royal blue		1
5	1536963-9005	Cross-brace single, SB36 ST32, black, glossy		1
5	1536963-9119	Cross-brace single, SB36 ST32, black, bronze	20091001 → 20100630	1
6	1536964-0411	Cross-brace single, SB38 ST32, black, matt	20100701 →	1
6	1536964-0419	Cross-brace single, SB38 ST32, silver grey		1
6	1536964-0442	Cross-brace single, SB38 ST32, night shimmer		1
6	1536964-0445	Cross-brace single, SB38 ST32, ivory white	20100701 →	1
6	1536964-0446	Cross-brace single, SB38 ST32, purple fire		1
6	1536964-0447	Cross-brace single, SB38 ST32, emerald green		1
6	1536964-0448	Cross-brace single, SB38 ST32, midnight blue		1
6	1536964-0449	Cross-brace single, SB38 ST32, umber glaze		1
6	1536964-0450	Cross-brace single, SB38 ST32, smokey grey		1
6	1536964-3524	Cross-brace single, SB38 ST32, royal red		1
6	1536964-5969	Cross-brace single, SB38 ST32, royal blue		1
6	1536964-9005	Cross-brace single, SB38 ST32, black, glossy		1
6	1536964-9119	Cross-brace single, SB38 ST32, black, bronze	20091001 → 20100630	1
7	1536965-0411	Cross-brace single, SB40 ST32, black, matt	20100701 →	1
7	1536965-0419	Cross-brace single, SB40 ST32, silver grey		1
7	1536965-0442	Cross-brace single, SB40 ST32, night shimmer		1
7	1536965-0445	Cross-brace single, SB40 ST32, ivory white	20100701 →	1
7	1536965-0446	Cross-brace single, SB40 ST32, purple fire		1
7	1536965-0447	Cross-brace single, SB40 ST32, emerald green		1
7	1536965-0448	Cross-brace single, SB40 ST32, midnight blue		1
7	1536965-0449	Cross-brace single, SB40 ST32, umber glaze		1

Cross brace single, SB28-42

Pos.	Part number	Description	Valid from → until	Qty
7	1536965-0450	Cross-brace single, SB40 ST32, smokey grey		1
7	1536965-3524	Cross-brace single, SB40 ST32, royal red		1
7	1536965-5969	Cross-brace single, SB40 ST32, royal blue		1
7	1536965-9005	Cross-brace single, SB40 ST32, black, glossy		1
7	1536965-9119	Cross-brace single, SB40 ST32, black, bronze	20091001 → 20100630	1
8	1536966-0411	Cross-brace single, SB42 ST32, black, matt	20100701 →	1
8	1536966-0419	Cross-brace single, SB42 ST32, silver grey		1
8	1536966-0442	Cross-brace single, SB42 ST32, night shimmer		1
8	1536966-0445	Cross-brace single, SB42 ST32, ivory white	20100701 →	1
8	1536966-0446	Cross-brace single, SB42 ST32, purple fire		1
8	1536966-0447	Cross-brace single, SB42 ST32, emerald green		1
8	1536966-0448	Cross-brace single, SB42 ST32, midnight blue		1
8	1536966-0449	Cross-brace single, SB42 ST32, umber glaze		1
8	1536966-0450	Cross-brace single, SB42 ST32, smokey grey		1
8	1536966-3524	Cross-brace single, SB42 ST32, royal red		1
8	1536966-5969	Cross-brace single, SB42 ST32, royal blue		1
8	1536966-9005	Cross-brace single, SB42 ST32, black, glossy		1
8	1536966-9119	Cross-brace single, SB42 ST32, black, bronze	20091001 → 20100630	1
9	1536971-0411	Cross-brace single, SB28 ST34, black, matt	20100701 →	1
9	1536971-0419	Cross-brace single, SB28 ST34, silver grey		1
9	1536971-0442	Cross-brace single, SB28 ST34, night shimmer		1
9	1536971-0445	Cross-brace single, SB28 ST34, ivory white	20100701 →	1
9	1536971-0446	Cross-brace single, SB28 ST34, purple fire		1
9	1536971-0447	Cross-brace single, SB28 ST34, emerald green		1
9	1536971-0448	Cross-brace single, SB28 ST34, midnight blue		1
9	1536971-0449	Cross-brace single, SB28 ST34, umber glaze		1
9	1536971-0450	Cross-brace single, SB28 ST34, smokey grey		1
9	1536971-3524	Cross-brace single, SB28 ST34, royal red		1
9	1536971-5969	Cross-brace single, SB28 ST34, royal blue		1
9	1536971-9005	Cross-brace single, SB28 ST34, black, glossy		1
9	1536971-9119	Cross-brace single, SB28 ST34, black, bronze	20091001 → 20100630	1
10	1536972-0411	Cross-brace single, SB30 ST34, black, matt	20100701 →	1
10	1536972-0419	Cross-brace single, SB30 ST34, silver grey		1
10	1536972-0442	Cross-brace single, SB30 ST34, night shimmer		1
10	1536972-0445	Cross-brace single, SB30 ST34, ivory white	20100701 →	1
10	1536972-0446	Cross-brace single, SB30 ST34, purple fire		1
10	1536972-0447	Cross-brace single, SB30 ST34, emerald green		1
10	1536972-0448	Cross-brace single, SB30 ST34, midnight blue		1
10	1536972-0449	Cross-brace single, SB30 ST34, umber glaze		1
10	1536972-0450	Cross-brace single, SB30 ST34, smokey grey		1
10	1536972-3524	Cross-brace single, SB30 ST34, royal red		1
10	1536972-5969	Cross-brace single, SB30 ST34, royal blue		1
10	1536972-9005	Cross-brace single, SB30 ST34, black, glossy		1
10	1536972-9119	Cross-brace single, SB30 ST34, black, bronze	20091001 → 20100630	1
11	1536973-0411	Cross-brace single, SB32 ST34, black, matt	20100701 →	1
11	1536973-0419	Cross-brace single, SB32 ST34, silver grey		1
11	1536973-0442	Cross-brace single, SB32 ST34, night shimmer		1
11	1536973-0445	Cross-brace single, SB32 ST34, ivory white	20100701 →	1
11	1536973-0446	Cross-brace single, SB32 ST34, purple fire		1
11	1536973-0447	Cross-brace single, SB32 ST34, emerald green		1
11	1536973-0448	Cross-brace single, SB32 ST34, midnight blue		1
11	1536973-0449	Cross-brace single, SB32 ST34, umber glaze		1
11	1536973-0450	Cross-brace single, SB32 ST34, smokey grey		1
11	1536973-3524	Cross-brace single, SB32 ST34, royal red		1
11	1536973-5969	Cross-brace single, SB32 ST34, royal blue		1

Cross brace single, SB28-42

Pos.	Part number	Description	Valid from → until	Qty
11	1536973-9005	Cross-brace single, SB32 ST34, black, glossy		1
11	1536973-9119	Cross-brace single, SB32 ST34, black, bronze	20091001 → 20100630	1
12	1536974-0411	Cross-brace single, SB34 ST34, black, matt	20100701 →	1
12	1536974-0419	Cross-brace single, SB34 ST34, silver grey		1
12	1536974-0442	Cross-brace single, SB34 ST34, night shimmer		1
12	1536974-0445	Cross-brace single, SB34 ST34, ivory white	20100701 →	1
12	1536974-0446	Cross-brace single, SB34 ST34, purple fire		1
12	1536974-0447	Cross-brace single, SB34 ST34, emerald green		1
12	1536974-0448	Cross-brace single, SB34 ST34, midnight blue		1
12	1536974-0449	Cross-brace single, SB34 ST34, umber glaze		1
12	1536974-0450	Cross-brace single, SB34 ST34, smokey grey		1
12	1536974-3524	Cross-brace single, SB34 ST34, royal red		1
12	1536974-5969	Cross-brace single, SB34 ST34, royal blue		1
12	1536974-9005	Cross-brace single, SB34 ST34, black, glossy		1
12	1536974-9119	Cross-brace single, SB34 ST34, black, bronze	20091001 → 20100630	1
13	1536975-0411	Cross-brace single, SB36 ST34, black, matt	20100701 →	1
13	1536975-0419	Cross-brace single, SB36 ST34, silver grey		1
13	1536975-0442	Cross-brace single, SB36 ST34, night shimmer		1
13	1536975-0445	Cross-brace single, SB36 ST34, ivory white	20100701 →	1
13	1536975-0446	Cross-brace single, SB36 ST34, purple fire		1
13	1536975-0447	Cross-brace single, SB36 ST34, emerald green		1
13	1536975-0448	Cross-brace single, SB36 ST34, midnight blue		1
13	1536975-0449	Cross-brace single, SB36 ST34, umber glaze		1
13	1536975-0450	Cross-brace single, SB36 ST34, smokey grey		1
13	1536975-3524	Cross-brace single, SB36 ST34, royal red		1
13	1536975-5969	Cross-brace single, SB36 ST34, royal blue		1
13	1536975-9005	Cross-brace single, SB36 ST34, black, glossy		1
13	1536975-9119	Cross-brace single, SB36 ST34, black, bronze	20091001 → 20100630	1
14	1536976-0411	Cross-brace single, SB38 ST34, black, matt	20100701 →	1
14	1536976-0419	Cross-brace single, SB38 ST34, silver grey		1
14	1536976-0442	Cross-brace single, SB38 ST34, night shimmer		1
14	1536976-0445	Cross-brace single, SB38 ST34, ivory white	20100701 →	1
14	1536976-0446	Cross-brace single, SB38 ST34, purple fire		1
14	1536976-0447	Cross-brace single, SB38 ST34, emerald green		1
14	1536976-0448	Cross-brace single, SB38 ST34, midnight blue		1
14	1536976-0449	Cross-brace single, SB38 ST34, umber glaze		1
14	1536976-0450	Cross-brace single, SB38 ST34, smokey grey		1
14	1536976-3524	Cross-brace single, SB38 ST34, royal red		1
14	1536976-5969	Cross-brace single, SB38 ST34, royal blue		1
14	1536976-9005	Cross-brace single, SB38 ST34, black, glossy		1
14	1536976-9119	Cross-brace single, SB38 ST34, black, bronze	20091001 → 20100630	1
15	1536977-0411	Cross-brace single, SB40 ST34, black, matt	20100701 →	1
15	1536977-0419	Cross-brace single, SB40 ST34, silver grey		1
15	1536977-0442	Cross-brace single, SB40 ST34, night shimmer		1
15	1536977-0445	Cross-brace single, SB40 ST34, ivory white	20100701 →	1
15	1536977-0446	Cross-brace single, SB40 ST34, purple fire		1
15	1536977-0447	Cross-brace single, SB40 ST34, emerald green		1
15	1536977-0448	Cross-brace single, SB40 ST34, midnight blue		1
15	1536977-0449	Cross-brace single, SB40 ST34, umber glaze		1
15	1536977-0450	Cross-brace single, SB40 ST34, smokey grey		1
15	1536977-3524	Cross-brace single, SB40 ST34, royal red		1
15	1536977-5969	Cross-brace single, SB40 ST34, royal blue		1
15	1536977-9005	Cross-brace single, SB40 ST34, black, glossy		1
15	1536977-9119	Cross-brace single, SB40 ST34, black, bronze	20091001 → 20100630	1
16	1536978-0411	Cross-brace single, SB42 ST34, black, matt	20100701 →	1

Cross brace single, SB28-42

Pos.	Part number	Description	Valid from → until	Qty
16	1536978-0419	Cross-brace single, SB42 ST34, silver grey		1
16	1536978-0442	Cross-brace single, SB42 ST34, night shimmer		1
16	1536978-0445	Cross-brace single, SB42 ST34, ivory white	20100701 →	1
16	1536978-0446	Cross-brace single, SB42 ST34, purple fire		1
16	1536978-0447	Cross-brace single, SB42 ST34, emerald green		1
16	1536978-0448	Cross-brace single, SB42 ST34, midnight blue		1
16	1536978-0449	Cross-brace single, SB42 ST34, umber glaze		1
16	1536978-0450	Cross-brace single, SB42 ST34, smokey grey		1
16	1536978-3524	Cross-brace single, SB42 ST34, royal red		1
16	1536978-5969	Cross-brace single, SB42 ST34, royal blue		1
16	1536978-9005	Cross-brace single, SB42 ST34, black, glossy		1
16	1536978-9119	Cross-brace single, SB42 ST34, black, bronze	20091001 → 20100630	1
17	1536983-0411	Cross-brace single, SB28 ST36, black, matt	20100701 →	1
17	1536983-0419	Cross-brace single, SB28 ST36, silver grey		1
17	1536983-0442	Cross-brace single, SB28 ST36, night shimmer		1
17	1536983-0445	Cross-brace single, SB28 ST36, ivory white	20100701 →	1
17	1536983-0446	Cross-brace single, SB28 ST36, purple fire		1
17	1536983-0447	Cross-brace single, SB28 ST36, emerald green		1
17	1536983-0448	Cross-brace single, SB28 ST36, midnight blue		1
17	1536983-0449	Cross-brace single, SB28 ST36, umber glaze		1
17	1536983-0450	Cross-brace single, SB28 ST36, smokey grey		1
17	1536983-3524	Cross-brace single, SB28 ST36, royal red		1
17	1536983-5969	Cross-brace single, SB28 ST36, royal blue		1
17	1536983-9005	Cross-brace single, SB28 ST36, black, glossy		1
17	1536983-9119	Cross-brace single, SB28 ST36, black, bronze	20091001 → 20100630	1
18	1536984-0411	Cross-brace single, SB30 ST36, black, matt	20100701 →	1
18	1536984-0419	Cross-brace single, SB30 ST36, silver grey		1
18	1536984-0442	Cross-brace single, SB30 ST36, night shimmer		1
18	1536984-0445	Cross-brace single, SB30 ST36, ivory white	20100701 →	1
18	1536984-0446	Cross-brace single, SB30 ST36, purple fire		1
18	1536984-0447	Cross-brace single, SB30 ST36, emerald green		1
18	1536984-0448	Cross-brace single, SB30 ST36, midnight blue		1
18	1536984-0449	Cross-brace single, SB30 ST36, umber glaze		1
18	1536984-0450	Cross-brace single, SB30 ST36, smokey grey		1
18	1536984-3524	Cross-brace single, SB30 ST36, royal red		1
18	1536984-5969	Cross-brace single, SB30 ST36, royal blue		1
18	1536984-9005	Cross-brace single, SB30 ST36, black, glossy		1
18	1536984-9119	Cross-brace single, SB30 ST36, black, bronze	20091001 → 20100630	1
19	1536985-0411	Cross-brace single, SB32 ST36, black, matt	20100701 →	1
19	1536985-0419	Cross-brace single, SB32 ST36, silver grey		1
19	1536985-0442	Cross-brace single, SB32 ST36, night shimmer		1
19	1536985-0445	Cross-brace single, SB32 ST36, ivory white	20100701 →	1
19	1536985-0446	Cross-brace single, SB32 ST36, purple fire		1
19	1536985-0447	Cross-brace single, SB32 ST36, emerald green		1
19	1536985-0448	Cross-brace single, SB32 ST36, midnight blue		1
19	1536985-0449	Cross-brace single, SB32 ST36, umber glaze		1
19	1536985-0450	Cross-brace single, SB32 ST36, smokey grey		1
19	1536985-3524	Cross-brace single, SB32 ST36, royal red		1
19	1536985-5969	Cross-brace single, SB32 ST36, royal blue		1
19	1536985-9005	Cross-brace single, SB32 ST36, black, glossy		1
19	1536985-9119	Cross-brace single, SB32 ST36, black, bronze	20091001 → 20100630	1
20	1536986-0411	Cross-brace single, SB34 ST36, black, matt	20100701 →	1
20	1536986-0419	Cross-brace single, SB34 ST36, silver grey		1
20	1536986-0442	Cross-brace single, SB34 ST36, night shimmer		1
20	1536986-0445	Cross-brace single, SB34 ST36, ivory white	20100701 →	1

Cross brace single, SB28-42

Pos.	Part number	Description	Valid from → until	Qty
20	1536986-0446	Cross-brace single, SB34 ST36, purple fire		1
20	1536986-0447	Cross-brace single, SB34 ST36, emerald green		1
20	1536986-0448	Cross-brace single, SB34 ST36, midnight blue		1
20	1536986-0449	Cross-brace single, SB34 ST36, umber glaze		1
20	1536986-0450	Cross-brace single, SB34 ST36, smokey grey		1
20	1536986-3524	Cross-brace single, SB34 ST36, royal red		1
20	1536986-5969	Cross-brace single, SB34 ST36, royal blue		1
20	1536986-9005	Cross-brace single, SB34 ST36, black, glossy		1
20	1536986-9119	Cross-brace single, SB34 ST36, black, bronze	20091001 → 20100630	1
21	1536987-0411	Cross-brace single, SB36 ST36, black, matt	20100701 →	1
21	1536987-0419	Cross-brace single, SB36 ST36, silver grey		1
21	1536987-0442	Cross-brace single, SB36 ST36, night shimmer		1
21	1536987-0445	Cross-brace single, SB36 ST36, ivory white	20100701 →	1
21	1536987-0446	Cross-brace single, SB36 ST36, purple fire		1
21	1536987-0447	Cross-brace single, SB36 ST36, emerald green		1
21	1536987-0448	Cross-brace single, SB36 ST36, midnight blue		1
21	1536987-0449	Cross-brace single, SB36 ST36, umber glaze		1
21	1536987-0450	Cross-brace single, SB36 ST36, smokey grey		1
21	1536987-3524	Cross-brace single, SB36 ST36, royal red		1
21	1536987-5969	Cross-brace single, SB36 ST36, royal blue		1
21	1536987-9005	Cross-brace single, SB36 ST36, black, glossy		1
21	1536987-9119	Cross-brace single, SB36 ST36, black, bronze	20091001 → 20100630	1
22	1536988-0411	Cross-brace single, SB38 ST36, black, matt	20100701 →	1
22	1536988-0419	Cross-brace single, SB38 ST36, silver grey		1
22	1536988-0442	Cross-brace single, SB38 ST36, night shimmer		1
22	1536988-0445	Cross-brace single, SB38 ST36, ivory white	20100701 →	1
22	1536988-0446	Cross-brace single, SB38 ST36, purple fire		1
22	1536988-0447	Cross-brace single, SB38 ST36, emerald green		1
22	1536988-0448	Cross-brace single, SB38 ST36, midnight blue		1
22	1536988-0449	Cross-brace single, SB38 ST36, umber glaze		1
22	1536988-0450	Cross-brace single, SB38 ST36, smokey grey		1
22	1536988-3524	Cross-brace single, SB38 ST36, royal red		1
22	1536988-5969	Cross-brace single, SB38 ST36, royal blue		1
22	1536988-9005	Cross-brace single, SB38 ST36, black, glossy		1
22	1536988-9119	Cross-brace single, SB38 ST36, black, bronze	20091001 → 20100630	1
23	1536989-0411	Cross-brace single, SB40 ST36, black, matt	20100701 →	1
23	1536989-0419	Cross-brace single, SB40 ST36, silver grey		1
23	1536989-0442	Cross-brace single, SB40 ST36, night shimmer		1
23	1536989-0445	Cross-brace single, SB40 ST36, ivory white	20100701 →	1
23	1536989-0446	Cross-brace single, SB40 ST36, purple fire		1
23	1536989-0447	Cross-brace single, SB40 ST36, emerald green		1
23	1536989-0448	Cross-brace single, SB40 ST36, midnight blue		1
23	1536989-0449	Cross-brace single, SB40 ST36, umber glaze		1
23	1536989-0450	Cross-brace single, SB40 ST36, smokey grey		1
23	1536989-3524	Cross-brace single, SB40 ST36, royal red		1
23	1536989-5969	Cross-brace single, SB40 ST36, royal blue		1
23	1536989-9005	Cross-brace single, SB40 ST36, black, glossy		1
23	1536989-9119	Cross-brace single, SB40 ST36, black, bronze	20091001 → 20100630	1
24	1536990-0411	Cross-brace single, SB42 ST36, black, matt	20100701 →	1
24	1536990-0419	Cross-brace single, SB42 ST36, silver grey		1
24	1536990-0442	Cross-brace single, SB42 ST36, night shimmer		1
24	1536990-0445	Cross-brace single, SB42 ST36, ivory white	20100701 →	1
24	1536990-0446	Cross-brace single, SB42 ST36, purple fire		1
24	1536990-0447	Cross-brace single, SB42 ST36, emerald green		1
24	1536990-0448	Cross-brace single, SB42 ST36, midnight blue		1

Cross brace single, SB28-42

Pos.	Part number	Description	Valid from → until	Qty
24	1536990-0449	Cross-brace single, SB42 ST36, umber glaze		1
24	1536990-0450	Cross-brace single, SB42 ST36, smokey grey		1
24	1536990-3524	Cross-brace single, SB42 ST36, royal red		1
24	1536990-5969	Cross-brace single, SB42 ST36, royal blue		1
24	1536990-9005	Cross-brace single, SB42 ST36, black, glossy		1
24	1536990-9119	Cross-brace single, SB42 ST36, black, bronze	20091001 → 20100630	1
25	1536995-0411	Cross-brace single, SB28 ST38, black, matt	20100701 →	1
25	1536995-0419	Cross-brace single, SB28 ST38, silver grey		1
25	1536995-0442	Cross-brace single, SB28 ST38, night shimmer		1
25	1536995-0445	Cross-brace single, SB28 ST38, ivory white	20100701 →	1
25	1536995-0446	Cross-brace single, SB28 ST38, purple fire		1
25	1536995-0447	Cross-brace single, SB28 ST38, emerald green		1
25	1536995-0448	Cross-brace single, SB28 ST38, midnight blue		1
25	1536995-0449	Cross-brace single, SB28 ST38, umber glaze		1
25	1536995-0450	Cross-brace single, SB28 ST38, smokey grey		1
25	1536995-3524	Cross-brace single, SB28 ST38, royal red		1
25	1536995-5969	Cross-brace single, SB28 ST38, royal blue		1
25	1536995-9005	Cross-brace single, SB28 ST38, black, glossy		1
25	1536995-9119	Cross-brace single, SB28 ST38, black, bronze	20091001 → 20100630	1
26	1536996-0411	Cross-brace single, SB30 ST38, black, matt	20100701 →	1
26	1536996-0419	Cross-brace single, SB30 ST38, silver grey		1
26	1536996-0442	Cross-brace single, SB30 ST38, night shimmer		1
26	1536996-0445	Cross-brace single, SB30 ST38, ivory white	20100701 →	1
26	1536996-0446	Cross-brace single, SB30 ST38, purple fire		1
26	1536996-0447	Cross-brace single, SB30 ST38, emerald green		1
26	1536996-0448	Cross-brace single, SB30 ST38, midnight blue		1
26	1536996-0449	Cross-brace single, SB30 ST38, umber glaze		1
26	1536996-0450	Cross-brace single, SB30 ST38, smokey grey		1
26	1536996-3524	Cross-brace single, SB30 ST38, royal red		1
26	1536996-5969	Cross-brace single, SB30 ST38, royal blue		1
26	1536996-9005	Cross-brace single, SB30 ST38, black, glossy		1
26	1536996-9119	Cross-brace single, SB30 ST38, black, bronze	20091001 → 20100630	1
27	1536997-0411	Cross-brace single, SB32 ST38, black, matt	20100701 →	1
27	1536997-0419	Cross-brace single, SB32 ST38, silver grey		1
27	1536997-0442	Cross-brace single, SB32 ST38, night shimmer		1
27	1536997-0445	Cross-brace single, SB32 ST38, ivory white	20100701 →	1
27	1536997-0446	Cross-brace single, SB32 ST38, purple fire		1
27	1536997-0447	Cross-brace single, SB32 ST38, emerald green		1
27	1536997-0448	Cross-brace single, SB32 ST38, midnight blue		1
27	1536997-0449	Cross-brace single, SB32 ST38, umber glaze		1
27	1536997-0450	Cross-brace single, SB32 ST38, smokey grey		1
27	1536997-3524	Cross-brace single, SB32 ST38, royal red		1
27	1536997-5969	Cross-brace single, SB32 ST38, royal blue		1
27	1536997-9005	Cross-brace single, SB32 ST38, black, glossy		1
27	1536997-9119	Cross-brace single, SB32 ST38, black, bronze	20091001 → 20100630	1
28	1536998-0411	Cross-brace single, SB34 ST38, black, matt	20100701 →	1
28	1536998-0419	Cross-brace single, SB34 ST38, silver grey		1
28	1536998-0442	Cross-brace single, SB34 ST38, night shimmer		1
28	1536998-0445	Cross-brace single, SB34 ST38, ivory white	20100701 →	1
28	1536998-0446	Cross-brace single, SB34 ST38, purple fire		1
28	1536998-0447	Cross-brace single, SB34 ST38, emerald green		1
28	1536998-0448	Cross-brace single, SB34 ST38, midnight blue		1
28	1536998-0449	Cross-brace single, SB34 ST38, umber glaze		1
28	1536998-0450	Cross-brace single, SB34 ST38, smokey grey		1
28	1536998-3524	Cross-brace single, SB34 ST38, royal red		1

Cross brace single, SB28-42

Pos.	Part number	Description	Valid from → until	Qty
28	1536998-5969	Cross-brace single, SB34 ST38, royal blue		1
28	1536998-9005	Cross-brace single, SB34 ST38, black, glossy		1
28	1536998-9119	Cross-brace single, SB34 ST38, black, bronze	20091001 → 20100630	1
29	1536999-0411	Cross-brace single, SB36 ST38, black, matt	20100701 →	1
29	1536999-0419	Cross-brace single, SB36 ST38, silver grey		1
29	1536999-0442	Cross-brace single, SB36 ST38, night shimmer		1
29	1536999-0445	Cross-brace single, SB36 ST38, ivory white	20100701 →	1
29	1536999-0446	Cross-brace single, SB36 ST38, purple fire		1
29	1536999-0447	Cross-brace single, SB36 ST38, emerald green		1
29	1536999-0448	Cross-brace single, SB36 ST38, midnight blue		1
29	1536999-0449	Cross-brace single, SB36 ST38, umber glaze		1
29	1536999-0450	Cross-brace single, SB36 ST38, smokey grey		1
29	1536999-3524	Cross-brace single, SB36 ST38, royal red		1
29	1536999-5969	Cross-brace single, SB36 ST38, royal blue		1
29	1536999-9005	Cross-brace single, SB36 ST38, black, glossy		1
29	1536999-9119	Cross-brace single, SB36 ST38, black, bronze	20091001 → 20100630	1
30	1537000-0411	Cross-brace single, SB38 ST38, black, matt	20100701 →	1
30	1537000-0419	Cross-brace single, SB38 ST38, silver grey		1
30	1537000-0442	Cross-brace single, SB38 ST38, night shimmer		1
30	1537000-0445	Cross-brace single, SB38 ST38, ivory white	20100701 →	1
30	1537000-0446	Cross-brace single, SB38 ST38, purple fire		1
30	1537000-0447	Cross-brace single, SB38 ST38, emerald green		1
30	1537000-0448	Cross-brace single, SB38 ST38, midnight blue		1
30	1537000-0449	Cross-brace single, SB38 ST38, umber glaze		1
30	1537000-0450	Cross-brace single, SB38 ST38, smokey grey		1
30	1537000-3524	Cross-brace single, SB38 ST38, royal red		1
30	1537000-5969	Cross-brace single, SB38 ST38, royal blue		1
30	1537000-9005	Cross-brace single, SB38 ST38, black, glossy		1
30	1537000-9119	Cross-brace single, SB38 ST38, black, bronze	20091001 → 20100630	1
31	1537001-0411	Cross-brace single, SB40 ST38, black, matt	20100701 →	1
31	1537001-0419	Cross-brace single, SB40 ST38, silver grey		1
31	1537001-0442	Cross-brace single, SB40 ST38, night shimmer		1
31	1537001-0445	Cross-brace single, SB40 ST38, ivory white	20100701 →	1
31	1537001-0446	Cross-brace single, SB40 ST38, purple fire		1
31	1537001-0447	Cross-brace single, SB40 ST38, emerald green		1
31	1537001-0448	Cross-brace single, SB40 ST38, midnight blue		1
31	1537001-0449	Cross-brace single, SB40 ST38, umber glaze		1
31	1537001-0450	Cross-brace single, SB40 ST38, smokey grey		1
31	1537001-3524	Cross-brace single, SB40 ST38, royal red		1
31	1537001-5969	Cross-brace single, SB40 ST38, royal blue		1
31	1537001-9005	Cross-brace single, SB40 ST38, black, glossy		1
31	1537001-9119	Cross-brace single, SB40 ST38, black, bronze	20091001 → 20100630	1
32	1537002-0411	Cross-brace single, SB42 ST38, black, matt	20100701 →	1
32	1537002-0419	Cross-brace single, SB42 ST38, silver grey		1
32	1537002-0442	Cross-brace single, SB42 ST38, night shimmer		1
32	1537002-0445	Cross-brace single, SB42 ST38, ivory white	20100701 →	1
32	1537002-0446	Cross-brace single, SB42 ST38, purple fire		1
32	1537002-0447	Cross-brace single, SB42 ST38, emerald green		1
32	1537002-0448	Cross-brace single, SB42 ST38, midnight blue		1
32	1537002-0449	Cross-brace single, SB42 ST38, umber glaze		1
32	1537002-0450	Cross-brace single, SB42 ST38, smokey grey		1
32	1537002-3524	Cross-brace single, SB42 ST38, royal red		1
32	1537002-5969	Cross-brace single, SB42 ST38, royal blue		1
32	1537002-9005	Cross-brace single, SB42 ST38, black, glossy		1
32	1537002-9119	Cross-brace single, SB42 ST38, black, bronze	20091001 → 20100630	1

Cross brace single, SB28-42

Pos.	Part number	Description	Valid from → until	Qty
33	1537007-0411	Cross-brace single, SB28 ST40, black, matt	20100701 →	1
33	1537007-0419	Cross-brace single, SB28 ST40, silver grey		1
33	1537007-0442	Cross-brace single, SB28 ST40, night shimmer		1
33	1537007-0445	Cross-brace single, SB28 ST40, ivory white	20100701 →	1
33	1537007-0446	Cross-brace single, SB28 ST40, purple fire		1
33	1537007-0447	Cross-brace single, SB28 ST40, emerald green		1
33	1537007-0448	Cross-brace single, SB28 ST40, midnight blue		1
33	1537007-0449	Cross-brace single, SB28 ST40, umber glaze		1
33	1537007-0450	Cross-brace single, SB28 ST40, smokey grey		1
33	1537007-3524	Cross-brace single, SB28 ST40, royal red		1
33	1537007-5969	Cross-brace single, SB28 ST40, royal blue		1
33	1537007-9005	Cross-brace single, SB28 ST40, black, glossy		1
33	1537007-9119	Cross-brace single, SB28 ST40, black, bronze	20091001 → 20100630	1
34	1537008-0411	Cross-brace single, SB30 ST40, black, matt	20100701 →	1
34	1537008-0419	Cross-brace single, SB30 ST40, silver grey		1
34	1537008-0442	Cross-brace single, SB30 ST40, night shimmer		1
34	1537008-0445	Cross-brace single, SB30 ST40, ivory white	20100701 →	1
34	1537008-0446	Cross-brace single, SB30 ST40, purple fire		1
34	1537008-0447	Cross-brace single, SB30 ST40, emerald green		1
34	1537008-0448	Cross-brace single, SB30 ST40, midnight blue		1
34	1537008-0449	Cross-brace single, SB30 ST40, umber glaze		1
34	1537008-0450	Cross-brace single, SB30 ST40, smokey grey		1
34	1537008-3524	Cross-brace single, SB30 ST40, royal red		1
34	1537008-5969	Cross-brace single, SB30 ST40, royal blue		1
34	1537008-9005	Cross-brace single, SB30 ST40, black, glossy		1
34	1537008-9119	Cross-brace single, SB30 ST40, black, bronze	20091001 → 20100630	1
35	1537009-0411	Cross-brace single, SB32 ST40, black, matt	20100701 →	1
35	1537009-0419	Cross-brace single, SB32 ST40, silver grey		1
35	1537009-0442	Cross-brace single, SB32 ST40, night shimmer		1
35	1537009-0445	Cross-brace single, SB32 ST40, ivory white	20100701 →	1
35	1537009-0446	Cross-brace single, SB32 ST40, purple fire		1
35	1537009-0447	Cross-brace single, SB32 ST40, emerald green		1
35	1537009-0448	Cross-brace single, SB32 ST40, midnight blue		1
35	1537009-0449	Cross-brace single, SB32 ST40, umber glaze		1
35	1537009-0450	Cross-brace single, SB32 ST40, smokey grey		1
35	1537009-3524	Cross-brace single, SB32 ST40, royal red		1
35	1537009-5969	Cross-brace single, SB32 ST40, royal blue		1
35	1537009-9005	Cross-brace single, SB32 ST40, black, glossy		1
35	1537009-9119	Cross-brace single, SB32 ST40, black, bronze	20091001 → 20100630	1
36	1537010-0411	Cross-brace single, SB34 ST40, black, matt	20100701 →	1
36	1537010-0419	Cross-brace single, SB34 ST40, silver grey		1
36	1537010-0442	Cross-brace single, SB34 ST40, night shimmer		1
36	1537010-0445	Cross-brace single, SB34 ST40, ivory white	20100701 →	1
36	1537010-0446	Cross-brace single, SB34 ST40, purple fire		1
36	1537010-0447	Cross-brace single, SB34 ST40, emerald green		1
36	1537010-0448	Cross-brace single, SB34 ST40, midnight blue		1
36	1537010-0449	Cross-brace single, SB34 ST40, umber glaze		1
36	1537010-0450	Cross-brace single, SB34 ST40, smokey grey		1
36	1537010-3524	Cross-brace single, SB34 ST40, royal red		1
36	1537010-5969	Cross-brace single, SB34 ST40, royal blue		1
36	1537010-9005	Cross-brace single, SB34 ST40, black, glossy		1
36	1537010-9119	Cross-brace single, SB34 ST40, black, bronze	20091001 → 20100630	1
37	1537011-0411	Cross-brace single, SB36 ST40, black, matt	20100701 →	1
37	1537011-0419	Cross-brace single, SB36 ST40, silver grey		1
37	1537011-0442	Cross-brace single, SB36 ST40, night shimmer		1

Cross brace single, SB28-42

Pos.	Part number	Description	Valid from → until	Qty
37	1537011-0445	Cross-brace single, SB36 ST40, ivory white	20100701 →	1
37	1537011-0446	Cross-brace single, SB36 ST40, purple fire		1
37	1537011-0447	Cross-brace single, SB36 ST40, emerald green		1
37	1537011-0448	Cross-brace single, SB36 ST40, midnight blue		1
37	1537011-0449	Cross-brace single, SB36 ST40, umber glaze		1
37	1537011-0450	Cross-brace single, SB36 ST40, smokey grey		1
37	1537011-3524	Cross-brace single, SB36 ST40, royal red		1
37	1537011-5969	Cross-brace single, SB36 ST40, royal blue		1
37	1537011-9005	Cross-brace single, SB36 ST40, black, glossy		1
37	1537011-9119	Cross-brace single, SB36 ST40, black, bronze	20091001 → 20100630	1
38	1537012-0411	Cross-brace single, SB38 ST40, black, matt	20100701 →	1
38	1537012-0419	Cross-brace single, SB38 ST40, silver grey		1
38	1537012-0442	Cross-brace single, SB38 ST40, night shimmer		1
38	1537012-0445	Cross-brace single, SB38 ST40, ivory white	20100701 →	1
38	1537012-0446	Cross-brace single, SB38 ST40, purple fire		1
38	1537012-0447	Cross-brace single, SB38 ST40, emerald green		1
38	1537012-0448	Cross-brace single, SB38 ST40, midnight blue		1
38	1537012-0449	Cross-brace single, SB38 ST40, umber glaze		1
38	1537012-0450	Cross-brace single, SB38 ST40, smokey grey		1
38	1537012-3524	Cross-brace single, SB38 ST40, royal red		1
38	1537012-5969	Cross-brace single, SB38 ST40, royal blue		1
38	1537012-9005	Cross-brace single, SB38 ST40, black, glossy		1
38	1537012-9119	Cross-brace single, SB38 ST40, black, bronze	20091001 → 20100630	1
39	1537013-0411	Cross-brace single, SB40 ST40, black, matt	20100701 →	1
39	1537013-0419	Cross-brace single, SB40 ST40, silver grey		1
39	1537013-0442	Cross-brace single, SB40 ST40, night shimmer		1
39	1537013-0445	Cross-brace single, SB40 ST40, ivory white	20100701 →	1
39	1537013-0446	Cross-brace single, SB40 ST40, purple fire		1
39	1537013-0447	Cross-brace single, SB40 ST40, emerald green		1
39	1537013-0448	Cross-brace single, SB40 ST40, midnight blue		1
39	1537013-0449	Cross-brace single, SB40 ST40, umber glaze		1
39	1537013-0450	Cross-brace single, SB40 ST40, smokey grey		1
39	1537013-3524	Cross-brace single, SB40 ST40, royal red		1
39	1537013-5969	Cross-brace single, SB40 ST40, royal blue		1
39	1537013-9005	Cross-brace single, SB40 ST40, black, glossy		1
39	1537013-9119	Cross-brace single, SB40 ST40, black, bronze	20091001 → 20100630	1
40	1537014-0411	Cross-brace single, SB42 ST40, black, matt	20100701 →	1
40	1537014-0419	Cross-brace single, SB42 ST40, silver grey		1
40	1537014-0442	Cross-brace single, SB42 ST40, night shimmer		1
40	1537014-0445	Cross-brace single, SB42 ST40, ivory white	20100701 →	1
40	1537014-0446	Cross-brace single, SB42 ST40, purple fire		1
40	1537014-0447	Cross-brace single, SB42 ST40, emerald green		1
40	1537014-0448	Cross-brace single, SB42 ST40, midnight blue		1
40	1537014-0449	Cross-brace single, SB42 ST40, umber glaze		1
40	1537014-0450	Cross-brace single, SB42 ST40, smokey grey		1
40	1537014-3524	Cross-brace single, SB42 ST40, royal red		1
40	1537014-5969	Cross-brace single, SB42 ST40, royal blue		1
40	1537014-9005	Cross-brace single, SB42 ST40, black, glossy		1
40	1537014-9119	Cross-brace single, SB42 ST40, black, bronze	20091001 → 20100630	1
41	1537019-0411	Cross-brace single, SB28 ST42, black, matt	20100701 →	1
41	1537019-0419	Cross-brace single, SB28 ST42, silver grey		1
41	1537019-0442	Cross-brace single, SB28 ST42, night shimmer		1
41	1537019-0445	Cross-brace single, SB28 ST42, ivory white	20100701 →	1
41	1537019-0446	Cross-brace single, SB28 ST42, purple fire		1
41	1537019-0447	Cross-brace single, SB28 ST42, emerald green		1

Cross brace single, SB28-42

Pos.	Part number	Description	Valid from → until	Qty
41	1537019-0448	Cross-brace single, SB28 ST42, midnight blue		1
41	1537019-0449	Cross-brace single, SB28 ST42, umber glaze		1
41	1537019-0450	Cross-brace single, SB28 ST42, smokey grey		1
41	1537019-3524	Cross-brace single, SB28 ST42, royal red		1
41	1537019-5969	Cross-brace single, SB28 ST42, royal blue		1
41	1537019-9005	Cross-brace single, SB28 ST42, black, glossy		1
41	1537019-9119	Cross-brace single, SB28 ST42, black, bronze	20091001 → 20100630	1
42	1537020-0411	Cross-brace single, SB30 ST42, black, matt	20100701 →	1
42	1537020-0419	Cross-brace single, SB30 ST42, silver grey		1
42	1537020-0442	Cross-brace single, SB30 ST42, night shimmer		1
42	1537020-0445	Cross-brace single, SB30 ST42, ivory white	20100701 →	1
42	1537020-0446	Cross-brace single, SB30 ST42, purple fire		1
42	1537020-0447	Cross-brace single, SB30 ST42, emerald green		1
42	1537020-0448	Cross-brace single, SB30 ST42, midnight blue		1
42	1537020-0449	Cross-brace single, SB30 ST42, umber glaze		1
42	1537020-0450	Cross-brace single, SB30 ST42, smokey grey		1
42	1537020-3524	Cross-brace single, SB30 ST42, royal red		1
42	1537020-5969	Cross-brace single, SB30 ST42, royal blue		1
42	1537020-9005	Cross-brace single, SB30 ST42, black, glossy		1
42	1537020-9119	Cross-brace single, SB30 ST42, black, bronze	20091001 → 20100630	1
43	1537021-0411	Cross-brace single, SB32 ST42, black, matt	20100701 →	1
43	1537021-0419	Cross-brace single, SB32 ST42, silver grey		1
43	1537021-0442	Cross-brace single, SB32 ST42, night shimmer		1
43	1537021-0445	Cross-brace single, SB32 ST42, ivory white	20100701 →	1
43	1537021-0446	Cross-brace single, SB32 ST42, purple fire		1
43	1537021-0447	Cross-brace single, SB32 ST42, emerald green		1
43	1537021-0448	Cross-brace single, SB32 ST42, midnight blue		1
43	1537021-0449	Cross-brace single, SB32 ST42, umber glaze		1
43	1537021-0450	Cross-brace single, SB32 ST42, smokey grey		1
43	1537021-3524	Cross-brace single, SB32 ST42, royal red		1
43	1537021-5969	Cross-brace single, SB32 ST42, royal blue		1
43	1537021-9005	Cross-brace single, SB32 ST42, black, glossy		1
43	1537021-9119	Cross-brace single, SB32 ST42, black, bronze	20091001 → 20100630	1
44	1537022-0411	Cross-brace single, SB34 ST42, black, matt	20100701 →	1
44	1537022-0419	Cross-brace single, SB34 ST42, silver grey		1
44	1537022-0442	Cross-brace single, SB34 ST42, night shimmer		1
44	1537022-0445	Cross-brace single, SB34 ST42, ivory white	20100701 →	1
44	1537022-0446	Cross-brace single, SB34 ST42, purple fire		1
44	1537022-0447	Cross-brace single, SB34 ST42, emerald green		1
44	1537022-0448	Cross-brace single, SB34 ST42, midnight blue		1
44	1537022-0449	Cross-brace single, SB34 ST42, umber glaze		1
44	1537022-0450	Cross-brace single, SB34 ST42, smokey grey		1
44	1537022-3524	Cross-brace single, SB34 ST42, royal red		1
44	1537022-5969	Cross-brace single, SB34 ST42, royal blue		1
44	1537022-9005	Cross-brace single, SB34 ST42, black, glossy		1
44	1537022-9119	Cross-brace single, SB34 ST42, black, bronze	20091001 → 20100630	1
45	1537023-0411	Cross-brace single, SB36 ST42, black, matt	20100701 →	1
45	1537023-0419	Cross-brace single, SB36 ST42, silver grey		1
45	1537023-0442	Cross-brace single, SB36 ST42, night shimmer		1
45	1537023-0445	Cross-brace single, SB36 ST42, ivory white	20100701 →	1
45	1537023-0446	Cross-brace single, SB36 ST42, purple fire		1
45	1537023-0447	Cross-brace single, SB36 ST42, emerald green		1
45	1537023-0448	Cross-brace single, SB36 ST42, midnight blue		1
45	1537023-0449	Cross-brace single, SB36 ST42, umber glaze		1
45	1537023-0450	Cross-brace single, SB36 ST42, smokey grey		1

Cross brace single, SB28-42

Pos.	Part number	Description	Valid from → until	Qty
45	1537023-3524	Cross-brace single, SB36 ST42, royal red		1
45	1537023-5969	Cross-brace single, SB36 ST42, royal blue		1
45	1537023-9005	Cross-brace single, SB36 ST42, black, glossy		1
45	1537023-9119	Cross-brace single, SB36 ST42, black, bronze	20091001 → 20100630	1
46	1537024-0411	Cross-brace single, SB38 ST42, black, matt	20100701 →	1
46	1537024-0419	Cross-brace single, SB38 ST42, silver grey		1
46	1537024-0442	Cross-brace single, SB38 ST42, night shimmer		1
46	1537024-0445	Cross-brace single, SB38 ST42, ivory white	20100701 →	1
46	1537024-0446	Cross-brace single, SB38 ST42, purple fire		1
46	1537024-0447	Cross-brace single, SB38 ST42, emerald green		1
46	1537024-0448	Cross-brace single, SB38 ST42, midnight blue		1
46	1537024-0449	Cross-brace single, SB38 ST42, umber glaze		1
46	1537024-0450	Cross-brace single, SB38 ST42, smokey grey		1
46	1537024-3524	Cross-brace single, SB38 ST42, royal red		1
46	1537024-5969	Cross-brace single, SB38 ST42, royal blue		1
46	1537024-9005	Cross-brace single, SB38 ST42, black, glossy		1
46	1537024-9119	Cross-brace single, SB38 ST42, black, bronze	20091001 → 20100630	1
47	1537025-0411	Cross-brace single, SB40 ST42, black, matt	20100701 →	1
47	1537025-0419	Cross-brace single, SB40 ST42, silver grey		1
47	1537025-0442	Cross-brace single, SB40 ST42, night shimmer		1
47	1537025-0445	Cross-brace single, SB40 ST42, ivory white	20100701 →	1
47	1537025-0446	Cross-brace single, SB40 ST42, purple fire		1
47	1537025-0447	Cross-brace single, SB40 ST42, emerald green		1
47	1537025-0448	Cross-brace single, SB40 ST42, midnight blue		1
47	1537025-0449	Cross-brace single, SB40 ST42, umber glaze		1
47	1537025-0450	Cross-brace single, SB40 ST42, smokey grey		1
47	1537025-3524	Cross-brace single, SB40 ST42, royal red		1
47	1537025-5969	Cross-brace single, SB40 ST42, royal blue		1
47	1537025-9005	Cross-brace single, SB40 ST42, black, glossy		1
47	1537025-9119	Cross-brace single, SB40 ST42, black, bronze	20091001 → 20100630	1
48	1537026-0411	Cross-brace single, SB42 ST42, black, matt	20100701 →	1
48	1537026-0419	Cross-brace single, SB42 ST42, silver grey		1
48	1537026-0442	Cross-brace single, SB42 ST42, night shimmer		1
48	1537026-0445	Cross-brace single, SB42 ST42, ivory white	20100701 →	1
48	1537026-0446	Cross-brace single, SB42 ST42, purple fire		1
48	1537026-0447	Cross-brace single, SB42 ST42, emerald green		1
48	1537026-0448	Cross-brace single, SB42 ST42, midnight blue		1
48	1537026-0449	Cross-brace single, SB42 ST42, umber glaze		1
48	1537026-0450	Cross-brace single, SB42 ST42, smokey grey		1
48	1537026-3524	Cross-brace single, SB42 ST42, royal red		1
48	1537026-5969	Cross-brace single, SB42 ST42, royal blue		1
48	1537026-9005	Cross-brace single, SB42 ST42, black, glossy		1
48	1537026-9119	Cross-brace single, SB42 ST42, black, bronze	20091001 → 20100630	1
49	1537031-0411	Cross-brace single, SB28 ST44, black, matt	20100701 →	1
49	1537031-0419	Cross-brace single, SB28 ST44, silver grey		1
49	1537031-0442	Cross-brace single, SB28 ST44, night shimmer		1
49	1537031-0445	Cross-brace single, SB28 ST44, ivory white	20100701 →	1
49	1537031-0446	Cross-brace single, SB28 ST44, purple fire		1
49	1537031-0447	Cross-brace single, SB28 ST44, emerald green		1
49	1537031-0448	Cross-brace single, SB28 ST44, midnight blue		1
49	1537031-0449	Cross-brace single, SB28 ST44, umber glaze		1
49	1537031-0450	Cross-brace single, SB28 ST44, smokey grey		1
49	1537031-3524	Cross-brace single, SB28 ST44, royal red		1
49	1537031-5969	Cross-brace single, SB28 ST44, royal blue		1
49	1537031-9005	Cross-brace single, SB28 ST44, black, glossy		1

Cross brace single, SB28-42

Pos.	Part number	Description	Valid from → until	Qty
49	1537031-9119	Cross-brace single, SB28 ST44, black, bronze	20091001 → 20100630	1
50	1537032-0411	Cross-brace single, SB30 ST44, black, matt	20100701 →	1
50	1537032-0419	Cross-brace single, SB30 ST44, silver grey		1
50	1537032-0442	Cross-brace single, SB30 ST44, night shimmer		1
50	1537032-0445	Cross-brace single, SB30 ST44, ivory white	20100701 →	1
50	1537032-0446	Cross-brace single, SB30 ST44, purple fire		1
50	1537032-0447	Cross-brace single, SB30 ST44, emerald green		1
50	1537032-0448	Cross-brace single, SB30 ST44, midnight blue		1
50	1537032-0449	Cross-brace single, SB30 ST44, umber glaze		1
50	1537032-0450	Cross-brace single, SB30 ST44, smokey grey		1
50	1537032-3524	Cross-brace single, SB30 ST44, royal red		1
50	1537032-5969	Cross-brace single, SB30 ST44, royal blue		1
50	1537032-9005	Cross-brace single, SB30 ST44, black, glossy		1
50	1537032-9119	Cross-brace single, SB30 ST44, black, bronze	20091001 → 20100630	1
51	1537033-0411	Cross-brace single, SB32 ST44, black, matt	20100701 →	1
51	1537033-0419	Cross-brace single, SB32 ST44, silver grey		1
51	1537033-0442	Cross-brace single, SB32 ST44, night shimmer		1
51	1537033-0445	Cross-brace single, SB32 ST44, ivory white	20100701 →	1
51	1537033-0446	Cross-brace single, SB32 ST44, purple fire		1
51	1537033-0447	Cross-brace single, SB32 ST44, emerald green		1
51	1537033-0448	Cross-brace single, SB32 ST44, midnight blue		1
51	1537033-0449	Cross-brace single, SB32 ST44, umber glaze		1
51	1537033-0450	Cross-brace single, SB32 ST44, smokey grey		1
51	1537033-3524	Cross-brace single, SB32 ST44, royal red		1
51	1537033-5969	Cross-brace single, SB32 ST44, royal blue		1
51	1537033-9005	Cross-brace single, SB32 ST44, black, glossy		1
51	1537033-9119	Cross-brace single, SB32 ST44, black, bronze	20091001 → 20100630	1
52	1537034-0411	Cross-brace single, SB34 ST44, black, matt	20100701 →	1
52	1537034-0419	Cross-brace single, SB34 ST44, silver grey		1
52	1537034-0442	Cross-brace single, SB34 ST44, night shimmer		1
52	1537034-0445	Cross-brace single, SB34 ST44, ivory white	20100701 →	1
52	1537034-0446	Cross-brace single, SB34 ST44, purple fire		1
52	1537034-0447	Cross-brace single, SB34 ST44, emerald green		1
52	1537034-0448	Cross-brace single, SB34 ST44, midnight blue		1
52	1537034-0449	Cross-brace single, SB34 ST44, umber glaze		1
52	1537034-0450	Cross-brace single, SB34 ST44, smokey grey		1
52	1537034-3524	Cross-brace single, SB34 ST44, royal red		1
52	1537034-5969	Cross-brace single, SB34 ST44, royal blue		1
52	1537034-9005	Cross-brace single, SB34 ST44, black, glossy		1
52	1537034-9119	Cross-brace single, SB34 ST44, black, bronze	20091001 → 20100630	1
53	1537035-0411	Cross-brace single, SB36 ST44, black, matt	20100701 →	1
53	1537035-0419	Cross-brace single, SB36 ST44, silver grey		1
53	1537035-0442	Cross-brace single, SB36 ST44, night shimmer		1
53	1537035-0445	Cross-brace single, SB36 ST44, ivory white	20100701 →	1
53	1537035-0446	Cross-brace single, SB36 ST44, purple fire		1
53	1537035-0447	Cross-brace single, SB36 ST44, emerald green		1
53	1537035-0448	Cross-brace single, SB36 ST44, midnight blue		1
53	1537035-0449	Cross-brace single, SB36 ST44, umber glaze		1
53	1537035-0450	Cross-brace single, SB36 ST44, smokey grey		1
53	1537035-3524	Cross-brace single, SB36 ST44, royal red		1
53	1537035-5969	Cross-brace single, SB36 ST44, royal blue		1
53	1537035-9005	Cross-brace single, SB36 ST44, black, glossy		1
53	1537035-9119	Cross-brace single, SB36 ST44, black, bronze	20091001 → 20100630	1
54	1537036-0411	Cross-brace single, SB38 ST44, black, matt	20100701 →	1
54	1537036-0419	Cross-brace single, SB38 ST44, silver grey		1

Cross brace single, SB28-42

Pos.	Part number	Description	Valid from → until	Qty
54	1537036-0442	Cross-brace single, SB38 ST44, night shimmer		1
54	1537036-0445	Cross-brace single, SB38 ST44, ivory white	20100701 →	1
54	1537036-0446	Cross-brace single, SB38 ST44, purple fire		1
54	1537036-0447	Cross-brace single, SB38 ST44, emerald green		1
54	1537036-0448	Cross-brace single, SB38 ST44, midnight blue		1
54	1537036-0449	Cross-brace single, SB38 ST44, umber glaze		1
54	1537036-0450	Cross-brace single, SB38 ST44, smokey grey		1
54	1537036-3524	Cross-brace single, SB38 ST44, royal red		1
54	1537036-5969	Cross-brace single, SB38 ST44, royal blue		1
54	1537036-9005	Cross-brace single, SB38 ST44, black, glossy		1
54	1537036-9119	Cross-brace single, SB38 ST44, black, bronze	20091001 → 20100630	1
55	1537037-0411	Cross-brace single, SB40 ST44, black matt	20100701 →	1
55	1537037-0419	Cross-brace single, SB40 ST44, silver grey		1
55	1537037-0442	Cross-brace single, SB40 ST44, night shimmer		1
55	1537037-0445	Cross-brace single, SB40 ST44, ivory white	20100701 →	1
55	1537037-0446	Cross-brace single, SB40 ST44, purple fire		1
55	1537037-0447	Cross-brace single, SB40 ST44, emerald green		1
55	1537037-0448	Cross-brace single, SB40 ST44, midnight blue		1
55	1537037-0449	Cross-brace single, SB40 ST44, umber glaze		1
55	1537037-0450	Cross-brace single, SB40 ST44, smokey grey		1
55	1537037-3524	Cross-brace single, SB40 ST44, royal red		1
55	1537037-5969	Cross-brace single, SB40 ST44, royal blue		1
55	1537037-9005	Cross-brace single, SB40 ST44, black, glossy		1
55	1537037-9119	Cross-brace single, SB40 ST44, black, bronze	20091001 → 20100630	1
56	1537038-0411	Cross-brace single, SB42 ST44, black, matt	20100701 →	1
56	1537038-0419	Cross-brace single, SB42 ST44, silver grey		1
56	1537038-0442	Cross-brace single, SB42 ST44, night shimmer		1
56	1537038-0445	Cross-brace single, SB42 ST44, ivory white	20100701 →	1
56	1537038-0446	Cross-brace single, SB42 ST44, purple fire		1
56	1537038-0447	Cross-brace single, SB42 ST44, emerald green		1
56	1537038-0448	Cross-brace single, SB42 ST44, midnight blue		1
56	1537038-0449	Cross-brace single, SB42 ST44, umber glaze		1
56	1537038-0450	Cross-brace single, SB42 ST44, smokey grey		1
56	1537038-3524	Cross-brace single, SB42 ST44, royal red		1
56	1537038-5969	Cross-brace single, SB42 ST44, royal blue		1
56	1537038-9005	Cross-brace single, SB42 ST44, black, glossy		1
56	1537038-9119	Cross-brace single, SB42 ST44, black, bronze	20091001 → 20100630	1
57	1537043-0411	Cross-brace single, SB28 ST46, black, matt	20100701 →	1
57	1537043-0419	Cross-brace single, SB28 ST46, silver grey		1
57	1537043-0442	Cross-brace single, SB28 ST46, night shimmer		1
57	1537043-0445	Cross-brace single, SB28 ST46, ivory white	20100701 →	1
57	1537043-0446	Cross-brace single, SB28 ST46, purple fire		1
57	1537043-0447	Cross-brace single, SB28 ST46, emerald green		1
57	1537043-0448	Cross-brace single, SB28 ST46, midnight blue		1
57	1537043-0449	Cross-brace single, SB28 ST46, umber glaze		1
57	1537043-0450	Cross-brace single, SB28 ST46, smokey grey		1
57	1537043-3524	Cross-brace single, SB28 ST46, royal red		1
57	1537043-5969	Cross-brace single, SB28 ST46, royal blue		1
57	1537043-9005	Cross-brace single, SB28 ST46, black, glossy		1
57	1537043-9119	Cross-brace single, SB28 ST46, black, bronze	20091001 → 20100630	1
58	1537044-0411	Cross-brace single, SB30 ST46, black, matt	20100701 →	1
58	1537044-0419	Cross-brace single, SB30 ST46, silver grey		1
58	1537044-0442	Cross-brace single, SB30 ST46, night shimmer		1
58	1537044-0445	Cross-brace single, SB30 ST46, ivory white	20100701 →	1
58	1537044-0446	Cross-brace single, SB30 ST46, purple fire		1

Cross brace single, SB28-42

Pos.	Part number	Description	Valid from → until	Qty
58	1537044-0447	Cross-brace single, SB30 ST46, emerald green		1
58	1537044-0448	Cross-brace single, SB30 ST46, midnight blue		1
58	1537044-0449	Cross-brace single, SB30 ST46, umber glaze		1
58	1537044-0450	Cross-brace single, SB30 ST46, smokey grey		1
58	1537044-3524	Cross-brace single, SB30 ST46, royal red		1
58	1537044-5969	Cross-brace single, SB30 ST46, royal blue		1
58	1537044-9005	Cross-brace single, SB30 ST46, black, glossy		1
58	1537044-9119	Cross-brace single, SB30 ST46, black, bronze	20091001 → 20100630	1
59	1537045-0411	Cross-brace single, SB32 ST46, black, matt	20100701 →	1
59	1537045-0419	Cross-brace single, SB32 ST46, silver grey		1
59	1537045-0442	Cross-brace single, SB32 ST46, night shimmer		1
59	1537045-0445	Cross-brace single, SB32 ST46, ivory white	20100701 →	1
59	1537045-0446	Cross-brace single, SB32 ST46, purple fire		1
59	1537045-0447	Cross-brace single, SB32 ST46, emerald green		1
59	1537045-0448	Cross-brace single, SB32 ST46, midnight blue		1
59	1537045-0449	Cross-brace single, SB32 ST46, umber glaze		1
59	1537045-0450	Cross-brace single, SB32 ST46, smokey grey		1
59	1537045-3524	Cross-brace single, SB32 ST46, royal red		1
59	1537045-5969	Cross-brace single, SB32 ST46, royal blue		1
59	1537045-9005	Cross-brace single, SB32 ST46, black, glossy		1
59	1537045-9119	Cross-brace single, SB32 ST46, black, bronze	20091001 → 20100630	1
60	1537046-0411	Cross-brace single, SB34 ST46, black, matt	20100701 →	1
60	1537046-0419	Cross-brace single, SB34 ST46, silver grey		1
60	1537046-0442	Cross-brace single, SB34 ST46, night shimmer		1
60	1537046-0445	Cross-brace single, SB34 ST46, ivory white	20100701 →	1
60	1537046-0446	Cross-brace single, SB34 ST46, purple fire		1
60	1537046-0447	Cross-brace single, SB34 ST46, emerald green		1
60	1537046-0448	Cross-brace single, SB34 ST46, midnight blue		1
60	1537046-0449	Cross-brace single, SB34 ST46, umber glaze		1
60	1537046-0450	Cross-brace single, SB34 ST46, smokey grey		1
60	1537046-3524	Cross-brace single, SB34 ST46, royal red		1
60	1537046-5969	Cross-brace single, SB34 ST46, royal blue		1
60	1537046-9005	Cross-brace single, SB34 ST46, black, glossy		1
60	1537046-9119	Cross-brace single, SB34 ST46, black, bronze	20091001 → 20100630	1
61	1537047-0411	Cross-brace single, SB36 ST46, black, matt	20100701 →	1
61	1537047-0419	Cross-brace single, SB36 ST46, silver grey		1
61	1537047-0442	Cross-brace single, SB36 ST46, night shimmer		1
61	1537047-0445	Cross-brace single, SB36 ST46, ivory white	20100701 →	1
61	1537047-0446	Cross-brace single, SB36 ST46, purple fire		1
61	1537047-0447	Cross-brace single, SB36 ST46, emerald green		1
61	1537047-0448	Cross-brace single, SB36 ST46, midnight blue		1
61	1537047-0449	Cross-brace single, SB36 ST46, umber glaze		1
61	1537047-0450	Cross-brace single, SB36 ST46, smokey grey		1
61	1537047-3524	Cross-brace single, SB36 ST46, royal red		1
61	1537047-5969	Cross-brace single, SB36 ST46, royal blue		1
61	1537047-9005	Cross-brace single, SB36 ST46, black, glossy		1
61	1537047-9119	Cross-brace single, SB36 ST46, black, bronze	20091001 → 20100630	1
62	1537048-0411	Cross-brace single, SB38 ST46, black, matt	20100701 →	1
62	1537048-0419	Cross-brace single, SB38 ST46, silver grey		1
62	1537048-0442	Cross-brace single, SB38 ST46, night shimmer		1
62	1537048-0445	Cross-brace single, SB38 ST46, ivory white	20100701 →	1
62	1537048-0446	Cross-brace single, SB38 ST46, purple fire		1
62	1537048-0447	Cross-brace single, SB38 ST46, emerald green		1
62	1537048-0448	Cross-brace single, SB38 ST46, midnight blue		1
62	1537048-0449	Cross-brace single, SB38 ST46, umber glaze		1

Cross brace single, SB28-42

Pos.	Part number	Description	Valid from → until	Qty
62	1537048-0450	Cross-brace single, SB38 ST46, smokey grey		1
62	1537048-3524	Cross-brace single, SB38 ST46, royal red		1
62	1537048-5969	Cross-brace single, SB38 ST46, royal blue		1
62	1537048-9005	Cross-brace single, SB38 ST46, black, glossy		1
62	1537048-9119	Cross-brace single, SB38 ST46, black, bronze	20091001 → 20100630	1
63	1537049-0411	Cross-brace single, SB40 ST46, black, matt	20100701 →	1
63	1537049-0419	Cross-brace single, SB40 ST46, silver grey		1
63	1537049-0442	Cross-brace single, SB40 ST46, night shimmer		1
63	1537049-0445	Cross-brace single, SB40 ST46, ivory white	20100701 →	1
63	1537049-0446	Cross-brace single, SB40 ST46, purple fire		1
63	1537049-0447	Cross-brace single, SB40 ST46, emerald green		1
63	1537049-0448	Cross-brace single, SB40 ST46, midnight blue		1
63	1537049-0449	Cross-brace single, SB40 ST46, umber glaze		1
63	1537049-0450	Cross-brace single, SB40 ST46, smokey grey		1
63	1537049-3524	Cross-brace single, SB40 ST46, royal red		1
63	1537049-5969	Cross-brace single, SB40 ST46, royal blue		1
63	1537049-9005	Cross-brace single, SB40 ST46, black, glossy		1
63	1537049-9119	Cross-brace single, SB40 ST46, black, bronze	20091001 → 20100630	1
64	1537050-0411	Cross-brace single, SB42 ST46, black, matt	20100701 →	1
64	1537050-0419	Cross-brace single, SB42 ST46, silver grey		1
64	1537050-0442	Cross-brace single, SB42 ST46, night shimmer		1
64	1537050-0445	Cross-brace single, SB42 ST46, ivory white	20100701 →	1
64	1537050-0446	Cross-brace single, SB42 ST46, purple fire		1
64	1537050-0447	Cross-brace single, SB42 ST46, emerald green		1
64	1537050-0448	Cross-brace single, SB42 ST46, midnight blue		1
64	1537050-0449	Cross-brace single, SB42 ST46, umber glaze		1
64	1537050-0450	Cross-brace single, SB42 ST46, smokey grey		1
64	1537050-3524	Cross-brace single, SB42 ST46, royal red		1
64	1537050-5969	Cross-brace single, SB42 ST46, royal blue		1
64	1537050-9005	Cross-brace single, SB42 ST46, black, glossy		1
64	1537050-9119	Cross-brace single, SB42 ST46, black, bronze	20091001 → 20100630	1
65	1537055-0411	Cross-brace single, SB28 ST48, black, matt	20100701 →	1
65	1537055-0419	Cross-brace single, SB28 ST48, silver grey		1
65	1537055-0442	Cross-brace single, SB28 ST48, night shimmer		1
65	1537055-0445	Cross-brace single, SB28 ST48, ivory white	20100701 →	1
65	1537055-0446	Cross-brace single, SB28 ST48, purple fire		1
65	1537055-0447	Cross-brace single, SB28 ST48, emerald green		1
65	1537055-0448	Cross-brace single, SB28 ST48, midnight blue		1
65	1537055-0449	Cross-brace single, SB28 ST48, umber glaze		1
65	1537055-0450	Cross-brace single, SB28 ST48, smokey grey		1
65	1537055-3524	Cross-brace single, SB28 ST48, royal red		1
65	1537055-5969	Cross-brace single, SB28 ST48, royal blue		1
65	1537055-9005	Cross-brace single, SB28 ST48, black, glossy		1
65	1537055-9119	Cross-brace single, SB28 ST48, black, bronze	20091001 → 20100630	1
66	1537056-0411	Cross-brace single, SB30 ST48, black, matt	20100701 →	1
66	1537056-0419	Cross-brace single, SB30 ST48, silver grey		1
66	1537056-0442	Cross-brace single, SB30 ST48, night shimmer		1
66	1537056-0445	Cross-brace single, SB30 ST48, ivory white	20100701 →	1
66	1537056-0446	Cross-brace single, SB30 ST48, purple fire		1
66	1537056-0447	Cross-brace single, SB30 ST48, emerald green		1
66	1537056-0448	Cross-brace single, SB30 ST48, midnight blue		1
66	1537056-0449	Cross-brace single, SB30 ST48, umber glaze		1
66	1537056-0450	Cross-brace single, SB30 ST48, smokey grey		1
66	1537056-3524	Cross-brace single, SB30 ST48, royal red		1
66	1537056-5969	Cross-brace single, SB30 ST48, royal blue		1

Cross brace single, SB28-42

Pos.	Part number	Description	Valid from → until	Qty
66	1537056-9005	Cross-brace single, SB30 ST48, black, glossy		1
66	1537056-9119	Cross-brace single, SB30 ST48, black, bronze	20091001 → 20100630	1
67	1537057-0411	Cross-brace single, SB32 ST48, black, matt	20100701 →	1
67	1537057-0419	Cross-brace single, SB32 ST48, silver grey		1
67	1537057-0442	Cross-brace single, SB32 ST48, night shimmer		1
67	1537057-0445	Cross-brace single, SB32 ST48, ivory white	20100701 →	1
67	1537057-0446	Cross-brace single, SB32 ST48, purple fire		1
67	1537057-0447	Cross-brace single, SB32 ST48, emerald green		1
67	1537057-0448	Cross-brace single, SB32 ST48, midnight blue		1
67	1537057-0449	Cross-brace single, SB32 ST48, umber glaze		1
67	1537057-0450	Cross-brace single, SB32 ST48, smokey grey		1
67	1537057-3524	Cross-brace single, SB32 ST48, royal red		1
67	1537057-5969	Cross-brace single, SB32 ST48, royal blue		1
67	1537057-9005	Cross-brace single, SB32 ST48, black, glossy		1
67	1537057-9119	Cross-brace single, SB32 ST48, black, bronze	20091001 → 20100630	1
68	1537058-0411	Cross-brace single, SB34 ST48, black, matt	20100701 →	1
68	1537058-0419	Cross-brace single, SB34 ST48, silver grey		1
68	1537058-0442	Cross-brace single, SB34 ST48, night shimmer		1
68	1537058-0445	Cross-brace single, SB34 ST48, ivory white	20100701 →	1
68	1537058-0446	Cross-brace single, SB34 ST48, purple fire		1
68	1537058-0447	Cross-brace single, SB34 ST48, emerald green		1
68	1537058-0448	Cross-brace single, SB34 ST48, midnight blue		1
68	1537058-0449	Cross-brace single, SB34 ST48, umber glaze		1
68	1537058-0450	Cross-brace single, SB34 ST48, smokey grey		1
68	1537058-3524	Cross-brace single, SB34 ST48, royal red		1
68	1537058-5969	Cross-brace single, SB34 ST48, royal blue		1
68	1537058-9005	Cross-brace single, SB34 ST48, black, glossy		1
68	1537058-9119	Cross-brace single, SB34 ST48, black, bronze	20091001 → 20100630	1
69	1537104-0411	Cross-brace single, SB36 ST48, black, matt	20100701 →	1
69	1537104-0419	Cross-brace single, SB36 ST48, silver grey		1
69	1537104-0442	Cross-brace single, SB36 ST48, night shimmer		1
69	1537104-0445	Cross-brace single, SB36 ST48, ivory white	20100701 →	1
69	1537104-0446	Cross-brace single, SB36 ST48, purple fire		1
69	1537104-0447	Cross-brace single, SB36 ST48, emerald green		1
69	1537104-0448	Cross-brace single, SB36 ST48, midnight blue		1
69	1537104-0449	Cross-brace single, SB36 ST48, umber glaze		1
69	1537104-0450	Cross-brace single, SB36 ST48, smokey grey		1
69	1537104-3524	Cross-brace single, SB36 ST48, royal red		1
69	1537104-5969	Cross-brace single, SB36 ST48, royal blue		1
69	1537104-9005	Cross-brace single, SB36 ST48, black, glossy		1
69	1537104-9119	Cross-brace single, SB36 ST48, black, bronze	20091001 → 20100630	1
70	1537105-0411	Cross-brace single, SB38 ST48, black, matt	20100701 →	1
70	1537105-0419	Cross-brace single, SB38 ST48, silver grey		1
70	1537105-0442	Cross-brace single, SB38 ST48, night shimmer		1
70	1537105-0445	Cross-brace single, SB38 ST48, ivory white	20100701 →	1
70	1537105-0446	Cross-brace single, SB38 ST48, purple fire		1
70	1537105-0447	Cross-brace single, SB38 ST48, emerald green		1
70	1537105-0448	Cross-brace single, SB38 ST48, midnight blue		1
70	1537105-0449	Cross-brace single, SB38 ST48, umber glaze		1
70	1537105-0450	Cross-brace single, SB38 ST48, smokey grey		1
70	1537105-3524	Cross-brace single, SB38 ST48, royal red		1
70	1537105-5969	Cross-brace single, SB38 ST48, royal blue		1
70	1537105-9005	Cross-brace single, SB38 ST48, black, glossy		1
70	1537105-9119	Cross-brace single, SB38 ST48, black, bronze	20091001 → 20100630	1
71	1537160-0411	Cross-brace single, SB40 ST48, black, matt	20100701 →	1

Cross brace single, SB28-42

Pos.	Part number	Description	Valid from → until	Qty
71	1537160-0419	Cross-brace single, SB40 ST48, silver grey		1
71	1537160-0442	Cross-brace single, SB40 ST48, night shimmer		1
71	1537160-0445	Cross-brace single, SB40 ST48, ivory white	20100701 →	1
71	1537160-0446	Cross-brace single, SB40 ST48, purple fire		1
71	1537160-0447	Cross-brace single, SB40 ST48, emerald green		1
71	1537160-0448	Cross-brace single, SB40 ST48, midnight blue		1
71	1537160-0449	Cross-brace single, SB40 ST48, umber glaze		1
71	1537160-0450	Cross-brace single, SB40 ST48, smokey grey		1
71	1537160-3524	Cross-brace single, SB40 ST48, royal red		1
71	1537160-5969	Cross-brace single, SB40 ST48, royal blue		1
71	1537160-9005	Cross-brace single, SB40 ST48, black, glossy		1
71	1537160-9119	Cross-brace single, SB40 ST48, black, bronze	20091001 → 20100630	1
72	1537161-0411	Cross-brace single, SB42 ST48, black, matt	20100701 →	1
72	1537161-0419	Cross-brace single, SB42 ST48, silver grey		1
72	1537161-0442	Cross-brace single, SB42 ST48, night shimmer		1
72	1537161-0445	Cross-brace single, SB42 ST48, ivory white	20100701 →	1
72	1537161-0446	Cross-brace single, SB42 ST48, purple fire		1
72	1537161-0447	Cross-brace single, SB42 ST48, emerald green		1
72	1537161-0448	Cross-brace single, SB42 ST48, midnight blue		1
72	1537161-0449	Cross-brace single, SB42 ST48, umber glaze		1
72	1537161-0450	Cross-brace single, SB42 ST48, smokey grey		1
72	1537161-3524	Cross-brace single, SB42 ST48, royal red		1
72	1537161-5969	Cross-brace single, SB42 ST48, royal blue		1
72	1537161-9005	Cross-brace single, SB42 ST48, black, glossy		1
72	1537161-9119	Cross-brace single, SB42 ST48, black, bronze	20091001 → 20100630	1
73	1537166-0411	Cross-brace single, SB28 ST50, black, matt	20100701 →	1
73	1537166-0419	Cross-brace single, SB28 ST50, silver grey		1
73	1537166-0442	Cross-brace single, SB28 ST50, night shimmer		1
73	1537166-0445	Cross-brace single, SB28 ST50, ivory white	20100701 →	1
73	1537166-0446	Cross-brace single, SB28 ST50, purple fire		1
73	1537166-0447	Cross-brace single, SB28 ST50, emerald green		1
73	1537166-0448	Cross-brace single, SB28 ST50, midnight blue		1
73	1537166-0449	Cross-brace single, SB28 ST50, umber glaze		1
73	1537166-0450	Cross-brace single, SB28 ST50, smokey grey		1
73	1537166-3524	Cross-brace single, SB28 ST50, royal red		1
73	1537166-5969	Cross-brace single, SB28 ST50, royal blue		1
73	1537166-9005	Cross-brace single, SB28 ST50, black, glossy		1
73	1537166-9119	Cross-brace single, SB28 ST50, black, bronze	20091001 → 20100630	1
74	1537167-0411	Cross-brace single, SB30 ST50, black, matt	20100701 →	1
74	1537167-0419	Cross-brace single, SB30 ST50, silver grey		1
74	1537167-0442	Cross-brace single, SB30 ST50, night shimmer		1
74	1537167-0445	Cross-brace single, SB30 ST50, ivory white	20100701 →	1
74	1537167-0446	Cross-brace single, SB30 ST50, purple fire		1
74	1537167-0447	Cross-brace single, SB30 ST50, emerald green		1
74	1537167-0448	Cross-brace single, SB30 ST50, midnight blue		1
74	1537167-0449	Cross-brace single, SB30 ST50, umber glaze		1
74	1537167-0450	Cross-brace single, SB30 ST50, smokey grey		1
74	1537167-3524	Cross-brace single, SB30 ST50, royal red		1
74	1537167-5969	Cross-brace single, SB30 ST50, royal blue		1
74	1537167-9005	Cross-brace single, SB30 ST50, black, glossy		1
74	1537167-9119	Cross-brace single, SB30 ST50, black, bronze	20091001 → 20100630	1
75	1537168-0411	Cross-brace single, SB32 ST50, black, matt	20100701 →	1
75	1537168-0419	Cross-brace single, SB32 ST50, silver grey		1
75	1537168-0442	Cross-brace single, SB32 ST50, night shimmer		1
75	1537168-0445	Cross-brace single, SB32 ST50, ivory white	20100701 →	1


Cross brace single, SB28-42

Pos.	Part number	Description	Valid from → until	Qty
75	1537168-0446	Cross-brace single, SB32 ST50, purple fire		1
75	1537168-0447	Cross-brace single, SB32 ST50, emerald green		1
75	1537168-0448	Cross-brace single, SB32 ST50, midnight blue		1
75	1537168-0449	Cross-brace single, SB32 ST50, umber glaze		1
75	1537168-0450	Cross-brace single, SB32 ST50, smokey grey		1
75	1537168-3524	Cross-brace single, SB32 ST50, royal red		1
75	1537168-5969	Cross-brace single, SB32 ST50, royal blue		1
75	1537168-9005	Cross-brace single, SB32 ST50, black, glossy		1
75	1537168-9119	Cross-brace single, SB32 ST50, black, bronze	20091001 → 20100630	1
76	1537194-0411	Cross-brace single, SB34 ST50, black, matt	20100701 →	1
76	1537194-0419	Cross-brace single, SB34 ST50, silver grey		1
76	1537194-0442	Cross-brace single, SB34 ST50, night shimmer		1
76	1537194-0445	Cross-brace single, SB34 ST50, ivory white	20100701 →	1
76	1537194-0446	Cross-brace single, SB34 ST50, purple fire		1
76	1537194-0447	Cross-brace single, SB34 ST50, emerald green		1
76	1537194-0448	Cross-brace single, SB34 ST50, midnight blue		1
76	1537194-0449	Cross-brace single, SB34 ST50, umber glaze		1
76	1537194-0450	Cross-brace single, SB34 ST50, smokey grey		1
76	1537194-3524	Cross-brace single, SB34 ST50, royal red		1
76	1537194-5969	Cross-brace single, SB34 ST50, royal blue		1
76	1537194-9005	Cross-brace single, SB34 ST50, black, glossy		1
76	1537194-9119	Cross-brace single, SB34 ST50, black, bronze	20091001 → 20100630	1
77	1537195-0411	Cross-brace single, SB36 ST50, black, matt	20100701 →	1
77	1537195-0419	Cross-brace single, SB36 ST50, silver grey		1
77	1537195-0442	Cross-brace single, SB36 ST50, night shimmer		1
77	1537195-0445	Cross-brace single, SB36 ST50, ivory white	20100701 →	1
77	1537195-0446	Cross-brace single, SB36 ST50, purple fire		1
77	1537195-0447	Cross-brace single, SB36 ST50, emerald green		1
77	1537195-0448	Cross-brace single, SB36 ST50, midnight blue		1
77	1537195-0449	Cross-brace single, SB36 ST50, umber glaze		1
77	1537195-0450	Cross-brace single, SB36 ST50, smokey grey		1
77	1537195-3524	Cross-brace single, SB36 ST50, royal red		1
77	1537195-5969	Cross-brace single, SB36 ST50, royal blue		1
77	1537195-9005	Cross-brace single, SB36 ST50, black, glossy		1
77	1537195-9119	Cross-brace single, SB36 ST50, black, bronze	20091001 → 20100630	1
78	1537196-0411	Cross-brace single, SB38 ST50, black, matt	20100701 →	1
78	1537196-0419	Cross-brace single, SB38 ST50, silver grey		1
78	1537196-0442	Cross-brace single, SB38 ST50, night shimmer		1
78	1537196-0445	Cross-brace single, SB38 ST50, ivory white	20100701 →	1
78	1537196-0446	Cross-brace single, SB38 ST50, purple fire		1
78	1537196-0447	Cross-brace single, SB38 ST50, emerald green		1
78	1537196-0448	Cross-brace single, SB38 ST50, midnight blue		1
78	1537196-0449	Cross-brace single, SB38 ST50, umber glaze		1
78	1537196-0450	Cross-brace single, SB38 ST50, smokey grey		1
78	1537196-3524	Cross-brace single, SB38 ST50, royal red		1
78	1537196-5969	Cross-brace single, SB38 ST50, royal blue		1
78	1537196-9005	Cross-brace single, SB38 ST50, black, glossy		1
78	1537196-9119	Cross-brace single, SB38 ST50, black, bronze	20091001 → 20100630	1
79	1537197-0411	Cross-brace single, SB40 ST50, black, matt	20100701 →	1
79	1537197-0419	Cross-brace single, SB40 ST50, silver grey		1
79	1537197-0442	Cross-brace single, SB40 ST50, night shimmer		1
79	1537197-0445	Cross-brace single, SB40 ST50, ivory white	20100701 →	1
79	1537197-0446	Cross-brace single, SB40 ST50, purple fire		1
79	1537197-0447	Cross-brace single, SB40 ST50, emerald green		1
79	1537197-0448	Cross-brace single, SB40 ST50, midnight blue		1

Cross brace single, SB28-42

Pos.	Part number	Description	Valid from → until	Qty
79	1537197-0449	Cross-brace single, SB40 ST50, umber glaze		1
79	1537197-0450	Cross-brace single, SB40 ST50, smokey grey		1
79	1537197-3524	Cross-brace single, SB40 ST50, royal red		1
79	1537197-5969	Cross-brace single, SB40 ST50, royal blue		1
79	1537197-9005	Cross-brace single, SB40 ST50, black, glossy		1
79	1537197-9119	Cross-brace single, SB40 ST50, black, bronze	20091001 → 20100630	1
80	1537681-0411	Cross-brace single, SB42 ST50, black, matt	20100701 →	1
80	1537681-0419	Cross-brace single, SB42 ST50, silver grey		1
80	1537681-0442	Cross-brace single, SB42 ST50, night shimmer		1
80	1537681-0445	Cross-brace single, SB42 ST50, ivory white	20100701 →	1
80	1537681-0446	Cross-brace single, SB42 ST50, purple fire		1
80	1537681-0447	Cross-brace single, SB42 ST50, emerald green		1
80	1537681-0448	Cross-brace single, SB42 ST50, midnight blue		1
80	1537681-0449	Cross-brace single, SB42 ST50, umber glaze		1
80	1537681-0450	Cross-brace single, SB42 ST50, smokey grey		1
80	1537681-3524	Cross-brace single, SB42 ST50, royal red		1
80	1537681-5969	Cross-brace single, SB42 ST50, royal blue		1
80	1537681-9005	Cross-brace single, SB42 ST50, black, glossy		1
80	1537681-9119	Cross-brace single, SB42 ST50, black, bronze	20091001 → 20100630	1
81	1536083	Retaining lever SB28/32/36/40, pair		1
82	1536084	Retaining lever SB30/34/38/42, pair		1
83	1537686	Seat lock kit		1
84	1537688	Seat lock kit with sleeves		1
85	1537689	Screw kit to Single crossbrace		1
86	1537687-0004	Cross-brace plug kit, black anodized		1
86	1537687-0400	Cross-brace plug kit, techno-anthrazit		1
86	1537687-0401	Cross-brace plug kit, lemon		1
86	1537687-0402	Cross-brace plug kit, apricot		1
86	1537687-0404	Cross-brace plug kit, b-blue		1
86	1537687-0440	Cross-brace plug kit, comet red		1
86	1537687-0441	Cross-brace plug kit, arctic blue		1
86	1537687-0444	Cross-brace plug kit, copper glaze		1

Cross brace double, SB44-50

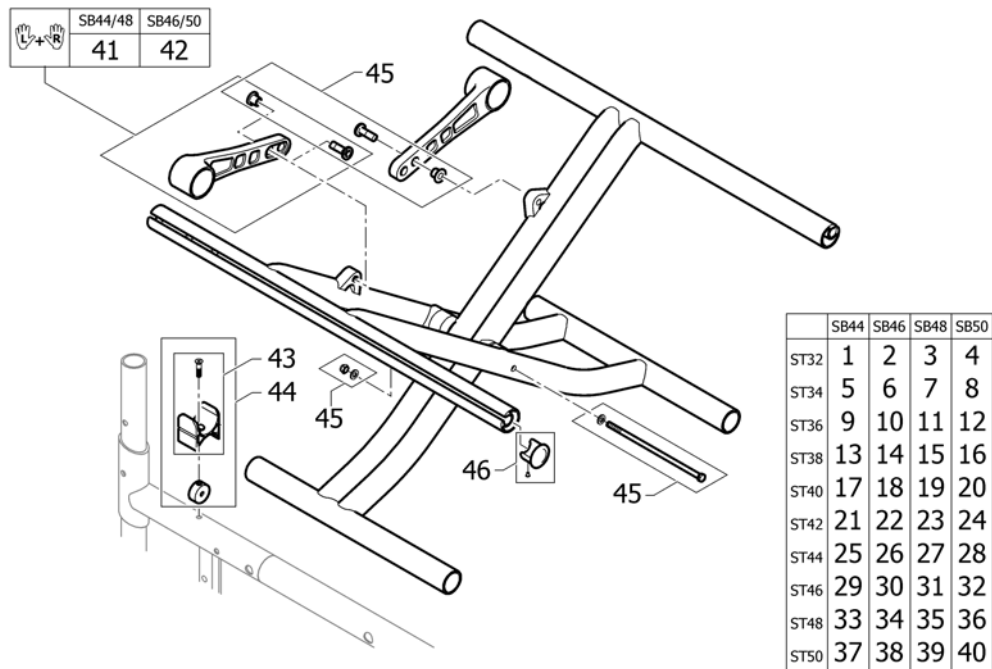

Pos.	Part number	Description	Valid from → until	Qty
1	1536967-0411	Cross-brace double, SB44 ST32, black, matt	20100701 →	1
1	1536967-0419	Cross-brace double, SB44 ST32, silver grey		1
1	1536967-0442	Cross-brace double, SB44 ST32, night shimmer		1
1	1536967-0445	Cross-brace double, SB44 ST32, ivory white	20100701 →	1
1	1536967-0446	Cross-brace double, SB44 ST32, purple fire		1
1	1536967-0447	Cross-brace double, SB44 ST32, emerald green		1
1	1536967-0448	Cross-brace double, SB44 ST32, midnight blue		1
1	1536967-0449	Cross-brace double, SB44 ST32, umber glaze		1
1	1536967-0450	Cross-brace double, SB44 ST32, smokey grey		1
1	1536967-3524	Cross-brace double, SB44 ST32, royal red		1
1	1536967-5969	Cross-brace double, SB44 ST32, royal blue		1
1	1536967-9005	Cross-brace double, SB44 ST32, black, glossy		1
1	1536967-9119	Cross-brace double, SB44 ST32, black, bronze	20091001 → 20100630	1
2	1536968-0411	Cross-brace double, SB46 ST32, black, matt	20100701 →	1
2	1536968-0419	Cross-brace double, SB46 ST32, silver grey		1
2	1536968-0442	Cross-brace double, SB46 ST32, night shimmer		1
2	1536968-0445	Cross-brace double, SB46 ST32, ivory white	20100701 →	1
2	1536968-0446	Cross-brace double, SB46 ST32, purple fire		1
2	1536968-0447	Cross-brace double, SB46 ST32, emerald green		1
2	1536968-0448	Cross-brace double, SB46 ST32, midnight blue		1
2	1536968-0449	Cross-brace double, SB46 ST32, umber glaze		1
2	1536968-0450	Cross-brace double, SB46 ST32, smokey grey		1
2	1536968-3524	Cross-brace double, SB46 ST32, royal red		1
2	1536968-5969	Cross-brace double, SB46 ST32, royal blue		1
2	1536968-9005	Cross-brace double, SB46 ST32, black, glossy		1
2	1536968-9119	Cross-brace double, SB46 ST32, black, bronze	20091001 → 20100630	1
3	1536969-0411	Cross-brace double, SB48 ST32, black, matt	20100701 →	1
3	1536969-0419	Cross-brace double, SB48 ST32, silver grey		1
3	1536969-0442	Cross-brace double, SB48 ST32, night shimmer		1
3	1536969-0445	Cross-brace double, SB48 ST32, ivory white	20100701 →	1
3	1536969-0446	Cross-brace double, SB48 ST32, purple fire		1

Cross brace double, SB44-50

Pos.	Part number	Description	Valid from → until	Qty
3	1536969-0447	Cross-brace double, SB48 ST32, emerald green		1
3	1536969-0448	Cross-brace double, SB48 ST32, midnight blue		1
3	1536969-0449	Cross-brace double, SB48 ST32, umber glaze		1
3	1536969-0450	Cross-brace double, SB48 ST32, smokey grey		1
3	1536969-3524	Cross-brace double, SB48 ST32, royal red		1
3	1536969-5969	Cross-brace double, SB48 ST32, royal blue		1
3	1536969-9005	Cross-brace double, SB48 ST32, black, glossy		1
3	1536969-9119	Cross-brace double, SB48 ST32, black, bronze	20091001 → 20100630	1
4	1536970-0411	Cross-brace double, SB50 ST32, black, matt	20100701 →	1
4	1536970-0419	Cross-brace double, SB50 ST32, silver grey		1
4	1536970-0442	Cross-brace double, SB50 ST32, night shimmer		1
4	1536970-0445	Cross-brace double, SB50 ST32, ivory white	20100701 →	1
4	1536970-0446	Cross-brace double, SB50 ST32, purple fire		1
4	1536970-0447	Cross-brace double, SB50 ST32, emerald green		1
4	1536970-0448	Cross-brace double, SB50 ST32, midnight blue		1
4	1536970-0449	Cross-brace double, SB50 ST32, umber glaze		1
4	1536970-0450	Cross-brace double, SB50 ST32, smokey grey		1
4	1536970-3524	Cross-brace double, SB50 ST32, royal red		1
4	1536970-5969	Cross-brace double, SB50 ST32, royal blue		1
4	1536970-9005	Cross-brace double, SB50 ST32, black, glossy		1
4	1536970-9119	Cross-brace double, SB50 ST32, black, bronze	20091001 → 20100630	1
5	1536979-0411	Cross-brace double, SB44 ST34, black, matt	20100701 →	1
5	1536979-0419	Cross-brace double, SB44 ST34, silver grey		1
5	1536979-0442	Cross-brace double, SB44 ST34, night shimmer		1
5	1536979-0445	Cross-brace double, SB44 ST34, ivory white	20100701 →	1
5	1536979-0446	Cross-brace double, SB44 ST34, purple fire		1
5	1536979-0447	Cross-brace double, SB44 ST34, emerald green		1
5	1536979-0448	Cross-brace double, SB44 ST34, midnight blue		1
5	1536979-0449	Cross-brace double, SB44 ST34, umber glaze		1
5	1536979-0450	Cross-brace double, SB44 ST34, smokey grey		1
5	1536979-3524	Cross-brace double, SB44 ST34, royal red		1
5	1536979-5969	Cross-brace double, SB44 ST34, royal blue		1
5	1536979-9005	Cross-brace double, SB44 ST34, black, glossy		1
5	1536979-9119	Cross-brace double, SB44 ST34, black, bronze	20091001 → 20100630	1
6	1536980-0411	Cross-brace double, SB46 ST34, black, matt	20100701 →	1
6	1536980-0419	Cross-brace double, SB46 ST34, silver grey		1
6	1536980-0442	Cross-brace double, SB46 ST34, night shimmer		1
6	1536980-0445	Cross-brace double, SB46 ST34, ivory white	20100701 →	1
6	1536980-0446	Cross-brace double, SB46 ST34, purple fire		1
6	1536980-0447	Cross-brace double, SB46 ST34, emerald green		1
6	1536980-0448	Cross-brace double, SB46 ST34, midnight blue		1
6	1536980-0449	Cross-brace double, SB46 ST34, umber glaze		1
6	1536980-0450	Cross-brace double, SB46 ST34, smokey grey		1
6	1536980-3524	Cross-brace double, SB46 ST34, royal red		1
6	1536980-5969	Cross-brace double, SB46 ST34, royal blue		1
6	1536980-9005	Cross-brace double, SB46 ST34, black, glossy		1
6	1536980-9119	Cross-brace double, SB46 ST34, black, bronze	20091001 → 20100630	1
7	1536981-0411	Cross-brace double, SB48 ST34, black, matt	20100701 →	1
7	1536981-0419	Cross-brace double, SB48 ST34, silver grey		1
7	1536981-0442	Cross-brace double, SB48 ST34, night shimmer		1
7	1536981-0445	Cross-brace double, SB48 ST34, ivory white	20100701 →	1
7	1536981-0446	Cross-brace double, SB48 ST34, purple fire		1
7	1536981-0447	Cross-brace double, SB48 ST34, emerald green		1
7	1536981-0448	Cross-brace double, SB48 ST34, midnight blue		1
7	1536981-0449	Cross-brace double, SB48 ST34, umber glaze		1

Cross brace double, SB44-50

Pos.	Part number	Description	Valid from → until	Qty
7	1536981-0450	Cross-brace double, SB48 ST34, smokey grey		1
7	1536981-3524	Cross-brace double, SB48 ST34, royal red		1
7	1536981-5969	Cross-brace double, SB48 ST34, royal blue		1
7	1536981-9005	Cross-brace double, SB48 ST34, black, glossy		1
7	1536981-9119	Cross-brace double, SB48 ST34, black, bronze	20091001 → 20100630	1
8	1536982-0411	Cross-brace double, SB50 ST34, black, matt	20100701 →	1
8	1536982-0419	Cross-brace double, SB50 ST34, silver grey		1
8	1536982-0442	Cross-brace double, SB50 ST34, night shimmer		1
8	1536982-0445	Cross-brace double, SB50 ST34, ivory white	20100701 →	1
8	1536982-0446	Cross-brace double, SB50 ST34, purple fire		1
8	1536982-0447	Cross-brace double, SB50 ST34, emerald green		1
8	1536982-0448	Cross-brace double, SB50 ST34, midnight blue		1
8	1536982-0449	Cross-brace double, SB50 ST34, umber glaze		1
8	1536982-0450	Cross-brace double, SB50 ST34, smokey grey		1
8	1536982-3524	Cross-brace double, SB50 ST34, royal red		1
8	1536982-5969	Cross-brace double, SB50 ST34, royal blue		1
8	1536982-9005	Cross-brace double, SB50 ST34, black, glossy		1
8	1536982-9119	Cross-brace double, SB50 ST34, black, bronze	20091001 → 20100630	1
9	1536991-0411	Cross-brace double, SB44 ST36, black, matt	20100701 →	1
9	1536991-0419	Cross-brace double, SB44 ST36, silver grey		1
9	1536991-0442	Cross-brace double, SB44 ST36, night shimmer		1
9	1536991-0445	Cross-brace double, SB44 ST36, ivory white	20100701 →	1
9	1536991-0446	Cross-brace double, SB44 ST36, purple fire		1
9	1536991-0447	Cross-brace double, SB44 ST36, emerald green		1
9	1536991-0448	Cross-brace double, SB44 ST36, midnight blue		1
9	1536991-0449	Cross-brace double, SB44 ST36, umber glaze		1
9	1536991-0450	Cross-brace double, SB44 ST36, smokey grey		1
9	1536991-3524	Cross-brace double, SB44 ST36, royal red		1
9	1536991-5969	Cross-brace double, SB44 ST36, royal blue		1
9	1536991-9005	Cross-brace double, SB44 ST36, black, glossy		1
9	1536991-9119	Cross-brace double, SB44 ST36, black, bronze	20091001 → 20100630	1
10	1536992-0411	Cross-brace double, SB46 ST36, black, matt	20100701 →	1
10	1536992-0419	Cross-brace double, SB46 ST36, silver grey		1
10	1536992-0442	Cross-brace double, SB46 ST36, night shimmer		1
10	1536992-0445	Cross-brace double, SB46 ST36, ivory white	20100701 →	1
10	1536992-0446	Cross-brace double, SB46 ST36, purple fire		1
10	1536992-0447	Cross-brace double, SB46 ST36, emerald green		1
10	1536992-0448	Cross-brace double, SB46 ST36, midnight blue		1
10	1536992-0449	Cross-brace double, SB46 ST36, umber glaze		1
10	1536992-0450	Cross-brace double, SB46 ST36, smokey grey		1
10	1536992-3524	Cross-brace double, SB46 ST36, royal red		1
10	1536992-5969	Cross-brace double, SB46 ST36, royal blue		1
10	1536992-9005	Cross-brace double, SB46 ST36, black, glossy		1
10	1536992-9119	Cross-brace double, SB46 ST36, black, bronze	20091001 → 20100630	1
11	1536993-0411	Cross-brace double, SB48 ST36, black, matt	20100701 →	1
11	1536993-0419	Cross-brace double, SB48 ST36, silver grey		1
11	1536993-0442	Cross-brace double, SB48 ST36, night shimmer		1
11	1536993-0445	Cross-brace double, SB48 ST36, ivory white	20100701 →	1
11	1536993-0446	Cross-brace double, SB48 ST36, purple fire		1
11	1536993-0447	Cross-brace double, SB48 ST36, emerald green		1
11	1536993-0448	Cross-brace double, SB48 ST36, midnight blue		1
11	1536993-0449	Cross-brace double, SB48 ST36, umber glaze		1
11	1536993-0450	Cross-brace double, SB48 ST36, smokey grey		1
11	1536993-3524	Cross-brace double, SB48 ST36, royal red		1
11	1536993-5969	Cross-brace double, SB48 ST36, royal blue		1

Cross brace double, SB44-50

Pos.	Part number	Description	Valid from → until	Qty
11	1536993-9005	Cross-brace double, SB48 ST36, black, glossy		1
11	1536993-9119	Cross-brace double, SB48 ST36, black, bronze	20091001 → 20100630	1
12	1536994-0411	Cross-brace double, SB50 ST36, black, matt	20100701 →	1
12	1536994-0419	Cross-brace double, SB50 ST36, silver grey		1
12	1536994-0442	Cross-brace double, SB50 ST36, night shimmer		1
12	1536994-0445	Cross-brace double, SB50 ST36, ivory white	20100701 →	1
12	1536994-0446	Cross-brace double, SB50 ST36, purple fire		1
12	1536994-0447	Cross-brace double, SB50 ST36, emerald green		1
12	1536994-0448	Cross-brace double, SB50 ST36, midnight blue		1
12	1536994-0449	Cross-brace double, SB50 ST36, umber glaze		1
12	1536994-0450	Cross-brace double, SB50 ST36, smokey grey		1
12	1536994-3524	Cross-brace double, SB50 ST36, royal red		1
12	1536994-5969	Cross-brace double, SB50 ST36, royal blue		1
12	1536994-9005	Cross-brace double, SB50 ST36, black, glossy		1
12	1536994-9119	Cross-brace double, SB50 ST36, black, bronze	20091001 → 20100630	1
13	1537003-0411	Cross-brace double, SB44 ST38, black, matt	20100701 →	1
13	1537003-0419	Cross-brace double, SB44 ST38, silver grey		1
13	1537003-0442	Cross-brace double, SB44 ST38, night shimmer		1
13	1537003-0445	Cross-brace double, SB44 ST38, ivory white	20100701 →	1
13	1537003-0446	Cross-brace double, SB44 ST38, purple fire		1
13	1537003-0447	Cross-brace double, SB44 ST38, emerald green		1
13	1537003-0448	Cross-brace double, SB44 ST38, midnight blue		1
13	1537003-0449	Cross-brace double, SB44 ST38, umber glaze		1
13	1537003-0450	Cross-brace double, SB44 ST38, smokey grey		1
13	1537003-3524	Cross-brace double, SB44 ST38, royal red		1
13	1537003-5969	Cross-brace double, SB44 ST38, royal blue		1
13	1537003-9005	Cross-brace double, SB44 ST38, black, glossy		1
13	1537003-9119	Cross-brace double, SB44 ST38, black, bronze	20091001 → 20100630	1
14	1537004-0411	Cross-brace double, SB46 ST38, black, matt	20100701 →	1
14	1537004-0419	Cross-brace double, SB46 ST38, silver grey		1
14	1537004-0442	Cross-brace double, SB46 ST38, night shimmer		1
14	1537004-0445	Cross-brace double, SB46 ST38, ivory white	20100701 →	1
14	1537004-0446	Cross-brace double, SB46 ST38, purple fire		1
14	1537004-0447	Cross-brace double, SB46 ST38, emerald green		1
14	1537004-0448	Cross-brace double, SB46 ST38, midnight blue		1
14	1537004-0449	Cross-brace double, SB46 ST38, umber glaze		1
14	1537004-0450	Cross-brace double, SB46 ST38, smokey grey		1
14	1537004-3524	Cross-brace double, SB46 ST38, royal red		1
14	1537004-5969	Cross-brace double, SB46 ST38, royal blue		1
14	1537004-9005	Cross-brace double, SB46 ST38, black, glossy		1
14	1537004-9119	Cross-brace double, SB46 ST38, black, bronze	20091001 → 20100630	1
15	1537005-0411	Cross-brace double, SB48 ST38, black, matt	20100701 →	1
15	1537005-0419	Cross-brace double, SB48 ST38, silver grey		1
15	1537005-0442	Cross-brace double, SB48 ST38, night shimmer		1
15	1537005-0445	Cross-brace double, SB48 ST38, ivory white	20100701 →	1
15	1537005-0446	Cross-brace double, SB48 ST38, purple fire		1
15	1537005-0447	Cross-brace double, SB48 ST38, emerald green		1
15	1537005-0448	Cross-brace double, SB48 ST38, midnight blue		1
15	1537005-0449	Cross-brace double, SB48 ST38, umber glaze		1
15	1537005-0450	Cross-brace double, SB48 ST38, smokey grey		1
15	1537005-3524	Cross-brace double, SB48 ST38, royal red		1
15	1537005-5969	Cross-brace double, SB48 ST38, royal blue		1
15	1537005-9005	Cross-brace double, SB48 ST38, black, glossy		1
15	1537005-9119	Cross-brace double, SB48 ST38, black, bronze	20091001 → 20100630	1
16	1537006-0411	Cross-brace double, SB50 ST38, black, matt	20100701 →	1

Cross brace double, SB44-50

Pos.	Part number	Description	Valid from → until	Qty
16	1537006-0419	Cross-brace double, SB50 ST38, silver grey		1
16	1537006-0442	Cross-brace double, SB50 ST38, night shimmer		1
16	1537006-0445	Cross-brace double, SB50 ST38, ivory white	20100701 →	1
16	1537006-0446	Cross-brace double, SB50 ST38, purple fire		1
16	1537006-0447	Cross-brace double, SB50 ST38, emerald green		1
16	1537006-0448	Cross-brace double, SB50 ST38, midnight blue		1
16	1537006-0449	Cross-brace double, SB50 ST38, umber glaze		1
16	1537006-0450	Cross-brace double, SB50 ST38, smokey grey		1
16	1537006-3524	Cross-brace double, SB50 ST38, royal red		1
16	1537006-5969	Cross-brace double, SB50 ST38, royal blue		1
16	1537006-9005	Cross-brace double, SB50 ST38, black, glossy		1
16	1537006-9119	Cross-brace double, SB50 ST38, black, bronze	20091001 → 20100630	1
17	1537015-0411	Cross-brace double, SB44 ST40, black, matt	20100701 →	1
17	1537015-0419	Cross-brace double, SB44 ST40, silver grey		1
17	1537015-0442	Cross-brace double, SB44 ST40, night shimmer		1
17	1537015-0445	Cross-brace double, SB44 ST40, ivory white	20100701 →	1
17	1537015-0446	Cross-brace double, SB44 ST40, purple fire		1
17	1537015-0447	Cross-brace double, SB44 ST40, emerald green		1
17	1537015-0448	Cross-brace double, SB44 ST40, midnight blue		1
17	1537015-0449	Cross-brace double, SB44 ST40, umber glaze		1
17	1537015-0450	Cross-brace double, SB44 ST40, smokey grey		1
17	1537015-3524	Cross-brace double, SB44 ST40, royal red		1
17	1537015-5969	Cross-brace double, SB44 ST40, royal blue		1
17	1537015-9005	Cross-brace double, SB44 ST40, black, glossy		1
17	1537015-9119	Cross-brace double, SB44 ST40, black, bronze	20091001 → 20100630	1
18	1537016-0411	Cross-brace double, SB46 ST40, black, matt	20100701 →	1
18	1537016-0419	Cross-brace double, SB46 ST40, silver grey		1
18	1537016-0442	Cross-brace double, SB46 ST40, night shimmer		1
18	1537016-0445	Cross-brace double, SB46 ST40, ivory white	20100701 →	1
18	1537016-0446	Cross-brace double, SB46 ST40, purple fire		1
18	1537016-0447	Cross-brace double, SB46 ST40, emerald green		1
18	1537016-0448	Cross-brace double, SB46 ST40, midnight blue		1
18	1537016-0449	Cross-brace double, SB46 ST40, umber glaze		1
18	1537016-0450	Cross-brace double, SB46 ST40, smokey grey		1
18	1537016-3524	Cross-brace double, SB46 ST40, royal red		1
18	1537016-5969	Cross-brace double, SB46 ST40, royal blue		1
18	1537016-9005	Cross-brace double, SB46 ST40, black, glossy		1
18	1537016-9119	Cross-brace double, SB46 ST40, black, bronze	20091001 → 20100630	1
19	1537017-0411	Cross-brace double, SB48 ST40, black, matt	20100701 →	1
19	1537017-0419	Cross-brace double, SB48 ST40, silver grey		1
19	1537017-0442	Cross-brace double, SB48 ST40, night shimmer		1
19	1537017-0445	Cross-brace double, SB48 ST40, ivory white	20100701 →	1
19	1537017-0446	Cross-brace double, SB48 ST40, purple fire		1
19	1537017-0447	Cross-brace double, SB48 ST40, emerald green		1
19	1537017-0448	Cross-brace double, SB48 ST40, midnight blue		1
19	1537017-0449	Cross-brace double, SB48 ST40, umber glaze		1
19	1537017-0450	Cross-brace double, SB48 ST40, smokey grey		1
19	1537017-3524	Cross-brace double, SB48 ST40, royal red		1
19	1537017-5969	Cross-brace double, SB48 ST40, royal blue		1
19	1537017-9005	Cross-brace double, SB48 ST40, black, glossy		1
19	1537017-9119	Cross-brace double, SB48 ST40, black, bronze	20091001 → 20100630	1
20	1537018-0411	Cross-brace double, SB50 ST40, black, matt	20100701 →	1
20	1537018-0419	Cross-brace double, SB50 ST40, silver grey		1
20	1537018-0442	Cross-brace double, SB50 ST40, night shimmer		1
20	1537018-0445	Cross-brace double, SB50 ST40, ivory white	20100701 →	1

Cross brace double, SB44-50

Pos.	Part number	Description	Valid from → until	Qty
20	1537018-0446	Cross-brace double, SB50 ST40, purple fire		1
20	1537018-0447	Cross-brace double, SB50 ST40, emerald green		1
20	1537018-0448	Cross-brace double, SB50 ST40, midnight blue		1
20	1537018-0449	Cross-brace double, SB50 ST40, umber glaze		1
20	1537018-0450	Cross-brace double, SB50 ST40, smokey grey		1
20	1537018-3524	Cross-brace double, SB50 ST40, royal red		1
20	1537018-5969	Cross-brace double, SB50 ST40, royal blue		1
20	1537018-9005	Cross-brace double, SB50 ST40, black, glossy		1
20	1537018-9119	Cross-brace double, SB50 ST40, black, bronze	20091001 → 20100630	1
21	1537027-0411	Cross-brace double, SB44 ST42, black, matt	20100701 →	1
21	1537027-0419	Cross-brace double, SB44 ST42, silver grey		1
21	1537027-0442	Cross-brace double, SB44 ST42, night shimmer		1
21	1537027-0445	Cross-brace double, SB44 ST42, ivory white	20100701 →	1
21	1537027-0446	Cross-brace double, SB44 ST42, purple fire		1
21	1537027-0447	Cross-brace double, SB44 ST42, emerald green		1
21	1537027-0448	Cross-brace double, SB44 ST42, midnight blue		1
21	1537027-0449	Cross-brace double, SB44 ST42, umber glaze		1
21	1537027-0450	Cross-brace double, SB44 ST42, smokey grey		1
21	1537027-3524	Cross-brace double, SB44 ST42, royal red		1
21	1537027-5969	Cross-brace double, SB44 ST42, royal blue		1
21	1537027-9005	Cross-brace double, SB44 ST42, black, glossy		1
21	1537027-9119	Cross-brace double, SB44 ST42, black, bronze	20091001 → 20100630	1
22	1537028-0411	Cross-brace double, SB46 ST42, black, matt	20100701 →	1
22	1537028-0419	Cross-brace double, SB46 ST42, silver grey		1
22	1537028-0442	Cross-brace double, SB46 ST42, night shimmer		1
22	1537028-0445	Cross-brace double, SB46 ST42, ivory white	20100701 →	1
22	1537028-0446	Cross-brace double, SB46 ST42, purple fire		1
22	1537028-0447	Cross-brace double, SB46 ST42, emerald green		1
22	1537028-0448	Cross-brace double, SB46 ST42, midnight blue		1
22	1537028-0449	Cross-brace double, SB46 ST42, umber glaze		1
22	1537028-0450	Cross-brace double, SB46 ST42, smokey grey		1
22	1537028-3524	Cross-brace double, SB46 ST42, royal red		1
22	1537028-5969	Cross-brace double, SB46 ST42, royal blue		1
22	1537028-9005	Cross-brace double, SB46 ST42, black, glossy		1
22	1537028-9119	Cross-brace double, SB46 ST42, black, bronze	20091001 → 20100630	1
23	1537029-0411	Cross-brace double, SB48 ST42, black, matt	20100701 →	1
23	1537029-0419	Cross-brace double, SB48 ST42, silver grey		1
23	1537029-0442	Cross-brace double, SB48 ST42, night shimmer		1
23	1537029-0445	Cross-brace double, SB48 ST42, ivory white	20100701 →	1
23	1537029-0446	Cross-brace double, SB48 ST42, purple fire		1
23	1537029-0447	Cross-brace double, SB48 ST42, emerald green		1
23	1537029-0448	Cross-brace double, SB48 ST42, midnight blue		1
23	1537029-0449	Cross-brace double, SB48 ST42, umber glaze		1
23	1537029-0450	Cross-brace double, SB48 ST42, smokey grey		1
23	1537029-3524	Cross-brace double, SB48 ST42, royal red		1
23	1537029-5969	Cross-brace double, SB48 ST42, royal blue		1
23	1537029-9005	Cross-brace double, SB48 ST42, black, glossy		1
23	1537029-9119	Cross-brace double, SB48 ST42, black, bronze	20091001 → 20100630	1
24	1537030-0411	Cross-brace double, SB50 ST42, black, matt	20100701 →	1
24	1537030-0419	Cross-brace double, SB50 ST42, silver grey		1
24	1537030-0442	Cross-brace double, SB50 ST42, night shimmer		1
24	1537030-0445	Cross-brace double, SB50 ST42, ivory white	20100701 →	1
24	1537030-0446	Cross-brace double, SB50 ST42, purple fire		1
24	1537030-0447	Cross-brace double, SB50 ST42, emerald green		1
24	1537030-0448	Cross-brace double, SB50 ST42, midnight blue		1

Cross brace double, SB44-50

Pos.	Part number	Description	Valid from → until	Qty
24	1537030-0449	Cross-brace double, SB50 ST42, umber glaze		1
24	1537030-0450	Cross-brace double, SB50 ST42, smokey grey		1
24	1537030-3524	Cross-brace double, SB50 ST42, royal red		1
24	1537030-5969	Cross-brace double, SB50 ST42, royal blue		1
24	1537030-9005	Cross-brace double, SB50 ST42, black, glossy		1
24	1537030-9119	Cross-brace double, SB50 ST42, black, bronze	20091001 → 20100630	1
25	1537039-0411	Cross-brace double, SB44 ST44, black, matt	20100701 →	1
25	1537039-0419	Cross-brace double, SB44 ST44, silver grey		1
25	1537039-0442	Cross-brace double, SB44 ST44, night shimmer		1
25	1537039-0445	Cross-brace double, SB44 ST44, ivory white	20100701 →	1
25	1537039-0446	Cross-brace double, SB44 ST44, purple fire		1
25	1537039-0447	Cross-brace double, SB44 ST44, emerald green		1
25	1537039-0448	Cross-brace double, SB44 ST44, midnight blue		1
25	1537039-0449	Cross-brace double, SB44 ST44, umber glaze		1
25	1537039-0450	Cross-brace double, SB44 ST44, smokey grey		1
25	1537039-3524	Cross-brace double, SB44 ST44, royal red		1
25	1537039-5969	Cross-brace double, SB44 ST44, royal blue		1
25	1537039-9005	Cross-brace double, SB44 ST44, black, glossy		1
25	1537039-9119	Cross-brace double, SB44 ST44, black, bronze	20091001 → 20100630	1
26	1537040-0411	Cross-brace double, SB46 ST44, black, matt	20100701 →	1
26	1537040-0419	Cross-brace double, SB46 ST44, silver grey		1
26	1537040-0442	Cross-brace double, SB46 ST44, night shimmer		1
26	1537040-0445	Cross-brace double, SB46 ST44, ivory white	20100701 →	1
26	1537040-0446	Cross-brace double, SB46 ST44, purple fire		1
26	1537040-0447	Cross-brace double, SB46 ST44, emerald green		1
26	1537040-0448	Cross-brace double, SB46 ST44, midnight blue		1
26	1537040-0449	Cross-brace double, SB46 ST44, umber glaze		1
26	1537040-0450	Cross-brace double, SB46 ST44, smokey grey		1
26	1537040-3524	Cross-brace double, SB46 ST44, royal red		1
26	1537040-5969	Cross-brace double, SB46 ST44, royal blue		1
26	1537040-9005	Cross-brace double, SB46 ST44, black, glossy		1
26	1537040-9119	Cross-brace double, SB46 ST44, black, bronze	20091001 → 20100630	1
27	1537041-0411	Cross-brace double, SB48 ST44, black, matt	20100701 →	1
27	1537041-0419	Cross-brace double, SB48 ST44, silver grey		1
27	1537041-0442	Cross-brace double, SB48 ST44, night shimmer		1
27	1537041-0445	Cross-brace double, SB48 ST44, ivory white	20100701 →	1
27	1537041-0446	Cross-brace double, SB48 ST44, purple fire		1
27	1537041-0447	Cross-brace double, SB48 ST44, emerald green		1
27	1537041-0448	Cross-brace double, SB48 ST44, midnight blue		1
27	1537041-0449	Cross-brace double, SB48 ST44, umber glaze		1
27	1537041-0450	Cross-brace double, SB48 ST44, smokey grey		1
27	1537041-3524	Cross-brace double, SB48 ST44, royal red		1
27	1537041-5969	Cross-brace double, SB48 ST44, royal blue		1
27	1537041-9005	Cross-brace double, SB48 ST44, black, glossy		1
27	1537041-9119	Cross-brace double, SB48 ST44, black, bronze	20091001 → 20100630	1
28	1537042-0411	Cross-brace double, SB50 ST44, black, matt	20100701 →	1
28	1537042-0419	Cross-brace double, SB50 ST44, silver grey		1
28	1537042-0442	Cross-brace double, SB50 ST44, night shimmer		1
28	1537042-0445	Cross-brace double, SB50 ST44, ivory white	20100701 →	1
28	1537042-0446	Cross-brace double, SB50 ST44, purple fire		1
28	1537042-0447	Cross-brace double, SB50 ST44, emerald green		1
28	1537042-0448	Cross-brace double, SB50 ST44, midnight blue		1
28	1537042-0449	Cross-brace double, SB50 ST44, umber glaze		1
28	1537042-0450	Cross-brace double, SB50 ST44, smokey grey		1
28	1537042-3524	Cross-brace double, SB50 ST44, royal red		1

Cross brace double, SB44-50

Pos.	Part number	Description	Valid from → until	Qty
28	1537042-5969	Cross-brace double, SB50 ST44, royal blue		1
28	1537042-9005	Cross-brace double, SB50 ST44, black, glossy		1
28	1537042-9119	Cross-brace double, SB50 ST44, black, bronze	20091001 → 20100630	1
29	1537051-0411	Cross-brace double, SB44 ST46, black, matt	20100701 →	1
29	1537051-0419	Cross-brace double, SB44 ST46, silver grey		1
29	1537051-0442	Cross-brace double, SB44 ST46, night shimmer		1
29	1537051-0445	Cross-brace double, SB44 ST46, ivory white	20100701 →	1
29	1537051-0446	Cross-brace double, SB44 ST46, purple fire		1
29	1537051-0447	Cross-brace double, SB44 ST46, emerald green		1
29	1537051-0448	Cross-brace double, SB44 ST46, midnight blue		1
29	1537051-0449	Cross-brace double, SB44 ST46, umber glaze		1
29	1537051-0450	Cross-brace double, SB44 ST46, smokey grey		1
29	1537051-3524	Cross-brace double, SB44 ST46, royal red		1
29	1537051-5969	Cross-brace double, SB44 ST46, royal blue		1
29	1537051-9005	Cross-brace double, SB44 ST46, black, glossy		1
29	1537051-9119	Cross-brace double, SB44 ST46, black, bronze	20091001 → 20100630	1
30	1537052-0411	Cross-brace double, SB46 ST46, black, matt	20100701 →	1
30	1537052-0419	Cross-brace double, SB46 ST46, silver grey		1
30	1537052-0442	Cross-brace double, SB46 ST46, night shimmer		1
30	1537052-0445	Cross-brace double, SB46 ST46, ivory white	20100701 →	1
30	1537052-0446	Cross-brace double, SB46 ST46, purple fire		1
30	1537052-0447	Cross-brace double, SB46 ST46, emerald green		1
30	1537052-0448	Cross-brace double, SB46 ST46, midnight blue		1
30	1537052-0449	Cross-brace double, SB46 ST46, umber glaze		1
30	1537052-0450	Cross-brace double, SB46 ST46, smokey grey		1
30	1537052-3524	Cross-brace double, SB46 ST46, royal red		1
30	1537052-5969	Cross-brace double, SB46 ST46, royal blue		1
30	1537052-9005	Cross-brace double, SB46 ST46, black, glossy		1
30	1537052-9119	Cross-brace double, SB46 ST46, black, bronze	20091001 → 20100630	1
31	1537053-0411	Cross-brace double, SB48 ST46, black, matt	20100701 →	1
31	1537053-0419	Cross-brace double, SB48 ST46, silver grey		1
31	1537053-0442	Cross-brace double, SB48 ST46, night shimmer		1
31	1537053-0445	Cross-brace double, SB48 ST46, ivory white	20100701 →	1
31	1537053-0446	Cross-brace double, SB48 ST46, purple fire		1
31	1537053-0447	Cross-brace double, SB48 ST46, emerald green		1
31	1537053-0448	Cross-brace double, SB48 ST46, midnight blue		1
31	1537053-0449	Cross-brace double, SB48 ST46, umber glaze		1
31	1537053-0450	Cross-brace double, SB48 ST46, smokey grey		1
31	1537053-3524	Cross-brace double, SB48 ST46, royal red		1
31	1537053-5969	Cross-brace double, SB48 ST46, royal blue		1
31	1537053-9005	Cross-brace double, SB48 ST46, black, glossy		1
31	1537053-9119	Cross-brace double, SB48 ST46, black, bronze	20091001 → 20100630	1
32	1537054-0411	Cross-brace double, SB50 ST46, black, matt	20100701 →	1
32	1537054-0419	Cross-brace double, SB50 ST46, silver grey		1
32	1537054-0442	Cross-brace double, SB50 ST46, night shimmer		1
32	1537054-0445	Cross-brace double, SB50 ST46, ivory white	20100701 →	1
32	1537054-0446	Cross-brace double, SB50 ST46, purple fire		1
32	1537054-0447	Cross-brace double, SB50 ST46, emerald green		1
32	1537054-0448	Cross-brace double, SB50 ST46, midnight blue		1
32	1537054-0449	Cross-brace double, SB50 ST46, umber glaze		1
32	1537054-0450	Cross-brace double, SB50 ST46, smokey grey		1
32	1537054-3524	Cross-brace double, SB50 ST46, royal red		1
32	1537054-5969	Cross-brace double, SB50 ST46, royal blue		1
32	1537054-9005	Cross-brace double, SB50 ST46, black, glossy		1
32	1537054-9119	Cross-brace double, SB50 ST46, black, bronze	20091001 → 20100630	1

Cross brace double, SB44-50

Pos.	Part number	Description	Valid from → until	Qty
33	1537162-0411	Cross-brace double, SB44 ST48, black, matt	20100701 →	1
33	1537162-0419	Cross-brace double, SB44 ST48, silver grey		1
33	1537162-0442	Cross-brace double, SB44 ST48, night shimmer		1
33	1537162-0445	Cross-brace double, SB44 ST48, ivory white	20100701 →	1
33	1537162-0446	Cross-brace double, SB44 ST48, purple fire		1
33	1537162-0447	Cross-brace double, SB44 ST48, emerald green		1
33	1537162-0448	Cross-brace double, SB44 ST48, midnight blue		1
33	1537162-0449	Cross-brace double, SB44 ST48, umber glaze		1
33	1537162-0450	Cross-brace double, SB44 ST48, smokey grey		1
33	1537162-3524	Cross-brace double, SB44 ST48, royal red		1
33	1537162-5969	Cross-brace double, SB44 ST48, royal blue		1
33	1537162-9005	Cross-brace double, SB44 ST48, black, glossy		1
33	1537162-9119	Cross-brace double, SB44 ST48, black, bronze	20091001 → 20100630	1
34	1537163-0411	Cross-brace double, SB46 ST48, black, matt	20100701 →	1
34	1537163-0419	Cross-brace double, SB46 ST48, silver grey		1
34	1537163-0442	Cross-brace double, SB46 ST48, night shimmer		1
34	1537163-0445	Cross-brace double, SB46 ST48, ivory white	20100701 →	1
34	1537163-0446	Cross-brace double, SB46 ST48, purple fire		1
34	1537163-0447	Cross-brace double, SB46 ST48, emerald green		1
34	1537163-0448	Cross-brace double, SB46 ST48, midnight blue		1
34	1537163-0449	Cross-brace double, SB46 ST48, umber glaze		1
34	1537163-0450	Cross-brace double, SB46 ST48, smokey grey		1
34	1537163-3524	Cross-brace double, SB46 ST48, royal red		1
34	1537163-5969	Cross-brace double, SB46 ST48, royal blue		1
34	1537163-9005	Cross-brace double, SB46 ST48, black, glossy		1
34	1537163-9119	Cross-brace double, SB46 ST48, black, bronze	20091001 → 20100630	1
35	1537164-0411	Cross-brace double, SB48 ST48, black, matt	20100701 →	1
35	1537164-0419	Cross-brace double, SB48 ST48, silver grey		1
35	1537164-0442	Cross-brace double, SB48 ST48, night shimmer		1
35	1537164-0445	Cross-brace double, SB48 ST48, ivory white	20100701 →	1
35	1537164-0446	Cross-brace double, SB48 ST48, purple fire		1
35	1537164-0447	Cross-brace double, SB48 ST48, emerald green		1
35	1537164-0448	Cross-brace double, SB48 ST48, midnight blue		1
35	1537164-0449	Cross-brace double, SB48 ST48, umber glaze		1
35	1537164-0450	Cross-brace double, SB48 ST48, smokey grey		1
35	1537164-3524	Cross-brace double, SB48 ST48, royal red		1
35	1537164-5969	Cross-brace double, SB48 ST48, royal blue		1
35	1537164-9005	Cross-brace double, SB48 ST48, black, glossy		1
35	1537164-9119	Cross-brace double, SB48 ST48, black, bronze	20091001 → 20100630	1
36	1537165-0411	Cross-brace double, SB50 ST48, black, matt	20100701 →	1
36	1537165-0419	Cross-brace double, SB50 ST48, silver grey		1
36	1537165-0442	Cross-brace double, SB50 ST48, night shimmer		1
36	1537165-0445	Cross-brace double, SB50 ST48, ivory white	20100701 →	1
36	1537165-0446	Cross-brace double, SB50 ST48, purple fire		1
36	1537165-0447	Cross-brace double, SB50 ST48, emerald green		1
36	1537165-0448	Cross-brace double, SB50 ST48, midnight blue		1
36	1537165-0449	Cross-brace double, SB50 ST48, umber glaze		1
36	1537165-0450	Cross-brace double, SB50 ST48, smokey grey		1
36	1537165-3524	Cross-brace double, SB50 ST48, royal red		1
36	1537165-5969	Cross-brace double, SB50 ST48, royal blue		1
36	1537165-9005	Cross-brace double, SB50 ST48, black, glossy		1
36	1537165-9119	Cross-brace double, SB50 ST48, black, bronze	20091001 → 20100630	1
37	1537682-0411	Cross-brace double, SB44 ST50, black, matt	20100701 →	1
37	1537682-0419	Cross-brace double, SB44 ST50, silver grey		1
37	1537682-0442	Cross-brace double, SB44 ST50, night shimmer		1


Cross brace double, SB44-50

Pos.	Part number	Description	Valid from → until	Qty
37	1537682-0445	Cross-brace double, SB44 ST50, ivory white	20100701 →	1
37	1537682-0446	Cross-brace double, SB44 ST50, purple fire		1
37	1537682-0447	Cross-brace double, SB44 ST50, emerald green		1
37	1537682-0448	Cross-brace double, SB44 ST50, midnight blue		1
37	1537682-0449	Cross-brace double, SB44 ST50, umber glaze		1
37	1537682-0450	Cross-brace double, SB44 ST50, smokey grey		1
37	1537682-3524	Cross-brace double, SB44 ST50, royal red		1
37	1537682-5969	Cross-brace double, SB44 ST50, royal blue		1
37	1537682-9005	Cross-brace double, SB44 ST50, black, glossy		1
37	1537682-9119	Cross-brace double, SB44 ST50, black, bronze	20091001 → 20100630	1
38	1537683-0411	Cross-brace double, SB46 ST50, black, matt	20100701 →	1
38	1537683-0419	Cross-brace double, SB46 ST50, silver grey		1
38	1537683-0442	Cross-brace double, SB46 ST50, night shimmer		1
38	1537683-0445	Cross-brace double, SB46 ST50, ivory white	20100701 →	1
38	1537683-0446	Cross-brace double, SB46 ST50, purple fire		1
38	1537683-0447	Cross-brace double, SB46 ST50, emerald green		1
38	1537683-0448	Cross-brace double, SB46 ST50, midnight blue		1
38	1537683-0449	Cross-brace double, SB46 ST50, umber glaze		1
38	1537683-0450	Cross-brace double, SB46 ST50, smokey grey		1
38	1537683-3524	Cross-brace double, SB46 ST50, royal red		1
38	1537683-5969	Cross-brace double, SB46 ST50, royal blue		1
38	1537683-9005	Cross-brace double, SB46 ST50, black, glossy		1
38	1537683-9119	Cross-brace double, SB46 ST50, black, bronze	20091001 → 20100630	1
39	1537684-0411	Cross-brace double, SB48 ST50, black, matt	20100701 →	1
39	1537684-0419	Cross-brace double, SB48 ST50, silver grey		1
39	1537684-0442	Cross-brace double, SB48 ST50, night shimmer		1
39	1537684-0445	Cross-brace double, SB48 ST50, ivory white	20100701 →	1
39	1537684-0446	Cross-brace double, SB48 ST50, purple fire		1
39	1537684-0447	Cross-brace double, SB48 ST50, emerald green		1
39	1537684-0448	Cross-brace double, SB48 ST50, midnight blue		1
39	1537684-0449	Cross-brace double, SB48 ST50, umber glaze		1
39	1537684-0450	Cross-brace double, SB48 ST50, smokey grey		1
39	1537684-3524	Cross-brace double, SB48 ST50, royal red		1
39	1537684-5969	Cross-brace double, SB48 ST50, royal blue		1
39	1537684-9005	Cross-brace double, SB48 ST50, black, glossy		1
39	1537684-9119	Cross-brace double, SB48 ST50, black, bronze	20091001 → 20100630	1
40	1537685-0411	Cross-brace double, SB50 ST50, black, matt	20100701 →	1
40	1537685-0419	Cross-brace double, SB50 ST50, silver grey		1
40	1537685-0442	Cross-brace double, SB50 ST50, night shimmer		1
40	1537685-0445	Cross-brace double, SB50 ST50, ivory white	20100701 →	1
40	1537685-0446	Cross-brace double, SB50 ST50, purple fire		1
40	1537685-0447	Cross-brace double, SB50 ST50, emerald green		1
40	1537685-0448	Cross-brace double, SB50 ST50, midnight blue		1
40	1537685-0449	Cross-brace double, SB50 ST50, umber glaze		1
40	1537685-0450	Cross-brace double, SB50 ST50, smokey grey		1
40	1537685-3524	Cross-brace double, SB50 ST50, royal red		1
40	1537685-5969	Cross-brace double, SB50 ST50, royal blue		1
40	1537685-9005	Cross-brace double, SB50 ST50, black, glossy		1
40	1537685-9119	Cross-brace double, SB50 ST50, black, bronze	20091001 → 20100630	1
41	1536085	Retaining lever SB44/48, pair		1
42	1536086	Retaining lever SB46/50, pair		1
43	1537686	Seat lock kit		1
44	1537688	Seat lock kit with sleeves		1
45	1537690	Screw kit to Double crossbrace		1
46	1537687-0004	Cross-brace plug kit, black anodized		1

Cross brace double, SB44-50

Pos.	Part number	Description	Valid from → until	Qty
46	1537687-0400	Cross-brace plug kit, techno-anthrazit		1
46	1537687-0401	Cross-brace plug kit, lemon		1
46	1537687-0402	Cross-brace plug kit, apricot		1
46	1537687-0404	Cross-brace plug kit, b-blue		1
46	1537687-0440	Cross-brace plug kit, comet red		1
46	1537687-0441	Cross-brace plug kit, arctic blue		1
46	1537687-0444	Cross-brace plug kit, copper glaze		1

Rear frames

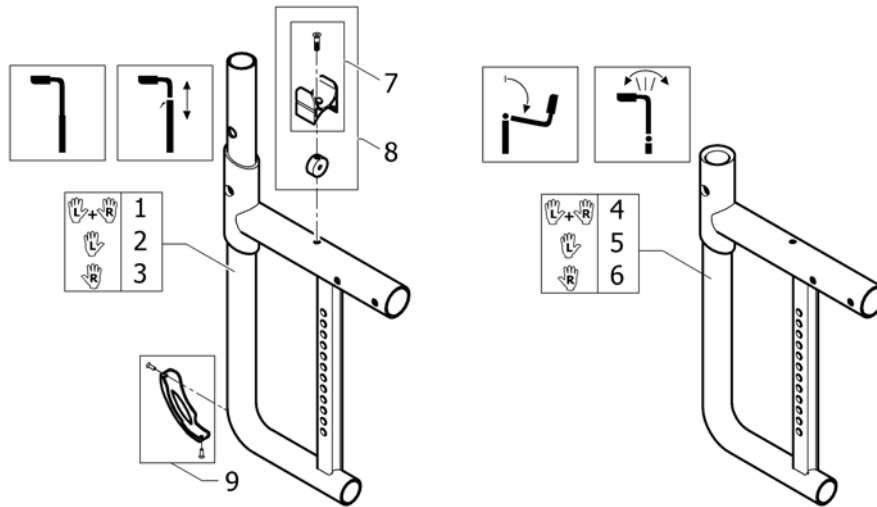

Pos.	Part number	Description	Valid from → until	Qty
1	1539779-0411	Rear frame, black, matt, Pair	20100701 →	1
1	1539779-0419	Rear frame, silver grey, Pair		1
1	1539779-0442	Rear frame, night shimmer, Pair		1
1	1539779-0445	Rear frame, ivory white, Pair	20100701 →	1
1	1539779-0446	Rear frame, purple fire, Pair		1
1	1539779-0447	Rear frame, emerald green, Pair		1
1	1539779-0448	Rear frame, midnight blue, Pair		1
1	1539779-0449	Rear frame, umber glaze, Pair		1
1	1539779-0450	Rear frame, smokey grey, Pair		1
1	1539779-3524	Rear frame, royal red, Pair		1
1	1539779-5969	Rear frame, royal blue, Pair		1
1	1539779-9005	Rear frame, black, glossy, Pair		1
1	1539779-9119	Rear frame, black, bronze, Pair	20091001 → 20100630	1
2	1539780-0411	Rear frame left, black, matt	20100701 →	1
2	1539780-0419	Rear frame left, silver grey		1
2	1539780-0442	Rear frame left, night shimmer		1
2	1539780-0445	Rear frame left, ivory white	20100701 →	1
2	1539780-0446	Rear frame left, purple fire		1
2	1539780-0447	Rear frame left, emerald green		1
2	1539780-0448	Rear frame left, midnight blue		1
2	1539780-0449	Rear frame left, umber glaze		1
2	1539780-0450	Rear frame left, smokey grey		1
2	1539780-3524	Rear frame left, royal red		1
2	1539780-5969	Rear frame left, royal blue		1
2	1539780-9005	Rear frame left, black, glossy		1
2	1539780-9119	Rear frame left, black, bronze	20091001 → 20100630	1
3	1539781-0411	Rear frame right, black, matt	20100701 →	1
3	1539781-0419	Rear frame right, silver grey		1
3	1539781-0442	Rear frame right, night shimmer		1
3	1539781-0445	Rear frame right, ivory white	20100701 →	1
3	1539781-0446	Rear frame right, purple fire		1


Rear frames

Pos.	Part number	Description	Valid from → until	Qty
3	1539781-0447	Rear frame right, emerald green		1
3	1539781-0448	Rear frame right, midnight blue		1
3	1539781-0449	Rear frame right, umber glaze		1
3	1539781-0450	Rear frame right, smokey grey		1
3	1539781-3524	Rear frame right, royal red		1
3	1539781-5969	Rear frame right, royal blue		1
3	1539781-9005	Rear frame right, black, glossy		1
3	1539781-9119	Rear frame right, black, bronze	20091001 → 20100630	1
4	1539974-0411	Rear frame short to foldable back or to angle adjustable back, black, matt , Pair	20100701 →	1
4	1539974-0419	Rear frame short to foldable back or to angle adjustable back, silver grey, Pair		1
4	1539974-0442	Rear frame short to foldable back or to angle adjustable back, night shimmer , Pair		1
4	1539974-0445	Rear frame short to foldable back or to angle adjustable back, ivory white, Pair	20100701 →	1
4	1539974-0446	Rear frame short to foldable back or to angle adjustable back, purple fire, Pair		1
4	1539974-0447	Rear frame short to foldable back or to angle adjustable back, emerald green , Pair		1
4	1539974-0448	Rear frame short to foldable back or to angle adjustable back, midnight blue , Pair		1
4	1539974-0449	Rear frame short to foldable back or to angle adjustable back, umber glaze, Pair		1
4	1539974-0450	Rear frame short to foldable back or to angle adjustable back, smokey grey, Pair		1
4	1539974-3524	Rear frame short to foldable back or to angle adjustable back, royal red, Pair		1
4	1539974-5969	Rear frame short to foldable back or to angle adjustable back, royal blue, Pair		1
4	1539974-9005	Rear frame short to foldable back or to angle adjustable back, black, glossy , Pair		1
4	1539974-9119	Rear frame short to foldable back or to angle adjustable back, black, bronze , Pair	20091001 → 20100630	1
5	1539975-0411	Rear frame left short to foldable back or to angle adjustable back, black, matt	20100701 →	1
5	1539975-0419	Rear frame left short to foldable back or to angle adjustable back, silver grey		1
5	1539975-0442	Rear frame left short to foldable back or to angle adjustable back, night shimmer		1
5	1539975-0445	Rear frame left short to foldable back or to angle adjustable back, ivory white	20100701 →	1
5	1539975-0446	Rear frame left short to foldable back or to angle adjustable back, purple fire		1
5	1539975-0447	Rear frame left short to foldable back or to angle adjustable back, emerald green		1
5	1539975-0448	Rear frame left short to foldable back or to angle adjustable back, midnight blue		1
5	1539975-0449	Rear frame left short to foldable back or to angle adjustable back, umber glaze		1
5	1539975-0450	Rear frame left short to foldable back or to angle adjustable back, smokey grey		1
5	1539975-3524	Rear frame left short to foldable back or to angle adjustable back, royal red		1
5	1539975-5969	Rear frame left short to foldable back or to angle adjustable back, royal blue		1
5	1539975-9005	Rear frame left short to foldable back or to angle adjustable back, black, glossy		1
5	1539975-9119	Rear frame left short to foldable back or to angle adjustable back, black, bronze	20091001 → 20100630	1
6	1539976-0411	Rear frame right short to foldable back or to angle adjustable back, black, matt	20100701 →	1

Rear frames


Pos.	Part number	Description	Valid from → until	Qty
6	1539976-0419	Rear frame right short to foldable back or to angle adjustable back, silver grey		1
6	1539976-0442	Rear frame right short to foldable back or to angle adjustable back, night shimmer		1
6	1539976-0445	Rear frame right short to foldable back or to angle adjustable back, ivory white	20100701 →	1
6	1539976-0446	Rear frame right short to foldable back or to angle adjustable back, purple fire		1
6	1539976-0447	Rear frame right short to foldable back or to angle adjustable back, emerald green		1
6	1539976-0448	Rear frame right short to foldable back or to angle adjustable back, midnight blue		1
6	1539976-0449	Rear frame right short to foldable back or to angle adjustable back, umber glaze		1
6	1539976-0450	Rear frame right short to foldable back or to angle adjustable back, smokey grey		1
6	1539976-3524	Rear frame right short to foldable back or to angle adjustable back, royal red		1
6	1539976-5969	Rear frame right short to foldable back or to angle adjustable back, royal blue		1
6	1539976-9005	Rear frame right short to foldable back or to angle adjustable back, black, glossy		1
6	1539976-9119	Rear frame right short to foldable back or to angle adjustable back, black, bronze	20091001 → 20100630	1
7	1537686	Seat lock kit		1
8	1537688	Seat lock kit with sleeves		1
9	1498774	Rear frame protection, pair		1

Rear wheel fixation

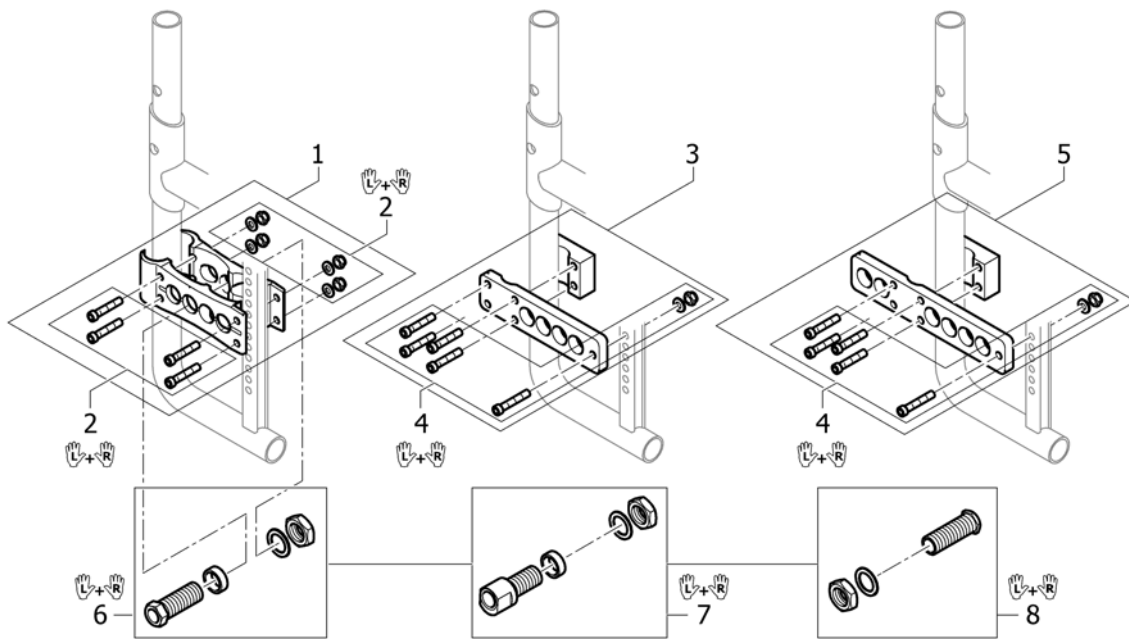


Pos.	Part number	Description	Valid from → until	Qty
1	1539962-0004	adapterplate set black		2
1	1539962-0400	adapterplate set Tecno		2
1	1539962-0401	adapterplate set Lemon		2
1	1539962-0402	adapterplate set Apricot		2
1	1539962-0404	adapterplate set B-Blue		2
1	1539962-0440	adapterplate set comet red		2
1	1539962-0441	adapterplate set arctic blue		2
1	1539962-0444	adapterplate set copper glaze		2
2	1539985	Fixation kit for adapter plate, standard		1
3	1539989	Adapterplate set for drumbrake or 4° with siderest		2
4	1539986	Fixation kit to adapter plate, drumbrake		1
5	1539990	Adapterplate set for Rearwheelextension		2
6	1539987	Adaptersleeve set 1°, pair		1
7	1539988	Adaptersleeve set 4°, pair		1
8	1527082	Adaptersleeve set 1° for siderest with desc function, pair		1

SEAT

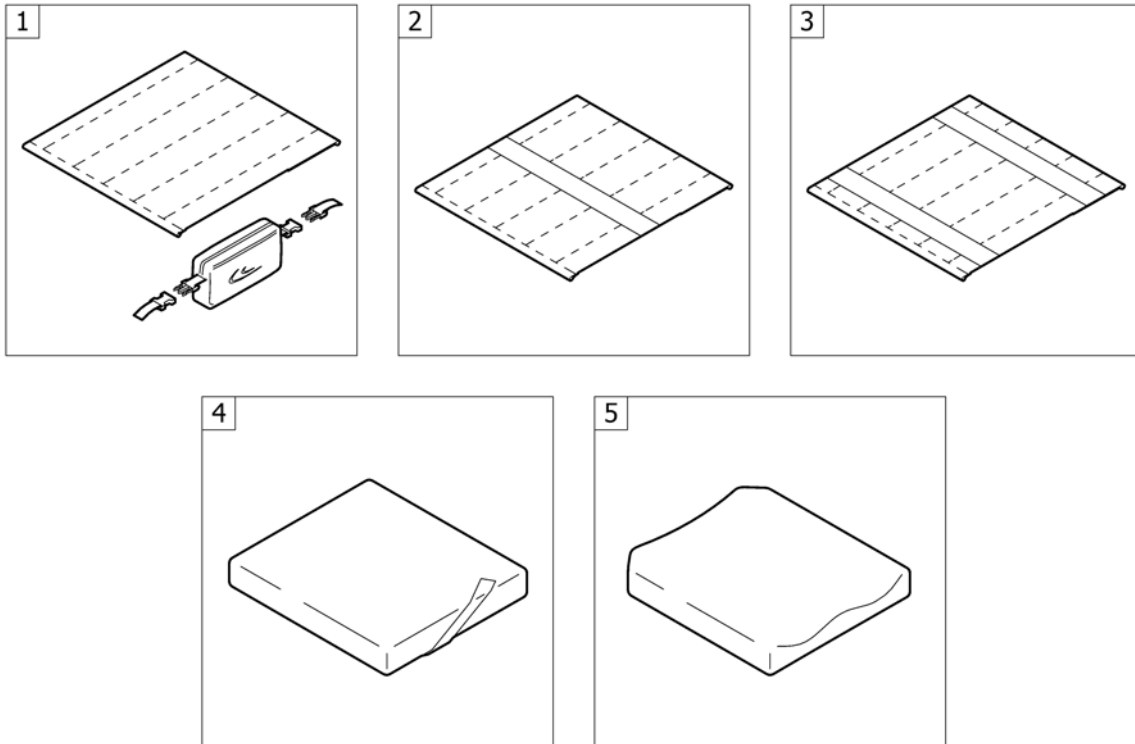


Pos.	Part number	Description	Valid from → until	Qty
1	-	Seat cover standard		1
2	-	Seat cover with 1 Velcro band (soft band)		1
3	-	Seat cover with 2 Velcro bands (rough band)		1
4	-	Seat cushion standard		1
5	-	Seat cushion Flo-tech		1

Seat cover standard

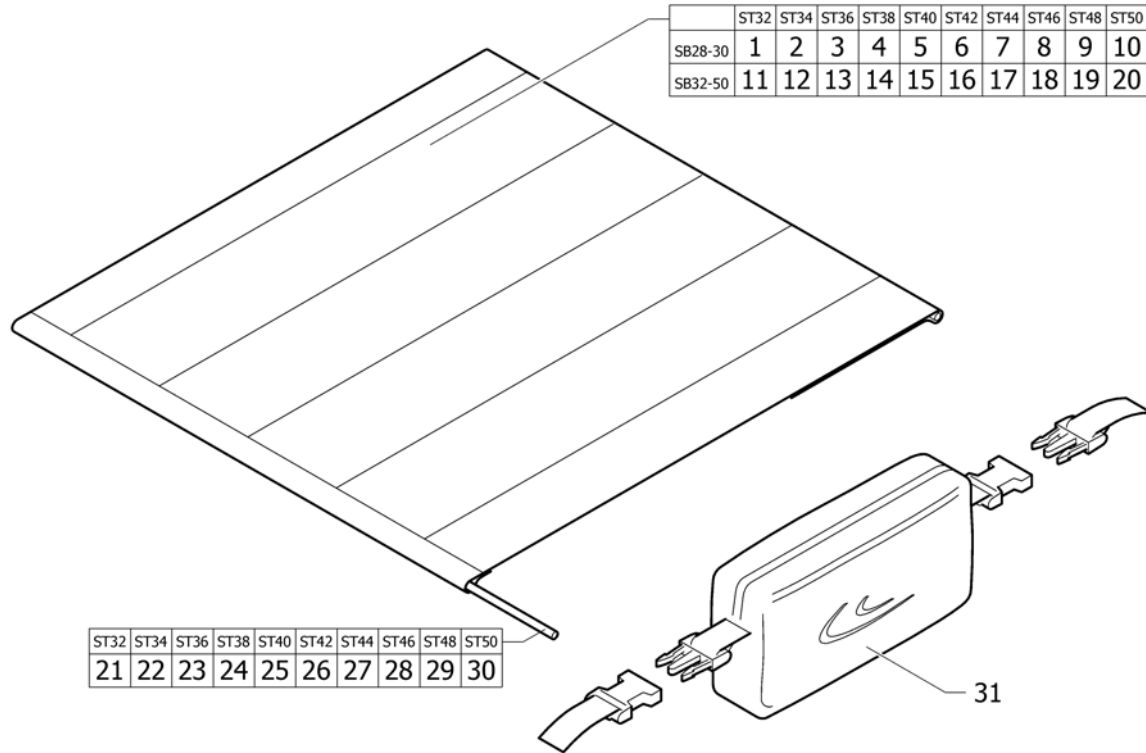

Pos.	Part number	Description	Valid from → until	Qty
1	1540150	Seat cover ST32 SB var. 28-30		1
2	1540151	Seat cover ST34 SB var. 28-30		1
3	1540152	Seat cover ST36 SB var. 28-30		1
4	1540153	Seat cover ST38 SB var. 28-30		1
5	1540154	Seat cover ST40 SB var. 28-30		1
6	1540155	Seat cover ST42 SB var. 28-30		1
7	1540156	Seat cover ST44 SB var. 28-30		1
8	1540157	Seat cover ST46 SB var. 28-30		1
9	1540158	Seat cover ST48 SB var. 28-30		1
10	1540159	Seat cover ST50 SB var. 28-30		1
11	1540160	Seat cover ST32 SB var. 32-50		1
12	1540161	Seat cover ST34 SB var. 32-50		1
13	1540162	Seat cover ST36 SB var. 32-50		1
14	1540163	Seat cover ST38 SB var. 32-50		1
15	1540164	Seat cover ST40 SB var. 32-50		1
16	1540165	Seat cover ST42 SB var. 32-50		1
17	1540166	Seat cover ST44 SB var. 32-50		1
18	1540167	Seat cover ST46 SB var. 32-50		1
19	1540168	Seat cover ST48 SB var. 32-50		1
20	1540169	Seat cover ST50 SB var. 32-50		1
21	1540170	Seat cover bar kit ST32		1
22	1540171	Seat cover bar kit ST34		1
23	1540172	Seat cover bar kit ST36		1
24	1540173	Seat cover bar kit ST38		1
25	1540174	Seat cover bar kit ST40		1
26	1540175	Seat cover bar kit ST42		1
27	1540176	Seat cover bar kit ST44		1
28	1540177	Seat cover bar kit ST46		1
29	1540178	Seat cover bar kit ST48		1
30	1540179	Seat cover bar kit ST50		1
31	1529341	Seat bag		1

Seat cover with 1 Velcro band (soft band)

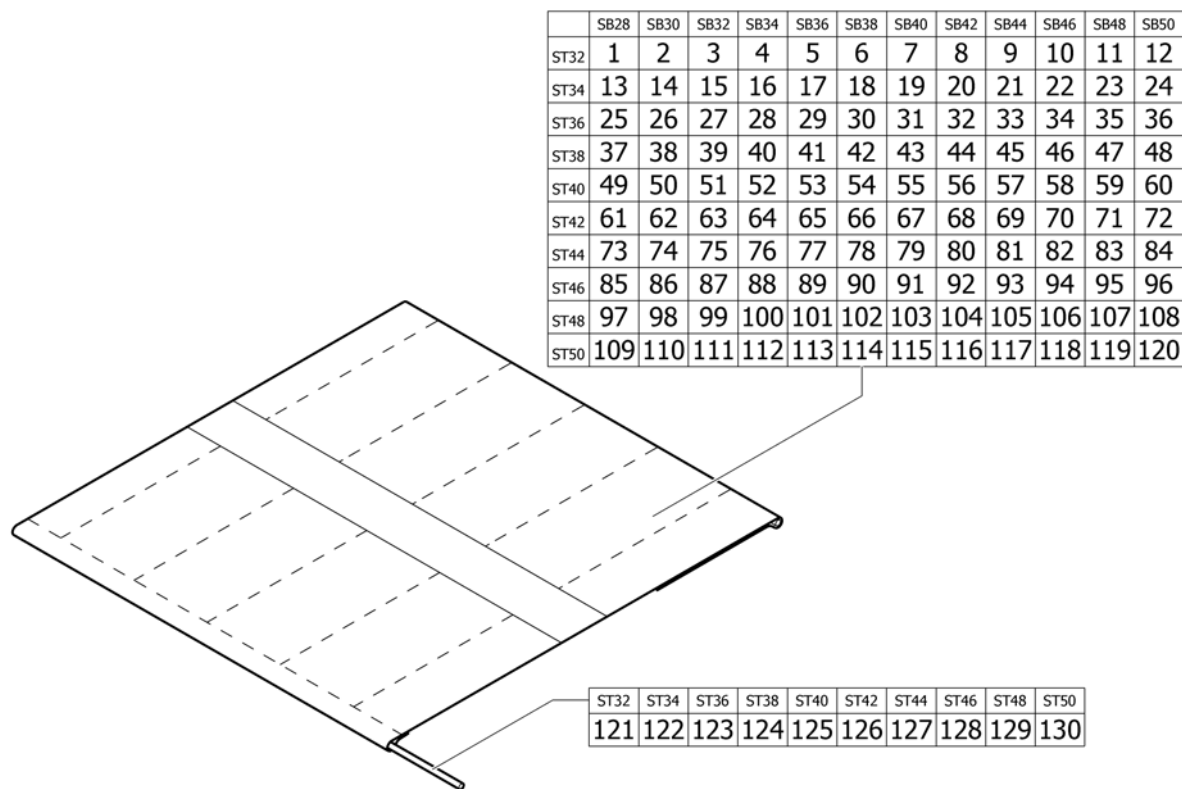

Pos.	Part number	Description	Valid from → until	Qty
1	1540180	Seat cover SB28 ST32 with 1 Velcro band (soft band)		1
2	1540181	Seat cover SB30 ST32 with 1 Velcro band (soft band)		1
3	1540182	Seat cover SB32 ST32 with 1 Velcro band (soft band)		1
4	1540183	Seat cover SB34 ST32 with 1 Velcro band (soft band)		1
5	1540184	Seat cover SB36 ST32 with 1 Velcro band (soft band)		1
6	1540185	Seat cover SB38 ST32 with 1 Velcro band (soft band)		1
7	1540186	Seat cover SB40 ST32 with 1 Velcro band (soft band)		1
8	1540187	Seat cover SB42 ST32 with 1 Velcro band (soft band)		1
9	1540188	Seat cover SB44 ST32 with 1 Velcro band (soft band)		1
10	1540189	Seat cover SB46 ST32 with 1 Velcro band (soft band)		1
11	1540190	Seat cover SB48 ST32 with 1 Velcro band (soft band)		1
12	1540191	Seat cover SB50 ST32 with 1 Velcro band (soft band)		1
13	1540192	Seat cover SB28 ST34 with 1 Velcro band (soft band)		1
14	1540193	Seat cover SB30 ST34 with 1 Velcro band (soft band)		1
15	1540194	Seat cover SB32 ST34 with 1 Velcro band (soft band)		1
16	1540195	Seat cover SB34 ST34 with 1 Velcro band (soft band)		1
17	1540196	Seat cover SB36 ST34 with 1 Velcro band (soft band)		1
18	1540197	Seat cover SB38 ST34 with 1 Velcro band (soft band)		1
19	1540198	Seat cover SB40 ST34 with 1 Velcro band (soft band)		1
20	1540199	Seat cover SB42 ST34 with 1 Velcro band (soft band)		1
21	1540200	Seat cover SB44 ST34 with 1 Velcro band (soft band)		1
22	1540201	Seat cover SB46 ST34 with 1 Velcro band (soft band)		1
23	1540202	Seat cover SB48 ST34 with 1 Velcro band (soft band)		1
24	1540203	Seat cover SB50 ST34 with 1 Velcro band (soft band)		1
25	1540204	Seat cover SB28 ST36 with 1 Velcro band (soft band)		1
26	1540205	Seat cover SB30 ST36 with 1 Velcro band (soft band)		1
27	1540206	Seat cover SB32 ST36 with 1 Velcro band (soft band)		1
28	1540207	Seat cover SB34 ST36 with 1 Velcro band (soft band)		1
29	1540208	Seat cover SB36 ST36 with 1 Velcro band (soft band)		1
30	1540209	Seat cover SB38 ST36 with 1 Velcro band (soft band)		1
31	1540210	Seat cover SB40 ST36 with 1 Velcro band (soft band)		1


Seat cover with 1 Velcro band (soft band)

Pos.	Part number	Description	Valid from → until	Qty
32	1540211	Seat cover SB42 ST36 with 1 Velcro band (soft band)		1
33	1540212	Seat cover SB44 ST36 with 1 Velcro band (soft band)		1
34	1540213	Seat cover SB46 ST36 with 1 Velcro band (soft band)		1
35	1540214	Seat cover SB48 ST36 with 1 Velcro band (soft band)		1
36	1540215	Seat cover SB50 ST36 with 1 Velcro band (soft band)		1
37	1540216	Seat cover SB28 ST38 with 1 Velcro band (soft band)		1
38	1540217	Seat cover SB30 ST38 with 1 Velcro band (soft band)		1
39	1540218	Seat cover SB32 ST38 with 1 Velcro band (soft band)		1
40	1540219	Seat cover SB34 ST38 with 1 Velcro band (soft band)		1
41	1540220	Seat cover SB36 ST38 with 1 Velcro band (soft band)		1
42	1540221	Seat cover SB38 ST38 with 1 Velcro band (soft band)		1
43	1540222	Seat cover SB40 ST38 with 1 Velcro band (soft band)		1
44	1540223	Seat cover SB42 ST38 with 1 Velcro band (soft band)		1
45	1540224	Seat cover SB44 ST38 with 1 Velcro band (soft band)		1
46	1540225	Seat cover SB46 ST38 with 1 Velcro band (soft band)		1
47	1540226	Seat cover SB48 ST38 with 1 Velcro band (soft band)		1
48	1540227	Seat cover SB50 ST38 with 1 Velcro band (soft band)		1
49	1540228	Seat cover SB28 ST40 with 1 Velcro band (soft band)		1
50	1540229	Seat cover SB30 ST40 with 1 Velcro band (soft band)		1
51	1540230	Seat cover SB32 ST40 with 1 Velcro band (soft band)		1
52	1540231	Seat cover SB34 ST40 with 1 Velcro band (soft band)		1
53	1540232	Seat cover SB36 ST40 with 1 Velcro band (soft band)		1
54	1540233	Seat cover SB38 ST40 with 1 Velcro band (soft band)		1
55	1540234	Seat cover SB40 ST40 with 1 Velcro band (soft band)		1
56	1540235	Seat cover SB42 ST40 with 1 Velcro band (soft band)		1
57	1540236	Seat cover SB44 ST40 with 1 Velcro band (soft band)		1
58	1540237	Seat cover SB46 ST40 with 1 Velcro band (soft band)		1
59	1540238	Seat cover SB48 ST40 with 1 Velcro band (soft band)		1
60	1540239	Seat cover SB50 ST40 with 1 Velcro band (soft band)		1
61	1540240	Seat cover SB28 ST42 with 1 Velcro band (soft band)		1
62	1540241	Seat cover SB30 ST42 with 1 Velcro band (soft band)		1
63	1540242	Seat cover SB32 ST42 with 1 Velcro band (soft band)		1
64	1540243	Seat cover SB34 ST42 with 1 Velcro band (soft band)		1
65	1540244	Seat cover SB36 ST42 with 1 Velcro band (soft band)		1
66	1540245	Seat cover SB38 ST42 with 1 Velcro band (soft band)		1
67	1540246	Seat cover SB40 ST42 with 1 Velcro band (soft band)		1
68	1540247	Seat cover SB42 ST42 with 1 Velcro band (soft band)		1
69	1540248	Seat cover SB44 ST42 with 1 Velcro band (soft band)		1
70	1540249	Seat cover SB46 ST42 with 1 Velcro band (soft band)		1
71	1540250	Seat cover SB48 ST42 with 1 Velcro band (soft band)		1
72	1540251	Seat cover SB50 ST42 with 1 Velcro band (soft band)		1
73	1540252	Seat cover SB28 ST44 with 1 Velcro band (soft band)		1
74	1540253	Seat cover SB30 ST44 with 1 Velcro band (soft band)		1
75	1540254	Seat cover SB32 ST44 with 1 Velcro band (soft band)		1
76	1540255	Seat cover SB34 ST44 with 1 Velcro band (soft band)		1
77	1540256	Seat cover SB36 ST44 with 1 Velcro band (soft band)		1
78	1540257	Seat cover SB38 ST44 with 1 Velcro band (soft band)		1
79	1540258	Seat cover SB40 ST44 with 1 Velcro band (soft band)		1
80	1540259	Seat cover SB42 ST44 with 1 Velcro band (soft band)		1
81	1540682	Seat cover SB44 ST44 with 1 Velcro band (soft band)		1
82	1540683	Seat cover SB46 ST44 with 1 Velcro band (soft band)		1
83	1540684	Seat cover SB48 ST44 with 1 Velcro band (soft band)		1
84	1540685	Seat cover SB50 ST44 with 1 Velcro band (soft band)		1
85	1540686	Seat cover SB28 ST46 with 1 Velcro band (soft band)		1
86	1540687	Seat cover SB30 ST46 with 1 Velcro band (soft band)		1

Seat cover with 1 Velcro band (soft band)

Pos.	Part number	Description	Valid from → until	Qty
87	1540688	Seat cover SB32 ST46 with 1 Velcro band (soft band)		1
88	1540689	Seat cover SB34 ST46 with 1 Velcro band (soft band)		1
89	1540690	Seat cover SB36 ST46 with 1 Velcro band (soft band)		1
90	1540691	Seat cover SB38 ST46 with 1 Velcro band (soft band)		1
91	1540692	Seat cover SB40 ST46 with 1 Velcro band (soft band)		1
92	1540693	Seat cover SB42 ST46 with 1 Velcro band (soft band)		1
93	1540694	Seat cover SB44 ST46 with 1 Velcro band (soft band)		1
94	1540695	Seat cover SB46 ST46 with 1 Velcro band (soft band)		1
95	1540696	Seat cover SB48 ST46 with 1 Velcro band (soft band)		1
96	1540697	Seat cover SB50 ST46 with 1 Velcro band (soft band)		1
97	1540698	Seat cover SB28 ST48 with 1 Velcro band (soft band)		1
98	1540699	Seat cover SB30 ST48 with 1 Velcro band (soft band)		1
99	1540700	Seat cover SB32 ST48 with 1 Velcro band (soft band)		1
100	1540701	Seat cover SB34 ST48 with 1 Velcro band (soft band)		1
101	1540702	Seat cover SB36 ST48 with 1 Velcro band (soft band)		1
102	1540703	Seat cover SB38 ST48 with 1 Velcro band (soft band)		1
103	1540704	Seat cover SB40 ST48 with 1 Velcro band (soft band)		1
104	1540705	Seat cover SB42 ST48 with 1 Velcro band (soft band)		1
105	1540706	Seat cover SB44 ST48 with 1 Velcro band (soft band)		1
106	1540707	Seat cover SB46 ST48 with 1 Velcro band (soft band)		1
107	1540708	Seat cover SB48 ST48 with 1 Velcro band (soft band)		1
108	1540709	Seat cover SB50 ST48 with 1 Velcro band (soft band)		1
109	1540710	Seat cover SB28 ST50 with 1 Velcro band (soft band)		1
110	1540711	Seat cover SB30 ST50 with 1 Velcro band (soft band)		1
111	1540712	Seat cover SB32 ST50 with 1 Velcro band (soft band)		1
112	1540713	Seat cover SB34 ST50 with 1 Velcro band (soft band)		1
113	1540714	Seat cover SB36 ST50 with 1 Velcro band (soft band)		1
114	1540715	Seat cover SB38 ST50 with 1 Velcro band (soft band)		1
115	1540716	Seat cover SB40 ST50 with 1 Velcro band (soft band)		1
116	1540717	Seat cover SB42 ST50 with 1 Velcro band (soft band)		1
117	1540718	Seat cover SB44 ST50 with 1 Velcro band (soft band)		1
118	1540719	Seat cover SB46 ST50 with 1 Velcro band (soft band)		1
119	1540720	Seat cover SB48 ST50 with 1 Velcro band (soft band)		1
120	1540721	Seat cover SB50 ST50 with 1 Velcro band (soft band)		1
121	1540170	Seat cover bar kit ST32		1
122	1540171	Seat cover bar kit ST34		1
123	1540172	Seat cover bar kit ST36		1
124	1540173	Seat cover bar kit ST38		1
125	1540174	Seat cover bar kit ST40		1
126	1540175	Seat cover bar kit ST42		1
127	1540176	Seat cover bar kit ST44		1
128	1540177	Seat cover bar kit ST46		1
129	1540178	Seat cover bar kit ST48		1
130	1540179	Seat cover bar kit ST50		1

Seat cover with 2 Velcro bands (rough band)

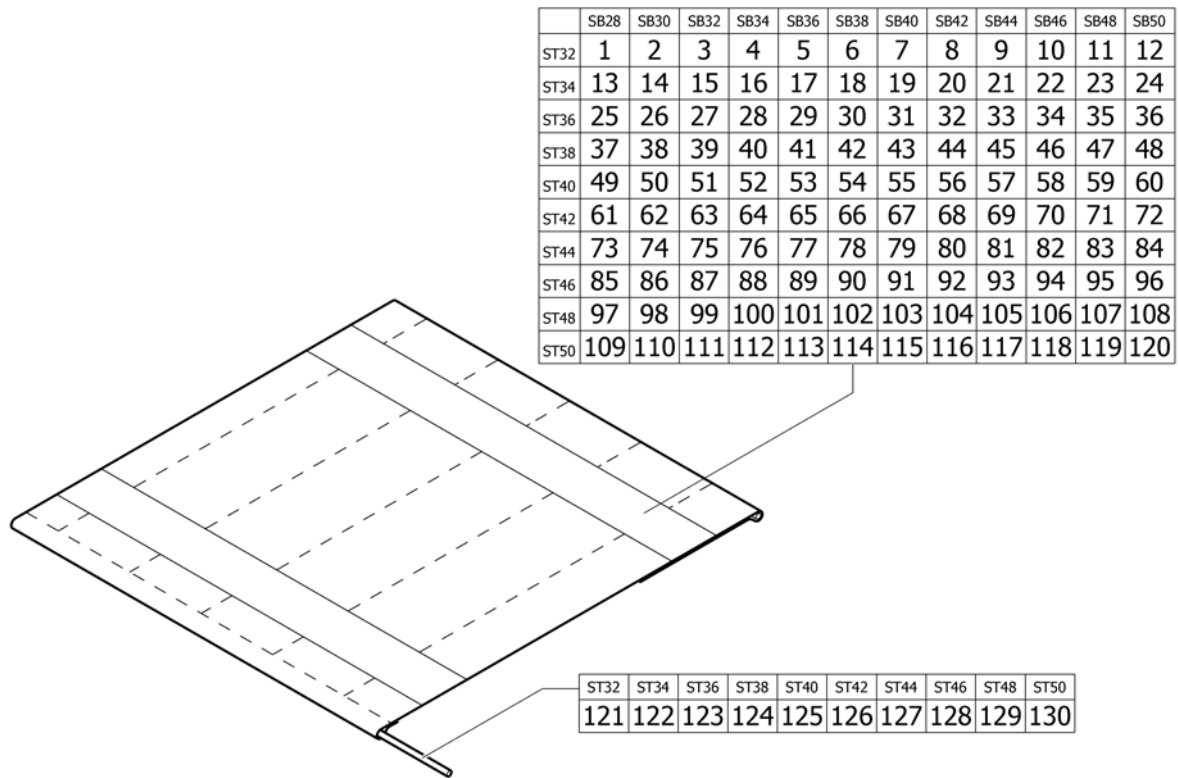

Pos.	Part number	Description	Valid from → until	Qty
1	1540722	Seat cover SB28 ST32 with 2 Velcro bands (rough band)		1
2	1540723	Seat cover SB30 ST32 with 2 Velcro bands (rough band)		1
3	1540724	Seat cover SB32 ST32 with 2 Velcro bands (rough band)		1
4	1540725	Seat cover SB34 ST32 with 2 Velcro bands (rough band)		1
5	1540726	Seat cover SB36 ST32 with 2 Velcro bands (rough band)		1
6	1540727	Seat cover SB38 ST32 with 2 Velcro bands (rough band)		1
7	1540728	Seat cover SB40 ST32 with 2 Velcro bands (rough band)		1
8	1540729	Seat cover SB42 ST32 with 2 Velcro bands (rough band)		1
9	1540730	Seat cover SB44 ST32 with 2 Velcro bands (rough band)		1
10	1540731	Seat cover SB46 ST32 with 2 Velcro bands (rough band)		1
11	1540732	Seat cover SB48 ST32 with 2 Velcro bands (rough band)		1
12	1540733	Seat cover SB50 ST32 with 2 Velcro bands (rough band)		1
13	1540734	Seat cover SB28 ST34 with 2 Velcro bands (rough band)		1
14	1540735	Seat cover SB30 ST34 with 2 Velcro bands (rough band)		1
15	1540736	Seat cover SB32 ST34 with 2 Velcro bands (rough band)		1
16	1540737	Seat cover SB34 ST34 with 2 Velcro bands (rough band)		1
17	1540738	Seat cover SB36 ST34 with 2 Velcro bands (rough band)		1
18	1540739	Seat cover SB38 ST34 with 2 Velcro bands (rough band)		1
19	1540740	Seat cover SB40 ST34 with 2 Velcro bands (rough band)		1
20	1540741	Seat cover SB42 ST34 with 2 Velcro bands (rough band)		1
21	1540742	Seat cover SB44 ST34 with 2 Velcro bands (rough band)		1
22	1540743	Seat cover SB46 ST34 with 2 Velcro bands (rough band)		1
23	1540744	Seat cover SB48 ST34 with 2 Velcro bands (rough band)		1
24	1540745	Seat cover SB50 ST34 with 2 Velcro bands (rough band)		1
25	1540746	Seat cover SB28 ST36 with 2 Velcro bands (rough band)		1
26	1540747	Seat cover SB30 ST36 with 2 Velcro bands (rough band)		1
27	1540748	Seat cover SB32 ST36 with 2 Velcro bands (rough band)		1
28	1540749	Seat cover SB34 ST36 with 2 Velcro bands (rough band)		1
29	1540750	Seat cover SB36 ST36 with 2 Velcro bands (rough band)		1
30	1540751	Seat cover SB38 ST36 with 2 Velcro bands (rough band)		1
31	1540752	Seat cover SB40 ST36 with 2 Velcro bands (rough band)		1


Seat cover with 2 Velcro bands (rough band)

Pos.	Part number	Description	Valid from → until	Qty
32	1540753	Seat cover SB42 ST36 with 2 Velcro bands (rough band)		1
33	1540754	Seat cover SB44 ST36 with 2 Velcro bands (rough band)		1
34	1540755	Seat cover SB46 ST36 with 2 Velcro bands (rough band)		1
35	1540756	Seat cover SB48 ST36 with 2 Velcro bands (rough band)		1
36	1540757	Seat cover SB50 ST36 with 2 Velcro bands (rough band)		1
37	1540758	Seat cover SB28 ST38 with 2 Velcro bands (rough band)		1
38	1540759	Seat cover SB30 ST38 with 2 Velcro bands (rough band)		1
39	1540760	Seat cover SB32 ST38 with 2 Velcro bands (rough band)		1
40	1540761	Seat cover SB34 ST38 with 2 Velcro bands (rough band)		1
41	1540762	Seat cover SB36 ST38 with 2 Velcro bands (rough band)		1
42	1540763	Seat cover SB38 ST38 with 2 Velcro bands (rough band)		1
43	1540764	Seat cover SB40 ST38 with 2 Velcro bands (rough band)		1
44	1540765	Seat cover SB42 ST38 with 2 Velcro bands (rough band)		1
45	1540766	Seat cover SB44 ST38 with 2 Velcro bands (rough band)		1
46	1540767	Seat cover SB46 ST38 with 2 Velcro bands (rough band)		1
47	1540768	Seat cover SB48 ST38 with 2 Velcro bands (rough band)		1
48	1540769	Seat cover SB50 ST38 with 2 Velcro bands (rough band)		1
49	1540770	Seat cover SB28 ST40 with 2 Velcro bands (rough band)		1
50	1540771	Seat cover SB30 ST40 with 2 Velcro bands (rough band)		1
51	1540772	Seat cover SB32 ST40 with 2 Velcro bands (rough band)		1
52	1540773	Seat cover SB34 ST40 with 2 Velcro bands (rough band)		1
53	1540774	Seat cover SB36 ST40 with 2 Velcro bands (rough band)		1
54	1540775	Seat cover SB38 ST40 with 2 Velcro bands (rough band)		1
55	1540776	Seat cover SB40 ST40 with 2 Velcro bands (rough band)		1
56	1540777	Seat cover SB42 ST40 with 2 Velcro bands (rough band)		1
57	1540778	Seat cover SB44 ST40 with 2 Velcro bands (rough band)		1
58	1540779	Seat cover SB46 ST40 with 2 Velcro bands (rough band)		1
59	1540780	Seat cover SB48 ST40 with 2 Velcro bands (rough band)		1
60	1540781	Seat cover SB50 ST40 with 2 Velcro bands (rough band)		1
61	1540782	Seat cover SB28 ST42 with 2 Velcro bands (rough band)		1
62	1540783	Seat cover SB30 ST42 with 2 Velcro bands (rough band)		1
63	1540784	Seat cover SB32 ST42 with 2 Velcro bands (rough band)		1
64	1540785	Seat cover SB34 ST42 with 2 Velcro bands (rough band)		1
65	1540786	Seat cover SB36 ST42 with 2 Velcro bands (rough band)		1
66	1540787	Seat cover SB38 ST42 with 2 Velcro bands (rough band)		1
67	1540788	Seat cover SB40 ST42 with 2 Velcro bands (rough band)		1
68	1540789	Seat cover SB42 ST42 with 2 Velcro bands (rough band)		1
69	1540790	Seat cover SB44 ST42 with 2 Velcro bands (rough band)		1
70	1540791	Seat cover SB46 ST42 with 2 Velcro bands (rough band)		1
71	1540792	Seat cover SB48 ST42 with 2 Velcro bands (rough band)		1
72	1540793	Seat cover SB50 ST42 with 2 Velcro bands (rough band)		1
73	1540794	Seat cover SB28 ST44 with 2 Velcro bands (rough band)		1
74	1540795	Seat cover SB30 ST44 with 2 Velcro bands (rough band)		1
75	1540796	Seat cover SB32 ST44 with 2 Velcro bands (rough band)		1
76	1540797	Seat cover SB34 ST44 with 2 Velcro bands (rough band)		1
77	1540798	Seat cover SB36 ST44 with 2 Velcro bands (rough band)		1
78	1540799	Seat cover SB38 ST44 with 2 Velcro bands (rough band)		1
79	1540800	Seat cover SB40 ST44 with 2 Velcro bands (rough band)		1
80	1540801	Seat cover SB42 ST44 with 2 Velcro bands (rough band)		1
81	1540802	Seat cover SB44 ST44 with 2 Velcro bands (rough band)		1
82	1540803	Seat cover SB46 ST44 with 2 Velcro bands (rough band)		1
83	1540804	Seat cover SB48 ST44 with 2 Velcro bands (rough band)		1
84	1540805	Seat cover SB50 ST44 with 2 Velcro bands (rough band)		1
85	1540806	Seat cover SB28 ST46 with 2 Velcro bands (rough band)		1
86	1540807	Seat cover SB30 ST46 with 2 Velcro bands (rough band)		1


Seat cover with 2 Velcro bands (rough band)

Pos.	Part number	Description	Valid from → until	Qty
87	1540808	Seat cover SB32 ST46 with 2 Velcro bands (rough band)		1
88	1540809	Seat cover SB34 ST46 with 2 Velcro bands (rough band)		1
89	1540810	Seat cover SB36 ST46 with 2 Velcro bands (rough band)		1
90	1540811	Seat cover SB38 ST46 with 2 Velcro bands (rough band)		1
91	1540812	Seat cover SB40 ST46 with 2 Velcro bands (rough band)		1
92	1540813	Seat cover SB42 ST46 with 2 Velcro bands (rough band)		1
93	1540814	Seat cover SB44 ST46 with 2 Velcro bands (rough band)		1
94	1540815	Seat cover SB46 ST46 with 2 Velcro bands (rough band)		1
95	1540816	Seat cover SB48 ST46 with 2 Velcro bands (rough band)		1
96	1540817	Seat cover SB50 ST46 with 2 Velcro bands (rough band)		1
97	1540818	Seat cover SB28 ST48 with 2 Velcro bands (rough band)		1
98	1540819	Seat cover SB30 ST48 with 2 Velcro bands (rough band)		1
99	1540820	Seat cover SB32 ST48 with 2 Velcro bands (rough band)		1
100	1540821	Seat cover SB34 ST48 with 2 Velcro bands (rough band)		1
101	1540822	Seat cover SB36 ST48 with 2 Velcro bands (rough band)		1
102	1540823	Seat cover SB38 ST48 with 2 Velcro bands (rough band)		1
103	1540824	Seat cover SB40 ST48 with 2 Velcro bands (rough band)		1
104	1540825	Seat cover SB42 ST48 with 2 Velcro bands (rough band)		1
105	1540826	Seat cover SB44 ST48 with 2 Velcro bands (rough band)		1
106	1540827	Seat cover SB46 ST48 with 2 Velcro bands (rough band)		1
107	1540828	Seat cover SB48 ST48 with 2 Velcro bands (rough band)		1
108	1540829	Seat cover SB50 ST48 with 2 Velcro bands (rough band)		1
109	1540830	Seat cover SB28 ST50 with 2 Velcro bands (rough band)		1
110	1540831	Seat cover SB30 ST50 with 2 Velcro bands (rough band)		1
111	1540832	Seat cover SB32 ST50 with 2 Velcro bands (rough band)		1
112	1540833	Seat cover SB34 ST50 with 2 Velcro bands (rough band)		1
113	1540834	Seat cover SB36 ST50 with 2 Velcro bands (rough band)		1
114	1540835	Seat cover SB38 ST50 with 2 Velcro bands (rough band)		1
115	1540836	Seat cover SB40 ST50 with 2 Velcro bands (rough band)		1
116	1540837	Seat cover SB42 ST50 with 2 Velcro bands (rough band)		1
117	1540838	Seat cover SB44 ST50 with 2 Velcro bands (rough band)		1
118	1540839	Seat cover SB46 ST50 with 2 Velcro bands (rough band)		1
119	1540840	Seat cover SB48 ST50 with 2 Velcro bands (rough band)		1
120	1540841	Seat cover SB50 ST50 with 2 Velcro bands (rough band)		1
121	1540170	Seat cover bar kit ST32		1
122	1540171	Seat cover bar kit ST34		1
123	1540172	Seat cover bar kit ST36		1
124	1540173	Seat cover bar kit ST38		1
125	1540174	Seat cover bar kit ST40		1
126	1540175	Seat cover bar kit ST42		1
127	1540176	Seat cover bar kit ST44		1
128	1540177	Seat cover bar kit ST46		1
129	1540178	Seat cover bar kit ST48		1
130	1540179	Seat cover bar kit ST50		1

Seat cushion standard

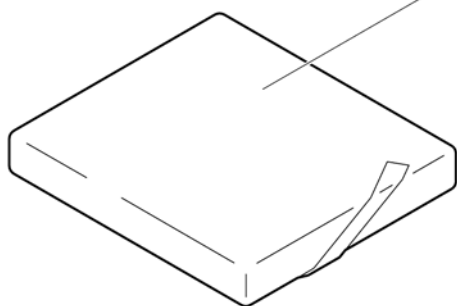

	SB28	SB30	SB32	SB34	SB36	SB38	SB40	SB42	SB44	SB46	SB48	SB50
ST32	1	2	3	4	5	6	7	8	9	10	11	12
ST34	13	14	15	16	17	18	19	20	21	22	23	24
ST36	25	26	27	28	29	30	31	32	33	34	35	36
ST38	37	38	39	40	41	42	43	44	45	46	47	48
ST40	49	50	51	52	53	54	55	56	57	58	59	60
ST42	61	62	63	64	65	66	67	68	69	70	71	72
ST44	73	74	75	76	77	78	79	80	81	82	83	84
ST46	85	86	87	88	89	90	91	92	93	94	95	96
ST48	97	98	99	100	101	102	103	104	105	106	107	108
ST50	109	110	111	112	113	114	115	116	117	118	119	120

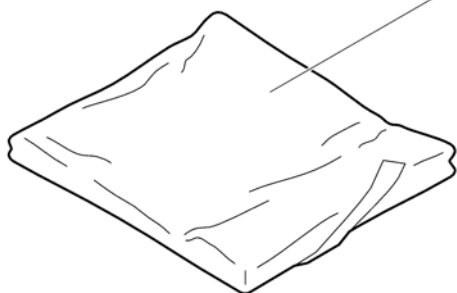

	SB28	SB30	SB32	SB34	SB36	SB38	SB40	SB42	SB44	SB46	SB48	SB50
ST32	121	122	123	124	125	126	127	128	129	130	131	132
ST34	133	134	135	136	137	138	139	140	141	142	143	144
ST36	145	146	147	148	149	150	151	152	153	154	155	156
ST38	157	158	159	160	161	162	163	164	165	166	167	168
ST40	169	170	171	172	173	174	175	176	177	178	179	180
ST42	181	182	183	184	185	186	187	188	189	190	191	192
ST44	193	194	195	196	197	198	199	200	201	202	203	204
ST46	205	206	207	208	209	210	211	212	213	214	215	216
ST48	217	218	219	220	221	222	223	224	225	226	227	228
ST50	229	230	231	232	233	234	235	236	237	238	239	240

Pos.	Part number	Description	Valid from → until	Qty
1	1540842	Seat cushion SB28 ST32 black complete		1
2	1516451	Seat cushion SB30 ST32 black complete		1
3	1516452	Seat cushion SB32 ST32 black complete		1
4	1516453	Seat cushion SB34 ST32 black complete		1
5	1516454	Seat cushion SB36 ST32 black complete		1
6	1516455	Seat cushion SB38 ST32 black complete		1
7	1516456	Seat cushion SB40 ST32 black complete		1
8	1516457	Seat cushion SB42 ST32 black complete		1
9	1516458	Seat cushion SB44 ST32 black complete		1
10	1516459	Seat cushion SB46 ST32 black complete		1
11	1516460	Seat cushion SB48 ST32 black complete		1
12	1516461	Seat cushion SB50 ST32 black complete		1
13	1540843	Seat cushion SB28 ST34 black complete		1
14	1516465	Seat cushion SB30 ST34 black complete		1
15	1516466	Seat cushion SB32 ST34 black complete		1
16	1516467	Seat cushion SB34 ST34 black complete		1
17	1516468	Seat cushion SB36 ST34 black complete		1
18	1516469	Seat cushion SB38 ST34 black complete		1
19	1516470	Seat cushion SB40 ST34 black complete		1
20	1516471	Seat cushion SB42 ST34 black complete		1
21	1516472	Seat cushion SB44 ST34 black complete		1
22	1516473	Seat cushion SB46 ST34 black complete		1
23	1516474	Seat cushion SB48 ST34 black complete		1
24	1516475	Seat cushion SB50 ST34 black complete		1
25	1540844	Seat cushion SB28 ST36 black complete		1
26	1516479	Seat cushion SB30 ST36 black complete		1
27	1516480	Seat cushion SB32 ST36 black complete		1
28	1516481	Seat cushion SB34 ST36 black complete		1
29	1516482	Seat cushion SB36 ST36 black complete		1
30	1516483	Seat cushion SB38 ST36 black complete		1
31	1516484	Seat cushion SB40 ST36 black complete		1

Seat cushion standard

Pos.	Part number	Description	Valid from → until	Qty
32	1516485	Seat cushion SB42 ST36 black complete		1
33	1516486	Seat cushion SB44 ST36 black complete		1
34	1516487	Seat cushion SB46 ST36 black complete		1
35	1516488	Seat cushion SB48 ST36 black complete		1
36	1516489	Seat cushion SB50 ST36 black complete		1
37	1540845	Seat cushion SB28 ST38 black complete		1
38	1540846	Seat cushion SB30 ST38 black complete		1
39	1516494	Seat cushion SB32 ST38 black complete		1
40	1516495	Seat cushion SB34 ST38 black complete		1
41	1516496	Seat cushion SB36 ST38 black complete		1
42	1516497	Seat cushion SB38 ST38 black complete		1
43	1516498	Seat cushion SB40 ST38 black complete		1
44	1516499	Seat cushion SB42 ST38 black complete		1
45	1516500	Seat cushion SB44 ST38 black complete		1
46	1516501	Seat cushion SB46 ST38 black complete		1
47	1516502	Seat cushion SB48 ST38 black complete		1
48	1516503	Seat cushion SB50 ST38 black complete		1
49	1516506	Seat cushion SB28 ST40 black complete		1
50	1516507	Seat cushion SB30 ST40 black complete		1
51	1516508	Seat cushion SB32 ST40 black complete		1
52	1516509	Seat cushion SB34 ST40 black complete		1
53	1516510	Seat cushion SB36 ST40 black complete		1
54	1516511	Seat cushion SB38 ST40 black complete		1
55	1516512	Seat cushion SB40 ST40 black complete		1
56	1516513	Seat cushion SB42 ST40 black complete		1
57	1516514	Seat cushion SB44 ST40 black complete		1
58	1516515	Seat cushion SB46 ST40 black complete		1
59	1516516	Seat cushion SB48 ST40 black complete		1
60	1516517	Seat cushion SB50 ST40 black complete		1
61	1540847	Seat cushion SB28 ST42 black complete		1
62	1540848	Seat cushion SB30 ST42 black complete		1
63	1516522	Seat cushion SB32 ST42 black complete		1
64	1516523	Seat cushion SB34 ST42 black complete		1
65	1516524	Seat cushion SB36 ST42 black complete		1
66	1516525	Seat cushion SB38 ST42 black complete		1
67	1516526	Seat cushion SB40 ST42 black complete		1
68	1516527	Seat cushion SB42 ST42 black complete		1
69	1516528	Seat cushion SB44 ST42 black complete		1
70	1516529	Seat cushion SB46 ST42 black complete		1
71	1516530	Seat cushion SB48 ST42 black complete		1
72	1516531	Seat cushion SB50 ST42 black complete		1
73	1540849	Seat cushion SB28 ST44 black complete		1
74	1540850	Seat cushion SB30 ST44 black complete		1
75	1516536	Seat cushion SB32 ST44 black complete		1
76	1516537	Seat cushion SB34 ST44 black complete		1
77	1516538	Seat cushion SB36 ST44 black complete		1
78	1516539	Seat cushion SB38 ST44 black complete		1
79	1516540	Seat cushion SB40 ST44 black complete		1
80	1516541	Seat cushion SB42 ST44 black complete		1
81	1516542	Seat cushion SB44 ST44 black complete		1
82	1516543	Seat cushion SB46 ST44 black complete		1
83	1516544	Seat cushion SB48 ST44 black complete		1
84	1516545	Seat cushion SB50 ST44 black complete		1
85	1540851	Seat cushion SB28 ST46 black complete		1
86	1540852	Seat cushion SB30 ST46 black complete		1

Seat cushion standard


Pos.	Part number	Description	Valid from → until	Qty
87	1516550	Seat cushion SB32 ST46 black complete		1
88	1516551	Seat cushion SB34 ST46 black complete		1
89	1516552	Seat cushion SB36 ST46 black complete		1
90	1516553	Seat cushion SB38 ST46 black complete		1
91	1516554	Seat cushion SB40 ST46 black complete		1
92	1516555	Seat cushion SB42 ST46 black complete		1
93	1516556	Seat cushion SB44 ST46 black complete		1
94	1516557	Seat cushion SB46 ST46 black complete		1
95	1516558	Seat cushion SB48 ST46 black complete		1
96	1516559	Seat cushion SB50 ST46 black complete		1
97	1540853	Seat cushion SB28 ST48 black complete		1
98	1540854	Seat cushion SB30 ST48 black complete		1
99	1516564	Seat cushion SB32 ST48 black complete		1
100	1516565	Seat cushion SB34 ST48 black complete		1
101	1516566	Seat cushion SB36 ST48 black complete		1
102	1516567	Seat cushion SB38 ST48 black complete		1
103	1516568	Seat cushion SB40 ST48 black complete		1
104	1516569	Seat cushion SB42 ST48 black complete		1
105	1516570	Seat cushion SB44 ST48 black complete		1
106	1516571	Seat cushion SB46 ST48 black complete		1
107	1516572	Seat cushion SB48 ST48 black complete		1
108	1516573	Seat cushion SB50 ST48 black complete		1
109	1540855	Seat cushion SB28 ST50 black complete		1
110	1540856	Seat cushion SB30 ST50 black complete		1
111	1516578	Seat cushion SB32 ST50 black complete		1
112	1516579	Seat cushion SB34 ST50 black complete		1
113	1516580	Seat cushion SB36 ST50 black complete		1
114	1516581	Seat cushion SB38 ST50 black complete		1
115	1516582	Seat cushion SB40 ST50 black complete		1
116	1516583	Seat cushion SB42 ST50 black complete		1
117	1516584	Seat cushion SB44 ST50 black complete		1
118	1516585	Seat cushion SB46 ST50 black complete		1
119	1516586	Seat cushion SB48 ST50 black complete		1
120	1516587	Seat cushion SB50 ST50 black complete		1
121	1540857	Cover to seat cushion SB28 ST32 black		1
122	1516633	Cover to seat cushion SB30 ST32 black		1
123	1516634	Cover to seat cushion SB32 ST32 black		1
124	1516635	Cover to seat cushion SB34 ST32 black		1
125	1516636	Cover to seat cushion SB36 ST32 black		1
126	1516637	Cover to seat cushion SB38 ST32 black		1
127	1516638	Cover to seat cushion SB40 ST32 black		1
128	1516639	Cover to seat cushion SB42 ST32 black		1
129	1516640	Cover to seat cushion SB44 ST32 black		1
130	1516641	Cover to seat cushion SB46 ST32 black		1
131	1516642	Cover to seat cushion SB48 ST32 black		1
132	1516643	Cover to seat cushion SB50 ST32 black		1
133	1540858	Cover to seat cushion SB28 ST34 black		1
134	1516647	Cover to seat cushion SB30 ST34 black		1
135	1516648	Cover to seat cushion SB32 ST34 black		1
136	1516649	Cover to seat cushion SB34 ST34 black		1
137	1516650	Cover to seat cushion SB36 ST34 black		1
138	1516651	Cover to seat cushion SB38 ST34 black		1
139	1516652	Cover to seat cushion SB40 ST34 black		1
140	1516653	Cover to seat cushion SB42 ST34 black		1
141	1516654	Cover to seat cushion SB44 ST34 black		1

Seat cushion standard

Pos.	Part number	Description	Valid from → until	Qty
142	1516655	Cover to seat cushion SB46 ST34 black		1
143	1516656	Cover to seat cushion SB48 ST34 black		1
144	1516657	Cover to seat cushion SB50 ST34 black		1
145	1540859	Cover to seat cushion SB28 ST36 black		1
146	1516661	Cover to seat cushion SB30 ST36 black		1
147	1516662	Cover to seat cushion SB32 ST36 black		1
148	1516663	Cover to seat cushion SB34 ST36 black		1
149	1516664	Cover to seat cushion SB36 ST36 black		1
150	1516665	Cover to seat cushion SB38 ST36 black		1
151	1516666	Cover to seat cushion SB40 ST36 black		1
152	1516667	Cover to seat cushion SB42 ST36 black		1
153	1516668	Cover to seat cushion SB44 ST36 black		1
154	1516669	Cover to seat cushion SB46 ST36 black		1
155	1516670	Cover to seat cushion SB48 ST36 black		1
156	1516671	Cover to seat cushion SB50 ST36 black		1
157	1540860	Cover to seat cushion SB28 ST38 black		1
158	1540861	Cover to seat cushion SB30 ST38 black		1
159	1516676	Cover to seat cushion SB32 ST38 black		1
160	1516677	Cover to seat cushion SB34 ST38 black		1
161	1516678	Cover to seat cushion SB36 ST38 black		1
162	1516679	Cover to seat cushion SB38 ST38 black		1
163	1516680	Cover to seat cushion SB40 ST38 black		1
164	1516681	Cover to seat cushion SB42 ST38 black		1
165	1516682	Cover to seat cushion SB44 ST38 black		1
166	1516683	Cover to seat cushion SB46 ST38 black		1
167	1516684	Cover to seat cushion SB48 ST38 black		1
168	1516685	Cover to seat cushion SB50 ST38 black		1
169	1516688	Cover to seat cushion SB28 ST40 black		1
170	1516689	Cover to seat cushion SB30 ST40 black		1
171	1516690	Cover to seat cushion SB32 ST40 black		1
172	1516691	Cover to seat cushion SB34 ST40 black		1
173	1516692	Cover to seat cushion SB36 ST40 black		1
174	1516693	Cover to seat cushion SB38 ST40 black		1
175	1516694	Cover to seat cushion SB40 ST40 black		1
176	1516695	Cover to seat cushion SB42 ST40 black		1
177	1516696	Cover to seat cushion SB44 ST40 black		1
178	1516697	Cover to seat cushion SB46 ST40 black		1
179	1516698	Cover to seat cushion SB48 ST40 black		1
180	1516699	Cover to seat cushion SB50 ST40 black		1
181	1540862	Cover to seat cushion SB28 ST42 black		1
182	1540863	Cover to seat cushion SB30 ST42 black		1
183	1516704	Cover to seat cushion SB32 ST42 black		1
184	1516705	Cover to seat cushion SB34 ST42 black		1
185	1516706	Cover to seat cushion SB36 ST42 black		1
186	1516707	Cover to seat cushion SB38 ST42 black		1
187	1516708	Cover to seat cushion SB40 ST42 black		1
188	1516709	Cover to seat cushion SB42 ST42 black		1
189	1516710	Cover to seat cushion SB44 ST42 black		1
190	1516711	Cover to seat cushion SB46 ST42 black		1
191	1516712	Cover to seat cushion SB48 ST42 black		1
192	1516713	Cover to seat cushion SB50 ST42 black		1
193	1540864	Cover to seat cushion SB28 ST44 black		1
194	1540865	Cover to seat cushion SB30 ST44 black		1
195	1516718	Cover to seat cushion SB32 ST44 black		1
196	1516719	Cover to seat cushion SB34 ST44 black		1


Seat cushion standard

Pos.	Part number	Description	Valid from → until	Qty
197	1516720	Cover to seat cushion SB36 ST44 black		1
198	1516721	Cover to seat cushion SB38 ST44 black		1
199	1516722	Cover to seat cushion SB40 ST44 black		1
200	1516723	Cover to seat cushion SB42 ST44 black		1
201	1516724	Cover to seat cushion SB44 ST44 black		1
202	1516725	Cover to seat cushion SB46 ST44 black		1
203	1516726	Cover to seat cushion SB48 ST44 black		1
204	1516727	Cover to seat cushion SB50 ST44 black		1
205	1540866	Cover to seat cushion SB28 ST46 black		1
206	1540867	Cover to seat cushion SB30 ST46 black		1
207	1516732	Cover to seat cushion SB32 ST46 black		1
208	1516733	Cover to seat cushion SB34 ST46 black		1
209	1516734	Cover to seat cushion SB36 ST46 black		1
210	1516735	Cover to seat cushion SB38 ST46 black		1
211	1516736	Cover to seat cushion SB40 ST46 black		1
212	1516737	Cover to seat cushion SB42 ST46 black		1
213	1516738	Cover to seat cushion SB44 ST46 black		1
214	1516739	Cover to seat cushion SB46 ST46 black		1
215	1516740	Cover to seat cushion SB48 ST46 black		1
216	1516741	Cover to seat cushion SB50 ST46 black		1
217	1540868	Cover to seat cushion SB28 ST48 black		1
218	1540869	Cover to seat cushion SB30 ST48 black		1
219	1516746	Cover to seat cushion SB32 ST48 black		1
220	1516747	Cover to seat cushion SB34 ST48 black		1
221	1516748	Cover to seat cushion SB36 ST48 black		1
222	1516749	Cover to seat cushion SB38 ST48 black		1
223	1516750	Cover to seat cushion SB40 ST48 black		1
224	1516751	Cover to seat cushion SB42 ST48 black		1
225	1516752	Cover to seat cushion SB44 ST48 black		1
226	1516753	Cover to seat cushion SB46 ST48 black		1
227	1516754	Cover to seat cushion SB48 ST48 black		1
228	1516755	Cover to seat cushion SB50 ST48 black		1
229	1540870	Cover to seat cushion SB28 ST50 black		1
230	1540871	Cover to seat cushion SB30 ST50 black		1
231	1516760	Cover to seat cushion SB32 ST50 black		1
232	1516761	Cover to seat cushion SB34 ST50 black		1
233	1516762	Cover to seat cushion SB36 ST50 black		1
234	1516763	Cover to seat cushion SB38 ST50 black		1
235	1516764	Cover to seat cushion SB40 ST50 black		1
236	1516765	Cover to seat cushion SB42 ST50 black		1
237	1516766	Cover to seat cushion SB44 ST50 black		1
238	1516767	Cover to seat cushion SB46 ST50 black		1
239	1516768	Cover to seat cushion SB48 ST50 black		1
240	1516769	Cover to seat cushion SB50 ST50 black		1

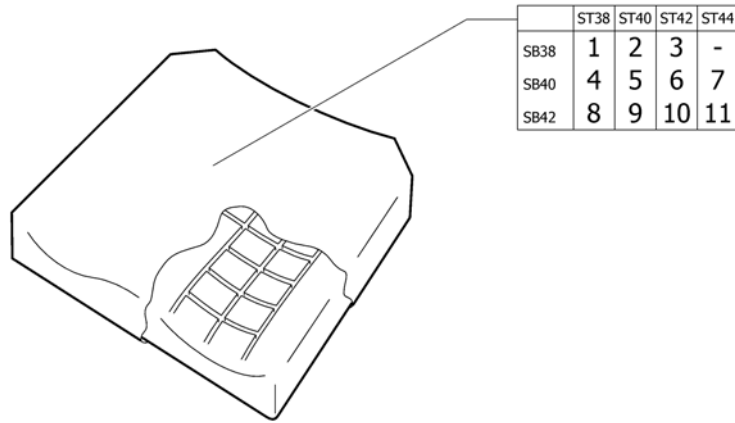Seat cushion Flo-tech


Pos.	Part number	Description	Valid from → until	Qty
1	1510329	Seat cushion Flo-tech Lite SB38 ST38		1
2	1510330	Seat cushion Flo-tech Lite SB38 ST40		1
3	1510331	Seat cushion Flo-tech Lite SB38 ST42		1
4	1510332	Seat cushion Flo-tech Lite SB40 ST38		1
5	1510333	Seat cushion Flo-tech Lite SB40 ST40		1
6	1510334	Seat cushion Flo-tech Lite SB40 ST42		1
7	1510380	Seat cushion Flo-tech Lite SB40 ST44		1
8	1510572	Seat cushion Flo-tech Lite SB42 ST38		1
9	1510795	Seat cushion Flo-tech Lite SB42 ST40		1
10	1510930	Seat cushion Flo-tech Lite SB42 ST42		1
11	1510931	Seat cushion Flo-tech Lite SB42 ST44		1

BACKREST

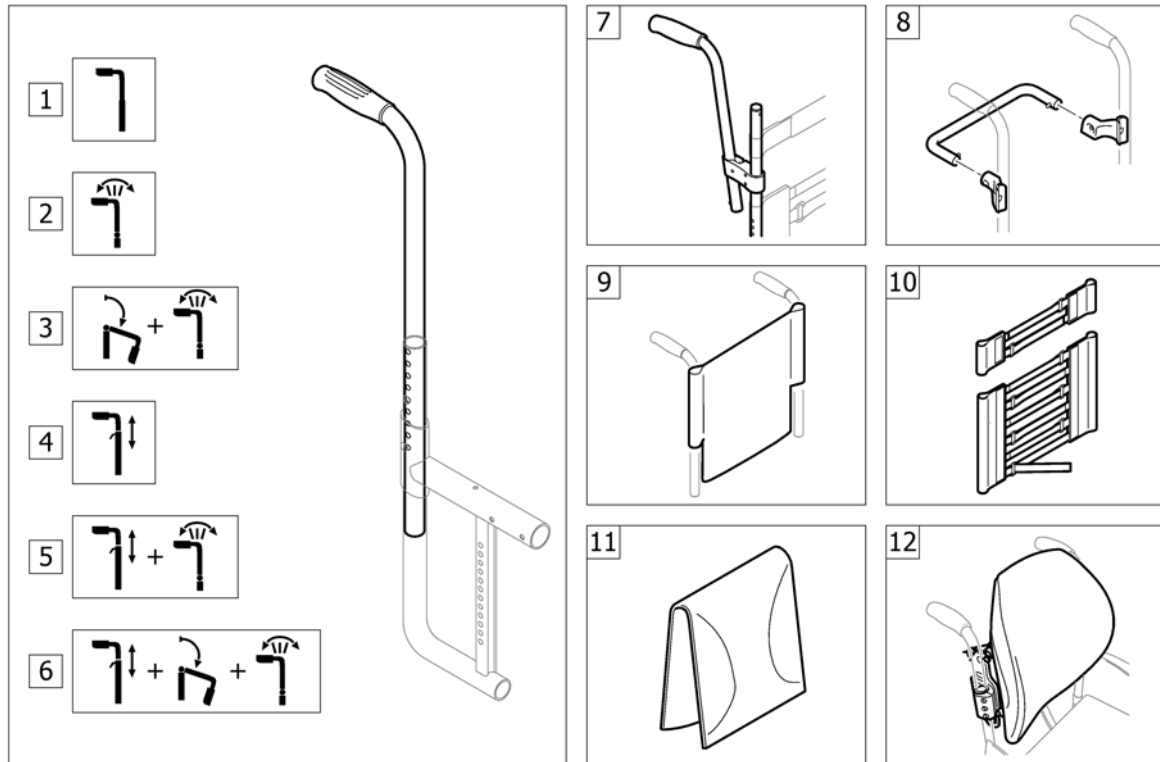


Pos.	Part number	Description	Valid from → until	Qty
1	-	Push handles for backrest standard		1
2	-	Push handles for backrest angle adjustable		1
3	-	Push handles for backrest foldable and angle adjustable		1
4	-	Push handles height adjustable		1
5	-	Push handles height adjustable for backrest angle adjustable		1
6	-	Push handles height adjustable for backrest foldable and angle adjustable		1
7	-	Push handles, inclined		1
8	-	Backrest stabilization bar		1
9	-	Back cover, standard		1
10	-	Bands to adjustable backrest		1
11	-	Cover to adjustable backrest		1
12	-	Sport back		1

Push handles for backrest standard

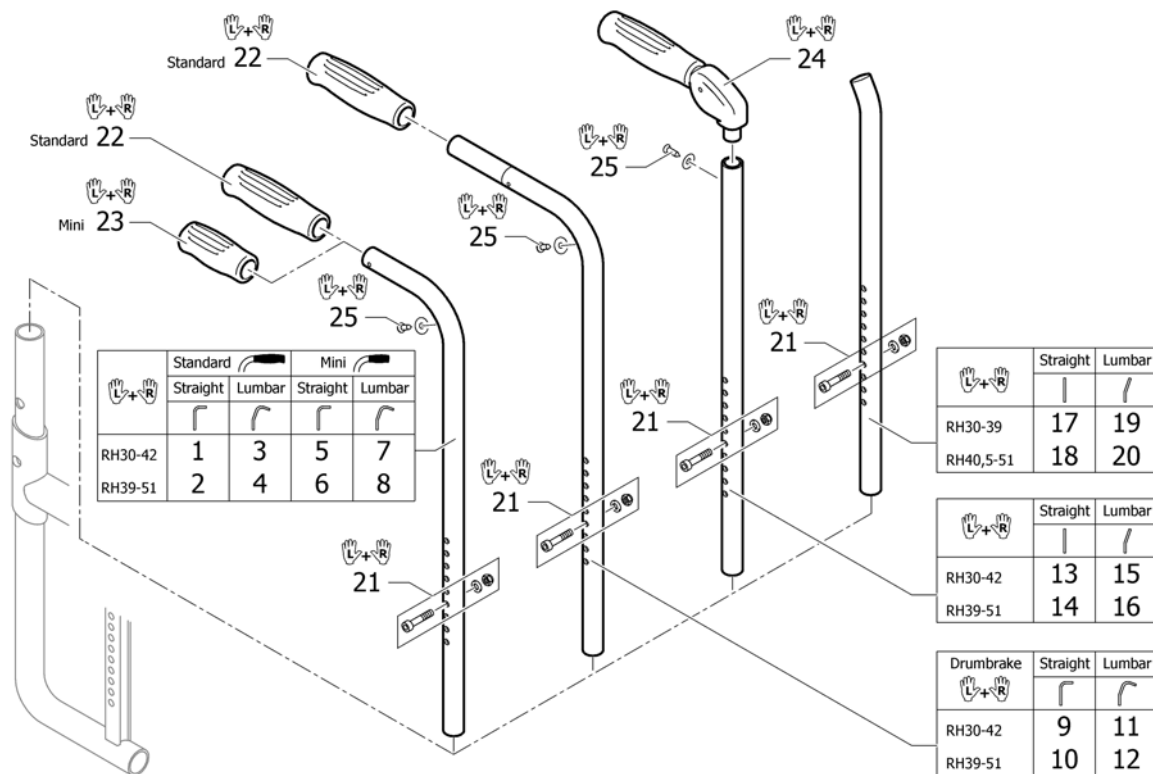


Pos.	Part number	Description	Valid from → until	Qty
1	1540072	Push handles standard, short, pair		1
2	1540073	Push handles standard, medium, pair		1
3	1540074	Push handles standard lumbar short, pair		1
4	1540075	Push handles standard lumbar medium, pair		1
5	1540076	Push handles mini, short, pair		1
6	1540077	Push handles mini, medium, pair		1
7	1540078	Push handles mini lumbar short, pair		1
8	1540079	Push handles mini lumbar medium, pair		1
9	1540084	Push handles short to drum brake, pair		1
10	1540085	Push handles medium to drum brake, pair		1
11	1540086	Push handles lumbar short to drum brake, pair		1
12	1540087	Push handles lumbar medium to drum brake, pair		1
13	1540080	Backrest tube straight short to foldable push handles, pair		1
14	1540081	Backrest tube straight medium to foldable push handles, pair		1
15	1540082	Backrest tube lumbar short to foldable push handles, pair		1
16	1540083	Backrest tube lumbar medium to foldable push handles, pair		1
17	1540089	Backrest tubes L without push handles, pair		1
18	1540090	Backrest tubes XL without push handles, pair		1
19	1540091	Backrest tubes lumbar L without push handles, pair		1
20	1540092	Backrest tubes lumbar XL without push handles, pair		1
21	1540088	Fixation kit to backrest tube		1
22	1507734	Handle black, pair		1
23	1525126	Handle black, mini, pair		1
24	1507733	Push handles foldable, pair		1
25	1432083	Fixation kit for backrest band		1

Push handles for backrest angle adjustable

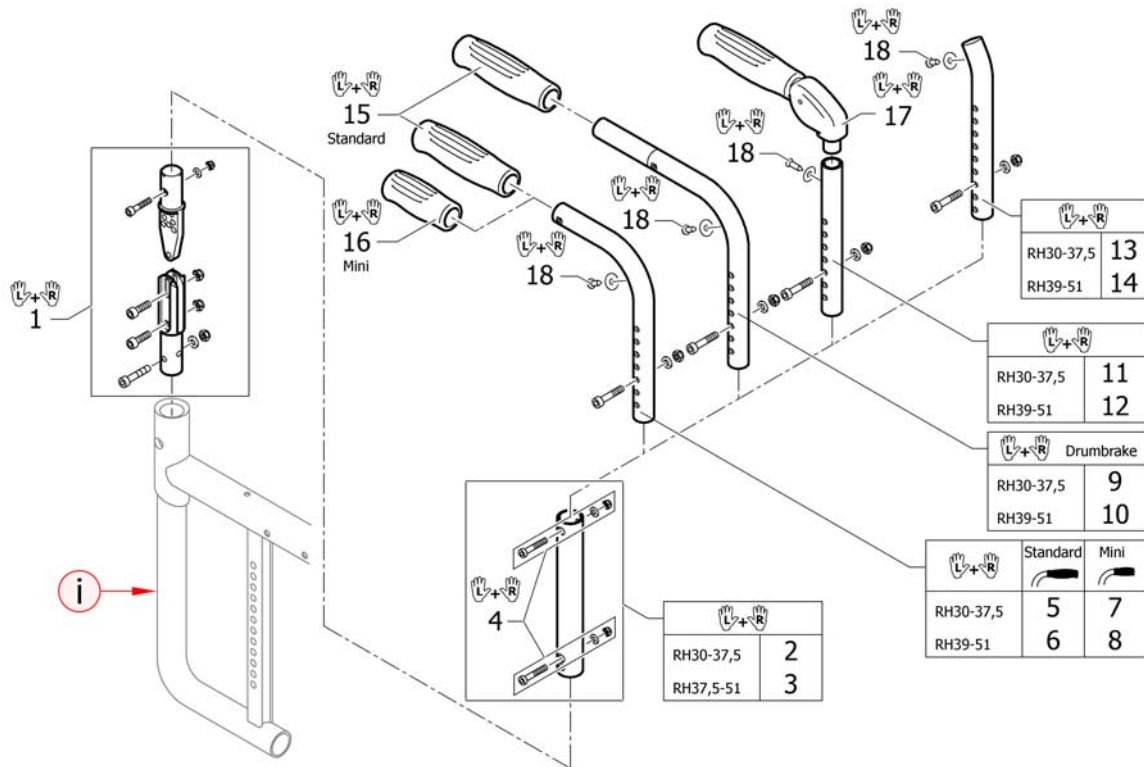


Pos.	Part number	Description	Valid from → until	Qty
i	-	A special rear frame is necessary. Please see chapter CHASSIS.		0
1	1540094	Backrest angle kit		1
2	1540063	Tube kit medium to angle adjustable back		1
3	1540095	Tube kit long to angle adjustable back		1
4	1540065	Screw kit for telescopic tube		1
5	1540055	Push handles standard, XS, pair		1
6	1540056	Push handles standard, S, pair		1
7	1540069	Push handles mini, XS, pair		1
8	1540070	Push handles mini, S, pair		1
9	1540057	Push handles XS to drum brake, pair		1
10	1540058	Push handles S to drum brake, pair		1
11	1540059	Backrest tube straight XS to foldable push handles, pair		1
12	1540060	Backrest tube straight S to foldable push handles, pair		1
13	1540061	Backrest tubes S without push handles, pair		1
14	1540062	Backrest tubes M without push handles, pair		1
15	1507734	Handle black, pair		1
16	1525126	Handle black, mini, pair		1
17	1507733	Push handles foldable, pair		1
18	1432083	Fixation kit for backrest band		1

Push handles for backrest foldable and angle adjustable

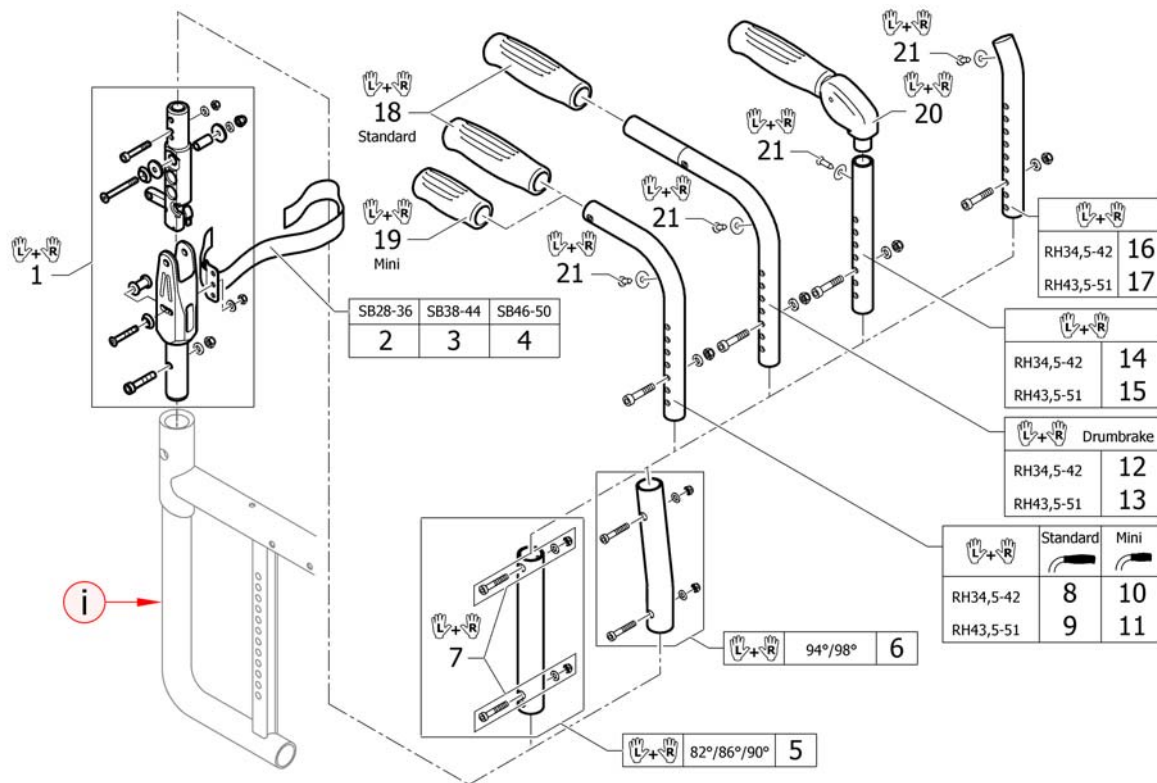


Pos.	Part number	Description	Valid from → until	Qty
i	-	A special rear frame is necessary. Please see chapter CHASSIS.		0
1	1540054	Hinge kit to backrest foldable, pair		1
2	1540066	Backrest band short for backrest foldable		1
3	1540067	Backrest band medium for backrest foldable		1
4	1540068	Backrest band long for backrest foldable		1
5	1540063	Tube kit medium to angle adjustable back		1
6	1540064	Tube kit lumbar to angle adjustable back		1
7	1540065	Screw kit for telescopic tube		1
8	1540055	Push handles standard, XS, pair		1
9	1540056	Push handles standard, S, pair		1
10	1540069	Push handles mini, XS, pair		1
11	1540070	Push handles mini, S, pair		1
12	1540057	Push handles XS to drum brake, pair		1
13	1540058	Push handles S to drum brake, pair		1
14	1540059	Backrest tube straight XS to foldable push handles, pair		1
15	1540060	Backrest tube straight S to foldable push handles, pair		1
16	1540061	Backrest tubes S without push handles, pair		1
17	1540062	Backrest tubes M without push handles, pair		1
18	1507734	Handle black, pair		1
19	1525126	Handle black, mini, pair		1
20	1507733	Push handles foldable, pair		1
21	1432083	Fixation kit for backrest band		1

Push handles height adjustable

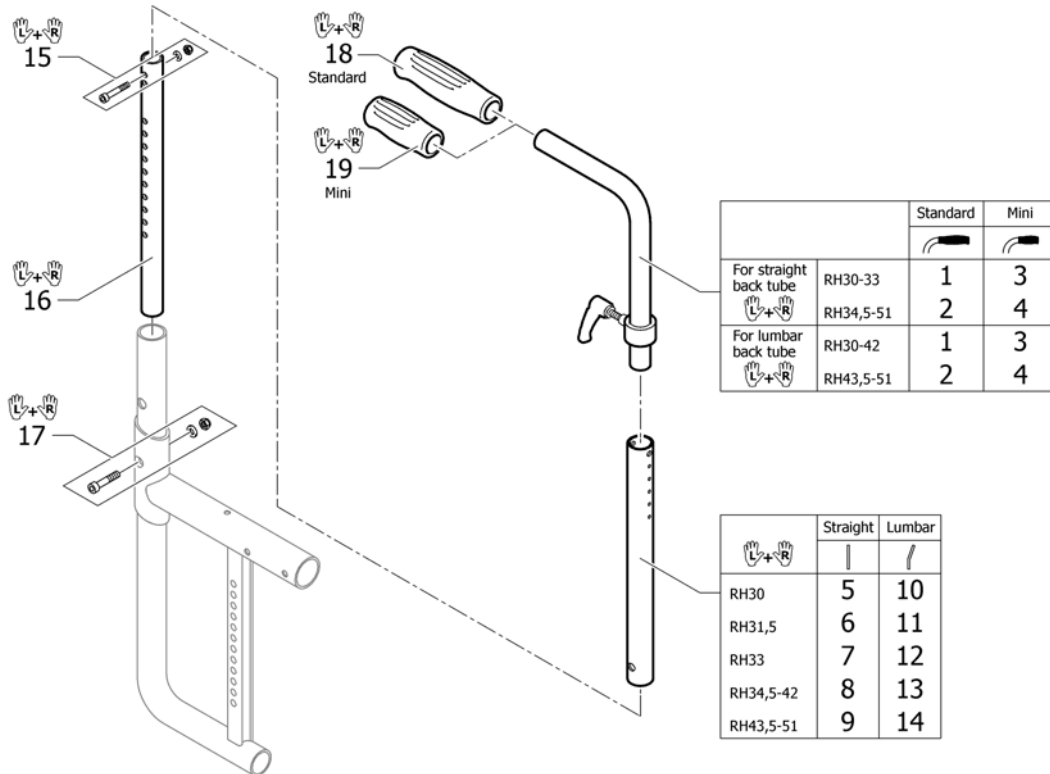


Pos.	Part number	Description	Valid from → until	Qty
1	1438203	Height adjustable, integrated push handles standard, kit short UL/C04		1
2	1438204	Height adjustable, integrated push handles standard, kit long UL/C04		1
3	1525423	Height adjustable, integrated push handles mini, kit short UL/C04		1
4	1525542	Height adjustable, integrated push handles mini, kit long UL/C04		1
5	1507750	Backrest tube L.235mm straight to integrated push handles, pair		1
6	1507751	Backrest tube L.250mm straight to integrated push handles, pair		1
7	1507752	Backrest tube L.265mm straight to integrated push handles, pair		1
8	1507753	Backrest tube L.280mm straight to integrated push handles, pair		1
9	1509103	Backrest tube L.355mm straight to integrated push handles, pair		1
10	1509114	Backrest tube L.235mm lumbar to integrated push handles, pair		1
11	1509115	Backrest tube L.250mm lumbar to integrated push handles, pair		1
12	1509116	Backrest tube L.265mm lumbar to integrated push handles, pair		1
13	1509117	Backrest tube L.280mm lumbar to integrated push handles, pair		1
14	1509122	Backrest tube L.355mm lumbar to integrated push handles, pair		1
15	1540071	Screw kit for backrest tube		1
16	1507747	Telescopic tube frame-backrest, pair		1
17	1540088	Fixation kit to backrest tube		1
18	1507734	Handle black, pair		1
19	1525126	Handle black, mini, pair		1

Push handles height adjustable for backrest angle adjustable

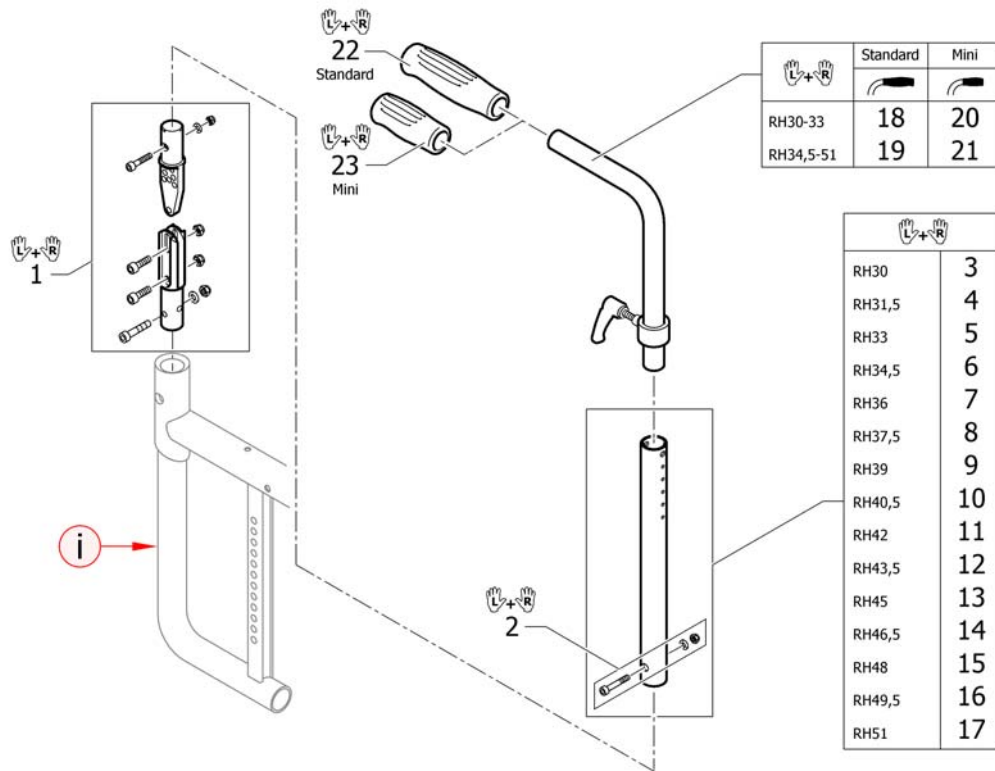

Pos.	Part number	Description	Valid from → until	Qty
i	-	A special rear frame is necessary. Please see chapter CHASSIS.		0
1	1540094	Backrest angle kit		1
2	1540071	Screw kit for backrest tube		1
3	1507751	Backrest tube L.250mm straight to integrated push handles, pair		1
4	1507752	Backrest tube L.265mm straight to integrated push handles, pair		1
5	1507753	Backrest tube L.280mm straight to integrated push handles, pair		1
6	1507862	Backrest tube L.295mm straight to integrated push handles, pair		1
7	1508438	Backrest tube L.310mm straight to integrated push handles, pair		1
8	1508466	Backrest tube L.325mm straight to integrated push handles, pair		1
9	1508498	Backrest tube L.340mm straight to integrated push handles, pair		1
10	1509103	Backrest tube L.355mm straight to integrated push handles, pair		1
11	1509104	Backrest tube L.370mm straight to integrated push handles, pair		1
12	1509105	Backrest tube L.385mm straight to integrated push handles, pair		1
13	1509106	Backrest tube L.400mm straight to integrated push handles, pair		1
14	1509107	Backrest tube L.415mm straight to integrated push handles, pair		1
15	1509108	Backrest tube L.430mm straight to integrated push handles, pair		1
16	1509109	Backrest tube L.445mm straight to integrated push handles, pair		1
17	1509110	Backrest tube L.460mm straight to integrated push handles, pair		1
18	1438203	Height adjustable, integrated push handles standard, kit short UL/C04		1
19	1438204	Height adjustable, integrated push handles standard, kit long UL/C04		1
20	1525423	Height adjustable, integrated push handles mini, kit short UL/C04		1
21	1525542	Height adjustable, integrated push handles mini, kit long UL/C04		1
22	1507734	Handle black, pair		1
23	1525126	Handle black, mini, pair		1

Push handles height adjustable for backrest foldable and angle adjustable

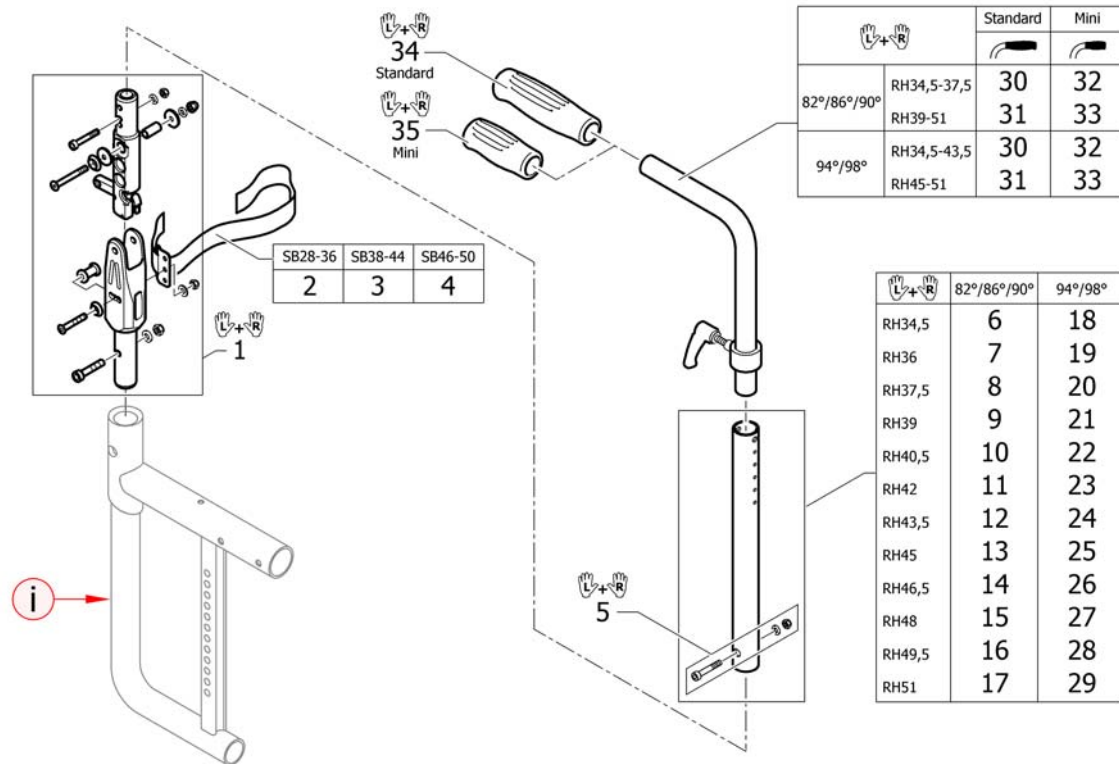


Pos.	Part number	Description	Valid from → until	Qty
i	-	A special rear frame is necessary. Please see chapter CHASSIS.		0
1	1540054	Hinge kit to backrest foldable, pair		1
2	1540066	Backrest band short for backrest foldable		1
3	1540067	Backrest band medium for backrest foldable		1
4	1540068	Backrest band long for backrest foldable		1
5	1540071	Screw kit for backrest tube		1
6	1507751	Backrest tube L.250mm straight to integrated push handles, pair		1
7	1507752	Backrest tube L.265mm straight to integrated push handles, pair		1
8	1507753	Backrest tube L.280mm straight to integrated push handles, pair		1
9	1507862	Backrest tube L.295mm straight to integrated push handles, pair		1
10	1508438	Backrest tube L.310mm straight to integrated push handles, pair		1
11	1508466	Backrest tube L.325mm straight to integrated push handles, pair		1
12	1508498	Backrest tube L.340mm straight to integrated push handles, pair		1
13	1509103	Backrest tube L.355mm straight to integrated push handles, pair		1
14	1509104	Backrest tube L.370mm straight to integrated push handles, pair		1
15	1509105	Backrest tube L.385mm straight to integrated push handles, pair		1
16	1509106	Backrest tube L.400mm straight to integrated push handles, pair		1
17	1509107	Backrest tube L.415mm straight to integrated push handles, pair		1
18	1509115	Backrest tube L.250mm lumbar to integrated push handles, pair		1
19	1509116	Backrest tube L.265mm lumbar to integrated push handles, pair		1
20	1509117	Backrest tube L.280mm lumbar to integrated push handles, pair		1
21	1509118	Backrest tube L.295mm lumbar to integrated push handles, pair		1
22	1509119	Backrest tube L.310mm lumbar to integrated push handles, pair		1
23	1509120	Backrest tube L.325mm lumbar to integrated push handles, pair		1
24	1509121	Backrest tube L.340mm lumbar to integrated push handles, pair		1
25	1509122	Backrest tube L.355mm lumbar to integrated push handles, pair		1
26	1509123	Backrest tube L.370mm lumbar to integrated push handles, pair		1
27	1509124	Backrest tube L.385mm lumbar to integrated push handles, pair		1
28	1509125	Backrest tube L.400mm lumbar to integrated push handles, pair		1
29	1509126	Backrest tube L.415mm lumbar to integrated push handles, pair		1
30	1438203	Height adjustable, integrated push handles standard, kit short UL/C04		1

Push handles height adjustable for backrest foldable and angle adjustable


Pos.	Part number	Description	Valid from → until	Qty
31	1438204	Height adjustable, integrated push handles standard, kit long UL/C04		1
32	1525423	Height adjustable, integrated push handles mini, kit short UL/C04		1
33	1525542	Height adjustable, integrated push handles mini, kit long UL/C04		1
34	1507734	Handle black, pair		1
35	1525126	Handle black, mini, pair		1

Push handles, inclined

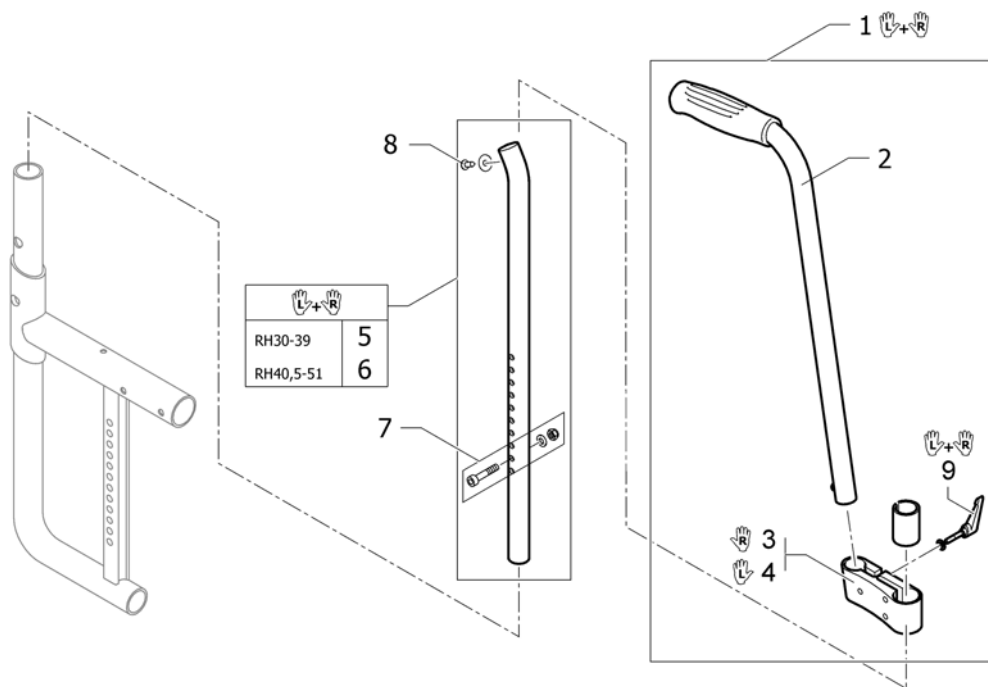

Pos.	Part number	Description	Valid from → until	Qty
1	1517876	Push handles, height adjustable, rear set, pair		1
2	1438938	Push handle tube complete, rearset		2
3	1438939	Mounting fixture right complete, for rearset push handles		1
4	1438940	Mounting fixture left complete, for rearset push handles		1
5	1540089	Backrest tubes L without push handles, pair		1
6	1540090	Backrest tubes XL without push handles, pair		1
7	1540088	Fixation kit to backrest tube		1
8	1432083	Fixation kit for backrest band		1
9	1529799	Adjusting lever, pair		1

Backrest stabilization bar

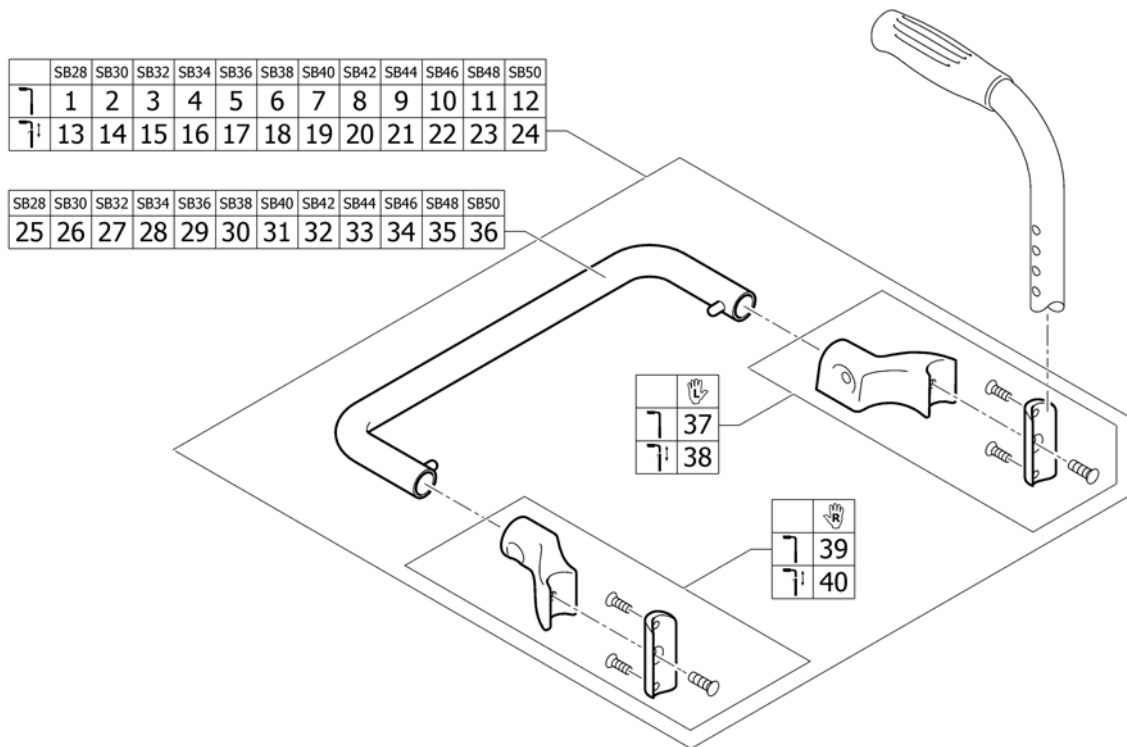


Pos.	Part number	Description	Valid from → until	Qty
1	1540096	Stabilizing bar complete SB28 for backrest tubes Ø20		1
2	1540097	Stabilizing bar complete SB30 for backrest tubes Ø20		1
3	1517978	Stabilizing bar complete SB32 for backrest tubes Ø20		1
4	1517979	Stabilizing bar complete SB34 for backrest tubes Ø20		1
5	1517980	Stabilizing bar complete SB36 for backrest tubes Ø20		1
6	1517981	Stabilizing bar complete SB38 for backrest tubes Ø20		1
7	1517982	Stabilizing bar complete SB40 for backrest tubes Ø20		1
8	1517983	Stabilizing bar complete SB42 for backrest tubes Ø20		1
9	1517984	Stabilizing bar complete SB44 for backrest tubes Ø20		1
10	1517985	Stabilizing bar complete SB46 for backrest tubes Ø20		1
11	1517986	Stabilizing bar complete SB48 for backrest tubes Ø20		1
12	1517987	Stabilizing bar complete SB50 for backrest tubes Ø20		1
13	1540098	Stabilizing bar complete SB28 for backrest tubes Ø25		1
14	1540099	Stabilizing bar complete SB30 for backrest tubes Ø25		1
15	1517988	Stabilizing bar complete SB32 for backrest tubes Ø25		1
16	1517989	Stabilizing bar complete SB34 for backrest tubes Ø25		1
17	1517990	Stabilizing bar complete SB36 for backrest tubes Ø25		1
18	1517991	Stabilizing bar complete SB38 for backrest tubes Ø25		1
19	1517992	Stabilizing bar complete SB40 for backrest tubes Ø25		1
20	1517993	Stabilizing bar complete SB42 for backrest tubes Ø25		1
21	1517994	Stabilizing bar complete SB44 for backrest tubes Ø25		1
22	1517995	Stabilizing bar complete SB46 for backrest tubes Ø25		1
23	1517996	Stabilizing bar complete SB48 for backrest tubes Ø25		1
24	1517997	Stabilizing bar complete SB50 for backrest tubes Ø25		1
25	1540100	Stabilizing bar SB28		1
26	1540101	Stabilizing bar SB30		1
27	1517998	Stabilizing bar SB32		1
28	1517999	Stabilizing bar SB34		1
29	1518000	Stabilizing bar SB36		1
30	1518001	Stabilizing bar SB38		1
31	1518002	Stabilizing bar SB40		1

Backrest stabilization bar

Pos.	Part number	Description	Valid from → until	Qty
32	1518003	Stabilizing bar SB42		1
33	1518004	Stabilizing bar SB44		1
34	1518005	Stabilizing bar SB46		1
35	1518006	Stabilizing bar SB48		1
36	1518007	Stabilizing bar SB50		1
37	1518009	Fixation kit to stabilizing bar, right, for backrest tubes Ø20		1
38	1518088	Fixation kit to stabilizing bar, right, for backrest tubes Ø25		1
39	1518008	Fixation kit to stabilizing bar, left, for backrest tubes Ø20		1
40	1518010	Fixation kit to stabilizing bar, left, for backrest tubes Ø25		1

Back cover, standard

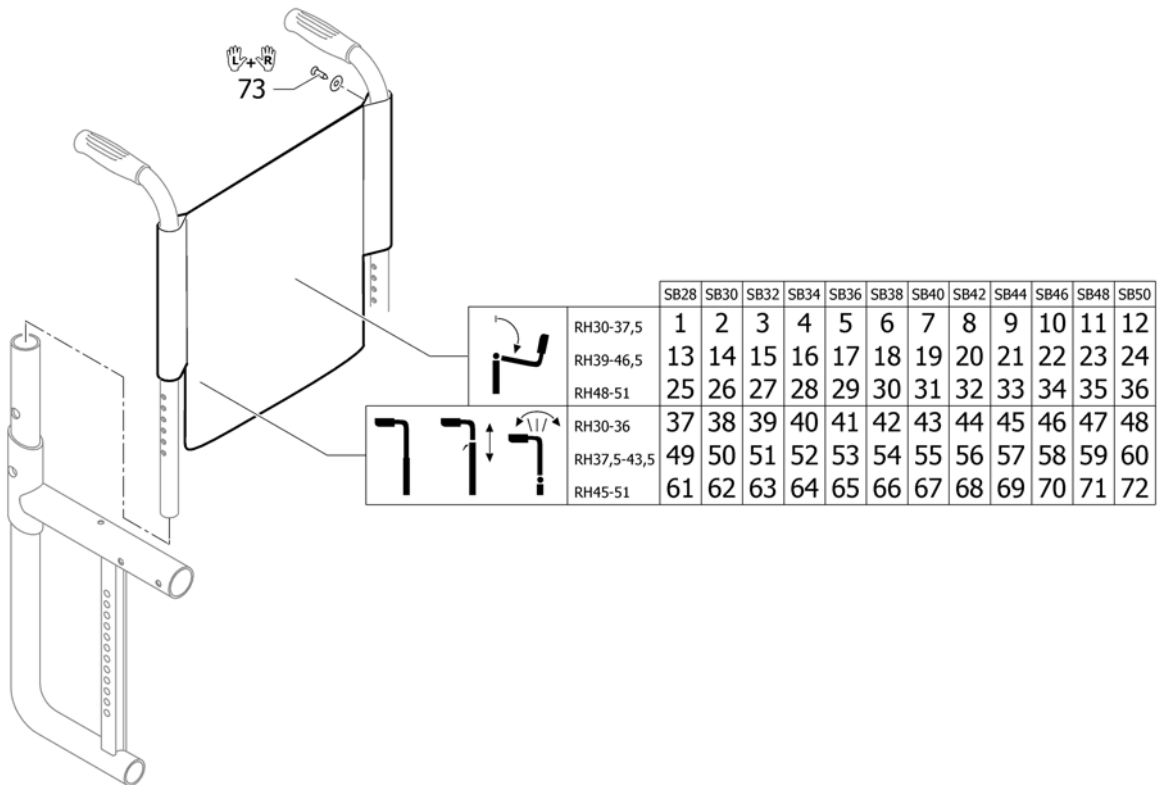

Pos.	Part number	Description	Valid from → until	Qty
1	1540108	Backrest cover SB28 S to backrest foldable		1
2	1540109	Backrest cover SB30 S to backrest foldable		1
3	1540110	Backrest cover SB32 S to backrest foldable		1
4	1540111	Backrest cover SB34 S to backrest foldable		1
5	1540112	Backrest cover SB36 S to backrest foldable		1
6	1540113	Backrest cover SB38 S to backrest foldable		1
7	1540114	Backrest cover SB40 S to backrest foldable		1
8	1540115	Backrest cover SB42 S to backrest foldable		1
9	1540116	Backrest cover SB44 S to backrest foldable		1
10	1540117	Backrest cover SB46 S to backrest foldable		1
11	1540118	Backrest cover SB48 S to backrest foldable		1
12	1540119	Backrest cover SB50 S to backrest foldable		1
13	1540120	Backrest cover SB28 M to backrest foldable		1
14	1540121	Backrest cover SB30 M to backrest foldable		1
15	1540122	Backrest cover SB32 M to backrest foldable		1
16	1540123	Backrest cover SB34 M to backrest foldable		1
17	1540124	Backrest cover SB36 M to backrest foldable		1
18	1540125	Backrest cover SB38 M to backrest foldable		1
19	1540126	Backrest cover SB40 M to backrest foldable		1
20	1540127	Backrest cover SB42 M to backrest foldable		1
21	1540128	Backrest cover SB44 M to backrest foldable		1
22	1540129	Backrest cover SB46 M to backrest foldable		1
23	1540130	Backrest cover SB48 M to backrest foldable		1
24	1540131	Backrest cover SB50 M to backrest foldable		1
25	1540132	Backrest cover SB28 L to backrest foldable		1
26	1540133	Backrest cover SB30 L to backrest foldable		1
27	1540134	Backrest cover SB32 L to backrest foldable		1
28	1540135	Backrest cover SB34 L to backrest foldable		1
29	1540136	Backrest cover SB36 L to backrest foldable		1
30	1540137	Backrest cover SB38 L to backrest foldable		1
31	1540138	Backrest cover SB40 L to backrest foldable		1


Back cover, standard

Pos.	Part number	Description	Valid from → until	Qty
32	1540139	Backrest cover SB42 L to backrest foldable		1
33	1540140	Backrest cover SB44 L to backrest foldable		1
34	1540141	Backrest cover SB46 L to backrest foldable		1
35	1540142	Backrest cover SB48 L to backrest foldable		1
36	1540143	Backrest cover SB50 L to backrest foldable		1
37	1540102	Backrest cover SB28 short		1
38	1540103	Backrest cover SB30 short		1
39	1517552	Backrest cover SB32 short <i>Replaces 1438180</i>		1
40	1517553	Backrest cover SB34 short <i>Replaces 1438181</i>		1
41	1517554	Backrest cover SB36 short <i>Replaces 1438182</i>		1
42	1517555	Backrest cover SB38 short <i>Replaces 1438183</i>		1
43	1517556	Backrest cover SB40 short <i>Replaces 1438184</i>		1
44	1517557	Backrest cover SB42 short <i>Replaces 1438185</i>		1
45	1517558	Backrest cover SB44 short <i>Replaces 1438186</i>		1
46	1517559	Backrest cover SB46 short <i>Replaces 1438187</i>		1
47	1517560	Backrest cover SB48 short <i>Replaces 1438188</i>		1
48	1517561	Backrest cover SB50 short <i>Replaces 1438189</i>		1
49	1540104	Backrest cover SB28 medium		1
50	1540105	Backrest cover SB30 medium		1
51	1517562	Backrest cover SB32 medium <i>Replaces 1438418</i>		1
52	1517563	Backrest cover SB34 medium <i>Replaces 1438419</i>		1
53	1517564	Backrest cover SB36 medium <i>Replaces 1438420</i>		1
54	1517565	Backrest cover SB38 medium <i>Replaces 1438421</i>		1
55	1517566	Backrest cover SB40 medium <i>Replaces 1438422</i>		1
56	1517567	Backrest cover SB42 medium <i>Replaces 1438423</i>		1
57	1517568	Backrest cover SB44 medium <i>Replaces 1438424</i>		1
58	1517569	Backrest cover SB46 medium <i>Replaces 1438425</i>		1
59	1517570	Backrest cover SB48 medium <i>Replaces 1438426</i>		1
60	1517571	Backrest cover SB50 medium <i>Replaces 1438427</i>		1
61	1540106	Backrest cover SB28 long		1
62	1540107	Backrest cover SB30 long		1
63	1517572	Backrest cover SB32 long <i>Replaces 1438190</i>		1
64	1517573	Backrest cover SB34 long <i>Replaces 1438191</i>		1
65	1517574	Backrest cover SB36 long <i>Replaces 1438192</i>		1
66	1517575	Backrest cover SB38 long <i>Replaces 1438193</i>		1
67	1517576	Backrest cover SB40 long <i>Replaces 1438194</i>		1
68	1517577	Backrest cover SB42 long <i>Replaces 1438195</i>		1

Back cover, standard


Pos.	Part number	Description	Valid from → until	Qty
69	1517578	Backrest cover SB44 long <i>Replaces 1438196</i>		1
70	1517579	Backrest cover SB46 long <i>Replaces 1438197</i>		1
71	1517580	Backrest cover SB48 long <i>Replaces 1438198</i>		1
72	1517581	Backrest cover SB50 long <i>Replaces 1438199</i>		1
73	1432083	Fixation kit for backrest band		1

Bands to adjustable backrest

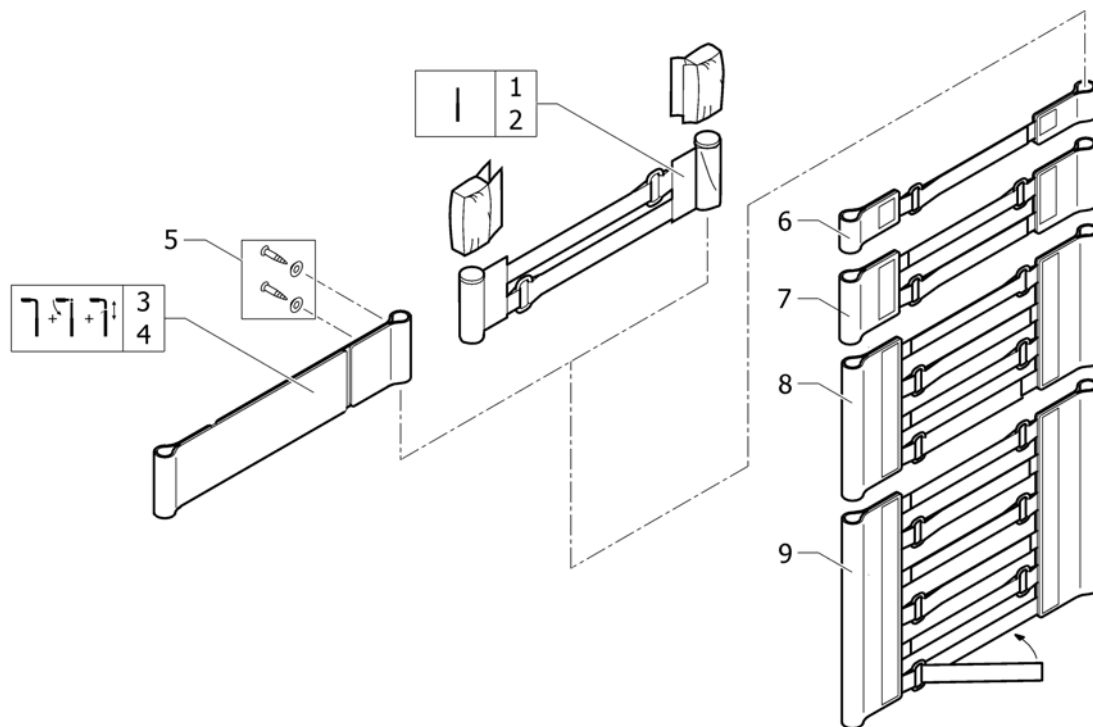

Pos.	Part number	Description	Valid from → until	Qty
1	1513172	Velcro end part, 5 cm <i>Replaces 1416975</i>		1
2	1513173	Velcro end part, 10cm <i>Replaces 1416974</i>		1
3	1513170	Backrest band 2 parts, width: 5cm <i>Replaces 709.1009</i>		1
4	1513171	Backrest band 2 parts, width: 10cm <i>Replaces 709.1018</i>		1
5	1432083	Fixation kit for backrest band		1
6	1513167	Velcro adjustment part with 1 strap, pair <i>Replaces 1439291</i>		1
7	1513168	Velcro adjustment strap with 2 straps, pair <i>Replaces 707.1088</i>		1
8	1513169	Velcro adjustment strap with 4 straps, pair <i>Replaces 707.1087</i>		1
9	1517520	Velcro adjustment strap with 6 straps, pair <i>Replaces 707.1075</i>		1


Cover to adjustable backrest

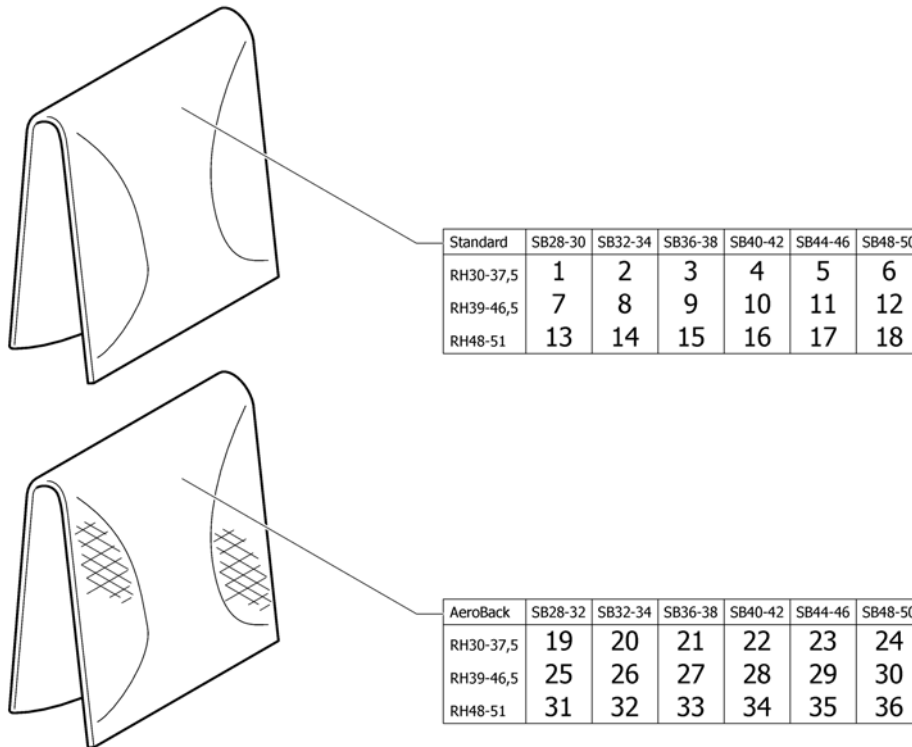

Pos.	Part number	Description	Valid from → until	Qty
1	1540144	Cover to adjustable backrest SB28-30 short		1
2	1517522	Cover to adjustable backrest SB32-34 short <i>Replaces 1438161</i>		1
3	1517523	Cover to adjustable backrest SB36-38 short <i>Replaces 1438163</i>		1
4	1517524	Cover to adjustable backrest SB40-42 short <i>Replaces 1438165</i>		1
5	1517525	Cover to adjustable backrest SB44-46 short <i>Replaces 1438167</i>		1
6	1517526	Cover to adjustable backrest SB48-50 short <i>Replaces 1438169</i>		1
7	1540145	Cover to adjustable backrest SB28-30 medium		1
8	1517527	Cover to adjustable backrest SB32-34 medium <i>Replaces 1438378</i>		1
9	1517528	Cover to adjustable backrest SB36-38 medium <i>Replaces 1438380</i>		1
10	1517529	Cover to adjustable backrest SB40-42 medium <i>Replaces 1438382</i>		1
11	1517530	Cover to adjustable backrest SB44-46 medium <i>Replaces 1438384</i>		1
12	1517531	Cover to adjustable backrest SB48-50 medium <i>Replaces 1438386</i>		1
13	1540146	Cover to adjustable backrest SB28-30 long		1
14	1517532	Cover to adjustable backrest SB32-34 long <i>Replaces 1438171</i>		1
15	1517533	Cover to adjustable backrest SB36-38 long <i>Replaces 1438173</i>		1
16	1517534	Cover to adjustable backrest SB40-42 long <i>Replaces 1438175</i>		1
17	1517535	Cover to adjustable backrest SB44-46 long <i>Replaces 1438177</i>		1
18	1517536	Cover to adjustable backrest SB48-50 long <i>Replaces 1438179</i>		1
19	1540147	Cover to adjustable backrest Aero SB28-30 short		1
20	1517537	Cover to adjustable backrest Aero SB32-34 short <i>Replaces 1438389</i>		1


Cover to adjustable backrest

Pos.	Part number	Description	Valid from → until	Qty
21	1517538	Cover to adjustable backrest Aero SB36-38 short <i>Replaces 1438391</i>		1
22	1517539	Cover to adjustable backrest Aero SB40-42 short <i>Replaces 1438393</i>		1
23	1517540	Cover to adjustable backrest Aero SB44-46 short <i>Replaces 1438395</i>		1
24	1517541	Cover to adjustable backrest Aero SB48-50 short <i>Replaces 1438397</i>		1
25	1540148	Cover to adjustable backrest Aero SB28-30 medium		1
26	1517542	Cover to adjustable backrest Aero SB32-34 medium <i>Replaces 1438399</i>		1
27	1517543	Cover to adjustable backrest Aero SB36-38 medium <i>Replaces 1438401</i>		1
28	1517544	Cover to adjustable backrest Aero SB40-42 medium <i>Replaces 1438403</i>		1
29	1517545	Cover to adjustable backrest Aero SB44-46 medium <i>Replaces 1438405</i>		1
30	1517546	Cover to adjustable backrest Aero SB48-50 medium <i>Replaces 1438407</i>		1
31	1540149	Cover to adjustable backrest Aero SB28-30 long		1
32	1517547	Cover to adjustable backrest Aero SB32-34 long <i>Replaces 1438409</i>		1
33	1517548	Cover to adjustable backrest Aero SB36-38 long <i>Replaces 1438411</i>		1
34	1517549	Cover to adjustable backrest Aero SB40-42 long <i>Replaces 1438413</i>		1
35	1517550	Cover to adjustable backrest Aero SB44-46 long <i>Replaces 1438415</i>		1
36	1517551	Cover to adjustable backrest Aero SB48-50 long <i>Replaces 1438417</i>		1

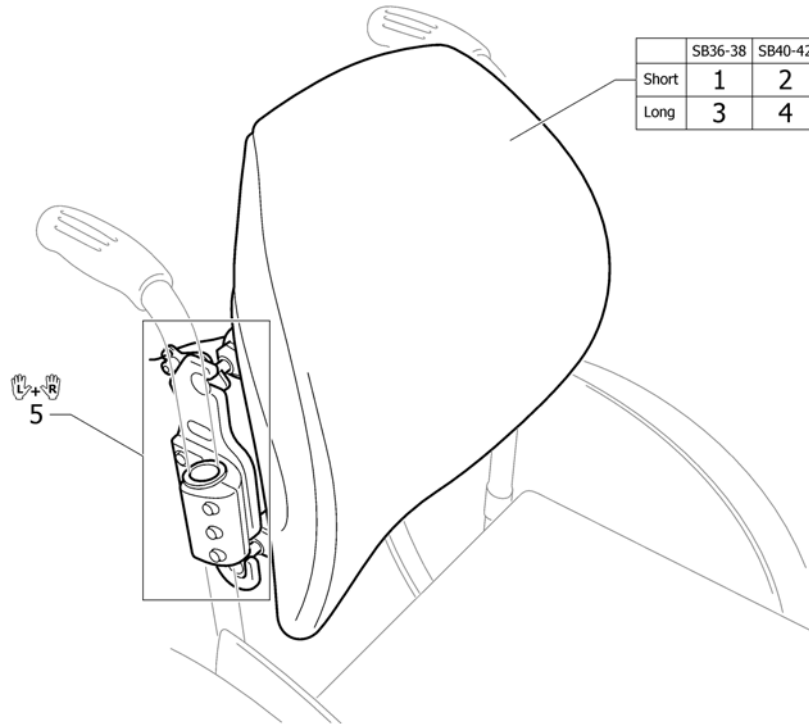
Sport back

Pos.	Part number	Description	Valid from → until	Qty
1	1529345	Sport back SB36-38 short		1
2	1529346	Sport back SB40-42 short		1
3	1529347	Sport back SB36-38 long		1
4	1529348	Sport back SB40-42 long		1
5	1529350	Fixation kit for sport back		1

LEGRESTS

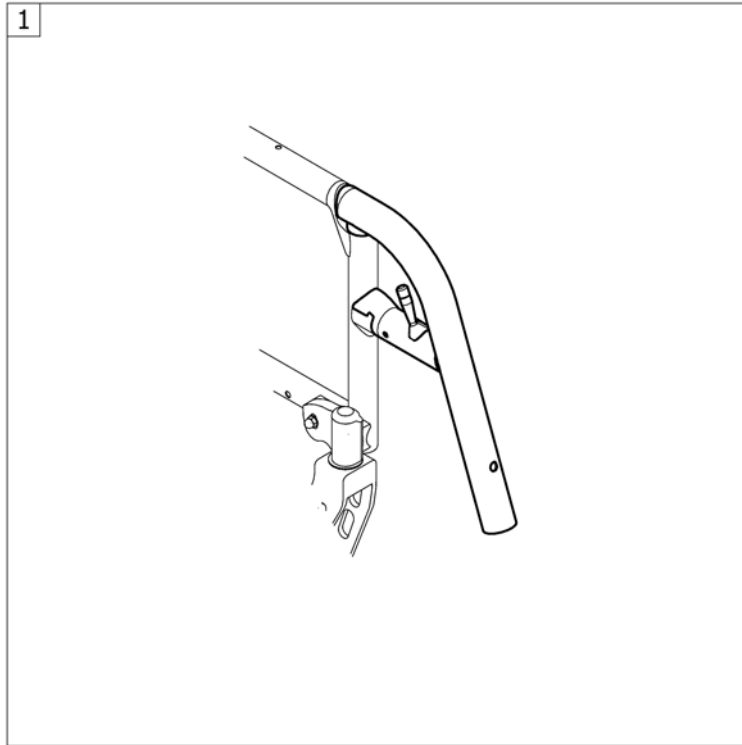

Pos.	Part number	Description	Valid from → until	Qty
1	-	Legrest hanger		1

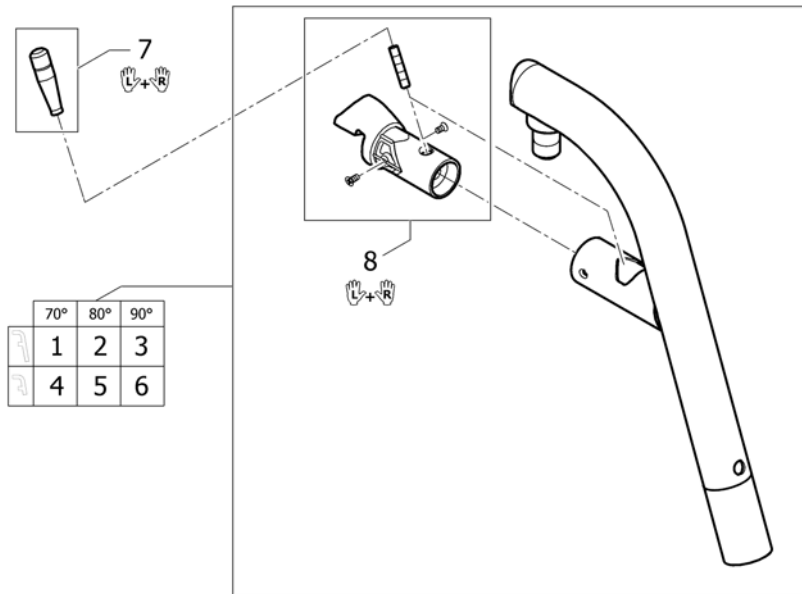





Pos.	Part number	Description	Valid from → until	Qty
1	1532016-0411	Swing Away 70°, black, matt	20100701 →	1
1	1532016-0419	Swing Away 70°, silver grey		1
1	1532016-0442	Swing Away 70°, night shimmer		1
1	1532016-0445	Swing Away 70°, ivory white	20100701 →	1
1	1532016-0446	Swing Away 70°, purple fire		1
1	1532016-0447	Swing Away 70°, emerald green		1
1	1532016-0448	Swing Away 70°, midnight blue		1
1	1532016-0449	Swing Away 70°, umber glaze		1
1	1532016-0450	Swing Away 70°, smokey grey		1
1	1532016-3524	Swing Away 70°, royal red		1
1	1532016-5969	Swing Away 70°, royal blue		1
1	1532016-9005	Swing Away 70°, black, glossy		1
1	1532016-9119	Swing Away 70°, black, bronze	20091001 → 20100630	1
2	1532017-0411	Swing Away 80°, black, matt	20100701 →	1
2	1532017-0419	Swing Away 80°, silver grey		1
2	1532017-0442	Swing Away 80°, night shimmer		1
2	1532017-0445	Swing Away 80°, ivory white	20100701 →	1
2	1532017-0446	Swing Away 80°, purple fire		1
2	1532017-0447	Swing Away 80°, emerald green		1
2	1532017-0448	Swing Away 80°, midnight blue		1
2	1532017-0449	Swing Away 80°, umber glaze		1
2	1532017-0450	Swing Away 80°, smokey grey		1
2	1532017-3524	Swing Away 80°, royal red		1
2	1532017-5969	Swing Away 80°, royal blue		1
2	1532017-9005	Swing Away 80°, black, glossy		1
2	1532017-9119	Swing Away 80°, black, bronze	20091001 → 20100630	1
3	1532018-0411	Swing Away 90°, black, matt	20100701 →	1
3	1532018-0419	Swing Away 90°, silver grey		1
3	1532018-0442	Swing Away 90°, night shimmer		1
3	1532018-0445	Swing Away 90°, ivory white	20100701 →	1
3	1532018-0446	Swing Away 90°, purple fire		1

Legrest hanger


Pos.	Part number	Description	Valid from → until	Qty
3	1532018-0447	Swing Away 90°, emerald green		1
3	1532018-0448	Swing Away 90°, midnight blue		1
3	1532018-0449	Swing Away 90°, umber glaze		1
3	1532018-0450	Swing Away 90°, smokey grey		1
3	1532018-3524	Swing Away 90°, royal red		1
3	1532018-5969	Swing Away 90°, royal blue		1
3	1532018-9005	Swing Away 90°, black, glossy		1
3	1532018-9119	Swing Away 90°, black, bronze	20091001 → 20100630	1
4	1545428-0411	Swing Away 70° short, black, matt	20100701 →	1
4	1545428-0419	Swing Away 70° short, silver grey		1
4	1545428-0442	Swing Away 70° short, night shimmer		1
4	1545428-0445	Swing Away 70° short, ivory white	20100701 →	1
4	1545428-0446	Swing Away 70° short, purple fire		1
4	1545428-0447	Swing Away 70° short, emerald green		1
4	1545428-0448	Swing Away 70° short, midnight blue		1
4	1545428-0449	Swing Away 70° short, umber glaze		1
4	1545428-0450	Swing Away 70° short, smokey grey		1
4	1545428-3524	Swing Away 70° short, royal red		1
4	1545428-5969	Swing Away 70° short, royal blue		1
4	1545428-9005	Swing Away 70° short, black, glossy		1
4	1545428-9119	Swing Away 70° short, black, bronze	20091001 → 20100630	1
5	1545429-0411	Swing Away 80° short, black, matt	20100701 →	1
5	1545429-0419	Swing Away 80° short, silver grey		1
5	1545429-0442	Swing Away 80° short, night shimmer		1
5	1545429-0445	Swing Away 80° short, ivory white	20100701 →	1
5	1545429-0446	Swing Away 80° short, purple fire		1
5	1545429-0447	Swing Away 80° short, emerald green		1
5	1545429-0448	Swing Away 80° short, midnight blue		1
5	1545429-0449	Swing Away 80° short, umber glaze		1
5	1545429-0450	Swing Away 80° short, smokey grey		1
5	1545429-3524	Swing Away 80° short, royal red		1
5	1545429-5969	Swing Away 80° short, royal blue		1
5	1545429-9005	Swing Away 80° short, black, glossy		1
5	1545429-9119	Swing Away 80° short, black, bronze	20091001 → 20100630	1
6	1545430-0411	Swing Away 90° short, black, matt	20100701 →	1
6	1545430-0419	Swing Away 90° short, silver grey		1
6	1545430-0442	Swing Away 90° short, night shimmer		1
6	1545430-0445	Swing Away 90° short, ivory white	20100701 →	1
6	1545430-0446	Swing Away 90° short, purple fire		1
6	1545430-0447	Swing Away 90° short, emerald green		1
6	1545430-0448	Swing Away 90° short, midnight blue		1
6	1545430-0449	Swing Away 90° short, umber glaze		1
6	1545430-0450	Swing Away 90° short, smokey grey		1
6	1545430-3524	Swing Away 90° short, royal red		1
6	1545430-5969	Swing Away 90° short, royal blue		1
6	1545430-9005	Swing Away 90° short, black, glossy		1
6	1545430-9119	Swing Away 90° short, black, bronze	20091001 → 20100630	1
7	1532019-0004	Swing Away release lever, black, pair		1
7	1532019-0400	Swing Away release lever, techno-anthrazit, pair		1
7	1532019-0401	Swing Away release lever, lemon, pair		1
7	1532019-0402	Swing Away release lever, apricot, pair		1
7	1532019-0404	Swing Away release lever, b-blue, pair		1
7	1532019-0440	Swing Away release lever, comet red, pair		1
7	1532019-0441	Swing Away release lever, arctic blue, pair		1
7	1532019-0444	Swing Away release lever, copper glaze, pair		1

Pos.	Part number	Description	Valid from → until	Qty
8	1536254	Locking mechanism for swing-away, kit, pair		1

FOOTREST

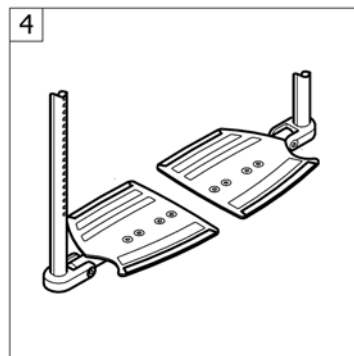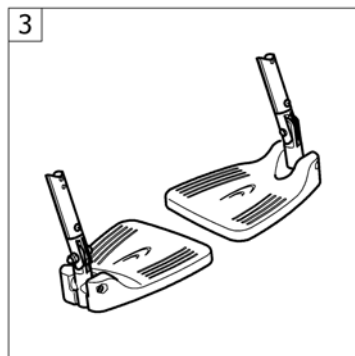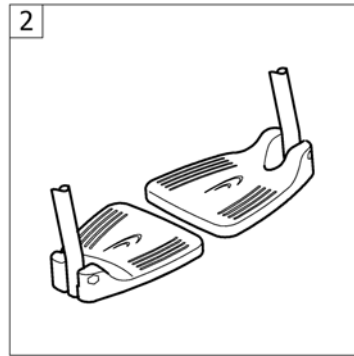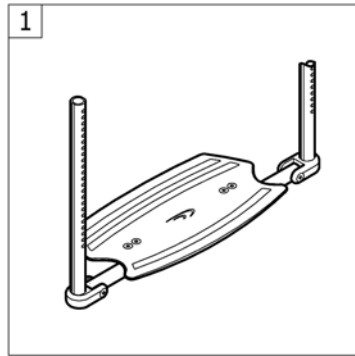


Pos.	Part number	Description	Valid from → until	Qty
1	-	Footrest 1-piece		1
2	-	Footrest 2-piece		1
3	-	Footrest 2-piece, angle adjustable		1
4	-	Footrest carbon 2-pieces		1

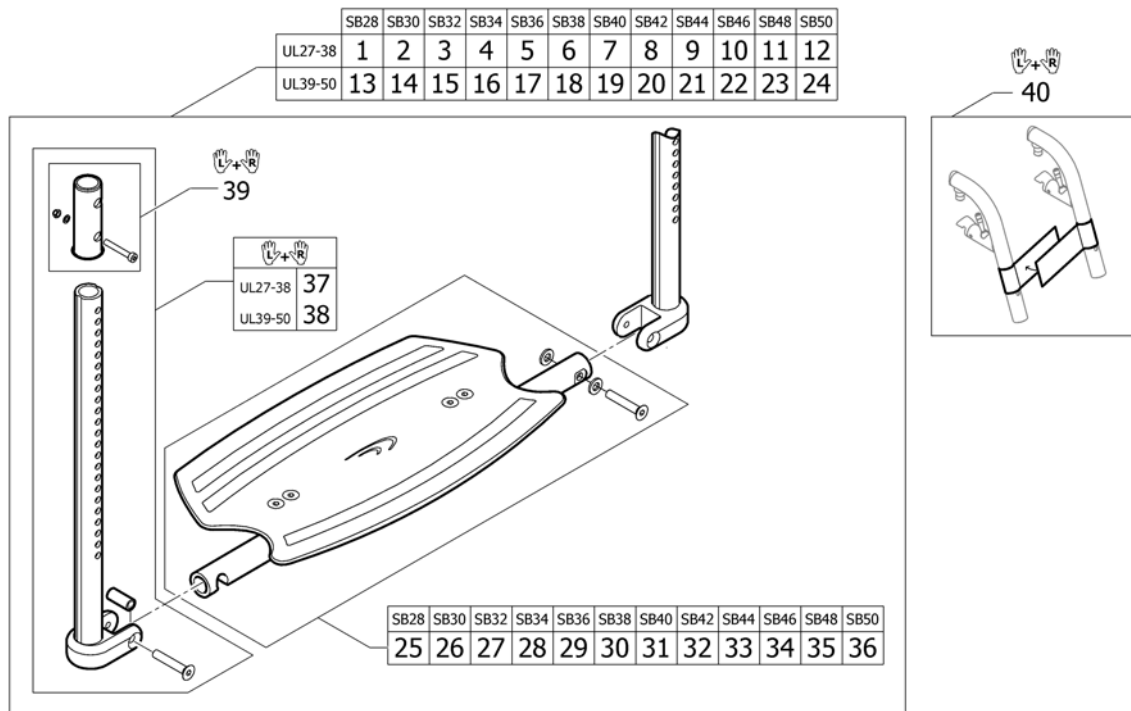

Pos.	Part number	Description	Valid from → until	Qty
1	1536064	Footrest 1-piece complete SB28 short		1
2	1536065	Footrest 1-piece complete SB30 short		1
3	1535912	Footrest 1-piece complete SB32 short		1
4	1535913	Footrest 1-piece complete SB34 short		1
5	1535914	Footrest 1-piece complete SB36 short		1
6	1535915	Footrest 1-piece complete SB38 short		1
7	1535916	Footrest 1-piece complete SB40 short		1
8	1535917	Footrest 1-piece complete SB42 short		1
9	1535918	Footrest 1-piece complete SB44 short		1
10	1535919	Footrest 1-piece complete SB46 short		1
11	1535920	Footrest 1-piece complete SB48 short		1
12	1535921	Footrest 1-piece complete SB50 short		1
13	1536066	Footrest 1-piece complete SB28 long		1
14	1536067	Footrest 1-piece complete SB30 long		1
15	1535922	Footrest 1-piece complete SB32 long		1
16	1535923	Footrest 1-piece complete SB34 long		1
17	1535924	Footrest 1-piece complete SB36 long		1
18	1535925	Footrest 1-piece complete SB38 long		1
19	1535926	Footrest 1-piece complete SB40 long		1
20	1535927	Footrest 1-piece complete SB42 long		1
21	1535928	Footrest 1-piece complete SB44 long		1
22	1535929	Footrest 1-piece complete SB46 long		1
23	1535930	Footrest 1-piece complete SB48 long		1
24	1535931	Footrest 1-piece complete SB50 long		1
25	1536068	Footrest 1-piece SB28		1
26	1536069	Footrest 1-piece SB30		1
27	1535932	Footrest 1-piece SB32		1
28	1535933	Footrest 1-piece SB34		1
29	1535974	Footrest 1-piece SB36		1
30	1535975	Footrest 1-piece SB38		1
31	1535976	Footrest 1-piece SB40		1

Footrest 1-piece

Pos.	Part number	Description	Valid from → until	Qty
32	1535977	Footrest 1-piece SB42		1
33	1535978	Footrest 1-piece SB44		1
34	1535979	Footrest 1-piece SB46		1
35	1535980	Footrest 1-piece SB48		1
36	1535981	Footrest 1-piece SB50		1
37	1535982	Telescopic tube kit for footrest 1-piece short, pair		1
38	1535983	Telescopic tube kit for footrest 1-piece long, pair		1
39	1532398	Fixation kit footplate telescope		1
40	1536082	Calf strap 2-pcs complete		1

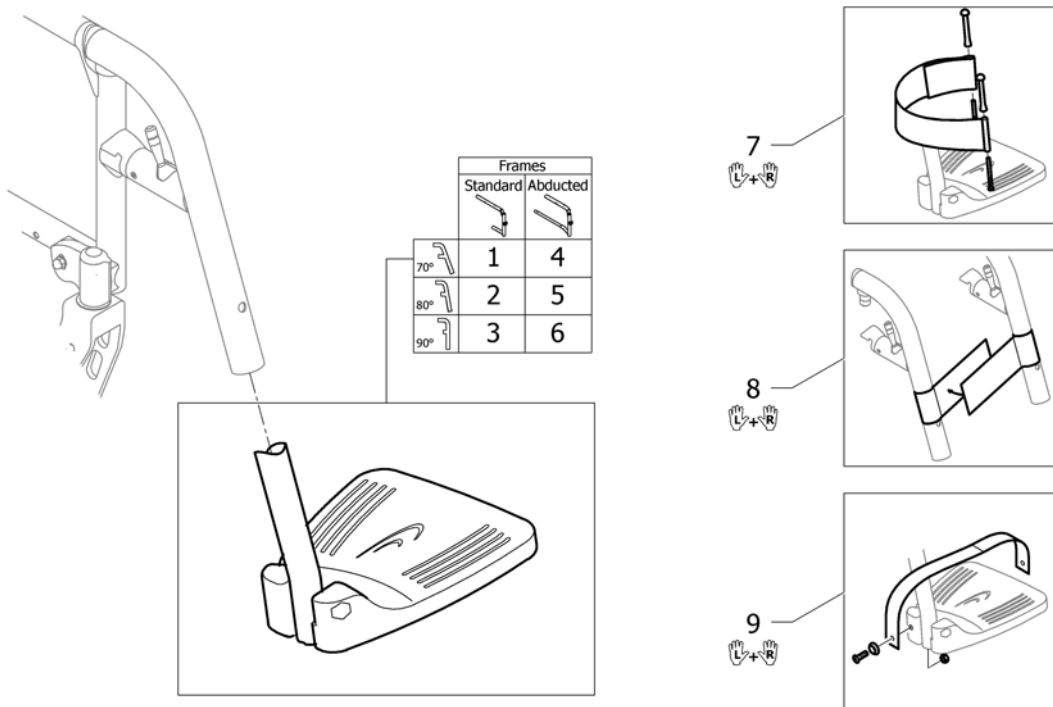

Pos.	Part number	Description	Valid from → until	Qty
1	-	Footrest 2-pieces for swing away 70° on front frames standard		1
2	-	Footrest 2-pieces for swing away 80° on front frames standard		1
3	-	Footrest 2-pieces for swing away 90° on front frames standard		1
4	-	Footrest 2-pieces for swing away 70° on front frames abducted		1
5	-	Footrest 2-pieces for swing away 80° on front frames abducted		1
6	-	Footrest 2-pieces for swing away 90° on front frames abducted		1
7	1437834	Heel loop kit short		1
8	1536082	Calf strap 2-pcs complete		1
9	1438143	instep strap UL/C04		1

Footrest 2-pieces for swing away 70° on front frames standard

Standard frames

70°	SB34	SB36	SB38	SB40	SB42	SB44	SB46	SB48	SB50
UL26-33	-	-	1	1	2	2	3	3	4
UL34-39	5	5	6	6	7	7	8	8	9
UL40-45	10	10	11	11	12	12	13	13	14
UL46-51	10	10	11	11	12	12	13	13	14

70°	SB34	SB36	SB38	SB40	SB42	SB44	SB46	SB48	SB50
UL26-33	-	-	15	15	16	16	17	17	18
UL34-39	19	19	20	20	21	21	22	22	23
UL40-45	24	24	25	25	26	26	27	27	28
UL46-51	24	24	25	25	26	26	27	27	28


70°	SB34	SB36	SB38	SB40	SB42	SB44	SB46	SB48	SB50
UL26-33	-	-	29	29	30	30	31	31	32
UL34-39	33	33	34	34	35	35	36	36	37
UL40-45	38	38	39	39	40	40	41	41	42
UL46-51	38	38	39	39	40	40	41	41	42

Pos.	Part number	Description	Valid from → until	Qty
1	1544821	Footrest 2-pc. 155mm, for frame 70° & 80° high mounted, pair, complete		1
2	1544822	Footrest 2-pc. 175mm, for frame 70° & 80° high mounted, pair, complete		1
3	1544823	Footrest 2-pc. 195mm, for frame 70° & 80° high mounted, pair, complete		1
4	1544824	Footrest 2-pc. 215mm, for frame 70° & 80° high mounted, pair, complete		1
5	1544826	Footrest 2-pc. 155mm, for frame 70° & 80° short, pair, complete		1
6	1544827	Footrest 2-pc. 175mm, for frame 70° & 80° short, pair, complete		1
7	1544828	Footrest 2-pc. 195mm, for frame 70° & 80° short, pair, complete		1
8	1544906	Footrest 2-pc. 215mm, for frame 70° & 80° short, pair, complete		1
9	1544907	Footrest 2-pc. 235mm, for frame 70° & 80° short, pair, complete		1
10	1544908	Footrest 2-pc. 155mm, for frame 70° & 80° long, pair, complete		1
11	1544909	Footrest 2-pc. 175mm, for frame 70° & 80° long, pair, complete		1
12	1544910	Footrest 2-pc. 195mm, for frame 70° & 80° long, pair, complete		1
13	1544911	Footrest 2-pc. 215mm, for frame 70° & 80° long, pair, complete		1
14	1544912	Footrest 2-pc. 235mm, for frame 70° & 80° long, pair, complete		1
15	1544913	Footrest 2-pc. 155mm, for frame 70° & 80° high mounted, left, complete		1
16	1544914	Footrest 2-pc. 175mm, for frame 70° & 80° high mounted, left, complete		1
17	1544915	Footrest 2-pc. 195mm, for frame 70° & 80° high mounted, left, complete		1
18	1544916	Footrest 2-pc. 215mm, for frame 70° & 80° high mounted, left, complete		1
19	1545357	Footrest 2-pc. 155mm, for frame 70° & 80° short, left, complete		1
20	1545358	Footrest 2-pc. 175mm, for frame 70° & 80° short, left, complete		1
21	1545359	Footrest 2-pc. 195mm, for frame 70° & 80° short, left, complete		1
22	1545360	Footrest 2-pc. 215mm, for frame 70° & 80° short, left, complete		1
23	1545361	Footrest 2-pc. 235mm, for frame 70° & 80° short, left, complete		1
24	1545362	Footrest 2-pc. 155mm, for frame 70° & 80° long, left, complete		1
25	1545363	Footrest 2-pc. 175mm, for frame 70° & 80° long, left, complete		1
26	1545364	Footrest 2-pc. 195mm, for frame 70° & 80° long, left, complete		1
27	1545365	Footrest 2-pc. 215mm, for frame 70° & 80° long, left, complete		1
28	1545366	Footrest 2-pc. 235mm, for frame 70° & 80° long, left, complete		1
29	1545367	Footrest 2-pc. 155mm, for frame 70° & 80° high mounted, right, complete		1
30	1545368	Footrest 2-pc. 175mm, for frame 70° & 80° high mounted, right, complete		1
31	1545369	Footrest 2-pc. 195mm, for frame 70° & 80° high mounted, right, complete		1

Footrest 2-pieces for swing away 70° on front frames standard

Pos.	Part number	Description	Valid from → until	Qty
32	1545370	Footrest 2-pc. 215mm, for frame 70° & 80° high mounted, right, complete		1
33	1545372	Footrest 2-pc. 155mm, for frame 70° & 80° short, right, complete		1
34	1545373	Footrest 2-pc. 175mm, for frame 70° & 80° short, right, complete		1
35	1545374	Footrest 2-pc. 195mm, for frame 70° & 80° short, right, complete		1
36	1545375	Footrest 2-pc. 215mm, for frame 70° & 80° short, right, complete		1
37	1545376	Footrest 2-pc. 235mm, for frame 70° & 80° short, right, complete		1
38	1545377	Footrest 2-pc. 155mm, for frame 70° & 80° long, right, complete		1
39	1545378	Footrest 2-pc. 175mm, for frame 70° & 80° long, right, complete		1
40	1545379	Footrest 2-pc. 195mm, for frame 70° & 80° long, right, complete		1
41	1545380	Footrest 2-pc. 215mm, for frame 70° & 80° long, right, complete		1
42	1545381	Footrest 2-pc. 235mm, for frame 70° & 80° long, right, complete		1
43	1544748	Telescopic tube kit for footrest 2-pc. for frame 70° and 80° high mounted, pair		1
44	1544749	Telescopic tube kit for footrest 2-pc. for frame 70° and 80° short, pair		1
45	1544750	Telescopic tube kit for footrest 2-pc. for frame 70° and 80° long, pair		1
46	1544613	Clamping kit for footrest high mounted, pair		1
47	1532398	Fixation kit footplate telescope		1

Footrest 2-pieces for swing away 80° on front frames standard

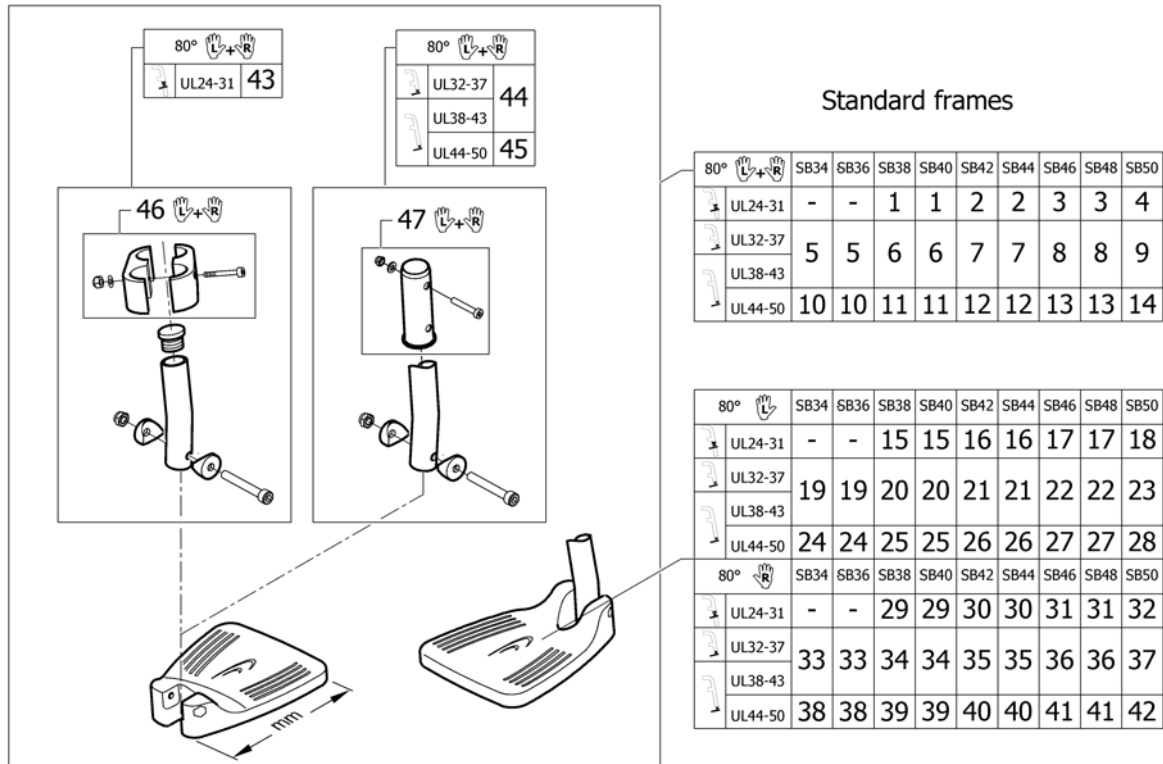


Pos.	Part number	Description	Valid from → until	Qty
1	1544821	Footrest 2-pc. 155mm, for frame 70° & 80° high mounted, pair, complete		1
2	1544822	Footrest 2-pc. 175mm, for frame 70° & 80° high mounted, pair, complete		1
3	1544823	Footrest 2-pc. 195mm, for frame 70° & 80° high mounted, pair, complete		1
4	1544824	Footrest 2-pc. 215mm, for frame 70° & 80° high mounted, pair, complete		1
5	1544826	Footrest 2-pc. 155mm, for frame 70° & 80° short, pair, complete		1
6	1544827	Footrest 2-pc. 175mm, for frame 70° & 80° short, pair, complete		1
7	1544828	Footrest 2-pc. 195mm, for frame 70° & 80° short, pair, complete		1
8	1544906	Footrest 2-pc. 215mm, for frame 70° & 80° short, pair, complete		1
9	1544907	Footrest 2-pc. 235mm, for frame 70° & 80° short, pair, complete		1
10	1544908	Footrest 2-pc. 155mm, for frame 70° & 80° long, pair, complete		1
11	1544909	Footrest 2-pc. 175mm, for frame 70° & 80° long, pair, complete		1
12	1544910	Footrest 2-pc. 195mm, for frame 70° & 80° long, pair, complete		1
13	1544911	Footrest 2-pc. 215mm, for frame 70° & 80° long, pair, complete		1
14	1544912	Footrest 2-pc. 235mm, for frame 70° & 80° long, pair, complete		1
15	1544913	Footrest 2-pc. 155mm, for frame 70° & 80° high mounted, left, complete		1
16	1544914	Footrest 2-pc. 175mm, for frame 70° & 80° high mounted, left, complete		1
17	1544915	Footrest 2-pc. 195mm, for frame 70° & 80° high mounted, left, complete		1
18	1544916	Footrest 2-pc. 215mm, for frame 70° & 80° high mounted, left, complete		1
19	1545357	Footrest 2-pc. 155mm, for frame 70° & 80° short, left, complete		1
20	1545358	Footrest 2-pc. 175mm, for frame 70° & 80° short, left, complete		1
21	1545359	Footrest 2-pc. 195mm, for frame 70° & 80° short, left, complete		1
22	1545360	Footrest 2-pc. 215mm, for frame 70° & 80° short, left, complete		1
23	1545361	Footrest 2-pc. 235mm, for frame 70° & 80° short, left, complete		1
24	1545362	Footrest 2-pc. 155mm, for frame 70° & 80° long, left, complete		1
25	1545363	Footrest 2-pc. 175mm, for frame 70° & 80° long, left, complete		1
26	1545364	Footrest 2-pc. 195mm, for frame 70° & 80° long, left, complete		1
27	1545365	Footrest 2-pc. 215mm, for frame 70° & 80° long, left, complete		1
28	1545366	Footrest 2-pc. 235mm, for frame 70° & 80° long, left, complete		1
29	1545367	Footrest 2-pc. 155mm, for frame 70° & 80° high mounted, right, complete		1
30	1545368	Footrest 2-pc. 175mm, for frame 70° & 80° high mounted, right, complete		1
31	1545369	Footrest 2-pc. 195mm, for frame 70° & 80° high mounted, right, complete		1

Footrest 2-pieces for swing away 80° on front frames standard

Pos.	Part number	Description	Valid from → until	Qty
32	1545370	Footrest 2-pc. 215mm, for frame 70° & 80° high mounted, right, complete		1
33	1545372	Footrest 2-pc. 155mm, for frame 70° & 80° short, right, complete		1
34	1545373	Footrest 2-pc. 175mm, for frame 70° & 80° short, right, complete		1
35	1545374	Footrest 2-pc. 195mm, for frame 70° & 80° short, right, complete		1
36	1545375	Footrest 2-pc. 215mm, for frame 70° & 80° short, right, complete		1
37	1545376	Footrest 2-pc. 235mm, for frame 70° & 80° short, right, complete		1
38	1545377	Footrest 2-pc. 155mm, for frame 70° & 80° long, right, complete		1
39	1545378	Footrest 2-pc. 175mm, for frame 70° & 80° long, right, complete		1
40	1545379	Footrest 2-pc. 195mm, for frame 70° & 80° long, right, complete		1
41	1545380	Footrest 2-pc. 215mm, for frame 70° & 80° long, right, complete		1
42	1545381	Footrest 2-pc. 235mm, for frame 70° & 80° long, right, complete		1
43	1544748	Telescopic tube kit for footrest 2-pc. for frame 70° and 80° high mounted, pair		1
44	1544749	Telescopic tube kit for footrest 2-pc. for frame 70° and 80° short, pair		1
45	1544750	Telescopic tube kit for footrest 2-pc. for frame 70° and 80° long, pair		1
46	1544613	Clamping kit for footrest high mounted, pair		1
47	1532398	Fixation kit footplate telescope		1

Footrest 2-pieces for swing away 90° on front frames standard

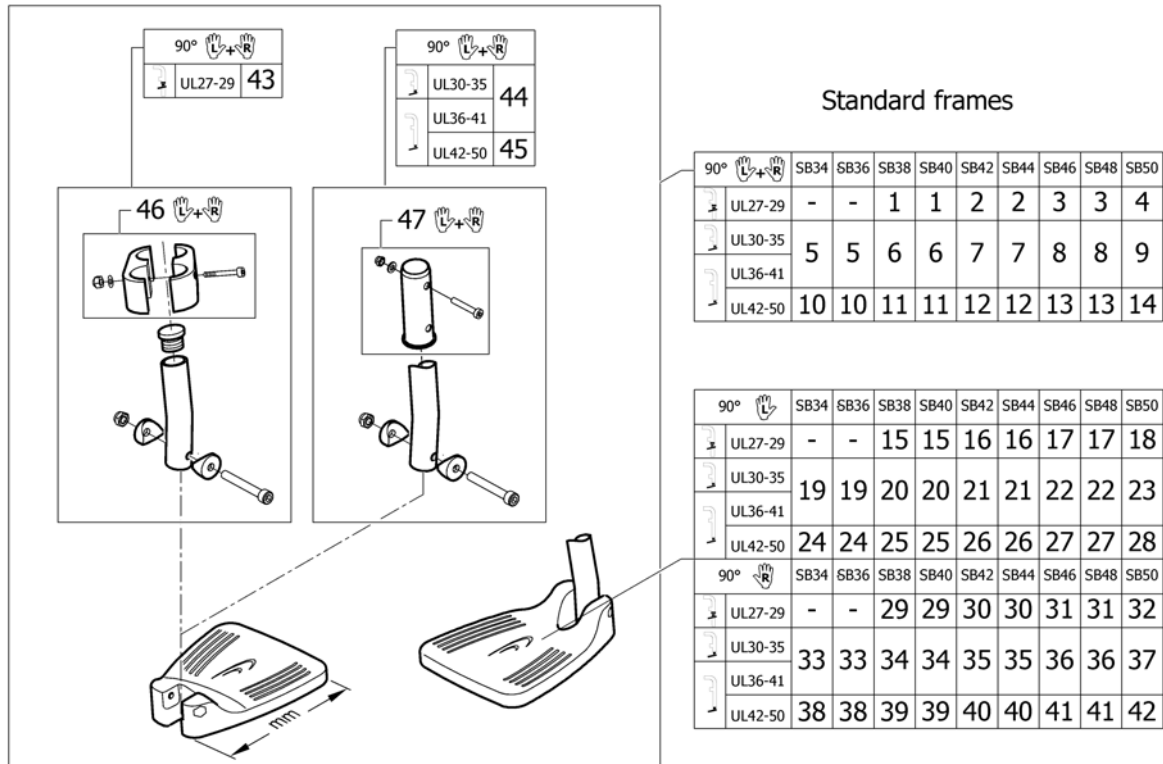


Pos.	Part number	Description	Valid from → until	Qty
1	1545382	Footrest 2-pc. 155mm, for frame 90° high mounted, pair, complete		1
2	1545383	Footrest 2-pc. 175mm, for frame 90° high mounted, pair, complete		1
3	1545384	Footrest 2-pc. 195mm, for frame 90° high mounted, pair, complete		1
4	1545385	Footrest 2-pc. 215mm, for frame 90° high mounted, pair, complete		1
5	1545387	Footrest 2-pc. 155mm, for frame 90° short, pair, complete		1
6	1545388	Footrest 2-pc. 175mm, for frame 90° short, pair, complete		1
7	1545389	Footrest 2-pc. 195mm, for frame 90° short, pair, complete		1
8	1545390	Footrest 2-pc. 215mm, for frame 90° short, pair, complete		1
9	1545391	Footrest 2-pc. 235mm, for frame 90° short, pair, complete		1
10	1545392	Footrest 2-pc. 155mm, for frame 90° long, pair, complete		1
11	1545393	Footrest 2-pc. 175mm, for frame 90° long, pair, complete		1
12	1545394	Footrest 2-pc. 195mm, for frame 90° long, pair, complete		1
13	1545395	Footrest 2-pc. 215mm, for frame 90° long, pair, complete		1
14	1545396	Footrest 2-pc. 235mm, for frame 90° long, pair, complete		1
15	1545397	Footrest 2-pc. 155mm, for frame 90° high mounted, left, complete		1
16	1545398	Footrest 2-pc. 175mm, for frame 90° high mounted, left, complete		1
17	1545399	Footrest 2-pc. 195mm, for frame 90° high mounted, left, complete		1
18	1545400	Footrest 2-pc. 215mm, for frame 90° high mounted, left, complete		1
19	1545402	Footrest 2-pc. 155mm, for frame 90° short, left, complete		1
20	1545403	Footrest 2-pc. 175mm, for frame 90° short, left, complete		1
21	1545404	Footrest 2-pc. 195mm, for frame 90° short, left, complete		1
22	1545405	Footrest 2-pc. 215mm, for frame 90° short, left, complete		1
23	1545406	Footrest 2-pc. 235mm, for frame 90° short, left, complete		1
24	1545407	Footrest 2-pc. 155mm, for frame 90° long, left, complete		1
25	1545408	Footrest 2-pc. 175mm, for frame 90° long, left, complete		1
26	1545409	Footrest 2-pc. 195mm, for frame 90° long, left, complete		1
27	1545410	Footrest 2-pc. 215mm, for frame 90° long, left, complete		1
28	1545411	Footrest 2-pc. 235mm, for frame 90° long, left, complete		1
29	1545412	Footrest 2-pc. 155mm, for frame 90° high mounted, right, complete		1
30	1545413	Footrest 2-pc. 175mm, for frame 90° high mounted, right, complete		1
31	1545414	Footrest 2-pc. 195mm, for frame 90° high mounted, right, complete		1

Footrest 2-pieces for swing away 90° on front frames standard

Pos.	Part number	Description	Valid from → until	Qty
32	1545415	Footrest 2-pc. 215mm, for frame 90° high mounted, right, complete		1
33	1545417	Footrest 2-pc. 155mm, for frame 90° short, right, complete		1
34	1545418	Footrest 2-pc. 175mm, for frame 90° short, right, complete		1
35	1545419	Footrest 2-pc. 195mm, for frame 90° short, right, complete		1
36	1545420	Footrest 2-pc. 215mm, for frame 90° short, right, complete		1
37	1545421	Footrest 2-pc. 235mm, for frame 90° short, right, complete		1
38	1545422	Footrest 2-pc. 155mm, for frame 90° long, right, complete		1
39	1545423	Footrest 2-pc. 175mm, for frame 90° long, right, complete		1
40	1545424	Footrest 2-pc. 195mm, for frame 90° long, right, complete		1
41	1545425	Footrest 2-pc. 215mm, for frame 90° long, right, complete		1
42	1545426	Footrest 2-pc. 235mm, for frame 90° long, right, complete		1
43	1544818	Telescopic tube kit for footrest 2-pc. for frame 90° high mounted, pair		1
44	1544819	Telescopic tube kit for footrest 2-pc. for frame 90° short, pair		1
45	1544820	Telescopic tube kit for footrest 2-pc. for frame 90° long, pair		1
46	1544613	Clamping kit for footrest high mounted, pair		1
47	1532398	Fixation kit footplate telescope		1

Footrest 2-pieces for swing away 70° on front frames abducted

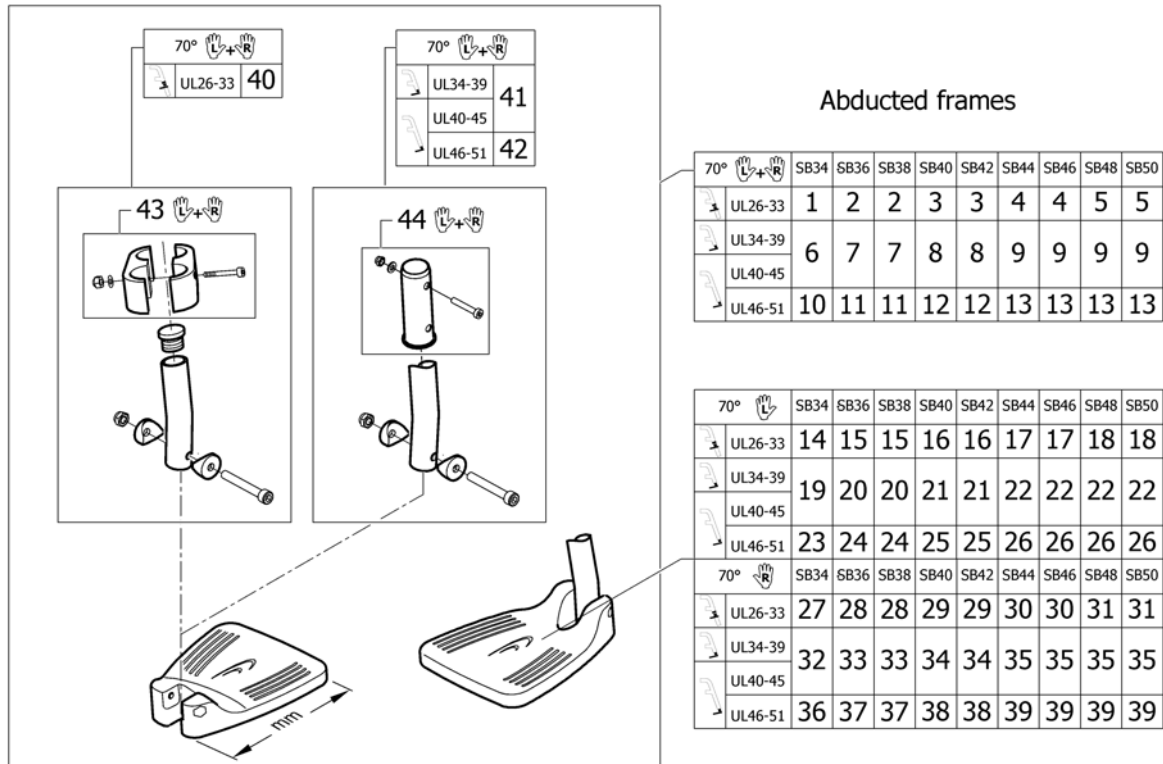


Pos.	Part number	Description	Valid from → until	Qty
1	1544821	Footrest 2-pc. 155mm, for frame 70° & 80° high mounted, pair, complete		1
2	1544822	Footrest 2-pc. 175mm, for frame 70° & 80° high mounted, pair, complete		1
3	1544823	Footrest 2-pc. 195mm, for frame 70° & 80° high mounted, pair, complete		1
4	1544824	Footrest 2-pc. 215mm, for frame 70° & 80° high mounted, pair, complete		1
5	1544825	Footrest 2-pc. 235mm, for frame 70° & 80° high mounted, pair, complete		1
6	1544827	Footrest 2-pc. 175mm, for frame 70° & 80° short, pair, complete		1
7	1544828	Footrest 2-pc. 195mm, for frame 70° & 80° short, pair, complete		1
8	1544906	Footrest 2-pc. 215mm, for frame 70° & 80° short, pair, complete		1
9	1544907	Footrest 2-pc. 235mm, for frame 70° & 80° short, pair, complete		1
10	1544909	Footrest 2-pc. 175mm, for frame 70° & 80° long, pair, complete		1
11	1544910	Footrest 2-pc. 195mm, for frame 70° & 80° long, pair, complete		1
12	1544911	Footrest 2-pc. 215mm, for frame 70° & 80° long, pair, complete		1
13	1544912	Footrest 2-pc. 235mm, for frame 70° & 80° long, pair, complete		1
14	1544913	Footrest 2-pc. 155mm, for frame 70° & 80° high mounted, left, complete		1
15	1544914	Footrest 2-pc. 175mm, for frame 70° & 80° high mounted, left, complete		1
16	1544915	Footrest 2-pc. 195mm, for frame 70° & 80° high mounted, left, complete		1
17	1544916	Footrest 2-pc. 215mm, for frame 70° & 80° high mounted, left, complete		1
18	1544917	Footrest 2-pc. 235mm, for frame 70° & 80° high mounted, left, complete		1
19	1545358	Footrest 2-pc. 175mm, for frame 70° & 80° short, left, complete		1
20	1545359	Footrest 2-pc. 195mm, for frame 70° & 80° short, left, complete		1
21	1545360	Footrest 2-pc. 215mm, for frame 70° & 80° short, left, complete		1
22	1545361	Footrest 2-pc. 235mm, for frame 70° & 80° short, left, complete		1
23	1545363	Footrest 2-pc. 175mm, for frame 70° & 80° long, left, complete		1
24	1545364	Footrest 2-pc. 195mm, for frame 70° & 80° long, left, complete		1
25	1545365	Footrest 2-pc. 215mm, for frame 70° & 80° long, left, complete		1
26	1545366	Footrest 2-pc. 235mm, for frame 70° & 80° long, left, complete		1
27	1545367	Footrest 2-pc. 155mm, for frame 70° & 80° high mounted, right, complete		1
28	1545368	Footrest 2-pc. 175mm, for frame 70° & 80° high mounted, right, complete		1
29	1545369	Footrest 2-pc. 195mm, for frame 70° & 80° high mounted, right, complete		1
30	1545370	Footrest 2-pc. 215mm, for frame 70° & 80° high mounted, right, complete		1
31	1545371	Footrest 2-pc. 235mm, for frame 70° & 80° high mounted, right, complete		1

Footrest 2-pieces for swing away 70° on front frames abducted

Pos.	Part number	Description	Valid from → until	Qty
32	1545373	Footrest 2-pc. 175mm, for frame 70° & 80° short, right, complete		1
33	1545374	Footrest 2-pc. 195mm, for frame 70° & 80° short, right, complete		1
34	1545375	Footrest 2-pc. 215mm, for frame 70° & 80° short, right, complete		1
35	1545376	Footrest 2-pc. 235mm, for frame 70° & 80° short, right, complete		1
36	1545378	Footrest 2-pc. 175mm, for frame 70° & 80° long, right, complete		1
37	1545379	Footrest 2-pc. 195mm, for frame 70° & 80° long, right, complete		1
38	1545380	Footrest 2-pc. 215mm, for frame 70° & 80° long, right, complete		1
39	1545381	Footrest 2-pc. 235mm, for frame 70° & 80° long, right, complete		1
40	1544748	Telescopic tube kit for footrest 2-pc. for frame 70° and 80° high mounted, pair		1
41	1544749	Telescopic tube kit for footrest 2-pc. for frame 70° and 80° short, pair		1
42	1544750	Telescopic tube kit for footrest 2-pc. for frame 70° and 80° long, pair		1
43	1544613	Clamping kit for footrest high mounted, pair		1
44	1532398	Fixation kit footplate telescope		1

Footrest 2-pieces for swing away 80° on front frames abducted

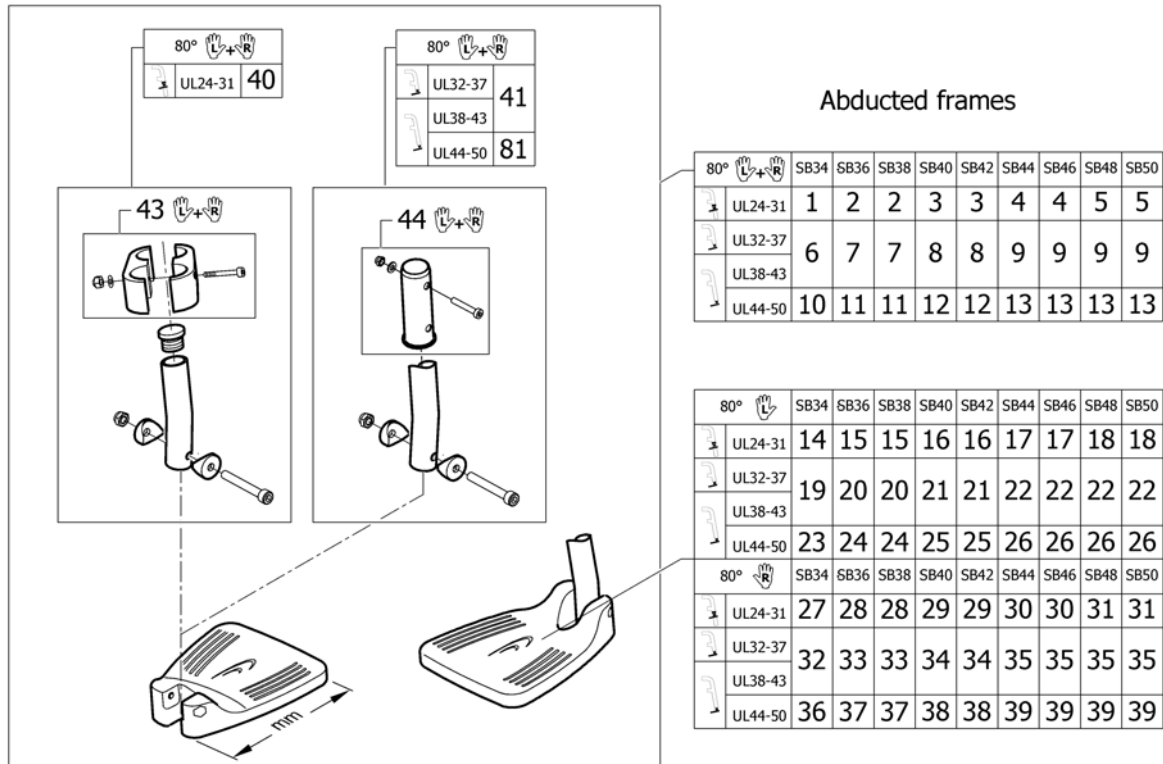


Pos.	Part number	Description	Valid from → until	Qty
1	1544821	Footrest 2-pc. 155mm, for frame 70° & 80° high mounted, pair, complete		1
2	1544822	Footrest 2-pc. 175mm, for frame 70° & 80° high mounted, pair, complete		1
3	1544823	Footrest 2-pc. 195mm, for frame 70° & 80° high mounted, pair, complete		1
4	1544824	Footrest 2-pc. 215mm, for frame 70° & 80° high mounted, pair, complete		1
5	1544825	Footrest 2-pc. 235mm, for frame 70° & 80° high mounted, pair, complete		1
6	1544827	Footrest 2-pc. 175mm, for frame 70° & 80° short, pair, complete		1
7	1544828	Footrest 2-pc. 195mm, for frame 70° & 80° short, pair, complete		1
8	1544906	Footrest 2-pc. 215mm, for frame 70° & 80° short, pair, complete		1
9	1544907	Footrest 2-pc. 235mm, for frame 70° & 80° short, pair, complete		1
10	1544909	Footrest 2-pc. 175mm, for frame 70° & 80° long, pair, complete		1
11	1544910	Footrest 2-pc. 195mm, for frame 70° & 80° long, pair, complete		1
12	1544911	Footrest 2-pc. 215mm, for frame 70° & 80° long, pair, complete		1
13	1544912	Footrest 2-pc. 235mm, for frame 70° & 80° long, pair, complete		1
14	1544913	Footrest 2-pc. 155mm, for frame 70° & 80° high mounted, left, complete		1
15	1544914	Footrest 2-pc. 175mm, for frame 70° & 80° high mounted, left, complete		1
16	1544915	Footrest 2-pc. 195mm, for frame 70° & 80° high mounted, left, complete		1
17	1544916	Footrest 2-pc. 215mm, for frame 70° & 80° high mounted, left, complete		1
18	1544917	Footrest 2-pc. 235mm, for frame 70° & 80° high mounted, left, complete		1
19	1545358	Footrest 2-pc. 175mm, for frame 70° & 80° short, left, complete		1
20	1545359	Footrest 2-pc. 195mm, for frame 70° & 80° short, left, complete		1
21	1545360	Footrest 2-pc. 215mm, for frame 70° & 80° short, left, complete		1
22	1545361	Footrest 2-pc. 235mm, for frame 70° & 80° short, left, complete		1
23	1545363	Footrest 2-pc. 175mm, for frame 70° & 80° long, left, complete		1
24	1545364	Footrest 2-pc. 195mm, for frame 70° & 80° long, left, complete		1
25	1545365	Footrest 2-pc. 215mm, for frame 70° & 80° long, left, complete		1
26	1545366	Footrest 2-pc. 235mm, for frame 70° & 80° long, left, complete		1
27	1545367	Footrest 2-pc. 155mm, for frame 70° & 80° high mounted, right, complete		1
28	1545368	Footrest 2-pc. 175mm, for frame 70° & 80° high mounted, right, complete		1
29	1545369	Footrest 2-pc. 195mm, for frame 70° & 80° high mounted, right, complete		1
30	1545370	Footrest 2-pc. 215mm, for frame 70° & 80° high mounted, right, complete		1
31	1545371	Footrest 2-pc. 235mm, for frame 70° & 80° high mounted, right, complete		1

Footrest 2-pieces for swing away 80° on front frames abducted

Pos.	Part number	Description	Valid from → until	Qty
32	1545373	Footrest 2-pc. 175mm, for frame 70° & 80° short, right, complete		1
33	1545374	Footrest 2-pc. 195mm, for frame 70° & 80° short, right, complete		1
34	1545375	Footrest 2-pc. 215mm, for frame 70° & 80° short, right, complete		1
35	1545376	Footrest 2-pc. 235mm, for frame 70° & 80° short, right, complete		1
36	1545378	Footrest 2-pc. 175mm, for frame 70° & 80° long, right, complete		1
37	1545379	Footrest 2-pc. 195mm, for frame 70° & 80° long, right, complete		1
38	1545380	Footrest 2-pc. 215mm, for frame 70° & 80° long, right, complete		1
39	1545381	Footrest 2-pc. 235mm, for frame 70° & 80° long, right, complete		1
40	1544748	Telescopic tube kit for footrest 2-pc. for frame 70° and 80° high mounted, pair		1
41	1544749	Telescopic tube kit for footrest 2-pc. for frame 70° and 80° short, pair		1
42	1544750	Telescopic tube kit for footrest 2-pc. for frame 70° and 80° long, pair		1
43	1544613	Clamping kit for footrest high mounted, pair		1
44	1532398	Fixation kit footplate telescope		1

Footrest 2-pieces for swing away 90° on front frames abducted

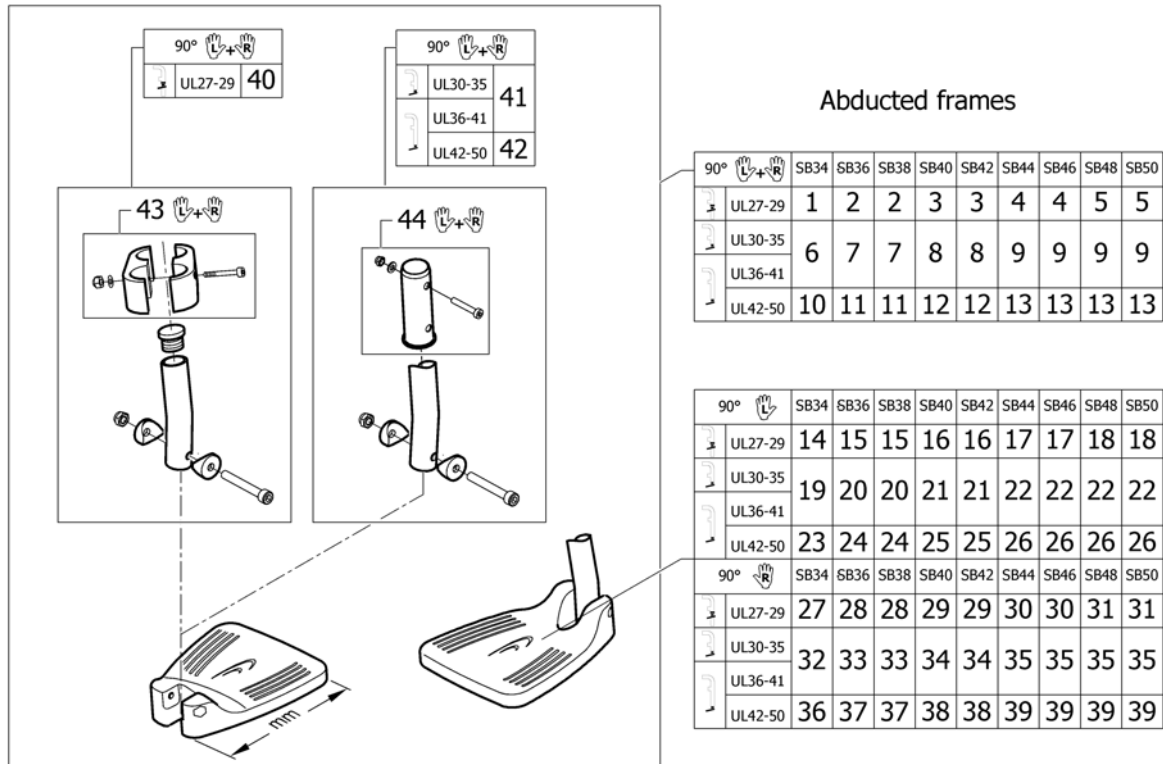


Pos.	Part number	Description	Valid from → until	Qty
1	1545382	Footrest 2-pc. 155mm, for frame 90° high mounted, pair, complete		1
2	1545383	Footrest 2-pc. 175mm, for frame 90° high mounted, pair, complete		1
3	1545384	Footrest 2-pc. 195mm, for frame 90° high mounted, pair, complete		1
4	1545385	Footrest 2-pc. 215mm, for frame 90° high mounted, pair, complete		1
5	1545386	Footrest 2-pc. 235mm, for frame 90° high mounted, pair, complete		1
6	1545388	Footrest 2-pc. 175mm, for frame 90° short, pair, complete		1
7	1545389	Footrest 2-pc. 195mm, for frame 90° short, pair, complete		1
8	1545390	Footrest 2-pc. 215mm, for frame 90° short, pair, complete		1
9	1545391	Footrest 2-pc. 235mm, for frame 90° short, pair, complete		1
10	1545393	Footrest 2-pc. 175mm, for frame 90° long, pair, complete		1
11	1545394	Footrest 2-pc. 195mm, for frame 90° long, pair, complete		1
12	1545395	Footrest 2-pc. 215mm, for frame 90° long, pair, complete		1
13	1545396	Footrest 2-pc. 235mm, for frame 90° long, pair, complete		1
14	1545397	Footrest 2-pc. 155mm, for frame 90° high mounted, left, complete		1
15	1545398	Footrest 2-pc. 175mm, for frame 90° high mounted, left, complete		1
16	1545399	Footrest 2-pc. 195mm, for frame 90° high mounted, left, complete		1
17	1545400	Footrest 2-pc. 215mm, for frame 90° high mounted, left, complete		1
18	1545401	Footrest 2-pc. 235mm, for frame 90° high mounted, left, complete		1
19	1545403	Footrest 2-pc. 175mm, for frame 90° short, left, complete		1
20	1545404	Footrest 2-pc. 195mm, for frame 90° short, left, complete		1
21	1545405	Footrest 2-pc. 215mm, for frame 90° short, left, complete		1
22	1545406	Footrest 2-pc. 235mm, for frame 90° short, left, complete		1
23	1545408	Footrest 2-pc. 175mm, for frame 90° long, left, complete		1
24	1545409	Footrest 2-pc. 195mm, for frame 90° long, left, complete		1
25	1545410	Footrest 2-pc. 215mm, for frame 90° long, left, complete		1
26	1545411	Footrest 2-pc. 235mm, for frame 90° long, left, complete		1
27	1545412	Footrest 2-pc. 155mm, for frame 90° high mounted, right, complete		1
28	1545413	Footrest 2-pc. 175mm, for frame 90° high mounted, right, complete		1
29	1545414	Footrest 2-pc. 195mm, for frame 90° high mounted, right, complete		1
30	1545415	Footrest 2-pc. 215mm, for frame 90° high mounted, right, complete		1
31	1545416	Footrest 2-pc. 235mm, for frame 90° high mounted, right, complete		1

Footrest 2-pieces for swing away 90° on front frames abducted

Pos.	Part number	Description	Valid from → until	Qty
32	1545418	Footrest 2-pc. 175mm, for frame 90° short, right, complete		1
33	1545419	Footrest 2-pc. 195mm, for frame 90° short, right, complete		1
34	1545420	Footrest 2-pc. 215mm, for frame 90° short, right, complete		1
35	1545421	Footrest 2-pc. 235mm, for frame 90° short, right, complete		1
36	1545423	Footrest 2-pc. 175mm, for frame 90° long, right, complete		1
37	1545424	Footrest 2-pc. 195mm, for frame 90° long, right, complete		1
38	1545425	Footrest 2-pc. 215mm, for frame 90° long, right, complete		1
39	1545426	Footrest 2-pc. 235mm, for frame 90° long, right, complete		1
40	1544818	Telescopic tube kit for footrest 2-pc. for frame 90° high mounted, pair		1
41	1544819	Telescopic tube kit for footrest 2-pc. for frame 90° short, pair		1
42	1544820	Telescopic tube kit for footrest 2-pc. for frame 90° long, pair		1
43	1544613	Clamping kit for footrest high mounted, pair		1
44	1532398	Fixation kit footplate telescope		1

Footrest 2-piece, angle adjustable

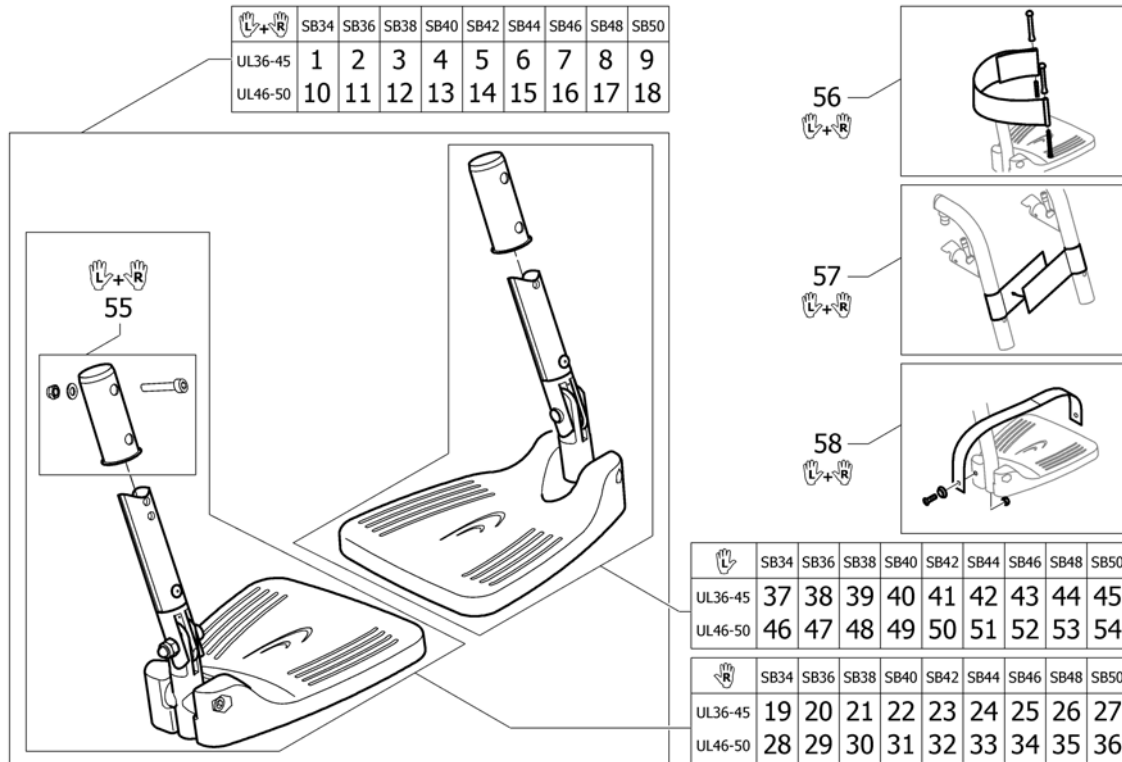


Pos.	Part number	Description	Valid from → until	Qty
1	1535164	Footrest 2-pc. angle adjustable complete, pair SB34 short		1
2	1535165	Footrest 2-pc. angle adjustable complete, pair SB36 short		1
3	1535166	Footrest 2-pc. angle adjustable complete, pair SB38 short		1
4	1535167	Footrest 2-pc. angle adjustable complete, pair SB40 short		1
5	1535168	Footrest 2-pc. angle adjustable complete, pair SB42 short		1
6	1535461	Footrest 2-pc. angle adjustable complete, pair SB44 short		1
7	1535495	Footrest 2-pc. angle adjustable complete, pair SB46 short		1
8	1535806	Footrest 2-pc. angle adjustable complete, pair SB48 short		1
9	1535866	Footrest 2-pc. angle adjustable complete, pair SB50 short		1
10	1535867	Footrest 2-pc. angle adjustable complete, pair SB34 long		1
11	1535868	Footrest 2-pc. angle adjustable complete, pair SB36 long		1
12	1535869	Footrest 2-pc. angle adjustable complete, pair SB38 long		1
13	1535870	Footrest 2-pc. angle adjustable complete, pair SB40 long		1
14	1535871	Footrest 2-pc. angle adjustable complete, pair SB42 long		1
15	1535872	Footrest 2-pc. angle adjustable complete, pair SB44 long		1
16	1535873	Footrest 2-pc. angle adjustable complete, pair SB46 long		1
17	1535874	Footrest 2-pc. angle adjustable complete, pair SB48 long		1
18	1535875	Footrest 2-pc. angle adjustable complete, pair SB50 long		1
19	1535876	Footrest 2-pc. angle adjustable complete SB34 short right		1
20	1535877	Footrest 2-pc. angle adjustable complete SB36 short right		1
21	1535878	Footrest 2-pc. angle adjustable complete SB38 short right		1
22	1535879	Footrest 2-pc. angle adjustable complete SB40 short right		1
23	1535880	Footrest 2-pc. angle adjustable complete SB42 short right		1
24	1535881	Footrest 2-pc. angle adjustable complete SB44 short right		1
25	1535882	Footrest 2-pc. angle adjustable complete SB46 short right		1
26	1535883	Footrest 2-pc. angle adjustable complete SB48 short right		1
27	1535884	Footrest 2-pc. angle adjustable complete SB50 short right		1
28	1535885	Footrest 2-pc. angle adjustable complete SB34 long right		1
29	1535886	Footrest 2-pc. angle adjustable complete SB36 long right		1
30	1535887	Footrest 2-pc. angle adjustable complete SB38 long right		1
31	1535888	Footrest 2-pc. angle adjustable complete SB40 long right		1

Footrest 2-piece, angle adjustable


Pos.	Part number	Description	Valid from → until	Qty
32	1535889	Footrest 2-pc. angle adjustable complete SB42 long right		1
33	1535890	Footrest 2-pc. angle adjustable complete SB44 long right		1
34	1535891	Footrest 2-pc. angle adjustable complete SB46 long right		1
35	1535892	Footrest 2-pc. angle adjustable complete SB48 long right		1
36	1535893	Footrest 2-pc. angle adjustable complete SB50 long right		1
37	1535894	Footrest 2-pc. angle adjustable complete SB34 short left		1
38	1535895	Footrest 2-pc. angle adjustable complete SB36 short left		1
39	1535896	Footrest 2-pc. angle adjustable complete SB38 short left		1
40	1535897	Footrest 2-pc. angle adjustable complete SB40 short left		1
41	1535898	Footrest 2-pc. angle adjustable complete SB42 short left		1
42	1535899	Footrest 2-pc. angle adjustable complete SB44 short left		1
43	1535900	Footrest 2-pc. angle adjustable complete SB46 short left		1
44	1535901	Footrest 2-pc. angle adjustable complete SB48 short left		1
45	1535902	Footrest 2-pc. angle adjustable complete SB50 short left		1
46	1535903	Footrest 2-pc. angle adjustable complete SB34 long left		1
47	1535904	Footrest 2-pc. angle adjustable complete SB36 long left		1
48	1535905	Footrest 2-pc. angle adjustable complete SB38 long left		1
49	1535906	Footrest 2-pc. angle adjustable complete SB40 long left		1
50	1535907	Footrest 2-pc. angle adjustable complete SB42 long left		1
51	1535908	Footrest 2-pc. angle adjustable complete SB44 long left		1
52	1535909	Footrest 2-pc. angle adjustable complete SB46 long left		1
53	1535910	Footrest 2-pc. angle adjustable complete SB48 long left		1
54	1535911	Footrest 2-pc. angle adjustable complete SB50 long left		1
55	1532398	Fixation kit footplate telescope		1
56	1437834	Heel loop kit short		1
57	1536082	Calf strap 2-pcs complete		1
58	1438143	instep strap UL/C04		1

Footrest carbon 2-pieces

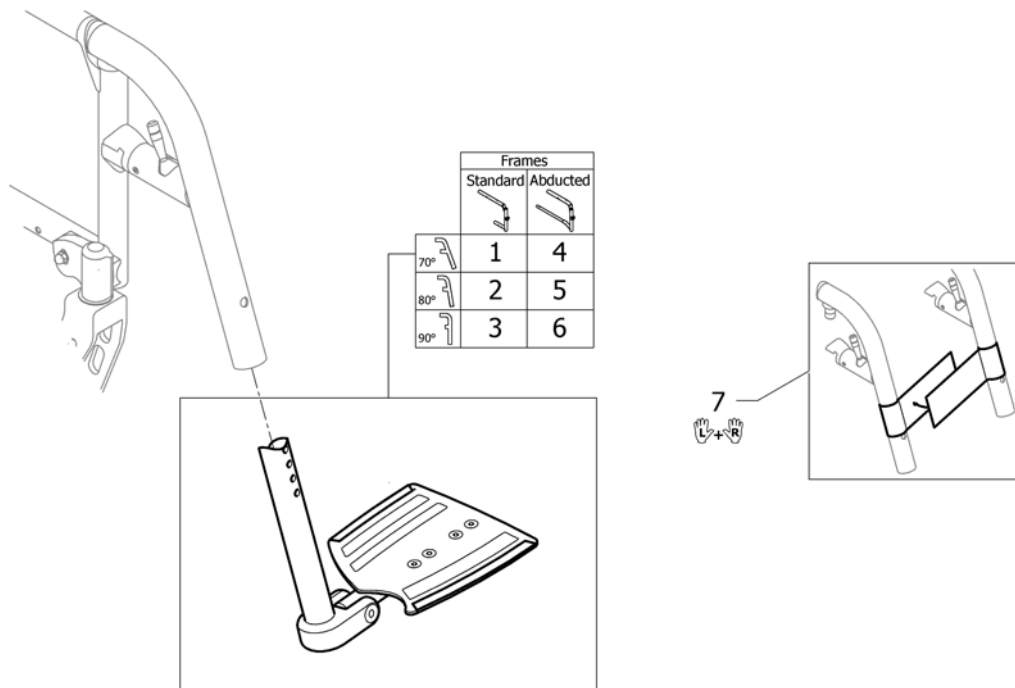

Pos.	Part number	Description	Valid from → until	Qty
1	-	Footrest carbon 2-pieces for swing away 70° on front frames standard		1
2	-	Footrest carbon 2-pieces for swing away 80° on front frames standard		1
3	-	Footrest carbon 2-pieces for swing away 90° on front frames standard		1
4	-	Footrest carbon 2-pieces for swing away 70° on front frames abducted		1
5	-	Footrest carbon 2-pieces for swing away 80° on front frames abducted		1
6	-	Footrest carbon 2-pieces for swing away 90° on front frames abducted		1
7	1536082	Calf strap 2-pcs complete		1

Footrest carbon 2-pieces for swing away 70° on front frames standard

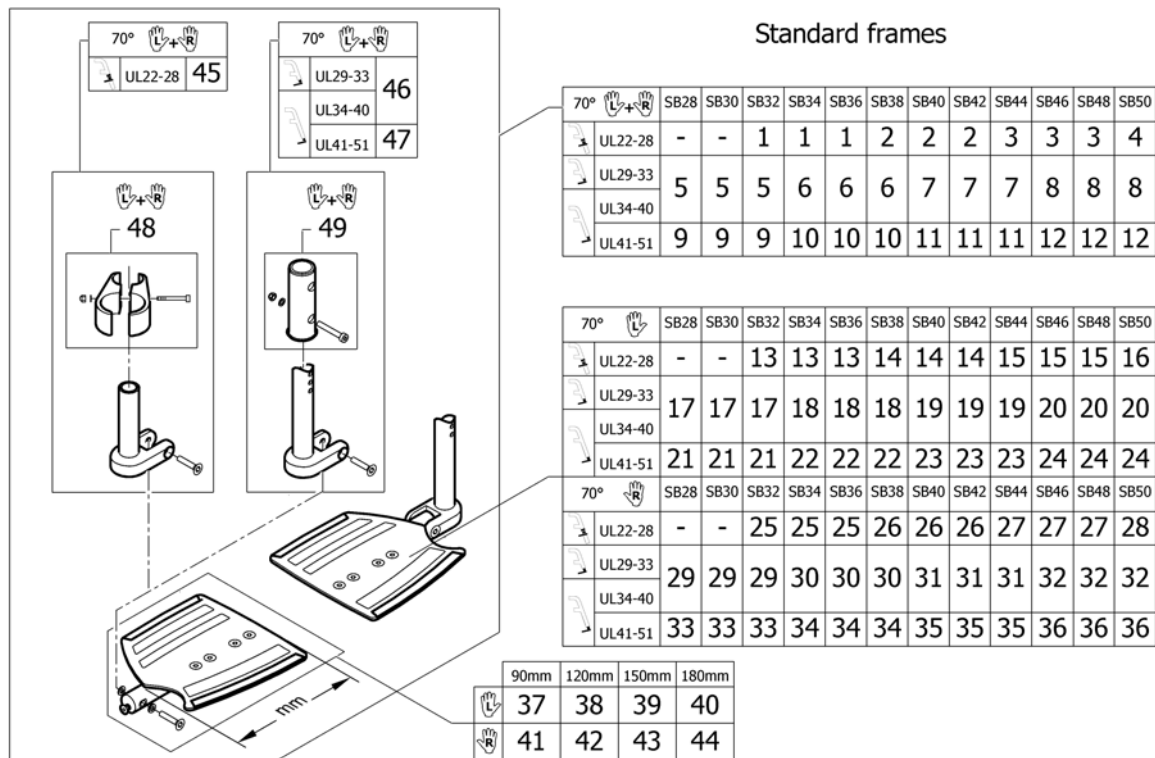


Pos.	Part number	Description	Valid from → until	Qty
1	1545434	Footrest carbon 2-pc. 90mm, high mounted, pair, complete		1
2	1545435	Footrest carbon 2-pc. 120mm, high mounted, pair, complete		1
3	1545436	Footrest carbon 2-pc. 150mm, high mounted, pair, complete		1
4	1545437	Footrest carbon 2-pc. 180mm, high mounted, pair, complete		1
5	1545438	Footrest carbon 2-pc. 90mm, short, pair, complete		1
6	1545439	Footrest carbon 2-pc. 120mm, short, pair, complete		1
7	1545440	Footrest carbon 2-pc. 150mm, short, pair, complete		1
8	1545441	Footrest carbon 2-pc. 180mm, short, pair, complete		1
9	1545442	Footrest carbon 2-pc. 90mm, long, pair, complete		1
10	1545443	Footrest carbon 2-pc. 120mm, long, pair, complete		1
11	1545444	Footrest carbon 2-pc. 150mm, long, pair, complete		1
12	1545445	Footrest carbon 2-pc. 180mm, long, pair, complete		1
13	1545772	Footrest carbon 2-pc. 90mm, high mounted, left, complete		1
14	1545773	Footrest carbon 2-pc. 120mm, high mounted, left, complete		1
15	1545774	Footrest carbon 2-pc. 150mm, high mounted, left, complete		1
16	1545775	Footrest carbon 2-pc. 180mm, high mounted, left, complete		1
17	1545776	Footrest carbon 2-pc. 90mm, short, left, complete		1
18	1545777	Footrest carbon 2-pc. 120mm, short, left, complete		1
19	1545778	Footrest carbon 2-pc. 150mm, short, left, complete		1
20	1545779	Footrest carbon 2-pc. 180mm, short, left, complete		1
21	1545780	Footrest carbon 2-pc. 90mm, long, left, complete		1
22	1545781	Footrest carbon 2-pc. 120mm, long, left, complete		1
23	1545782	Footrest carbon 2-pc. 150mm, long, left, complete		1
24	1545783	Footrest carbon 2-pc. 180mm, long, left, complete		1
25	1545760	Footrest carbon 2-pc. 90mm, high mounted, right, complete		1
26	1545761	Footrest carbon 2-pc. 120mm, high mounted, right, complete		1
27	1545762	Footrest carbon 2-pc. 150mm, high mounted, right, complete		1
28	1545763	Footrest carbon 2-pc. 180mm, high mounted, right, complete		1
29	1545764	Footrest carbon 2-pc. 90mm, short, right, complete		1
30	1545765	Footrest carbon 2-pc. 120mm, short, right, complete		1
31	1545766	Footrest carbon 2-pc. 150mm, short, right, complete		1

Footrest carbon 2-pieces for swing away 70° on front frames standard

Pos.	Part number	Description	Valid from → until	Qty
32	1545767	Footrest carbon 2-pc. 180mm, short, right, complete		1
33	1545768	Footrest carbon 2-pc. 90mm, long, right, complete		1
34	1545769	Footrest carbon 2-pc. 120mm, long, right, complete		1
35	1545770	Footrest carbon 2-pc. 150mm, long, right, complete		1
36	1545771	Footrest carbon 2-pc. 180mm, long, right, complete		1
37	1545450	Footrest carbon 2-pc. 90mm, left		1
38	1545451	Footrest carbon 2-pc. 120mm, left		1
39	1545452	Footrest carbon 2-pc. 150mm, left		1
40	1545517	Footrest carbon 2-pc. 180mm, left		1
41	1545446	Footrest carbon 2-pc. 90mm, right		1
42	1545447	Footrest carbon 2-pc. 120mm, right		1
43	1545448	Footrest carbon 2-pc. 150mm, right		1
44	1545449	Footrest carbon 2-pc. 180mm, right		1
45	1545784	Telescopic tube kit for footrest carbon 2-pc. high mounted, pair		1
46	1545785	Telescopic tube kit for footrest carbon 2-pc. short, pair		1
47	1545786	Telescopic tube kit for footrest carbon 2-pc. long, pair		1
48	1544613	Clamping kit for footrest high mounted, pair		1
49	1532398	Fixation kit footplate telescope		1

Footrest carbon 2-pieces for swing away 80° on front frames standard

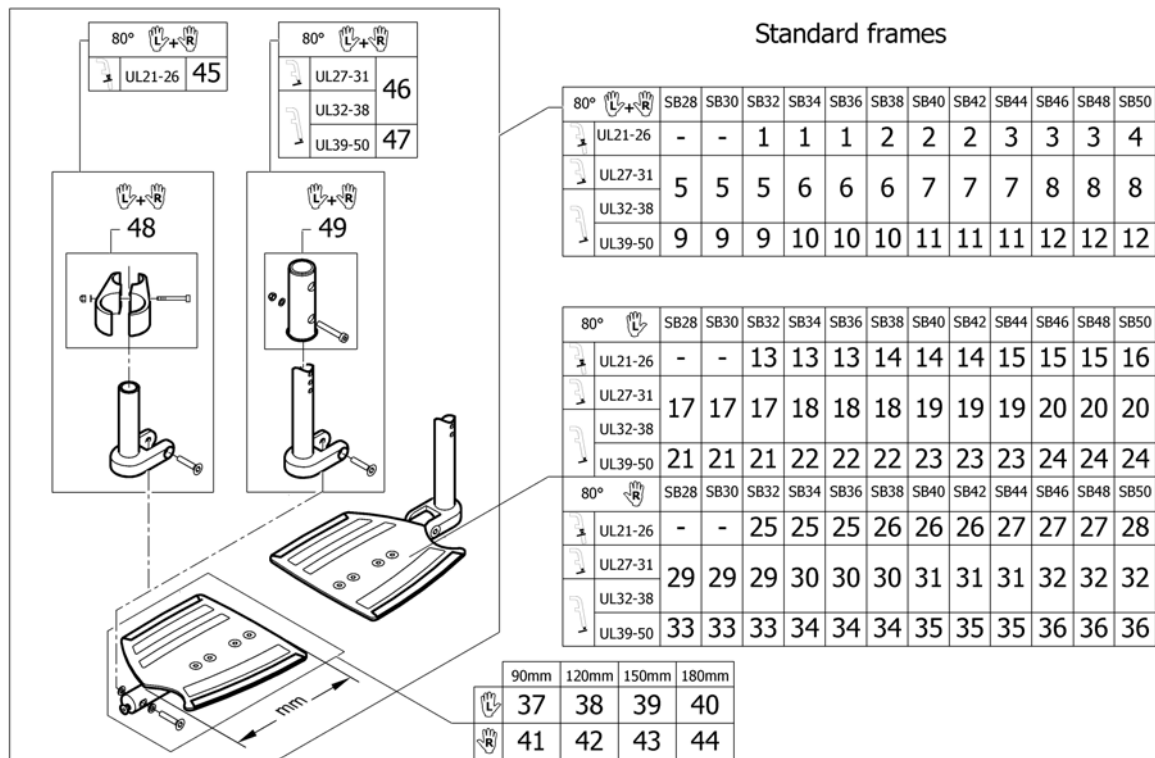

Pos.	Part number	Description	Valid from → until	Qty
1	1545434	Footrest carbon 2-pc. 90mm, high mounted, pair, complete		1
2	1545435	Footrest carbon 2-pc. 120mm, high mounted, pair, complete		1
3	1545436	Footrest carbon 2-pc. 150mm, high mounted, pair, complete		1
4	1545437	Footrest carbon 2-pc. 180mm, high mounted, pair, complete		1
5	1545438	Footrest carbon 2-pc. 90mm, short, pair, complete		1
6	1545439	Footrest carbon 2-pc. 120mm, short, pair, complete		1
7	1545440	Footrest carbon 2-pc. 150mm, short, pair, complete		1
8	1545441	Footrest carbon 2-pc. 180mm, short, pair, complete		1
9	1545442	Footrest carbon 2-pc. 90mm, long, pair, complete		1
10	1545443	Footrest carbon 2-pc. 120mm, long, pair, complete		1
11	1545444	Footrest carbon 2-pc. 150mm, long, pair, complete		1
12	1545445	Footrest carbon 2-pc. 180mm, long, pair, complete		1
13	1545772	Footrest carbon 2-pc. 90mm, high mounted, left, complete		1
14	1545773	Footrest carbon 2-pc. 120mm, high mounted, left, complete		1
15	1545774	Footrest carbon 2-pc. 150mm, high mounted, left, complete		1
16	1545775	Footrest carbon 2-pc. 180mm, high mounted, left, complete		1
17	1545776	Footrest carbon 2-pc. 90mm, short, left, complete		1
18	1545777	Footrest carbon 2-pc. 120mm, short, left, complete		1
19	1545778	Footrest carbon 2-pc. 150mm, short, left, complete		1
20	1545779	Footrest carbon 2-pc. 180mm, short, left, complete		1
21	1545780	Footrest carbon 2-pc. 90mm, long, left, complete		1
22	1545781	Footrest carbon 2-pc. 120mm, long, left, complete		1
23	1545782	Footrest carbon 2-pc. 150mm, long, left, complete		1
24	1545783	Footrest carbon 2-pc. 180mm, long, left, complete		1
25	1545760	Footrest carbon 2-pc. 90mm, high mounted, right, complete		1
26	1545761	Footrest carbon 2-pc. 120mm, high mounted, right, complete		1
27	1545762	Footrest carbon 2-pc. 150mm, high mounted, right, complete		1
28	1545763	Footrest carbon 2-pc. 180mm, high mounted, right, complete		1
29	1545764	Footrest carbon 2-pc. 90mm, short, right, complete		1
30	1545765	Footrest carbon 2-pc. 120mm, short, right, complete		1
31	1545766	Footrest carbon 2-pc. 150mm, short, right, complete		1

Footrest carbon 2-pieces for swing away 80° on front frames standard

Pos.	Part number	Description	Valid from → until	Qty
32	1545767	Footrest carbon 2-pc. 180mm, short, right, complete		1
33	1545768	Footrest carbon 2-pc. 90mm, long, right, complete		1
34	1545769	Footrest carbon 2-pc. 120mm, long, right, complete		1
35	1545770	Footrest carbon 2-pc. 150mm, long, right, complete		1
36	1545771	Footrest carbon 2-pc. 180mm, long, right, complete		1
37	1545450	Footrest carbon 2-pc. 90mm, left		1
38	1545451	Footrest carbon 2-pc. 120mm, left		1
39	1545452	Footrest carbon 2-pc. 150mm, left		1
40	1545517	Footrest carbon 2-pc. 180mm, left		1
41	1545446	Footrest carbon 2-pc. 90mm, right		1
42	1545447	Footrest carbon 2-pc. 120mm, right		1
43	1545448	Footrest carbon 2-pc. 150mm, right		1
44	1545449	Footrest carbon 2-pc. 180mm, right		1
45	1545784	Telescopic tube kit for footrest carbon 2-pc. high mounted, pair		1
46	1545785	Telescopic tube kit for footrest carbon 2-pc. short, pair		1
47	1545786	Telescopic tube kit for footrest carbon 2-pc. long, pair		1
48	1544613	Clamping kit for footrest high mounted, pair		1
49	1532398	Fixation kit footplate telescope		1

Footrest carbon 2-pieces for swing away 90° on front frames standard

Standard frames

90°	SB28	SB30	SB32	SB34	SB36	SB38	SB40	SB42	SB44	SB46	SB48	SB50
UL21-25	-	-	1	1	1	2	2	2	3	3	3	4
UL26-30	5	5	5	6	6	6	7	7	7	8	8	8
UL31-37	9	9	9	10	10	10	11	11	11	12	12	12
UL38-50	9	9	9	10	10	10	11	11	11	12	12	12

90°	SB28	SB30	SB32	SB34	SB36	SB38	SB40	SB42	SB44	SB46	SB48	SB50
UL21-25	-	-	13	13	13	14	14	14	15	15	15	16
UL26-30	17	17	17	18	18	18	19	19	19	20	20	20
UL31-37	21	21	21	22	22	22	23	23	23	24	24	24
UL38-50	21	21	21	22	22	22	23	23	23	24	24	24

90°	SB28	SB30	SB32	SB34	SB36	SB38	SB40	SB42	SB44	SB46	SB48	SB50
UL21-25	-	-	25	25	25	26	26	26	27	27	27	28
UL26-30	29	29	29	30	30	30	31	31	31	32	32	32
UL31-37	33	33	33	34	34	34	35	35	35	36	36	36
UL38-50	33	33	33	34	34	34	35	35	35	36	36	36


	90mm	120mm	150mm	180mm
L	37	38	39	40
R	41	42	43	44

Pos.	Part number	Description	Valid from → until	Qty
1	1545434	Footrest carbon 2-pc. 90mm, high mounted, pair, complete		1
2	1545435	Footrest carbon 2-pc. 120mm, high mounted, pair, complete		1
3	1545436	Footrest carbon 2-pc. 150mm, high mounted, pair, complete		1
4	1545437	Footrest carbon 2-pc. 180mm, high mounted, pair, complete		1
5	1545438	Footrest carbon 2-pc. 90mm, short, pair, complete		1
6	1545439	Footrest carbon 2-pc. 120mm, short, pair, complete		1
7	1545440	Footrest carbon 2-pc. 150mm, short, pair, complete		1
8	1545441	Footrest carbon 2-pc. 180mm, short, pair, complete		1
9	1545442	Footrest carbon 2-pc. 90mm, long, pair, complete		1
10	1545443	Footrest carbon 2-pc. 120mm, long, pair, complete		1
11	1545444	Footrest carbon 2-pc. 150mm, long, pair, complete		1
12	1545445	Footrest carbon 2-pc. 180mm, long, pair, complete		1
13	1545772	Footrest carbon 2-pc. 90mm, high mounted, left, complete		1
14	1545773	Footrest carbon 2-pc. 120mm, high mounted, left, complete		1
15	1545774	Footrest carbon 2-pc. 150mm, high mounted, left, complete		1
16	1545775	Footrest carbon 2-pc. 180mm, high mounted, left, complete		1
17	1545776	Footrest carbon 2-pc. 90mm, short, left, complete		1
18	1545777	Footrest carbon 2-pc. 120mm, short, left, complete		1
19	1545778	Footrest carbon 2-pc. 150mm, short, left, complete		1
20	1545779	Footrest carbon 2-pc. 180mm, short, left, complete		1
21	1545780	Footrest carbon 2-pc. 90mm, long, left, complete		1
22	1545781	Footrest carbon 2-pc. 120mm, long, left, complete		1
23	1545782	Footrest carbon 2-pc. 150mm, long, left, complete		1
24	1545783	Footrest carbon 2-pc. 180mm, long, left, complete		1
25	1545760	Footrest carbon 2-pc. 90mm, high mounted, right, complete		1
26	1545761	Footrest carbon 2-pc. 120mm, high mounted, right, complete		1
27	1545762	Footrest carbon 2-pc. 150mm, high mounted, right, complete		1
28	1545763	Footrest carbon 2-pc. 180mm, high mounted, right, complete		1
29	1545764	Footrest carbon 2-pc. 90mm, short, right, complete		1
30	1545765	Footrest carbon 2-pc. 120mm, short, right, complete		1
31	1545766	Footrest carbon 2-pc. 150mm, short, right, complete		1

Footrest carbon 2-pieces for swing away 90° on front frames standard

Pos.	Part number	Description	Valid from → until	Qty
32	1545767	Footrest carbon 2-pc. 180mm, short, right, complete		1
33	1545768	Footrest carbon 2-pc. 90mm, long, right, complete		1
34	1545769	Footrest carbon 2-pc. 120mm, long, right, complete		1
35	1545770	Footrest carbon 2-pc. 150mm, long, right, complete		1
36	1545771	Footrest carbon 2-pc. 180mm, long, right, complete		1
37	1545450	Footrest carbon 2-pc. 90mm, left		1
38	1545451	Footrest carbon 2-pc. 120mm, left		1
39	1545452	Footrest carbon 2-pc. 150mm, left		1
40	1545517	Footrest carbon 2-pc. 180mm, left		1
41	1545446	Footrest carbon 2-pc. 90mm, right		1
42	1545447	Footrest carbon 2-pc. 120mm, right		1
43	1545448	Footrest carbon 2-pc. 150mm, right		1
44	1545449	Footrest carbon 2-pc. 180mm, right		1
45	1545784	Telescopic tube kit for footrest carbon 2-pc. high mounted, pair		1
46	1545785	Telescopic tube kit for footrest carbon 2-pc. short, pair		1
47	1545786	Telescopic tube kit for footrest carbon 2-pc. long, pair		1
48	1544613	Clamping kit for footrest high mounted, pair		1
49	1532398	Fixation kit footplate telescope		1

Footrest carbon 2-pieces for swing away 70° on front frames abducted

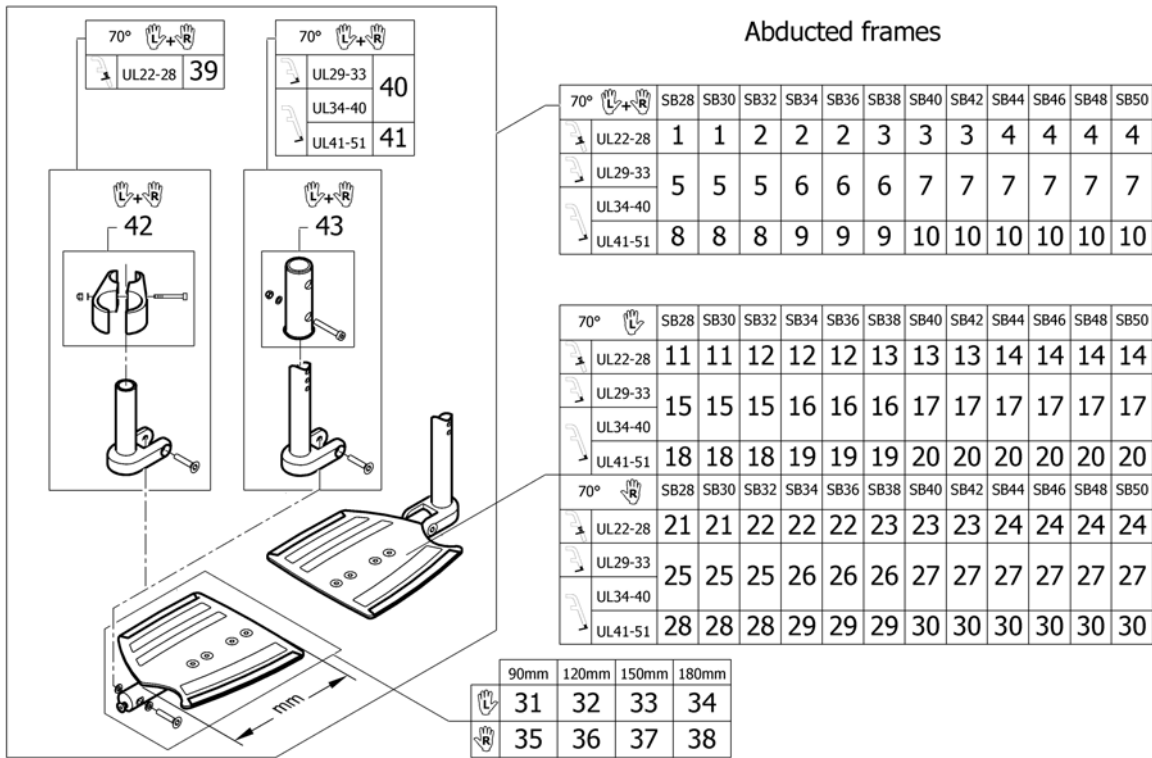


Pos.	Part number	Description	Valid from → until	Qty
1	1545434	Footrest carbon 2-pc. 90mm, high mounted, pair, complete		1
2	1545435	Footrest carbon 2-pc. 120mm, high mounted, pair, complete		1
3	1545436	Footrest carbon 2-pc. 150mm, high mounted, pair, complete		1
4	1545437	Footrest carbon 2-pc. 180mm, high mounted, pair, complete		1
5	1545439	Footrest carbon 2-pc. 120mm, short, pair, complete		1
6	1545440	Footrest carbon 2-pc. 150mm, short, pair, complete		1
7	1545441	Footrest carbon 2-pc. 180mm, short, pair, complete		1
8	1545443	Footrest carbon 2-pc. 120mm, long, pair, complete		1
9	1545444	Footrest carbon 2-pc. 150mm, long, pair, complete		1
10	1545445	Footrest carbon 2-pc. 180mm, long, pair, complete		1
11	1545772	Footrest carbon 2-pc. 90mm, high mounted, left, complete		1
12	1545773	Footrest carbon 2-pc. 120mm, high mounted, left, complete		1
13	1545774	Footrest carbon 2-pc. 150mm, high mounted, left, complete		1
14	1545775	Footrest carbon 2-pc. 180mm, high mounted, left, complete		1
15	1545777	Footrest carbon 2-pc. 120mm, short, left, complete		1
16	1545778	Footrest carbon 2-pc. 150mm, short, left, complete		1
17	1545779	Footrest carbon 2-pc. 180mm, short, left, complete		1
18	1545781	Footrest carbon 2-pc. 120mm, long, left, complete		1
19	1545782	Footrest carbon 2-pc. 150mm, long, left, complete		1
20	1545783	Footrest carbon 2-pc. 180mm, long, left, complete		1
21	1545760	Footrest carbon 2-pc. 90mm, high mounted, right, complete		1
22	1545761	Footrest carbon 2-pc. 120mm, high mounted, right, complete		1
23	1545762	Footrest carbon 2-pc. 150mm, high mounted, right, complete		1
24	1545763	Footrest carbon 2-pc. 180mm, high mounted, right, complete		1
25	1545765	Footrest carbon 2-pc. 120mm, short, right, complete		1
26	1545766	Footrest carbon 2-pc. 150mm, short, right, complete		1
27	1545767	Footrest carbon 2-pc. 180mm, short, right, complete		1
28	1545769	Footrest carbon 2-pc. 120mm, long, right, complete		1
29	1545770	Footrest carbon 2-pc. 150mm, long, right, complete		1
30	1545771	Footrest carbon 2-pc. 180mm, long, right, complete		1
31	1545450	Footrest carbon 2-pc. 90mm, left		1

Footrest carbon 2-pieces for swing away 70° on front frames abducted

Pos.	Part number	Description	Valid from → until	Qty
32	1545451	Footrest carbon 2-pc. 120mm, left		1
33	1545452	Footrest carbon 2-pc. 150mm, left		1
34	1545517	Footrest carbon 2-pc. 180mm, left		1
35	1545446	Footrest carbon 2-pc. 90mm, right		1
36	1545447	Footrest carbon 2-pc. 120mm, right		1
37	1545448	Footrest carbon 2-pc. 150mm, right		1
38	1545449	Footrest carbon 2-pc. 180mm, right		1
39	1545784	Telescopic tube kit for footrest carbon 2-pc. high mounted, pair		1
40	1545785	Telescopic tube kit for footrest carbon 2-pc. short, pair		1
41	1545786	Telescopic tube kit for footrest carbon 2-pc. long, pair		1
42	1544613	Clamping kit for footrest high mounted, pair		1
43	1532398	Fixation kit footplate telescope		1

Footrest carbon 2-pieces for swing away 80° on front frames abducted

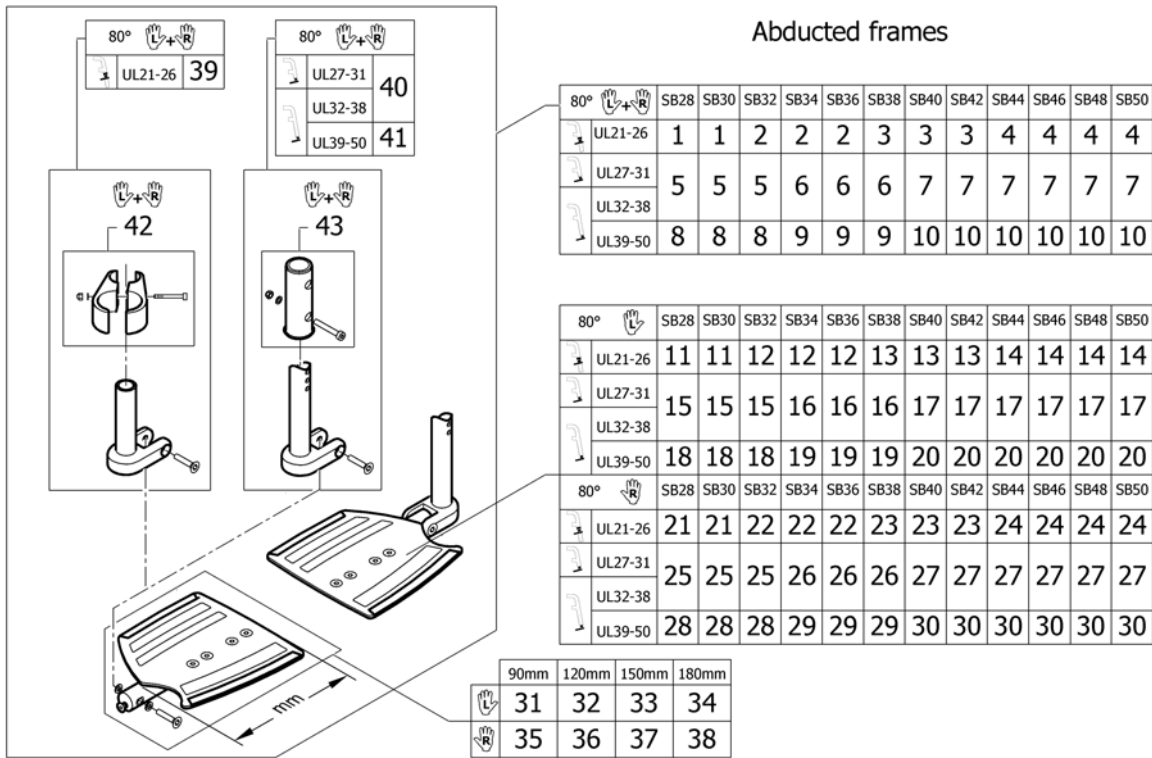


Pos.	Part number	Description	Valid from → until	Qty
1	1545434	Footrest carbon 2-pc. 90mm, high mounted, pair, complete		1
2	1545435	Footrest carbon 2-pc. 120mm, high mounted, pair, complete		1
3	1545436	Footrest carbon 2-pc. 150mm, high mounted, pair, complete		1
4	1545437	Footrest carbon 2-pc. 180mm, high mounted, pair, complete		1
5	1545439	Footrest carbon 2-pc. 120mm, short, pair, complete		1
6	1545440	Footrest carbon 2-pc. 150mm, short, pair, complete		1
7	1545441	Footrest carbon 2-pc. 180mm, short, pair, complete		1
8	1545443	Footrest carbon 2-pc. 120mm, long, pair, complete		1
9	1545444	Footrest carbon 2-pc. 150mm, long, pair, complete		1
10	1545445	Footrest carbon 2-pc. 180mm, long, pair, complete		1
11	1545772	Footrest carbon 2-pc. 90mm, high mounted, left, complete		1
12	1545773	Footrest carbon 2-pc. 120mm, high mounted, left, complete		1
13	1545774	Footrest carbon 2-pc. 150mm, high mounted, left, complete		1
14	1545775	Footrest carbon 2-pc. 180mm, high mounted, left, complete		1
15	1545777	Footrest carbon 2-pc. 120mm, short, left, complete		1
16	1545778	Footrest carbon 2-pc. 150mm, short, left, complete		1
17	1545779	Footrest carbon 2-pc. 180mm, short, left, complete		1
18	1545781	Footrest carbon 2-pc. 120mm, long, left, complete		1
19	1545782	Footrest carbon 2-pc. 150mm, long, left, complete		1
20	1545783	Footrest carbon 2-pc. 180mm, long, left, complete		1
21	1545760	Footrest carbon 2-pc. 90mm, high mounted, right, complete		1
22	1545761	Footrest carbon 2-pc. 120mm, high mounted, right, complete		1
23	1545762	Footrest carbon 2-pc. 150mm, high mounted, right, complete		1
24	1545763	Footrest carbon 2-pc. 180mm, high mounted, right, complete		1
25	1545765	Footrest carbon 2-pc. 120mm, short, right, complete		1
26	1545766	Footrest carbon 2-pc. 150mm, short, right, complete		1
27	1545767	Footrest carbon 2-pc. 180mm, short, right, complete		1
28	1545769	Footrest carbon 2-pc. 120mm, long, right, complete		1
29	1545770	Footrest carbon 2-pc. 150mm, long, right, complete		1
30	1545771	Footrest carbon 2-pc. 180mm, long, right, complete		1
31	1545450	Footrest carbon 2-pc. 90mm, left		1

Footrest carbon 2-pieces for swing away 80° on front frames abducted

Pos.	Part number	Description	Valid from → until	Qty
32	1545451	Footrest carbon 2-pc. 120mm, left		1
33	1545452	Footrest carbon 2-pc. 150mm, left		1
34	1545517	Footrest carbon 2-pc. 180mm, left		1
35	1545446	Footrest carbon 2-pc. 90mm, right		1
36	1545447	Footrest carbon 2-pc. 120mm, right		1
37	1545448	Footrest carbon 2-pc. 150mm, right		1
38	1545449	Footrest carbon 2-pc. 180mm, right		1
39	1545784	Telescopic tube kit for footrest carbon 2-pc. high mounted, pair		1
40	1545785	Telescopic tube kit for footrest carbon 2-pc. short, pair		1
41	1545786	Telescopic tube kit for footrest carbon 2-pc. long, pair		1
42	1544613	Clamping kit for footrest high mounted, pair		1
43	1532398	Fixation kit footplate telescope		1


Footrest carbon 2-pieces for swing away 90° on front frames abducted

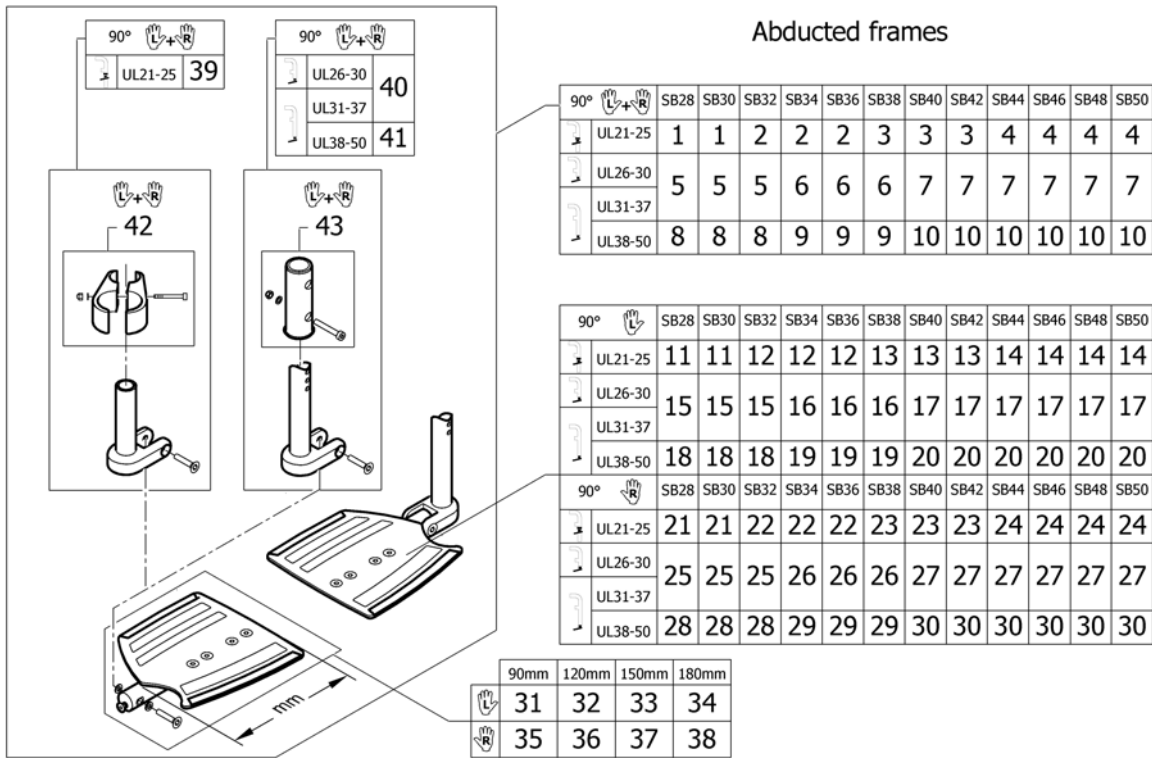

Pos.	Part number	Description	Valid from → until	Qty
1	1545434	Footrest carbon 2-pc. 90mm, high mounted, pair, complete		1
2	1545435	Footrest carbon 2-pc. 120mm, high mounted, pair, complete		1
3	1545436	Footrest carbon 2-pc. 150mm, high mounted, pair, complete		1
4	1545437	Footrest carbon 2-pc. 180mm, high mounted, pair, complete		1
5	1545439	Footrest carbon 2-pc. 120mm, short, pair, complete		1
6	1545440	Footrest carbon 2-pc. 150mm, short, pair, complete		1
7	1545441	Footrest carbon 2-pc. 180mm, short, pair, complete		1
8	1545443	Footrest carbon 2-pc. 120mm, long, pair, complete		1
9	1545444	Footrest carbon 2-pc. 150mm, long, pair, complete		1
10	1545445	Footrest carbon 2-pc. 180mm, long, pair, complete		1
11	1545772	Footrest carbon 2-pc. 90mm, high mounted, left, complete		1
12	1545773	Footrest carbon 2-pc. 120mm, high mounted, left, complete		1
13	1545774	Footrest carbon 2-pc. 150mm, high mounted, left, complete		1
14	1545775	Footrest carbon 2-pc. 180mm, high mounted, left, complete		1
15	1545777	Footrest carbon 2-pc. 120mm, short, left, complete		1
16	1545778	Footrest carbon 2-pc. 150mm, short, left, complete		1
17	1545779	Footrest carbon 2-pc. 180mm, short, left, complete		1
18	1545781	Footrest carbon 2-pc. 120mm, long, left, complete		1
19	1545782	Footrest carbon 2-pc. 150mm, long, left, complete		1
20	1545783	Footrest carbon 2-pc. 180mm, long, left, complete		1
21	1545760	Footrest carbon 2-pc. 90mm, high mounted, right, complete		1
22	1545761	Footrest carbon 2-pc. 120mm, high mounted, right, complete		1
23	1545762	Footrest carbon 2-pc. 150mm, high mounted, right, complete		1
24	1545763	Footrest carbon 2-pc. 180mm, high mounted, right, complete		1
25	1545765	Footrest carbon 2-pc. 120mm, short, right, complete		1
26	1545766	Footrest carbon 2-pc. 150mm, short, right, complete		1
27	1545767	Footrest carbon 2-pc. 180mm, short, right, complete		1
28	1545769	Footrest carbon 2-pc. 120mm, long, right, complete		1
29	1545770	Footrest carbon 2-pc. 150mm, long, right, complete		1
30	1545771	Footrest carbon 2-pc. 180mm, long, right, complete		1
31	1545450	Footrest carbon 2-pc. 90mm, left		1


Footrest carbon 2-pieces for swing away 90° on front frames abducted

Pos.	Part number	Description	Valid from → until	Qty
32	1545451	Footrest carbon 2-pc. 120mm, left		1
33	1545452	Footrest carbon 2-pc. 150mm, left		1
34	1545517	Footrest carbon 2-pc. 180mm, left		1
35	1545446	Footrest carbon 2-pc. 90mm, right		1
36	1545447	Footrest carbon 2-pc. 120mm, right		1
37	1545448	Footrest carbon 2-pc. 150mm, right		1
38	1545449	Footrest carbon 2-pc. 180mm, right		1
39	1545784	Telescopic tube kit for footrest carbon 2-pc. high mounted, pair		1
40	1545785	Telescopic tube kit for footrest carbon 2-pc. short, pair		1
41	1545786	Telescopic tube kit for footrest carbon 2-pc. long, pair		1
42	1544613	Clamping kit for footrest high mounted, pair		1
43	1532398	Fixation kit footplate telescope		1

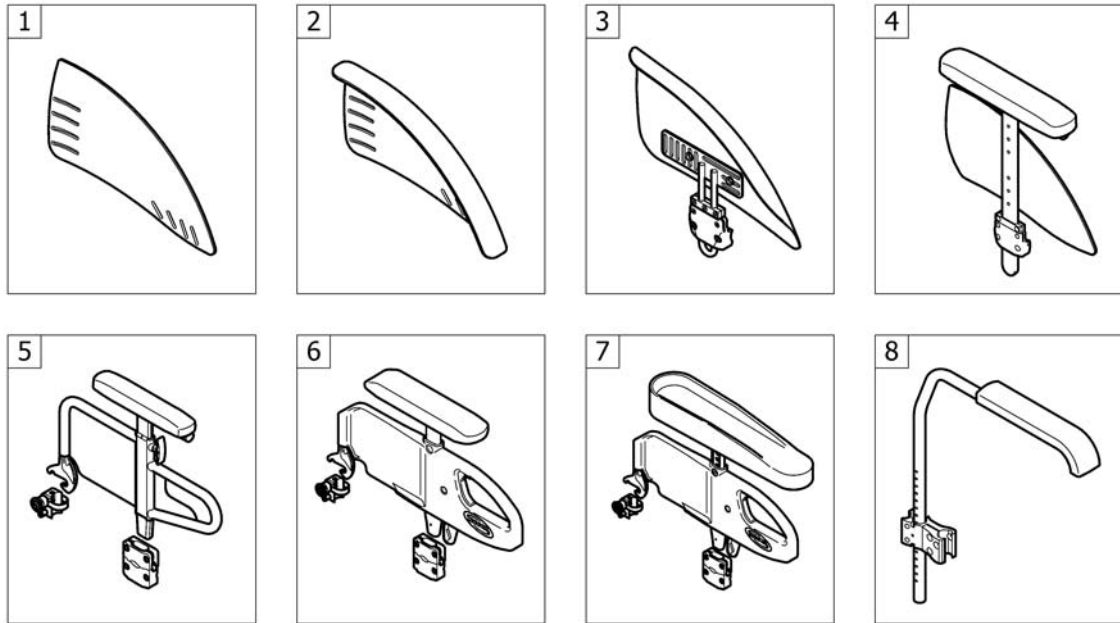


Pos.	Part number	Description	Valid from → until	Qty
1	-	Clothes-guards		1
2	-	Mudguards fixe		1
3	-	Mudguards removable		1
4	-	Siderest height adjustable		1
5	-	Siderest in colours, flip-back and height adjustable		1
6	-	Siderest with desk function		1
7	-	Armrest Hemi		1
8	-	Tubular armrest		1

Clothes-guards

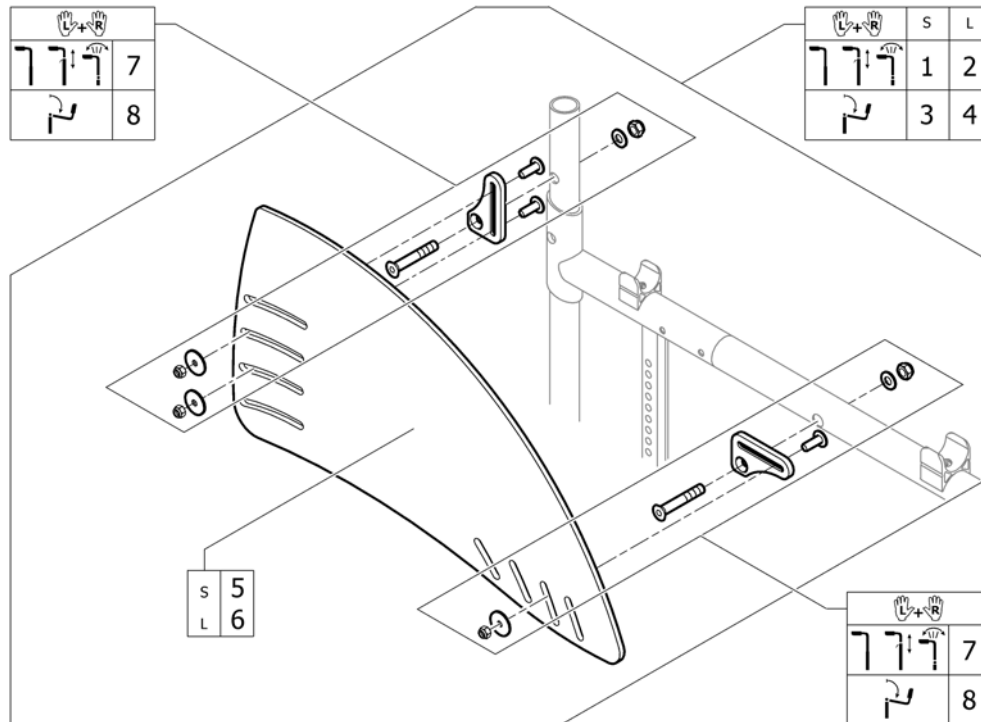


Pos.	Part number	Description	Valid from → until	Qty
1	1540001	Clothes-guards S with fixation kit, pair		1
2	1540002	Clothes-guards L with fixation kit, pair		1
3	1540914	Clothes-guards S with fixation kit to backrest foldable, pair		1
4	1540915	Clothes-guards L with fixation kit to backrest foldable, pair		1
5	1540003	Clothes-guard S		2
6	1540004	Clothes-guard L		2
7	1540005	Fixation kit for clothes-guard and mudguard		1
8	1540916	Fixation kit for clothes-guard and mudguard to backrest foldable		1

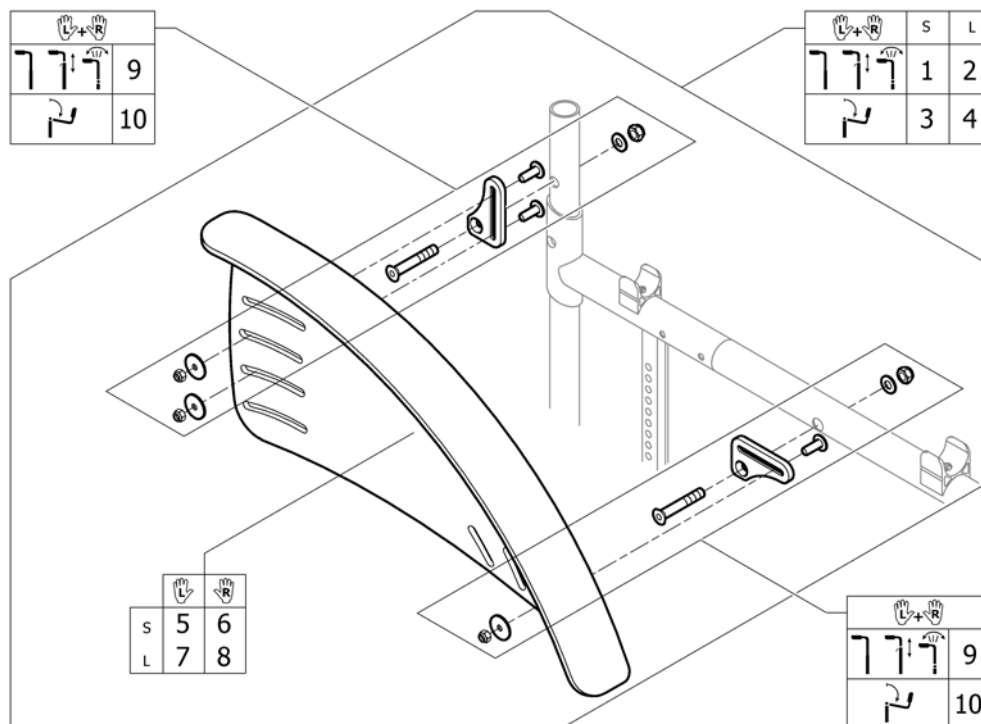


Pos.	Part number	Description	Valid from → until	Qty
1	1540006	Mudguard Carbon S complete, pair		1
2	1540007	Mudguard Carbon L complete, pair		1
3	1540917	Mudguard Carbon S complete to backrest foldable, pair		1
4	1540918	Mudguard Carbon L complete to backrest foldable, pair		1
5	1529781	Mudguard Carbon S short left		1
6	1529782	Mudguard Carbon S short right		1
7	1529783	Mudguard Carbon L long left		1
8	1529784	Mudguard Carbon L long right		1
9	1540005	Fixation kit for clothes-guard and mudguard		1
10	1540916	Fixation kit for clothes-guard and mudguard to backrest foldable		1

Mudguards removable

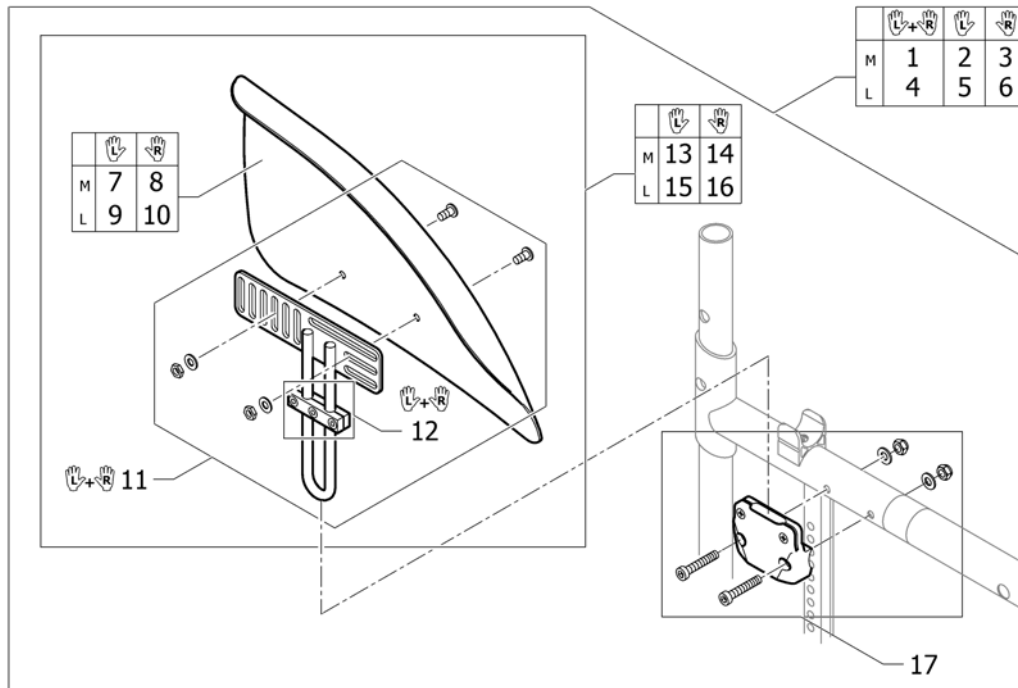


Pos.	Part number	Description	Valid from → until	Qty
1	1540037	Mudguard Carbon M long complete, pair		1
2	1540033	Mudguard Carbon left M long complete		1
3	1540034	Mudguard Carbon right M long complete		1
4	1540038	Mudguard Carbon L long complete, pair		1
5	1540035	Mudguard Carbon left L long complete		1
6	1540036	Mudguard Carbon right L long complete		1
7	1510275	Mudguard Carbon left M		1
8	1510276	Mudguard Carbon right M		1
9	1510277	Mudguard Carbon left L		1
10	1510278	Mudguard Carbon right L		1
11	1488459	Mudguard holder long, pair		1
12	1437465	Siderests height clamp kit, pair		1
13	1529304	Mudguard Carbon left M long		1
14	1529306	Mudguard Carbon right M long		1
15	1529305	Mudguard Carbon left L long		1
16	1529307	Mudguard Carbon right L long		1
17	1540010	Sidepart Adaption complete		1

Siderest height adjustable

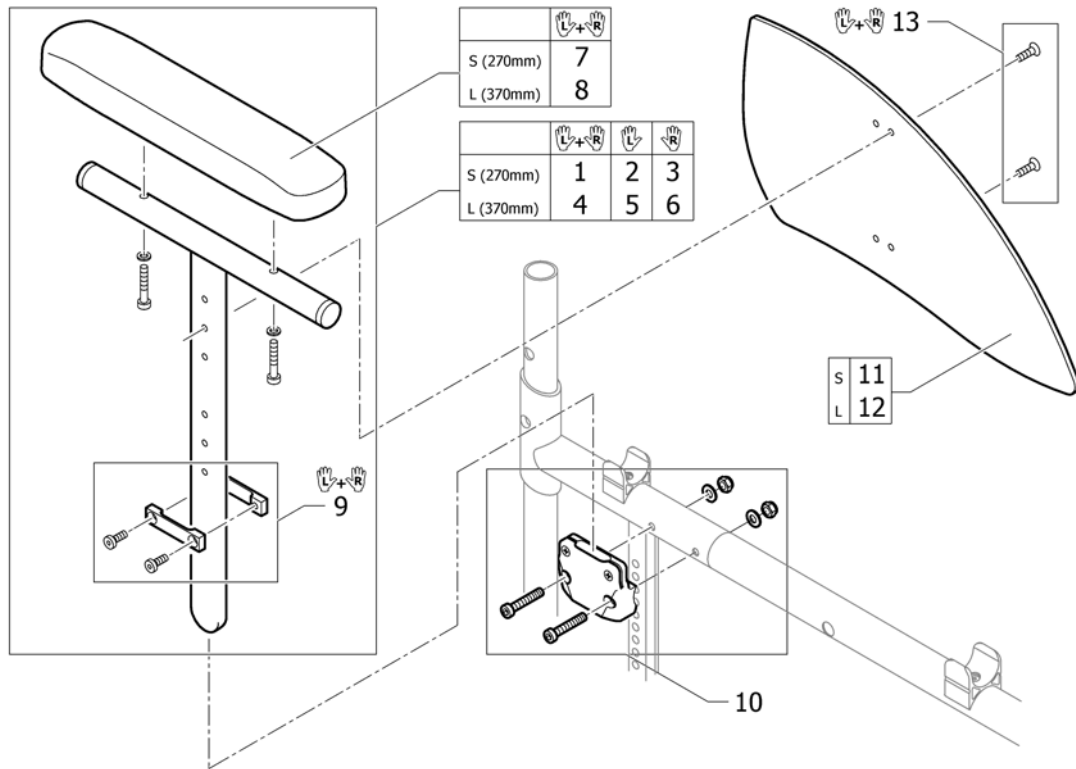

Pos.	Part number	Description	Valid from → until	Qty
1	1517682	Siderest height adjustable short, pair		1
2	1517678	Siderest height adjustable short, left		1
3	1517677	Siderest height adjustable short, right		1
4	1517683	Siderest height adjustable long, pair		1
5	1517680	Siderest height adjustable long, left		1
6	1517679	Siderest height adjustable long, right		1
7	1439497	Armpad S (270mm) incl. screws, pair		1
8	1439498	Armpad L (370mm) incl. screws, pair		1
9	1517681	Siderests height clamp kit, pair		1
10	1540010	Sidepart Adaption complete		2
11	1540008	Clothes-guard S to siderest		2
12	1540009	Clothes-guard L to siderest		2
13	1540011	Clothes-guard fixation kit		1

Siderest in colours, flip-back and height adjustable

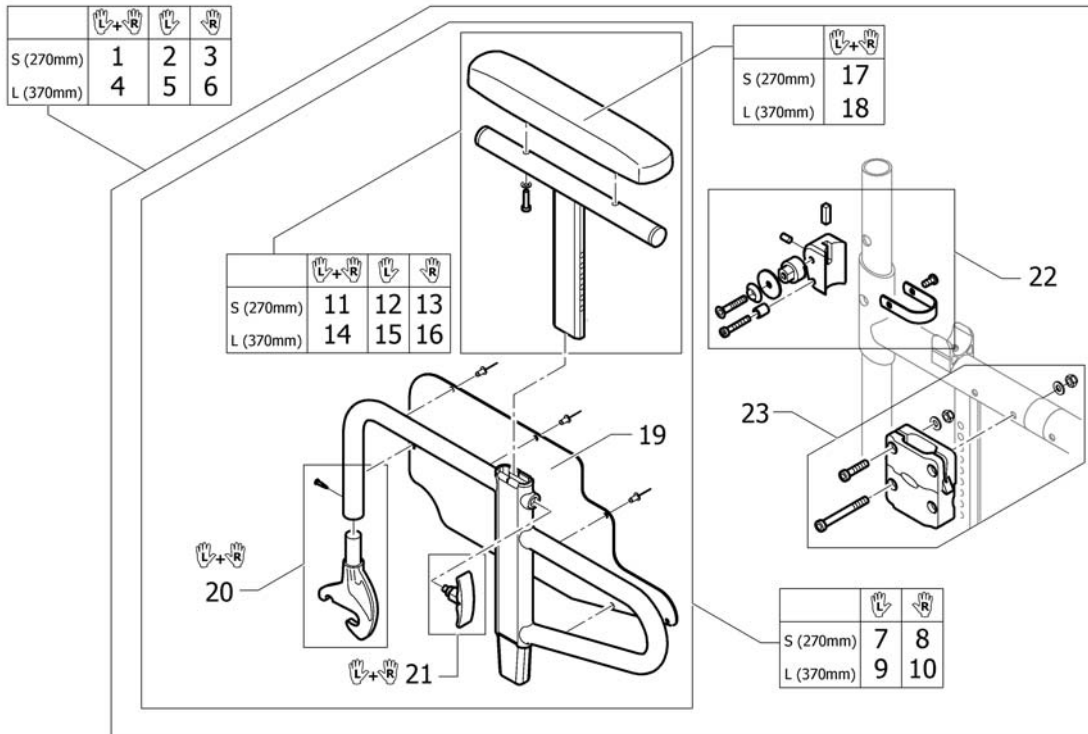

Pos.	Part number	Description	Valid from → until	Qty
1	1540012-0411	Siderest flip-back and height adjustable short, black, matt, complete, pair	20100701 →	1
1	1540012-0419	Siderest flip-back and height adjustable short, silver grey, complete, pair		1
1	1540012-0442	Siderest flip-back and height adjustable short, night shimmer, complete, pair		1
1	1540012-0445	Siderest flip-back and height adjustable short, ivory white, complete, pair	20100701 →	1
1	1540012-0446	Siderest flip-back and height adjustable short, purple fire, complete, pair		1
1	1540012-0447	Siderest flip-back and height adjustable short, emerald green, complete, pair		1
1	1540012-0448	Siderest flip-back and height adjustable short, midnight blue, complete, pair		1
1	1540012-0449	Siderest flip-back and height adjustable short, umber glaze, complete, pair		1
1	1540012-0450	Siderest flip-back and height adjustable short, smokey grey, complete, pair		1
1	1540012-3524	Siderest flip-back and height adjustable short, royal red, complete, pair		1
1	1540012-5969	Siderest flip-back and height adjustable short, royal blue, complete, pair		1
1	1540012-9005	Siderest flip-back and height adjustable short, black, glossy, complete, pair		1
1	1540012-9119	Siderest flip-back and height adjustable short, black, bronze, complete, pair	20100101 → 20100630	1
2	1540014-0411	Siderest flip-back and height adjustable short, black, matt, complete, left	20100701 →	1
2	1540014-0419	Siderest flip-back and height adjustable short, silver grey, complete, left		1
2	1540014-0442	Siderest flip-back and height adjustable short, night shimmer, complete, left		1
2	1540014-0445	Siderest flip-back and height adjustable short, ivory white, complete, left	20100701 →	1
2	1540014-0446	Siderest flip-back and height adjustable short, purple fire, complete, left		1
2	1540014-0447	Siderest flip-back and height adjustable short, emerald green, complete, left		1
2	1540014-0448	Siderest flip-back and height adjustable short, midnight blue, complete, left		1
2	1540014-0449	Siderest flip-back and height adjustable short, umber glaze, complete, left		1
2	1540014-0450	Siderest flip-back and height adjustable short, smokey grey, complete, left		1
2	1540014-3524	Siderest flip-back and height adjustable short, royal red, complete, left		1
2	1540014-5969	Siderest flip-back and height adjustable short, royal blue, complete, left		1
2	1540014-9005	Siderest flip-back and height adjustable short, black, glossy, complete, left		1

Siderest in colours, flip-back and height adjustable

Pos.	Part number	Description	Valid from → until	Qty
2	1540014-9119	Siderest flip-back and height adjustable short, black, bronze, complete,	20100101 → 20100630	1
3	1540016-0411	Siderest flip-back and height adjustable short, black, matt, complete, right	20100701 →	1
3	1540016-0419	Siderest flip-back and height adjustable short, silver grey, complete, right		1
3	1540016-0442	Siderest flip-back and height adjustable short, night shimmer, complete, right		1
3	1540016-0445	Siderest flip-back and height adjustable short, ivory white, complete, right	20100701 →	1
3	1540016-0446	Siderest flip-back and height adjustable short, purple fire, complete, right		1
3	1540016-0447	Siderest flip-back and height adjustable short, emerald green, complete, right		1
3	1540016-0448	Siderest flip-back and height adjustable short, midnight blue, complete, right		1
3	1540016-0449	Siderest flip-back and height adjustable short, umber glaze, complete, right		1
3	1540016-0450	Siderest flip-back and height adjustable short, smokey grey, complete, right		1
3	1540016-3524	Siderest flip-back and height adjustable short, royal red, complete, right		1
3	1540016-5969	Siderest flip-back and height adjustable short, royal blue, complete, right		1
3	1540016-9005	Siderest flip-back and height adjustable short, black, glossy, complete, right		1
3	1540016-9119	Siderest flip-back and height adjustable short, black, bronze, complete, right	20100101 → 20100630	1
4	1540013-0411	Siderest flip-back and height adjustable long, black, matt, complete, pair	20100701 →	1
4	1540013-0419	Siderest flip-back and height adjustable long, silver grey, complete, pair		1
4	1540013-0442	Siderest flip-back and height adjustable long, night shimmer, complete, pair		1
4	1540013-0445	Siderest flip-back and height adjustable long, ivory white, complete, pair	20100701 →	1
4	1540013-0446	Siderest flip-back and height adjustable long, purple fire, complete, pair		1
4	1540013-0447	Siderest flip-back and height adjustable long, emerald green, complete, pair		1
4	1540013-0448	Siderest flip-back and height adjustable long, midnight blue, complete,		1
4	1540013-0449	Siderest flip-back and height adjustable long, umber glaze, complete, pair		1
4	1540013-0450	Siderest flip-back and height adjustable long, smokey grey, complete,		1
4	1540013-3524	Siderest flip-back and height adjustable long, royal red, complete, pair		1
4	1540013-5969	Siderest flip-back and height adjustable long, royal blue, complete, pair		1
4	1540013-9005	Siderest flip-back and height adjustable long, black, glossy, complete, pair		1
4	1540013-9119	Siderest flip-back and height adjustable long, black, bronze, complete,	20100101 → 20100630	1
5	1540015-0411	Siderest flip-back and height adjustable long, black, matt, complete, left	20100701 →	1
5	1540015-0419	Siderest flip-back and height adjustable long, silver grey, complete, left		1
5	1540015-0442	Siderest flip-back and height adjustable long, night shimmer, complete,		1
5	1540015-0445	Siderest flip-back and height adjustable long, ivory white, complete, left	20100701 →	1
5	1540015-0446	Siderest flip-back and height adjustable long, purple fire, complete, left		1
5	1540015-0447	Siderest flip-back and height adjustable long, emerald green, complete, left		1
5	1540015-0448	Siderest flip-back and height adjustable long, midnight blue, complete, left		1
5	1540015-0449	Siderest flip-back and height adjustable long, umber glaze, complete, left		1
5	1540015-0450	Siderest flip-back and height adjustable long, smokey grey, complete, left		1
5	1540015-3524	Siderest flip-back and height adjustable long, royal red, complete, left		1
5	1540015-5969	Siderest flip-back and height adjustable long, royal blue, complete, left		1
5	1540015-9005	Siderest flip-back and height adjustable long, black, glossy, complete, left		1
5	1540015-9119	Siderest flip-back and height adjustable long, black, bronze, complete, left	20100101 → 20100630	1
6	1540017-0411	Siderest flip-back and height adjustable long, black, matt, complete, right	20100701 →	1
6	1540017-0419	Siderest flip-back and height adjustable long, silver grey, complete, right		1
6	1540017-0442	Siderest flip-back and height adjustable long, night shimmer, complete, right		1
6	1540017-0445	Siderest flip-back and height adjustable long, ivory white, complete, right	20100701 →	1
6	1540017-0446	Siderest flip-back and height adjustable long, purple fire, complete, right		1
6	1540017-0447	Siderest flip-back and height adjustable long, emerald green, complete, right		1


Siderest in colours, flip-back and height adjustable

Pos.	Part number	Description	Valid from → until	Qty
6	1540017-0448	Siderest flip-back and height adjustable long, midnight blue, complete, right		1
6	1540017-0449	Siderest flip-back and height adjustable long, umber glaze, complete, right		1
6	1540017-0450	Siderest flip-back and height adjustable long, smokey grey, complete, right		1
6	1540017-3524	Siderest flip-back and height adjustable long, royal red, complete, right		1
6	1540017-5969	Siderest flip-back and height adjustable long, royal blue, complete, right		1
6	1540017-9005	Siderest flip-back and height adjustable long, black, glossy, complete, right		1
6	1540017-9119	Siderest flip-back and height adjustable long, black, bronze, complete, right	20100101 → 20100630	1
7	1540018-0411	Siderest flip-back and height adjustable short, black, matt, left	20100701 →	1
7	1540018-0419	Siderest flip-back and height adjustable short, silver grey, left		1
7	1540018-0442	Siderest flip-back and height adjustable short, night shimmer, left		1
7	1540018-0445	Siderest flip-back and height adjustable short, ivory white, left	20100701 →	1
7	1540018-0446	Siderest flip-back and height adjustable short, purple fire, left		1
7	1540018-0447	Siderest flip-back and height adjustable short, emerald green, left		1
7	1540018-0448	Siderest flip-back and height adjustable short, midnight blue, left		1
7	1540018-0449	Siderest flip-back and height adjustable short, umber glaze, left		1
7	1540018-0450	Siderest flip-back and height adjustable short, smokey grey, left		1
7	1540018-3524	Siderest flip-back and height adjustable short, royal red, left		1
7	1540018-5969	Siderest flip-back and height adjustable short, royal blue, left		1
7	1540018-9005	Siderest flip-back and height adjustable short, black, glossy, left		1
7	1540018-9119	Siderest flip-back and height adjustable short, black, bronze, left	20100101 → 20100630	1
8	1540020-0411	Siderest flip-back and height adjustable short, black, matt, right	20100701 →	1
8	1540020-0419	Siderest flip-back and height adjustable short, silver grey, right		1
8	1540020-0442	Siderest flip-back and height adjustable short, night shimmer, right		1
8	1540020-0445	Siderest flip-back and height adjustable short, ivory white, right	20100701 →	1
8	1540020-0446	Siderest flip-back and height adjustable short, purple fire, right		1
8	1540020-0447	Siderest flip-back and height adjustable short, emerald green, right		1
8	1540020-0448	Siderest flip-back and height adjustable short, midnight blue, right		1
8	1540020-0449	Siderest flip-back and height adjustable short, umber glaze, right		1
8	1540020-0450	Siderest flip-back and height adjustable short, smokey grey, right		1
8	1540020-3524	Siderest flip-back and height adjustable short, royal red, right		1
8	1540020-5969	Siderest flip-back and height adjustable short, royal blue, right		1
8	1540020-9005	Siderest flip-back and height adjustable short, black, glossy, right		1
8	1540020-9119	Siderest flip-back and height adjustable short, black, bronze, right	20100101 → 20100630	1
9	1540019-0411	Siderest flip-back and height adjustable long, black, matt, left	20100701 →	1
9	1540019-0419	Siderest flip-back and height adjustable long, silver grey, left		1
9	1540019-0442	Siderest flip-back and height adjustable long, night shimmer, left		1
9	1540019-0445	Siderest flip-back and height adjustable long, ivory white, left	20100701 →	1
9	1540019-0446	Siderest flip-back and height adjustable long, purple fire, left		1
9	1540019-0447	Siderest flip-back and height adjustable long, emerald green, left		1
9	1540019-0448	Siderest flip-back and height adjustable long, midnight blue, left		1
9	1540019-0449	Siderest flip-back and height adjustable long, umber glaze, left		1
9	1540019-0450	Siderest flip-back and height adjustable long, smokey grey, left		1
9	1540019-3524	Siderest flip-back and height adjustable long, royal red, left		1
9	1540019-5969	Siderest flip-back and height adjustable long, royal blue, left		1
9	1540019-9005	Siderest flip-back and height adjustable long, black, glossy, left		1
9	1540019-9119	Siderest flip-back and height adjustable long, black, bronze, left	20100101 → 20100630	1
10	1540021-0411	Siderest flip-back and height adjustable long, black, matt, right	20100701 →	1
10	1540021-0419	Siderest flip-back and height adjustable long, silver grey, right		1
10	1540021-0442	Siderest flip-back and height adjustable long, night shimmer, right		1
10	1540021-0445	Siderest flip-back and height adjustable long, ivory white, right	20100701 →	1
10	1540021-0446	Siderest flip-back and height adjustable long, purple fire, right		1
10	1540021-0447	Siderest flip-back and height adjustable long, emerald green, right		1

Siderest in colours, flip-back and height adjustable


Pos.	Part number	Description	Valid from → until	Qty
10	1540021-0448	Siderest flip-back and height adjustable long, midnight blue, right		1
10	1540021-0449	Siderest flip-back and height adjustable long, umber glaze, right		1
10	1540021-0450	Siderest flip-back and height adjustable long, smokey grey, right		1
10	1540021-3524	Siderest flip-back and height adjustable long, royal red, right		1
10	1540021-5969	Siderest flip-back and height adjustable long, royal blue, right		1
10	1540021-9005	Siderest flip-back and height adjustable long, black, glossy, right		1
10	1540021-9119	Siderest flip-back and height adjustable long, black, bronze, right	20100101 → 20100630	1
11	1540022	Holder with armpad S (270mm), pair		1
12	1540024	Holder left with armpad S (270mm)		1
13	1540026	Holder left with armpad L (370mm)		1
14	1540023	Holder with armpad L (370mm), pair		1
15	1540025	Holder right with armpad S (270mm)		1
16	1540027	Holder right with armpad L (370mm)		1
17	1439497	Armpad S (270mm) incl. screws, pair		1
18	1439498	Armpad L (370mm) incl. screws, pair		1
19	1540030	Clothes-guard to siderest in colours		2
20	1540031	Hook bracket fr siderest, pair		1
21	1439581	Closing mechanism for Siderest height adj. Flip back		1
22	1540028	Fixation kit rear for Siderest in colours		2
23	1540029	Fixation kit front for siderest		2

Siderest with desk function

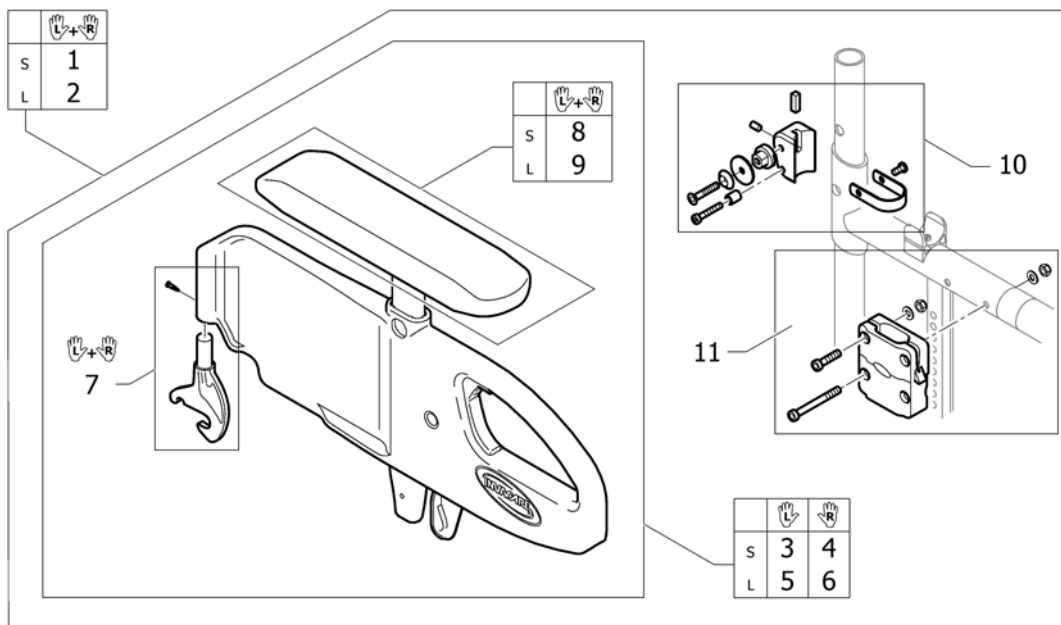


Pos.	Part number	Description	Valid from → until	Qty
1	1540043	Siderest with desk function, flip-back, height adj., removable, pair short, complete		1
2	1540044	Siderest with desk function, flip-back, height adj., removable, pair long, complete		1
3	1540040	Siderest with desk function, flip-back, height adj., removable, short, left		1
4	1540039	Siderest with desk function, flip-back, height adj., removable, short, right		1
5	1540042	Siderest with desk function, flip-back, height adj., removable, long left		1
6	1540041	Siderest with desk function, flip-back, height adj., removable, long, right		1
7	1540031	Hook bracket fr siderest, pair		1
8	1521532	Armpad short incl. Screws, pair		1
9	1521533	Armpad long incl. Screws, pair		1
10	1540032	Siderest with desk function fixation kit		1
11	1540029	Fixation kit front for siderest		1

Armrest Hemi


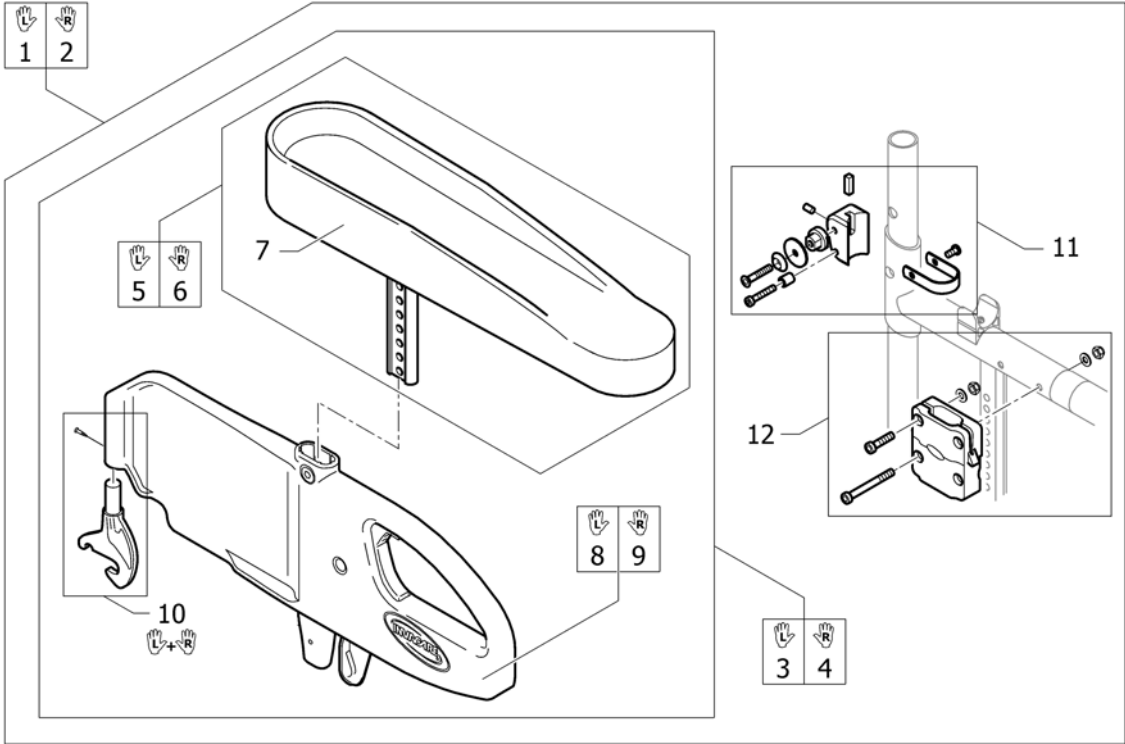


Pos.	Part number	Description	Valid from → until	Qty
1	1540045	Armrest Hemi left, complete		1
2	1540046	Armrest Hemi right, complete		1
3	1540047	Armrest Hemi left		1
4	1540048	Armrest Hemi right		1
5	1540050	Holder left with Armpad for Armrest Hemi		1
6	1540051	Holder right with Armpad for Armrest Hemi		1
7	1540049	Armpad for Armrest Hemi		1
8	1540052	Armrest Hemi left, lower part		1
9	1540053	Armrest Hemi right, lower part		1
10	1540031	Hook bracket fr siderest, pair		1
11	1540032	Siderest with desk function fixation kit		1
12	1540029	Fixation kit front for siderest		1

Tubular armrest

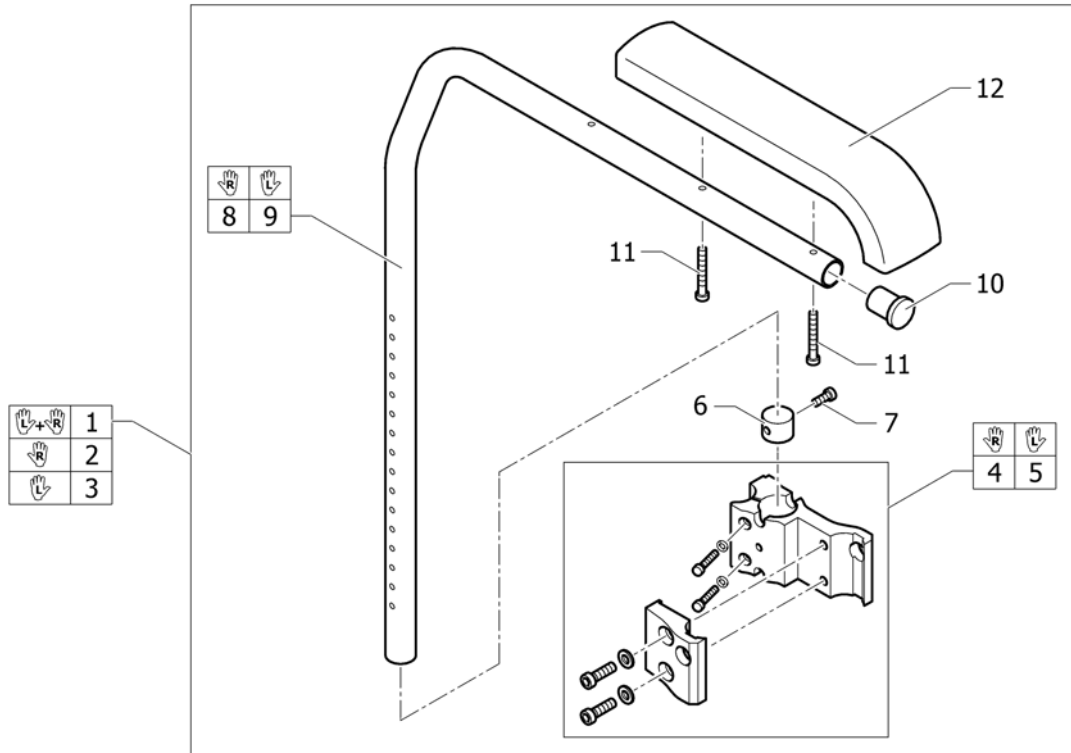


Pos.	Part number	Description	Valid from → until	Qty
1	913.1004	Tubular armrest pair, complete, with standard cushion		1
2	913.1157.000	Tubular armrest right, complete, with standard cushion		1
3	913.1158.000	Tubular armrest left, complete, with standard cushion		1
4	913.1103	Clamping bracket complete, right		1
5	913.1009	Clamping bracket complete, left		1
6	305.1063	Armrest clamping part sleeve		2
7	100.1084	M5x12 12.9 screw		2
8	456.1516.956	Armrest short right		1
9	456.1517.956	Armrest short left		1
10	220.1013	Tube plug Ø24mm		2
11	100.5091	M5x35 shank 8.8 screw		4
12	702.1004	Standard cushion, bent		2

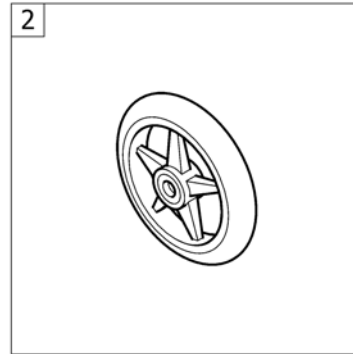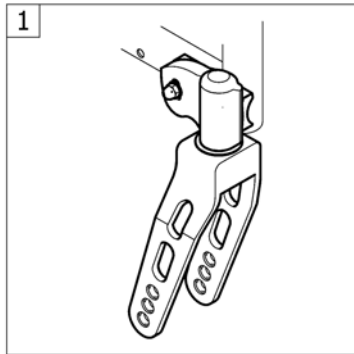

Pos.	Part number	Description	Valid from → until	Qty
1	-	Front wheel supporter		1
2	-	Front wheels		1


Front wheel supporter

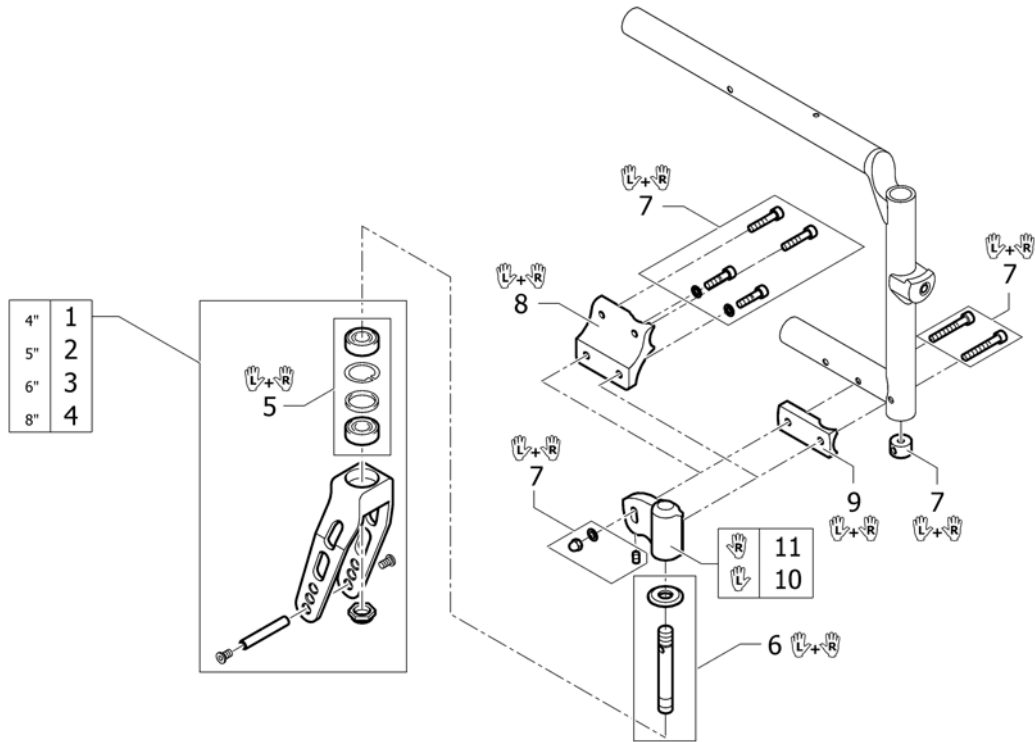








Pos.	Part number	Description	Valid from → until	Qty
1	1532028-0004	Front fork 4", black anodized		2
1	1532028-0400	Front fork 4", techno-anthrazit		2
1	1532028-0401	Front fork 4", lemon		2
1	1532028-0402	Front fork 4", apricot		2
1	1532028-0404	Front fork 4", b-blue		2
1	1532028-0440	Front fork 4", comet red		2
1	1532028-0441	Front fork 4", arctic blue		2
1	1532028-0444	Front fork 4", copper glaze		2
2	1532029-0004	Front fork 5", black anodized		2
2	1532029-0400	Front fork 5", techno-anthrazit		2
2	1532029-0401	Front fork 5", lemon		2
2	1532029-0402	Front fork 5", apricot		2
2	1532029-0404	Front fork 5", b-blue		2
2	1532029-0440	Front fork 5", comet red		2
2	1532029-0441	Front fork 5", arctic blue		2
2	1532029-0444	Front fork 5", copper glaze		2
3	1532030-0004	Front fork 6", black anodized		2
3	1532030-0400	Front fork 6", techno-anthrazit		2
3	1532030-0401	Front fork 6", lemon		2
3	1532030-0402	Front fork 6", apricot		2
3	1532030-0404	Front fork 6", b-blue		2
3	1532030-0440	Front fork 6", comet red		2
3	1532030-0441	Front fork 6", arctic blue		2
3	1532030-0444	Front fork 6", copper glaze		2
4	1532031-0004	Front fork 8", black anodized		2
4	1532031-0400	Front fork 8", techno-anthrazit		2
4	1532031-0401	Front fork 8", lemon		2
4	1532031-0402	Front fork 8", apricot		2
4	1532031-0404	Front fork 8", b-blue		2
4	1532031-0440	Front fork 8", comet red		2
4	1532031-0441	Front fork 8", arctic blue		2

Front wheel supporter

Pos.	Part number	Description	Valid from → until	Qty
4	1532031-0444	Front fork 8", copper glaze		2
5	1510219	Bearing kit		1
6	1532032	Front fork pin kit, pair		1
7	1532034	Fixation kit for front fork castor Compact		1
8	1532035-0004	Fixation parts for front fork castor high or low mounted, black anodized, Compact, pair		1
8	1532035-0400	Fixation parts for front fork castor high or low mounted, techno-anthrazit, Compact, pair		1
8	1532035-0401	Fixation parts for front fork castor high or low mounted, lemon, Compact, pair		1
8	1532035-0402	Fixation parts for front fork castor high or low mounted, apricot, Compact, pair		1
8	1532035-0404	Fixation parts for front fork castor high or low mounted, b-blue, Compact, pair		1
8	1532035-0440	Fixation parts for front fork castor high or low mounted, comet red, Compact, pair		1
8	1532035-0441	Fixation parts for front fork castor high or low mounted, arctic blue, Compact, pair		1
8	1532035-0444	Fixation parts for front fork castor high or low mounted, copper glaze, Compact, pair		1
9	1532037	Fixation parts Standard for front fork castor, Compact, pair		1
10	1529375-0004	Front fork castor left, black anodized		1
10	1529375-0400	Front fork castor left, techno-anthrazit		1
10	1529375-0401	Front fork castor left, lemon		1
10	1529375-0402	Front fork castor left, apricot		1
10	1529375-0404	Front fork castor left, b-blue		1
10	1529375-0440	Front fork castor left, comet red		1
10	1529375-0441	Front fork castor left, arctic blue		1
10	1529375-0444	Front fork castor left, copper glaze		1
11	1529376-0004	Front fork castor right, black anodized		1
11	1529376-0400	Front fork castor right, techno-anthrazit		1
11	1529376-0401	Front fork castor right, lemon		1
11	1529376-0402	Front fork castor right, apricot		1
11	1529376-0404	Front fork castor right, b-blue		1
11	1529376-0440	Front fork castor right, comet red		1
11	1529376-0441	Front fork castor right, arctic blue		1
11	1529376-0444	Front fork castor right, copper glaze		1

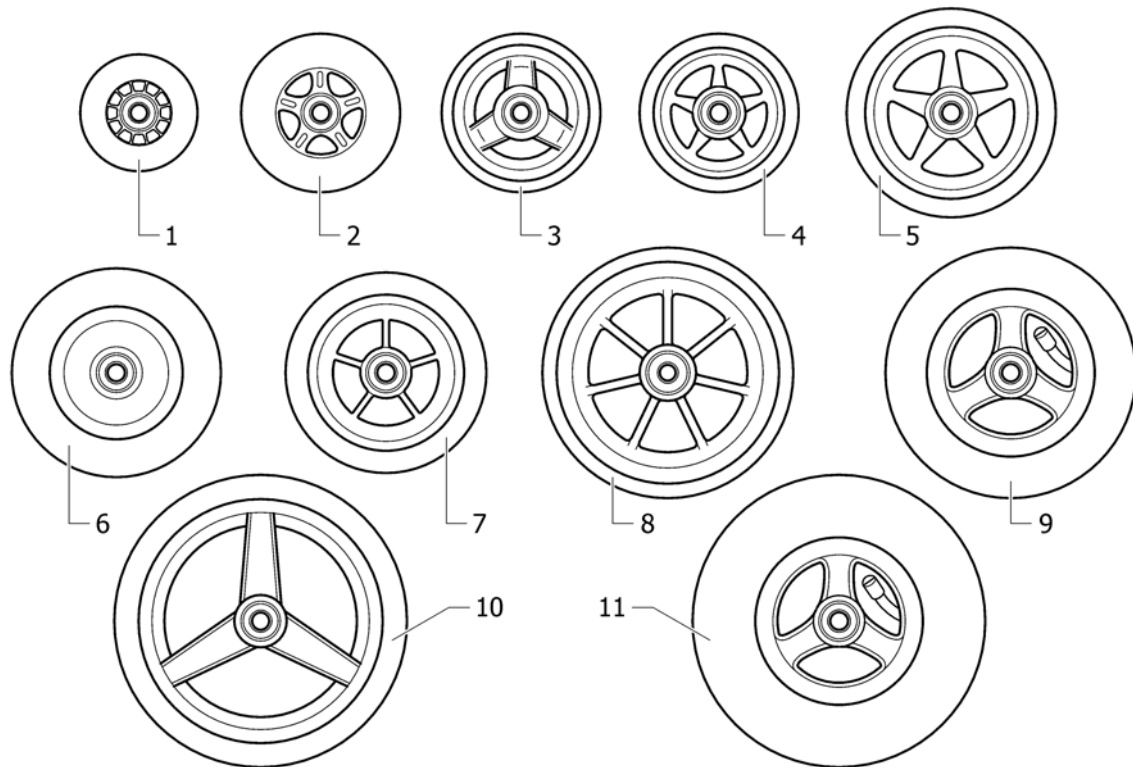
Front wheels

Pos.	Part number	Description	Valid from → until	Qty
1	1513180	3" Sports wheel, transparent		2
2	1513181	4" Skater wheel, transparent		2
3	1513182	4" Everyday wheel, 3 spokes		2
4	1513183-0407	4" Starec wheel, colourless		2
4	1513183-0400	4" Starec wheel, techno-anthrazit		2
4	1513183-0401	4" Starec wheel, lemon		2
4	1513183-0402	4" Starec wheel, apricot		2
4	1513183-0404	4" Starec wheel, b-blue		2
4	1513183-0440	4" Starec wheel, comet red		2
4	1513183-0441	4" Starec wheel, arctic blue		2
4	1513183-0444	4" Starec wheel, copper glaze		2
4	1513183-0408	4" Starec wheel, glossy black		2
5	1513187-0407	5" Starec wheel, colourless		2
5	1513187-0400	5" Starec wheel, techno-anthrazit		2
5	1513187-0401	5" Starec wheel, lemon		2
5	1513187-0402	5" Starec wheel, apricot		2
5	1513187-0404	5" Starec wheel, b-blue		2
5	1513187-0440	5" Starec wheel, comet red		2
5	1513187-0441	5" Starec wheel, arctic blue		2
5	1513187-0444	5" Starec wheel, copper glaze		2
5	1513187-0408	5" Starec wheel, black		2
6	1513186	5" Soft roll wheel		2
7	1513185	5" Light wheel		2
8	1513190	6" Light wheel, 7 spokes		2
9	1513189	6" Pneumatic wheel		2
10	1513192	7" Sports wheel		2
11	1513191	7" Pneumatic wheel		2

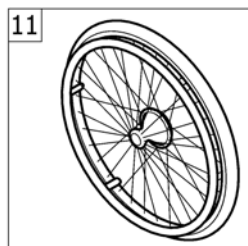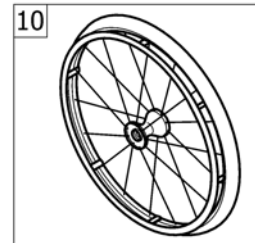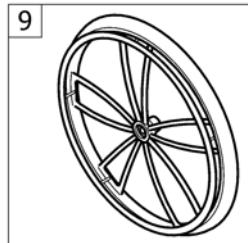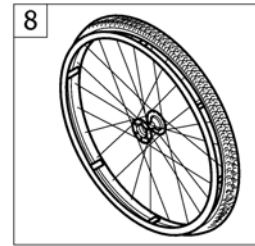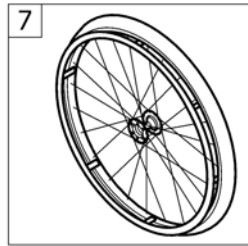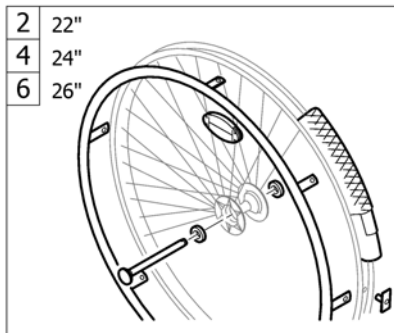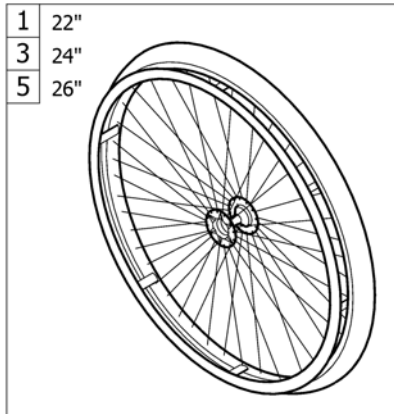


Pos.	Part number	Description	Valid from → until	Qty
1	-	22" Küschall rear wheels complete		1
2	-	22" Küschall rear wheel accessories		1
3	-	24" Küschall rear wheels complete		1
4	-	24" Küschall rear wheel accessories		1
5	-	26" Küschall rear wheels complete		1
6	-	26" Küschall rear wheel accessories		1
7	-	High Performance wheel		1
8	-	Mountain bike wheel		1
9	-	Spider wheel		1
10	-	Spinergy wheel		1
11	-	Drumbrake wheel		1

22" Küschall rear wheels complete

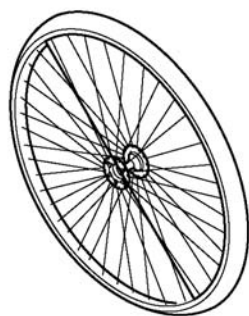

	⊘ no tyre			
	1	2	3	4
	5	6	7	8
	9	10	11	12

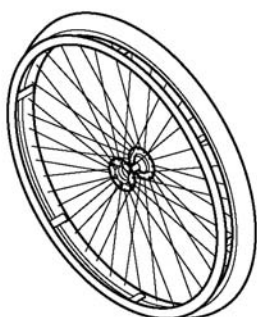


	⊘ no tyre			
		13	14	15
		16	17	18
		19	20	21

Pos.	Part number	Description	Valid from → until	Qty
1	904.1077	Rear wheel 22", standard hub, without handrim, without tyre		2
2	904.1180	Rear wheel 22", Standard hub, without handrim, light wall		2
3	904.1169	Rear wheel 22", standard hub, without handrim, Tread		2
4	904.1170	Rear wheel 22", standard hub, without handrim, Indoor sports		2
5	904.1081	Rear wheel 22", Starec hub R, without handrim, without tyre		2
5	904.1081.950	Rear wheel 22" Starec R, tecno, without handrim, without tyre		2
5	904.1081.951	Rear wheel 22" Starec R, lemon, without handrim, without tyre		2
5	904.1081.954	Rear wheel 22" Starec R, apricot, without handrim, without tyre		2
5	904.1081.957	Rear wheel 22" Starec R, b-blue, without handrim, without tyre		2
5	904.1081-0440	Rear wheel 22" Starec R, comet red, without handrim, without tyre		2
5	904.1081-0441	Rear wheel 22" Starec R, arctic blue, without handrim, without tyre		2
5	904.1081-0444	Rear wheel 22" Starec R, copper glaze, without handrim, without tyre		2
6	904.1195	Rear wheel 22 ", Starec hub R, without handrim, light wall		2
7	904.1158	Rear wheel 22", Starec hub R, without handrim, Tread		2
8	904.1159	Rear wheel 22", Starec hub R, without handrim, Indoor sports		2
9	904.1131	Rear wheel 22", Starec hub X, without handrim, without tyre		2
9	904.1131.950	Rear wheel 22" Starec X Tecno without handrim, without tyre		2
9	904.1131.951	Rear wheel 22" Starec X Lemon without handrim, without tyre		2
9	904.1131.954	Rear wheel 22" Starec X Apricot without handrim, without tyre		2
9	904.1131.957	Rear wheel 22" Starec X B-blue without handrim, without tyre		2
9	904.1131-0440	Rear wheel 22" Starec X Comet red without handrim, without tyre		2
9	904.1131-0441	Rear wheel 22" Starec X arctic blue, without handrim, without tyre		2
9	904.1131-0444	Rear wheel 22" Starec X, copper glaze, without handrim, without tyre		2
10	904.1189	Rear wheel 22", Starec hub X, without handrim, light wall		2
11	904.1139	Rear wheel 22", Starec hub X, without handrim, Tread		2
12	904.1140	Rear wheel 22", Starec hub X, without handrim, Indoor sports		2
13	904.1178	Rear wheel 22", Standard hub, handrim, light wall		2
14	904.1054	Rear wheel 22", standard hub, handrim, Tread		2
15	904.1066	Rear wheel 22", standard hub, handrim, indoors tyre		2
16	904.1192	Rear wheel 22", Starec hub R, handrim, light wall		2
17	904.1149	Rear wheel 22", Starec hub R, handrim, Tread		2

22" Küschall rear wheels complete


Pos.	Part number	Description	Valid from → until	Qty
18	904.1150	Rear wheel 22", Starec hub R, handrim, Indoor sports		2
19	904.1186	Rear wheel 22", Starec hub X, handrim, light wall		2
20	904.1123	Rear wheel 22", Starec hub X, handrim, Tread		2
21	904.1124	Rear wheel 22", Starec hub X, handrim, Indoor sports		2

22" Küschall rear wheel accessories

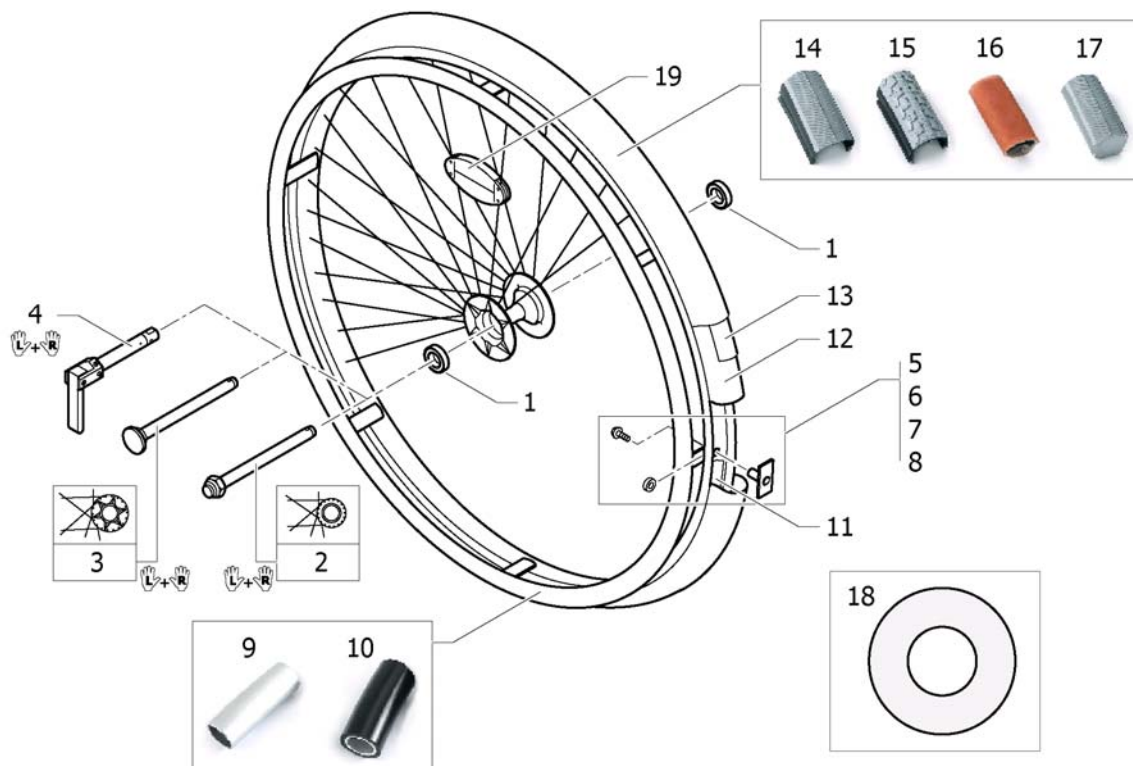

Pos.	Part number	Description	Valid from → until	Qty
1	270.1003	R8-2RS 1/2"/1.1/8"x5/16"		4
2	1512031	Quick release axles long, pair		1
3	1509137-0407	Quick release axles long with cap, pair, colourless		1
3	1509137-0400	Quick release axles long with cap, pair, tecno anthracite		1
3	1509137-0401	Quick release axles long with cap, pair, lemon		1
3	1509137-0402	Quick release axles long with cap, pair, apricot		1
3	1509137-0404	Quick release axles long with cap, pair, b-blue		1
3	1509137-0440	Quick release axles long with cap, pair, comet red		1
3	1509137-0441	Quick release axles long with cap, pair, arctic blue		1
3	1509137-0444	Quick release axles long with cap, pair, copper glaze		1
4	1493755	Tetra clip with quick release axle, pair, long		1
5	1495958	Fixation kit handrim, 3 cm		2
6	1495959	Fixation kit handrim, 4 cm		2
7	1495960	Fixation kit handrim, 3 cm with spokes guard		2
8	1495961	Fixation kit handrim, 4 cm with spokes guard		2
9	457.1023	Handrim 22" Alu		2
10	457.1045.429	Handrim 22" Super-Grip		2
11	726.1005	Rim protection 22"		2
12	727.1007	Inner Tube 22" x 1"		2
13	1424917	Rear wheel inset anti-flat for 20,22,24,26" tyre		2
14	728.1027	Tyre Lightwall 22"		2
15	1493768	Treaded tyre 22"		2
16	729.1006	Indoor sports tyre 22" / 10 bar		2
17	728.1031	Solid tyre 22", light, grey		2
18	736.1005	Spokes guard Transparent 22"		2
19	220.1033	Spokereflection		2

24" Küschall rear wheels complete

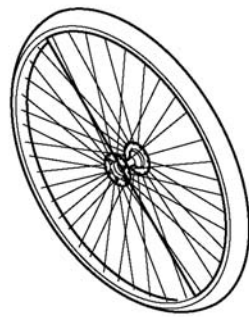

	∅ no tyre			
	1	2	3	4
	5	6	7	8
	9	10	11	12




	∅ no tyre			
		13	14	15
		16	17	
		18	19	20

Pos.	Part number	Description	Valid from → until	Qty
1	1416882	Rear wheel 24", Standard hub, without handrim, without tyre		2
2	1416883	Rear wheel 24", Standard hub, without handrim, light wall		2
3	904.1171	Rear wheel 24", standard hub, without handrim, Tread		2
4	904.1174	Rear wheel 24", standard hub, without handrim, Indoor sports		2
5	904.1082	Rear wheel 24", Starec hub R, without handrim, without tyre		2
5	904.1082.950	Rear wheel 24" Starec R, tecno, without handrim, without tyre		2
5	904.1082.951	Rear wheel 24" Starec R, lemon, without handrim, without tyre		2
5	904.1082.954	Rear wheel 24" Starec R, apricot, without handrim, without tyre		2
5	904.1082.957	Rear wheel 24" Starec R, b-blue, without handrim, without tyre		2
5	904.1082-0440	Rear wheel 24" Starec R, comet red, without handrim, without tyre		2
5	904.1082-0441	Rear wheel 24" Starec R, arctic blue, without handrim, without tyre		2
5	904.1082-0444	Rear wheel 24" Starec R, copper glaze, without handrim, without tyre		2
6	904.1161	Rear wheel 24", Starec hub R, without handrim, light wall		2
7	904.1160	Rear wheel 24", Starec hub R, without handrim, Tread		2
8	904.1163	Rear wheel 24", Starec hub R, without handrim, Indoor sports		2
9	1416885	Rear wheel 24", Starec hub X, without handrim, without tyre		2
9	904.1132.950	Rear wheel 24" Starec X Tecno without handrim, without tyre		2
9	904.1132.951	Rear wheel 24" Starec X Lemon without handrim, without tyre		2
9	904.1132.954	Rear wheel 24" Starec X Apricot without handrim, without tyre		2
9	904.1132.957	Rear wheel 24" Starec X B-blue without handrim, without tyre		2
9	904.1132-0440	Rear wheel 24" Starec X Comet red without handrim, without tyre		2
9	904.1132-0441	Rear wheel 24" Starec X arctic blue without handrim, without tyre		2
9	904.1132-0444	Rear wheel 24" Starec X Copper glaze without handrim, without tyre		2
10	1416886	Rear wheel 24", Starec hub X, without handrim, light wall		2
11	904.1141	Rear wheel 24", Starec hub X, without handrim, Tread		2
12	904.1144	Rear wheel 24", Starec hub X, without handrim, Indoor sports		2
13	1416881	Rear wheel 24", Standard hub, handrim, light wall		2
14	904.1005	Rear wheel 24", standard hub, handrim, Tread		2
15	904.1006	Rear wheel 24", standard hub, handrim, indoors tyre		2
16	904.1152	Rear wheel 24", Starec hub R, handrim, light wall		2
17	904.1151	Rear wheel 24", Starec hub R, handrim, Tread		2

24" Küschall rear wheels complete

Pos.	Part number	Description	Valid from → until	Qty
18	1416884	Rear wheel 24", Starec hub X, handrim, light wall		2
19	904.1125	Rear wheel 24", Starec hub X, handrim, Tread		2
20	904.1135	Rear wheel 24", Starec hub X, handrim, Indoor sports		2

24" Küschall rear wheel accessories

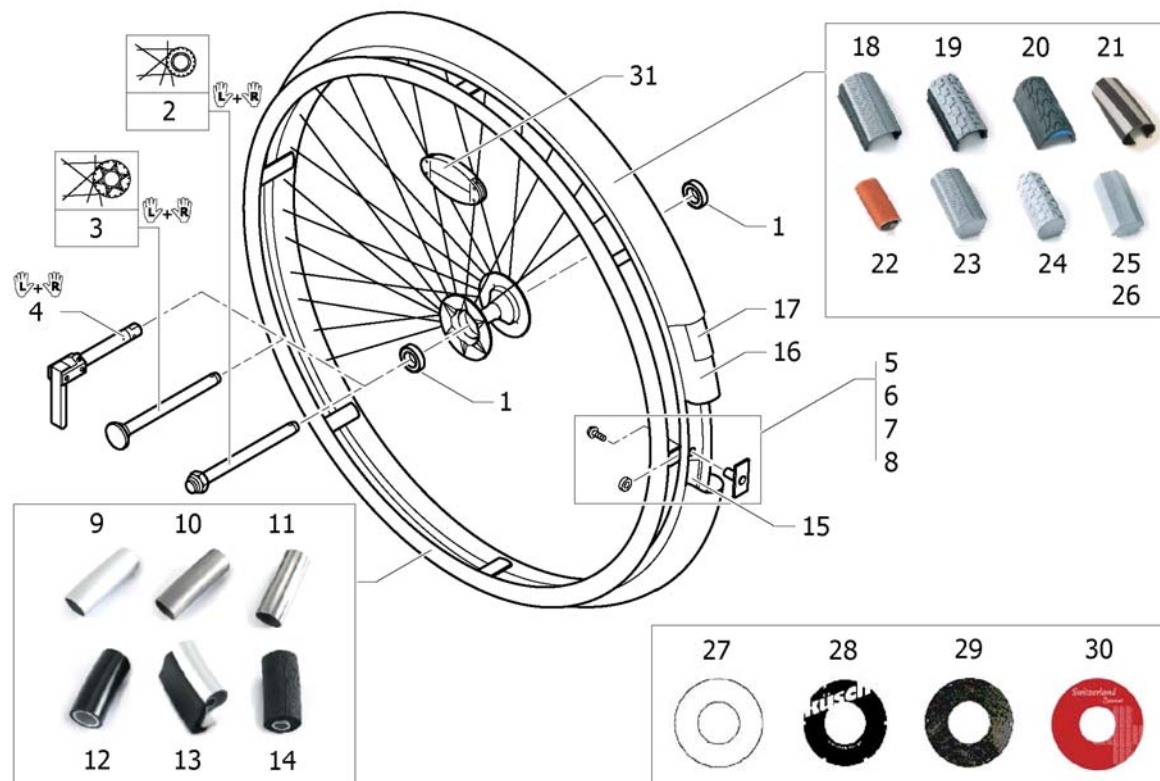


Pos.	Part number	Description	Valid from → until	Qty
1	270.1003	R8-2RS 1/2"/1.1/8"x5/16"		4
2	1512031	Quick release axles long, pair		1
3	1509137-0407	Quick release axles long with cap, pair, colourless		1
3	1509137-0400	Quick release axles long with cap, pair, tecno anthracite		1
3	1509137-0401	Quick release axles long with cap, pair, lemon		1
3	1509137-0402	Quick release axles long with cap, pair, apricot		1
3	1509137-0404	Quick release axles long with cap, pair, b-blue		1
3	1509137-0440	Quick release axles long with cap, pair, comet red		1
3	1509137-0441	Quick release axles long with cap, pair, arctic blue		1
3	1509137-0444	Quick release axles long with cap, pair, copper glaze		1
4	1493755	Tetra clip with quick release axle, pair, long		1
5	1495958	Fixation kit handrim, 3 cm		2
6	1495959	Fixation kit handrim, 4 cm		2
7	1495960	Fixation kit handrim, 3 cm with spokes guard		2
8	1495961	Fixation kit handrim, 4 cm with spokes guard		2
9	1432287-0005	Handrim 24" colourless anod. light V2003		2
10	457.1051	Handrim 24" Titanium		2
11	457.1009	Handrim 24" steel polished		2
12	457.1017.429	Handrim 24" Super-Grip		2
13	457.1052.000	Handrim 24" Tetra 2000		2
14	905.5002	Handrim 24" Maxgrip		2
15	726.1008.000	Rim protection 24" Large		2
16	727.1002	Inner Tube 24" x 1"		2
17	1424917	Rear wheel inset anti-flat for 20,22,24,26" tyre		2
18	1424749	Tyre Light wall 24"		2
19	728.1002	Treaded Tyre 24"		2
20	1438521	Tyre 24" Marathon Evolution		2
21	1528649	Tyre 24" Speedrun		2
22	729.1001	Indoor sports tyre 24" / 10 bar		2
23	728.1032	Solid tyre 24", light, grey		2
24	728.1029	Solid tyre 24", profile		2

24" Küschall rear wheel accessories

Pos.	Part number	Description	Valid from → until	Qty
25	728.1015	KIK-Type 24", black, fail-safe		2
26	728.1023	KIK-Type 24", grey, fail-safe		2
27	736.1004	Spokes guard Transparent 24"		2
28	736.1002	Spokes guard Küschall, black 24"		2
29	736.1014	Spokes guard Carbon 24"		2
30	1497108	Spokes guard Switzerland 24"		2
31	220.1033	Spokereflection		2

26" Küschall rear wheels complete

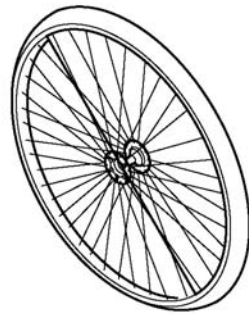


	no tyre		
	1	2	3
	4	5	6
	7	8	9



	no tyre		
		10	11
		12	13
		14	15


Pos.	Part number	Description	Valid from → until	Qty
1	904.1079	Rear wheel 26", standard hub, without handrim, without tyre		2
2	904.1184	Rear wheel 26", Standard hub, without handrim, light wall		2
3	904.1176	Rear wheel 26", standard hub, without handrim, Indoor sports		2
4	904.1083	Rear wheel 26", Starec hub R, without handrim, without tyre		2
4	904.1083.950	Rear wheel 26" Starec R, tecno, without handrim, without tyre		2
4	904.1083.951	Rear wheel 26" Starec R, lemon, without handrim, without tyre		2
4	904.1083.954	Rear wheel 26" Starec R, apricot, without handrim, without tyre		2
4	904.1083.957	Rear wheel 26" Starec R, b-blue, without handrim, without tyre		2
4	904.1083-0440	Rear wheel 26" Starec R, comet red, without handrim, without tyre		2
4	904.1083-0441	Rear wheel 26" Starec R, arctic blue, without handrim, without tyre		2
4	904.1083-0444	Rear wheel 26" Starec R, copper glaze, without handrim, without tyre		2
5	904.1196	Rear wheel 26", Starec hub R, without handrim, light wall		2
6	904.1165	Rear wheel 26", Starec hub R, without handrim, indoor sports		2
7	904.1133	Rear wheel 26", Starec hub X, without handrim, without tyre		2
7	904.1133.950	Rear wheel 26" Starec X Tecno without handrim, without tyre		2
7	904.1133.951	Rear wheel 26" Starec X Lemon without handrim, without tyre		2
7	904.1133.954	Rear wheel 26" Starec X Apricot without handrim, without tyre		2
7	904.1133.957	Rear wheel 26" Starec X B-blue without handrim, without tyre		2
7	904.1133-0440	Rear wheel 26" Starec X Comet red without handrim, without tyre		2
7	904.1133-0441	Rear wheel 26" Starec X arctic blue without handrim, without tyre		2
7	904.1133-0444	Rear wheel 26" Starec X copper glaze without handrim, without tyre		2
8	904.1190	Rear wheel 26", Starec hub X, without handrim, light wall		2
9	904.1146	Rear wheel 26", Starec hub X, without handrim, Indoor sports		2
10	904.1183	Rear wheel 26", Standard hub, handrim, light wall		2
11	904.1088	Rear wheel 26", standard hub, handrim, indoors tyre		2
12	904.1193	Rear wheel 26", Starec hub R, handrim, light wall		2
13	904.1155	Rear wheel 26", Starec hub R, handrim, Indoor sports		2
14	904.1187	Rear wheel 26", Starec hub X, handrim, light wall		2
15	904.1129	Rear wheel 26", Starec hub X, handrim, Indoor sports		2

26" Küschall rear wheel accessories

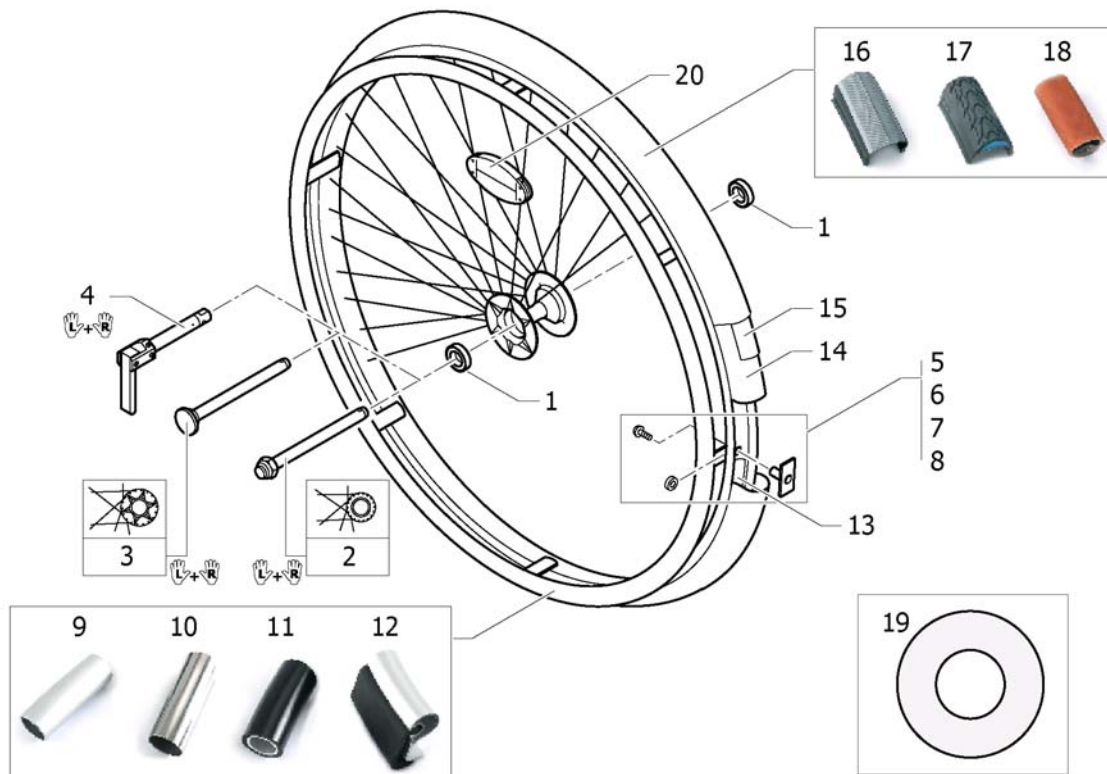

Pos.	Part number	Description	Valid from → until	Qty
1	270.1003	R8-2RS 1/2"/1.1/8"x5/16"		4
2	1512031	Quick release axles long, pair		1
3	1509137-0407	Quick release axles long with cap, pair, colourless		1
3	1509137-0400	Quick release axles long with cap, pair, tecno anthracite		1
3	1509137-0401	Quick release axles long with cap, pair, lemon		1
3	1509137-0402	Quick release axles long with cap, pair, apricot		1
3	1509137-0404	Quick release axles long with cap, pair, b-blue		1
3	1509137-0440	Quick release axles long with cap, pair, comet red		1
3	1509137-0441	Quick release axles long with cap, pair, arctic blue		1
3	1509137-0444	Quick release axles long with cap, pair, copper glaze		1
4	1493755	Tetra clip with quick release axle, pair, long		1
5	1495958	Fixation kit handrim, 3 cm		2
6	1495959	Fixation kit handrim, 4 cm		2
7	1495960	Fixation kit handrim, 3 cm with spokes guard		2
8	1495961	Fixation kit handrim, 4 cm with spokes guard		2
9	457.1043	Handrim 26" Alu		2
10	457.1010	Handrim 26" steel polished		2
11	457.1046.429	Handrim 26" Super Grip		2
12	1436971	Handrim 26" Tetra		2
13	726.1003	Rim protection 26"		2
14	727.1003	Inner Tube 26" x 1"		2
15	1424917	Rear wheel inset anti-flat for 20,22,24,26" tyre		2
16	728.1007	Tyre Lightwall 26"		2
17	1517977	Tyre 26" Marathon Evolution		2
18	729.1002	Indoor sports tyre 26" / 10 bar		2
19	736.1003	Spokes guard Transparent 26"		2
20	220.1033	Spokereflection		2

High Performance wheel

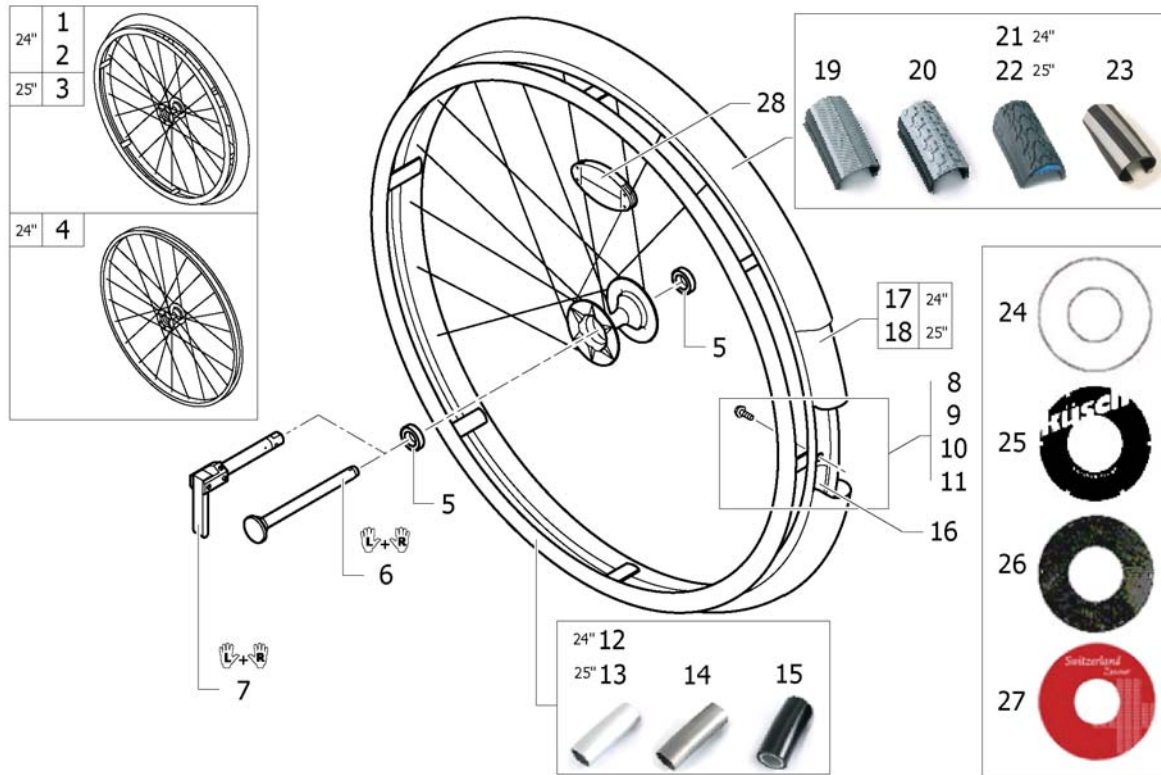


Pos.	Part number	Description	Valid from → until	Qty
1	1420958	Wheel 24", High Performance, colourless, handrim alu, tyre light wall		2
1	1420958-0004	Wheel 24", High Performance, black, handrim alu, tyre light		2
1	1420958-0400	Wheel 24", High Performance, techno-anthracite, handrim alu, tyre light		2
1	1420958-0401	Wheel 24", High Performance, lemon, handrim alu, tyre light wall		2
1	1420958-0402	Wheel 24", High Performance, apricot, handrim alu, tyre light wall		2
1	1420958-0404	Wheel 24", High Performance, b-blue, handrim alu, tyre light wall		2
1	1420958-0440	Wheel 24", High Performance, comet red, handrim alu, tyre light wall		2
1	1420958-0441	Wheel 24", High Performance, arctic blue, handrim alu, tyre light wall		2
1	1420958-0444	Wheel 24", High Performance, copper glaze, handrim alu, tyre light wall		2
2	1527046	Wheel 24", High Performance, colourless, handrim alu, tyre Marathon Evolution		2
2	1527046-0004	Wheel 24", High Performance, black, handrim alu, tyre Marathon Evolution		2
2	1527046-0400	Wheel 24", High Performance, techno-anthracite, handrim alu, tyre Marathon Evolution		2
2	1527046-0401	Wheel 24", High Performance, lemon, handrim alu, tyre Marathon Evolution		2
2	1527046-0402	Wheel 24", High Performance, apricot, handrim alu, tyre Marathon Evolution		2
2	1527046-0404	Wheel 24", High Performance, b-blue, handrim alu, tyre Marathon Evolution		2
2	1527046-0440	Wheel 24", High Performance, comet red, handrim alu, tyre Marathon Evolution		2
2	1527046-0441	Wheel 24", High Performance, arctic blue, handrim alu, tyre Marathon Evolution		2
2	1527046-0444	Wheel 24", High Performance, copper glaze, handrim alu, tyre Marathon Evolution		2
3	1445479	Wheel 25", High Performance, colourless, handrim alu, tyre Marathon Evolution		2
3	1445479-0004	Wheel 25", High Performance, black, handrim alu, tyre Marathon Evolution		2
3	1445479-0400	Wheel 25", High Performance, techno-anthracite, handrim alu, tyre Marathon Evolution		2

High Performance wheel


Pos.	Part number	Description	Valid from → until	Qty
3	1445479-0401	Wheel 25", High Performance, lemon, handrim alu, tyre Marathon Evolution		2
3	1445479-0402	Wheel 25", High Performance, apricot, handrim alu, tyre Marathon Evolution		2
3	1445479-0404	Wheel 25", High Performance, b-blue, handrim alu, tyre Marathon Evolution		2
3	1445479-0440	Wheel 25", High Performance, comet red, handrim alu, tyre Marathon Evolution		2
3	1445479-0441	Wheel 25", High Performance, arctic blue, handrim alu, tyre Marathon Evolution		2
3	1445479-0444	Wheel 25", High Performance, copper glaze, handrim alu, tyre Marathon Evolution		2
4	1517723-0004	Wheel 24", High Performance, black, without handrim, without tyre		2
4	1517723-0407	Wheel 24", High Performance, colourless, without handrim, without tyre		2
4	1517723-0400	Wheel 24", High Performance, techno-anthracite, without handrim, without tyre		2
4	1517723-0401	Wheel 24", High Performance, lemon, without handrim, without tyre		2
4	1517723-0402	Wheel 24", High Performance, apricot, without handrim, without tyre		2
4	1517723-0404	Wheel 24", High Performance, b-blue, without handrim, without tyre		2
4	1517723-0440	Wheel 24", High Performance, comet red, without handrim, without tyre		2
4	1517723-0441	Wheel 24", High Performance, arctic blue, without handrim, without tyre		2
4	1517723-0444	Wheel 24", High Performance, copper glaze, without handrim, without tyre		2
5	270.1003	R8-2RS 1/2"/1.1/8"x5/16"		4
6	1509137-0004	Quick release axles long with cap, pair, black		1
6	1509137-0407	Quick release axles long with cap, pair, colourless		1
6	1509137-0400	Quick release axles long with cap, pair, tecno anthracite		1
6	1509137-0401	Quick release axles long with cap, pair, lemon		1
6	1509137-0402	Quick release axles long with cap, pair, apricot		1
6	1509137-0404	Quick release axles long with cap, pair, b-blue		1
6	1509137-0440	Quick release axles long with cap, pair, comet red		1
6	1509137-0441	Quick release axles long with cap, pair, arctic blue		1
6	1509137-0444	Quick release axles long with cap, pair, copper glaze		1
7	1493755	Tetra clip with quick release axle, pair, long		1
8	1496019	Fixation kit handrim for High Performance wheel, 3 cm		2
9	1496020	Fixation kit handrim for High Performance wheel, 4 cm		2
10	1497110	Fixation kit handrim for High Performance wheel, 3 cm with spokes guard		2
11	1497111	Fixation kit handrim for High Performance wheel, 4 cm with spokes guard		2
12	1432287-0005	Handrim 24" colourless anod. light V2003		2
13	1420890-0005	Handrim 26" aluminium light for high performance		2
15	457.1017.429	Handrim 24" Super-Grip		2
14	457.1051	Handrim 24" Titanium		2
16	1495962	Rim Protection for High Performance / Spinergy wheel		2
17	727.1002	Inner Tube 24" x 1"		2
18	1430886	Inner tube 26" for High-Performance wheel		2
19	1424749	Tyre Light wall 24"		2
20	728.1002	Treaded Tyre 24"		2
21	1438521	Tyre 24" Marathon Evolution		2
22	1493769	Tyre 25" Marathon Evolution		2
23	1528649	Tyre 24" Speedrun		2
24	736.1004	Spokes guard Transparent 24"		2
25	736.1002	Spokes guard Küschall, black 24"		2
26	736.1014	Spokes guard Carbon 24"		2
27	1497108	Spokes guard Switzerland 24"		2
28	220.1033	Spokereflection		2

Mountain bike wheel

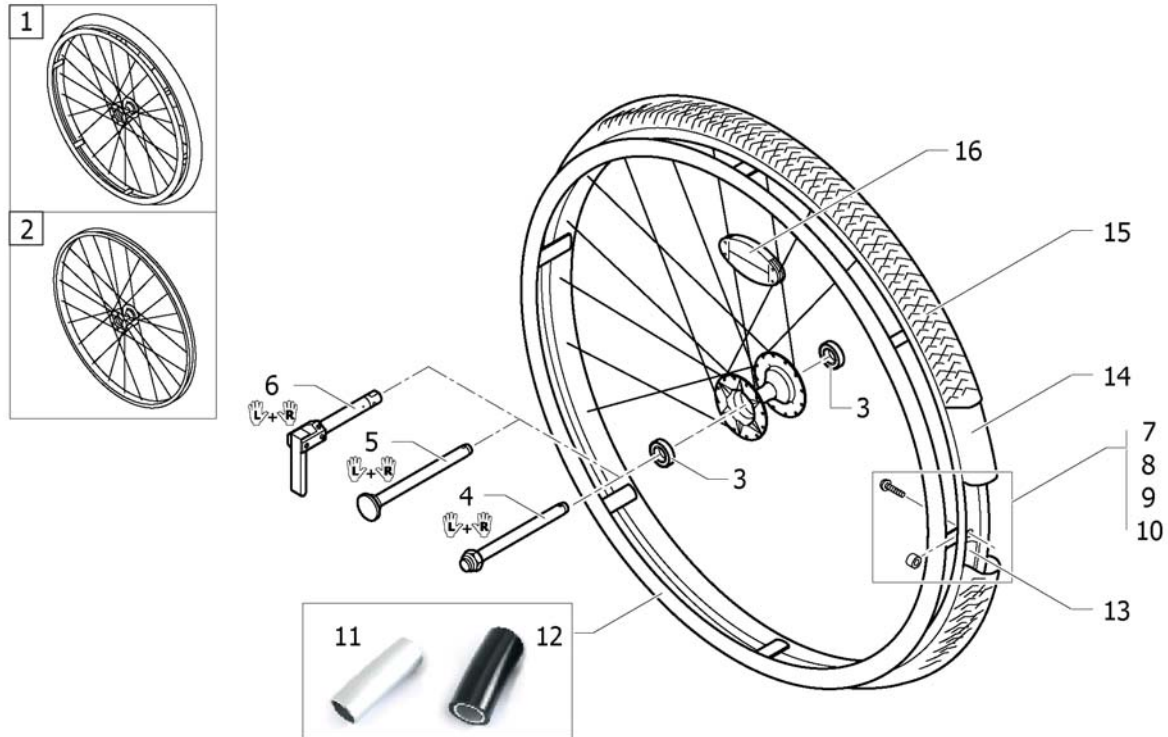


Pos.	Part number	Description	Valid from → until	Qty
1	1517875	Rear wheel 24", Mountain Bike, handrim alu, tyre MTB		2
2	1529327	Rear wheel Mountain Bike 24", without handrim, without tyre		2
3	270.1003	R8-2RS 1/2"/1.1/8"x5/16"		4
4	1512031	Quick release axles long, pair		1
5	1509137-0407	Quick release axles long with cap, pair, colourless		1
6	1493755	Tetra clip with quick release axle, pair, long		1
7	1496015	Fixation kit handrim for Mountain bike wheel, 3 cm		2
8	1496016	Fixation kit handrim for Mountain bike wheel, 4 cm		2
9	1496017	Fixation kit handrim for Mountain bike wheel, 3 cm with spokes guard		2
10	1496018	Fixation kit handrim for Mountain bike wheel, 4 cm with spokes guard		2
11	457.1023	Handrim 22" Alu		2
12	457.1045.429	Handrim 22" Super-Grip		2
13	1420881	Rim protection for Mountain bike		2
14	1420884	Inner tube for Mountain bike		2
15	1420887	Tyre Mountain bike, 24"		2
16	220.1033	Spokereflection		2

Spider wheel

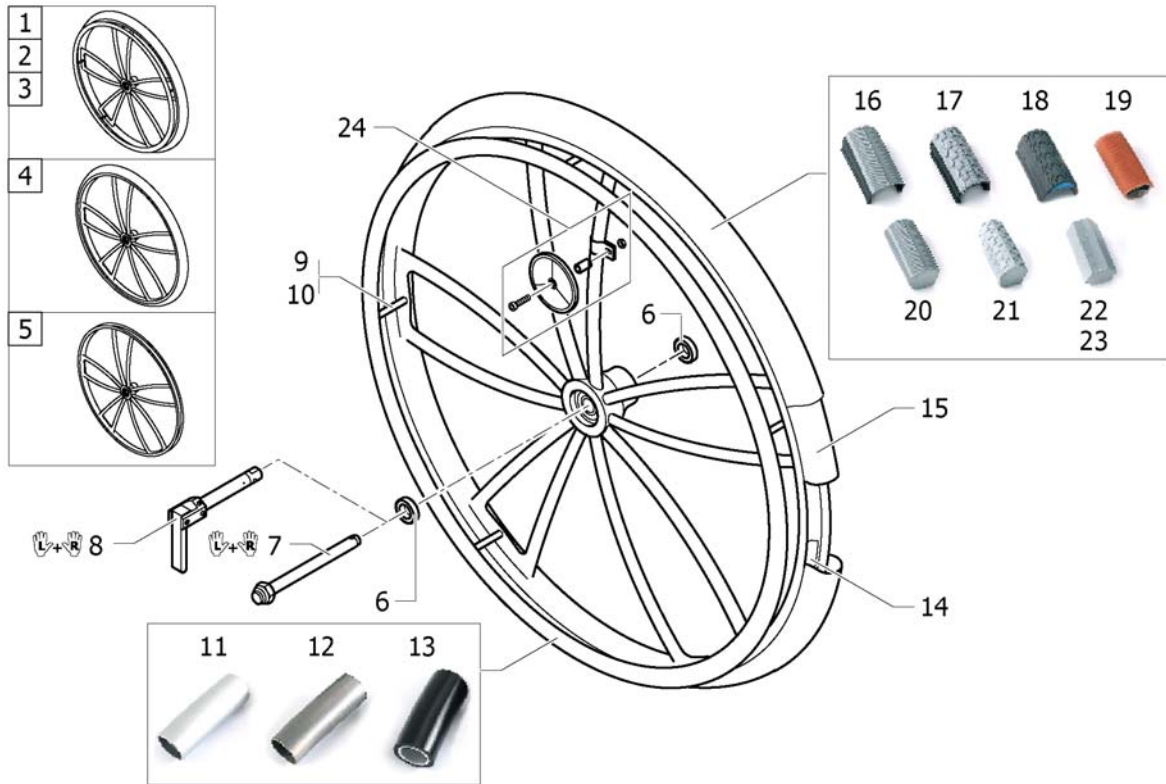


Pos.	Part number	Description	Valid from → until	Qty
1	904.1201.000	Rear wheel 24", Spiderwheel, handrim alu, tyre light wall		2
2	1416977	Rear wheel 24", Spider wheel, handrim Titan, tyre light wall		2
3	1416978	Rear wheel 24", Spider wheel, handrim Super-Grip, tyre light wall		2
4	1420961	Rear wheel 24", Spiderwheel, without handrim, tyre light wall		2
5	1424944	Rear wheel 24", Spiderwheel without handrim, without tyre		2
6	1416804	Bearing for Spider wheel		4
7	1512031	Quick release axles long, pair		1
8	1493755	Tetra clip with quick release axle, pair, long		1
9	1418931	Fixation kit handrim for Spider wheel, 3 cm		2
10	1418932	Fixation kit handrim for Spider wheel, 4 cm		2
11	457.1053.000	Handrim 24" Alu for Spider wheel		2
12	1416801	Handrim 24" Titan for Spider Wheel		2
13	1416802	Handrim 24" Super-Grip for Spider wheel		2
14	726.1008.000	Rim protection 24" Large		2
15	727.1002	Inner Tube 24" x 1"		2
16	1424749	Tyre Light wall 24"		2
17	728.1002	Treaded Tyre 24"		2
18	1438521	Tyre 24" Marathon Evolution		2
19	729.1001	Indoor sports tyre 24" / 10 bar		2
20	728.1032	Solid tyre 24", light, grey		2
21	728.1029	Solid tyre 24", profile		2
22	728.1015	KIK-Type 24", black, fail-safe		2
23	728.1023	KIK-Type 24", grey, fail-safe		2
24	1439352	Spoke reflection set for Spider wheel		2

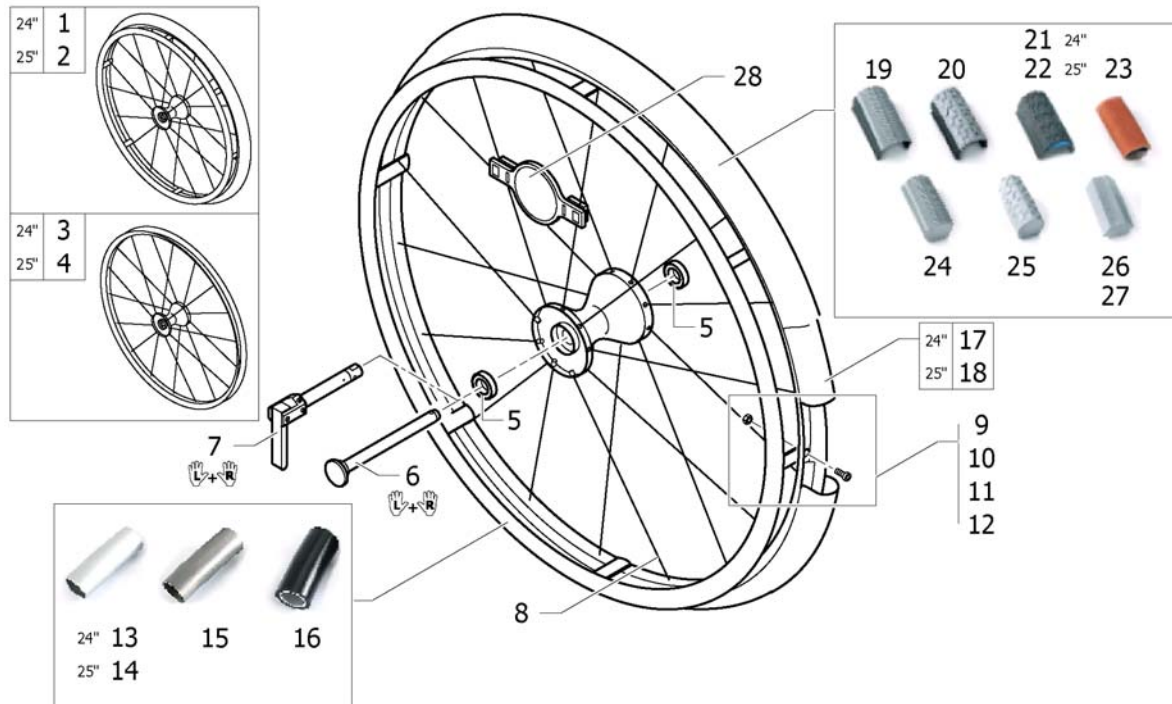

Pos.	Part number	Description	Valid from → until	Qty
1	1485805	Rear wheel 24", Spinergy, handrim Alu, tyre light wall		2
2	1525127	Rear wheel 25", Spinergy, handrim Alu, tyre Marathon Evolution		2
3	1438747	Rear wheel 24", Spinergy, without handrim, without tyre		2
4	1525128	Rear wheel 25", Spinergy, without handrim, without tyre		2
5	270.1003	R8-2RS 1/2"/1.1/8"x5/16"		4
6	1507627	Quickrelease axle long with cap for Spinergy wheel, pair		1
7	1512029	Tetra clip with quick release axle, long, for Spinergy wheel, pair		1
8	1493648	Spoke for Spinergy wheel 24"		36
9	1495967	Fixation kit handrim for Spinergy wheel, 3 cm		2
10	1495968	Fixation kit handrim for Spinergy wheel, 4 cm		2
11	1495969	Fixation kit handrim for Spinergy wheel, 3 cm with spokes guard		2
12	1495970	Fixation kit handrim for Spinergy wheel, 4 cm with spokes guard		2
13	1432287-0005	Handrim 24" colourless anod. light V2003		2
14	1420890-0005	Handrim 26" aluminium light for high performance		2
15	457.1051	Handrim 24" Titanium		2
16	457.1017.429	Handrim 24" Super-Grip		2
17	727.1002	Inner Tube 24" x 1"		2
18	1430886	Inner tube 26" for High-Performance wheel		2
19	1424749	Tyre Light wall 24"		2
20	728.1002	Treaded Tyre 24"		2
21	1438521	Tyre 24" Marathon Evolution		2
22	1493769	Tyre 25" Marathon Evolution		2
23	729.1001	Indoor sports tyre 24" / 10 bar		2
24	728.1032	Solid tyre 24", light, grey		2
25	728.1029	Solid tyre 24", profile		2
26	728.1015	KIK-Type 24", black, fail-safe		2
27	728.1023	KIK-Type 24", grey, fail-safe		2
28	1451931	Spoke reflection for Spinergy wheel		2

Drumbrake wheel


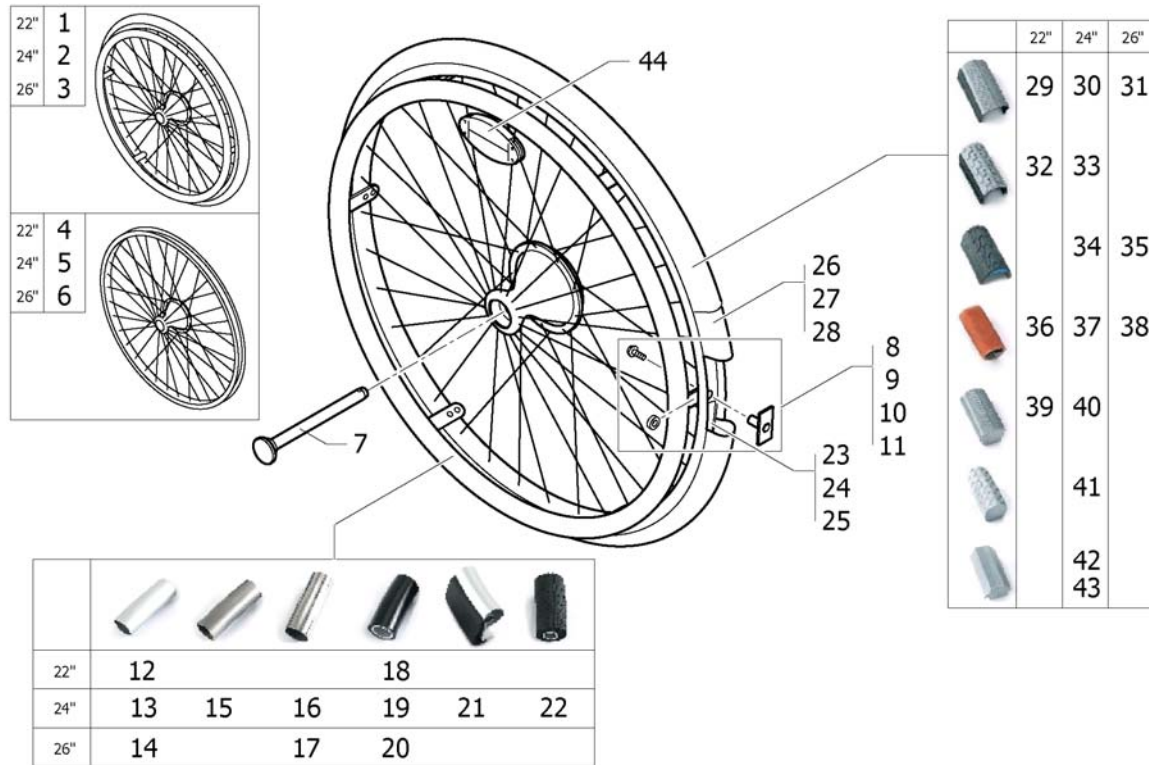


Pos.	Part number	Description	Valid from → until	Qty
1	1445495	Rear wheel 22" Drumbrake Octopus, handrim, tyre light wall		2
2	1445496	Rear wheel 24" Drumbrake Octopus, handrim, tyre light weight		2
3	1445497	Rear wheel 26" Drumbrake Octopus, handrim, tyre light weight		2
4	1445492	Rear wheel 22" Drumbrake Octopus, without handrim, without tyre		2
5	1445493	Rear wheel 24" Drumbrake Octopus, without handrim, without tyre		2
6	1445494	Rear wheel 26" Drumbrake Octopus, without handrim, without tyre		2
7	1438987	Quickrelease axle for drum brake		2
8	1495958	Fixation kit handrim, 3 cm		2
9	1495959	Fixation kit handrim, 4 cm		2
10	1495960	Fixation kit handrim, 3 cm with spokes guard		2
11	1495961	Fixation kit handrim, 4 cm with spokes guard		2
12	457.1023	Handrim 22" Alu		2
13	1432287-0005	Handrim 24" colourless anod. light V2003		2
14	457.1043	Handrim 26" Alu		2
15	457.1051	Handrim 24" Titanium		2
16	457.1009	Handrim 24" steel polished		2
17	457.1010	Handrim 26" steel polished		2
18	457.1045.429	Handrim 22" Super-Grip		2
19	457.1017.429	Handrim 24" Super-Grip		2
20	457.1046.429	Handrim 26" Super Grip		2
21	457.1052.000	Handrim 24" Tetra 2000		2
22	905.5002	Handrim 24" Maxgrip		2
23	726.1005	Rim protection 22"		2
24	726.1008.000	Rim protection 24" Large		2
25	726.1003	Rim protection 26"		2
26	727.1007	Inner Tube 22" x 1"		2
27	727.1002	Inner Tube 24" x 1"		2
28	727.1003	Inner Tube 26" x 1"		2
29	728.1027	Tyre Lightwall 22"		2
30	1424749	Tyre Light wall 24"		2
31	728.1007	Tyre Lightwall 26"		2


Drumbrake wheel

Pos.	Part number	Description	Valid from → until	Qty
32	1493768	Treaded tyre 22"		2
33	728.1002	Treaded Tyre 24"		2
34	1438521	Tyre 24" Marathon Evolution		2
35	1517977	Tyre 26" Marathon Evolution		2
36	729.1006	Indoor sports tyre 22" / 10 bar		2
37	729.1001	Indoor sports tyre 24" / 10 bar		2
38	729.1002	Indoor sports tyre 26" / 10 bar		2
39	728.1031	Solid tyre 22", light, grey		2
40	728.1032	Solid tyre 24", light, grey		2
41	728.1029	Solid tyre 24", profile		2
42	728.1015	KIK-Type 24", black, fail-safe		2
43	728.1023	KIK-Type 24", grey, fail-safe		2
44	220.1033	Spokereflection		2

BRAKES

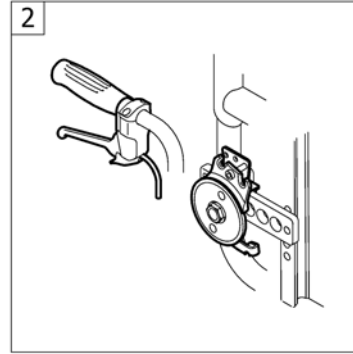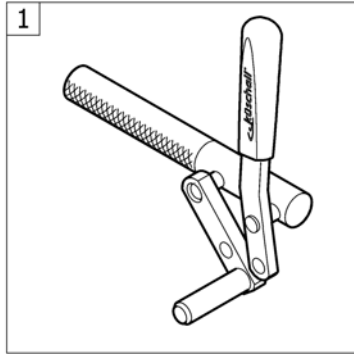


Pos.	Part number	Description	Valid from → until	Qty
1	-	Brakes		1
2	-	Drum brake		1






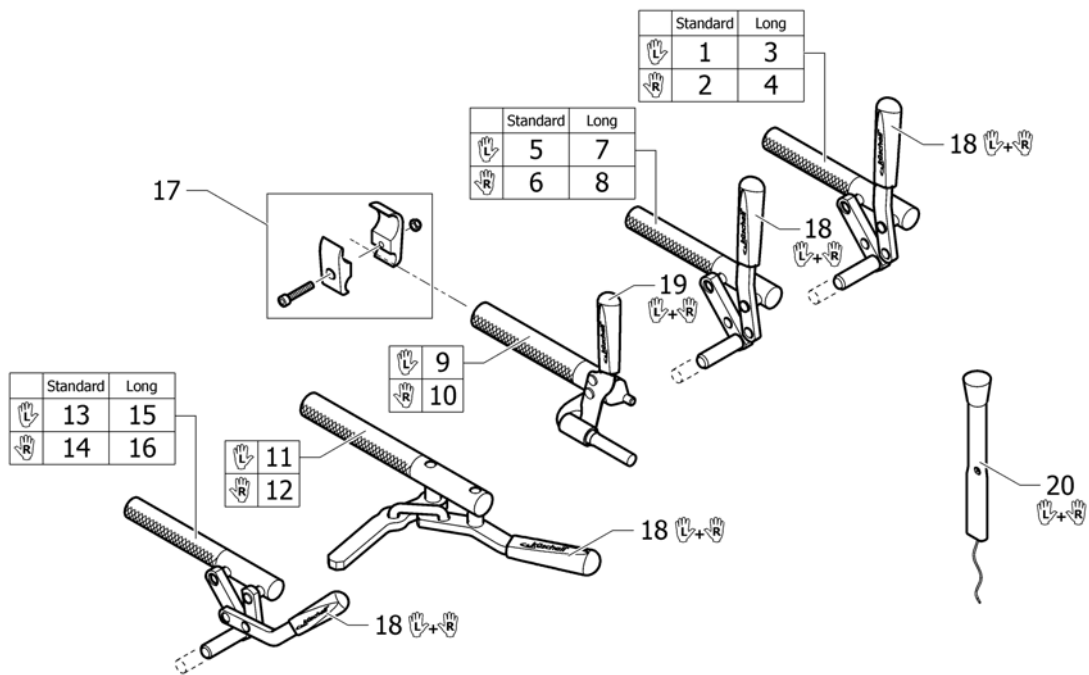


Pos.	Part number	Description	Valid from → until	Qty
1	1512127	Brake, straight lever, left		1
2	1512128	Brake, straight lever, right		1
3	1517658	Brake, straight lever, long shoe, left		1
4	1517657	Brake, straight lever, long shoe, right		1
5	1512125	Brake, bent lever, left		1
6	1512126	Brake, bent lever, right		1
7	1517656	Brake, lever bent, long shoe, left		1
8	1517655	Brake, lever bent, long shoe, right		1
9	1512131	Performance brake left		1
10	1512132	Performance brake right		1
11	1512133	Active brake left		1
12	1512134	Active brake right		1
13	1512129	Pull brake left		1
14	1512130	Pull brake right		1
15	1517660	Pull brake, long shoe left		1
16	1517659	Pull brake, long shoe right		1
17	1513176	Clamping piece to brake, set		2
18	1513177	Brake handle standard, pair		1
19	1513178	Brake handle for performance brake, pair		1
20	1513179	Brake lever extension, pair		1

Drum brake

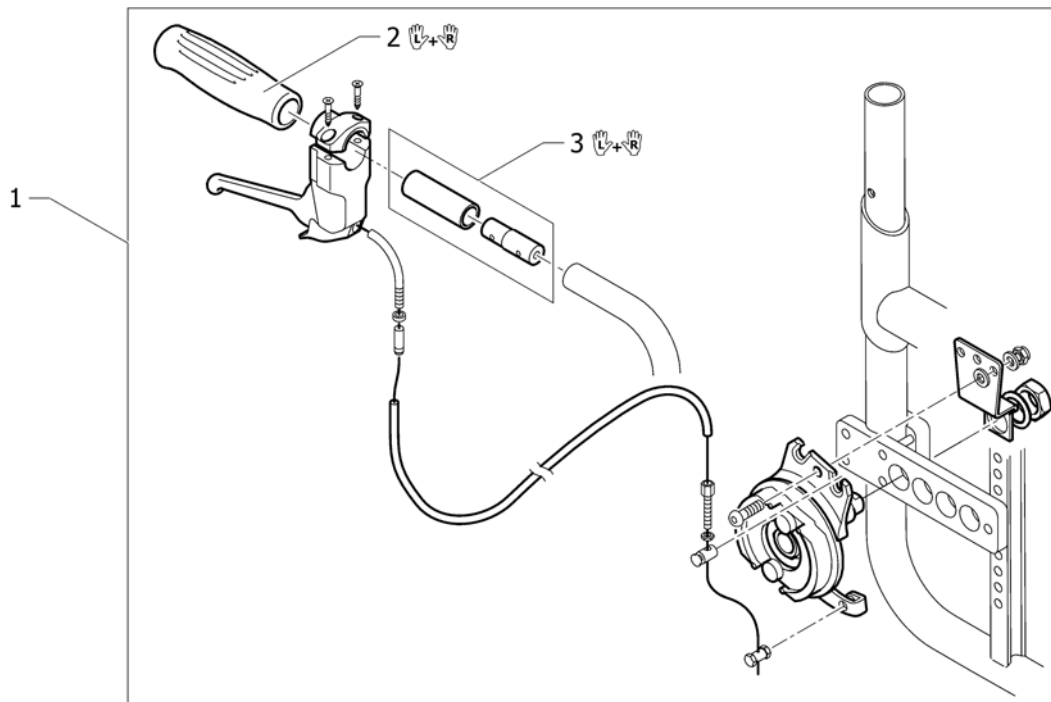


Pos.	Part number	Description	Valid from → until	Qty
1	1532014	Drumbrake set Compact / Ultra Light 09		1
2	1507734	Handle black, pair		1
3	1527108	Push handle extension for drumbrake, pair		1

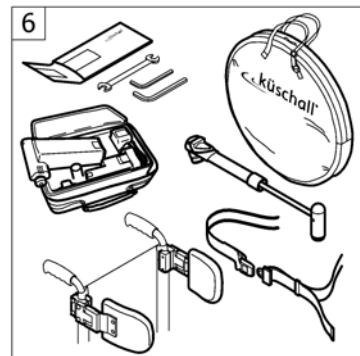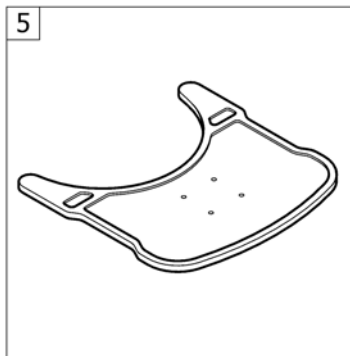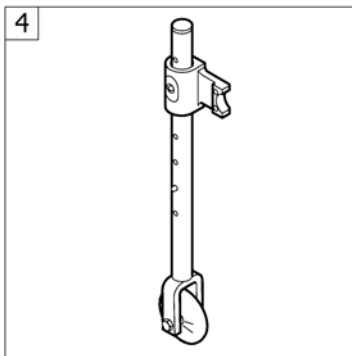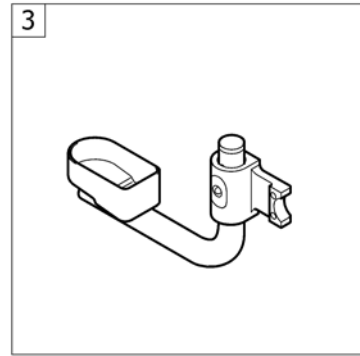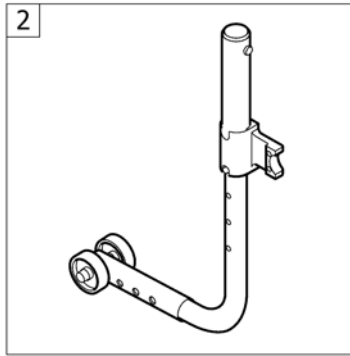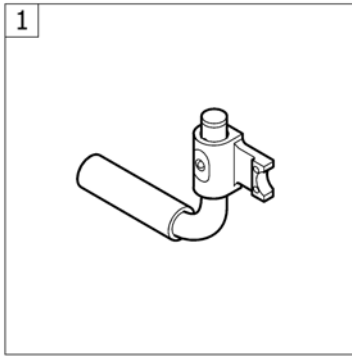
OPTIONS & ACCESSORIES

Pos.	Part number	Description	Valid from → until	Qty
1	-	Tipper aid		1
2	-	Antitipper		1
3	-	Cane holder		1
4	-	Transit wheels		1
5	-	Therapy table		1
6	-	Chair accessories		1
7	-	Winter accessories		1

Tipper aid

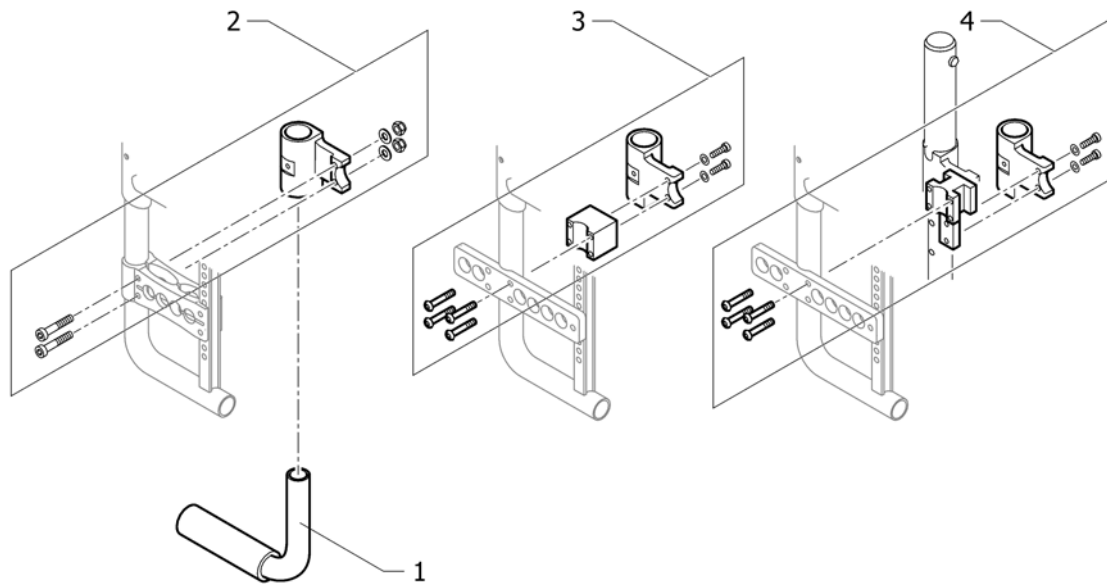


Pos.	Part number	Description	Valid from → until	Qty
1	1438213	Tipper-Aid complete UL/C04		1
2	1532010	Adaptation holder set (standard adapter plate)		1
3	1532011	Adaptation holder kit (adapter plate for rear wheel ext. or drum brake)		2
4	1532012	Adaptation holder kit for 2nd Adaptation (adapter plate for rear wheel ext. or drum brake)		2

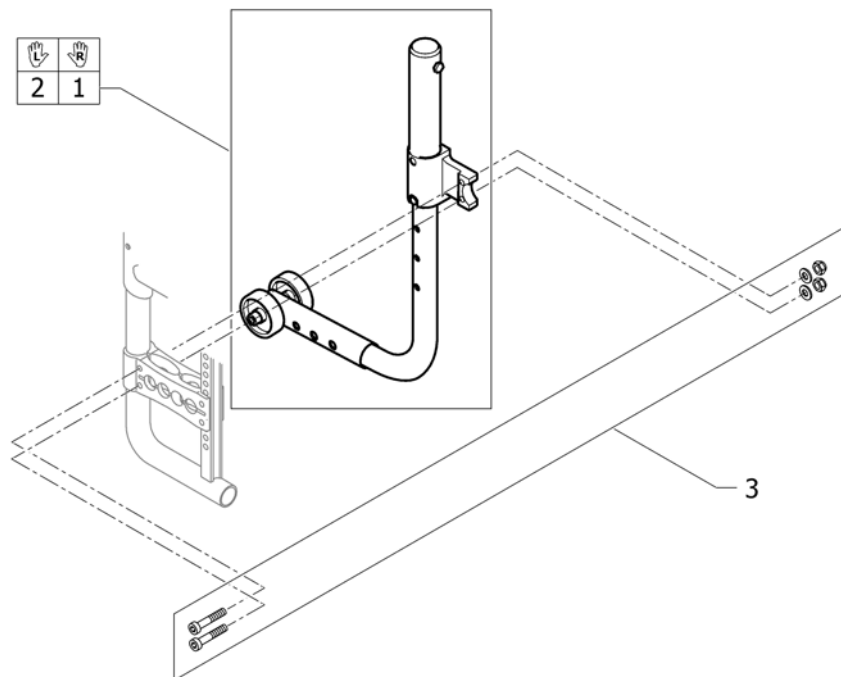


Pos.	Part number	Description	Valid from → until	Qty
1	1438211	Antitipper right complete UL/C04		1
2	1438212	Antitipper left complete UL/C04		1
3	1532013	Fixation kit to Antitipper (standard adapter plate)		1


Cane holder

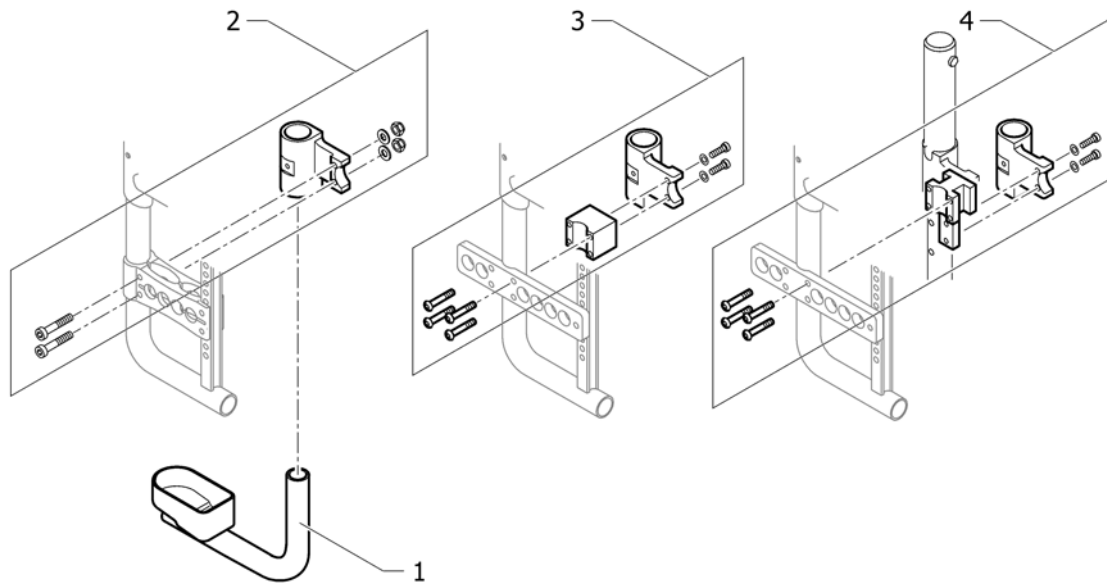

Pos.	Part number	Description	Valid from → until	Qty
1	1438214	cane holder complete UL/C04		1
2	1532010	Adaptation holder set (standard adapter plate)		1
3	1532011	Adaptation holder kit (adapter plate for rear wheel ext. or drum brake)		2
4	1532012	Adaptation holder kit for 2nd Adaptation (adapter plate for rear wheel ext. or drum brake)		2

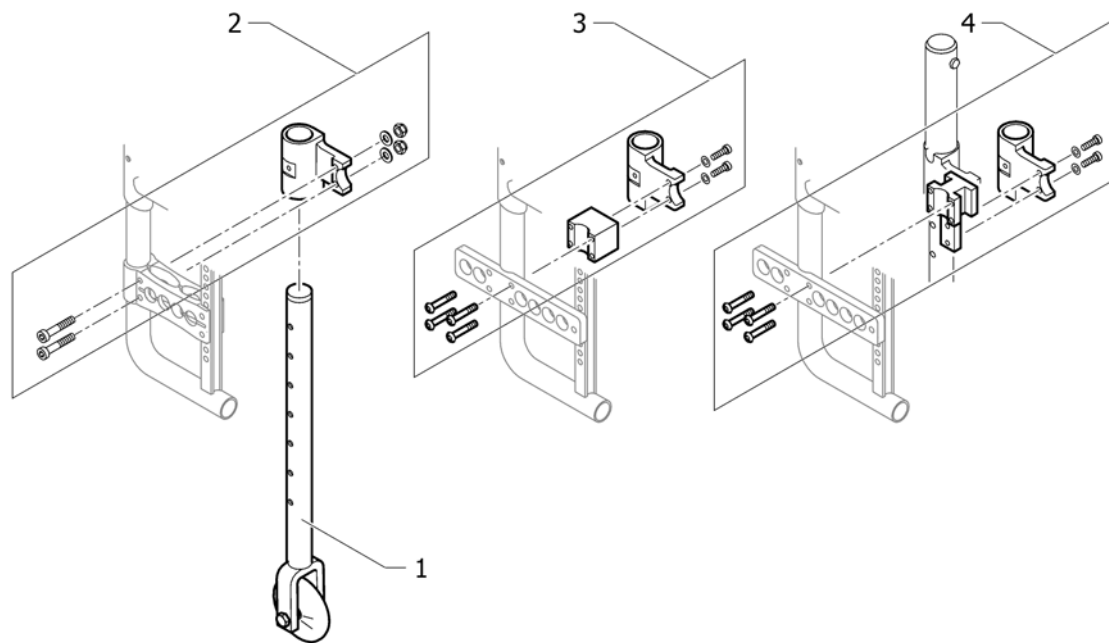

Pos.	Part number	Description	Valid from → until	Qty
1	1438215	pair transit wheels complete UL/C04		1
2	1532010	Adaptation holder set (standard adapter plate)		1
3	1532011	Adaptation holder kit (adapter plate for rear wheel ext. or drum brake)		2
4	1532012	Adaptation holder kit for 2nd Adaptation (adapter plate for rear wheel ext. or drum brake)		2

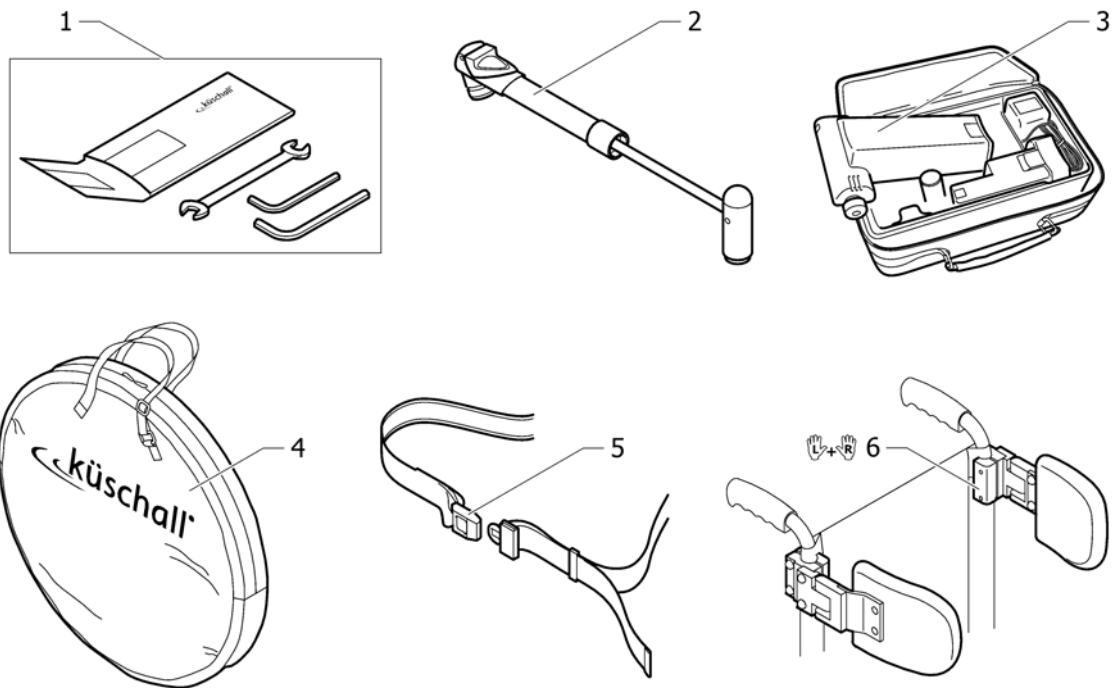Chair accessories

Pos.	Part number	Description	Valid from → until	Qty
1	1518285	Tool Kit		1
2	1517924	Air pump		1
3	1518286	Air man		1
4	1518287	Wheel bag 24" - 26"		1
5	1518288	Seat belt		1
6	1518289	Lateralsupport, pair		1

Winter accessories

Pos.	Part number	Description	Valid from → until	Qty
1	1532007	Seat warmer		1
2	1532008	Feet bag for body height <1,60m		1
3	1532009	Feet bag for body height >1,60m		1



küschall[®] distributors

Belgium & Luxemburg: Invacare nv • Autobaan 22 • B-8210 Loppem
Tel: (32) (0)50 83 10 10 • Fax: (32) (0)50 83 10 11 • belgium@invacare.com

Danmark: Invacare A/S • Sdr. Ringvej 37 • DK-2605 Brøndby
Tel: (45) (0)36 90 00 00 • Fax: (45) (0)36 90 00 01 • denmark@invacare.com

Deutschland: Invacare Aquatec GmbH • Alemannenstraße 10 • D-88316 Isny
Tel: (49) (0)75 62 7 00 0 • Fax: (49) (0)75 62 7 00 66 • info@invacare-aquatec.com

European Distributor Organisation: Invacare • Kleiststraße 49 • D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540 • Fax: (49) (0)57 31 754 541 • edo@invacare.com

España: Invacare SA • c/Areny s/n • Poligon Industrial de Celrà • E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00 • Fax: (34) (0)972 49 32 20 • contactsp@invacare.com

France: Invacare Poirier SAS • Route de St Roch • F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66 • Fax: (33) (0)2 47 42 12 24 • contactfr@invacare.com

Ireland: Invacare Ireland Ltd • Unit 5 Seatown Business Campus • Seatown Road • Swords • County Dublin – Ireland
Tel: (353) 1 810 7084 • fax: (353) 1 810 7085 • ireland@invacare.com

Italia: Invacare Mecc San s.r.l. • Via dei Pini 62 • I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59 • Fax: (39) 0445 38 00 34 • italia@invacare.com

Nederland: Invacare BV • Celsiusstraat 46 • NL-6716 BZ Ede
Tel: (31) (0)318 695 757 • Fax: (31) (0)318 695 758 • nederland@invacare.com

Norge: Invacare AS • Grensesvingen 9 • Postboks 6230 • Etterstad • N-0603 Oslo
Tel: (47) (0)22 57 95 00 • Fax: (47) (0)22 57 95 01 • norway@invacare.com • island@invacare.com

Österreich: Mobitec Mobilitätshilfen GmbH • Herzog Odilostrasse 101 • A-5310 Mondsee
Tel.: (43) 6232 5535 0 • Fax.: (43) 6232 5535 4 • office@mobitec-austria.com • austria@invacare.com

Portugal: Invacare Lda • Rua Estrada Velha • 949 • P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47 • Fax: (351) (0)225 1057 39 • portugal@invacare.com

Sverige & Suomi: Invacare AB • Fagerstagatan 9 • S-163 91 Spånga
Tel: (46) (0)8 761 70 90 • Fax: (46) (0)8 761 81 08 • sweden@invacare.com • finland@invacare.com

Switzerland: Küschall AG • Benkenstrasse 260 • CH-4108 Witterswil
Tel.: (41) (0)61 487 70 80 • Fax.: (41) (0)61 487 70 81 • kueschall@invacare.com

United Kingdom: Invacare Limited • Pencoed Technology Park • Pencoed, Bridgend CF35 5HZ
Tel: (44) (0)1656 776200 • Fax: (44) (0)1656 776201 • UK@invacare.com