


# DOLOMITE SYMPHONY

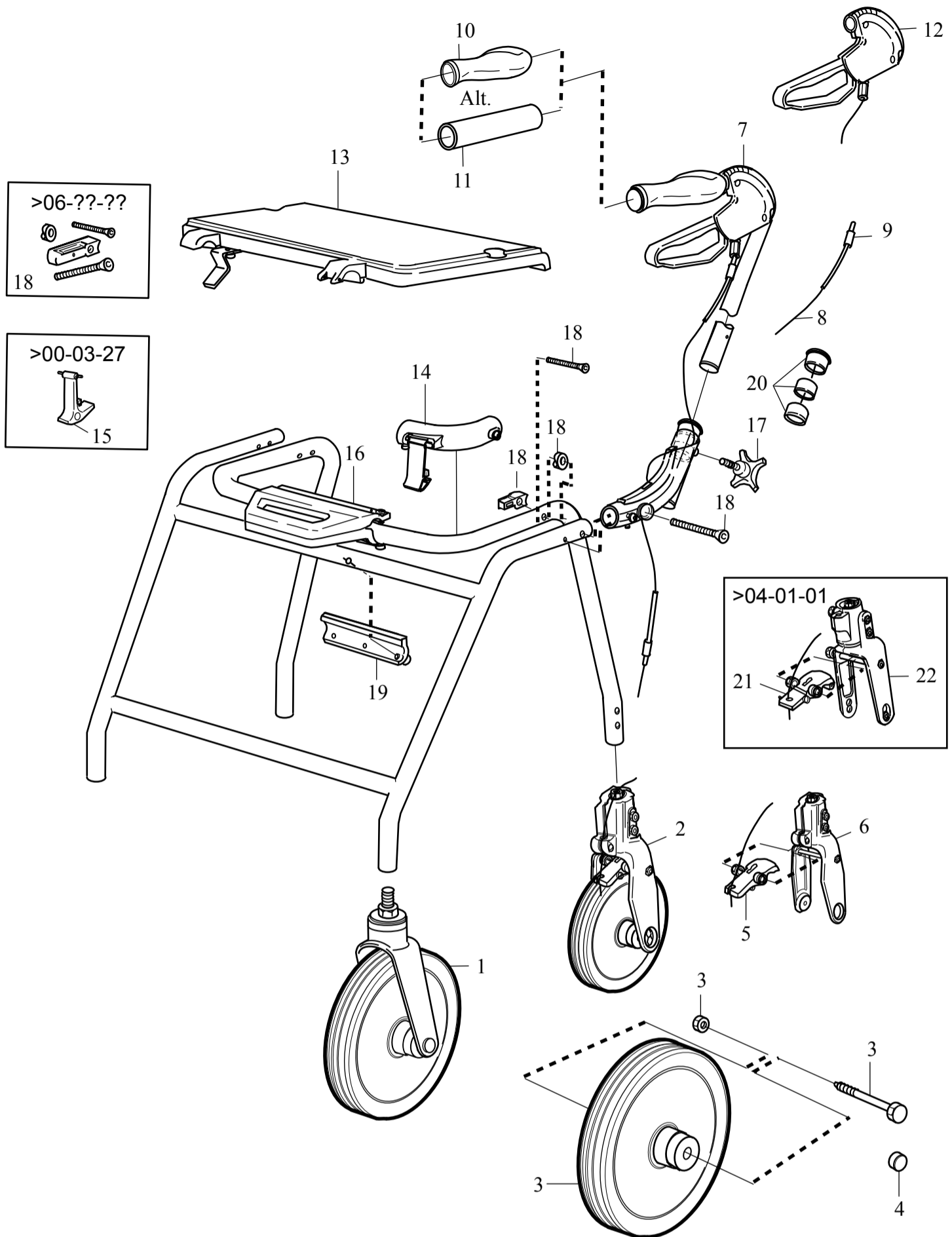

1	13565-25	SLII-wheel in front fork (pcs)
2	12590-25	SL II-wheel complete in plastic rear fork (pcs) DABS II from Jan-04
3	12609	SL II-wheel kit complete (pcs)
4	70050	Cover (pcs)
5	12620	Brake stopper complete DABS II (pcs), grey
6	13096	Rear fork, plastic (pcs) DABS II from January 2004
7	13509-L	Handle kit DABSII, left. Incl. grip and wire (pcs)
	13509-R	Handle kit DABSII, right. Incl. grip and wire (pcs)
8	80068	Wire (pcs)
9	80090	Wire cover L=630 mm (pcs) SYMPHONY 600
	80089	Wire cover L=550 mm (pcs) SYMPHONY 520
	80088	Wire cover L=500 mm (pcs) SYMPHONY 450
10	12519	Anatomic grip incl. Cover (pair)
11	12593	Soft grip, incl. Covers (pair)
12	12619-L	Brake handle kit, left incl. wire (pcs)
	12619-R	Brake handle kit, right incl. wire (pcs)
13	12545	Flip-up seat, compl w. attachment part w/o safety latch (pcs)
14	12656	Safety latch, frame assembly (pcs)
	70202	Safety latch without holder (pcs)
15	12615	Safety latch, seat assembly (pcs)
16	12622	Folding handle kit (pcs)
17	70253	Knob (pcs)
18	1452148	Folding hinge kit (pair) new
	12648	Folding hinge kit (pair)
19	12647-S	Plastic rail for folding mec. incl. Screws (pcs)
20	12687	Components for hinge (pair)
21	12617	Brake flap, complete DABS (pcs) black
22	13093	Rear fork, plastic (pcs) until Dec 2004
	82104	Operating instructions (unit)