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GB [1] Connection for Speed-Stop-Limit switch

A Speed-Stop-Limit switch provides the option to regulate the speed in accordance with the switch position. For example, in the case of wheelchairs with a stand-up and lift function the speed of these two positions can be reduced, or travelling blocked. When the Speed-Stop-Limit switch is pressed for the first time, the speed is reduced to 50% of the maximum speed. If the switch is closed, the e-fix continues to travel at the maximum speed. Your specialist dealer can make a change in the reduced speed.

[2] Interface for connection to external controllers

This is used for communication between the e-fix and external controllers and serves at the same time as the connection to the optional intuitive attendant controller.

[3] 24V power supply

Connection option for a power supply for an external consumer.

Warning:

- **Do not fully discharge the rechargeable batteries through external consumers!**
- **The power supply can not be used to operate live conserving machines!**

[4] Connection for an external on/off switch

Option to switch the e-fix on and off by means of an external button.

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