


# e-fix E35/E36


## Components

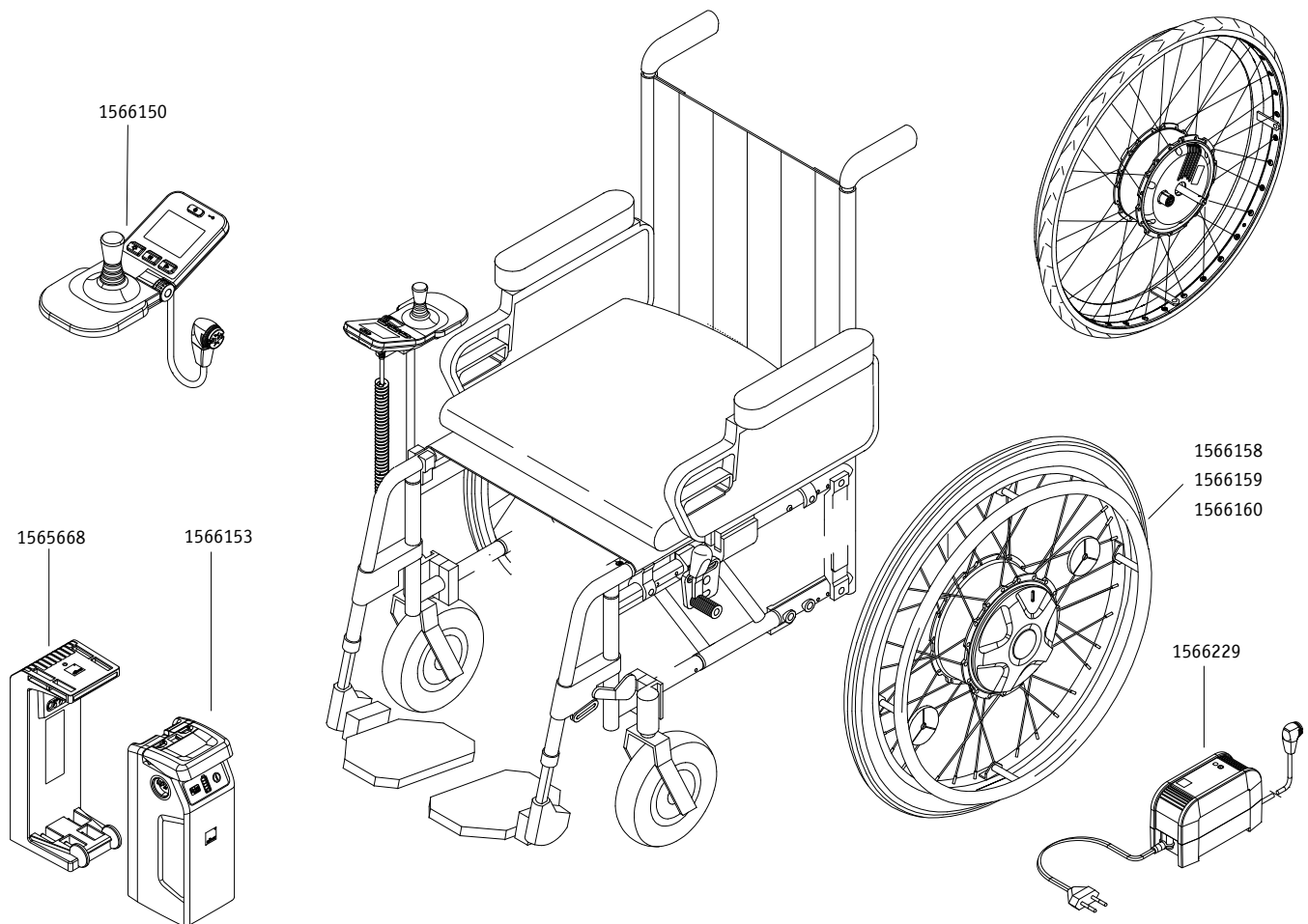

Order number	Description
1565639	Battery pack e-fix E35/E36, 6 Ah
1565639	Battery pack e-fix E35/E36, 6 Ah
1565668	Battery-fixation with interface e-fix E35/E36
1566150	Control unit e-fix E35/E36, complete
1566153	Battery pack e-fix E35/36, 7,5 Ah
1566158	Drive wheel e-fix E35 24"
1566159	Drive wheel e-fix E35 22"
1566160	Drive wheel e-fix E36 24"
1566229	Charger E35 / E36 / V25
1566250	Documentation Kit E35/E36 German
1566251	Documentation Kit E35/E36 multilingual




## Drive wheel 22", 24"

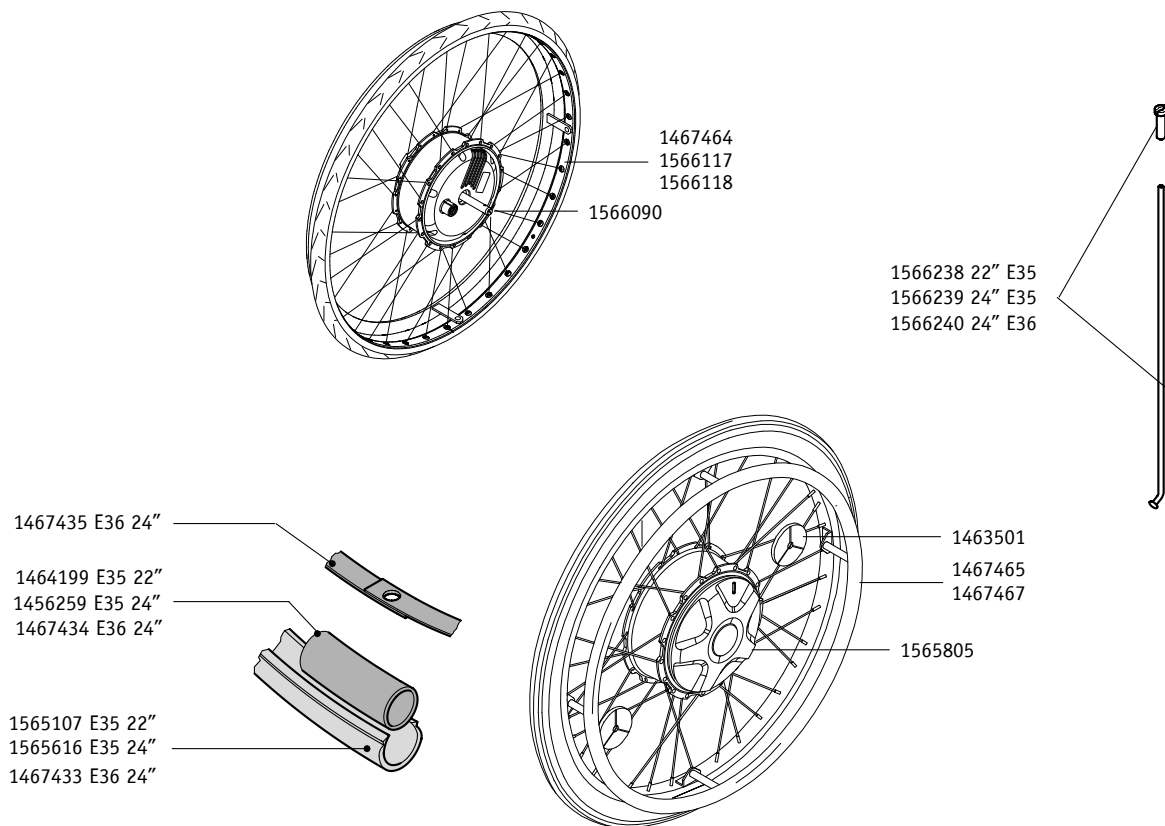

Order number	Description
1456259	Tube (airless) for e-fix 24" x 1 3/8" for E25 24"
1464199	Tube (airless) for e-fix E25 22"
1463501	Reflector e-fix
1467433	Tyre Marathon Plus grey/black 24 x 1 3/8 (37-540)
1467434	Tube with car-type valve 38/47-507/541 (air pressure 5 bar)
1467435	Rim strip Super HP 24" (16-540) pressure resistant
1467464	Rim Tour silver 22" (489-21)
1467465	Push rim stainless steel 24" reinforced, complete
1467467	Push rim stainless steel 22" reinforced, complete
1565107	Tyre C63 grey 22 x 1 3/8 (37-489)
1565616	Tyre Downtown grey/black 24 x 1 3/8 (37-540)
1565805	Cover clutch ring, white
1566090	Stop screw stub axle e-fix
1566117	Rim Tour silver 24", eyed
1566118	Rim Tour black 24", reinforced and eyed
1566238	Spoke set E35 22" Leader
1566239	Spoke set E35 24" Leader
1566240	Spoke set E36 24" Race


## Control unit

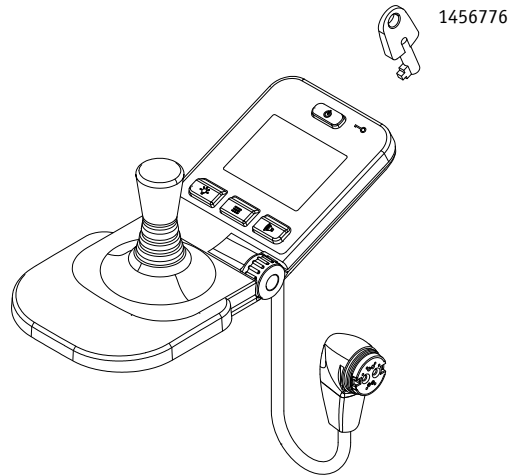

Order number	Description
1456567	Screw Ejot PT K 30 x 8 WN 1451
1456776	Immobiliser key
1459095	Screw for width adjustment X6
1459626	Hexagon nut ISO 4032 - M5 - 6 AU
1463520	Screw Ejot PT K 30 x 14 WN
1463627	Screw Ejot PT K 30 x 12 WN 1423
1520860	Cable for control unit V25 / E35 / E36
1565513	Hand rest for control unit E35/E36
1565514	Headlight glass for control unit E35/E36
1565573	Switch ON/OFF
1565574	Switch MENU
1565575	Gaiter for joystick E35/E36
1565576	Joystickknob standard E35/E36
1565579	Control unit housing lower part E35/E36
1565620	Control unit service window E35/E36
1565634	Cable for joystick E35/E36
1566267	Joystick E35/E36 complete
1566134	Rail for control unit E35/E36
1566241	Housing upper part for control unit e-fix E35/E36




## Battery-Pack / Battery-interface

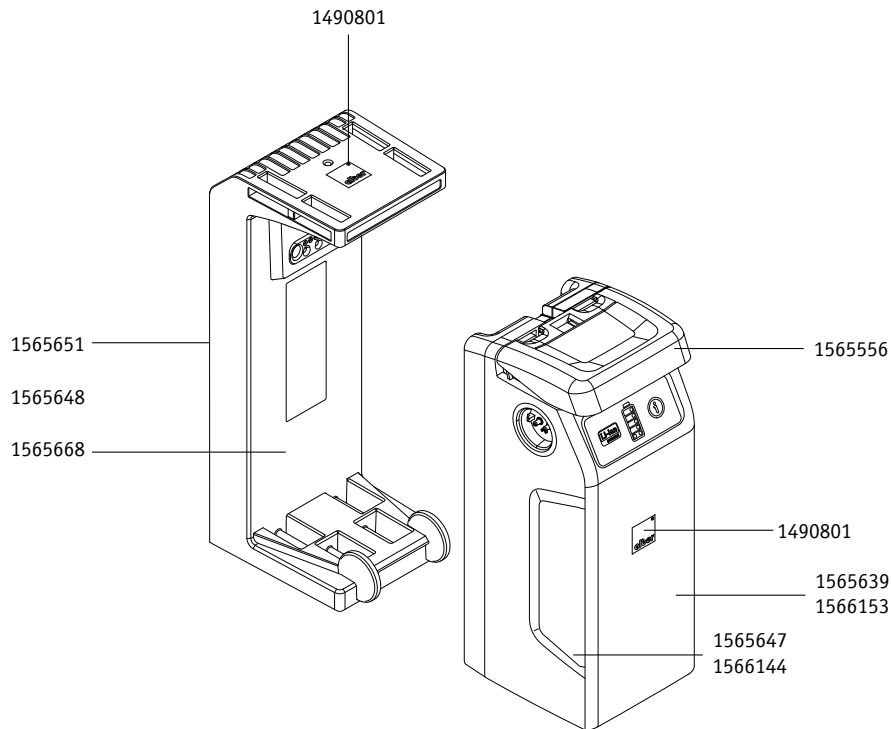

Order number	Description
1490801	Alber Logo 17 x 17mm
1565556	Handle for battery-pack E35/E36
1565639	Battery pack e-fix E35/E36, 6 Ah
1565647	Sticker on handle standard battery-pack 6,0 Ah
1565648	Motor cable E35/E36
1565651	Cover battery-interface with sealing E35 / E36
1565668	Battery-fixation with interface e-fix E35/E36
1566144	Sticker handle powerline battery-pack 7,5 Ah
1566153	Battery pack e-fix E35/36, 7,5 Ah
1566258	Velcro strap kit for e-fix E35 / E36


## Charger

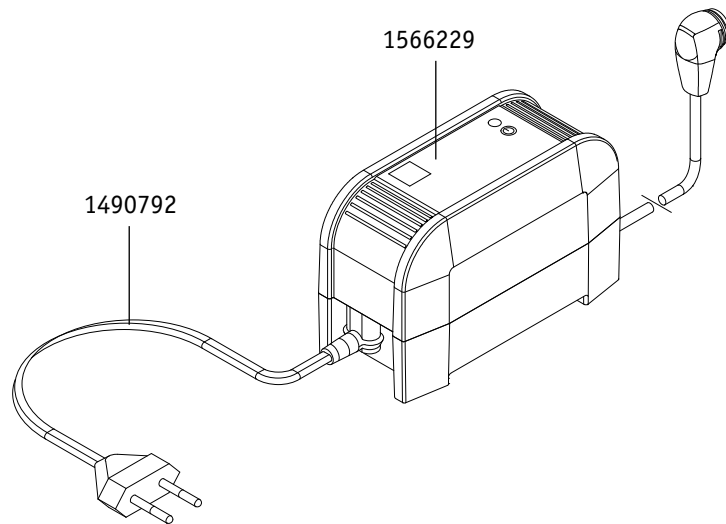

Order number	Description
1490792	Main cable EU for charger, angled
1566229	Charger E35 / E36 / V25




## Brackets

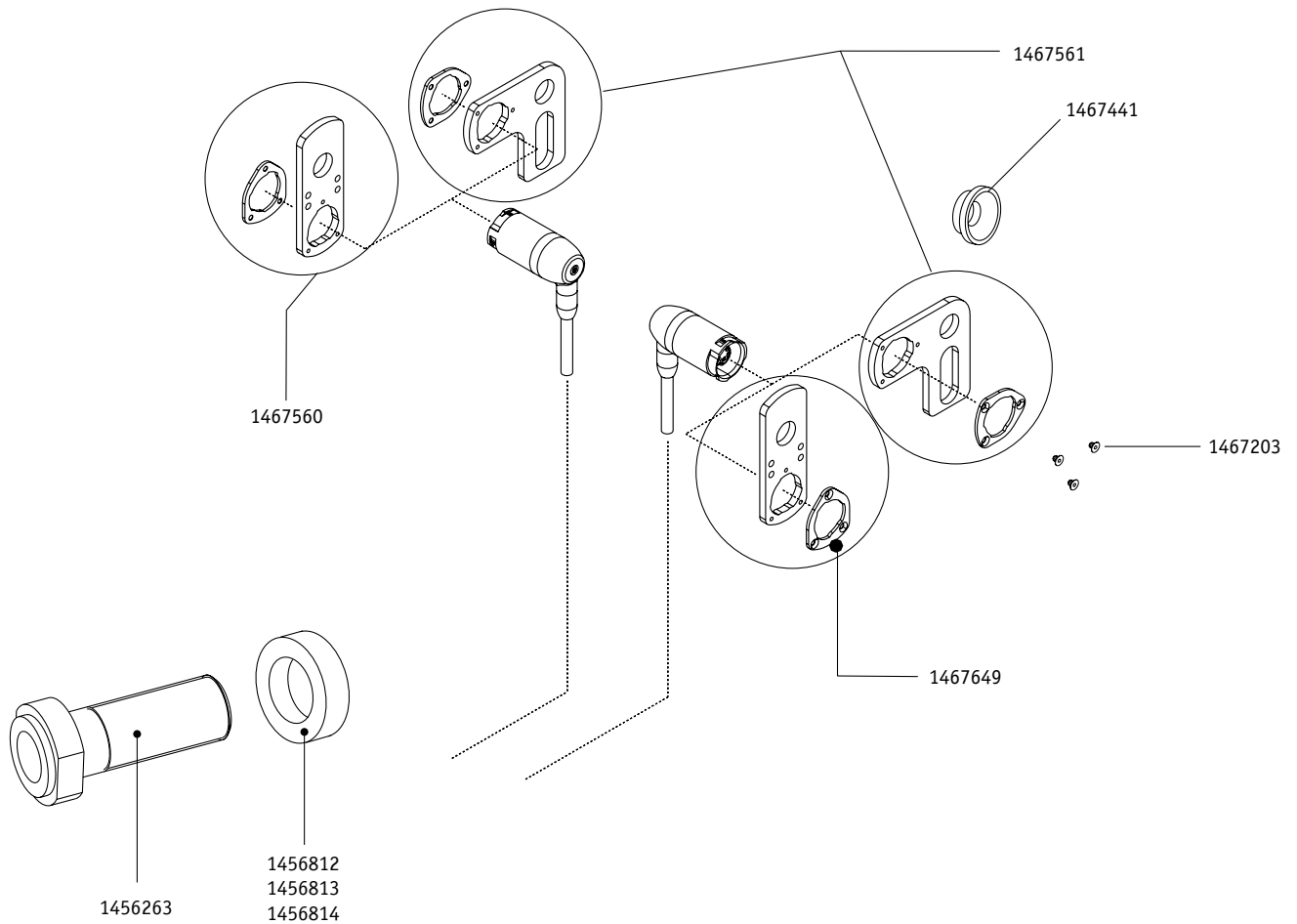

Order number	Description
1456263	Adapter sleeve 1/2" Length 50.8mm with nut
1456812	Spacer ring aluminium 25 x 17 x 10 long
1456813	Spacer ring aluminium 25 x 17 x 8 long
1456814	Spacer ring aluminium 25 x 17 x 5 long
1463520	Screw Ejot PT K 30 x 14 WN
1467203	Countersunk head screw Torx M 3 x 6
1467441	Sliding block e-fix E25/E26
1467560	Installation kit for straight torque receiver e-fix E25/E26
1467561	Installation kit for right-angled torque receiver e-fix E25/E26
1467649	Fixation plate for e-motion/E25/E26

**e-fix E35/E36**


**Anti-tippers with jack-up function**

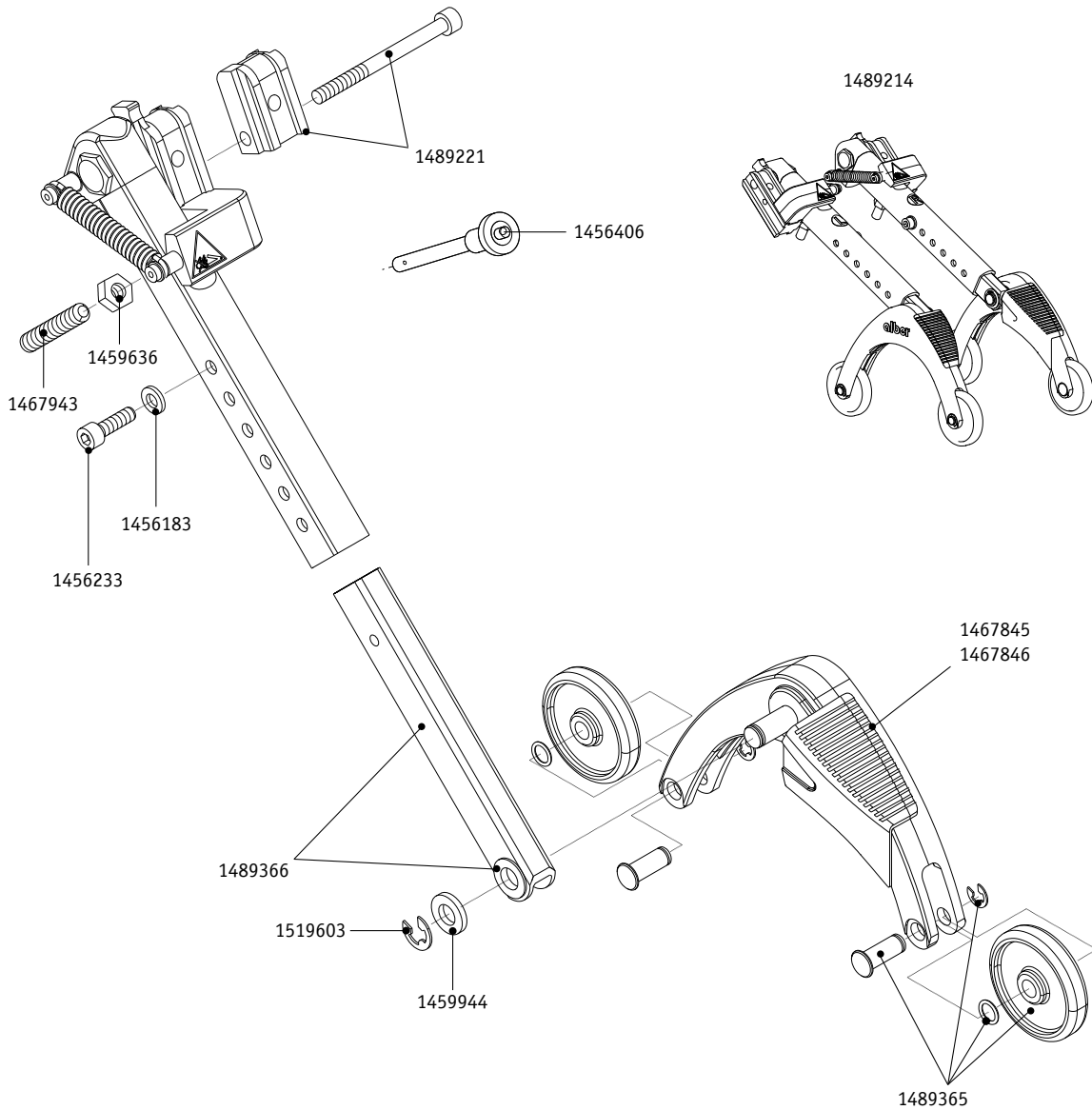

Order number	Description
1456183	Scheibe DIN 125 - A 6,4
1456233	Zylinderschraube DIN 7984 M 6 x 20
1456406	Quickpin Sicherungstift
1459636	Sechskantmutter M8
1459944	Paßscheibe DIN 988 - 10 x 16 x 1
1467845	Stützwinkel rechts für Kippstütze
1467846	Stützwinkel links für Kippstütze
1467943	Gewindestift M 8 x 45
1489214	Kippstützen mit Aufbockfunktion (Paar)
1489221	Aufnahmeschuh-Set 4 x 12mm incl. Schrauben
1489365	Rad für Kippstütze komplett
1489366	Schiebestange Kippstütze montiert
1519603	Sicherungsscheibe DIN 6799 Gr.8

## e-fix E35/E36

### Anti-tippers without jack-up function

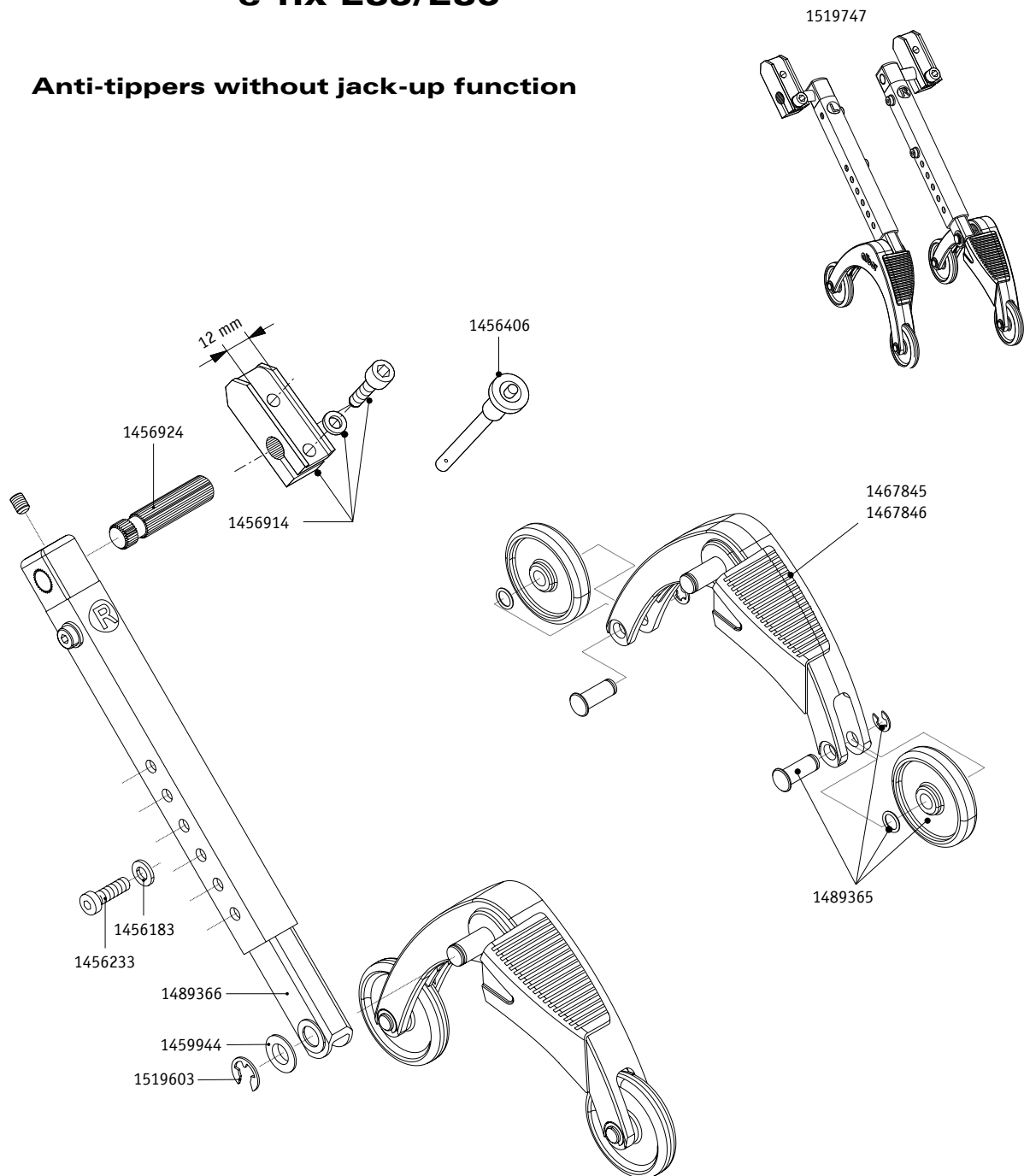

Order number	Description
1456183	Washer DIN 125 - A 6,4
1456233	Cylinder head screw M 6 x 20
1456406	Quickpin quick release bolt
1456914	Fixation for anti-tippers 12mm
1456924	Toothed bar for anti-tippers e-fix
1459944	Washer DIN 988 - 10 x 16 x 1
1467845	Support angle right for anti-tippers
1467846	Support angle left for anti-tippers
1489365	Wheel for anti-tippers
1489366	Slide rod anti-tippers
1519603	Securing disk DIN 6799-Gr.8 vz
1519747	Anti-tippers without jack-up function 12mm