---

**INVACARE® INTERNATIONAL**

**Belgium & Luxembourg:** Autobaan 14 B-8210 Loppem

**Danmark:** Sdr. Ringvej 39 DK-2605 Brøndby

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## General

### SHOWER SEAT

The shower seat provides an aid to washing and the care of patients and residents in wet cells.

The following shower seat models are available:

- Basic model (no backrest); type R8802
- Basic model with backrest; type R8803
- Basic model with backrest and armrests; type R8804
- All basic models are available with optional support leg (option ,60)

### Introduction

We at Invacare are delighted that you have chosen to purchase a quality aid.

A long history of developing and manufacturing bathroom, shower and toilet aids, has enabled Invacare to become a leading company in this field.

These shower seats meet the relevant Essential Requirements from the Medical Devices Directive of 14 June 1993 no. 93/42 EEC.

### Warranty

All Invacare toilet aids come with a 2-year manufacturer's warranty. Normal wear and tear and improper use do not fall under the terms of this warranty.

Invacare does not accept any liability for personal injury which is caused by the improper use of the shower seats.

### General safety

Ensure that the shower seat is installed in keeping with the specifications and drawings attached.

The shower seat should preferably be installed by a qualified workman.

For the installation you should use the R6600 tool set provided by Invacare.

The shower seat can only be mounted on a wall, which has a sufficient load-bearing strength (brickwork or similar). In all other cases you should use the support leg (option ,60), or position the shower seat on a console fixed to the floor (model R8806).

It is also possible to install the shower seat using counter plates type R6630 (only on walls of sufficient thickness) (see figure 3).

In the event of queries or defects, please contact Invacare.

**Please note: When raising / lowering the seat, take care not to trap your fingers in the hinge point.**

### Maintenance

The shower seat and optional support leg (,60) can be rinsed down after use with water and normal household detergents; do not use a scourer or aggressive solvents for this purpose.

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## Installation

### Determining the height

To use the shower seat ergonomically, it should be positioned at the correct height. To determine the user's ideal sitting height, measure the height of the user's knee (see figure 1) and subtract 10 cm. The shower seat can be installed at the height, which maximises the comfort and safety of the user.

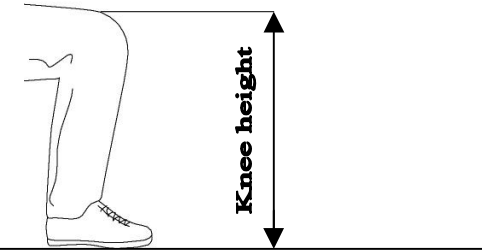
Figure 1. Determining the user's knee height

### Attaching the shower seat to the wall

- Use the drill-template that came with this set.
- Caution : For attaching the type R8802 and R8802,60 just drill the lower three holes for the seat.**
- Drill holes (drill diameter 10mm) in the wall and insert Rawlplugs.
- Screw the shower seat to the wall preferably using the screws in the R6600 tool set (see figure 2).

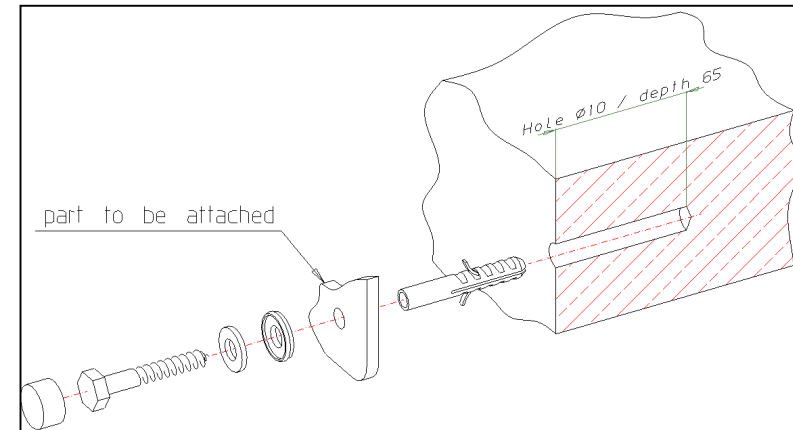
Figure 2. Wall attachment

### Folding the seat

The seat can easily be lowered and raised. When in raised position, the seat should remain firmly in position. If not consult your dealer.


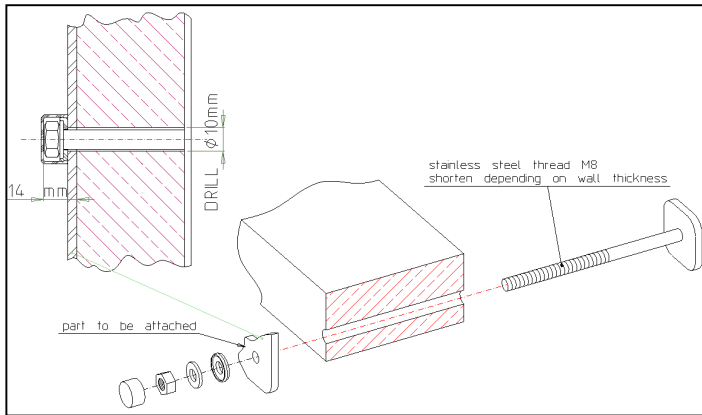
Figure 3. Counter plate attachment R6630

## Technical specifications

- Frame Aluminium anodised.
- Seat and backrest ABS
- Armresttubing St.37 with shockproof and impact-resistant Rilsan coating.
- Polyurethane armrests.
- Maximum capacity: 135 kg.
- Dead weight (basic model)
 

R8802:	4.8 kg
R8802,60:	5.9 kg
R8803:	6.6 kg
R8803,60:	7.7 kg
R8804:	8 kg
R8804,60:	9.1 kg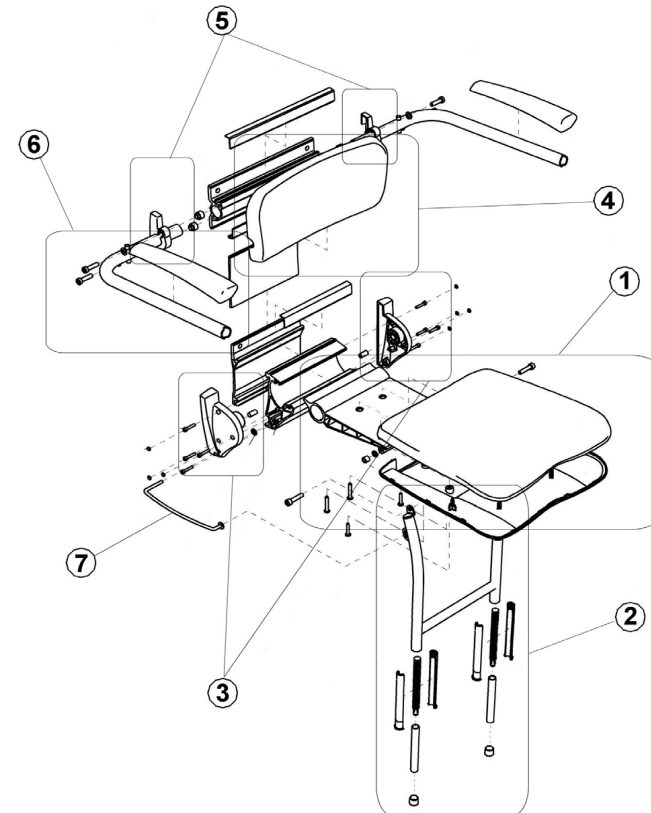

Position	Description	Art. Nr
1	Assembled tray	1426765
2	legs	1426767
3	Side lids	1426766
4	Backrest	3840,1000
5	Subset side lids for backrest	1426840
6	Subset side lids for armrest	1426839
7	Thread of guide	1426841
8	Subset of screws	1426842
9	Subset of caps	1426843