
Operator's Manual

HomeFill® II **Oxygen Conserving** **Cylinder**

HF2PCE6
HF2PCE9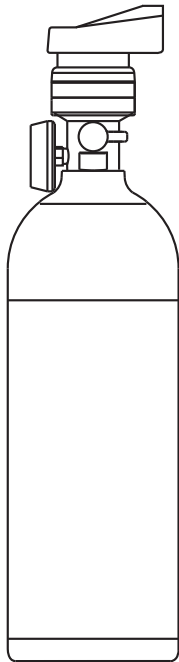

Dealer: This manual
MUST be given to the
end user.

User: Before using this
cylinder, read this
manual and save for
future reference.

**For more information regarding
Invacare products, parts, and services,
please visit www.invacare.com**

Yes, you can.®

⚠ WARNING

DO NOT OPERATE THIS EQUIPMENT WITHOUT FIRST READING AND UNDERSTANDING THIS MANUAL. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS, AND INSTRUCTIONS, CONTACT A HEALTHCARE PROFESSIONAL, DEALER OR A QUALIFIED TECHNICIAN BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE INJURY OR DAMAGE MAY RESULT.

⚠ WARNING

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

⚠ DANGER

DO NOT SMOKE while using this device. Keep all matches, lighted cigarettes or other sources of ignition out of the room in which this product is located and away from where oxygen is being delivered.

Textiles and other materials that normally would not burn are easily ignited and burn with great intensity in oxygen enriched air. Failure to observe this warning can result in severe fire, property damage and cause physical injury or death.

NOTE: The information contained in this document is subject to change without notice.

NOTE: Updated versions of this manual are available on www.invacare.com.

TABLE OF CONTENTS

SPECIAL NOTES	4
RECEIVING INSPECTION	4
INTENDED USE	4
SECTION 1—GENERAL GUIDELINES	6
SECTION 2—FEATURES	10
SECTION 3—TYPICAL PRODUCT PARAMETERS .	11
SECTION 4—INSPECTION AND COMPRESSOR INTERFACE	12
Cylinder Prefill Inspection	12
Connecting/Disconnecting the Cylinder to/from the Compressor.....	16
SECTION 5—OPERATING INSTRUCTIONS	22
SECTION 6—MAINTENANCE	24
Replacement Parts – Accessories.....	24
SECTION 7—CYLINDER FILL TIMES	25
Troubleshooting	25
WARRANTY INFORMATION	26

SPECIAL NOTES

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. Refer to the following table for definitions of the signal words.

SIGNAL WORD	MEANING
DANGER	Danger indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.
WARNING	Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
CAUTION	Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage.

RECEIVING INSPECTION

Remove product from package and inspect for damage. If there is any damage, DO NOT use. Contact your dealer/ Home Care equipment provider.

INTENDED USE


To store, regulate, and deliver supplemental oxygen to patients who may have difficulty extracting oxygen from the air they breathe. It is for patients who would normally receive the oxygen via a nasal cannula. The device delivers USP 93% oxygen at a number of different flow/pulse settings. It is intended to be used as an oxygen saving device that may reduce the drying out of the airways.

This product has been supplied from an environmentally aware manufacturer that complies with the Waste Electrical and Electronic Equipment (WEEE) Directive 2002/96/CE.

This product may contain substances that could be harmful to the environment if disposed of in places (landfills) that are not appropriate according to legislation.

The 'crossed out wheelee bin' symbol is placed on this product to encourage you to recycle wherever possible.

Please be environmentally responsible and recycle this product through your recycling facility at its end of life.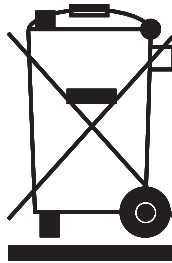

SECTION I—GENERAL GUIDELINES

WARNING

SECTION I - GENERAL GUIDELINES contains important information for the safe operation and use of this product. **DO NOT** use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as **Owner's Manuals, Service Manuals or Instruction Sheets** supplied with this product or optional equipment. If you are unable to understand the **Warnings, Cautions or Instructions**, contact a health care professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

WARNING

Only personnel instructed and trained in the use of the HomeFill II O2 Conserving Cylinder should operate this product.

Product contains magnetic, ferrous material that may affect the results of an MRI.

Be sure all connections are tight and leak free.

DO NOT use a liquid leak detector to test for leaks.

DO NOT autoclave.

DO NOT gas sterilize with ethylene oxide.

DO NOT clean with aromatic hydrocarbons.

DO NOT attempt to repair product.

DO NOT immerse product in any kind of liquid.

Store product in a clean area when not in use.

⚠ WARNING

Only use Invacare Corporation carrying bag HF2PC9BAG carrying bag.

NEVER smoke in an area where oxygen is being administered.

ALWAYS confirm prescribed dose before administering to patient and monitor on a frequent basis.

DO NOT use if dirt or contaminants are present on or around cylinder, valve, product or connecting devices.

NEVER use near any type of flame or flammable/explosive substances, vapours or atmosphere.

NO OXYGEN is delivered when the pointer is aligned with “Off”.

DO NOT block the outlet fitting or kink the cannula when the product is in use, as this may damage the unit.

The use of this device is limited to the oxygen patient. Cylinders MUST be used only by the oxygen patient and are not to be distributed to any other individual for any purpose.

This product is NOT to be used by patients who breathe through their mouths.

DO NOT allow oil from your hands or other sources to come into contact with the compressor coupling or bottle fittings. These substances may become flammable in the presence of oxygen and cause injury. Avoid touching the bottle fitting whenever possible. Example of items to avoid are hand creams or lotions, cooking oils, suntan oils or sunblocks and similar oily products. If you use these types of products Invacare suggests you wash your hands prior to using the HomeFill II unit. If the bottle fitting should come in contact with such substances it should be cleaned off with a damp cloth prior to connecting it to the compressor.

⚠ WARNING

NEVER oil or lubricate the compressor coupler or bottle fittings. NEVER use any penetrating oil or lubricant such as WD-40 or 3 in One. If connection difficulty is experienced verify that the bottle fitting is being inserted straight down into the compressor coupler. If continued difficulty is experienced contact your supplier for assistance.

DO NOT use a humidifier with this product.

DO NOT allow cylinders to tip or fall. Secure gas cylinders so they cannot fall. Ensure cylinder is always in upright position.

DO NOT store cylinders near sources of heat or flame.

This product is equipped with a relief valve. If you hear a loud hissing or popping sound coming from the product, discontinue use of product, turn the flow selector to the “Off” position, and contact your dealer/Home Care equipment provider.

DO NOT use while sleeping without consulting your healthcare provider.

This product is designed to operate with a standard single lumen, adult oxygen nasal cannula with a maximum length of 2.1 m (7 ft).

The cannula is for single patient use only.

Cannula MUST be installed on outlet connection prior to use.

DO NOT place the unit under clothing while in use. When this product is in use a small amount of oxygen is vented. Wearing the unit under clothing may saturate fabrics with oxygen and cause them to burn rapidly if exposed to sparks or flame. It may take several hours for oxygen levels in fabrics to return to normal.

⚠ WARNING

DO NOT use IOH200AW, HomeFill II unit, to fill cylinder outdoors or anywhere below 13°C (55°F).

SECTION 2—FEATURES

⚠ DANGER

This product is not intended as a life-sustaining or life-supporting device.

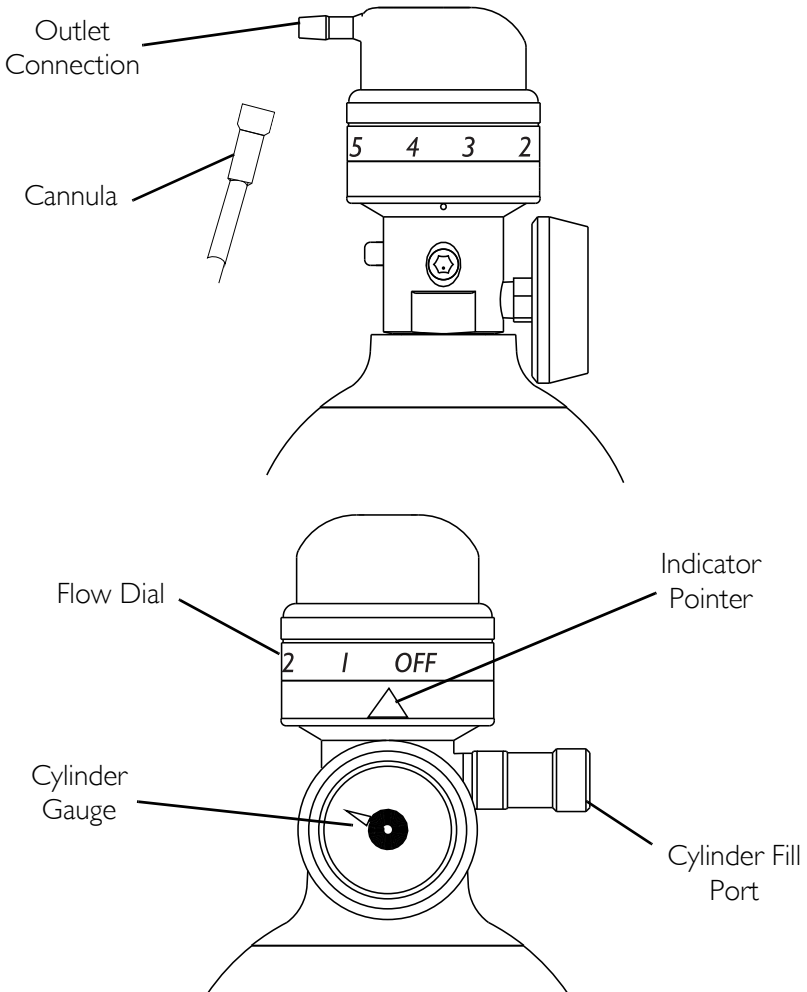
FIGURE 2.1 Product Diagram

SECTION 3—TYPICAL PRODUCT PARAMETERS

INLET PRESSURE RANGE:	2068 kpa - 14479 kpa (300 - 2100 psig)
INTERNAL REGULATED PRESSURE:	345 kpa (50 psig)
DIMENSIONS:	HF2PCE6 - 1.0 Liters HF2PCE9 - 1.7 Liters
HF2PCE6: WEIGHT	1.86 kg (4.1 lbs)
HF2PCE6: OVERALL LENGTH	27.7 cm (10.9 in)
HF2PCE6: DIAMETER	11.1 cm (4.4 in)
HF2PCE9: WEIGHT	2.31 kg (5.1 lbs)
HF2PCE9: OVERALL LENGTH	35.5 cm (14 in)
HF2PCE9: DIAMETER	11.1 cm (4.4 in)
PULSE SETTINGS:	1, 2, 3, 4 and 5
FLOW SETTINGS:	2 LPM Continuous
SAVINGS RATIO:	Up to 5.7:1
CANNULA REQUIREMENT:	1.21 m (4 ft) recommended (MS0204) 2.1 m (7 ft) long maximum standard adult single lumen oxygen nasal cannula
OPERATING CONDITIONS	
TEMPERATURE	
PULSE:	1.7° C (35° F) to 40.6° C (105° F)
CONTINUOUS:	-17° C (0° F) to 40.6° C (105° F)
ALTITUDE:	0 to 3071 m (0 to 10,000 ft)
STORAGE CONDITIONS	
TEMPERATURE:	-40° C (-40° F) to 60° C (140° F)
MAXIMUM HUMIDITY:	95%
PRODUCT CLASSIFICATION:	Class IIB
REGULATORY LISTING:	In compliance with the medical device directive 93/42/EEC

SECTION 4—INSPECTION AND COMPRESSOR INTERFACE

NOTE: Refer to the HomeFill II owner's manual for compressor operating instructions.

Cylinder Prefill Inspection

DANGER

Improper filling or abuse of this cylinder, or failure to follow these safety instructions may cause serious bodily injury or death. High pressure oxidizing gas vigorously accelerates combustion.

DO NOT alter or modify cylinder or related components.

DO NOT use oil or grease. Keep cylinder, cylinder valve, all equipment and connections free of oil and grease to avoid violent ignition resulting in serious bodily injury or death.

WARNING

For medical applications, use only as directed by a licensed practitioner. Uninterrupted use of high concentrations of oxygen over a long duration, without monitoring its effect on oxygen content of arterial blood, may be harmful.

Oxygen cylinders filled by the HomeFill II shall be used for personal use only. “Not to be filled for resale or use by professional users”.

This cylinder is to be used with the HomeFill II system. This cylinder contains 93±3% oxygen produced by an oxygen concentration using the pressure swing adsorption process.

DO NOT handle cylinder or use contents until you are professionally trained, including emergency procedures.

⚠ WARNING

Use this cylinder in accordance with the HomeFill II owner's manual. Open valve slowly, Close valve after each use and when empty. Secure cylinder during storage and use. No smoking in cylinder area. Keep away from heat, flame or spark. Keep out of the reach of children. **DO NOT** drop.

Transfilling of this gas is performed by HomeFill II only.

Cylinders that have been refinished and/or exposed to elevated temperatures **MUST** be hydrostatically tested before filling. Cylinder temperature should not exceed 52° C (125° F). Aluminum cylinders subject to the action of fire or heated temperatures in excess of 177° C (350° F) **MUST** be withdrawn from service and condemned by trained personnel. **DO NOT** use caustic paint strippers. Valve and safety relief devices **MUST** be removed and replaced by trained and authorized personnel. **DO NOT** alter or change valve assembly.

All cylinders **MUST** be inspected before attempting to fill; otherwise, injury or damage may occur.

NEVER smoke in an area where oxygen is being administered.

NEVER use near any type of flame or flammable/explosive substances, vapours or atmosphere.

NOTE: For this procedure, refer to FIGURE 4.1.

The Hydrostatic Testing Date

Aluminum cylinders **MUST** undergo testing every five years.


⚠ WARNING

DO NOT fill cylinders that have not been tested in the past five years. Otherwise, serious personal injury may result. Contact your dealer for replacement.

External Examination

1. Examine the outside of the cylinder for the following conditions, and replace the cylinder if they exist:
 - Dents or dings
 - Arc burns
 - Oil or grease
 - Any other signs of damage that might cause a cylinder to be unacceptable or unsafe for use.
2. Examine the cylinder for evidence of fire or thermal damage. Evidence includes charring or blistering of the paint, or other protective coating or heat sensitive indicator. If fire or thermal damage is found, replace the cylinder.
3. Inspect the cylinder fillport for the following:
 - Debris, oil or grease
 - Noticeable signs of damage
 - Signs of corrosion inside the valve
 - Signs of excessive heat or fire damage

NOTE: DO NOT use the cylinder if any of these items are found. Contact your dealer for replacement.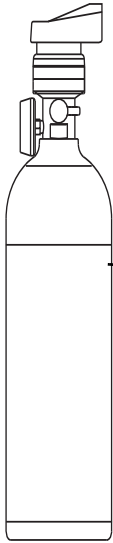

Warning Label

This cylinder is to be used with the HomeFill II system.
This cylinder contains 93±3% oxygen.
Keep cylinder in a well ventilated place.
Keep away from combustible material.
Keep free from oil and grease.
Refer to the HomeFill II owner's manual for additional warnings, cautions and safety instructions.

FIGURE 4.1 External Examination

Connecting/Disconnecting the Cylinder to/from the Compressor

Connecting the Cylinder to the Compressor

⚠ WARNING

NEVER use tools of any kind to connect/disconnect the cylinder and the compressor. Otherwise, severe injury and/or damage may occur.

DO NOT drop oxygen cylinders. Use two hands when handling/transporting oxygen cylinders. Otherwise, injury or damage may occur.

DO NOT use a liquid leak detector to test for leaks.

NOTE: For this procedure, refer to FIGURE 4.2 and FIGURE 4.3.

1. Examine the cylinder gauge. If the cylinder pressure is less than 10342 kpa (1500 psig), proceed to STEP 2 (FIGURE 4.3).

NOTE: If the cylinder pressure is greater than 10342 kpa (1500 psig), DO NOT attempt to top off this cylinder. It may not fill.

2. Set the regulator flow dial on the cylinder to Off (FIGURE 4.3).
 3. Remove the cylinder and connector fillport covers.
-

CAUTION

DO NOT connect the outlet (cannula) connection to the HomeFill II unit, otherwise the conserving cylinder will not work properly.

4. Momentarily push DOWN on the outer ring (sleeve) of the connector fillport to reset the connector as shown in FIGURE 4.2.

NOTE: If the outer ring (sleeve) is in the UP position (GREEN dots not visible), the connector fillport will not be able to accept the cylinder fillport. Pushing DOWN momentarily will reset the connector fillport (GREEN dots visible) to accept the cylinder fillport.


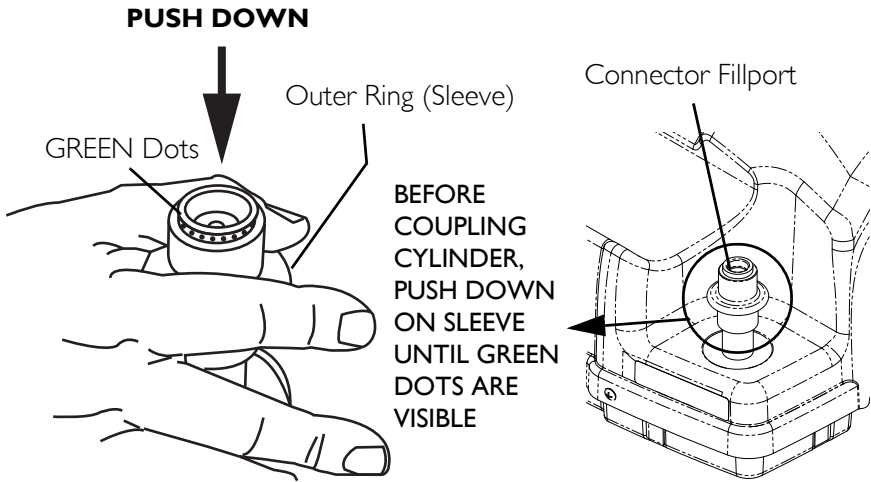
FIGURE 4.2 Resetting Connector Fillport

5. Grasp the cylinder/regulator assembly in the area behind the cylinder gauge (FIGURE 4.3).
6. Position the cylinder in the compressor cradle (FIGURE 4.3).
7. Align the cylinder fillport with the connector fillport (FIGURE 4.3).
8. Pull UP on the outer ring (sleeve) of the connector fillport while pushing DOWN on the cylinder/regulator assembly to couple the cylinder fillport into the connector fillport (FIGURE 4.3).

NOTE: The cylinder is properly connected when an audible “click” is heard.

SECTION 4—INSPECTION AND COMPRESSOR INTERFACE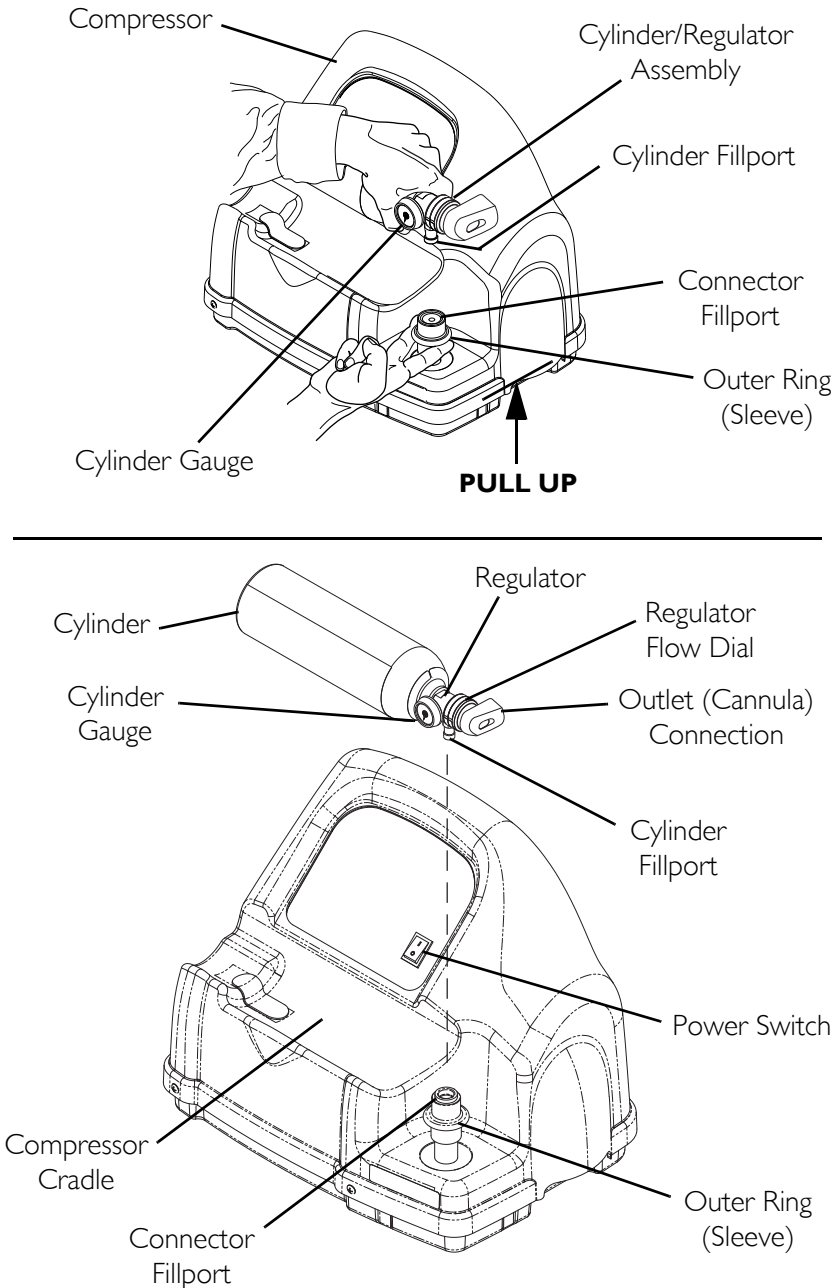

NOTE: Fillport covers not shown for clarity.

FIGURE 4.3 Connecting the Cylinder to the Compressor

Disconnecting the Cylinder from the Compressor

⚠ WARNING

NEVER use tools of any kind to connect/disconnect the cylinder and the compressor. Otherwise, severe injury and/or damage may occur.

NOTE: For this procedure, refer to FIGURE 4.4.

1. Press the compressor power switch to the Off (O) position.
2. Grasp the cylinder/regulator assembly in the area behind the cylinder gauge.
3. With the other hand, grasp the outer ring (sleeve) of the connector fillport and push DOWN.
4. Lift up on the cylinder/regulator assembly to remove from the connector fillport.

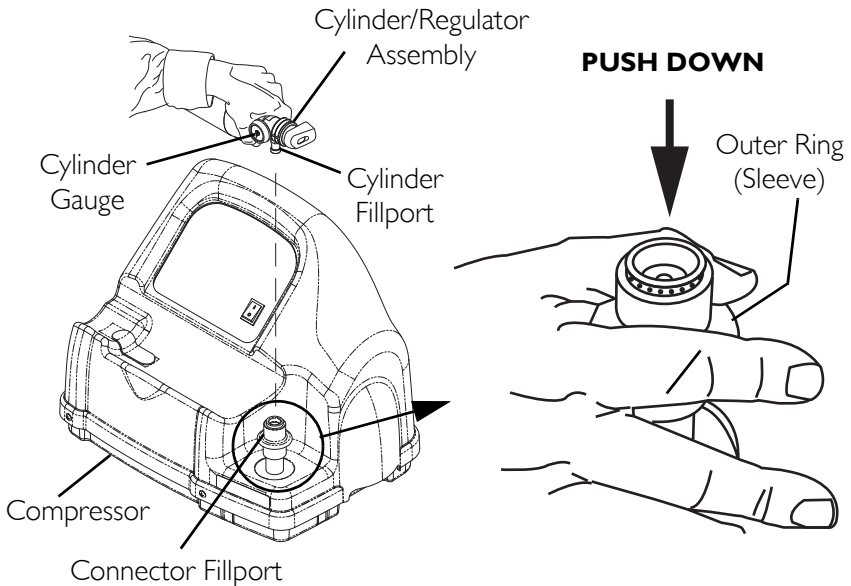
FIGURE 4.4 Disconnecting the Cylinder from the Compressor



5. When the cylinder fillport is disconnected from the connector fillport, release the outer ring (sleeve) of the connector fillport and use two hands to remove the cylinder from the compressor cradle.

⚠ WARNING

The fillport cover on the connector fillport and the cylinder fillport MUST be replaced after filling and whenever not in use. If either of the fillport covers are missing contact your healthcare provider before using.

6. Place the fillport covers onto the connector fillport and cylinder fillport.
7. Examine the cylinder gauge to ensure cylinder is full (needle pointing into GREEN area). If the cylinder is not full, perform the following:
 - Repeat STEPS 1-8 of Connecting the Cylinder to the Compressor on page 16.
 - If cylinder is not full after second attempt at filling, DO NOT use the cylinder and contact the provider.

Cylinder Fillport Cover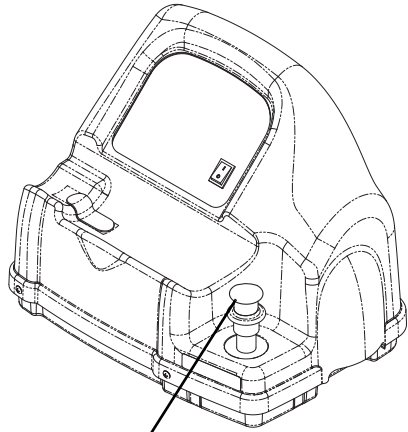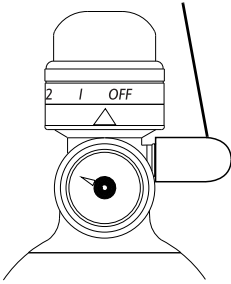

Connector Fillport Cover

FIGURE 4.5 Fillport Covers

SECTION 5—OPERATING INSTRUCTIONS

WARNING

NEVER smoke in an area where oxygen is being administered.

NEVER use near any type of flame or flammable/explosive substances, vapours or atmosphere.

NOTE: For this procedure, refer to FIGURE 5.1.

1. Prior to each use, inspect product for visible damage. DO NOT use if any damage is found.

NOTE: If for any reason, any label becomes illegible or lost, contact your dealer/Home Care equipment provider.

2. Be sure regulator is in the “Off” position. Align “Off” with the indicating pointer.
 3. Position the cylinder so the oxygen cylinder valve outlet is pointing away from the user and any other person(s).
-

WARNING

DO NOT direct flow of oxygen at any person, or flammable material when adjusting the flow dial.

4. Attach a standard adult single lumen oxygen nasal cannula, no longer than seven feet to the product’s outlet according to the cannula manufacturer’s instructions. DO NOT use pediatric, low flow nasal cannulas or oxygen masks with this product.

NOTE: A unit without cannula attached may pulse if set at position 1-5.

5. Place the cannula over your ears and position the prongs in your nose as instructed by your Health Care provider or cannula manufacturer. Refer to FIGURE 5.1.

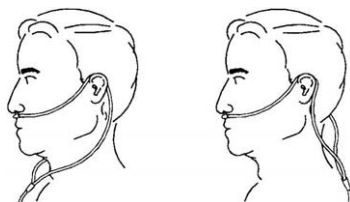
FIGURE 5.1 Cannula Placement

6. Turn dial until indicating pointer is aligned with the prescribed setting.

⚠ WARNING

When pointer on contents gauge enters red zone, it is recommended to refill the cylinder on the Invacare HomeFill Compressor.

NO OXYGEN is delivered between settings. To obtain desired flow, the indicating pointer must align with a specific number on the dial.

To avoid injury to patient, ALWAYS confirm prescribed setting before administering to patient and monitor flow on a frequent basis.

Only use the Invacare Corporation HF2PC9BAG carrying bag.

7. Breathe through the nose and feel a pulse of oxygen as you inhale.

NOTE: When on a pulse setting, there is flow, or a pulse, only at the beginning of each breath. If you do not feel the pulse at the beginning of each breath, check the setting.

NOTE: Because each patient's breathing pattern is different and the environment varies, it may be difficult to feel some low setting pulses.

NOTE: If operating the cylinder below 2° C (35° F) use the 2 L/Min continuous setting only.

SECTION 6—MAINTENANCE

1. After each use, clean exterior of the product with a dry, lint free cloth only.
2. Store product in a clean area free from grease, oil, and other sources of contamination.

CAUTION

DO NOT use cleaning solutions (i.e coupler and fillport fitting).

DO NOT immerse product in any kind of liquid.

All repairs MUST be done by Invacare Corporation.

Replacement Parts – Accessories

HF2PC6BAG Carrying Bag

HF2PC9BAG Carrying Bag

SECTION 7—CYLINDER FILL TIMES

NOTE: All filling times are approximate and may vary on environmental conditions.

Cylinder Filling Times	Concentrator Flow Rate to Patient: (Platinum 5 up to 2.5 L/min.) (Platinum 9 up to 5 L/min.)	
	HF2PCE6	HF2PCE9
NORMAL	1 hr 25 min	2 hr 20 min

Troubleshooting

If Conserving Cylinder fails to function, consult the Troubleshooting guide. If problem cannot be corrected, consult your Dealer/Home Care equipment provider.

PROBLEM	POSSIBLE CAUSE	SOLUTION
No flow.	<ol style="list-style-type: none"> 1. Regulator in “Off” position. 2. Cylinder empty. 3. Dial not set at increment. 4. Conserving not sensing breath. 	<ol style="list-style-type: none"> 1. Set to prescribed setting. 2. Replace/fill cylinder. 3. Set dial so indicator points to a setting. 4. Check position of cannula in nose. Do not breathe through mouth. 5. Check to see if the cannula is kinked.

WARRANTY INFORMATION

For warranty information contact customer service at one of the following locations:

United Kingdom:

Invacare Ltd, South Road, Bridgend Industrial Estate,
UK-Bridgend CF31 3PY

Tel: (44) (0)1656 664 321, Fax: (44) (0)1656 667 532

UK@invacare.com

Ireland:

Invacare Ireland Ltd, Unit 5 Seatwon Business Campus,
Seatown Road, Swords, County Dublin - Ireland

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All rights reserved. Trademarks are identified by [™] and [®].
All trademarks are owned by or licensed to Invacare Corporation unless otherwise noted.
© 2008 Invacare Corporation