

OPERATING THE BED

1. Operating beds with 4-sectioned mattress platforms

The Medley bed is equipped with **HB 70** hand control. Please see *illustration 1*


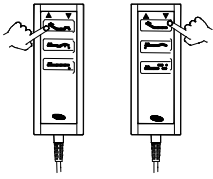


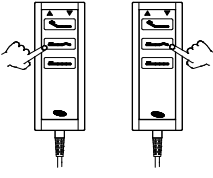

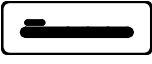
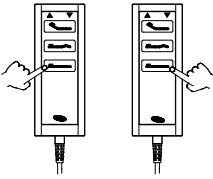
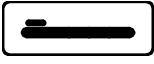
<p>Backrest up</p>  <p>Button with the symbol shown above. Push the left-hand side of the button.</p>		<p>Backrest down</p>  <p>Button with the symbol shown above. Push the right-hand side of the button.</p>
<p>Thigh section up</p>  <p>Button with the symbol shown above. Push the left-hand side of the button.</p>		<p>Thigh section down</p>  <p>Button with the symbol shown above. Push the right-hand side of the button.</p>
<p>Height up</p>  <p>Button with the symbol shown above. Push the left-hand side of the button.</p>		<p>Height down</p>  <p>Button with the symbol shown above. Push the right-hand side of the button.</p>

Illustration 1

2. Operation of brakes. Please see *illustration 2*

Each of the bed's 4 castors is provided with brakes for both lengthwise and crosswise locking.
The brake is foot operated.

When the bed is positioned correctly, at least one castor at the head section and one castor at the foot section must be locked.

Braking: Step on the pedal.

Releasing the brake: Step on the release pedal.



The bed's castors must be locked during care of a client in bed or when using any positioning function.

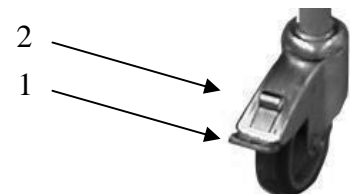


Illustration 2

OPERATING THE ACCESSORIES

1. Operation of side rails – steel and wood

UP: Pull up the top side rail bar until the locking pin engages with an audible click. *Please see illustration 3*



Illustration 3

DOWN: Lift the top side rail bar and press the slat inside the rail to disengage the locks. Lower the side rail. *Please see illustration 3*




Illustration 4



There is a risk of entrapment when operating the side rails.

2. Adjusting the height of the lifting pole handle (optional accessory)

Loosen the cord as shown in *illustration A*. The lifting handle can now be adjusted to the desired height. Press the cord together as shown in *illustration B*, and check that the cord is locked in the cord lock by pulling the handle.

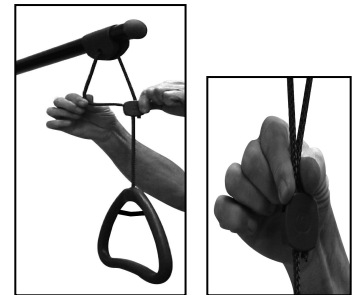

A

B

Please notice that the string between the pole handle and lifting pole can be adjusted under consideration of the mattress height and the beds inclinations.

For a flexible use, we recommend, that both upper and lower bows in the handle are used.



Always position the lifting pole in such a way that the handle extends inwards across the bed. If the lifting pole is used while the handle has been turned away from the bed, the bed can tip when used.

Please note:

Disconnect the plug from the mains before moving the bed. The cables must be mounted in such a way that they are kept clear of the floor and do not block the castors.

We recommend to mount the mains cable on the hooks on the frame of the bed. *Please see illustration 5.*



Illustration 5

CLEANING AND DISINFECTION

Remove the mains plug from the socket outlet before cleaning.

The bed should be washed down using a sponge, cloth or brush. Use ordinary disinfection detergents.

When disinfecting, only use spray or wiping methods. Only officially recognized disinfectants must be used. If in doubt, please contact the Community Equipment store that loaned the bed.

Dry the bed after cleaning.

Never use acids, alkalines or solvents such as acetone or cellulose thinner.

The hand control, motors and control unit may be washed with brush and water, but not with pressurized water. Never immerse any electrical components in water.