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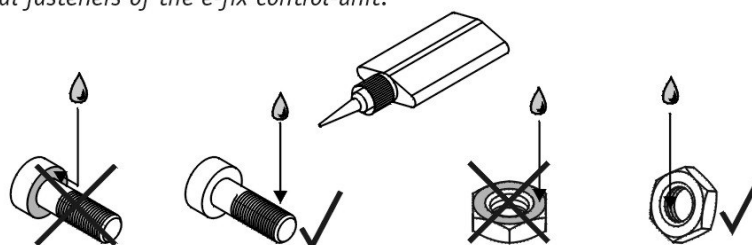
Locking of screws and screw nuts

If there is no other specification in our mounting instruction: Save all screw fittings which have no locking washer with screw locking adhesive (see pictures below). Use WEICON adhesive AN 302-43 (blue) / LOCTITE 243 adhesive to stick adapter sockets and screws. Use WEICON adhesive AN 306-01 (green) / LOCTITE 601 adhesive to stick the mechanical fasteners of the e-fix control-unit.

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Allowed wheel camber of the e-fix and e-motion

Don't pass over the following maximum values, if you mount the bracket at a wheelchair which offers the adjustemt of the wheel camber.

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Fastening torques for screw fittings

Unless otherwise noted, seize the screw fittings as shown in the following table:

Schraubverbindungen / Screw fitting

Gewinde/Thread	Anzugsmoment/Fastening torque
M3	1,1 Nm
M4	3 Nm
M5	5 Nm
M6	10 Nm
M8	25 Nm
M10	50 Nm
M12	85 Nm

Adapterhülsen / Adapter sockets

Gewinde/Thread	Anzugsmoment/Fastening torque
M16x1	30 Nm

Montageanweisung [Mounting instruction]

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Important Liability Note

A combination of the mounting systems introduced by Ulrich Alber GmbH (Alber) together with auxiliary drives not manufactured by Alber is not authorized by Alber. Such combination does not comply with the intended use to be defined in accordance with the medical device directive (MDD) and therefore constitutes non-conformity according to the MDD.

By way of precaution, Ulrich Alber GmbH rejects any liability regarding warranty services or possibly resulting property damages and personal injuries due to a combination of Alber mounting systems with third party brands.

Pos. [Pos.]	Menge [Quantity]	Einheit [Unit]	Art.-Nr. [Art.-No.]	Lagerplatz/Alber [Stock/Alber]	Bezeichnung [Description]
005	2	St [Pcs]	1565767	715/01/016	Halterung IVC Azalea
010	12	St [Pcs]	1467022	718/04/016-017	Klemmstück [Clamp colourless anodized]
015	12	St [Pcs]	1520032	721/04/008-009	Einlegeprofil Prisma [Inlet profile prism]
020	2	St [Pcs]	1463160	719/03/007-009	Standardgabel blau vz [Standard fork blue galvanized]
025	2	St [Pcs]	1456406	721/02/006-007	Quickpin Sicherungsstift [Quickpin quick release bolt]
030	4	St [Pcs]	1463716	1465/02/002	Zylinderschraube DIN 7984 - M 6 x 12 - 8.8 vz [Cylinder head screw DIN 7984 - M 6 x 12 - 8.8]
035	12	St [Pcs]	1459361	790/02/020	Zylinderschraube DIN 912 - M 6 x 50 - 8.8 vz [Cylinder head screw DIN 912 - M 6 x 50 - 8.8]
040	4	St [Pcs]	1456223	1468/00/003	Sicherungsscheibe S - 6 - FSt [Securing disk S - 6 - FSt Schnorr]

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