# Aquatec® Ocean / Ocean XL

en	Shower and Toilet Commode User Manual .....	3
de	Dusch- und Toilettenrollstuhl Gebrauchsanweisung .....	27
fr	Chaise roulante pour douche/toilettes Manuel d'utilisation .....	53
it	Carrozzina da doccia e da toilette Manuale d'uso .....	77
es	Silla de ducha e inodoro Manual del usuario .....	101
pt	Cadeira de ducha e sanitária Manual de utilização .....	125
nl	Douche- en toiletstoel Gebruiksaanwijzing .....	149

This manual **MUST** be given to the user of the product.  
**BEFORE** using this product, read this manual and save for future reference.

**AQUATEC®**

**Yes, you can.®**

©2018 Invacare Corporation

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

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# Contents

---

This manual MUST be given to the user of the product.  
BEFORE using this product, read this manual and save for future reference.

<b>1 General</b>	<b>4</b>
1.1 General information	4
1.2 Symbols in this user manual	4
1.3 Warranty	4
1.4 Compliance	4
1.5 Intended use	5
1.6 Service life	5
<b>2 Safety</b>	<b>6</b>
2.1 Safety information	6
2.2 Labels and symbols on the product	6
<b>3 Components</b>	<b>8</b>
3.1 Overview	8
<b>4 Setup</b>	<b>10</b>
4.1 Safety information	10
4.2 Assembling the shower and toilet commode	10
4.3 Accessory (optional)	15
<b>5 Usage</b>	<b>16</b>
5.1 Safety information	16
5.2 Using the shower and toilet commode	17
<b>6 Transport</b>	<b>19</b>
6.1 Safety information	19
<b>7 Maintenance</b>	<b>20</b>
7.1 Maintenance and inspection	20
7.2 Cleaning and disinfection	20
7.3 Changing the castors	21
<b>8 After Use</b>	<b>22</b>
8.1 Storage	22

8.2 Re-use	22
8.3 Disposal	22
<b>9 Troubleshooting</b>	<b>23</b>
9.1 Identifying and repairing faults	23
<b>10 Technical Data</b>	<b>24</b>
10.1 Dimensions and weight	24
10.2 Materials	24
10.3 Environmental parameters	25

# 1 General

## 1.1 General information

This User Manual contains important information about using the product. In order to ensure safety when using the product, read the User Manual carefully and follow the safety information.

If your vision is impaired, you can view the user manual as a PDF file on the Internet and enlarge it on-screen as required. If you cannot enlarge the text and graphics sufficiently, please contact the Invacare® distributor for your country. For addresses, see back page of this manual. If necessary, we will provide you with a high resolution PDF file of the user manual.

## 1.2 Symbols in this user manual

In this user manual, warnings are marked with symbols. The title next to the warning symbol indicates the level of danger.

### WARNING

Indicates a dangerous situation which could lead to death or serious injury if not avoided.

### CAUTION

Indicates a dangerous situation which could lead to minor injuries if not avoided.

### IMPORTANT

Indicates a dangerous situation which could lead to damage if not avoided.

Useful tips, recommendations and information to ensure efficient and smooth operation.

This product complies with the directive 93/42/EEC for medical products. The launch date for this product is specified in the CE declaration of conformity.

Manufacturer

## 1.3 Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries. Warranty claims can only be made through the provider from whom the appliance was obtained.

## 1.4 Compliance

Quality is of the utmost importance for our company; all processes are based on the ISO 13485 standard.

This product features the CE mark, in compliance with the Medical Device Directive 93/42/EEC Class 1.

Invacare is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

- We only use REACH compliant materials and components.

For further information please contact Invacare in your country (for addresses, see end of this document).

## 1.5 Intended use

The shower and toilet commode is designed exclusively as an aid for showering or bathing, going to the toilet, for use during personal hygiene (e.g. in front of the basin) or for transferring patients while indoors. It is intended to be used by adult patient, with or without caregiver assistance, who have:

- Difficulty standing or keeping their balance
- Weak legs or lower limb disability
- Medical conditions that impair their balance

The product is intended for institutional and private household use. Any other use is prohibited.

### **WARNING!**

#### **Risk of injury**

- Never use the product as an aid for getting into or out of a bathtub, for climbing or descending, as a support or for similar purposes.
- Do not use the product outdoors.

As a toilet commode, the product can be used over a standard toilet or independently with the commode pan (optional).

The shower and toilet commode may only be used as a bathing stool in a pool, if the pool does not contain salt water and an attendant is present at all times.

Temperature for use: 10 - 65 °C

## 1.6 Service life

We estimate a life expectancy of five years for these products, provided they are used in strict accordance

with the intended use as set out in this document and all maintenance and service requirements are met. The estimated life expectancy can be exceeded if the product is carefully used and properly maintained, and provided technical and scientific advances do not result in technical limitations. The life expectancy can also be considerably reduced by extreme or incorrect usage. The fact that we estimate a life expectancy for these products does not constitute an additional warranty.

## 2 Safety

### 2.1 Safety information

#### WARNING!

- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, service manuals or instruction sheets supplied with this product or optional equipment. Invacare product manuals are available on the internet or at your local dealer (Addresses are displayed on the back cover of this manual).
- If you are unable to understand the warnings, cautions or instructions, please contact a healthcare professional, dealer or technical personnel before attempting to use this equipment – otherwise, injury or damage may occur.

#### WARNING!

#### Risk of injury

- Do not stand on the product.
- Do not use as a climbing aid.
- Do not use outdoors.

#### WARNING!

#### Risk of injury

- Do not use if defective.
- In the event of a malfunction, contact your dealer immediately.
- Do not make any unauthorized changes or alterations to the product.
- Only use accessories intended for this product.


#### IMPORTANT!

The information contained in this document is subject to change without notice.

### 2.2 Labels and symbols on the product

#### Identification label

The identification label contains important information:

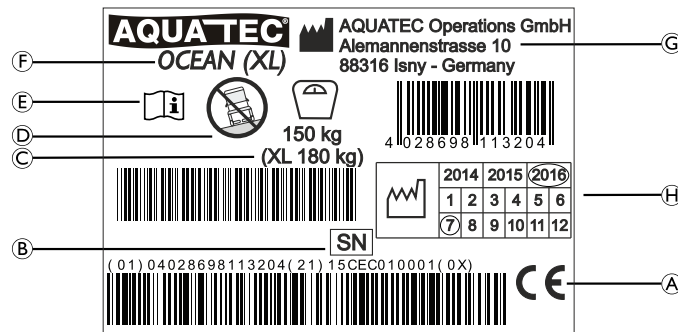Ⓐ	Conformity mark
Ⓑ	Serial number
Ⓒ	Maximum user weight
Ⓓ	Note (use only on flat surfaces)
Ⓔ	Read user manual
Ⓕ	Product designation
Ⓖ	Manufacturer address
Ⓗ	Date of manufacture


The identification label is attached to the inside of the right side section.

## 3 Components

### 3.1 Overview


The following components are included within the scope of delivery: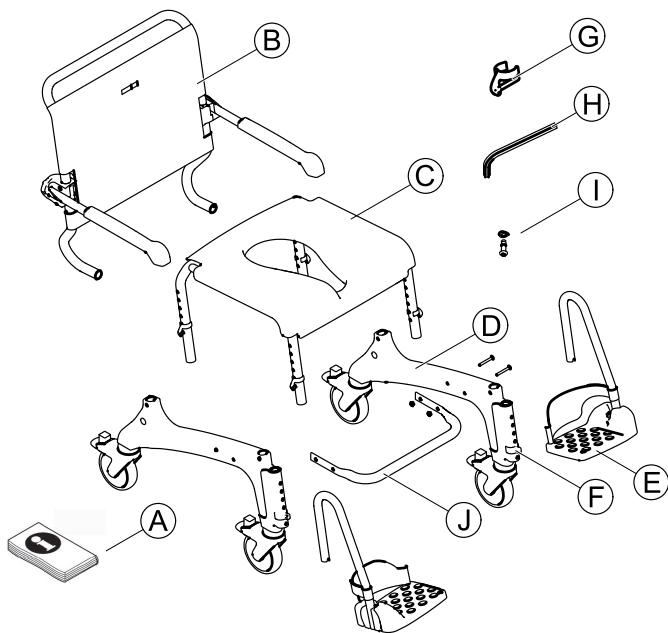

(A)	User Manual
(B)	Backrest with backrest cover and armrests

(C)	Seat frame with seat plate
(D)	Side parts with wheels (2 x)
(E)	Footrests with heel strap (2 x)
(F)	Clips (2 x) for footrest (inserted)
(G)	Clips (4 x) for seat frame
(H)	Allen key AF 4
(I)	Screws (4 x) and washers (4 x)
(J)	Base stabilization tube with screws (4 x) and nuts (4 x) - Ocean XL only

The following parts are included within the scope of delivery of the self-propelling wheels with parking brake:

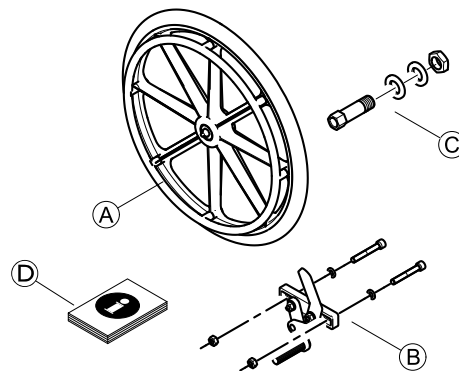Ⓐ	Self-propelling wheels (2 x)
Ⓑ	Parking brakes (2 x)
Ⓒ	Hubs incl. axles (2 x)

Note the change in dimensions when the self-propelling wheels are fitted. For details see chapter Technical Data.

## 4 Setup

### 4.1 Safety information


#### WARNING!

- Check the parts for transport damage before commissioning and contact your dealer if required.
- Ensure during assembly that the parts are positioned correctly with respect to one another.
- Self locking screws, nuts or thread locking adhesive are used for several connections. If these are loosened, they must be replaced by new self locking screws, nuts or they must be secured using new thread locking adhesive.

### 4.2 Assembling the shower and toilet commode

This section describes the assembly. Disassembly takes place in the reverse order.

#### Fitting the seat frame tube for height adjustment

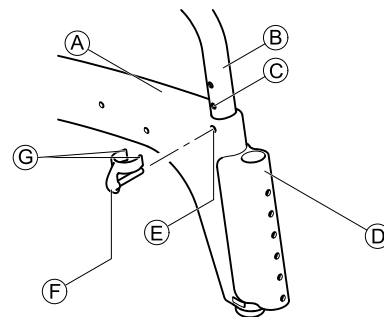1. Remove the enclosed clips (F) from the bag.
2. Place the seat frame on its side.

#### IMPORTANT

- Push the side part evenly onto the frame and do not twist.
- Secure the side part at the same height, front and rear.

3. Push the side part (A) with the guides for the footrests (D) forwards on the seat frame (B).
4. Bring the clip mounts (E) on the front and back of the side part in line with a hole in the seat frame (C).
5. Insert the clips (F) from the outside into the side part, until both ends (G) enclose the seat frame tube (B).

#### IMPORTANT

- Ensure that the left and the right side parts are set to the same height.

6. Turn the seat frame and repeat the procedure on the second side part.
7. Set the seat frame with the fitted side parts on its wheels.

### Adjusting the seat height


#### WARNING!

- The seat height must be adjusted to suit the user. As a rule, the seat height is correctly adjusted when the user is sitting on the commode and both feet touch the ground.
- The seat height must be adjusted to the same height on both sides.
- Do not adjust the seat height when someone is sitting on the shower and toilet commode.

1. Raise the shower and toilet commode slightly on one side by the seat frame ⑥.
2. Remove both clips ⑤ on this side.
3. Pull the seat frame out of the side part ④ or push it into the side part to adjust the seat height.
4. Push both clips back in completely.
5. Repeat step 1 – 4 on the other side.

### Fitting the base stabilization tube (Ocean XL only)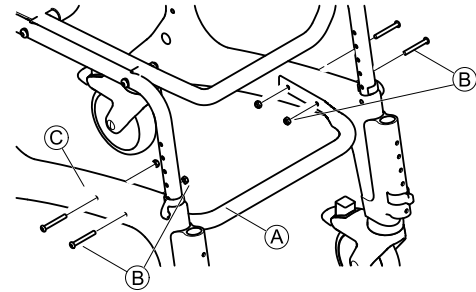

1. Fit the base stabilization tube ④ using the screws and nuts ⑤ supplied between the side parts ⑥.


### Fitting the backrest

1. Push down all four foot pedals for the castors to lock them.

#### IMPORTANT!

- Make sure the armrests are facing forward when folded up.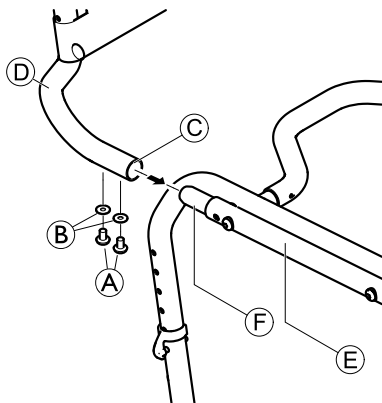

2.

Insert the backrest ④ from behind into the seat frame ⑤, until the collar ③ touches the seat frame tube ⑥.


**!** **IMPORTANT!**

- Firmly tighten the screws using the Allen key supplied.
- Check the screws regularly to ensure they are sitting securely.

3. Position a washer ② on each screw ① and secure the backrest on each side.
4. Tighten the screws with a torque of 5 Nm.

**Adjusting the backrest cover**

The tension of the backrest can be adjusted as desired.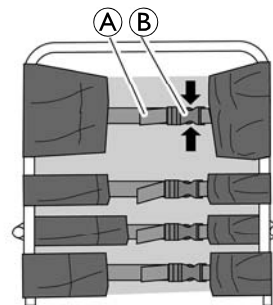

**WARNING!**


– Make sure that at least two belts are always passed by the backrest frame (on the outside) and their belt locks are closed.

Tightening:

1. Pull the belts ① of the backrest cover tighter.

Loosening:


1. Open the buckles ② by squeezing the retaining lugs together (arrows).
2. Push the belts back a little through the buckle.
3. Close the belt buckles.
4. Retighten the belts gently.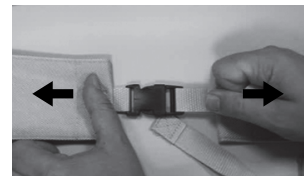

**WARNING!****Risk of injury due to damaged backrest buckles!**

Check for secure fastening of the backrest buckles each time the product is maintained or reconditioned.

- Check all buckle clips for secure fastening by closing the buckles and pulling on both ends.
- Replace the backrest/backrest buckles if a buckle opens under the applied force.

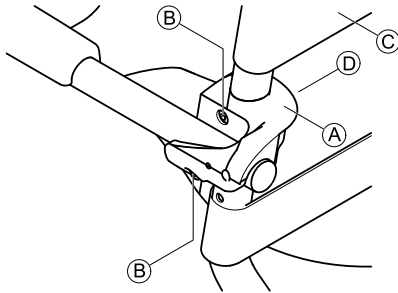**Adjusting the height of the armrests****IMPORTANT!**

- Adjust the height of the armrests to suit the user.
- Change the height of the armrests evenly on both sides.
- Set the height of the right and left armrest in the exact same manner.

1. On the backrest cover, open the buckle of the belt that passes on the inside of the backrest frame.
2. On the backrest cover, open the buckle of the belt © that runs above the armrest.
3. Pass this belt on the inside of the backrest frame and re-close.

The nuts are inserted loosely in the retaining clamp and can fall out.

4. Unscrew the screws © of the armrest bracket © using the Allen key supplied.
5. Remove the securing clamp © with the nuts from the inside of the backrest frame.
6. Remove the backrest bracket © with the armrest and move it upwards.


The smaller hole serves to fix the retaining clamp.

7. Insert the screws in the armrest bracket and guide them through the holes in the backrest frame.
8. If required, put the nuts back into the retaining clamp.
9. Place the retaining clamp on the backrest frame so that the fixing on the retaining clamp projects into the smaller hole.
10. Turn the screws in the nuts and tighten.
11. Pass the belt of the backrest cover that is still open on the outside of the backrest frame, close and tighten all belts.

**!** **IMPORTANT!**

The armrests are lowered using the same procedure.  
 – When converting to the lower position, the belt that lies beneath the armrest must be opened and passed inside the backrest frame.

**Fitting the footrests**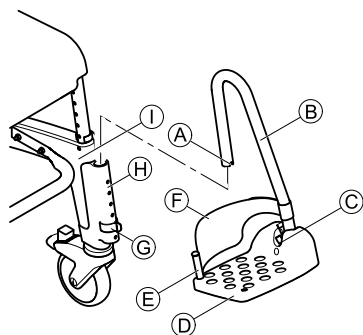

1. Push the footrest **B** with the footplate **D** into the guide **H** at the front of the side part **I** until the groove **A** engages with the pin of the clip **G**. The footrest can no longer be turned to the side.

**i** The assembly procedure is the same for the right and the left footrests.

**Adjusting the height of the footrests**

1. Pull up the footrest **B** slightly.
2. Pull out the clip **G** and put it in again at the required position. Pull out the footrest slightly further if necessary.
3. Push down the footrest until the groove **A** catches the pin of the clip.
4. Set the other footrest to the same height as described.


**Adjusting the inclination of the foot plates**

**!** **IMPORTANT!**

– The inclination of the foot plates can be adjusted infinitely variably.  
 – Make sure that the right and left foot plate is adjusted in the same way.

1. Loosen the Allen screw **C** on the foot plate using a size 4 Allen key.
2. Turn the foot plate **D** to the desired position.
3. Re-tighten the Allen screw.

## Fitting/removing the self-propelling wheels (optional)

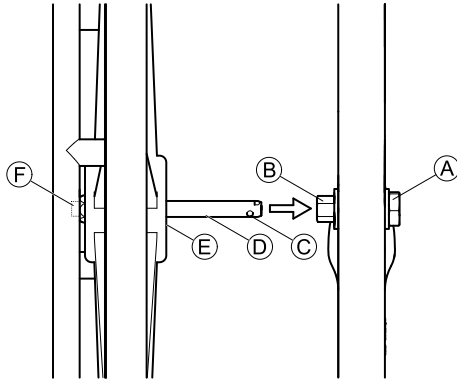5. Raise the shower and toilet commode a little.
6. Insert the axle into the hub (B) and push in the self-propelling wheel onto the axle until the end stop (E).
7. Release the button (F).
8. Press the retaining balls outward so that they are behind the hex nut (A).

To remove the wheels, carry out the same procedure in the reverse order.

### 4.3 Accessory (optional)

#### **WARNING!**

#### **Risk of injury**

- The assembly of accessories (optional) must be carried out according to the corresponding instructions.

#### **!** IMPORTANT!

- When the self-propelling wheels are fitted, the rear wheels must remain locked.
- The assembly is the same for the right and left self-propelling wheel.
- The hubs and the parking brakes for the self-propelling wheels must be fitted by a qualified technician. Assembly is described in the assembly instructions „Assembling the hubs and brakes for Aquatec® Ocean 24" wheels“.

1. Align all four castors in the driving direction.
2. Lock all castors.
3. Push the axle (D) on the self-propelling wheel inwards. (The handrims project outwards.)
4. Press and hold the button (F) on the outside of the axle. This releases the retaining balls (C) at the end of the axle.

## 5 Usage

### 5.1 Safety information

#### **WARNING!**

- Only use the shower and toilet commode on a flat surface and indoors.
- Observe the specifications on the identification label. Do not overload the shower and toilet commode.
- Do not place excessive weight on the shower and toilet commode.
- Always perform a visual inspection of the product for external damage before each use.

#### **WARNING!**

##### **Risk of slipping and rolling**

- Before use, make sure that the seat is securely fastened to the seat frame.
- Lock all castors with levers using the foot levers before sitting down on the shower and toilet commode or standing up from it, as well as in the parking position.
- Apply the parking brake when the self-propelling wheels are fitted (optional).

#### **WARNING!**

##### **Risk of tipping**

- Swivel the footrests to the side or fold up the foot plates before sitting down or standing up.
- Before sitting down or standing up, turn the front castors so that the lock is pointing forwards.
- Never stand on the footrests.
- Do not lean excessively forwards.

#### **WARNING!**

##### **Risk of tipping forwards**

- Front antitippers (Art. No. 1535936) are available to prevent the shower commode chair from tipping forward.

#### **WARNING!**

##### **Risk of tipping**

- The shower and toilet commode must only be moved by using the push bar or the self-propelled wheels (optional).

#### **CAUTION!**

##### **Risk of burning your skin**

- The product components can heat up when exposed to external sources of heat.
- Do not expose the product to strong sunlight before use.
  - Allow the product to reach room temperature before use.

**CAUTION!****Risk of pinching**

- When using the self-propelling wheels (optional), do not insert your fingers between the spokes or the rotating self-propelling wheels and the seat frame while moving.

**IMPORTANT!****Risk of damage to the product**

- Do not lift the shower and toilet commode using the footrests.


## 5.2 Using the shower and toilet commode

### Swiveling the armrests

**CAUTION!****Risk of getting fingers caught in the folding mechanism**

- Do not insert fingers in the articulated joint when folding the armrests up or down.
- Do not insert fingers between the armrest and the backrest when folding back the armrests.
- Make sure that no parts get caught.

### Swiveling the footrests

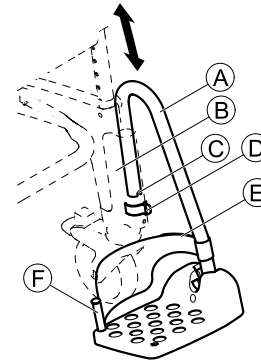**Entry position/exit position:**

1. Pull the footrest **A** in the guide **B** a little upwards, until the groove **C** no longer engages in the pin of the clip **D**.
2. Swivel the footrest to the side.

**Driving position:****CAUTION!****Risk of accident due to loose footrest**

- Fix footrests in the driving position.

1. Turn the footrest **A** inwards.
2. Push the footrest downward, until the groove **C** engages in the pin of the clip **D**.

### Removing/fitting the heel strap

1. Open the Velcro® strip (E) on the heel strap.
2. Remove the screw (E) from the underside of the foot plate.
3. Remove the heel strap from the flexible plastic pin.

To attach the strap, carry out the same procedure in the reverse order.

#### **IMPORTANT!**

- When installing ensure that the Velcro® strip of the heel strap is facing backwards.

### Removing/fitting the seat plate

#### Removing:

1. Pull the seat plate upwards on both sides.

#### Fitting:

#### **CAUTION!**

##### **Risk of getting fingers caught**

- Do not insert fingers between the seat surface and the seat frame when pressing down on the seat plate.

1. Place the seat plate on the seat frame with the cutout to the front.
2. Push the seat plate backwards as far as it will go.
3. With your hand flat, press the seat plate down at the sides and then at the front and back until the seat plate snaps into the seat frame.

---

## 6 Transport

---

### 6.1 Safety information

The product can be dismantled for transport into the parts of the scope of delivery, see chapter 3.1 *Overview, page 8*.

1. Disassemble the product, see chapter 4.2 *Assembling the shower and toilet commode, page 10* by following the assembly steps in reverse order.

#### **IMPORTANT!**

##### **Risk of damage to the product**

- Transport the product in a dry area.
- Do not expose the product to direct sunlight.
- Refer to the storage and shipping conditions in chapter 10.3 *Environmental parameters, page 25*.

#### **IMPORTANT!**

- Self locking screws, nuts or thread locking adhesive are used for several connections. If these are loosened, they must be replaced by new self locking screws, nuts or they must be secured using new thread locking adhesive.

## 7 Maintenance

### 7.1 Maintenance and inspection

Maintenance is required to be done at least once a year.

To ensure good performance and safety you should regularly check that the following is carried out:

- Check the product and accessories regularly for damage. Do not use the product if defective.
- Check tilting function of the seat frame and backrest (Ocean Dual VIP Ergo). Easy tilting and secure locking of the position is mandatory.
- In case of damage or malfunction we recommend to contact your provider.
- Ensure that the product is securely assembled incl. all protection covers and that screws are properly tightened.

#### **WARNING!**

##### **Risk of injury and damage**

If the product is improperly adjusted it can cause accidents and severe injuries.

- The product must be repaired by a qualified technician.
- Damaged parts have to be replaced by a qualified technician.
- Only use original Invacare spare parts.

### 7.2 Cleaning and disinfection

The product can be cleaned and disinfected using commercially available agents.

#### **IMPORTANT!**

In the sensitive area of the bathroom and toilet, hygiene is particularly important.

- Keep the product clean at all times and wipe it down regularly.

1. Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

#### **IMPORTANT!**

All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Healthcare Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control [www.nice.org.uk/CG139](http://www.nice.org.uk/CG139) and your local infection control policy.

- Clean by hand or in an autoclave.
- Clean without scouring agent.

#### **Cleaning in a domestic environment**

#### **IMPORTANT!**

Risk of damage to the product

- Do not disassemble the holders for the accessories.

1. Remove any accessories fitted.
2. Pull the footrests out of the guides.
3. Remove the seat plate and the backrest cover.
4. Remove the side parts.
5. Reinsert the clips in the seat frame immediately so that they are not misplaced.
6. Wipe down the individual parts using a cloth or brush and commercially available cleaning agents and disinfectants.
7. Rinse the parts with warm water.
8. Dry the parts with a cloth.
9. Re-assemble the product, see chapter 4 Setup.

### Cleaning in a clinical environment

In addition to cleaning as described above, clean the product after the release of each patient, after ending of the period of use or on a minimum monthly basis (depending on which occurs first) at 85 °C for 3 minutes in the autoclave.

#### CAUTION!

##### Risk of burning your skin

- Allow the product to cool after cleaning in the autoclave.

### Cleaning the backrest cover (optional)

1. Remove the backrest cover.
2. Machine-wash the cover at max. 60 °C using a mild detergent.

#### IMPORTANT!

##### Risk of damage

- Washing at higher temperatures can cause shrinkage.
- Do not tumble dry covers.

#### IMPORTANT!

Excessive cleaning can damage textile.

- We recommend to change the backrest cover every two years for hygiene reasons.

### Cleaning the toilet pan (optional)

The toilet pan can be cleaned in an autoclave at following temperatures/times:

- 90 °C for 1 minute
- 85 °C for 3 minutes
- 80 °C for 10 minutes.

## 7.3 Changing the castors

#### WARNING!

##### Risk of accident due to incorrectly fitted castors.

- The castors must be replaced by a specialist dealer.

---

## 8 After Use

---

### 8.1 Storage

#### **IMPORTANT!**

#### **Risk of damage to the product**

- Store the product in a dry area.
- Do not store the product near heat sources.
- Do not expose the product to direct sunlight.
- Refer to the storage and shipping conditions in chapter *10.3 Environmental parameters*, page 25.

### 8.2 Re-use

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly (→ *7.2 Cleaning and disinfection*, page 20) and check the safety functions, and reset if necessary.

### 8.3 Disposal

The disposal and recycling of used devices and packaging must comply with the applicable legal regulations.

Please refer to the manufacturer for details of the proper and complete disposal.

## 9 Troubleshooting

### 9.1 Identifying and repairing faults


The following table provides information on malfunctions and their possible causes. If you cannot correct the malfunction using the remedies named, please contact your specialist dealer directly.

Fault	Possible cause	Remedy
Difficulty in adjusting the seat height	Dirt between the seat frame and the side section	Disassemble product into the parts of the scope of delivery and clean thoroughly
The castors are difficult or impossible to turn	Dirt between the fork and the rim	Thoroughly clean the castors
Difficulty pushing the footrests to the side	Deposits in the guides	Thoroughly clean the footrests and the guides
Parking brake of the self-propelling wheels (optional) does not engage	Parking brake wrongly adjusted	The parking brake must be adjusted by a qualified technician.

## 10 Technical Data

### 10.1 Dimensions and weight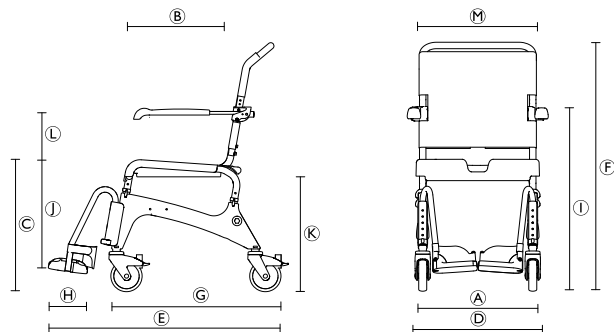

Ⓐ	Seat width	480 mm
Ⓑ	Seat depth	450 mm
Ⓒ	Seat height	475 - 600 mm
Ⓓ	Total width (Ocean) Total width (Ocean XL) Total width with self-propelling wheels	560 mm 640 mm 675 mm
Ⓔ	Total depth Total depth with self-propelling wheels	900 mm 1060 mm
Ⓕ	Total height	964 - 1094 mm
Ⓖ	Depth without footrests	700 mm

Ⓜ	Depth of foot plate	160 mm
Ⓝ	Height to armrests	680 - 850 mm
Ⓣ	Height from footrest to seat	255 - 495 mm
Ⓚ	Min. distance between the floor and the bracket for the commode pan	approx. 408 mm
	Min. width between the side parts	approx. 381 mm
	Product weight	approx. 15 kg
	Product weight with self-propelling wheels	approx. 19.5 kg
	Maximum User weight (Ocean)	150 kg
	Maximum User weight (Ocean XL)	180 kg

### 10.2 Materials

Frame	Stainless steel, powder-coated
Seat plate	Polypropylene
Backrest cover	Nylon, PVC-coated
Footrests / Side parts	Polypropylene, fiber-glass reinforced parts
Armrest pads	Polypropylene

Castors	Plastics and stainless steel
Running surface of 24" wheels	PUR

All product components are corrosion resistant and do not contain natural rubber latex.

### 10.3 Environmental parameters

<b>Operating conditions</b>	
Ambient temperature	10 - 65 °C
Relative humidity	15% - 93%, non-condensing
Atmospheric pressure	795 - 1060 hPa
<b>Storage and shipping conditions</b>	
Ambient temperature	0 - 65°C
Relative humidity	30% - 75%, non-condensing
Atmospheric pressure	795 - 1060 hPa

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The label contains the following information:

- Brand and Model:** AQUATEC® OCEAN (XL)
- Manufacturer:** AQUATEC Operations GmbH, Alemannenstrasse 10, 88316 Isny - Germany
- Weight:** 150 kg (XL 180 kg)
- Barcode 1:** 4 028698 113204
- Barcode 2:** (01) 04028698113204 (21) 15CEC010001 (0X)
- SN (Serial Number):** 15CEC010001 (0X)
- CE Marking:** CE
- Year of Manufacture:** 2014, 2015, 2016
- Production Line:** 1 2 3 4 5 6, 7 8 9 10 11 12
- Symbols:** Information icon (i), No climbing icon, and a triangle with a slash.

Labels A through G point to specific features on the label:

- A:** CE Marking
- B:** SN (Serial Number)
- C:** Weight (150 kg)
- D:** Weight (XL 180 kg)
- E:** Information icon (i)
- F:** No climbing icon
- G:** Manufacturer information
- H:** Year of manufacture and production line

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Aquatec Operations GmbH