Invacare® Delta™ P429/2 - P429/2C
Rollator

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Manual de instrucciones

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User Manual

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DE

ES

FR

GB

IT

NL

PT

CE

Yes, you can.®

Bedienungsanleitung DE	2
Manual de instrucciones ES	8
Manuel d'utilisation FR	14
User Manual GB	20
Istruzioni per l'uso IT	26
Gebruiksaanwijzing NL	32
Manual de instruções PT	38

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User Manual

1 GENERAL	21
General Information.....	21
Symbols	21
Product Life Cycle	21
Product Labeling.....	21
User Height Specifications and Weight Limitation	22
2 SAFETY	22
Intended Use.....	22
General Warnings.....	22
3 USAGE	23
Assembling the Rollator	23
Operating Position.....	23
Folding the Rollator.....	23
Adjusting the Handle Height	23
Installing the Bag, Basket and Tray.....	24
Adjustment of the Braking Effect	24
Using the Hand Brake.....	24
Using the Parking Brake	24
4 CARE AND MAINTENANCE	25
5 AFTER USE	25
Disposal.....	25
Reuse	25
6 WARRANTY	25
Warranty	25

I GENERAL

I.1 General Information

Read carefully through the operating instructions before use.
The rollator is adapted for both indoor and outdoor use.

I.2 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

Gives useful tips, recommendations and information for efficient, trouble-free use.

I.3 Product Life Cycle


The expected product life cycle of this product is 5 years when used in accordance with safety instructions, maintenance intervals and correct use, stated in this manual. The effective product life cycle can vary according to frequency and intensity of use.

NB. Only to be used as a walking support. User's max. weight:
P429/2 - P429/2C - DELTA 135 kg.

INVACARE will not accept responsibility for adaptations, faulty repairs etc. that are outside its control.

I.4 Product Labeling

- 1: Article number
- 3: Lot number
- 4: Date of manufacture
- 5: Max. user weight
- 6: NB. See operating instructions.
- 7: In- and outdoor rollator.
- 8: Max. width of rollator.
- 9: Manufacturer's name

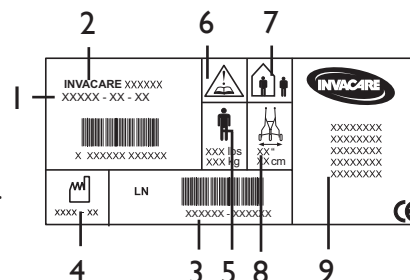I.5 User Height Specifications and Weight Limitation

Item	User Height Min. / Max	Weight Limitations	Product Specifications
Delta Rollator Model P429/2 - P429/2C	160-183 cm	135 kg *	
Basket	n/a	5 kg	
Tray		5 kg	
Bag		5 kg	

* The weight limitation includes the contents of the basket, tray or bag and the weight of the user.

2 SAFETY

- ❗ Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instruction.

2.1 Intended Use

The rollator is intended to be used solely as a walking aid and all other use can cause risk of injury.

2.2 General Warnings

WARNING

Exercise caution when walking on sloping ground.

WARNING

If the push handles are exposed to extreme temperature (above 41°C) there might be risk for first degree burns. The rollator must be fully unfolded and locked when used as a walking support. NB! Only to be used as a walking support. Do not go up or down stairs with the rollator. Do not use the rollator on escalators. DO NOT hang anything from the frame of the rollator. Items placed in either the basket or on the tray should not protrude from the basket, bag or on the tray. Exercise caution when there is a heavy load in the basket. Loads should be transported solely in the basket, bag or on the tray. The handlebar must not be angled outwards as this reduces the rollator's lateral stability.

3 USAGE

3.1 Assembling the Rollator

- ❗ The locking lever with the washer shall be installed on the outside.
- Open the rollator into operating position (see below).


3.2 Operating Position

1. Before the rollator is assembled remove plastic caps in the side tubes.
2. Fit the locking lever (A) with the bolt (B). Ensure that the head of the bolt slots into the recess on the fitting.
3. Insert the handle tubes into the side uprights and tighten the locking levers. These must be pointing downwards during use.


WARNING:

ALWAYS check that the safety latch is in locked position.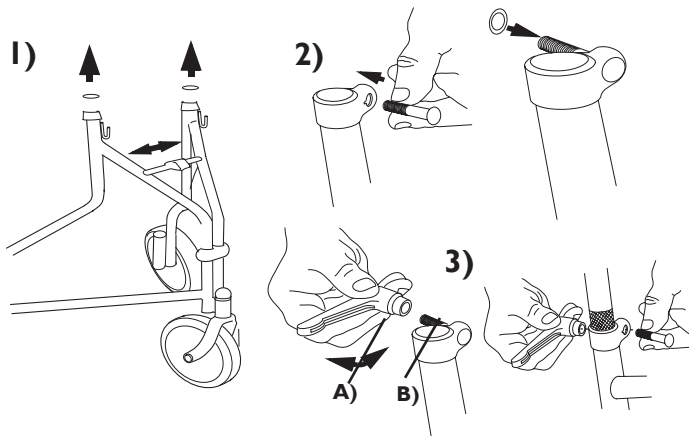

3.3 Folding the Rollator

1. Pull the safety catch upwards - backwards (A).
2. Fold the rollator. NB. Be aware of the risk of crushing between the parts of the frame.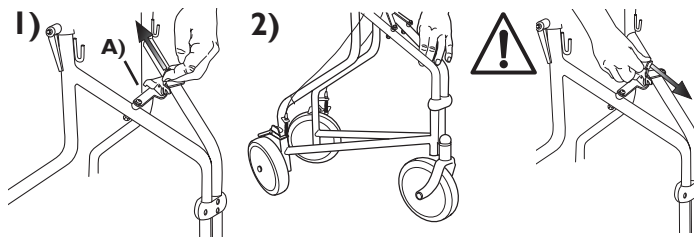

3.4 Adjusting the handle height

To adjust the height of the handles: Undo the locking levers and adjust the handles to a suitable height.


Tighten the locking levers, which should then be adjusted so that they point downwards. (The locking levers can be freely turned by first pulling them outwards). Check that the handles are set at the same height. The handles should normally be positioned at wrist height.

WARNING: Risk of falling


The handlebars must point directly backwards when using the rollator. Under no circumstances should they be angled outwards. The vertical line in the knurling must be located in the middle of the indentation in the frame, (see figure A).

The handles must never be set above their highest position, indicated by the knurling on the tubes. Handles should be set to the same height on either side.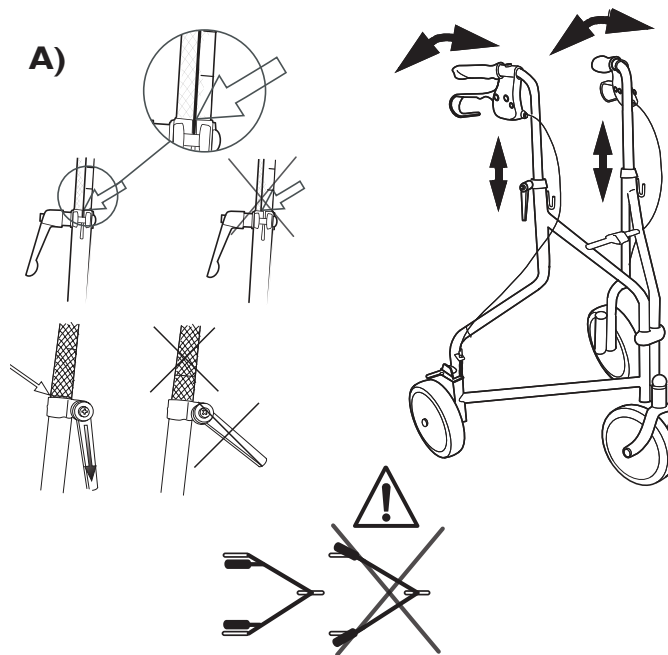

3.5 Installing the Basket, Bag and Tray

1. Fitting the basket
2. Fitting the tray.
3. Fitting the bag.

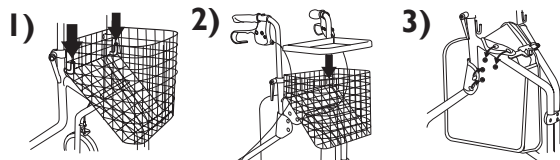
3.6 Adjustment of the Braking effect

Two setting screws, one in the brake lever (A) and one in the middle of the brake cable (B) adjusts the tension of the brake and the application of the brake to the user's capability. The braking power increases and the movement required in the brake lever is reduced when the setting screws are loosened.


WARNING:

Check the braking effect after height adjustment.
Make sure to tighten the lock nuts after adjustment.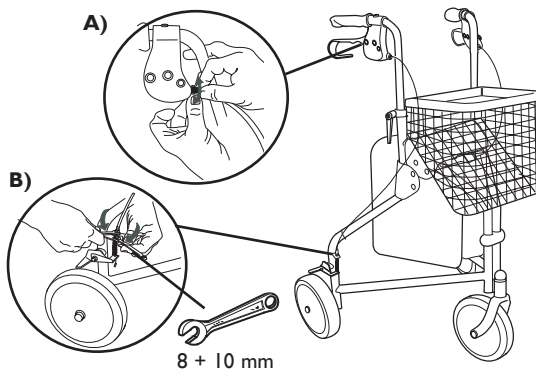

3.7 Using the handbrake

Apply the brakes by pulling both brake handles upwards.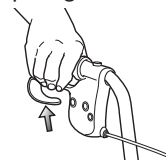


3.8 Using the Parking Brake

Press the brake handles carefully downwards to apply the parking brakes. A distinct click indicates that the brake is in the correct position.

- ❗ It is important that both the brakes are put into parking position when they are to be used.

To release the parking brake, pull the brake handle upwards.

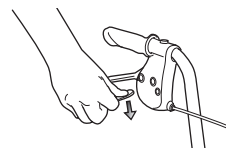4 CARE AND MAINTENANCE

- ① Maintenance should be done at least every six months.
DO NOT use an abrasive detergent, cleaner or cloth to clean or dry the walker.

To ensure good performance and safety you should regularly check that the following is carried out:

- Clean the rollator with ordinary cleaning agent. Wipe it dry. Keep the wheels clean.
- Check that screws and adjustments are properly tightened.
- If your rollator is not functioning perfectly contact your Invacare representative immediately.
- Never use a defective rollator. Do not attempt repairs yourself!
- Verify brake operation. Contact your Invacare dealer if brake adjustment is required.
- If hand grips are loose, Do not use the rollator. Contact your Invacare representative.
- Periodically inspect the casters for tightness, wear and damage.
- Make sure that the wheels are free of hair, lint and other debris.

5 AFTER USE

5.1 Disposal

When the product can no longer be used, the product, accessories and the packaging materials should be sorted correctly. The individual parts can be separated and dealt with according to the list specification.

Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

5.3 After Use

WARNING: Risk of Injury

ALWAYS have an Invacare representative inspect the product for damage before transferring the product to a different user. If any damage is found, Do not use the product. Otherwise injury may occur.

6 WARRANTY

We supply a 2 year guarantee ** from the date of delivery. Damage caused by wear to items such as wheels, handles and brakes is not covered by the guarantee. Damage that arises due to abnormal use is excluded.

Damage caused by users who weigh more than 135 kg is excluded. Maintenance and operating instructions must be observed for the guarantee to be valid.

- ① ** Deviations may occur due to local agreements.

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Equipment (Suzhou) Co., Ltd.
Invacare Rea AB, Box 200