
Robin™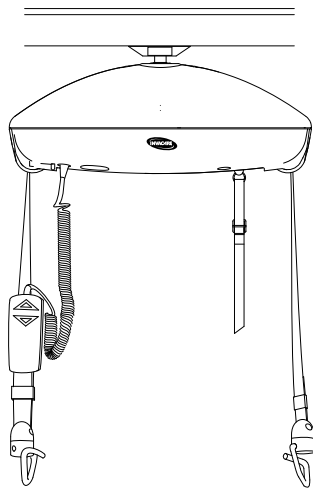

Robin Mover™

Invacare® Robin™

Robin™ , Robin Mover™

en	Ceiling Hoist	
	User Manual	3
da	Loftspersonløfter	
	Brugsanvisning	29
fr	Lève-personne sur rail	
	Manuel d'utilisation	57
it	Sollevatore a soffitto	
	Manuale d'uso.....	87
nl	Plafondlift	
	Gebruiksaanwijzing	117
no	Takmontert løfter	
	Bruksanvisning	145
sv	Taklyft	
	Bruksanvisning	173

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

© 2016 Invacare Corporation

All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by [™] and [®]. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.

The ceiling hoist is CE-marked and in accordance with the Medical Device Directive MDD 93/42/EU.

The ceiling hoist has been tested and approved in accordance with Lift standard EN/ISO 10535.

Contents

This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

I General	4
1.1 Introduction	4
1.2 Intended Use	4
1.3 Service Life	4
1.4 Warranty Information	4
2 Safety	5
2.1 Safety information	5
2.2 Operating information	5
2.3 Load Limits	7
2.4 Radio Frequency Interference	7
2.5 Product Labeling	7
3 Components	9
3.1 Main parts of the ceiling hoist	9
3.2 Accessories	9
4 Setup	11
4.1 Included items	11
4.2 Before Use	11
4.3 Installing the ceiling hoist	11
4.4 Inserting hand control plug	12
5 Usage	13
5.1 General operating information	13
5.2 Operating the ceiling hoist	13
5.3 Indicator light and audio signal	14
5.4 Charging the Battery	14

5.5 Using the external back-up battery	15
5.6 Emergency stop	15
5.7 Emergency lowering	16
5.8 Resetting the emergency stop or lowering feature	17
6 Lifting the Patient	18
6.1 Sling Information	18
6.2 Attaching the Sling	18
6.3 Lifting Information	19
6.4 Lifting to and from a seated position	19
6.5 Lifting to and from a lying position	20
7 After Use	21
7.1 Transport and Storage	21
7.2 Dismounting the ceiling hoist	21
7.3 Disposal	21
8 Troubleshooting	22
8.1 Troubleshooting Table	22
9 Maintenance	24
9.1 Safe Maintenance	24
9.2 Maintenance and Safety Inspection	24
9.3 LOLER Statement	24
9.4 Cleaning and disinfection	25
9.5 Battery Maintenance	25
10 Technical data	26
10.1 Dimensions and weight	26
10.2 Electrical System	26
10.3 Environmental conditions	27
10.4 Electromagnetic compliance (EMC) information	27
10.4.1 Guidance and manufacturer's declaration	27

I General

I.1 Introduction

Thank you for choosing an Invacare product.

This user manual contains important information about the handling of the product. In order to ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Please note that there may be sections in this user manual, which are not relevant to your product, since this manual applies to all available models (on the date of printing). If not otherwise stated, each section in this manual refers to all models of the product.

I.2 Intended Use

A ceiling hoist is a battery-powered transfer device, designed to be used indoor in normal temperature and humidity.

Common lifting situations are:

- Between the bed and a wheelchair
- To and from the toilet
- Lowering and raising patients to/from the floor

The ceiling hoist can be used to transfer and position completely or partially immobile patients in the recommended track system. All position changes are possible without assistance of the patient. The ceiling hoist is only intended to lift patients up to the maximum weight limit stated in the user manual.

There are no known contraindications for this product. Using the hoist for purposes other than transfer of people is not advised.

Selecting the appropriate slings and accessories for each individual is important to assure safety when using a ceiling hoist. See Invacare's sling and accessory user manuals for further information on those devices.

I.3 Service Life

The ceiling hoist has an expected lifetime of 10 years when used in accordance with safety instructions, maintenance intervals and correct use, stated in this manual. After 10 years use, it must be inspected by a specialized service technician before further use.

The effective product lifetime can vary according to frequency and intensity of use. If the hoist is used in localities with increased humidity, a reduction in lifetime must be expected.

I.4 Warranty Information

Terms and conditions of the warranty are part of the general terms and conditions particular to the individual countries in which this product is sold.

Contact information for your local Invacare office is located inside the back cover of this manual.

2 Safety

2.1 Safety information

WARNING!

Risk of injury or damage

- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this product.

The information contained in this document is subject to change without notice.

2.2 Operating information

This section of the manual contains general safety information about your product. For specific safety information, refer to the appropriate section of the manual and procedures within that section.

WARNING!

- Check all parts for shipping damage before use.
- In case of damage, do not use the equipment. Contact the dealer or Invacare representative for further instructions.

WARNING!

- The ceiling hoist must only be operated by trained personnel.
- Read the instructions carefully before using the ceiling hoist and in connection with cleaning and service of the hoist.
- Do not perform any modification to the equipment.
- The ceiling hoist's maximum load of 200 kg must never be exceeded.
- If accessories are rated with a lower load than 200 kg, then this determines the ceiling hoist's maximum load.
- The ceiling hoist is advised only to be used to lift or move a patient.
- The red emergency strap must be adjusted to the caregiver's reach, and must not be removed.
- If the hoist fails, lower patient onto a suitable surface using the red emergency strap before continuing the transfer by a different method.

WARNING!

Risk of falling

DO NOT attempt any transfer without the consent of the patient's healthcare providers. Read this user manual and observe other personnel lifting patients. Then practice lifting under supervision with a capable person acting as a patient.

- Use common sense in all lifts. Special care **MUST BE** taken with people with disabilities who cannot cooperate while being lifted.
- Check that the sling attachments are secure on the lifting strap hooks each time the sling is removed and replaced, to ensure that the sling is properly attached before the patient is removed from a stationary object.

WARNING!

Risk of entrapment / strangulation

- Before lifting, check that the patient is completely free of his/her surroundings.
- While lifting, do not allow the hand control cord or the lifting straps to become entangled around the patient or caregiver.

WARNING!

Risk of entrapment

There is a risk of entrapment between the hook and the hook housing.

There is a risk of entrapment between the hooks and the sling, when lifting.

To avoid injury from entrapment:

- Use caution when lifting
- **NEVER** put hands or fingers on or near the hook when lifting.
- Ensure the patient's hands and fingers are away from the hooks before lifting.

WARNING!

Risk of injury

There is a risk of injury to the patient and caregiver caused by dangling hooks or the hand control.

- Always be aware of the position of the hooks and hand control during lifting procedures.

WARNING!

Risk due to electronic parts

This product is mechanically protected against derailing and jamming.

- For safety reasons the top cover must only be removed by specialized service technicians.
- The electronics may only be serviced by specialized service technicians.

The hand control is protected against water splashes (IPx4).

- DO NOT immerse the hand control in liquids, even for a short time. Avoid any contact with water or other liquids.

2.3 Load Limits

WARNING!

The maximum load limit must not be exceeded.

- Read the Safe Working Load label (indicating the maximum load limit) for each component, e.g. hanger bar, sling, etc.
- The component, labelled with the lowest load limit determines the maximum load limit for the entire system.

2.4 Radio Frequency Interference


WARNING!

- Most electronic equipment is influenced by Radio Frequency Interference (RFI). CAUTION should be exercised with regard to the use of portable communication equipment in the area around such equipment. If RFI causes erratic behavior, stop the unit using the hand control or emergency stop. DO NOT use the unit while transmission is in progress.

2.5 Product Labeling

Hoist

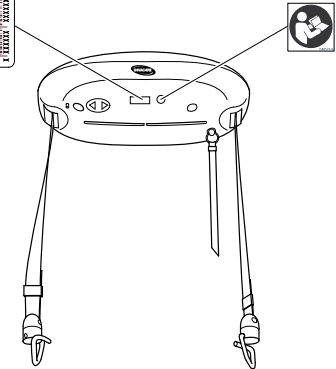Symbols

Safe Working Load (= max. load including patient + sling + other accessories)

Class II Equipment

Type B Applied Part

Read Manual

Manufacturer Address

Date of Manufacture

Reference Number


Serial Number

WEEE Conform. Refer to 7.3 Disposal, page 21.

3 Components

3.1 Main parts of the ceiling hoist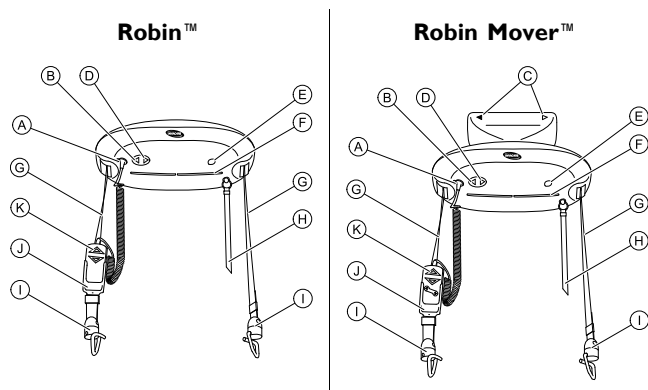

(A)	Connection for hand control
(B)	Control button UP
(C)	Directional arrows on Robin Mover™
(D)	Control button DOWN
(E)	Connection for external back-up battery
(F)	Indicator light
(G)	Lifting straps
(H)	Red emergency strap
(I)	Strap hooks

(J)	Retainer for hand control (accessory)
(K)	Hand control

3.2 Accessories

CAUTION!

Compatibility of slings and hanger bars / strap hooks

Invacare® uses a "Loop and Coat Hanger Bar / Strap Hook System" as do many other manufacturers. Therefore other suitable patient transfer systems (slings), manufactured by other companies, can be used on the Invacare patient lift range as well.

However we do recommend:

- A risk assessment is always to be carried out by a professional prior to issuing lifting equipment. It is important that the Task, Individual, Load, Environment and Equipment are considered in the risk assessment.
- Always choose the sling design and size according to the patient's weight, size and physical ability whilst considering the type of transfer to be carried out.
- Do only use slings that are suitable for a "Loop and Coat Hanger Bar / Strap Hook System".
- Do not use slings for "Keyhole Hanger Bar" or for "Tilting Frame Hanger Bar" designs.


Available accessories

- 2- and 4-point hanger bar including flexiscope version
- Stretcher for horizontal lift
- Scale including hanger bar
- Holder for hand control
- 24 V charger

- Charger set (24 V charger and holder for hand control)
- Extension straps and extension cables for hand control
- External back-up battery
- Charger for external back-up battery
- Sling models for “Loop and Coat Hanger Bar System”:
 - Full body support slings – without head support
 - Full body support slings – with head support
 - Slings for dress/toileting – with or without head support
 - Slings for amputee
 - Slings for walking training

4 Setup

4.1 Included items

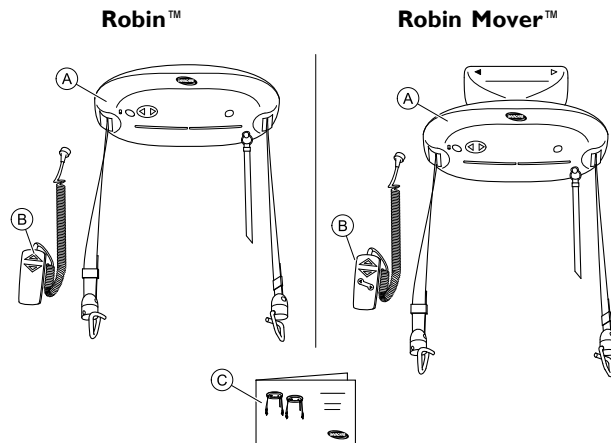- (A) Ceiling hoist including two lifting straps with hooks and an emergency strap. Robin Mover also includes the trolley.
- (B) Hand control
- (C) User manual

4.2 Before Use

CAUTION!

- The ceiling hoist is not intended to be used together with MEDICAL ELECTRICAL EQUIPMENT.

Before first use, all features of the hoist system must be inspected by specialized service technicians.

Perform a visual inspection of the ceiling hoist. If the packaging is damaged upon arrival, the ceiling hoist parts must be thoroughly examined for visible damage or defects. If there is any suspicion of damage, the ceiling hoist must not be used before specialized service technicians have approved it.

The emergency stop is activated during shipment. Reset the emergency stop before first use. See 5.8 Resetting the emergency stop or lowering feature, page 17.

It is possible that the hoist's battery is discharged on account of self-discharge. Charge the hoist's battery before first use. See 5.4 Charging the Battery, page 14.

4.3 Installing the ceiling hoist


The ceiling hoist has to be mounted to a rail system for use. The rail system has to be installed and approved according to EN 10535 by a specialized dealer or specialized service technician.

Installing Robin™


The Robin™ requires a trolley, holding the ceiling hoist in the rail system. The trolley has to be pre-installed by a specialized dealer or service technician.

To mount the ceiling hoist to the trolley, do the following:

1. Locate the hoist trolley at the Ø 32 mm round recess in the rail.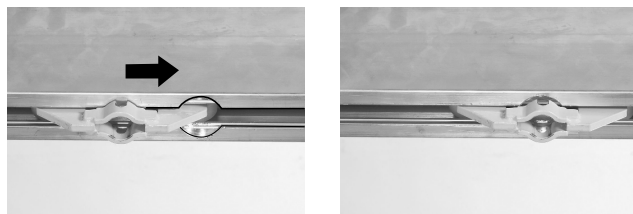

2. Lift the hoist and push the suspension pin through the trolley opening.
3. Turn the hoist by 90 degrees to fix it in the trolley.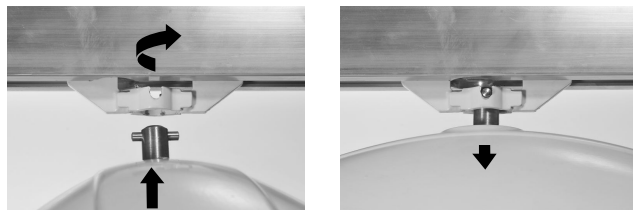


4. Adjust the length of the emergency stop/lowering cord to within the user's reach.

Installing Robin Mover™

Robin Mover™ is a ceiling hoist with the trolley already attached. The trolley can therefore not be pre-installed in the rail system.

The ceiling hoist has to be installed by a specialized dealer or service technician.

4.4 Inserting hand control plug

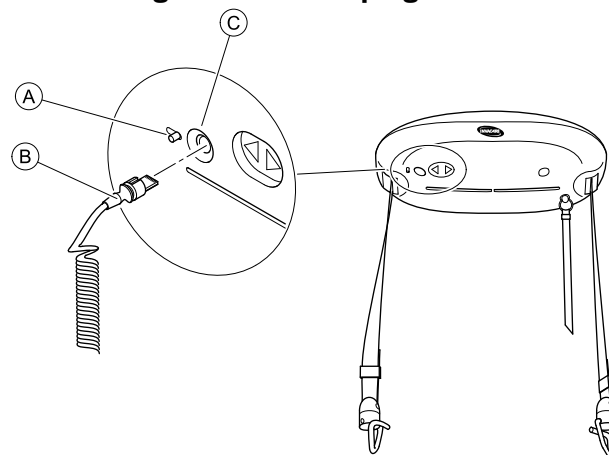1. Slide and hold locking lever **A** away from the aperture **C**. A gate in the aperture will open.
2. Make sure the plug **B** is in the correct orientation with the slot upwards (see image). Insert plug into aperture.
3. Release lever, so that the gate closes and hold the plug in the aperture.

Removing the hand control plug

1. Slide and hold locking lever **A** away from the aperture **C**.
2. Pull the plug **B** out of the aperture.
3. Release lever.

5 Usage

5.1 General operating information

CAUTION!

If the lifting straps are guided incorrectly in the hoist (e.g. the straps become twisted), the ceiling hoist will automatically turn off.

- Keep the straps straight to ensure they feed into the hoist correctly. The straps are held taught by the weight of the hooks.
- Keep the straps freely suspended when the unloaded hoist is lifting/lowering.

If a user unintentionally traps the hand in the lifting strap during lifting, the hoist will automatically turn off. The hoist stops immediately when a hand gets in contact with the strap inlet.

- Keep your hands away from the strap inlet during lifting.


CAUTION!

The hand control is protected against splashed water (IPX4). Extended contact with water will damage the hand control.

- Do not immerse the hand control in liquids, even for a short time.
- Avoid any contact with water or other liquids.

5.2 Operating the ceiling hoist

The ceiling hoist is in operation mode as soon as a button on the hand control is pressed.

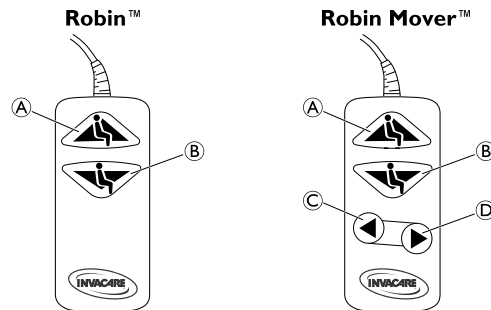Lifting and lowering the straps

When the ceiling hoist is not loaded, it will lift/lower at high speed. When loaded with more than 10 kg, the lifting/lowering speed is automatically reduced to a safe level.

1. To lift the straps, press the UP button (A) on the hand control.
2. To lower the straps, press the DOWN button (B) on the hand control.

The ceiling hoist can also be operated by pressing the UP and DOWN buttons on the underside of the hoist.

Traverse Robin™ along the track

1. Use the two lifting straps to pull the ceiling hoist along the track.

DO NOT use the red emergency strap to traverse the ceiling hoist.

Traverse Robin Mover™ along the track

- I. Perform one of the following:
 - Hoist is loaded: Press button LEFT © or RIGHT © on the hand control to traverse the ceiling hoist along the track.
 - Hoist is not loaded: Use the two lifting straps to pull the hoist along the track.

DO NOT use the red emergency strap to traverse the ceiling hoist.

It is not possible to activate both traversing and lifting/lowering function simultaneously. When the traversing function is activated, the blue indicator light on the bottom of the hoist is off. See 5.3 Indicator light and audio signal, page 14.

5.3 Indicator light and audio signal

Hoist

Status	Indicator light	Audio signal
Off / Stand by Hoist traversing (Robin Mover™ only)	Off	No
Hoist in use (Up/Down)	Blue	No
Low battery	Blue Flashing	Yes

The blue indicator light on the underside of the hoist turns off automatically after 30 seconds.

Wall Mounted Charger

Status	Indicator light	Duration
Battery not connected	Yellow	n/a
Initializing and analysis of battery	Yellow	12–14 seconds
Fast charge	Red	5 min — 3 hours
Charged (trickle charge)	Green	n/a
Fault condition (battery defect)	Alternating red and green	n/a

5.4 Charging the Battery

If the charge status becomes low, an audible indicator sounds and the indicator lamp on the bottom of the ceiling hoist flashes (see also 5.3 Indicator light and audio signal, page 14). When this happens, there is usually sufficient power to lower the patient.

It is not possible to use the hoist while the battery is charging.

- I. Finish the current lift in progress
If the current lift in process discharges the battery completely, an external backup battery may be used to temporarily power the lift. See 5.5 Using the external back-up battery, page 15.
2. Move the ceiling hoist to the charging station.
3. Place the hand control into the charging station.
A yellow indicator lamp on the charger indicates the hand control is connected. The indicator lamp illuminates red while charging.

4. When fully charged, the indicator lamp on the charger turns green.
As long as the ceiling hoist is not in use, leave the hand control in the charging station.
5. To use the ceiling hoist, remove the hand control from the charger.

Keeping the batteries charged guarantees hoist functionality and maintains the battery to ensure a long lifetime.

If the audible indicator beeps and the blue indicator on the hoist flashes after the hoist has been fully charged and only been in use for a short period of time, the battery may need exercise or replacement. See 9.5 Battery Maintenance, page 25.

If the ceiling hoist will not be used for more than 4 weeks, Invacare recommends disconnecting the charger and pulling the emergency stop.


The charger is connected to the mains with a mains plug. Make sure that the mains plug is accessible and can be unplugged if required.

5.5 Using the external back-up battery

If the ceiling hoist has to be used immediately with its battery discharged, an external back-up battery can be used.

Use the external back-up battery only as an exception and when absolutely necessary. The external back-up battery is available as an accessory.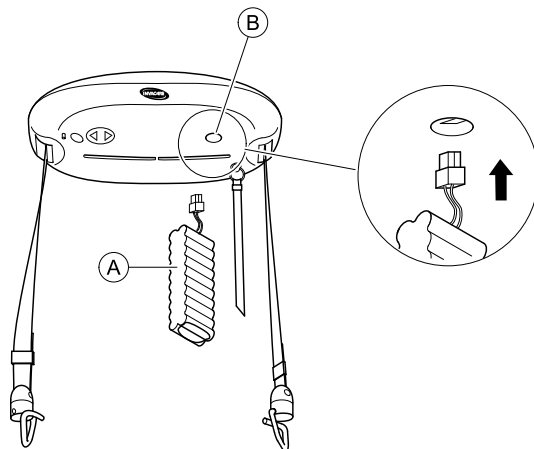


1. Insert the plug of the external back-up battery (A) into aperture (B) underside of the hoist. The external back-up battery should hang freely during patient transfer.
2. Operate the ceiling hoist as described in chapter 5.2 Operating the ceiling hoist, page 13.
3. Recharge the external back-up battery in order to have it fully charged for the next usage.

Charging the external battery:


1. Place the plug in the charger socket (a special adapter is necessary).
2. The LED light turns green to indicate the unit is fully charged (approx. charging time 2 hours).

5.6 Emergency stop

The emergency stop function is used to stop the patient from lowering or lifting if the ceiling hoist does not stop or react to the hand control.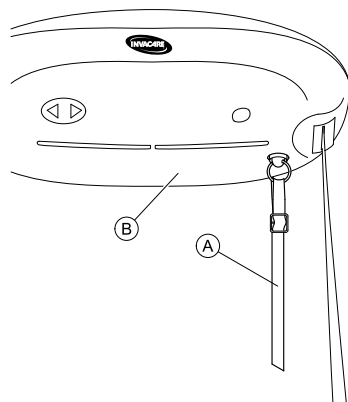

1. Pull and release the red emergency strap (A) underside of the hoist (B) to activate the emergency stop.
The ceiling hoist stops any lifting or lowering movement immediately.

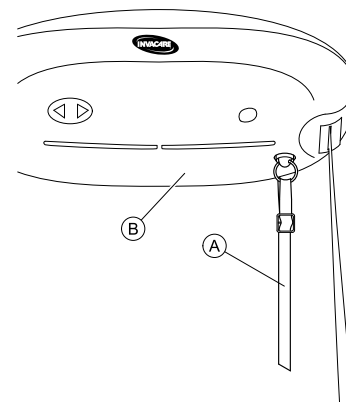1. Pull and hold the red emergency strap (A) underside of the hoist (B) to lower the patient.
Keep on pulling, until the patient is lowered to a safe position.
2. Release the red emergency strap to stop lowering the patient.

IMPORTANT!

When the red emergency strap has been pulled, the hoist functions, including the end stop, are deactivated. The blue indicator light turns off.

- To activate the hoist functions again, reset the emergency stop. See 5.8 Resetting the emergency stop or lowering feature, page 17.

5.7 Emergency lowering

The emergency lowering function is used to lower the patient only if the hoist has stopped working.

IMPORTANT!

When the red emergency strap has been pulled, the hoist functions, including the end stop, are deactivated. The blue indicator light turns off.

With the end stop function deactivated, the lifting straps will not stop automatically when lowered to their full length. Continued lowering could result in rolling up the lifting straps unevenly and in the wrong way.


- Pull the red emergency strap only until the patient is lowered to a safe position.
- DO NOT lower the lifting straps to their full length or more.
- To activate the hoist functions again, reset the emergency stop. See 5.8 Resetting the emergency stop or lowering feature, page 17.

5.8 Resetting the emergency stop or lowering feature

WARNING!

- If the emergency stop or emergency lowering must be used, there is a problem with the system. Contact your dealer or Invacare representative for service before using the system again.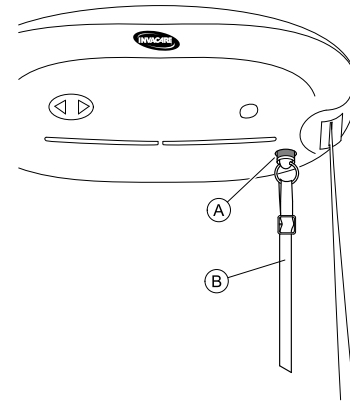

When the red emergency strap ⑥ has been pulled, the reset button ④ pops out so that a red ring is visible. The hoist will not function until the emergency stop or lowering feature is reset.

1. Press the reset button ④ to reset the emergency stop or lowering feature.
2. Activate the hand control to resume use of the hoist.

6 Lifting the Patient

6.1 Sling Information

WARNING!

Invacare shall not be liable for damage or injury that may occur as a result of using slings made by other manufacturers.

Invacare shall not be liable for damage or injury due to incorrect use of the sling or inadequate attention by the caregiver or user.

- Use the sling that is recommended by the individual's healthcare team for the comfort and safety of the individual that is being lifted. If there is any doubt about the selection or use of a sling, please contact your Invacare dealer.

A sling with four to six straps designed for mounting on hooks should be used when using the ceiling hoist.

Refer to the sling user manual for more details about the slings. Contact your Invacare Representative for more information.


6.2 Attaching the Sling

CAUTION!

Be careful when attaching the sling on the hooks.

- Check that the sling straps have been positioned completely onto the hooks.
- When pressing the up button to lift the patient, check again that all the sling straps remain correctly placed in the hooks.

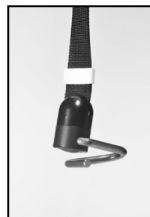The sling straps are to be hooked in pairs (one from the back and one from the leg) on their own suspension hook.

When attaching the sling to the hooks, and the straps are twisted, the straps will rotate and stretch out, thus ensuring trouble-free operation.

The information in this section is only a general guideline. Refer to the Invacare sling DVD and the sling user manuals in how to apply each of the Invacare sling models. Contact Invacare with any questions regarding sling attachment.

6.3 Lifting Information

WARNING!

Only personnel who have received training regarding the use of lifting equipment and fitting of slings should use the hoist.

- Plan the move. Avoid leaving the client in the sling unattended.
- The ceiling hoist lifts quickly. Before lifting, check that the client is completely free of his/her surroundings.
- The patient's body parts must not be in danger of becoming trapped.
- Check that the red emergency strap, hand control and hand control cable are free of the lifting straps, client and other objects before the ceiling hoist is activated and moved up or down.
- When the ceiling hoist is used correctly, the client should only be lifted to the extent that she/he is clear of the surface and should be moved at this height.

6.4 Lifting to and from a seated position

Follow this procedure when lifting a patient from a seated position (wheelchair, commode, etc.).

1. Traverse the ceiling hoist along the track towards the patient to be lifted.
2. Position the suspension hooks at the same height as the patient's chest and no closer than mid-thigh.
3. Position the suspension hooks so they are parallel to the patient's shoulders.
4. Position the patient in the sling. Refer to the sling user manual.
5. Attach the sling to the suspension hooks. Refer to 6.2 Attaching the Sling, page 18.

6.5 Lifting to and from a lying position

Follow this procedure when lifting a patient from a lying position.

1. Traverse the ceiling hoist along the track towards the patient to be lifted.
2. Position the suspension hooks above the patient to be lifted.
3. Align the suspension hooks with the patient's shoulders.
4. Position the patient in the sling. Refer to the sling user manual.
5. Elevate the head of the bed, so that the user is sitting up.
6. Attach the sling to the suspension hooks. Refer to 6.2 Attaching the Sling, page 18.

7 After Use

7.1 Transport and Storage

Invacare recommends that the ceiling hoist is always transported and stored in the original packaging. Place the ceiling hoist on a soft surface, such as a cloth or foam rubber mat.



For long term storage, the emergency stop must be activated. This will reduce the discharge of the battery. Refer to 5.6 Emergency stop, page 15.

For storage conditions of the ceiling hoist, refer to 10.3 Environmental conditions, page 27.

7.2 Dismounting the ceiling hoist

The ceiling hoist can be dismounted for transport or storage.

Robin™

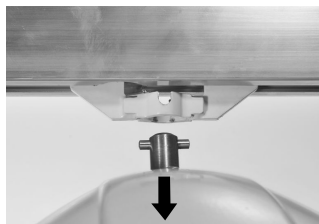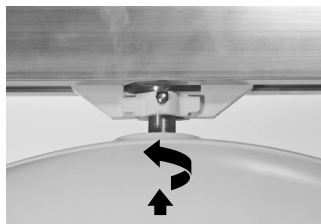1. Lift the hoist slightly and push the suspension pin out of its holder in the trolley.
2. Turn the hoist by 90 degrees and pull it out of the trolley.

Robin Mover™

Contact your Invacare representative or specialized dealer to have the ceiling hoist dismounted by a specialized service technician.

I586745-B

7.3 Disposal

WARNING!

Environmental Hazard

Device contains batteries.

This product may contain substances that could be harmful to the environment if disposed of in places (landfills) that are not appropriate according to legislation.

- DO NOT dispose of batteries in normal household waste. They **MUST** be taken to a proper disposal site. Contact your local waste management company for information.

Environmental Policy Statement

Invacare® is continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

- We comply with the current environmental legislation (e.g. WEEE and RoHS directives).
- We only use REACH compliant materials and components.

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

8 Troubleshooting

8.1 Troubleshooting Table

SYMPTOMS	PROBLEM	SOLUTION
Ceiling hoist does not respond to the hand control's buttons and up/down buttons on the ceiling hoist are not responding.	Emergency stop activated	Check that the emergency stop is not activated. Refer to 5.8 Resetting the emergency stop or lowering feature, page 17.
	System does not have power or battery is discharged	Charge the battery or use emergency battery. Refer to 5.4 Charging the Battery, page 14.
	Fuse is blown	The fuse needs to be replaced. Contact an Invacare representative or specialized dealer for service.
Ceiling hoist does not respond to the hand control's buttons, but up/down buttons on the ceiling hoist are responding.	Hand control not connected properly	Check the hand control plug. Refer to 4.4 Inserting hand control plug, page 12.
	Hand control defective	Replace the hand control.
Lifting straps will only move down and not up.	Battery nearly discharged	Charge the battery or use emergency battery. Refer to 5.4 Charging the Battery, page 14.
	Load is too high	Reduce the load. The hoist is equipped with an over-current safety system.
Lifting straps will only move up and not down.	Hand control defective	Replace the hand control.
	Loose strap function activated	Contact an Invacare representative or specialized dealer for service.
Ceiling hoist speed is the same with and without load.	Non-load quick movement function activated permanently	Contact an Invacare representative or specialized dealer for service.
Lifting straps are twisted and do not straighten out.	Lifting hooks do not turn freely enough	Clean or replace the hooks. Contact an Invacare representative or specialized dealer for service.
	Straps are frayed	Contact an Invacare representative or specialized dealer to replace the straps.

SYMPTOMS	PROBLEM	SOLUTION
Ceiling hoist does not traverse the rails.	Rails require service or cleaning	Contact an Invacare representative or specialized dealer for service.
	Trolley worn or damaged (Robin™ only)	Trolley must be replaced. Contact an Invacare representative or specialized dealer for service.
	Driving wheels worn or damaged (Robin Mover™ only)	Driving wheels must be adjusted. Contact an Invacare representative or specialized dealer for service.
	Hand control defective (Robin Mover™ only)	Replace the hand control.
Ceiling hoist emits excessive noise when activated.	Bearings, gearwheels or motor defective	The hoist must be replaced. Contact an Invacare representative or specialized dealer for service.
Ceiling hoist will not charge	System has no power	Check that the power supply is switched on and connected to the charging station.
		Contact an Invacare representative or specialized dealer for service.
No indicator light on hoist	System has no power or battery is discharged	Check that the ceiling hoist has power supply and that the battery is recharged.
		Place the hand control in the charging station and recharge the ceiling hoist.
The emergency strap does not turn ceiling hoist off	Ceiling hoist requires service.	Contact an Invacare representative or specialized dealer for service.
The emergency strap does not lower the ceiling hoist.	Ceiling hoist requires service.	Contact an Invacare representative or specialized dealer for service.

If problems are not remedied by the suggested means, contact an Invacare representative or specialized dealer for service.

9 Maintenance

9.1 Safe Maintenance

WARNING!

Regular maintenance of patient lifts and accessories is necessary to assure proper operation.

- Maintenance **MUST** be performed only by a specialized service technician.

9.2 Maintenance and Safety Inspection

Service Interval

At normal daily operation, a service check-up should take place every year, according to the Safety Inspection Checklist in the Service Manual. When performing annual or regular maintenance, all parts designed to carry load must be, as a minimum, tested with load. All safety features must be checked according to EN ISO 10535 Annex B. This inspection must be performed by a specialized technician.

General Maintenance

Follow the maintenance procedures described in this manual to keep your patient lift in continuous service.

The hoist is designed to provide a safe, efficient and satisfactory service with minimum care and maintenance.

It is important to inspect slings, hanger bar (optional), lifting straps and strap hooks for signs of cracking, fraying, deformation or deterioration. If damage is found, **DO NOT** use. Contact your Invacare representative or specialized dealer **IMMEDIATELY**.

If you question the safety of any part of the lift, contact your Invacare dealer immediately and advise him/her of your problem.

Replacement of parts must only be performed by a specialized technician.

Daily Inspections

The hoist should be checked each time it is used. Perform the following checks in addition to those performed in the annual service inspection. If you question the safety of any part of the lift, **DO NOT** use. Contact your Invacare representative or specialized dealer **IMMEDIATELY**.

- ☐ Visually inspect slings, hanger bar (optional), lifting straps and strap hooks. Check all parts for damage or wear. If damage is found, **DO NOT** use. Contact your Invacare representative or specialized dealer **IMMEDIATELY**.
- ☐ Check the emergency stop and emergency lowering function. Check all parts for external damage or wear. If damage is found, **DO NOT** use. Contact your Invacare representative or specialized dealer **IMMEDIATELY**.
- ☐ Check that all hardware and attachment points for damage or wear. Check all parts for external damage or wear. If damage is found, **DO NOT** use. Contact your Invacare representative or specialized dealer **IMMEDIATELY**.
- ☐ Verify that the hand control is functional.
- ☐ Check the battery charger.
- ☐ Charge the battery every day the lift is used.

Safety Inspection Checklist

The annual safety inspection checklist is available in the service manual.

9.3 LOLER Statement

The UK Health and Safety Executive's Lifting Operations and Lifting Equipment Regulations 1998, require any equipment that is used in the workplace to lift a load be subject to safety inspection on

a six monthly basis. Please refer to the HSE web site for guidance www.hse.gov.uk.

The person responsible for the equipment must ensure adherence to LOLER regulations.

9.4 Cleaning and disinfection

Cleaning and Disinfecting the Hoist

CAUTION!

- To prevent cross-infection, the parts of the hoist that come in contact with caregivers and patients must be cleaned and disinfected on a daily basis.
- Any parts that come in contact with the patient must be cleaned and disinfected before the hoist is used for a next patient.

A soft cloth, dampened with water and a small amount of dish liquid, is all that is needed to clean the hoist, lifting strap and strap hooks. The hoist can be cleaned with non-abrasive cleaners.

Never use acids, alkaline or solvents for cleaning the hoist. Dry the hoist carefully after cleaning.

The hoist, hand control and mounting parts can be damaged if the hoist is cleaned any other way than stated above.

For disinfection, the hoist must be wiped with a moistened, firmly wrung cloth with ordinary household disinfectants. Only use disinfection detergents approved by the facility.

Cleaning the Sling

Refer to the washing instructions on the sling and to the sling manual for cleaning details.

9.5 Battery Maintenance

If the audible indicator beeps and the blue indicator on the hoist flashes after the hoist has been fully charged and only been in use for a short period of time, the battery may need to be exercised.

To enable the battery to charge fully, the battery should be exercised from time to time.

Following a long period of storage, it could also be necessary to exercise the battery.

It is also suggested that the battery is exercised in situations where the hoist is used infrequently, and/or where the hand control is placed in the charger for long periods of time. In this situation, there will always be capacity enough for a few lifts, but we recommend to exercise the battery as soon as possible afterwards.

Exercise of the battery can be done by charging and discharging the hoist completely several times.

1. Charge the battery until the LED turns green.
2. Remove the hand control from the charger.
3. Raise and lower the straps with a load for one minute.
4. Wait ten minutes.
5. Repeat Steps 3–4 until the battery is completely discharged.


Continue this process after the audible alert sounds and the blue light flashes, indicating the battery is low.

6. Repeat Steps 1–5 two to three times.

10 Technical data

10.1 Dimensions and weight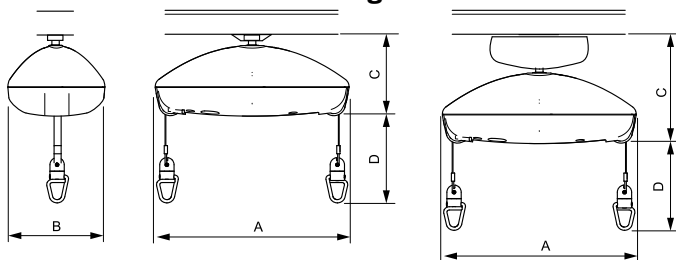

Dimensions

	Robin™	Robin Mover™
Hoist length (A)	470 mm	470 mm
Hoist width (B)	250 mm	250 mm
Hoist height (C)	203 mm	266 mm
Maximum lifting range (D)	2.5 m	2.5 m

Weights

	Robin™	Robin Mover™
Maximum lifting capacity (Safe Working Load)	200 kg	200 kg
Total weight without sling	7.3 kg	10 kg

10.2 Electrical System

	Robin™	Robin Mover™
Maximum current input	0.9 A	
Protection class ¹	Entire device: IPx4 (ceiling hoist: IP24; hand control: IP44)	
Insulation class	Class II equipment	
	Type B applied part Applied part complying with the specified requirements for protection against electrical shock according to IEC60601-1.	
Sound level	50 – 55 dB (A)	
Number of lifts per charge	65 lifts of 0.5 m with 80 kg	
Intermittent (periodic motor operation)	10%, max. 2 min. / 18 min.	
Battery	NiMH Battery 24V / 2.0 Ah	
Charging time	Up to 3 hours at 20°C	
Lifting speed with 200 kg load	2.1 cm/s	2.1 cm/s
Traverse speed in the tracks	n/a	14 cm/s

Charger

Voltage output charger	24 V DC, max. 0.9 A
Voltage supply charger	230 V AC, 50 Hz
Protection class ¹ charger	IP41 or IP67 (depending on version)

¹ See product label and label on each electric device for correct protection class. The lowest IP classification decides the overall classification of the device.

- IPx4: Protected against water splashed from any direction.
- IP24: Protected against objects larger than 12.5 mm and protected against water splashed from any direction.
- IP44: Protected against objects larger than 1 mm and protected against water splashed from any direction.
- IP41: Protected against objects larger than 1 mm and protected against vertically dripping water.
- IP67: Dust-tight and protected against temporary immersion into water.

10.3 Environmental conditions

	Storage and transportation	Operation
Temperature	-30 °C to +50 °C	+5 °C to +40 °C
Relative humidity	10 % to 70 %	20 % to 90 % at 30 °C, not condensing
Atmospheric pressure	700 hPa to 1060 hPa	

10.4 Electromagnetic compliance (EMC) information

Medical Electrical Equipment needs to be installed and used according to the EMC information in this manual.

This equipment has been tested and found to comply with EMC limits specified by IEC/EN 60601-1-2 for Class B equipment.

Portable and mobile RF communications equipment can affect the operation of this equipment.

Other devices may experience interference from even the low levels of electromagnetic emissions permitted by the above standard. To determine if the emission from the lift is causing the interference, run and stop running the lift. If the interference with the other device operation stops, then the lift is causing the interference. In such rare cases, interference may be reduced or corrected by the following:

- Reposition, relocate, or increase the separation between the devices.

10.4.1 Guidance and manufacturer's declaration

Test	Basic Standard	Compliance Level / Frequency Range
Mains Port Conducted Emissions	EN 55011:2007	Group I Class B
Radiated Emissions	EN 55011:2007	Group I Class B
Mains Harmonics	EN 61000-3-2:2006 + A1:2009 + A2:2009	Class A

Test	Basic Standard	Compliance Level / Frequency Range
Voltage Fluctuations	EN 61000-3-3:2008	dc%/dmax/d(t)/Pst
Electrostatic Discharge	EN 61000-4-2:1995	±2/4/6kV Contact, ±2/4/8kV Air
Radiated Field Immunity	EN 61000-4-3:2006	3V/m, 80% 1kHz AM, 80MHz - 2.5 GHz
EFT/Bursts	EN 61000-4-4:2004	±2kV L/N/E/LNE
Surge Immunity	EN 61000-4-5:2006	±0.5/1kV L-N, ±0.5/1/2kV L-E/N-E
Conducted RF Immunity	EN 61000-4-6:2007	3V, 80% 1kHz AM, 0.15MHz - 80MHz
Magnetic Field Immunity	EN 61000-4-8:1993	3A/m, 50Hz & 60Hz
Dips & Interruptions	EN 61000-4-11:2004	-95%/-60%/-30%

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Invacare Ireland Ltd,
Invacare AB