
**Yes, you can.<sup>®</sup>**

## *Invacare<sup>®</sup> Sit-up support*

---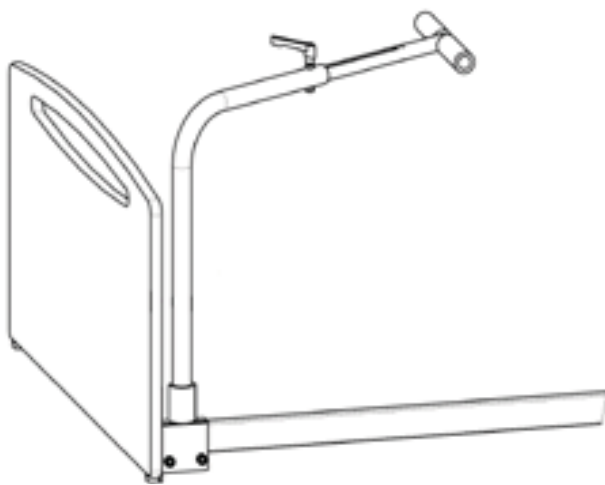

***User Manual (GB)***

***Brugermanual (DK)***

***Bruksanvisning (SE)***

***Brukerveiledning (NO)***

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***Manuale d'uso (IT)***

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## Sit-up Support

### Installation:

The mounting bracket is clamped onto the side frame of the bed in the same side as the user gets in and out of the bed. The clamp must be placed as close to the foot end as possible, so that the distance between the vertical tube and the bed end is less than 6 cm (Fig. 1).

The grooves in the brackets must be on the inside, towards the bed.

Tighten the two bolts well.

On some bed models, it can be necessary to loosen the screw for the bed end bracket while the bracket for the support is being installed. Refasten afterwards.

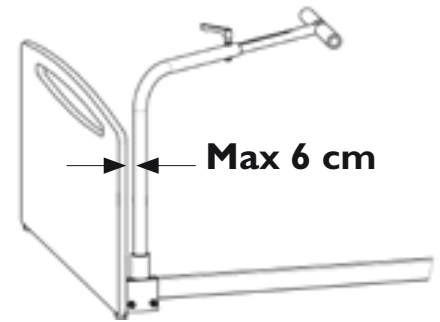
Fig. 1

### Adjusting the handle:

Loosen the grip on top of the arm, and adjust the length of the handle. The handle cannot be pulled out.

Retighten the grip afterwards.

### Use:

Be aware that the pin in the arm is locked into the groove of the mounting bracket, so that the handle is over the centre of the bed (Fig. 2).

When the sit-up support is not in use, it can be turned across the bed by lifting it slightly.

At least two wheels must be locked, when the Sit-up support is used.

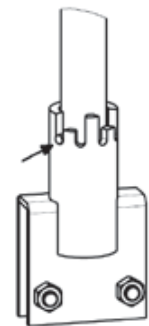
Fig. 2

Always respect the safety distances. If the distance between the bed end and the arm of the sit-up support is between 6 cm and 32 cm, there is risk of serious injury due to entrapment or suffocation.

Tighten the screws. If this is not done, there is risk that the handle will loosen and the user can fall or get entrapped.

The arm must not be turned so that the handle is outside the bed; there is risk of overturning the bed.

Do not load the handle with more than 60 kg.

### Application:

This Sit-up support can be used on the following bed types: *Etude*, *SB400™*, *SB750™* and *SB755™*.

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#### **INVACARE AUSTRIA GmbH**
#### **INVACARE EC-Høng A/S**