**SAVE THESE INSTRUCTIONS**

*NOTE: Check all parts for shipping damage. If damage is found, DO NOT use. Contact Carrier/Invacare for further instructions.*

**Safety Summary**

To ensure the safe and proper assembly of the cart, these instructions and warnings **MUST** be followed:

**⚠ WARNING**

DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

After any adjustments, repair or service and before use, make sure all attaching hardware is tightened securely - otherwise injury or damage may result.

**⚠ ACCESSORIES WARNING**

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

**Cart Assembly**

*NOTE: The following tools are required to perform this procedure:*


- #3 Phillips Head Screwdriver
- 10mm Wrench

1. Remove base assembly, handle assembly, and hardware packet from packaging.

*NOTE: Hardware packet should contain two each button head screws, nuts, screw retainers, and thumb screws.*

2. Insert handle assembly posts into square brackets located on the base assembly. Refer to FIGURE 1 (Detail "A").
3. Adjust handle and base assemblies until screw holes are directly aligned. Refer to FIGURE 1 (Detail "B").


For more information regarding Invacare products, parts, and services, please visit [www.invacare.com](http://www.invacare.com)

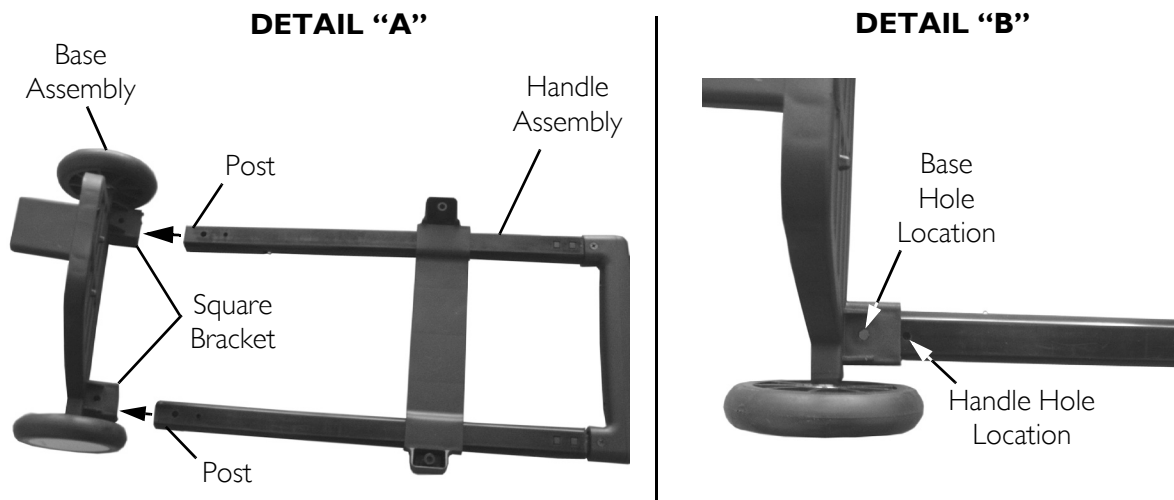**FIGURE 1** Inserting Handle into Base Assembly


4. Insert screw through base assembly bracket, ensuring screw protrudes through opposite side of bracket facing toward base assembly platform (FIGURE 2).
5. Thread nut onto screw and tighten securing the handle assembly to the base assembly (FIGURE 2).

*NOTE: Repeat STEP 4 thru STEP 5 on the opposite side of base assembly.*

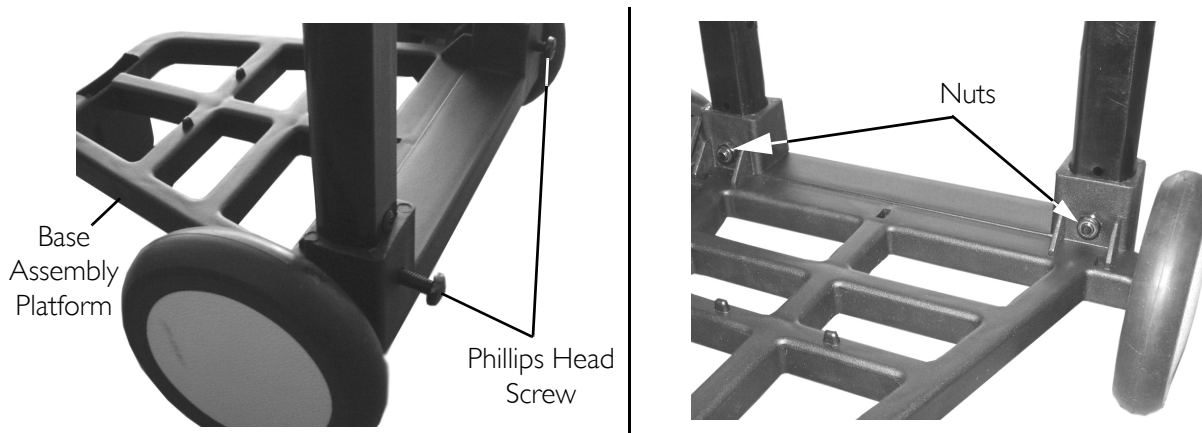
**FIGURE 2** Securing Handle to Base Assembly

6. Insert the two thumb screws into bracket located on the handle assembly (FIGURE 3).
7. Secure screws with retaining rings (FIGURE 3).

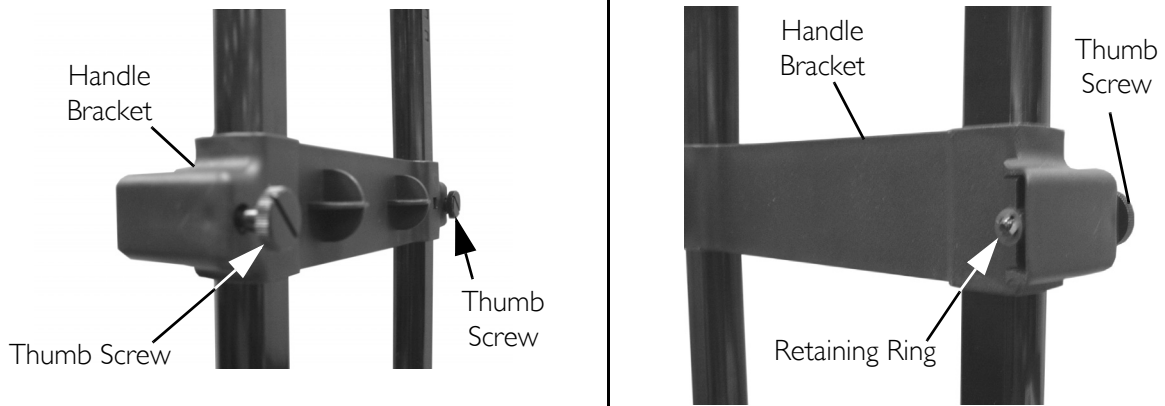
**FIGURE 3** Attaching Handle Bracket

8. Position concentrator unit onto base assembly by aligning holes in the bottom side of the unit with protrusions on cart base (FIGURE 4).

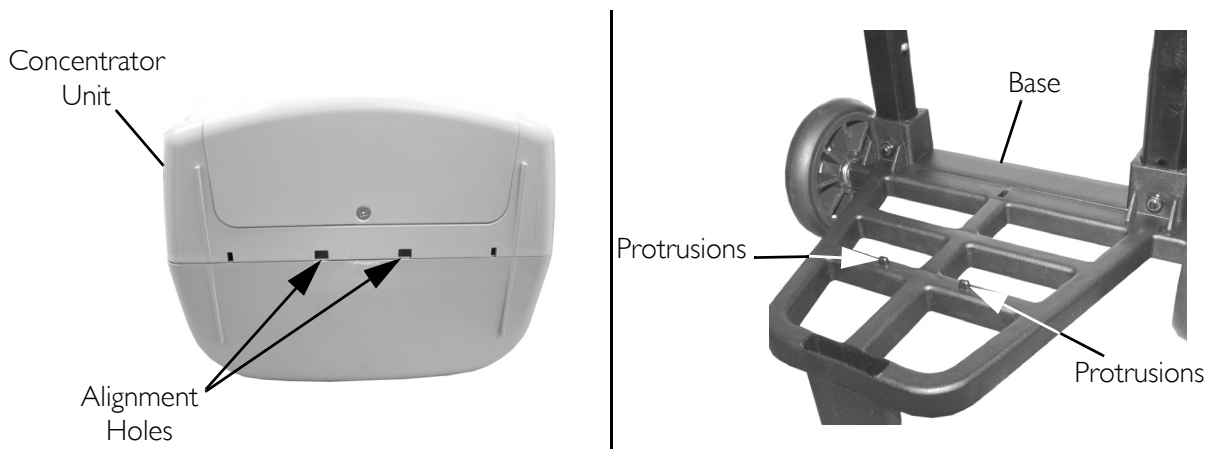**FIGURE 4** Positioning Concentrator Unit onto Cart

- 
9. Align thumb screws with holes located on the back of concentrator unit, hand tighten screws to secure the unit to the cart (FIGURE 5).

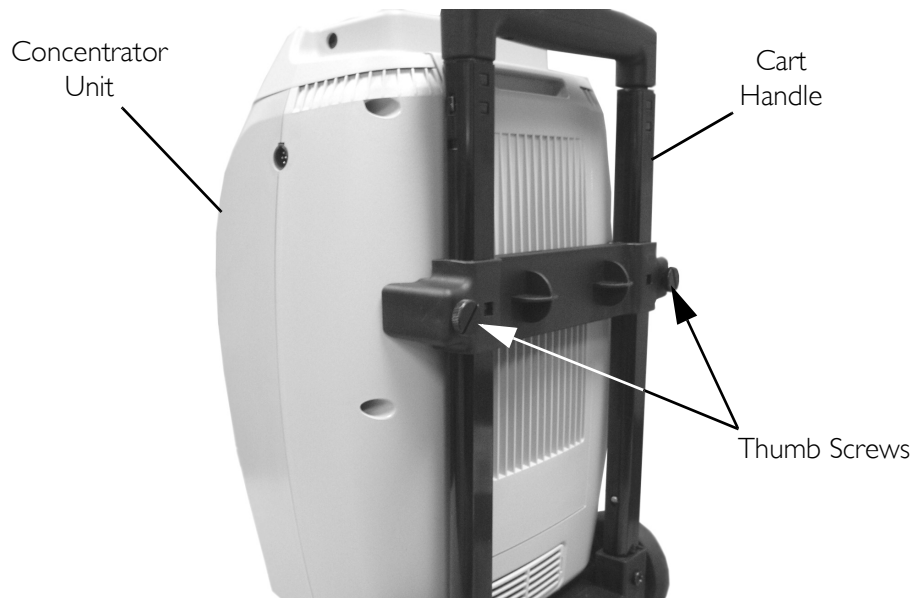**FIGURE 5** Securing Concentrator Unit to Cart

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