# NAVIGATING THE SOLO<sub>2</sub><sup>™</sup> LCD SCREEN

## QUICK REFERENCE GUIDE


*NOTE: This guide is meant to be used only as a brief, general reference. For complete operating instructions, always refer to the Operator's Manual.*

### Turning Concentrator On/Off

*NOTE: For this procedure, refer to FIGURE 1.1.*

1. To turn the SOLO<sub>2</sub> concentrator on, press and hold the On/Off button  until the three warning lights display and the unit beeps.
2. To turn the SOLO<sub>2</sub> concentrator off, press and hold the On/Off button .

*NOTE: GREEN light will flash and you will hear the machine stop running.*

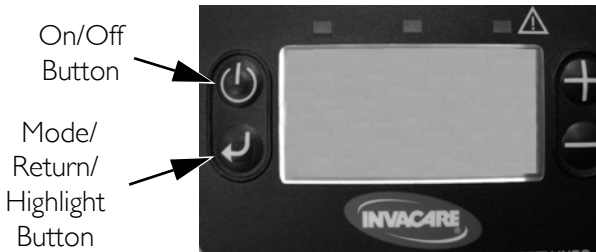**FIGURE 1.1** Turning Concentrator On/Off

### Menu Screens Navigation

*NOTE: For this procedure, refer to FIGURE 1.1.*

1. After turning on unit, press and hold the Mode/Return/Highlight Button  to scroll through the three SOLO<sub>2</sub> concentrator menu screens.

*NOTE: Screens will continue scrolling until the Mode/Return/Highlight Button is released.*

### Normal Operating/Standby Screen

The Normal Operating/Standby screen shows either battery strength gauge, fan operating, external power on and battery charging icons, when applicable, or the current/last mode of operation and output flow level selected.

---

## LCD Adjust Screen

The LCD Adjust screen allows user to change the brightness and contrast of the screen (FIGURE 1.2).

**FIGURE 1.2** LCD Adjust Screen

## LCD Adjust Screen Navigation

*NOTE: For this procedure, refer to FIGURE 1.2.*

1. Perform one of the following, while the LCD Adjust screen is displayed:
  - A. Tap the Return/Highlight Button  to move the black highlight bar from left to right, or right to left.
  - B. Tap the Up/Increase Button  or the Down/Decrease Button  while the black highlight bar is on the "LEFT SIDE" of the screen to move the highlight bar up or down.
  - C. Tap the Up/Increase Button  or Down/Decrease Button  while the highlight bar is on the "RIGHT SIDE" of the screen to change the value of the item highlighted.
  - D. Hold down the Mode/Return/Highlight Button  to continue scrolling through the three menu screens.

*NOTE: If no buttons are pressed for approximately 10 seconds, the screen will change back to the Standby/Normal Operating screen. Any changes made will NOT be saved.*

---

2. To save changes made on the LCD Adjust screen:

- A. Press and hold the Mode/Return/Highlight Button  until a different screen appears.


## Mode & Flow Adjust Screen

The Mode & Flow Adjust screen allows user to change the current mode of operation and output flow (FIGURE 1.3).

*NOTE: To adjust either the mode or flow, the Mode & Flow Adjust screen must be displayed. If the battery level screen is displayed, press the Mode/Return/Highlight , Up/Increase , or*

*Down/Decrease  button for approximately two seconds (screen should change to Mode & Flow Adjust).*

*NOTE: Any changes to the setting will not take effect until after the screen flashes three times.*

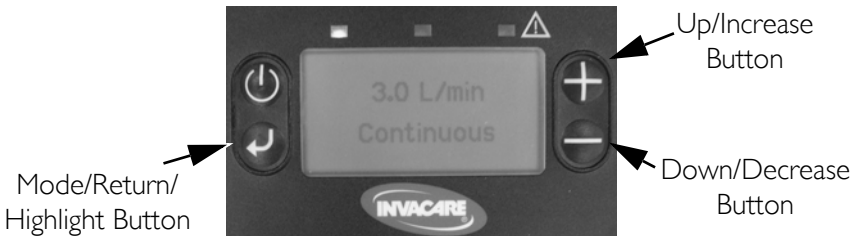**FIGURE 1.3** Mode & Flow Adjust Screen

---

## Changing Mode

*NOTE: For this procedure, refer to FIGURE 1.3.*

1. With the Mode & Flow screen displayed, press and hold the Return/Highlight Button  for one second.
2. Wait until the Mode & Flow screen flashes three times.

## Adjusting Flow

*NOTE: For this procedure, refer to FIGURE 1.3.*

1. With the Mode & Flow screen displayed, press and hold the Up/Increase Button  or Down/Decrease Button  until desired setting is achieved.
2. Wait until the Mode & Flow screen flashes three times.

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© 2010 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
**Invacare Corporation** [www.invacare.com](http://www.invacare.com)