


DOLOMITE ALPHA BASIC

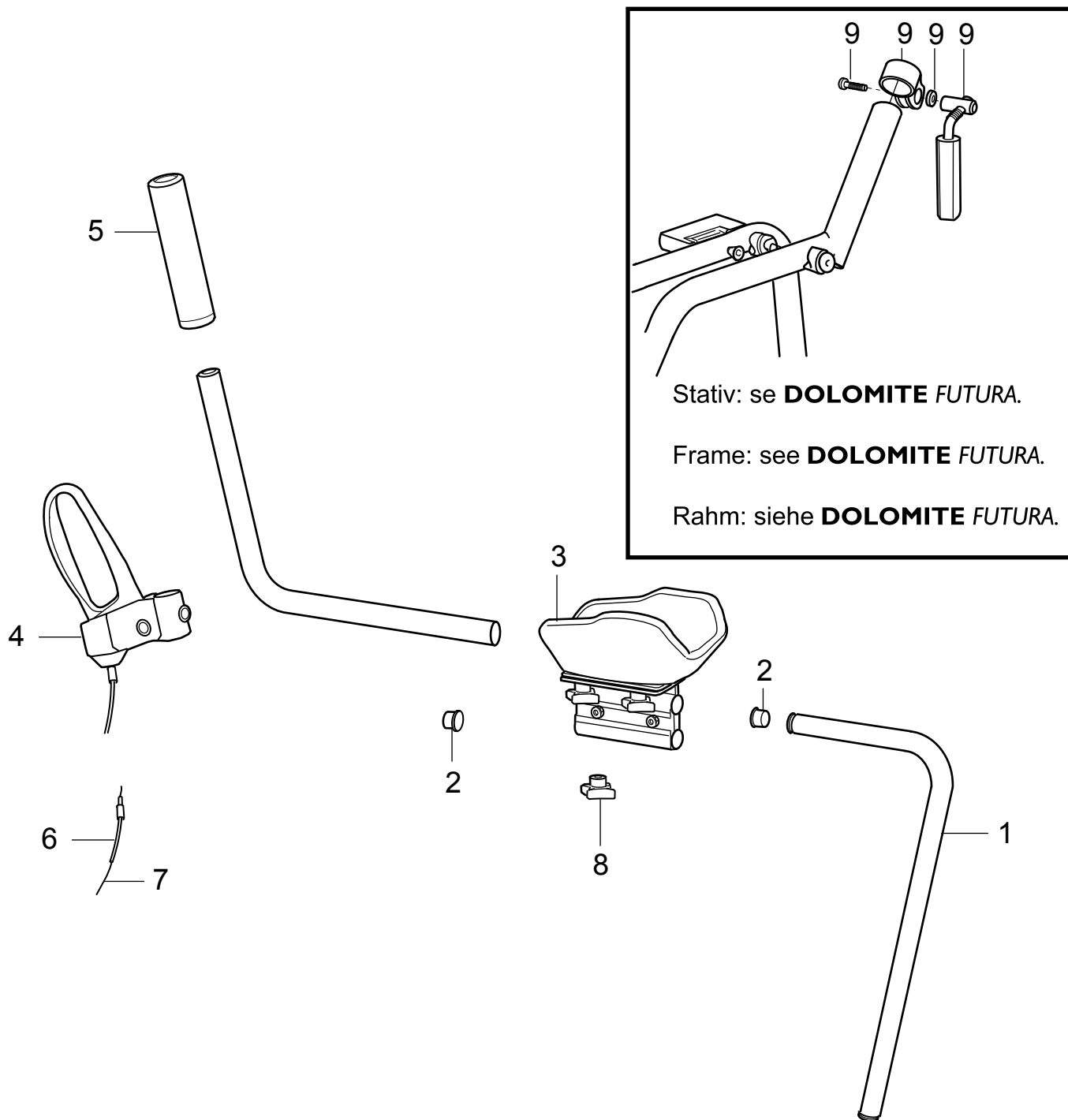

- | | | |
|---|---------|-----------------------------------|
| 1 | 13627 | Height adjustment tube |
| 2 | 1452465 | End plug 10-pack |
| 3 | 1452469 | Arm rest plate pair |
| 4 | 13098-L | Brake handle cpl, left |
| | 13098-R | Brake handle cpl, right |
| 5 | 12593 | Soft grip L=150 mm (pair) |
| 6 | 80075 | Wire cover L=1000 mm |
| 7 | 1452466 | Wire L=1330 mm, 10-pack |
| 8 | 1452467 | Knob 10-pack |
| 9 | 1452468 | Locking device with clamp, 2-pack |
| | 82056 | Operating instructions |