


Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
 Serial No. :

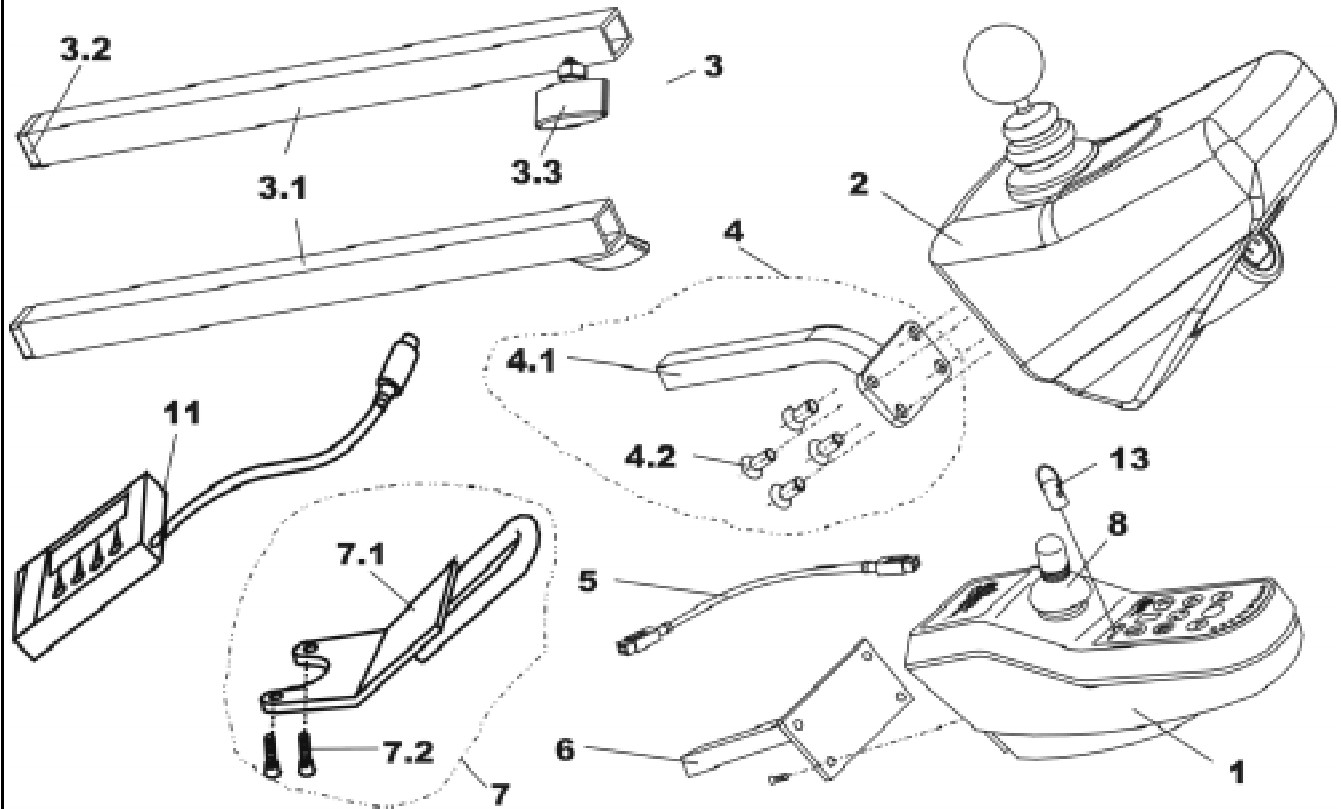


Item	Qty	Part-No.	Part-Designation	
-	1	SP1422779	ACS STD. REMOTE CPL.	->02/04
1	1	P10150	REMOTE REM 24 SC	->02/04
-	1	GSM9917	KEYPAD WITH UPPER HOUSING FOR REM 24SC	->02/04
-	-		The joystick box support for REM 24SD is not the same a...	
-	-		Please order the new version if you have the old one!	
-	1	1436541	REMOTE REM 24 SD	03/04->
-	1	SP1422797	REMOTE G80 I CPL.	
2	1	P10156	REMOTE REM G80 I	
13	1	SP1419695	MAGNETIC KEY W. POUCH CPL.	
3	1	SPS183040	MOTION ROD REMOTE CPL.	
3.1	1		GUIDE F. REMOTE STD.RAL 9005 DEEP BLACK	
3.2	1		TUBE END PLUG E 15 X 15	
3.3	1	L31001	HAND KNOB M6 X 10	
4	1	SPZ183302	REMOTE G80 I HOLDER CPL.	
4.1	1		REMOTE HOLDER F. >REMOTE REM G80	
4.2	4		COUNTERSUNK CROSS HEAD SCREW M5 X 12	
5	1	1419386	ACS BUS CABLE L=1900 MM	
6	1	F283001.P40	GUIDE F. REMOTE STD.RAL 9005 DEEP BLACK	
7	1	SP1422828	GUARD BOW F. REM 24 REMOTE CPL.	
7.1	1		GUARD BOW F. ACS REMOTE	
7.2	2		COUNTERSUNK CROSS HEAD SCREW M5 X 12	
8	1	Z183229	JOYSTICK KNOB CPL.	
11	1	6001450	ACS-HAND PROGRAMMER, "F; GER; ESP"	
-	1	P10062	WIZARD SOFTWARE	

Storm 3



Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
 Serial No. :

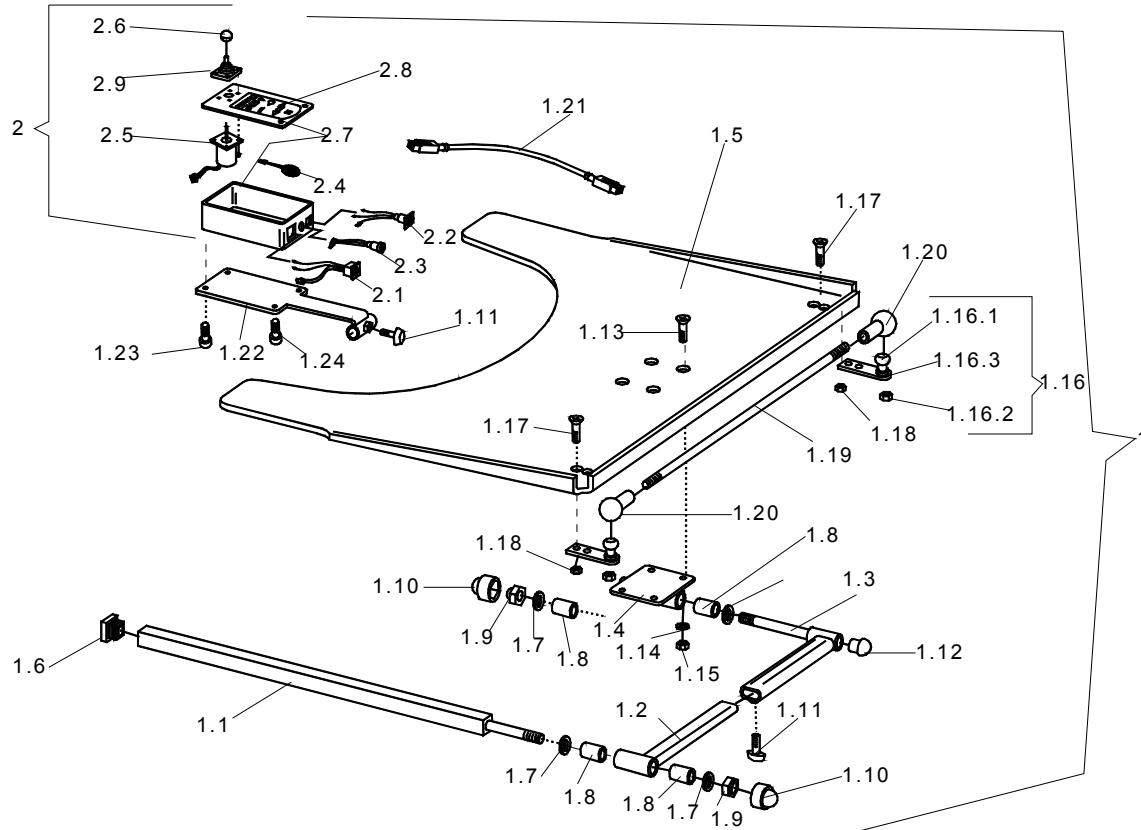


Item	Qty	Part-No.	Part-Designation
1	1	SP1422798	TRAY CONTROL CPL.
-	1	SPZ177011	TRAY SUPPORT FIXING KIT CPL.
1.1	1		SUPPORT TUBE; CHROME PLATED
1.2	1		SWING ARM; CHROME PLATED
1.3	1		SWING ARM EXTENSION; CHROME PLATED
1.4	1		TRAY SUPPORT BRACKET; CHROME PLATED
1.6	1		TUBE END PLUG E 15 X 15
1.7	4		WASHER 13 X 24 X 2,5
1.8	4		BUSH RD. 19 X 3.5 X 25
1.9	2		NYLOC NUT M12
1.10	2		NUT CAP M12
1.11	2		HAND KNOB M6 X 10
1.12	1		TUBE END PLUG SL 22
1.14	4		WASHER 6,4 X 12,5 X 1,6
1.15	4		DOMED NYLOC NUT M6, SELF-LOCKING
1.5	1	Z17024A	TRAY 660 X 540 X 8; TRANSPARENT
1.13	4		COUNTERSUNK CROSS HEAD SCREW M6 X 20
1.16	2	SP4100136	SUPPORT ROD ; CPL..
1.16.1	2		SWIVEL JOINT AS 13-M8 RH
1.16.2	2		NYLOC NUT M8 DIN 985
1.16.3	2		BRACKET 20 X 70 X 5 DIN-1652; ZINC PLATED
1.17	4		COUNTERSUNK CROSS HEAD SCREW M5 X 12
1.18	4		NYLOC NUT M5
1.19	1		ROD. M. L=560 MM
1.20	2		SWIVEL JOINT AS 13-M8 RH

Storm 3



Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
 Serial No. :

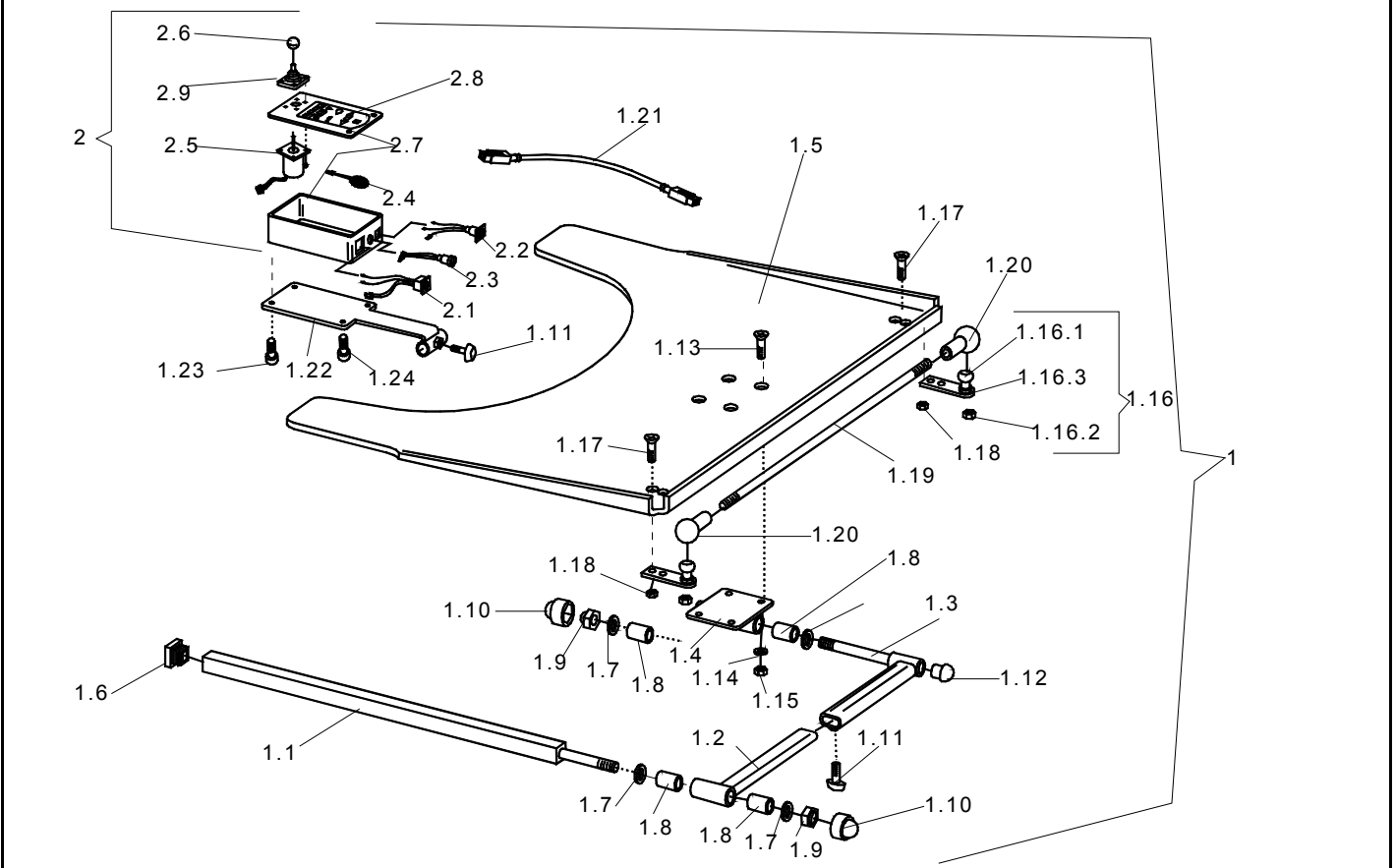


Item	Qty	Part-No.	Part-Designation
1.22	1		SUPPORT BRACKET
1.21	1	1423209	ACS BUS CABLE L=2900 MM
1.23	2	L60999	FOOT M4 X 13
1.24	2		COUNTERSUNK CROSS HEAD SCREW M4 X 10
2	1	P10066	ACS-TRAY CONTROL BOX
2.1	1	P41108	ACS-BUS CABLE SOCKET
2.2	1	P41106	CHARGER SOCKET
2.3	1	P41107	PROGRAMMER SOCKET
2.4	1	P30493	HORN
2.5	1	GSM55120	ACS-JOYSTICK F. TRAYCONTROL
2.6	1	L36999	KNOB 32 MM, HOLE 5,8 MM
2.7	1		ACS CONTROL BOX HOUSING
2.8	1	P30907	FOIL KEYPAD
2.9	1	Z183001A	RUBBER GAITER; BLACK
---	1	Q99081	OPERATION INSTRUCTION, TRAY-/CENTRAL STEERING

Storm 3



Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
 Serial No. :

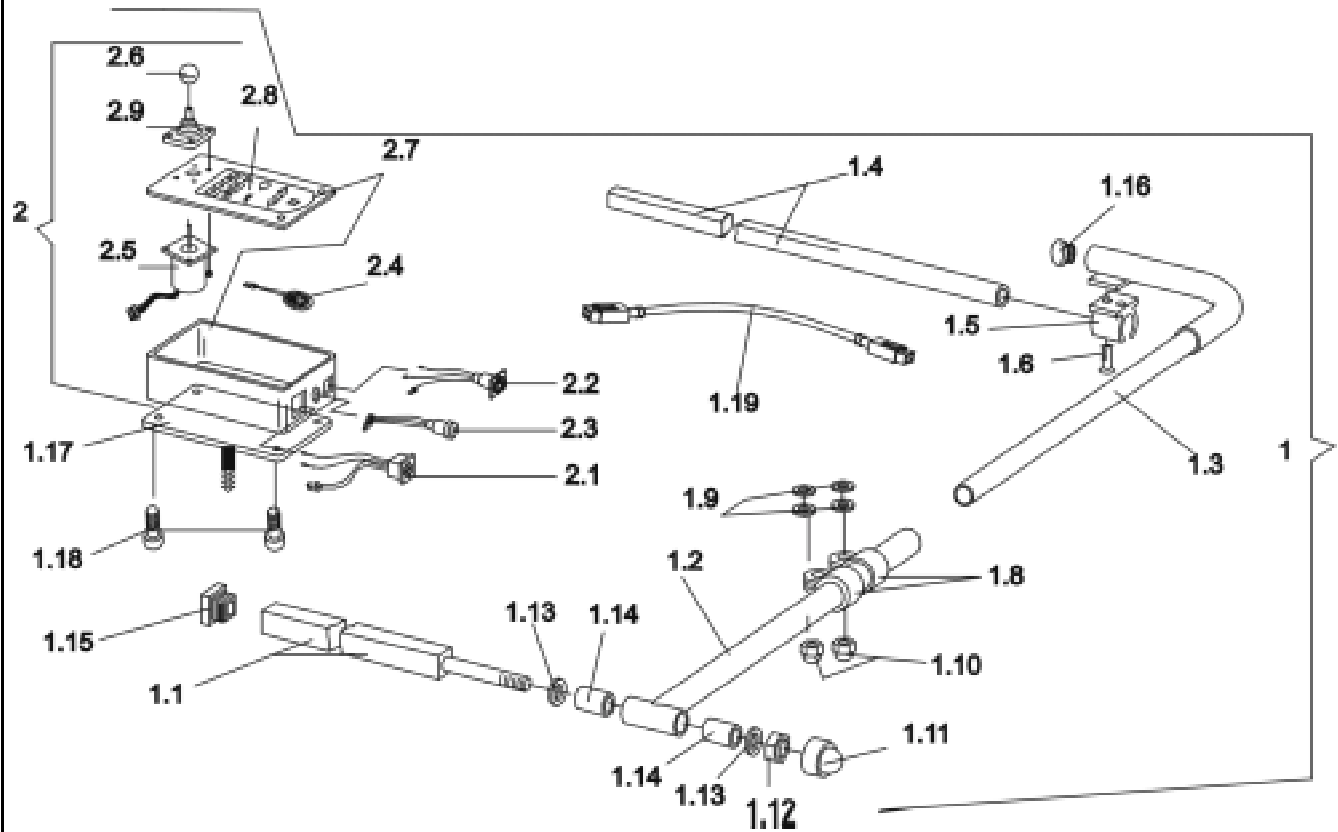


Item	Qty	Part-No.	Part-Designation
1	1	SP1422799	SWING-OUT MIDDLE CONTROL LH CPL.
1	1	SP1422800	SWING-OUT MIDDLE CONTROL RH CPL.
1.1	1		SUPPORT TUBE; CHROME PLATED
1.2	1		SWING ARM; CHROME PLATED
1.3	1		SUPPORT ARM LH; CHROME PLATED
1.3	1		SUPPORT ARM RH; CHROME PLATED
1.4	1		MIRROR SUPPORT RAIL; CHROME PLATED
1.5	1	Z16001A	CLAMP
1.6	2		COUNTERSUNK CROSS HEAD SCREW M5 X 12
1.8	2		CLAMP 22 MM
1.9	4		WASHER 8,4 X 17 X 1,6
1.10	2		DOMED NYLOC NUT M8 X 1
1.11	1		NUT CAP M12
1.12	1		NYLOC NUT M12
1.13	2		WASHER 13 X 24 X 2,5
1.14	2		BUSH RD. 19 X 3.5 X 25
1.15	1		TUBE END PLUG E 15 X 15
1.16	2		TUBE END PLUG SL 22
1.17	1		SUPPORT BRACKET; RAL 9005 BLACK
1.18	4		COUNTERSUNK CROSS HEAD SCREW M4 X 10
1.19	1		ACS BUS CABLE L=2900 MM

Storm 3



Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
Serial No. :

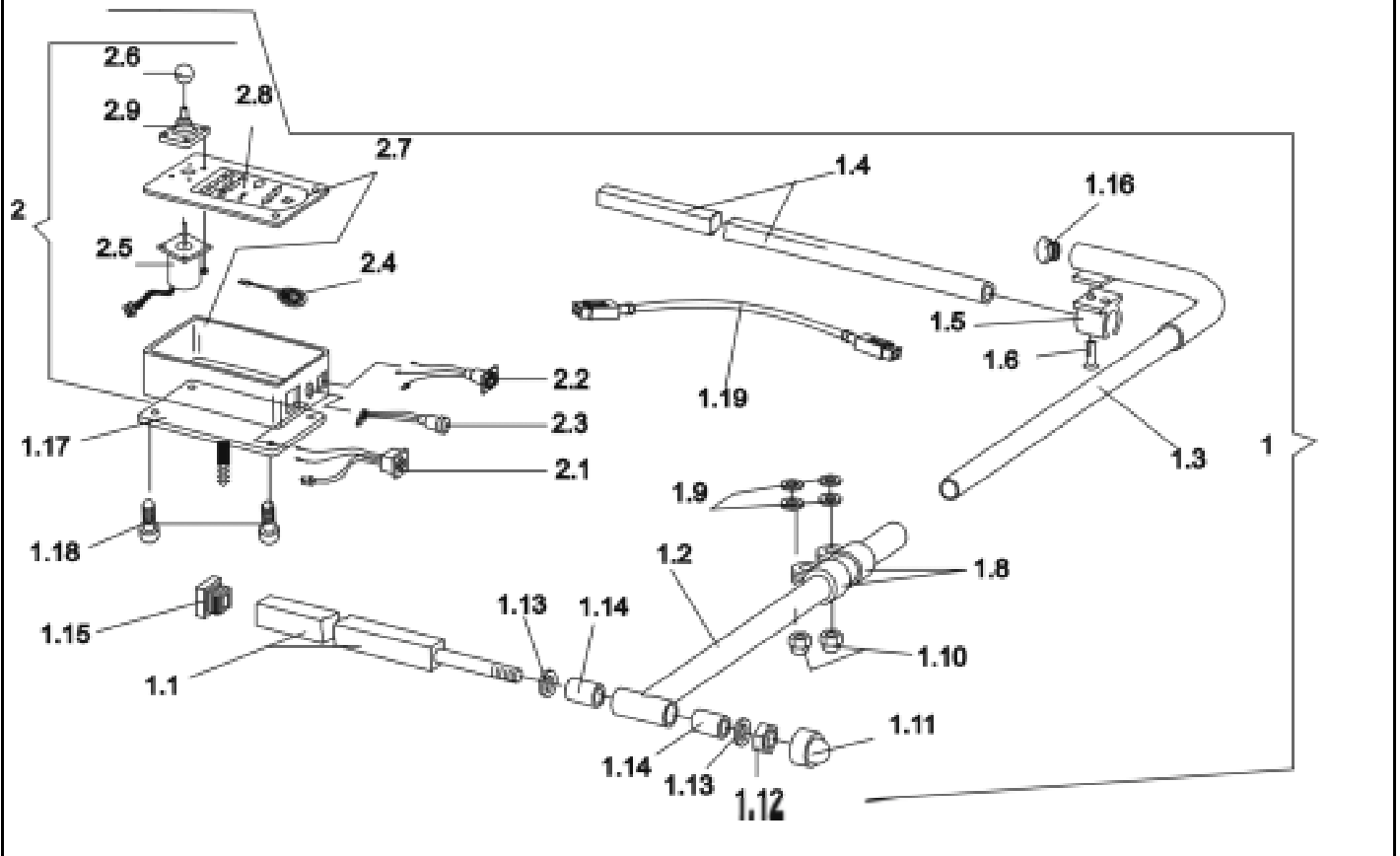


Item	Qty	Part-No.	Part-Designation
2	1	P10066	ACS-TRAY CONTROL BOX
2.1	1	P41108	ACS-BUS CABLE SOCKET
2.2	1	P41106	CHARGER SOCKET
2.3	1	P41107	PROGRAMMER SOCKET
2.4	1	P30493	HORN
2.5	1	GSM55120	ACS-JOYSTICK F. TRAYCONTROL
2.6	1	L36999	KNOB 32 MM, HOLE 5,8 MM
2.7	1		ACS CONTROL BOX HOUSING
2.8	1	P30907	FOIL KEYPAD
2.9	1	Z183001A	RUBBER GAITER; BLACK

Storm 3



Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
 Serial No. :

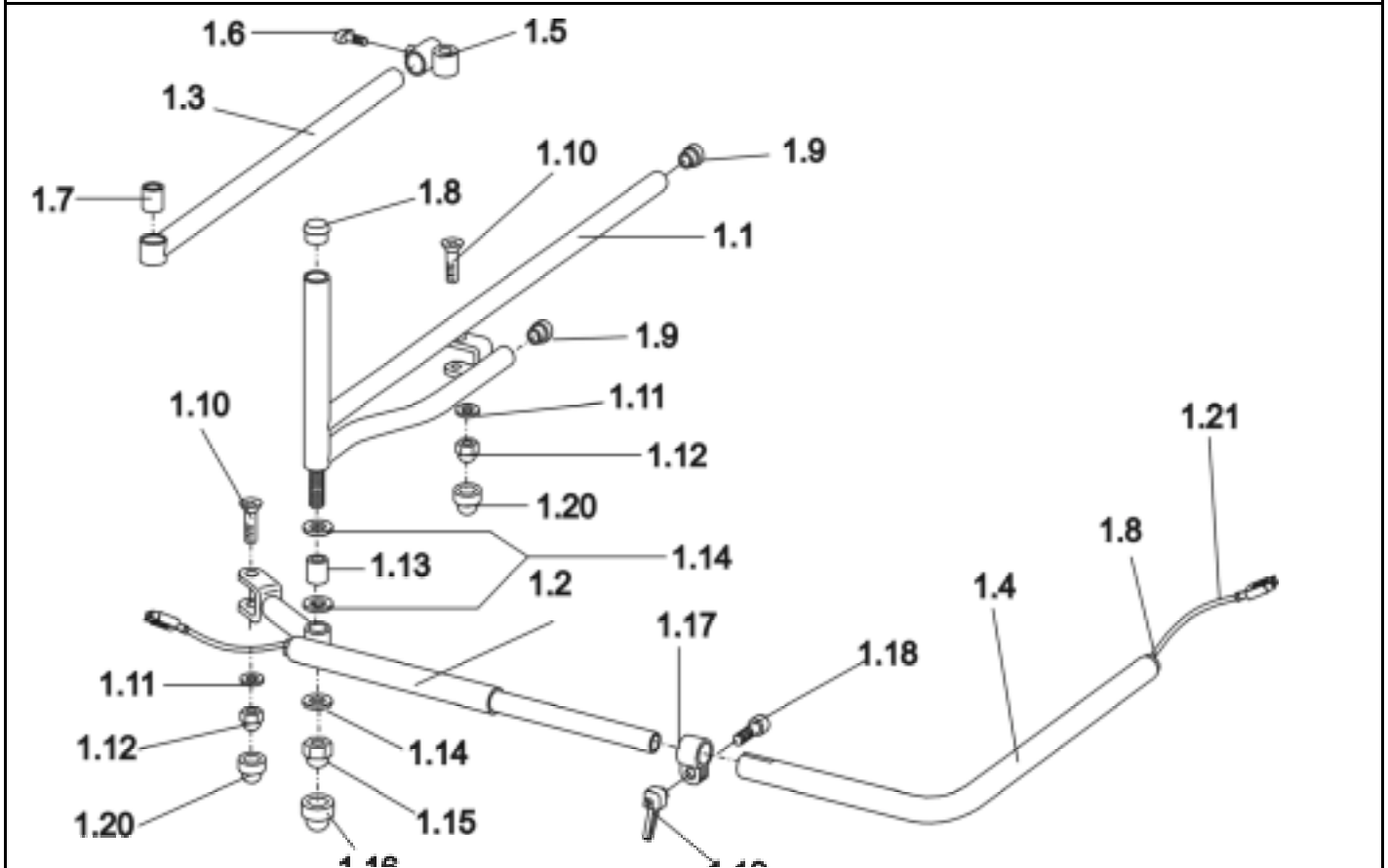


Item	Qty	Part-No.	Part-Designation
1	1	SP1422943	ASSEMBLY KIT FOR ELECTR. HORIZONTAL SWING-OUT CHIN CONT...
1	1	SP1422942	ASSEMBLY KIT RH F. MANUAL HORIZONTAL SWING-OUT CHIN CON...
1.1	1		SUPPORT TUBE LH; CHROME PLATED
1.1	1		SUPPORT TUBE RH; CHROME PLATED
1.2	1		CHIN CONTROL SWING ARM.; CHROME PLATED
1.3	1		CHIN CONTROL PUSH ROD.; CHROME PLATED
1.4	1		CHIN CONTROL SUPPORT TUBE; CHROME PLATED
1.5	1		PUSH-ROD GUIDE; CHROME PLATED
1.6	1	L31001	HAND KNOB M6 X 10
1.7	1		BUSH RD. 18 X 22
1.8	2		TUBE END PLUG SL 22
1.9	2		TUBE END PLUG SL 18
1.10	2		COUNTERSUNK SOCKET HEAD SCREW M8 X 45
1.11	2		WASHER 8,4 X 17 X 1,6
1.12	2		NYLOC NUT M8
1.13	1		BUSH RD. 18 X 3 X 22
1.14	3		WASHER 13 X 24 X 2,5
1.15	1		NYLOC NUT M12 X 1,5
1.16	1		NUT CAP M12
1.17	1		CLAMP 22 MM
1.18	1		CUP HEAD NIBBED BOLT M8 X 25
1.19	1	L32901	LEVER M8; BLACK
1.20	2		NUT CAP M8
1.21	1	1419367	ACS BUS CABLE L=2950 MM

Storm 3



Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
 Serial No. :

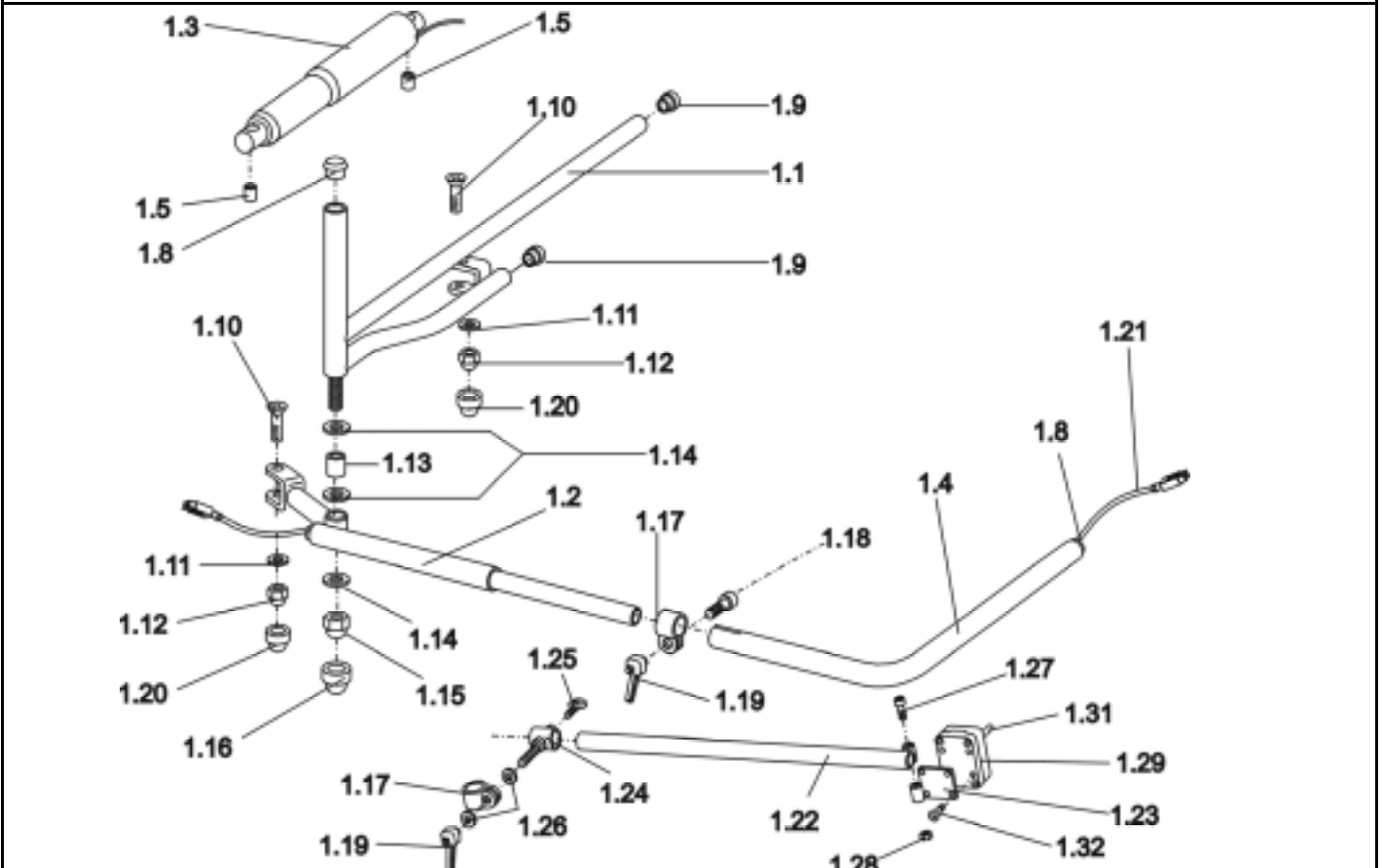


Item	Qty	Part-No.	Part-Designation
1	1	SP1422944	ASSEMBLY KIT RH F. EL. HORIZONTAL SWING-OUT CHIN CONTRO...
1	1	SP1422945	ASSEMBLY KIT FOR ELECTR. HORIZONTAL SWING-OUT CHIN CONT...
1.1	1		SUPPORT TUBE LH; CHROME PLATED
1.1	1		SUPPORT TUBE RH; CHROME PLATED
1.2	1		CHIN CONTROL SWING ARM.; CHROME PLATED
1.3	1	P00082	ACTUATOR LA 22.5E-75-24 INCL CABLE AND SWITCH
1.4	1		CHIN CONTROL SUPPORT TUBE; CHROME PLATED
1.5	1		BUSH 10 X 1 X 22,5
1.8	1		TUBE END PLUG SL 22
1.9	1		TUBE END PLUG SL 18
1.10	2		COUNTERSUNK SOCKET HEAD SCREW M8 X 45
1.11	2		WASHER 6,4 X 12,5 X 1,6
1.12	2		NYLOC NUT M8
1.13	2		BUSH RD. 18 X 3 X 22
1.14	2		WASHER 13 X 24 X 2,5
1.15	1		NYLOC NUT M12 X 1,5
1.16	3		NUT CAP M12
1.17	1		CLAMP 22 MM
1.18	1		CUP HEAD NIBBED BOLT M8 X 25
1.19	1	L32901	LEVER M8; BLACK
1.20	1		NUT CAP M8
1.21	1	1419367	ACS BUS CABLE L=2950 MM
1.22	2		SWITCH BOX TUBE; CHROME PLATED
1.23	1		SWITCH BOX MOUNTING PLATE; CHROME PLATED
1.24	1		CLAMP; CHROME PLATED

Storm 3



Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
 Serial No. :

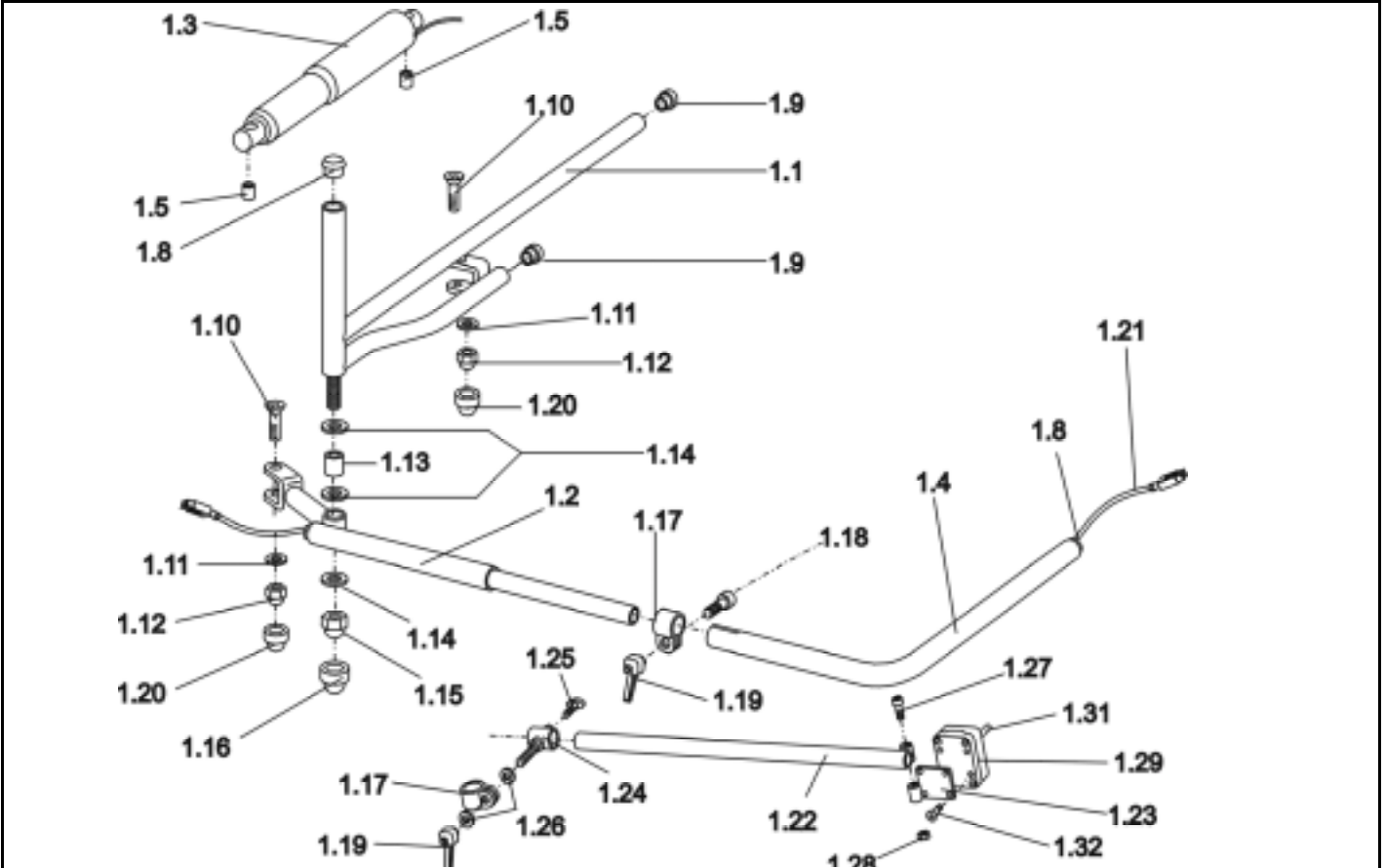


Item	Qty	Part-No.	Part-Designation
1.25	1	L31001	HAND KNOB M6 X 10
1.26	2		WASHER 8,4 X 17 X 1,6
1.27	1		HEX HEAD SCREW M6 X 40
1.28	1		NYLOC NUT M6
1.29	1	L90994	MULTIBOX SMALL
1.30	1	P30109	TOGGLE SWITCH SINGLE POLE
1.31	1	Z183011A	TOGGLE SWITCH EXTENSION; GREY
1.32	2		POP RIVET 3,2 X 11,5

Storm 3



Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
 Serial No. :

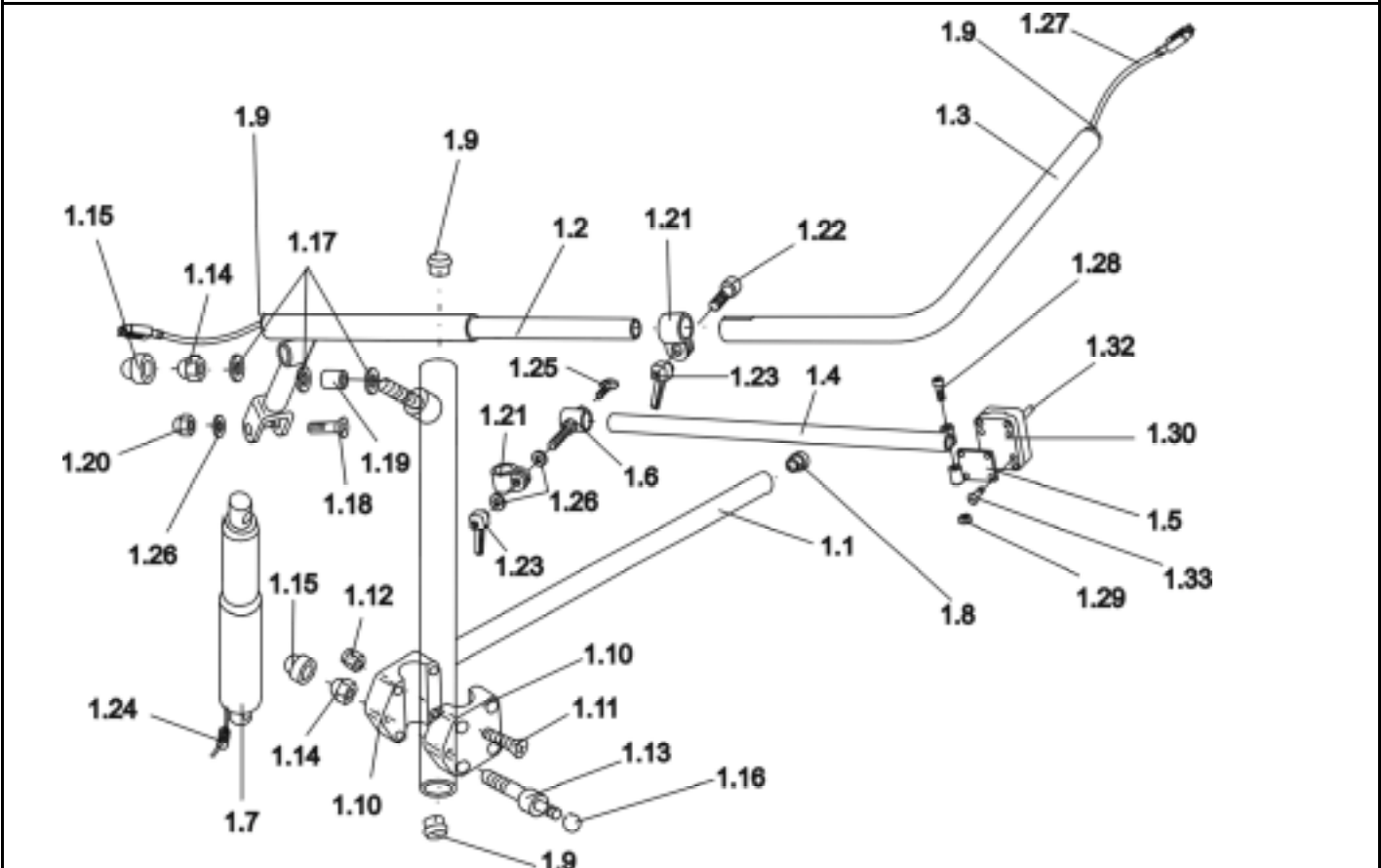


Item	Qty	Part-No.	Part-Designation
1	1	SP1422946	ELECTR. VERTICAL SWING-OUT CHIN CONTROL ASSEMBLY KIT
1.1	1		CHIN CONTROL, SWING ARM ELECTRICAL VERTICAL SWING-AWAY
1.2	1		CHIN CONTROL SWING ARM.; CHROME PLATED
1.3	1		CHIN CONTROL SUPPORT TUBE; CHROME PLATED
1.4	1		SWITCH BOX TUBE; CHROME PLATED
1.5	1		SWITCH BOX MOUNTING PLATE; CHROME PLATED
1.6	1		CLAMP; CHROME PLATED
1.7	1	P00081	ACTUATOR LA 22.5E-100-24 WITH STOP-SWI
1.8	1		TUBE END PLUG SL 18
1.9	4		TUBE END PLUG SL 22
1.10	2		CLAMP
1.11	4		SOCKET HEAD CAP SCREW M6 X 25
1.12	4		NYLOC NUT M6
1.13	1		BOLT RD. 16 X 88 M M12
1.14	2		NYLOC NUT M12 X 1,5
1.15	2		NUT CAP M12
1.16	1		CAP A10
1.17	3		WASHER 13 X 24 X 2,5
1.18	1		COUNTERSUNK SOCKET HEAD SCREW M8 X 45
1.19	1		BUSH 10 X 1 X 22,5
1.20	1		DOMED NYLOC NUT M8 X 1
1.21	2		CLAMP 22 MM
1.22	1		CUP HEAD NIBBED BOLT M8 X 25
1.23	2	L32901	LEVER M8; BLACK
1.24	1		FERRITE SLEEVE C=8 D=1

Storm 3



Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
 Serial No. :

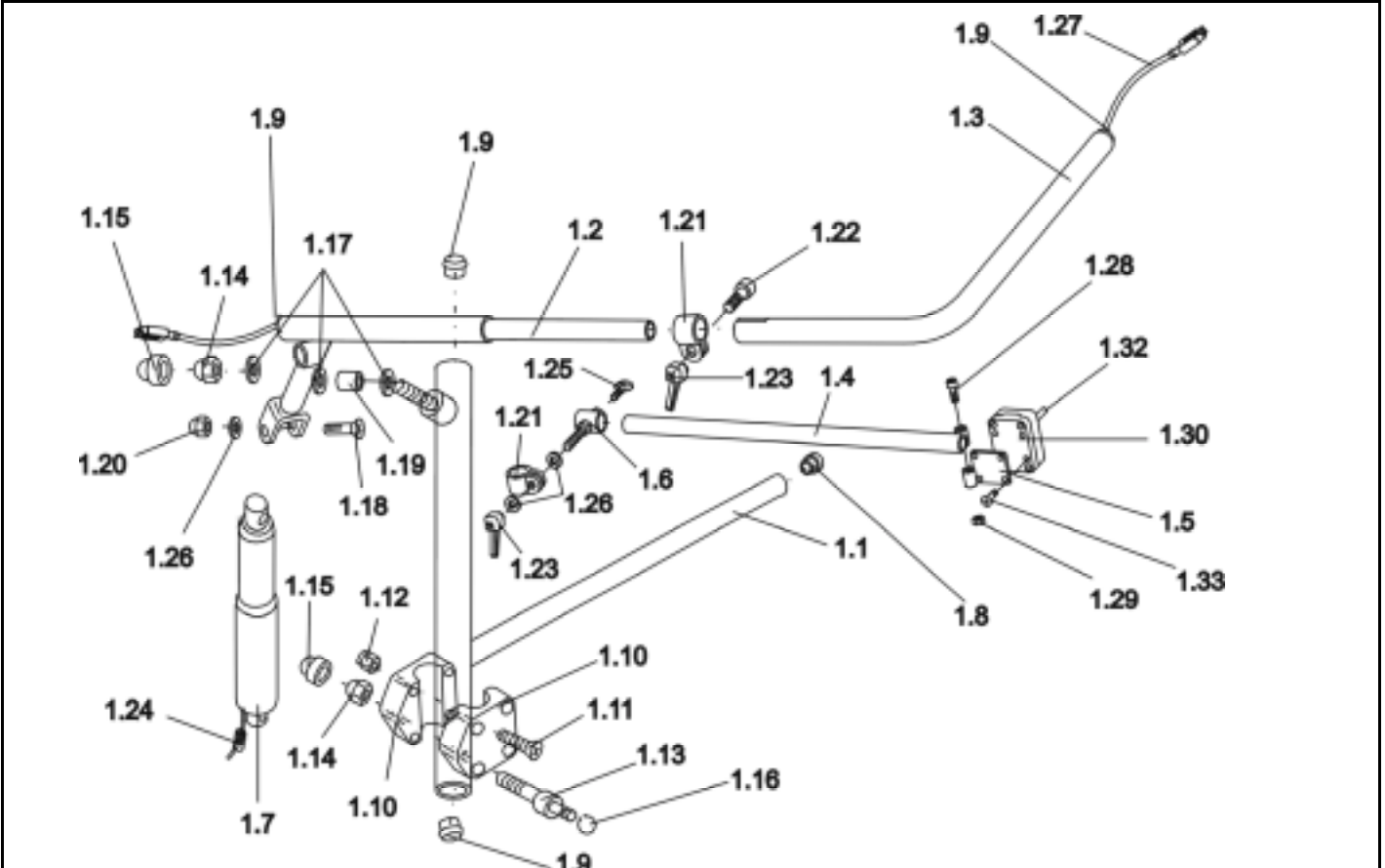


Item	Qty	Part-No.	Part-Designation
1.25	1	L31001	HAND KNOB M6 X 10
1.26	3		WASHER 8,4 X 17 X 1,6
1.27	1	1419367	ACS BUS CABLE L=2950 MM
1.28	1		HEX HEAD SCREW M6 X 40
1.29	1		NYLOC NUT M6
1.30	1	L90994	MULTIBOX SMALL
1.31	1	P30109	TOGGLE SWITCH SINGLE POLE
1.32	1	Z183011A	TOGGLE SWITCH EXTENSION; GREY
1.33	2		POP RIVET 3,2 X 11,5

Storm 3



Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
 Serial No. :

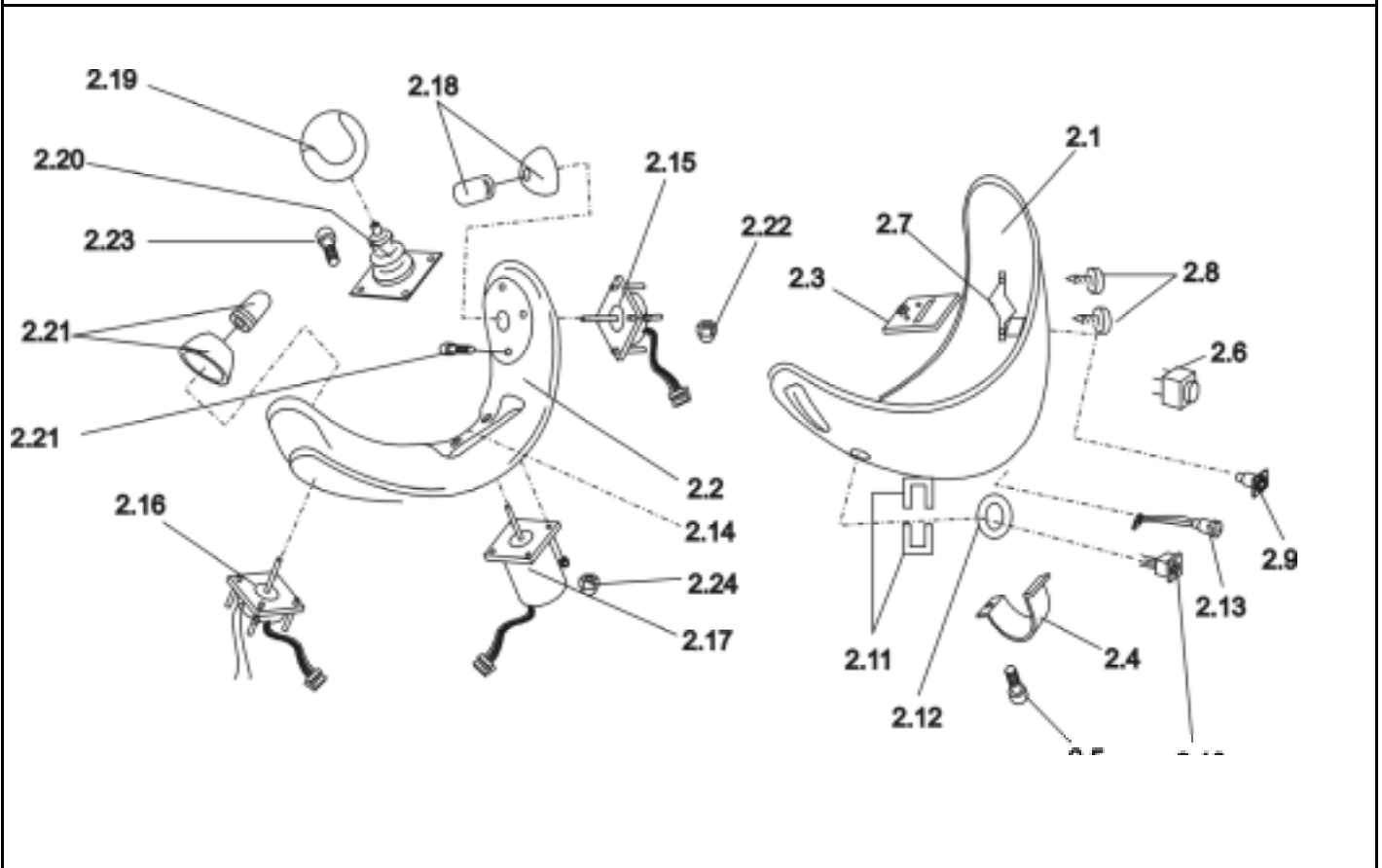


Item	Qty	Part-No.	Part-Designation
2	1	SP1422801	ACS CHIN CONTROL LH CPL.
2	1	SP1422802	ACS CHIN CONTROL RH CPL.
2.1	1	Z183242	LOWER HOUSING LH
2.1	1	Z183241	LOWER HOUSING RH
2.2	1	F383019	UPPER HOUSING; GREY
2.3	1		MOUNTING PLATE
2.4	1		CLAMP
2.5	1		SOCKET HEAD CAP SCREW M6 X 16
2.6	1		HORN (DIGISOUND)
2.7	1		HORN CLAMP
2.8	2		DOMED HEAD SELF TAPPING SCREW 2,9 X 25
2.9	1	P41106	CHARGER SOCKET
2.10	1	P41108	ACS-BUS CABLE SOCKET
2.11	2	P41103	CHARGER SOCKET
2.12	1	T95099	O-RING 14-3 MM
2.13	1	P41104	PROGRAMMER SOCKET
2.14	1	P10475	STICKERSET (ACS CHIN CONTROL)
2.15	1	Z183243	ACS-CHIN CONTROL JOYSTICK SWITCH LH
2.16	1	Z183244	ACS-CHIN CONTROL JOYSTICK SWITCH RH
2.17	1	P10304	JOYSTICK
2.18	2	Z183229	JOYSTICK KNOB CPL.
2.19	1	Z183265	FOAM CONTROL KNOB CPL.
2.20	1	Z183001A	RUBBER GAITER; BLACK
2.21	8		CROSS HEAD SCREW M2,5 X 22
2.22	8		NYLOC NUT M2,5

Storm 3



Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
Serial No. :

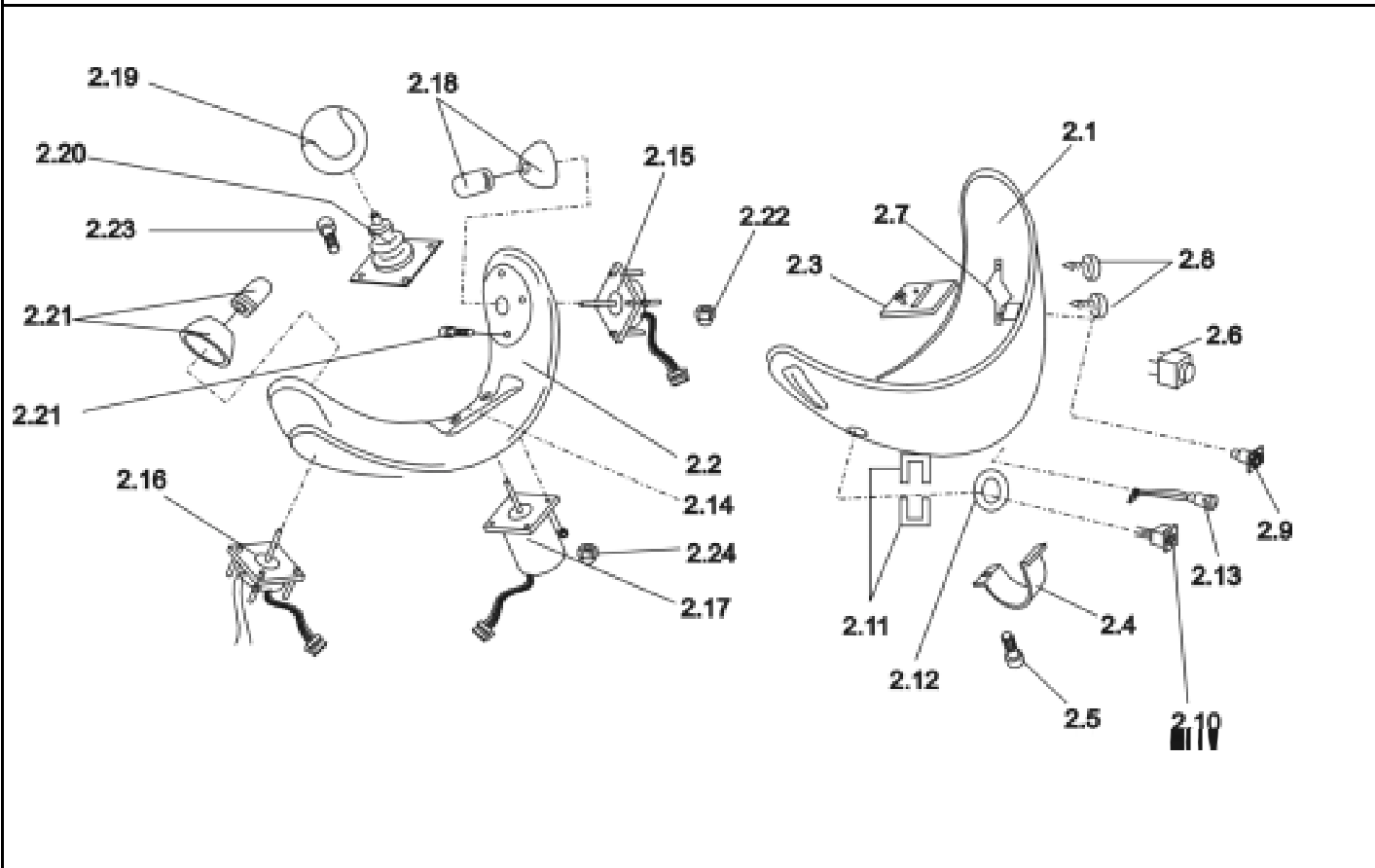

Item	Qty	Part-No.	Part-Designation
2.23	4		CROSS HEAD SCREW M3 X 20
2.24	8		HEX NUT M3


Storm 3



Release
 07 / 2004
 Chapter 13
 Sheet 12

Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
 Serial No. :

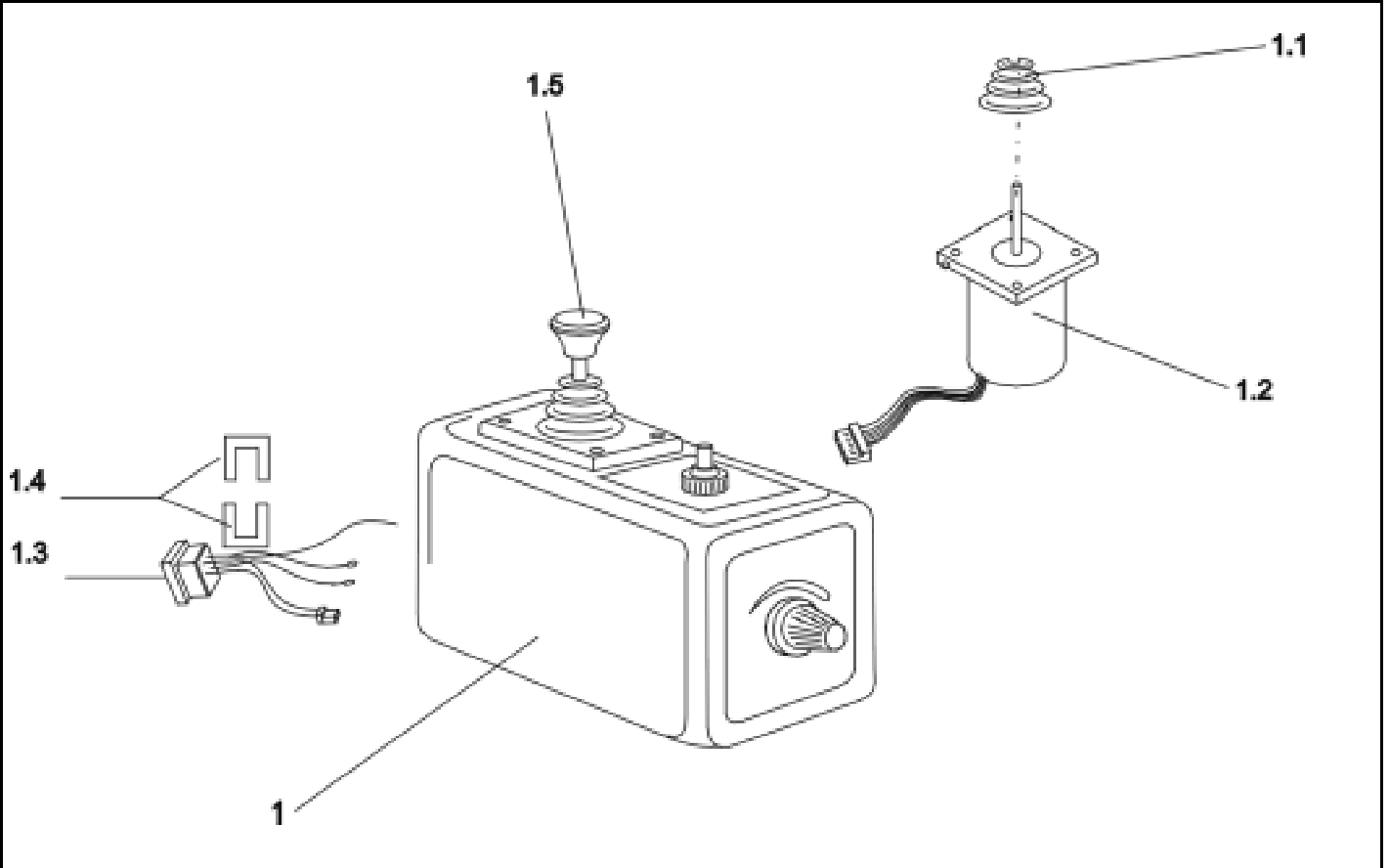


Item	Qty	Part-No.	Part-Designation
1	1	6001425	ACS CTRL. FOR ATTENDANT SWITCHABLE
1.1	1	Z183001A	RUBBER GAITER; BLACK
1.2	1	P10304	JOYSTICK
1.3	1	P41105	BUS CABLE SOCKET
1.4	1	GME61927	FIXING CLAMP
1.5	1	P40120	DRIVE LEVER KNOB F. SWITCHABLE ACS CARER CONTROL

Storm 3



Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
 Serial No. :

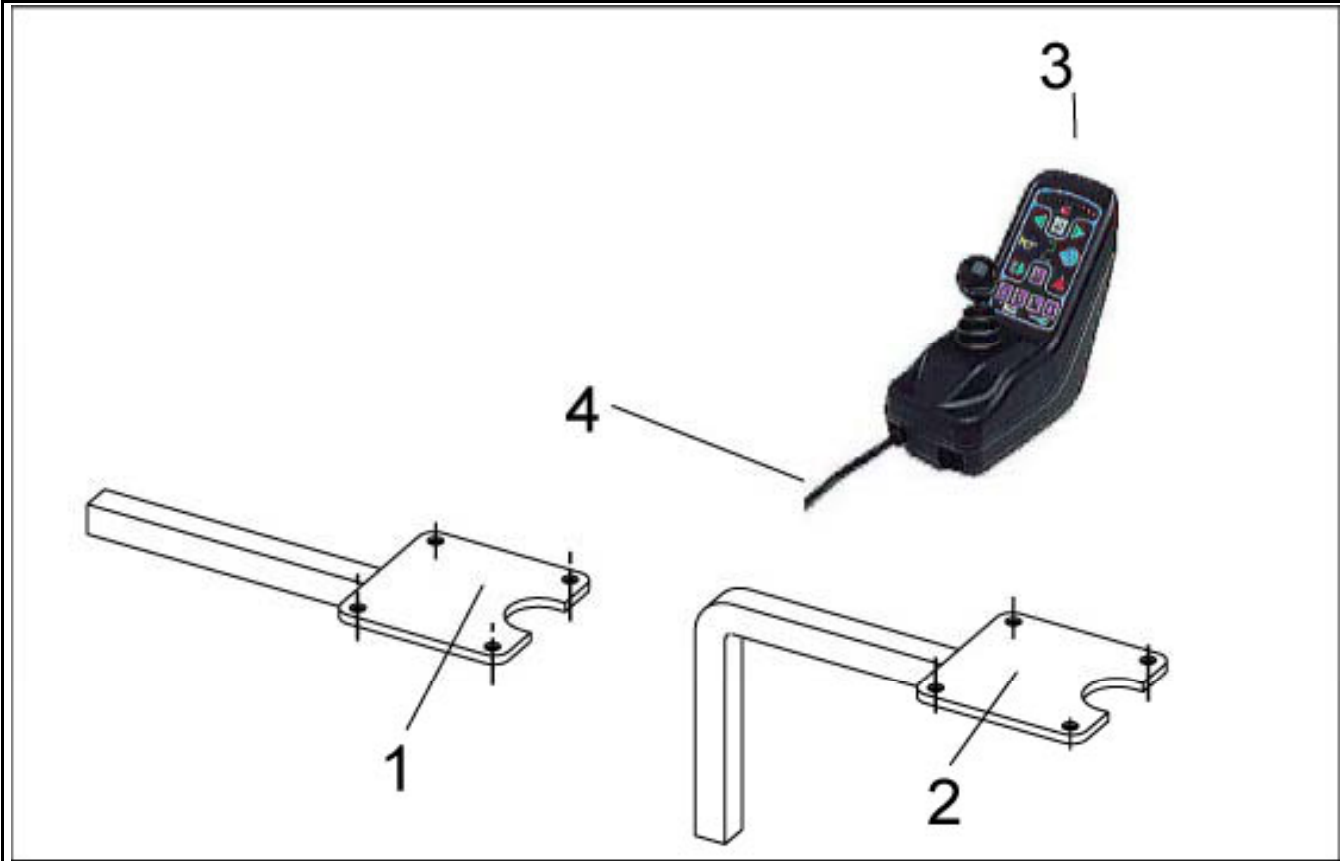

Item	Qty	Part-No.	Part-Designation
1	1	1419372-9005	JOYSTICK BOX SUPPORT REM 34
2	1	1434462-9005	JOYSTICK BOX SUPPORT REM 34 F.SWIVEL MECHANISM
-	4	D10124	COUNTERSUNK CROSS HEAD SCREW M4 X 10
3	1	SP1434920	JOYSTICK BOX REM 34 (DOLPHIN) CPL.
4	1	1419386	ACS BUS CABLE L=1900 MM
-	1	SP1419695	MAGNETIC KEY W. POUCH CPL.


Storm 3



Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
 Serial No. :

umruet_rem24sd.dwg

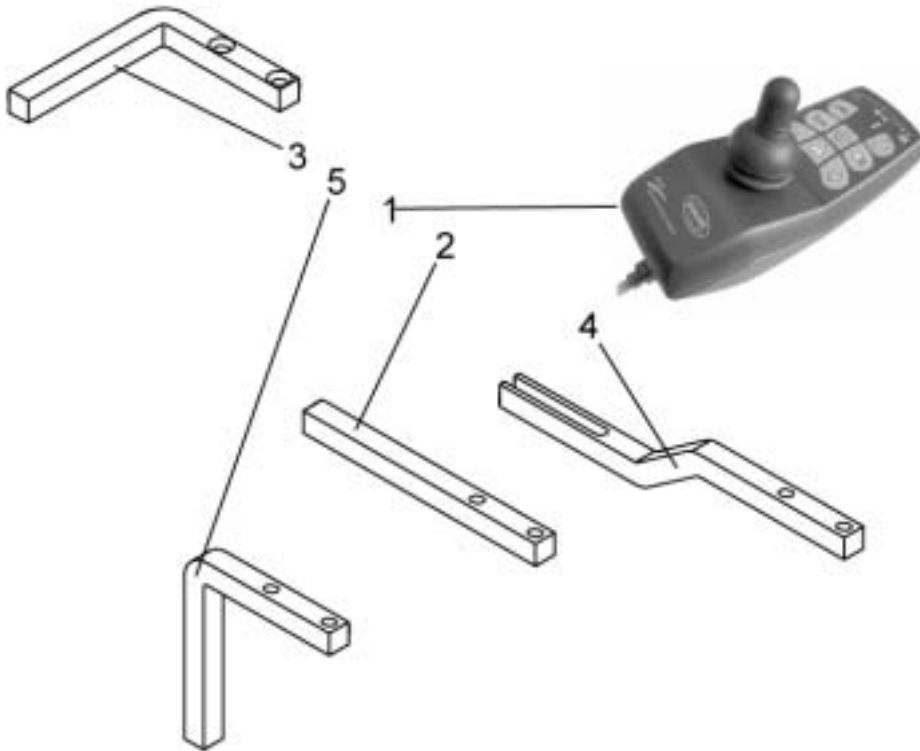

Item	Qty	Part-No.	Part-Designation	
-	-		The joystick box support for REM 24SD is not the same a...	
-	-		Please order the new version if you have the old one!	
1	1	1436541	REMOTE REM 24 SD	03/04→
2	1	1440437-9005	JOYSTICK BOX SUPPORT REM 24SD STANDARD	03/04→
3	1	1440438-9005	ATTENDANT JOYSTICK BOX SUPPORT REM 24SD	03/04→
4	1	1440439-9005	JOYSTICK BOX SUPPORT REM 24SD F. "ARNE JENSEN" SEAT	03/04→
5	1	1440441-9005	REMOTE HOLDER SWING AWAY REM 24SD	03/04→

Storm 3



Release
 07 / 2004
 Chapter 13
 Sheet 15