


Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
 Serial No. :

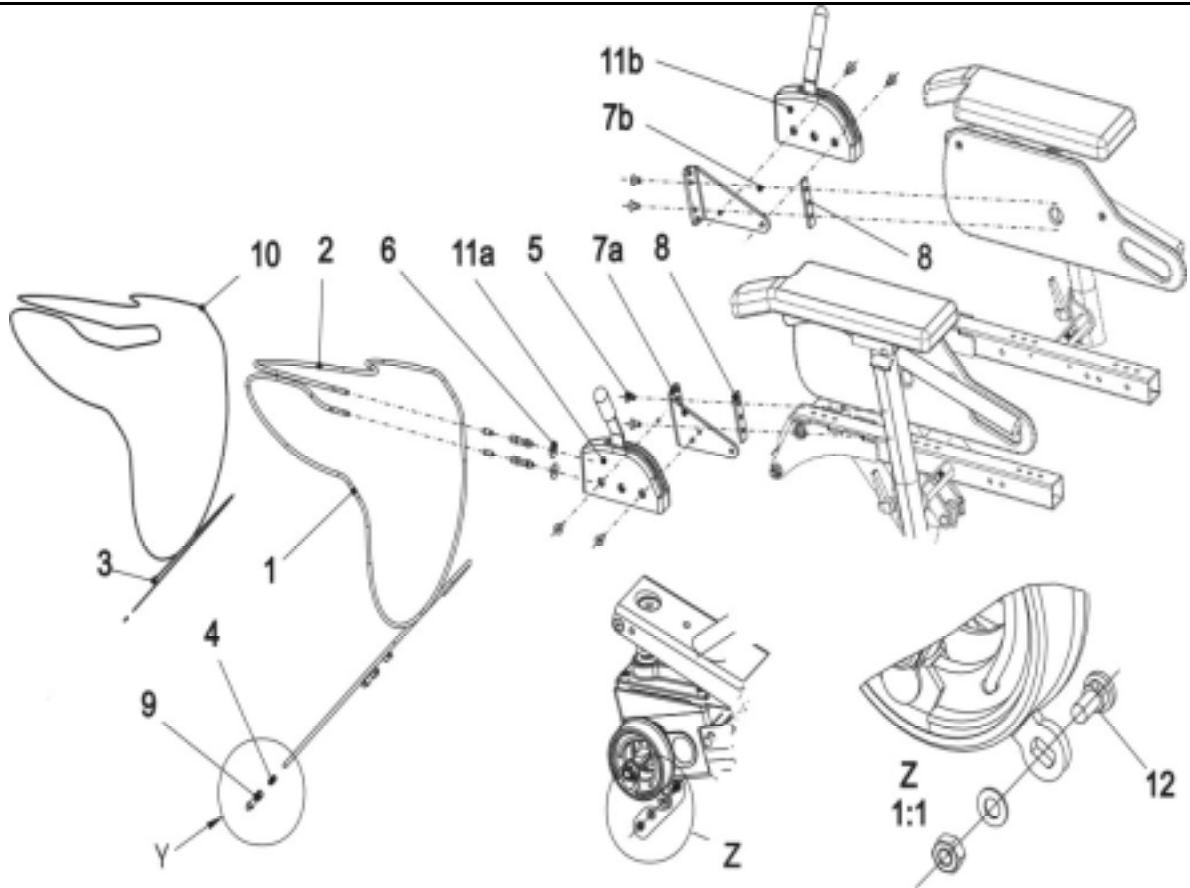

Item	Qty	Part-No.	Part-Designation
-	1		WHEEL BRAKE RH, CPL.
-	1		WHEEL BRAKE LH, CPL.
-	1	SP1423473	BOWDEN CABLE SET CPL.
1	1		BOWDEN-CABLE ENVELOPE L=1130 MM
2	1		BOWDEN-CABLE ENVELOPE L=1470 MM
3	2		INNER CABLE END CAP
4	4		CABLE END-CAP
10	1		BOWDEN-CABLE INNER LINE L=3200 MM
12	2		CABLE-CLAMP SCREW 6 MM
5	4		BOLT M6 X 12
6	2		WASHER 6,4 X 18 X 1,6
7a	1	1428076-9005	BRAKE BOX HOLDER RH
7b	1	1428075-9005	BRAKE BOX HOLDER LH
8	1	1421720-0002	GROVE PLATE
9	4	M30050	ADJUSTING SCREW M6 X 24; ZINC PLATED
11a	1	SPF331038	BRAKE ASSY RH
11b	1	SPF331028	BRAKE ASSY LH

Storm 3

