


Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
Serial No. :

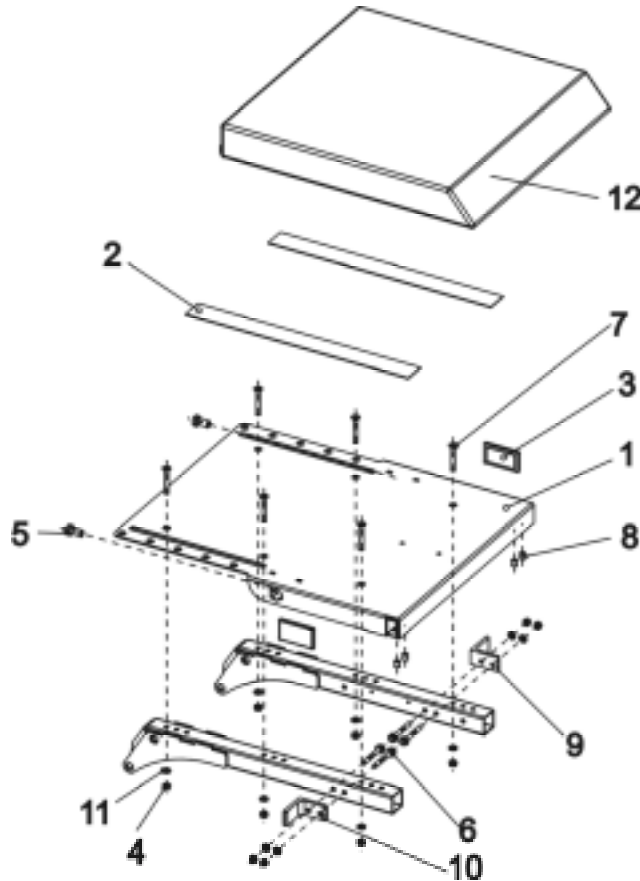


Item	Qty	Part-No.	Part-Designation
-	1		SEAT (ONE-PIECE) CPL.
1	1	1419999-9005	SEAT PLATE
2	2	U21087	CUSHION FIXING STRIP - VELCRO FASTENER (HOOKED) 38 X 40...
3	2	P31298	REFLECTOR, YELLOW
-	1	SP1423205	SEAT MOUNTING KIT (ONE-PIECE VERSION)
4	10		NYLOC NUT M6
5	2		BOLT M10 X 20
6	4		BOLT M6 X 50
7	6		COUNTERSUNK SOCKET HEAD SCREW M6 X 45 10.9
8	4		HEX HEAD SCREW M8 X 20
9	1		ANGLE BRACKET LH, SB 43
10	1		ANGLE BRACKET RH, SB 43
11	10		WASHER 6,4 X 12,5 X 1,6
12	1	1420460	CUSHION ST=41 CM
12	1	1420461	CUSHION ST=46 CM
12	1	1420462	CUSHION ST=51 CM

Storm 3



Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
 Serial No. :

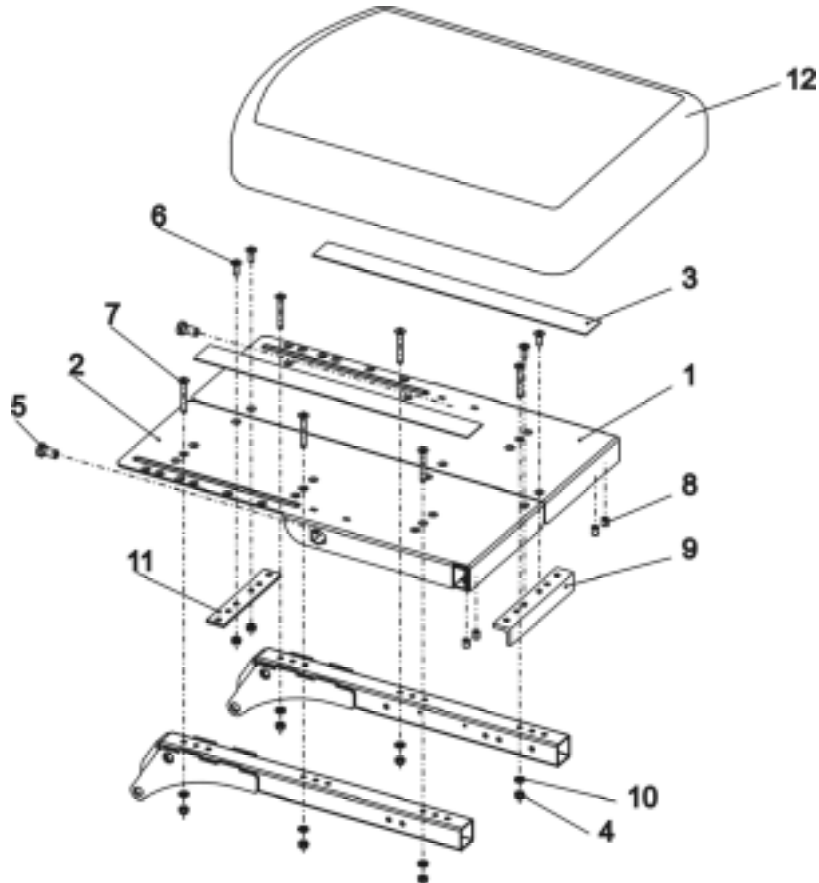


Item	Qty	Part-No.	Part-Designation
-	1	SP1422121	SEAT PLATE, FLEX F. SB 39 CPL.
1	1	1420458-9005	SEAT PLATE LH CPL.
2	1	1420457-9005	SEAT PLATE RH CPL.
3	2	U21087	CUSHION FIXING STRIP - VELCRO FASTENER (HOOKED) 38 X 40...
-	1	SP1423206	SEAT (SPLIT) ASSEMBLY KIT
4	8		NYLOC NUT M6
5	2		BOLT M10 X 20
6	4		COUNTERSUNK SOCKET HEAD SCREW M6 X 16
7	6		COUNTERSUNK SOCKET HEAD SCREW M6 X 45 10.9
8	4		HEX HEAD SCREW M8 X 20
9	1		ANGLE BRACE
10	6		WASHER 6,4 X 12,5 X 1,6
11	1		REINFORCING PLATE
12	1	1422529	CUSHION SB 39; BLUE
12	1	5314542	CUSHION SB 39; GREY
12	1	5252582-TR23	SEAT CUSHION SB 39; DARTEX

Storm 3



Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
 Serial No. :

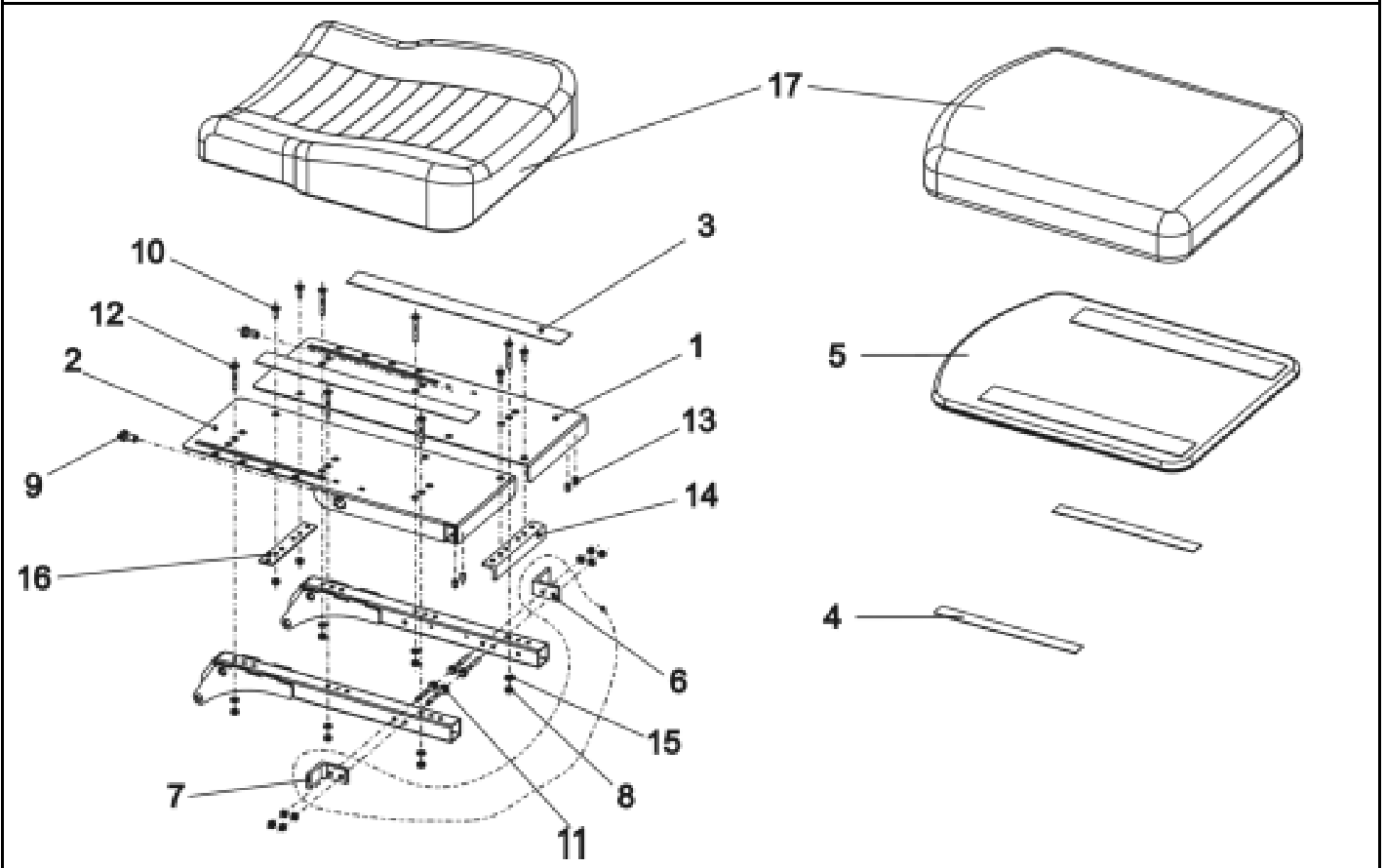


Item	Qty	Part-No.	Part-Designation
-	1	SP1422064	SPLIT SEAT SB 43 CPL.
1	1	1420458-9005	SEAT PLATE LH CPL.
2	1	1420457-9005	SEAT PLATE RH CPL.
3	2	U21087	CUSHION FIXING STRIP - VELCRO FASTENER (HOOKED) 38 X 40...
4	2	5816479	DURALOCK SJ 3552
5	1	5314532	SEAT PLATE 410 X 520 MM
6	1	1421701-9005	ANGLE BRACKET LH, SB 43
7	1	1421700-9005	ANGLE BRACKET RH, SB 43
-	1	SP1423206	SEAT (SPLIT) ASSEMBLY KIT
8	12		NYLOC NUT M6
9	2		BOLT M10 X 20
10	4		COUNTERSUNK SOCKET HEAD SCREW M6 X 16
11	4		BOLT M6 X 50
12	6		COUNTERSUNK SOCKET HEAD SCREW M6 X 45 10.9
13	4		HEX HEAD SCREW M8 X 20
14	1		ANGLE BRACE
15	10		WASHER 6,4 X 12,5 X 1,6
16	1		REINFORCING PLATE
17	1	1422071	CUSHION CONTOUR SB 43; GREY
17	1	1422072	CUSHION CONTOUR SB 43; BLACK
17	1	5314525	SEAT CUSHION FLEX GREY, 55 X 44 X 7
17	1	1422530	CUSHION FLEX SB 43; BLUE
17	1	1422531	CUSHION SB 43 VIC AIR
17	1	1428090	SEATCHSHION SB 43; DARTEX

Storm 3



Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
 Serial No. :

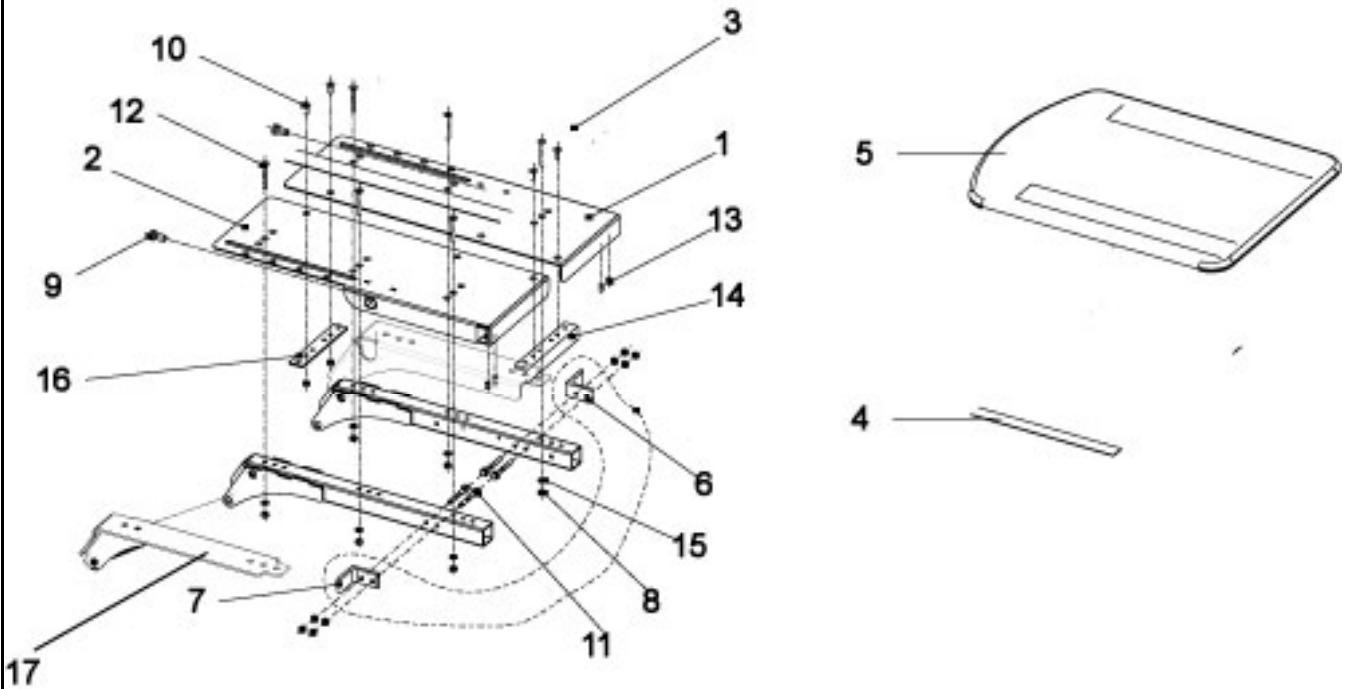

Item	Qty	Part-No.	Part-Designation
-	1	SP1422065	SPLIT SEAT SB 48 CPL.
1	1	1420458-9005	SEAT PLATE LH CPL.
2	1	1420457-9005	SEAT PLATE RH CPL.
3	2	U21087	CUSHION FIXING STRIP - VELCRO FASTENER (HOOKED) 38 X 40...
4	2	5816479	DURALOCK SJ 3552
5	1	5314533	SEAT PLATE 460 X 520 MM
6	1	1421705-9005	SEAT ANGLE LH SB 48
7	1	1421706-9005	SEAT ANGLE RH, SB 48
-	1	SP1423206	SEAT (SPLIT) ASSEMBLY KIT
8	12		NYLOC NUT M6
9	2		BOLT M10 X 20
10	4		COUNTERSUNK SOCKET HEAD SCREW M6 X 16
11	4		BOLT M6 X 50
12	6		COUNTERSUNK SOCKET HEAD SCREW M6 X 45 10.9
13	4		HEX HEAD SCREW M8 X 20
14	1		ANGLE BRACE
15	10		WASHER 6,4 X 12,5 X 1,6
16	1		REINFORCING PLATE
17	1	1434870-9005	BRACKET SEAT REINFORCEMENT RH
17	1	1434869-9005	BRACKET SEAT REINFORCEMENT LH
-	1	1436355	INSTALLATION KIT BRACKET SEAT REINFORCEMENT (RH + LH)


Storm 3



Release
 07 / 2004
 Chapter 9
 Sheet 4

Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
 Serial No. :

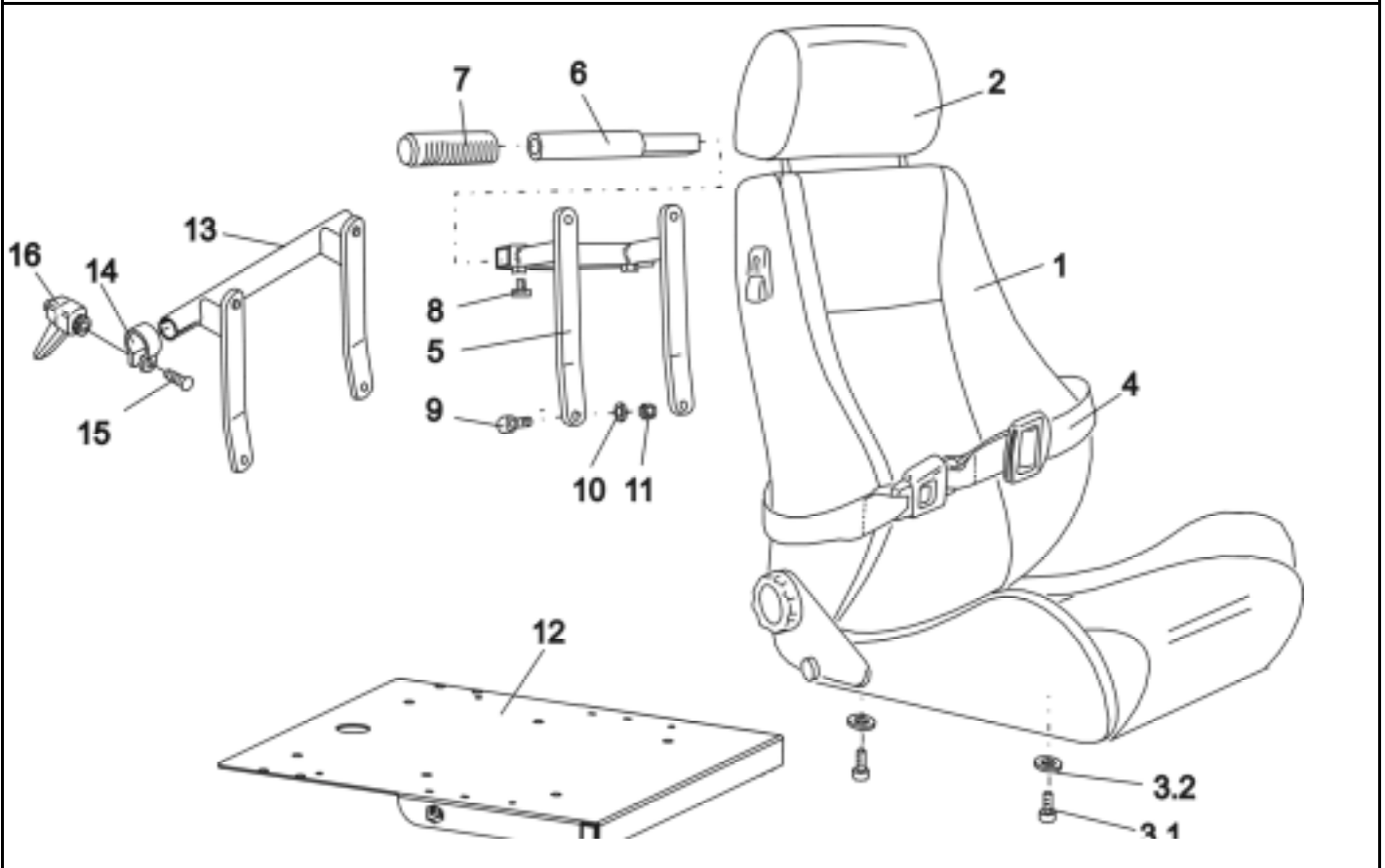


Item	Qty	Part-No.	Part-Designation
1	1	1422864	RECARO SEAT MOBILITY LT W. ELECTR. ADJUSTMENT
2	1	N89999	HEADREST - SPORT NARDO
1	1	N89995	RECARO SEAT "N-JOY" WITH "LE MANS" SEAT COVER
2	1	N89987	RECARO SEAT "N-JOY" HEADREST
3	1	SP1423173	RECARO SEAT MOUNTING KIT
3.1	4		SOCKET HEAD CAP SCREW M6 X 20
3.2	4		WASHER 6,4 X 12,5 X 1,6
-	4		HEX HEAD SCREW M8 X 20
4	1	Z163001A	SAFETY BELT WITH SNAP LOCK AND VELCRO FASTNER FOR RECARO...
4	1	F367002	SAFETY BELT WITH VELCRO FASTNER F. RECARO
-	1	1415001	CONNECTOR (2-PIN VERS.)
-	1	1415002	CAP
-	1	SP1423177	HOLDER F. PUSH HANDLE RECARO CPL.
			WE RECOMMEND RETROFITTING AT THE FACTORY!
5	1		HOLDER FOR PUSH HANDLE RECARO; RAL 9005, DEEP BLACK
6	2		HANDLE; RAL 9005 DEEP BLACK
7	2	L10009	HANDLE
8	2		HEX HEAD SCREW M8 X 20
9	4		HEX HEAD SCREW M6 X 40
10	4		WASHER 6,4 X 12,5 X 1,6
11	4		NYLOC NUT M6
12	1	1423174-9005	RECARO SEAT PLATE
-	1	SP1422951	ASSEMBLY KIT F. CHIN CONTROL HOLDER RECARO CPL.
13	1	4100067.00	CHIN CONTROL SUPPORT; CHROME PLATED
14	2		CLAMP 22 MM

Storm 3



Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
 Serial No. :

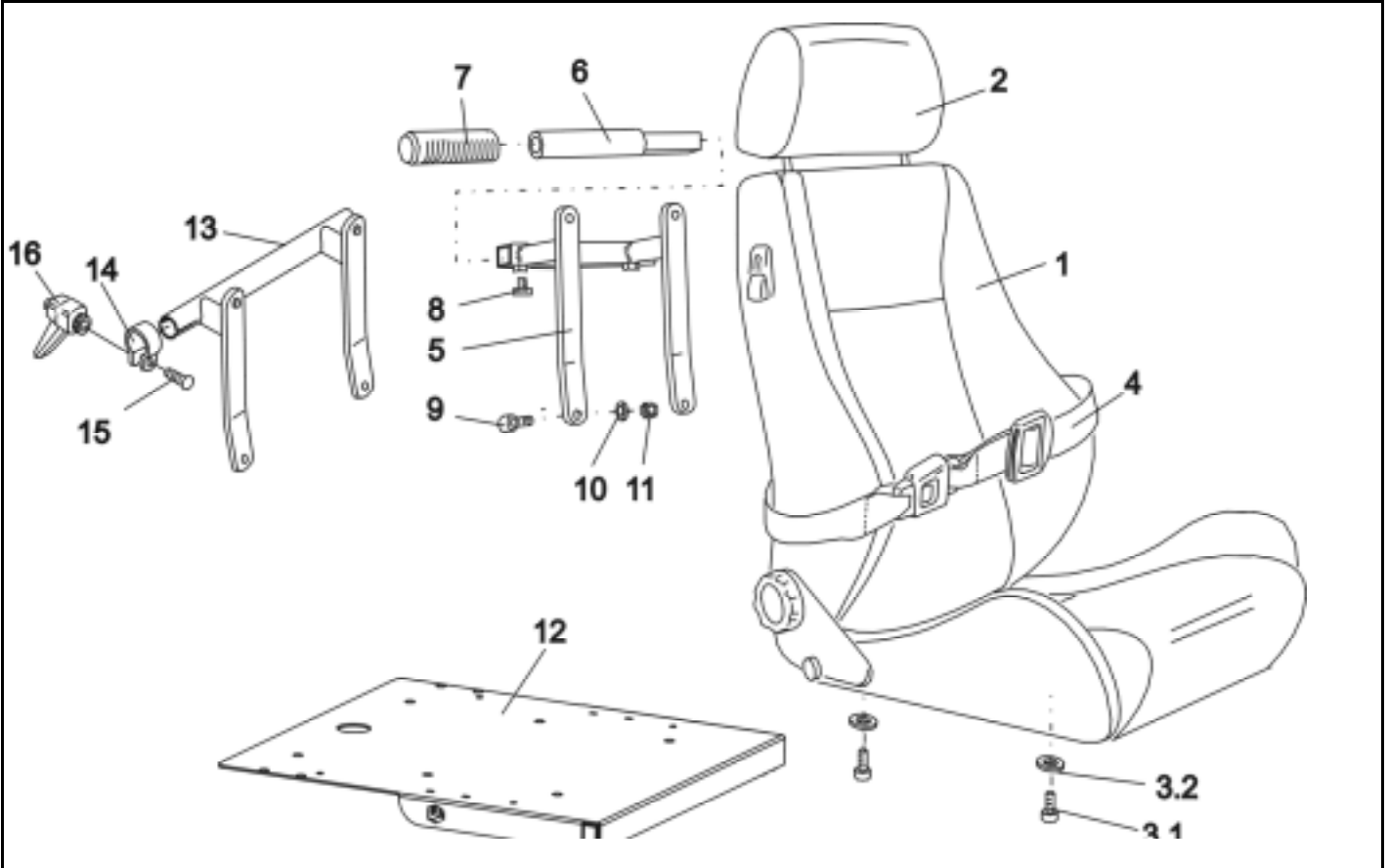

Item	Qty	Part-No.	Part-Designation
15	2		CUP HEAD NIBBED BOLT M8 X 25
16	2	L32901	LEVER M8; BLACK
-	1	SP1423183	SEAT PLATE RECARO CPL.
-	1	1421705-9005	SEAT ANGLE LH SB 48
-	1	1421706-9005	SEAT ANGLE RH, SB 48
12	1	1423174-9005	RECARO SEAT PLATE
-	2	P31298	REFLECTOR, YELLOW
-	2		MOUNTING TUBE FOR LEGRESTS


Storm 3



Release
 07 / 2004
 Chapter 9
 Sheet 7

Customer :
 Customer No. :
 Phone :

Contact :
 Commission :
 Serial No. :

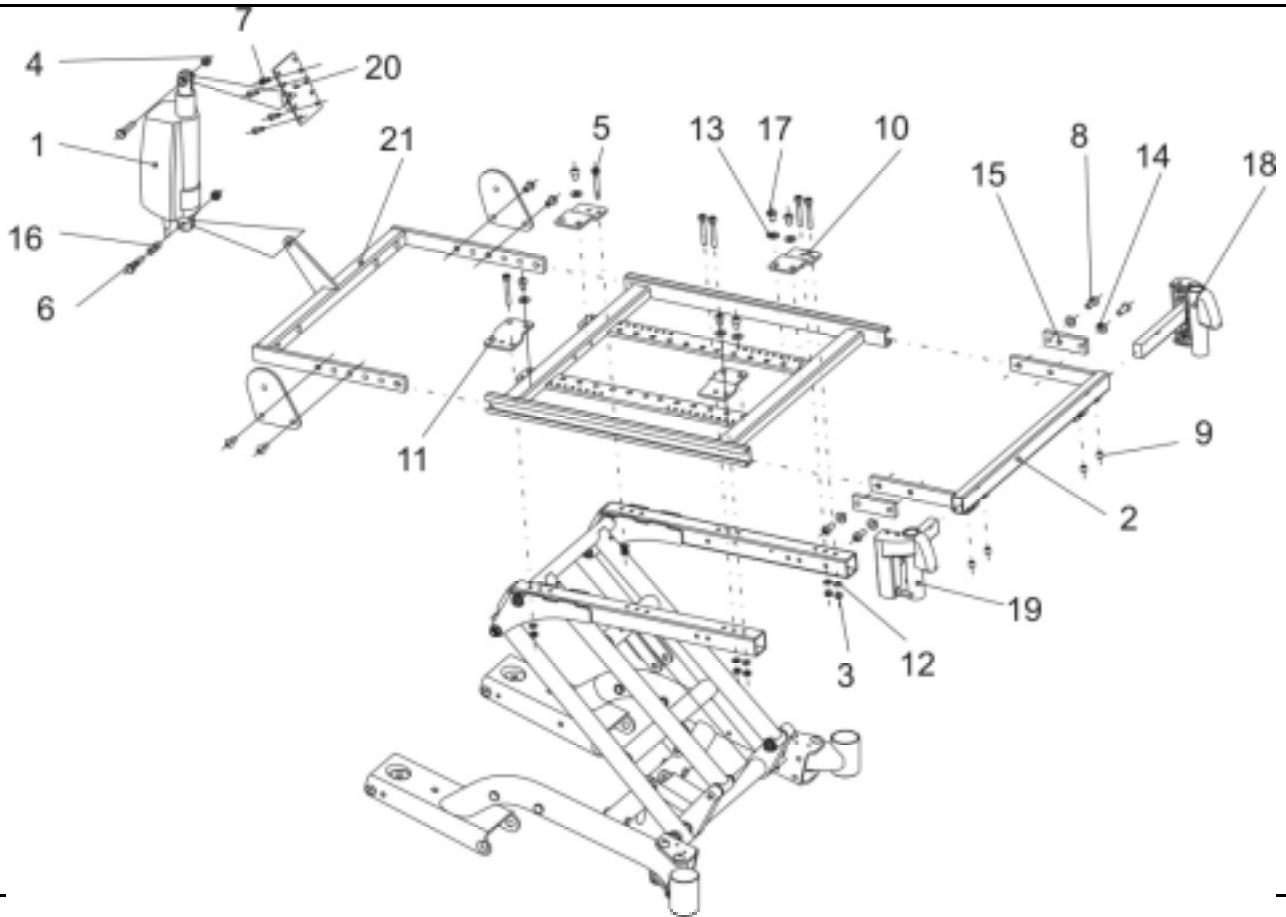

Item	Qty	Part-No.	Part-Designation
-	1		OPTIMIST SEAT w. EL.BACKREST ADJ.(ONLY SCANDINAVIA!...
-	1		OPTIMIST SEAT SB=38 M. EL.BACKREST ADJ. (ONLY SCAND :
-	1		OPTIMIST SEAT SB=50 M. EL.BACKREST ADJ. (ONLY SCANDINAV...
1	1	1419417	MOTOR LA 28.1-100-24-001
2	1	1418558-9005	FRONT TUBE "ARNE JENSEN" SB 45 (STD.)
2	1		FRAME TUBE AJ SB=45CM FRONT (STD. VERS.)
2	1	1418559-9005	CROSS TUBE SEAT FRAMES AJ SB=50CM
3	6	F20109	NYLOC NUT M6
4	2	F00002	NYLOC NUT M8
5	6	D50122	SOCKET HEAD CAP SCREW M6 X 45
6	2	D08111	HEX HEAD SCREW M8 X 35
7	4	1415005	SCREW M6 X 16
8	4	5824259	SCREW M8 X 20
9	4	D08144	HEX HEAD SCREW M8 X 20
10	2		AJ ADAPTER REAR LH ELECTRO-GALVANISED
11	2		AJ ADAPTER REAR RH ELECTRO-GALVANISED
12	6	G00110	WASHER 6,4 X 12,5 X 1,6
13	6	G00106	WASHER 8,4 X 17 X 1,6
14	4	G00107	WASHER 8,4 X 17 X 1,6
15	2		PLATE
16	1	F312038.89	BUSH RD. 10 X 1 X 26
17	6	5824302	SCREW M8 X 12
18	1		LEGREST SUPPORT LH HS-SEATFRAME
19	1		LEGREST SUPPORT RH HS-SEATFRAME

Storm 3



Release
 07 / 2004
 Chapter 9
 Sheet 8

