

Reservdels Katalog

Version: 2014/12/08a SV



Invacare® SB750™



Yes, you can.®

Allmän information.

Vid beställning av reservdelar måste alltid följande anges:

- Typ av produkt.
- Produktens namn och modell.
- Produktens serienummer.
- Reservdelens artikelnummer.
- Reservdelens benämning.

Sprängskisser och reservdelars positionsnummer är inte bindande och kan ändras utan föregående notis, allt i enlighet med Invacare's ständiga strävan att förbättra sina produkter och att tillmötesgå sina kunders behov.

All kopiering eller användning av detta material, eller av någon väsentlig del av det, oavsett format eller metod, elektronisk, mekanisk eller på annat sätt, inklusive fotokopiering, skanning eller inspelning, och lagring i elektroniskt media eller i informationssystem, inklusive elektroniska överföringar, får ej utföras utan skriftligt medgivande från Invacare® International Sarl.



Yes, you can.®

Innehållsförteckning

Färgspecifikation

Färger stål och plast	7
Trätyper	8

(Mattress supports)

(Mattress support 85 cm width with wooden lamellas)	9
Sänggram huvuddel 183+200 cm	10
(Top frame, head end, 183+200 cm length, with head rest)	11
(Head rest)	12
(Top frame, foot end, 183 cm length)	13
(Top frame, foot end, 200 cm length)	14
(Mattress support 85 cm width with aluminium lamellas)	15
(Top frame, head end, 183+200 cm length)	16
(Top frame, head end, 183+200 cm length, with head rest)	17
(Head rest)	18
(Top frame, foot end, 183 cm length)	19
(Top frame, foot end, 200 cm length)	20
(Mattress support 90 cm width with wooden lamellas)	21
(Top frame, head end, 183+200 cm length)	22
(Top frame, head end, 183+200 cm length, with head rest)	23
(Head rest)	12
(Top frame, foot end, 183 cm length)	24
(Top frame, foot end, 200 cm length)	25
(Mattress support 90 cm width with aluminium lamellas)	26
(Top frame, head end, 183+200 cm length)	27
(Top frame, head end, 183+200 cm length, with head rest)	28
(Head rest)	18
(Top frame, foot end, 183 cm length)	29
(Top frame, foot end, 200 cm length)	30
(Mattress support 105 cm width with steel lamellas)	31
(Top frame, head end, 200 cm length)	32
(Top frame, foot end, 200 cm length)	33
(Mattress support 120 cm width with steel lamellas)	34
(Top frame, head end, 200 cm length)	35
(Top frame, foot end, 200 cm length)	36

(Bases)

(Base for 85+90+105 cm width)	37
(Base for 120 cm width)	38

(Castors and brake systems)

(Castors with brake)	39
(Central brake system for 85+90+105 cm width)	40
(Ø80 mm castors with central brake)	41
(Base for 85+90+105 cm width)	37
(Ø100 mm castors with central brake)	42
(Base for 85+90+105 cm width)	37
(Ø125 mm castors with central brake)	43
(Base for 85+90+105 cm width)	37



Yes, you can.®

Innehållsförteckning

(Ø150 mm castors with central brake)	44
(Base for 85+90+105 cm width)	37
(Central brake system with dir. control for 85+90+105 cm width)	45
(Ø100 mm castors with central brake and dir. control)	46
(Base for 85+90+105 cm width)	37
(Ø125 mm castors with central brake and dir. control)	47
(Base for 85+90+105 cm width)	37
(Ø150 mm castors with central brake and dir. control)	48
(Base for 85+90+105 cm width)	37
(Central brake system with dir. control for 120 cm width)	49
(Ø125 mm castors with central brake and dir. control, 120 cm width)	50
(Base for 120 cm width)	38
(Lowering kit, 75mm castor)	51
(Lowering kit, Ø75 mm castor)	52

(Bed ends and -plates)

(Bed end and -brackets, 85 cm width)	53
(Oda bed end plates, 85 cm width)	54
(Piggy bed end plates, 85 cm width)	55
(Karin bed end plates, 85 cm width)	56
(Sanne bed end plates, 85 cm width)	57
(Sanne bed end plates, 85 cm width, increased)	58
(Flex III bed end plates, 85 cm width)	59
(Oda and Piggy bed end brackets, 85 cm width)	60
(Sanne bed end brackets)	61
(Flex III bed end brackets)	62
(Bed end and -brackets, 90 cm width)	63
(Oda bed ends, 90 cm width)	64
(Piggy bed ends, 90 cm width)	65
(Karin bed end)	66
(Sanne bed end plates)	67
(Sanne bed end plates, 90 cm width, increased)	68
(Flex III bed end plates)	69
(Fittings with U-profile for Oda and Piggy)	70
(Sanne bed end brackets)	61
(Flex III bed end brackets)	62
(Bed end and -brackets, 105 cm width)	71
(Piggy bed end plates, 105 cm width)	72
(Sanne bed end plates, 105 cm width)	73
(Sanne bed end plates, 105 cm width, increased)	74
(Oda and Piggy bed end brackets, 105 cm width)	75
(Sanne bed end brackets)	61
(Flex III bed end brackets)	62
(Bed end and -brackets, 120 cm width)	76
(Sanne bed end plates, 120 cm width)	77
(Sanne bed end plates, 120 cm width, increased)	78
(Flex III bed end plates, 120 cm width)	79
(Sanne bed end brackets)	61
(Flex III bed end brackets)	62

(Electrical parts and gas springs)



Yes, you can.®

Innehållsförteckning

(ACP box with accessories)	80
(Motor for back rest)	81
(Motor for backrest)	82
(Motor for backrest, with emergency lowering handle)	83
(Motor for backrest, V0)	84
(Motor for thigh section)	85
(Motor for thigh section)	86
(Motor for thigh section, V0)	87
(Motor and gas spring for leg section)	88
(Motor for leg section)	89
(Motor for leg section, V0)	90
(Gas spring for 85 cm width)	91
(Gas spring for 90 cm width)	92
(Motor for height adjustment)	93
(Motor for height adjustment)	94
(Motor for height adjustment, 120 cm width)	95
(Motor for height adjustment, 85+90+105 cm width, V0)	96
(Motor for height adjustment, 120 cm width, V0)	97
(Control unit for 85+90 cm width)	98
(Control unit without accumulator backup, 4 motors, EU)	99
(Control unit without accumulator backup, 3 motors, EU)	100
(Control unit with accumulator backup, 4 motors, EU)	101
(Control unit with accumulator backup, 3 motors, EU)	102
(Control unit without accumulator backup, 4 motors, UK)	103
(Control unit with accumulator backup, 4 motors, UK)	104
(Control unit without accumulator backup, 4 motors, Japan)	105
(Control unit with battery backup)	106
(Control unit without accumulator backup, 4 motors, V0, EU)	107
(Control unit for 105+120 cm width)	108
(Control unit without accumulator backup, 4 motors, EU)	109
(Control unit without accumulator backup, 4 motors, V0, EU)	110
(Hand control)	111
(Extension cable, handcontrol)	112

(Side rails)

(Britt III side rail, for 183 cm length)	113
Grind Britt III	114
(Britt II side rail, extended)	115
(Britt IV side rail)	116















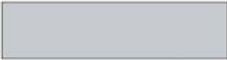

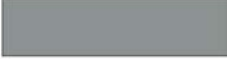


(Lifting pole)

(Lifting pole, Carebed, light)	117
--------------------------------------	-----

(Accessories)

(Support handles)	118
(Drip stand IV)	119
(Support for hand control)	120
(Sit-up handle)	121

Färger stål och plast





Chromated, glossy 0001		0153		7035
Zinc, natural 0002		0154		7039
Zinc, black 0003		1015		7042
Anodized, black 0004		3004		7046
Anodized, natural 0005		5002		8016
Chromated, matte 0011		7012		9005
		7016		9006
		7032		9010
		0152		

NB: Colours are for reference only
Not all colours are available for all products

(Not for separate sale - please contact Invacare)

1

Trätyper

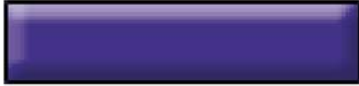 Beech	0101		0112
 Cherry	0102		0113
 Mahogany	0103		0114
 Light beech	0105		0115
	0110		0116
	0111		0117

-


(Not for separate sale - please contact Invacare)

1


(Mattress support 85 cm width with wooden lamellas)

-	Sängram huvuddel 183+200 cm	1
-	(Top frame, head end, 183+200 cm length, with head rest)	1
-	(Top frame, foot end, 183 cm length)	1
-	(Top frame, foot end, 200 cm length)	1

Sänggram huvuddel 183+200 cm

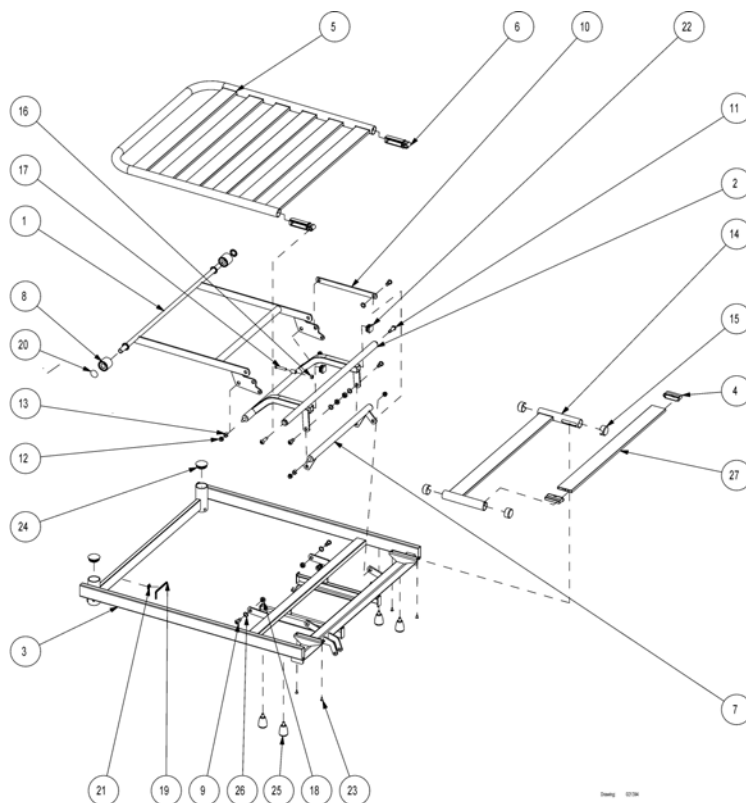


1	219567-1015	Lifter arm for backrest	1
2	221150-1015	Steering arm head end 85 and 90 cm width	1
3	221149-1015	Top frame head end 85 cm width	1
4	314884-7035	Beslag sänglatta SB750 ljusgrå	16
5	319787	Rörbygel ryggdel SB750	1
6	319800	Hängselled zink SB750	2
7	320881-1015	Parallel arm for backrest section 85 and 90 cm width	1
8	405479-7035	Nylonrulle d 40/17x38	2
9	1493130	Bult ryggdel SB750 ingår i 1435195	6
10	420620-1015	Fläns SB750	1
11	420623	Bult m8x3.4	2
12	420625-7035	Plastbussning styrarmar SB750 grå	4
13	420649-7035	Fixeringsrör styrarmar SB750 grå	2
14	421459	Rörstycke SB750	2
15	421460-7035	Doppsko sätesdel SB750	4
16	802729	Mutter m8 fzb	1
17	803008	Skruv m8x40 till SB750 lyftarm	1
18	806651	Låsmutter m6m 10loc-king fzb	6
19	811408	Sexkantnyckel nv 5mm hfc 753 din 911	1
20	815764	Starlock dia 16 SB750	2
21	816488	Kabelclips 46x41 hfc 2540	1
22	1423912-7035	Doppsko SB750	2
23	1423996	Blind rivet ø48x9	4
24	1432700-7035	Doppsko	2
25	1435325-7035	Handgrepp/vred sängbotten SB750/SB200	4
26	1446475	Fibre washer ø13xø19x05 red	6
27	420628	Trälatta 85/90 cm SB750	8

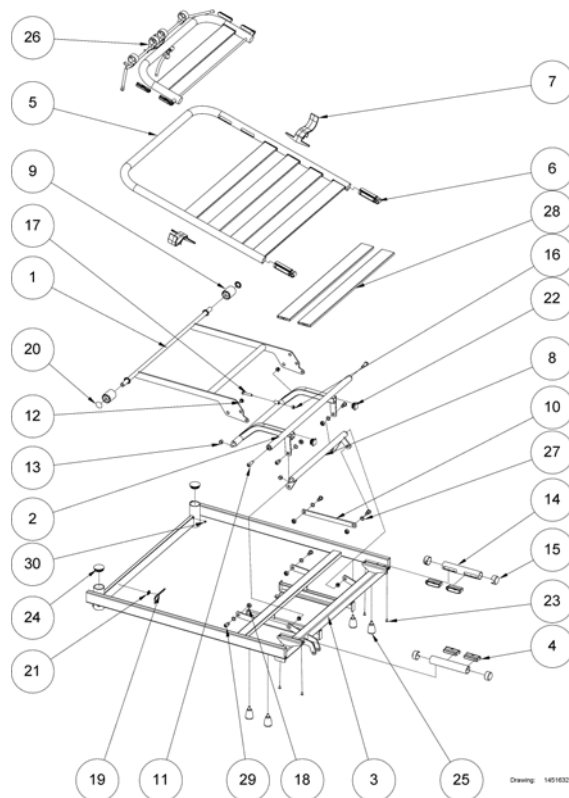(Top frame, head end, 183+200 cm length, with head rest)


26	-	(Head rest)	1
1	219567-1015	Lifter arm for backrest	1
2	221150-1015	Steering arm head end 85 and 90 cm width	1
3	221149-1015	Top frame head end 85 cm width	1
4	314884-7035	Beslag sänglatta SB750 ljusgrå	12
5	319787	Rörbygel ryggdel SB750	1
6	319800	Hängselled zink SB750	2
7	320584-7035	Madrassöd grå SB750 90cm	2
8	320881-1015	Parallel arm for backrest section 85 and 90 cm width	1
9	405479-7035	Nylonrulle d 40/17x38	2
10	420620-1015	Fläns SB750	1
11	420623	Bult m8x3.4	2
12	420625-7035	Plastbussning styrarmar SB750 grå	4
13	420649-7035	Fixeringsrör styrarmar SB750 grå	2
14	421459	Rörstycke SB750	2
15	421460-7035	Doppsko sätesdel SB750	4
16	802729	Mutter m8 fzb	1
17	803008	Skruv m8x40 till SB750 lyftarm	1
18	806651	Låsmutter m6m 10loc-king fzb	6
19	811408	Sexkantnyckel nv 5mm hfc 753 din 911	1
20	815764	Starlock dia 16 SB750	2
21	816488	Kabelclips 46x41 hfc 2540	1
22	1423912-7035	Doppsko SB750	2
23	1423996	Blind rivet ø48x9	4
24	1432700-7035	Doppsko	2
25	1435325-7035	Handgrepp/vred sängbotten SB750/SB200	4
27	1446475	Fibre washer ø13xø19x05 red	6
28	420628	Trälatta 85/90 cm SB750	6
29	1493130	Bult ryggdel SB750 ingår i 1435195	6
30	1538319	(Plastic cap Ø8)	2

(Head rest)

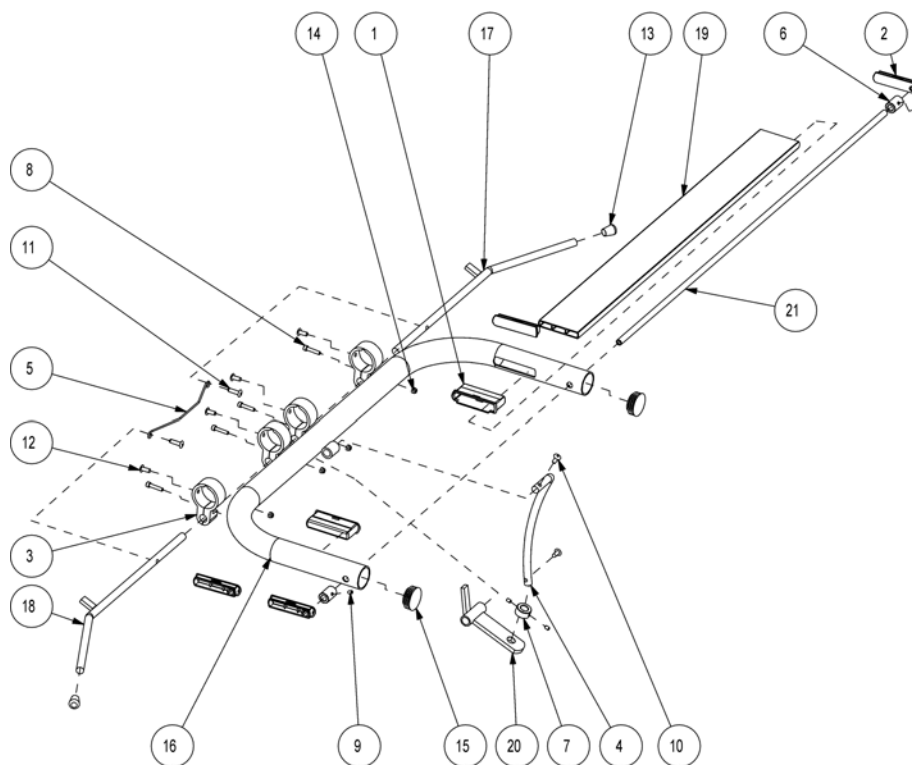


Drawing: 1441324

1	314884-7035	Beslag sänglatta SB750 ljusgrå	2
2	316825-7035	Bracket for head rest r al 7035	4
3	316839-7035	Holder for head rest ra l 7035	4
4	409155.60	Regleringsstag nacke obeh	1
5	410739	Utlösningsfjäder dia 2 rostfri	1
6	416785	Bussning till SB710	2
7	416788.60	Stop ring ø20/ø105x10	1
8	803024	Screw m4x208.8 iso 4762	4
9	803216	Stoppskruv m4x6 s6ss m4x6	4
10	803809	Popnit tlp/d/bs 639	2
11	803811	Popnit tlp/d/bs 665 lf 48x16 hfc 1460	2
12	803833	Popnit SB700/750	4
13	804387	Ändpropp dia 9 gummi	2
14	804464	Låsmutter m4 fzb hfc 840 din 985-8	4
15	809982	Ändpropp inv. sfl 32 plast	2
17	1441322-1015	Release arm for head rest right	1
18	1441323-1015	Release arm for head rest left	1
19	1443906	Lamella wooden 602x62x 95	1
20	1443909-1015	Locking lug with bushing/stop for head rest	1
21	1443910-0002	Rotation tube for head rest ø8x1,5x710	1

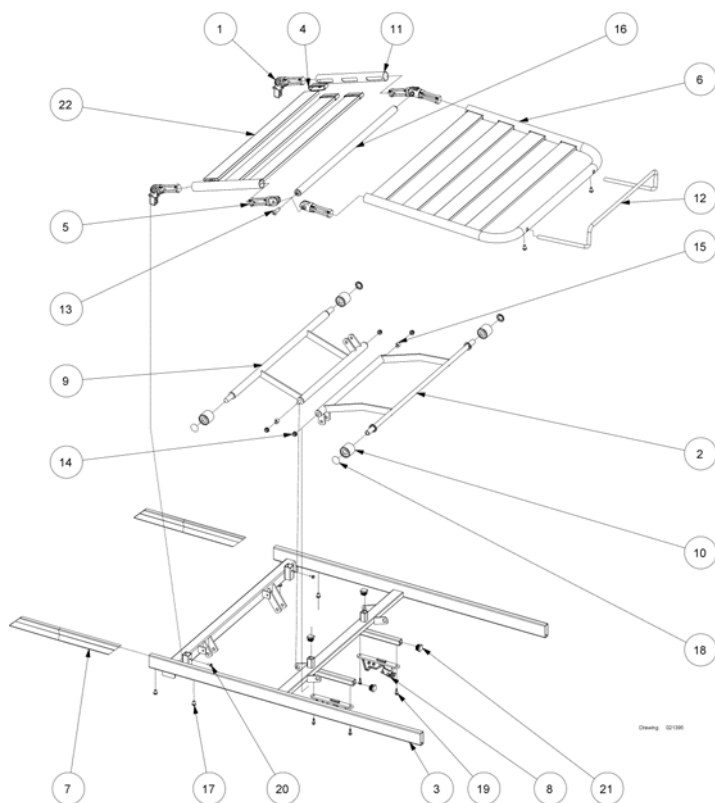(Top frame, foot end, 183 cm length)

1	021351	Hängselled SB750	2
2	221146-1015	Lifter arm for leg section 85 and 90 cm width	1
3	221134-1015	Top frame foot end 85x183 cm	1
4	314884-7035	Beslag sänglatta SB750 ljusgrå	12
5	314885	Hängselled sängbotten SB750	4
6	320850	Tubular bow for leg section length 183 cm ø32	1
7	320516.60	Insticksrör SB750	2
8	320529-7035	Låsbeslag SB750 grå sängbotten/lyft st	2
9	320882-1015	Lifter arm for thigh section 85 and 90 cm width	1
10	405479-7035	Nylonrulle d 40/17x38	4
11	420127	Rörstycke rördel SB750	2
13	420623	Bult m8x3.4	2
14	420625-7035	Plastbussning styrarmar SB750 grå	4
15	420649-7035	Fixeringsrör styrarmar SB750 grå	2
16	421458-1015	Distance tube for lifter arm 85 and 90 cm width	1
17	812875	Skruv mc6s m8x12 fzb hfc 9729	6
18	815764	Starlock dia 16 SB750	4
19	819407	Skruv självgående 5.5x19 SB750	4
20	821235	Screw ø5x12 thread form ing	2
21	1423915	Doppsko 20x20 SB750	4
22	420628	Trälatta 85/90 cm SB750	6
12	420163-0005	Madrass stopp fotände SB750	1

(Top frame, foot end, 200 cm length)

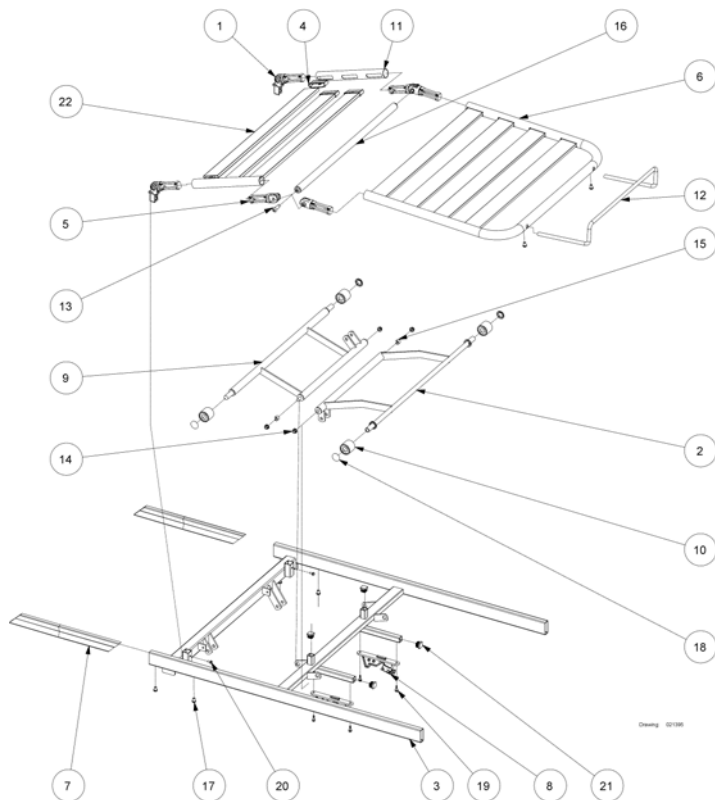


1	021351	Hängselled SB750	2
2	221155-1015	Lifter arm for leg section 85 and 90 cm width	1
3	221154-1015	Top frame foot end 85x200 cm	1
4	314884-7035	Beslag sänglatta SB750 ljusgrå	14
5	314885	Hängselled sängbotten SB750	4
6	319788	Rörbygel bendel SB750	1
7	320516.60	Insticksrör SB750	2
8	320529-7035	Låsbeslag SB750 grå sängbotten/lyft st	2
9	320882-1015	Lifter arm for thigh section 85 and 90 cm width	1
10	405479-7035	Nylonrulle d 40/17x38	4
11	420127	Rörstycke rördel SB750	2
13	420623	Bult m8x3.4	2
14	420625-7035	Plastbussning styrarmar SB750 grå	4
15	420649-7035	Fixeringsrör styrarmar SB750 grå	2
16	421458-1015	Distance tube for lifter arm 85 and 90 cm width	1
17	812875	Skruv mc6s m8x12 fzb hfc 9729	6
18	815764	Starlock dia 16 SB750	4
19	819407	Skruv självgående 5.5x19 SB750	4
20	821235	Screw ø5x12 thread form ing	2
21	1423915	Doppsko 20x20 SB750	4
22	420628	Trälatta 85/90 cm SB750	7
12	420163-0005	Madrass stopp fotände SB750	1


(Mattress support 85 cm width with aluminium lamellas)

-	(Top frame, head end, 183+200 cm length)	1
-	(Top frame, head end, 183+200 cm length, with head rest)	1
-	(Top frame, foot end, 183 cm length)	1
-	(Top frame, foot end, 200 cm length)	1

(Top frame, head end, 183+200 cm length)

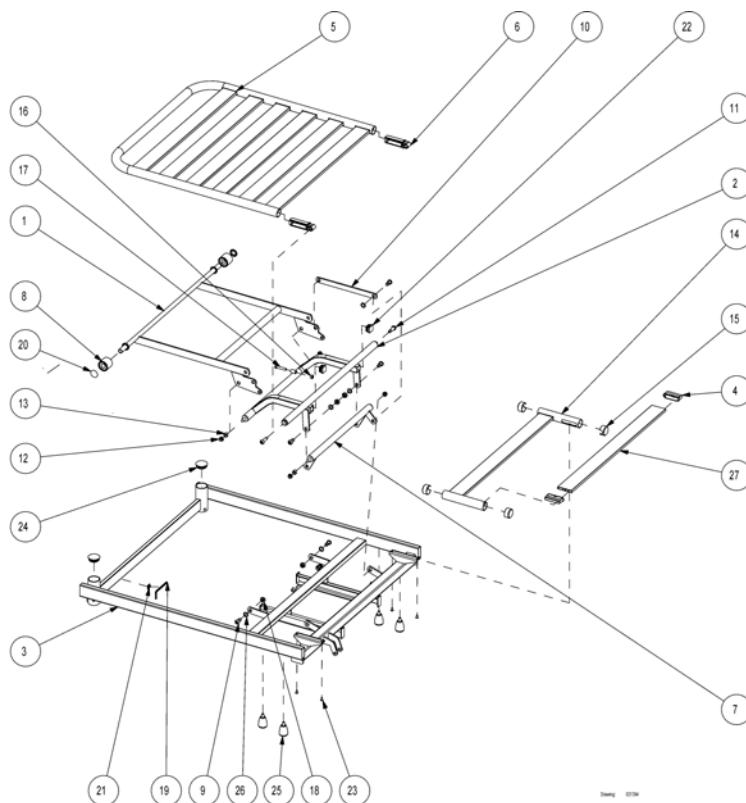


1	219567-1015	Lifter arm for backrest	1
2	221150-1015	Steering arm head end 85 and 90 cm width	1
3	221149-1015	Top frame head end 85 cm width	1
4	314884-7035	Beslag sänglatta SB750 ljusgrå	16
5	319787	Rörbygel ryggdel SB750	1
6	319800	Hängselled zink SB750	2
7	320881-1015	Parallel arm for backrest section 85 and 90 cm width	1
8	405479-7035	Nylonrulle d 40/17x38	2
9	1493130	Bult ryggdel SB750 ingår i 1435195	6
10	420620-1015	Fläns SB750	1
11	420623	Bult m8x3.4	2
12	420625-7035	Plastbussning styrarmar SB750 grå	4
13	420649-7035	Fixeringsrör styrarmar SB750 grå	2
14	421459	Rörstycke SB750	2
15	421460-7035	Doppsko sätesdel SB750	4
16	802729	Mutter m8 fzb	1
17	803008	Skruv m8x40 till SB750 lyftarm	1
18	806651	Låsmutter m6m 10loc-king fzb	6
19	811408	Sexkantnyckel nv 5mm hfc 753 din 911	1
20	815764	Starlock dia 16 SB750	2
21	816488	Kabelclips 46x41 hfc 2540	1
22	1423912-7035	Doppsko SB750	2
23	1423996	Blind rivet ø48x9	4
24	1432700-7035	Doppsko	2
25	1435325-7035	Handgrepp/vred sängbotten SB750/SB200	2
26	1446475	Fibre washer ø13xø19x05 red	6
27	1484059	Lamell 85/90cm SB750 70cm	8

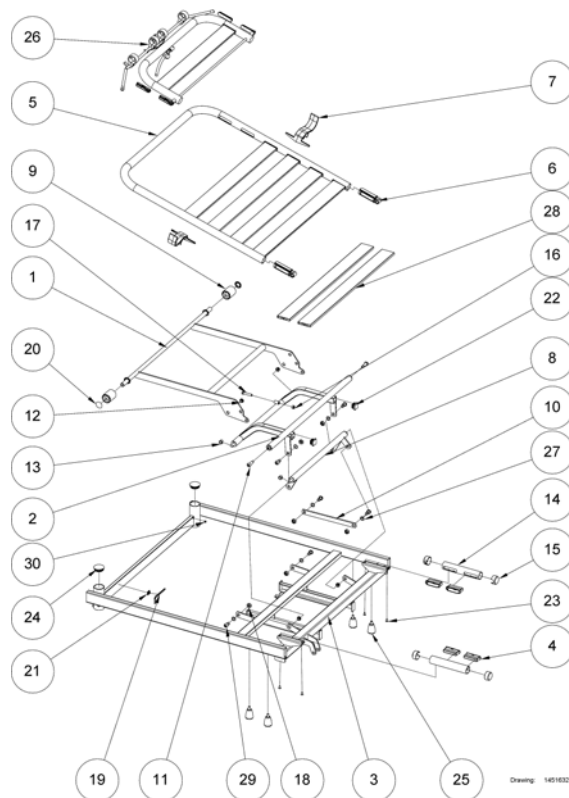(Top frame, head end, 183+200 cm length, with head rest)


26	-	(Head rest)	1
1	219567-1015	Lifter arm for backrest	1
2	221150-1015	Steering arm head end 85 and 90 cm width	1
3	221149-1015	Top frame head end 85 cm width	1
4	314884-7035	Beslag sänglatta SB750 ljusgrå	12
5	319787	Rörbygel ryggdel SB750	1
6	319800	Hängselled zink SB750	2
7	320584-7035	Madrasstöd grå SB750 90cm	2
8	320881-1015	Parallel arm for backrest section 85 and 90 cm width	1
9	405479-7035	Nylonrulle d 40/17x38	2
10	420620-1015	Fläns SB750	1
11	420623	Bult m8x3.4	2
12	420625-7035	Plastbussning styrarmar SB750 grå	4
13	420649-7035	Fixeringsrör styrarmar SB750 grå	2
14	421459	Rörstycke SB750	2
15	421460-7035	Doppsko sätesdel SB750	4
16	802729	Mutter m8 fzb	1
17	803008	Skruv m8x40 till SB750 lyftarm	1
18	806651	Låsmutter m6m 10loc-king fzb	6
19	811408	Sexkantnyckel nv 5mm hfc 753 din 911	1
20	815764	Starlock dia 16 SB750	2
21	816488	Kabelclips 46x41 hfc 2540	1
22	1423912-7035	Doppsko SB750	2
23	1423996	Blind rivet ø48x9	4
24	1432700-7035	Doppsko	2
25	1435325-7035	Handgrepp/vred sängbotten SB750/SB200	4
27	1446475	Fibre washer ø13xø19x05 red	6
28	1484059	Lamell 85/90cm SB750 70cm	6
29	1493130	Bult ryggdel SB750 ingår i 1435195	6
30	1538319	(Plastic cap Ø8)	2

(Head rest)

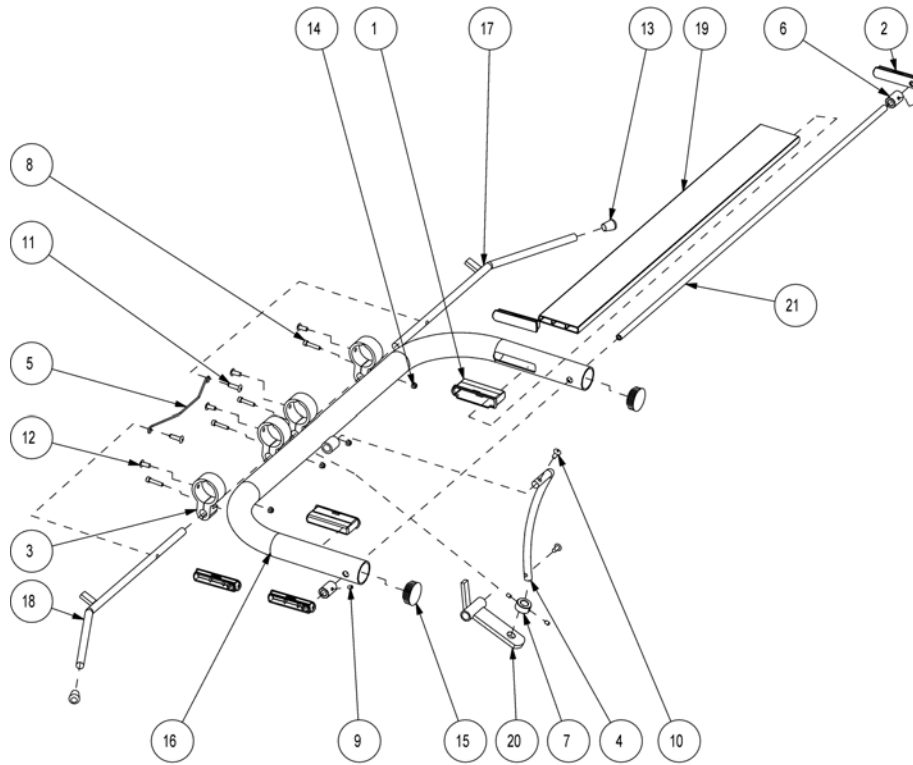


Drawing: 1441324

1	314884-7035	Beslag sänglatta SB750 ljusgrå	2
2	316825-7035	Bracket for head rest r al 7035	4
3	316839-7035	Holder for head rest ra l 7035	4
4	409155.60	Regleringsstag nacke obeh	1
5	410739	Utlösningsfjäder dia 2 rostfri	1
6	416785	Bussning till SB710	2
7	416788.60	Stop ring $\varnothing 20/\varnothing 105 \times 10$	1
8	803024	Screw m4x208.8 iso 4762	4
9	803216	Stoppskruv m4x6 s6ss m4x6	4
10	803809	Popnit tlp/d/bs 639	2
11	803811	Popnit tlp/d/bs 665 lf 48x16 hfc 1460	2
12	803833	Popnit SB700/750	4
13	804387	Ändpropp dia 9 gummi	2
14	804464	Låsmutter m4 fzb hfc 840 din 985-8	4
15	809982	Ändpropp inv. sfl 32 plast	2
17	1441322-1015	Release arm for head rest right	1
18	1441323-1015	Release arm for head rest left	1
20	1443909-1015	Locking lug with bushing/stop for head rest	1
21	1443910-0002	Rotation tube for head rest $\varnothing 8 \times 1,5 \times 710$	1

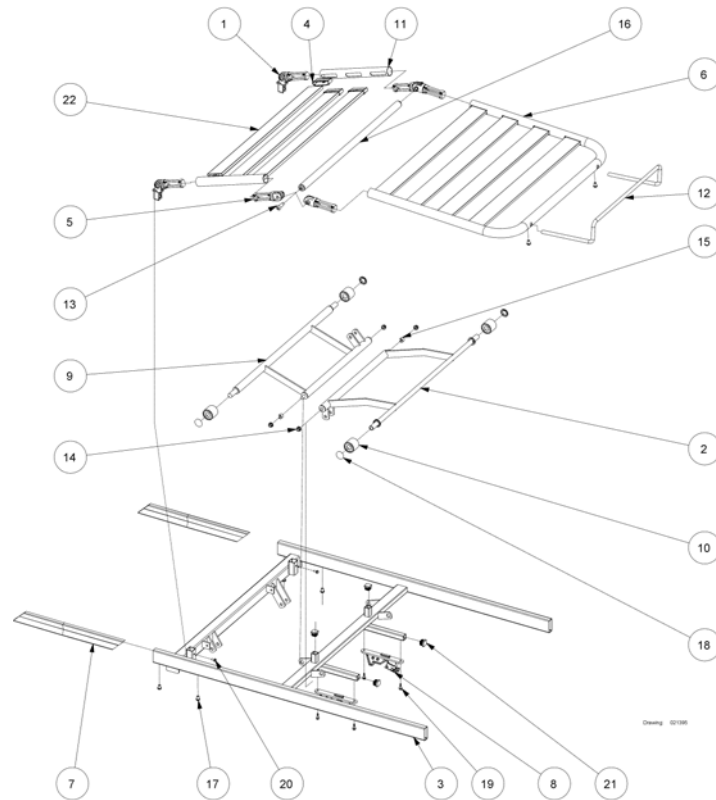(Top frame, foot end, 183 cm length)

1	021351	Hängselled SB750	2
2	221146-1015	Lifter arm for leg section 85 and 90 cm width	1
3	221134-1015	Top frame foot end 85x183 cm	1
4	314884-7035	Beslag sänglatta SB750 ljusgrå	12
5	314885	Hängselled sängbotten SB750	4
6	320850	Tubular bow for leg section length 183 cm ø32	1
7	320516.60	Insticksrör SB750	2
8	320529-7035	Låsbeslag SB750 grå sängbotten/lyft st	2
9	320882-1015	Lifter arm for thigh section 85 and 90 cm width	1
10	405479-7035	Nylonrulle d 40/17x38	4
11	420127	Rörstycke rördel SB750	2
13	420623	Bult m8x3.4	2
14	420625-7035	Plastbussning styrarmar SB750 grå	4
15	420649-7035	Fixeringsrör styrarmar SB750 grå	2
16	421458-1015	Distance tube for lifter arm 85 and 90 cm width	1
17	812875	Skruv mc6s m8x12 fzb hfc 9729	6
18	815764	Starlock dia 16 SB750	4
19	819407	Skruv självgående 5.5x19 SB750	4
20	821235	Screw ø5x12 thread form ing	2
21	1423915	Doppsko 20x20 SB750	4
22	1484059	Lamell 85/90cm SB750 70cm	7
12	420163-0005	Madrass stopp fotände SB750	1

(Top frame, foot end, 200 cm length)

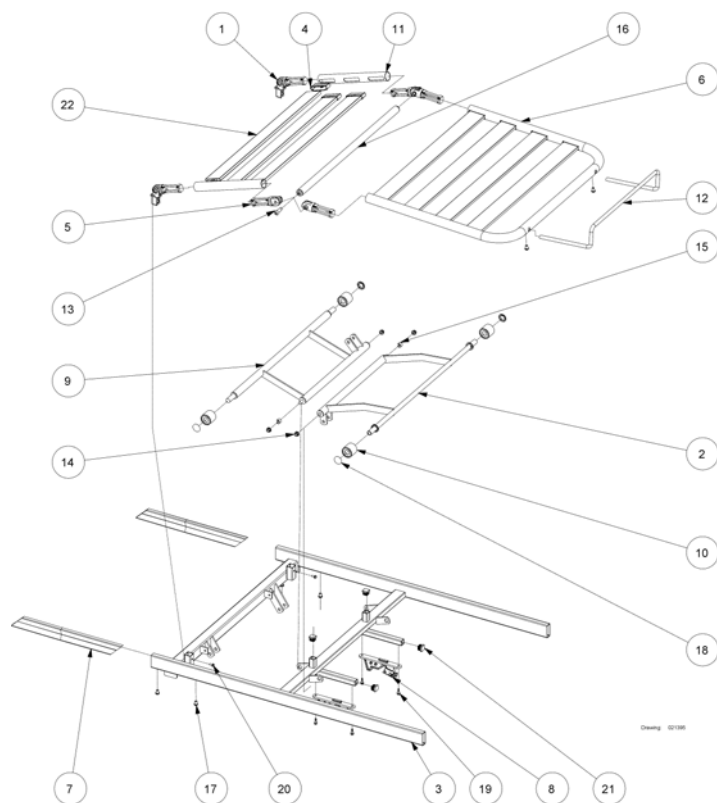


1	021351	Hängselled SB750	2
2	221155-1015	Lifter arm for leg section 85 and 90 cm width	1
3	221154-1015	Top frame foot end 85x200 cm	1
3	221162-1015	Top frame foot end 90x200 cm	1
4	314884-7035	Beslag sänglatta SB750 ljusgrå	14
5	314885	Hängselled sängbotten SB750	4
6	319788	Rörbygel bendel SB750	1
7	320516.60	Insticksrör SB750	2
8	320529-7035	Låsbeslag SB750 grå sängbotten/lyft st	2
9	320882-1015	Lifter arm for thigh section 85 and 90 cm width	1
10	405479-7035	Nylonrulle d 40/17x38	4
11	420127	Rörstycke rördel SB750	2
12	420163-0005	Madress stopp fotände SB750	1
13	420623	Bult m8x3.4	2
14	420625-7035	Plastbussning styrarmar SB750 grå	4
15	420649-7035	Fixeringsrör styrarmar SB750 grå	2
16	421458-1015	Distance tube for lifter arm 85 and 90 cm width	1
17	812875	Skruv mc6s m8x12 fzb hfc 9729	6
18	815764	Starlock dia 16 SB750	4
19	819407	Skruv självgängande 5.5x19 SB750	4
20	821235	Screw ø5x12 thread form ing	2
21	1423915	Doppsko 20x20 SB750	4
22	1484059	Lamell 85/90cm SB750 70cm	7


(Mattress support 90 cm width with wooden lamellas)

-	(Top frame, head end, 183+200 cm length)	1
-	(Top frame, head end, 183+200 cm length, with head rest)	1
-	(Top frame, foot end, 183 cm length)	1
-	(Top frame, foot end, 200 cm length)	1

(Top frame, head end, 183+200 cm length)

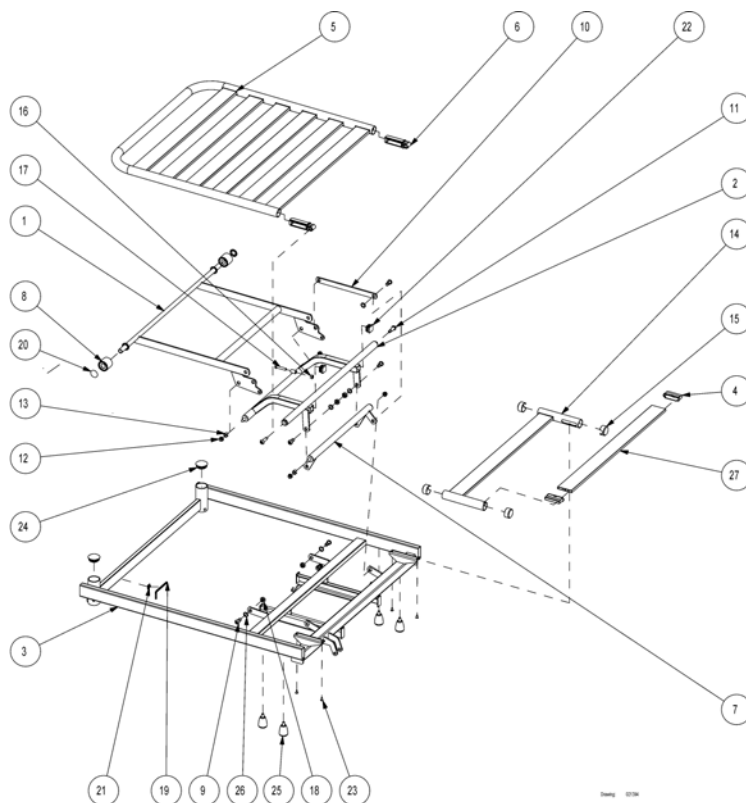


1	219567-1015	Lifter arm for backrest	1
2	221150-1015	Steering arm head end 85 and 90 cm width	1
3	221161-1015	Top frame head end 90 cm width	1
4	314884-7035	Beslag sänglatta SB750 ljusgrå	16
5	319787	Rörbygel ryggdel SB750	1
6	319800	Hängselled zink SB750	2
7	320881-1015	Parallel arm for backrest section 85 and 90 cm width	1
8	405479-7035	Nylonrulle d 40/17x38	2
9	1493130	Bult ryggdel SB750 ingår i 1435195	6
10	420620-1015	Fläns SB750	1
11	420623	Bult m8x3.4	2
12	420625-7035	Plastbussning styrarmar SB750 grå	4
13	420649-7035	Fixeringsrör styrarmar SB750 grå	2
14	421459	Rörstycke SB750	2
15	421460-7035	Doppsko sätesdel SB750	4
16	802729	Mutter m8 fzb	1
17	803008	Skruv m8x40 till SB750 lyftarm	1
18	806651	Låsmutter m6m 10loc-king fzb	6
19	811408	Sexkantnyckel nv 5mm hfc 753 din 911	1
20	815764	Starlock dia 16 SB750	2
21	816488	Kabelclips 46x41 hfc 2540	1
22	1423912-7035	Doppsko SB750	2
23	1423996	Blind rivet ø48x9	4
24	1432700-7035	Doppsko	2
25	1435325-7035	Handgrepp/vred sängbotten SB750/SB200	4
26	1446475	Fibre washer ø13xø19x05 red	6
27	420628	Trälatta 85/90 cm SB750	8

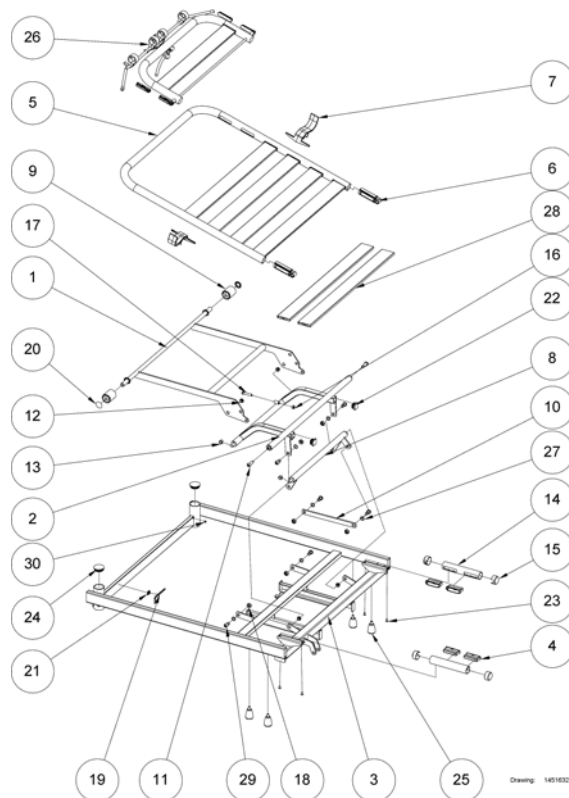
(Top frame, head end, 183+200 cm length, with head rest)

26	-	(Head rest)	1
1	219567-1015	Lifter arm for backrest	1
2	221150-1015	Steering arm head end 85 and 90 cm width	1
3	221161-1015	Top frame head end 90 cm width	1
4	314884-7035	Beslag sänglatta SB750 ljusgrå	14
5	319787	Rörbygel ryggdel SB750	1
6	319800	Hängselled zink SB750	2
7	320584-7035	Madrasstöd grå SB750 90cm	2
8	320881-1015	Parallel arm for backrest section 85 and 90 cm width	1
9	405479-7035	Nylonrulle d 40/17x38	2
10	420620-1015	Fläns SB750	1
11	420623	Bult m8x3.4	2
12	420625-7035	Plastbussning styrarmar SB750 grå	4
13	420649-7035	Fixeringsrör styrarmar SB750 grå	2
14	421459	Rörstycke SB750	2
15	421460-7035	Doppsko sätesdel SB750	4
16	802729	Mutter m8 fzb	1
17	803008	Skruv m8x40 till SB750 lyftarm	1
18	806651	Låsmutter m6m 10loc-king fzb	6
19	811408	Sexkantnyckel nv 5mm hfc 753 din 911	1
20	815764	Starlock dia 16 SB750	2
21	816488	Kabelclips 46x41 hfc 2540	1
22	1423912-7035	Doppsko SB750	2
23	1423996	Blind rivet ø48x9	4
24	1432700-7035	Doppsko	2
25	1435325-7035	Handgrepp/vred sängbotten SB750/SB200	2
27	1446475	Fibre washer ø13xø19x05 red	6
28	420628	Trälatta 85/90 cm SB750	6
29	1493130	Bult ryggdel SB750 ingår i 1435195	6
30	1538319	(Plastic cap Ø8)	2

(Top frame, foot end, 183 cm length)

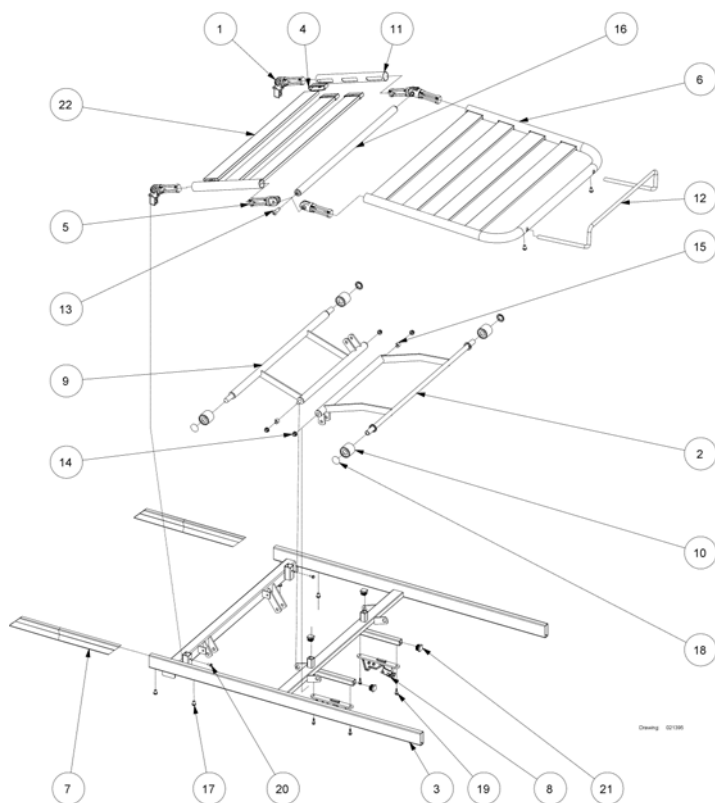


1	021351	Hängselled SB750	2
2	221146-1015	Lifter arm for leg section 85 and 90 cm width	1
3	221165-1015	Top frame foot end 90x183 cm	1
4	314884-7035	Beslag sänglatta SB750 ljusgrå	12
5	314885	Hängselled sängbotten SB750	4
6	320850	Tubular bow for leg section length 183 cm ø32	1
7	320516.60	Insticksrör SB750	2
8	320529-7035	Låsbeslag SB750 grå sängbotten/lyft st	2
9	320882-1015	Lifter arm for thigh section 85 and 90 cm width	1
10	405479-7035	Nylonrulle d 40/17x38	4
11	420127	Rörstycke rördel SB750	2
13	420623	Bult m8x3.4	2
14	420625-7035	Plastbussning styrarmar SB750 grå	4
15	420649-7035	Fixeringsrör styrarmar SB750 grå	2
16	421458-1015	Distance tube for lifter arm 85 and 90 cm width	1
17	812875	Skruv mc6s m8x12 fzb hfc 9729	6
18	815764	Starlock dia 16 SB750	4
19	819407	Skruv självgående 5.5x19 SB750	4
20	821235	Skrub ø5x12 thread form ing	2
21	1423915	Doppsko 20x20 SB750	4
22	420628	Trälatta 85/90 cm SB750	7
12	420163-0005	Madrass stopp fotände SB750	1

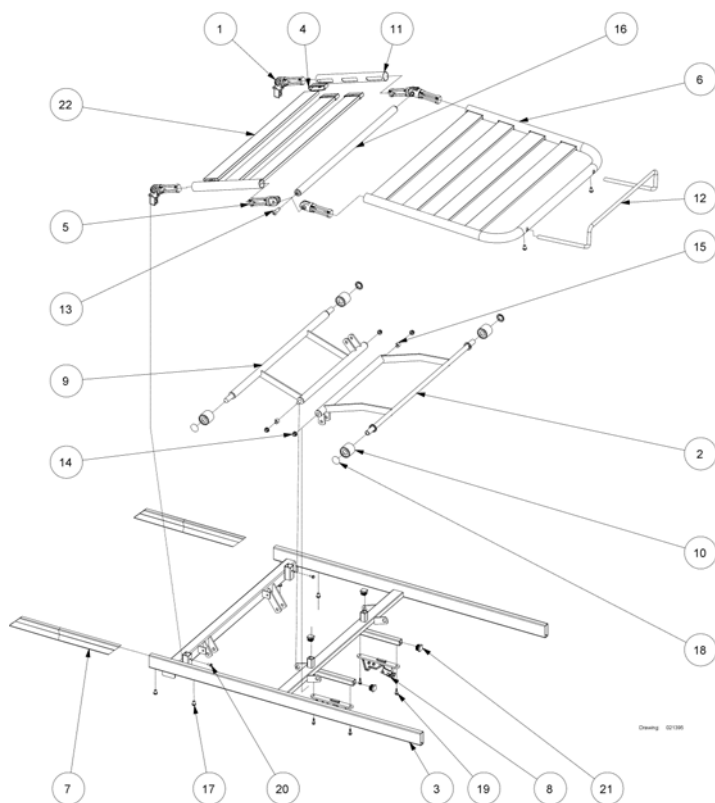(Top frame, foot end, 200 cm length)


1	021351	Hängselled SB750	2
2	221155-1015	Lifter arm for leg section 85 and 90 cm width	1
3	221162-1015	Top frame foot end 90x200 cm	1
4	314884-7035	Beslag sänglatta SB750 ljusgrå	14
5	314885	Hängselled sängbotten SB750	4
6	319788	Rörbygel bendel SB750	1
7	320516.60	Insticksrör SB750	2
8	320529-7035	Låsbeslag SB750 grå sängbotten/lyft st	2
9	320882-1015	Lifter arm for thigh section 85 and 90 cm width	1
10	405479-7035	Nylonrulle d 40/17x38	4
11	420127	Rörstycke rördel SB750	2
13	420623	Bult m8x3.4	2
14	420625-7035	Plastbussning styrarmar SB750 grå	4
15	420649-7035	Fixeringsrör styrarmar SB750 grå	2
16	421458-1015	Distance tube for lifter arm 85 and 90 cm width	1
17	812875	Skruv mc6s m8x12 fzb hfc 9729	6
18	815764	Starlock dia 16 SB750	4
19	819407	Skruv självgående 5.5x19 SB750	4
20	821235	Screw ø5x12 thread form ing	2
21	1423915	Doppsko 20x20 SB750	4
22	420628	Trälatta 85/90 cm SB750	7
12	420163-0005	Madrass stopp fotände SB750	1

(Mattress support 90 cm width with aluminium lamellas)


-	(Top frame, head end, 183+200 cm length)	1
-	(Top frame, head end, 183+200 cm length, with head rest)	1
-	(Top frame, foot end, 183 cm length)	1
-	(Top frame, foot end, 200 cm length)	1

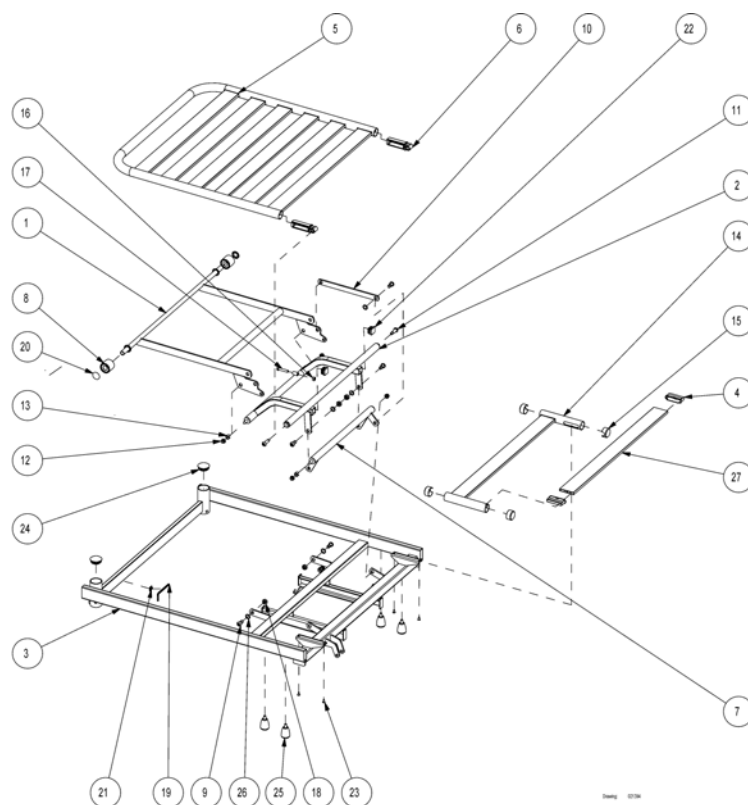(Top frame, head end, 183+200 cm length)

1	219567-1015	Lifter arm for backrest	1
2	221150-1015	Steering arm head end 85 and 90 cm width	1
3	221161-1015	Top frame head end 90 cm width	1
4	314884-7035	Beslag sänglatta SB750 ljusgrå	16
5	319787	Rörbygel ryggdel SB750	1
6	319800	Hängselled zink SB750	2
7	320881-1015	Parallel arm for backrest section 85 and 90 cm width	1
8	405479-7035	Nylonrulle d 40/17x38	2
9	1493130	Bult ryggdel SB750 ingår i 1435195	6
10	420620-1015	Fläns SB750	1
11	420623	Bult m8x3.4	2
12	420625-7035	Plastbussning styrarmar SB750 grå	4
13	420649-7035	Fixeringsrör styrarmar SB750 grå	2
14	421459	Rörstycke SB750	2
15	421460-7035	Doppsko sätesdel SB750	4
16	802729	Mutter m8 fzb	1
17	803008	Skruv m8x40 till SB750 lyftarm	1
18	806651	Låsmutter m6m 10loc-king fzb	6
19	811408	Sexkantnyckel nv 5mm hfc 753 din 911	1
20	815764	Starlock dia 16 SB750	2
21	816488	Kabelclips 46x41 hfc 2540	1
22	1423912-7035	Doppsko SB750	2
23	1423996	Blind rivet ø48x9	4
24	1432700-7035	Doppsko	2
25	1435325-7035	Handgrepp/vred sängbotten SB750/SB200	4
26	1446475	Fibre washer ø13xø19x05 red	6
27	1484059	Lamell 85/90cm SB750 70cm	8

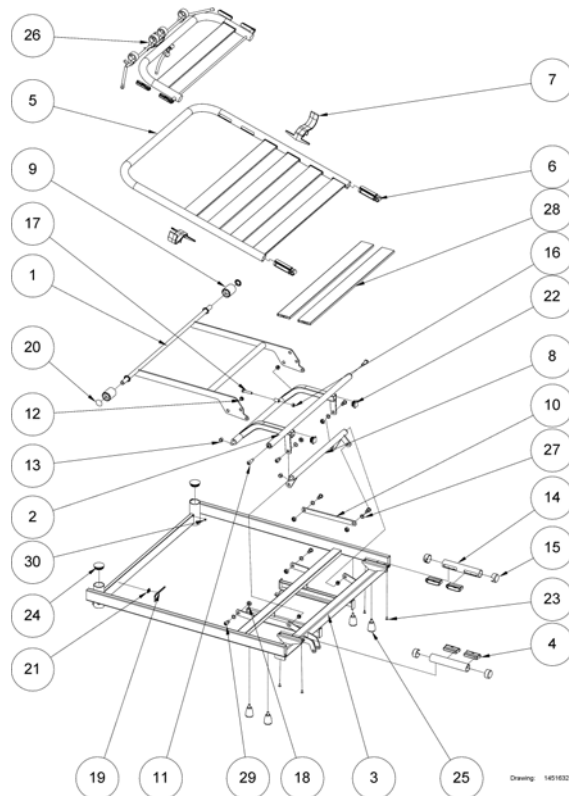
(Top frame, head end, 183+200 cm length, with head rest)

26	-	(Head rest)	1
1	219567-1015	Lifter arm for backrest	1
2	221150-1015	Steering arm head end 85 and 90 cm width	1
3	221161-1015	Top frame head end 90 cm width	1
4	314884-7035	Beslag sänglatta SB750 ljusgrå	12
5	319787	Rörbygel ryggdel SB750	1
6	319800	Hängselled zink SB750	2
7	320584-7035	Madrasstöd grå SB750 90cm	2
8	320881-1015	Parallel arm for backrest section 85 and 90 cm width	1
9	405479-7035	Nylonrulle d 40/17x38	2
10	420620-1015	Fläns SB750	1
11	420623	Bult m8x3.4	2
12	420625-7035	Plastbussning styrarmar SB750 grå	4
13	420649-7035	Fixeringsrör styrarmar SB750 grå	2
14	421459	Rörstycke SB750	2
15	421460-7035	Doppsko sätesdel SB750	4
16	802729	Mutter m8 fzb	1
17	803008	Skruv m8x40 till SB750 lyftarm	1
18	806651	Låsmutter m6m 10loc-king fzb	6
19	811408	Sexkantnyckel nv 5mm hfc 753 din 911	1
20	815764	Starlock dia 16 SB750	2
21	816488	Kabelclips 46x41 hfc 2540	1
22	1423912-7035	Doppsko SB750	2
23	1423996	Blind rivet ø48x9	0
24	1432700-7035	Doppsko	2
25	1435325-7035	Handgrepp/vred sängbotten SB750/SB200	4
27	1446475	Fibre washer ø13xø19x05 red	6
28	1484059	Lamell 85/90cm SB750 70cm	6
29	1493130	Bult ryggdel SB750 ingår i 1435195	6
30	1538319	(Plastic cap Ø8)	2

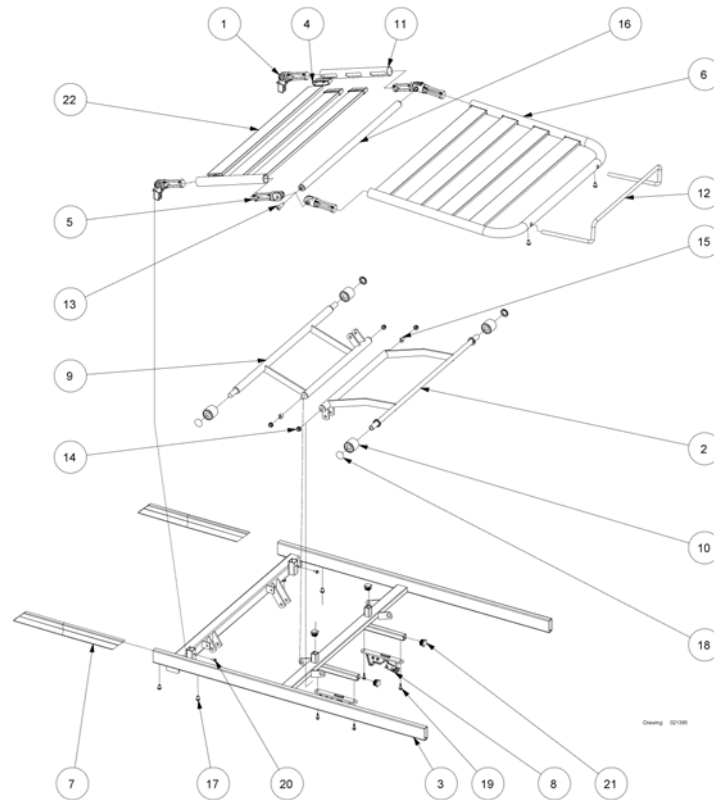(Top frame, foot end, 183 cm length)

1	021351	Hängselled SB750	2
2	221146-1015	Lifter arm for leg section 85 and 90 cm width	1
3	221165-1015	Top frame foot end 90x183 cm	1
4	314884-7035	Beslag sänglatta SB750 ljusgrå	12
5	314885	Hängselled sängbotten SB750	4
6	320850	Tubular bow for leg section length 183 cm ø32	1
7	320516.60	Insticksrör SB750	2
8	320529-7035	Låsbeslag SB750 grå sängbotten/lyft st	2
9	320882-1015	Lifter arm for thigh section 85 and 90 cm width	1
10	405479-7035	Nylonrulle d 40/17x38	4
11	420127	Rörstycke rördel SB750	2
13	420623	Bult m8x3.4	2
14	420625-7035	Plastbussning styrarmar SB750 grå	4
15	420649-7035	Fixeringsrör styrarmar SB750 grå	2
16	421458-1015	Distance tube for lifter arm 85 and 90 cm width	1
17	812875	Skruv mc6s m8x12 fzb hfc 9729	6
18	815764	Starlock dia 16 SB750	4
19	819407	Skruv självgående 5.5x19 SB750	4
20	821235	Screw ø5x12 thread form ing	2
21	1423915	Doppsko 20x20 SB750	4
22	1484059	Lamell 85/90cm SB750 70cm	7
12	420163-0005	Madrass stopp fotände SB750	1

(Top frame, foot end, 200 cm length)

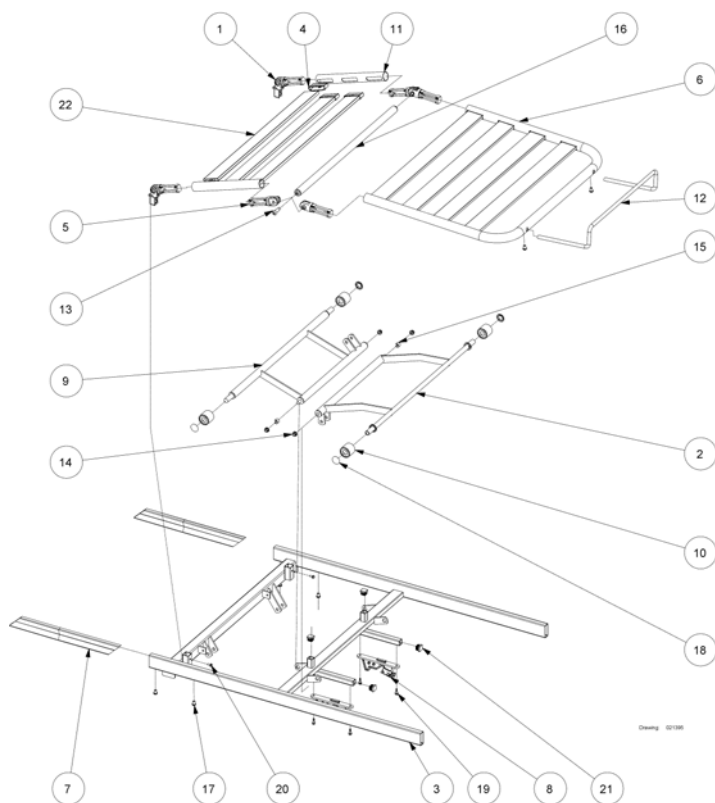


1	021351	Hängselled SB750	2
2	221155-1015	Lifter arm for leg section 85 and 90 cm width	1
3	221162-1015	Top frame foot end 90x200 cm	1
4	314884-7035	Beslag sänglatta SB750 ljusgrå	14
5	314885	Hängselled sängbotten SB750	4
6	319788	Rörbygel bendel SB750	1
7	320516.60	Insticksrör SB750	2
8	320529-7035	Låsbeslag SB750 grå sängbotten/lyft st	2
9	320882-1015	Lifter arm for thigh section 85 and 90 cm width	1
10	405479-7035	Nylonrulle d 40/17x38	4
11	420127	Rörstycke rördel SB750	2
13	420623	Bult m8x3.4	2
14	420625-7035	Plastbussning styrarmar SB750 grå	4
15	420649-7035	Fixeringsrör styrarmar SB750 grå	2
16	421458-1015	Distance tube for lifter arm 85 and 90 cm width	1
17	812875	Skruv mc6s m8x12 fzb hfc 9729	6
18	815764	Starlock dia 16 SB750	4
19	819407	Skruv självgående 5.5x19 SB750	4
20	821235	Screw ø5x12 thread form ing	2
21	1423915	Doppsko 20x20 SB750	4
22	1484059	Lamell 85/90cm SB750 70cm	7
12	420163-0005	Madrass stopp fotände SB750	1


(Mattress support 105 cm width with steel lamellas)

-	(Top frame, head end, 200 cm length)	1
-	(Top frame, foot end, 200 cm length)	1

(Top frame, head end, 200 cm length)

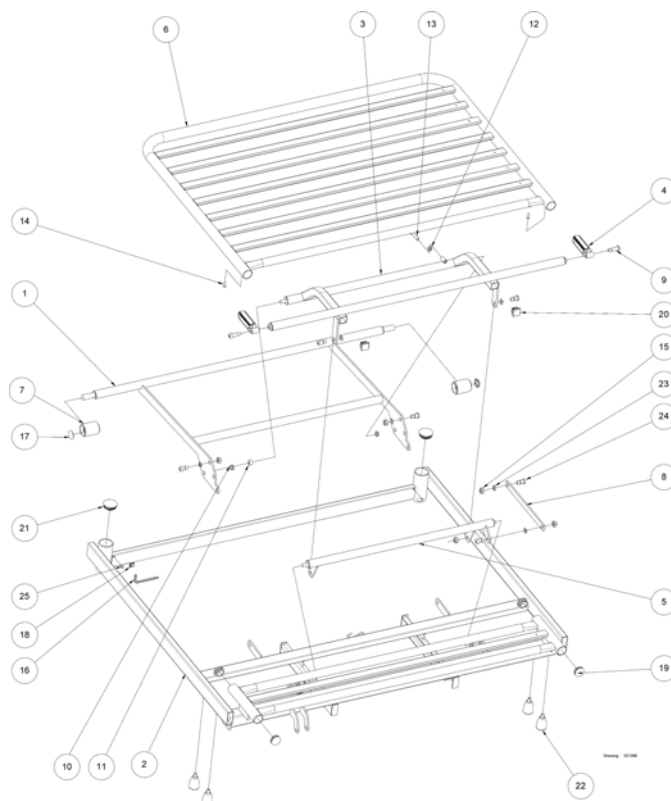


1	221127-1015	Lifter arm for backrest 105 cm width	1
2	221133-1015	Top frame head end 105 cm width	1
3	221124-1015	Steering arm head end 105 cm width	1
4	319800	Hängselled zink SB750	2
5	320847-1015	Paralellarm SB750 105 cm	1
6	320877-1015	Ryggdel SB750 105 cm	1
7	405479-7035	Nylonrulle d 40/17x38	2
8	420620-1015	Fläns SB750	1
9	420623	Bult m8x3.4	2
10	420625-7035	Plastbussning styrarmar SB750 grå	4
11	420649-7035	Fixeringsrör styrarmar SB750 grå	2
12	802729	Mutter m8 fzb	1
13	803008	Skruv m8x40 till SB750 lyftarm	1
14	803811	Popnit tlp/d/bs 665 lf 48x16 hfc 1460	2
15	806651	Låsmutter m6m 10loc-king fzb	6
16	811408	Sexkantnyckel nv 5mm hfc 753 din 911	1
17	815764	Starlock dia 16 SB750	2
18	816488	Kabelclips 46x41 hfc 2540	1
19	1423911	Doppsko sfl32	4
20	1423912-7035	Doppsko SB750	2
21	1432700-7035	Doppsko	2
22	1435325-7035	Handgrepp/vred sängbotten SB750/SB200	4
23	1446475	Fibre washer ø13xø19x05 red	6
24	1493130	Bult ryggdel SB750 ingår i 1435195	6
25	1538319	(Plastic cap Ø8)	2

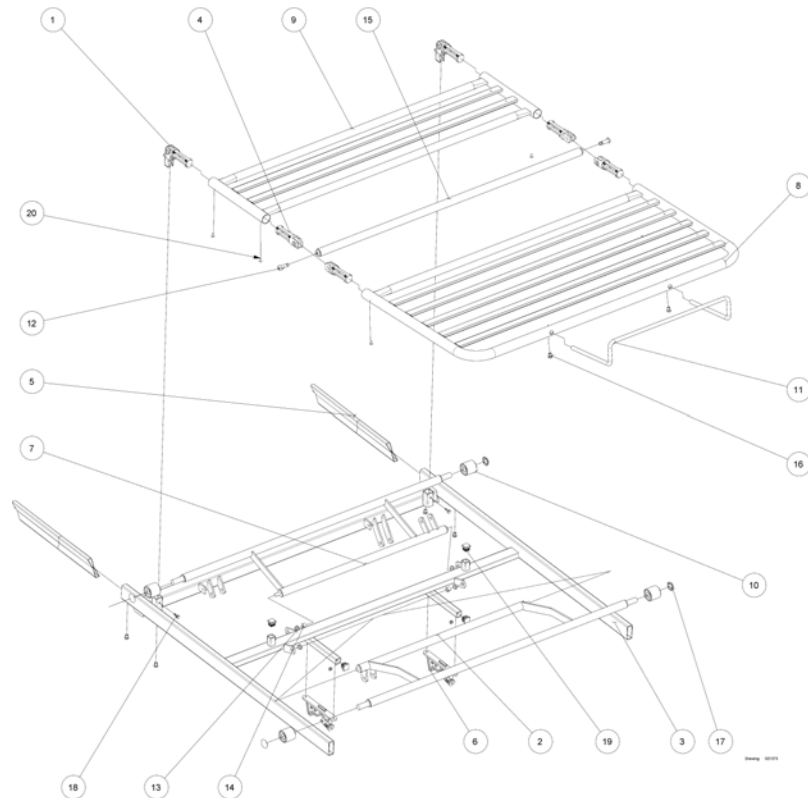(Top frame, foot end, 200 cm length)

1	1530100	Hängselled kpl säte/lårdel SB750 105/120 cm	2
2	221126-1015	Lifter arm for leg section 105 cm width	1
3	221132-1015	Top frame foot end 105 cm width	1
4	1529664	Hängselled förstärkt 105/120 cm	4
5	320516.60	Insticksrör SB750	2
6	320529-7035	Låsbeslag SB750 grå sängbotten/lyft st	2
7	320846-1015	Lifter arm for thigh section 105 cm width	1
8	320875-1015	Leg mattress section 105 cm width	1
9	320876-1015	Thigh mattress section 105 cm width	1
10	405479-7035	Nylonrulle d 40/17x38	4
11	420163-0005	Madrass stopp fotände SB750	1
12	420623	Bult m8x3.4	2
13	420625-7035	Plastbussning styrarmar SB750 grå	4
14	420649-7035	Fixeringsrör styrarmar SB750 grå	2
15	420975-1015	Distance tube for lifter arm 105 cm width	1
16	812875	Skruv mc6s m8x12 fzb hfc 9729	6
17	815764	Starlock dia 16 SB750	4
18	819407	Skruv självgående 5.5x19 SB750	2
19	1423915	Doppsko 20x20 SB750	4
20	803811	Popnit tlp/d/bs 665 lf 48x16 hfc 1460	1


(Mattress support 120 cm width with steel lamellas)

-	(Top frame, head end, 200 cm length)	1
-	(Top frame, foot end, 200 cm length)	1

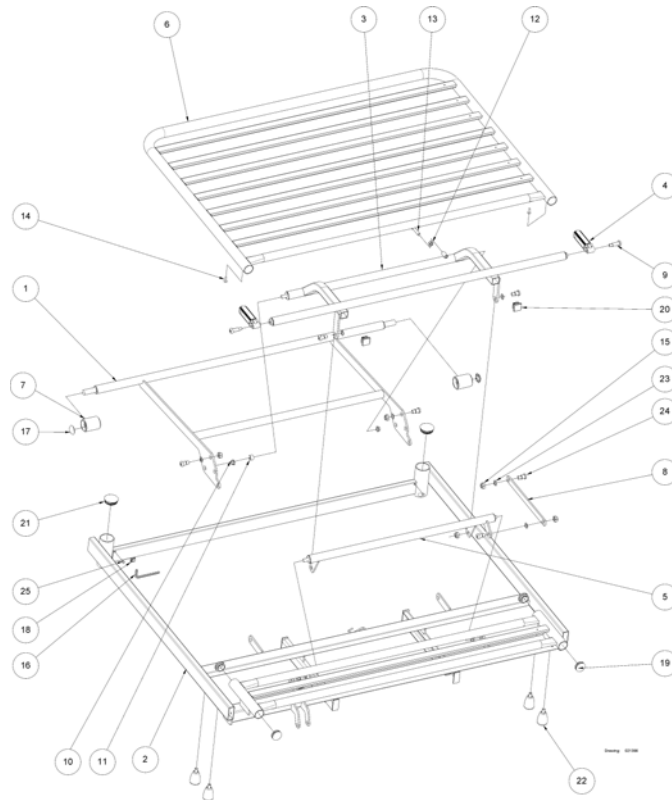
(Top frame, head end, 200 cm length)

1	221139-1015	Lifter arm for backrest 120 cm width	1
2	221142-1015	Top frame head end 120 cm width	1
3	221143-1015	Steering arm head end 120 cm width	1
4	319800	Hängselled zink SB750	2
5	320847-1015	Paralellarm SB750 105 cm	1
6	320892-1015	Backrest 120 cm width	1
7	405479-7035	Nylonrulle d 40/17x38	2
8	420620-1015	Fläns SB750	1
9	420623	Bult m8x3.4	2
10	420625-7035	Plastbussning styrarmar SB750 grå	4
11	420649-7035	Fixeringsrör styrarmar SB750 grå	2
12	802729	Mutter m8 fzb	1
13	803008	Skruv m8x40 till SB750 lyftarm	1
14	803811	Popnit tlp/d/bs 665 lf 48x16 hfc 1460	2
15	806651	Låsmutter m6m 10loc-king fzb	6
16	811408	Sexkantnyckel nv 5mm hfc 753 din 911	1
17	815764	Starlock dia 16 SB750	2
18	816488	Kabelclips 46x41 hfc 2540	1
19	1423911	Doppsko sfl32	4
20	1423912-7035	Doppsko SB750	2
21	1432700-7035	Doppsko	2
22	1435325-7035	Handgrepp/vred sängbotten SB750/SB200	4
23	1446475	Fibre washer ø13xø19x05 red	6
24	1493130	Bult ryggedel SB750 ingår i 1435195	6

(Top frame, foot end, 200 cm length)

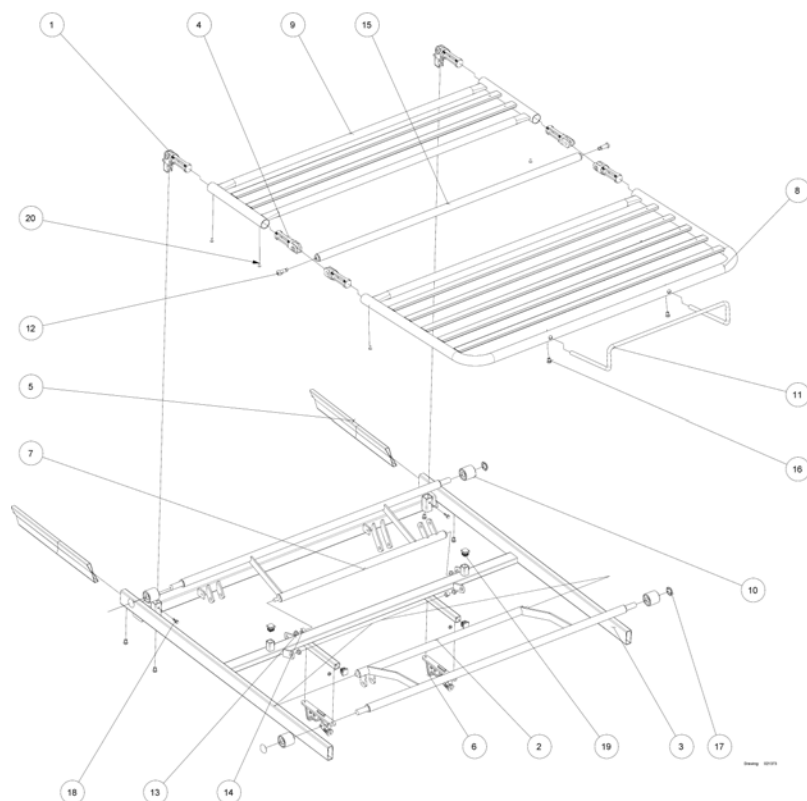


1	1530100	Hängselled kpl säte/lårdel SB750 105/120 cm	2
2	221140-1015	Lifter arm for leg section 120 cm width	1
3	221144-1015	Top frame foot end 105 cm width	1
4	1529664	Hängselled förstärkt 105/120 cm	1
5	320516.60	Insticksrör SB750	2
6	320529-7035	Låsbeslag SB750 grå sängbotten/lyft st	2
7	320868-1015	Lyftarm SB750 120 lårdel	1
8	321502-1015	Leg mattress section 120 cm width	1
9	321503-1015	Thigh mattress section 120 cm width	1
10	405479-7035	Nylonrulle d 40/17x38	4
11	420163-0005	Madrass stopp fotände SB750	1
12	420623	Bult m8x3.4	2
13	420625-7035	Plastbussning styrarmar SB750 grå	4
14	420649-7035	Fixeringsrör styrarmar SB750 grå	2
15	421430-1015	Distance tube for lifter arm 120 cm width	1
16	812875	Skruv mc6s m8x12 fzb hfc 9729	6
17	815764	Starlock dia 16 SB750	4
18	819407	Skruv självgående 5.5x19 SB750	2
19	1423915	Doppsko 20x20 SB750	4
20	803811	Popnit tlp/d/bs 665 lf 48x16 hfc 1460	1

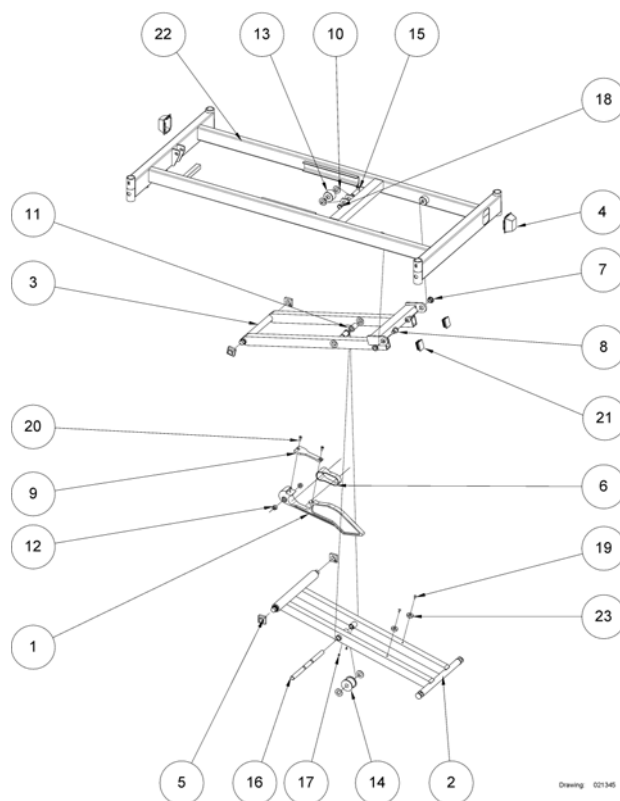


1	221103	Kil till underrede till SB700/750	1
2	221122-1015	Shear arm inner	1
3	221123-1015	Shear arm outer	1
4	315626-7035	Täcklock cb SB750/400 sänglyft	2
5	318648	Glidkuts sänglyft 700/710/750	4
6	318673	Glidlager til underrede /kil SB700/750	1
7	409645	Lagerbussning	2
8	418477	Bult/bröstbult lyftkryss SB700	2
9	418896	Kilhållare till underrede SB700	1
10	419298	Plastskiva SB750 sänglyft	4
11	420154	Nylonbussning SB750	2
12	420171	Bussning till lyftkryss och kil SB750	2
13	421442	Stödrolle nedre SB750	1
14	421443	Stödrolle övre till säng SB750	1
15	421452	Sprint m krage u-rede SB750	1
16	421455	Axle ø16x250	1
17	803221	Skruv m6x6 till SB750	2
18	803247	Segerring SB750 sänglyft	1
19	803833	Popnit SB700/750	2
20	816401	Skruv m6x16 till underrede till SB700	2
21	1423916	Doppsko sänglyft SB750	2
22	1451305-1015	Base for SB750	1
23	1507221	Distance piece	2


(Base for 120 cm width)

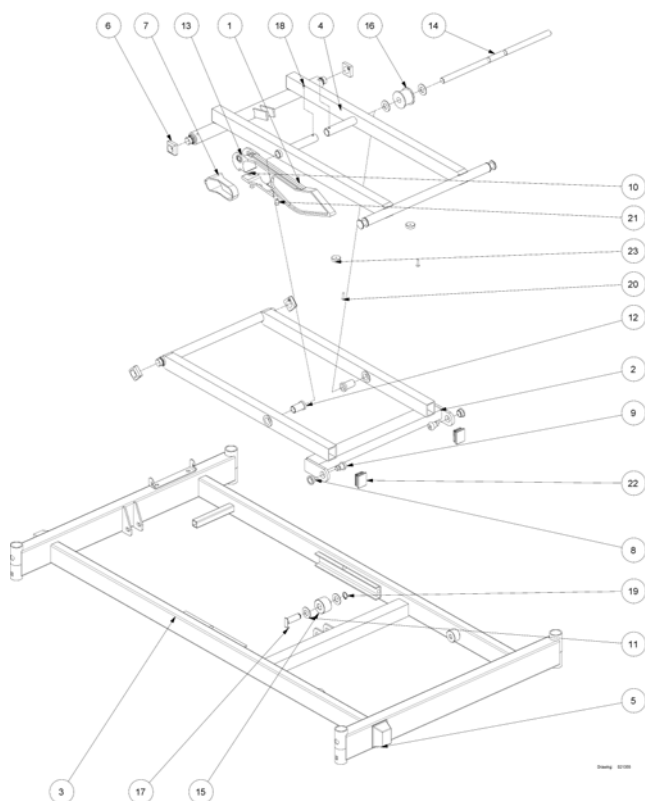

1	221103	Kil till underrede till SB700/750	1
2	221135-1015	Shear arm outer 120 cm	1
3	221136-1015	Base 120 cm width	1
4	221138-1015	Shear arm inner 120 cm	1
5	315626-7035	Täcklock cb SB750/400 sänglyft	2
6	318648	Glidkuts sänglyft 700/710/750	4
7	318673	Glidlager til underrede /kil SB700/750	1
8	409645	Lagerbussning	2
9	418477	Bult/bröstbult lyftkryss SB700	2
10	418896	Kilhållare till underrede SB700	1
11	419298	Plastskiva SB750 sänglyft	4
12	420154	Nylonbussning SB750	2
13	420171	Bussning till lyftkryss och kil SB750	2
14	421422	Axle ø16x400 120 cm wid th	1
15	421442	Stödrulle nedre SB750	1
16	421443	Stödrulle övre till säng SB750	1
17	421452	Sprint m krage u-rede SB750	1
18	803221	Skruv m6x6 till SB750	2
19	803247	Segerring SB750 sänglyft	1
20	803835	Popnit	2
21	816401	Skruv m6x16 till underrede till SB700	2
22	1423916	Doppsko sänglyft SB750	2
23	1507221	Distance piece	2

(Castors with brake)

	80	1
	100	2
mm	125	3

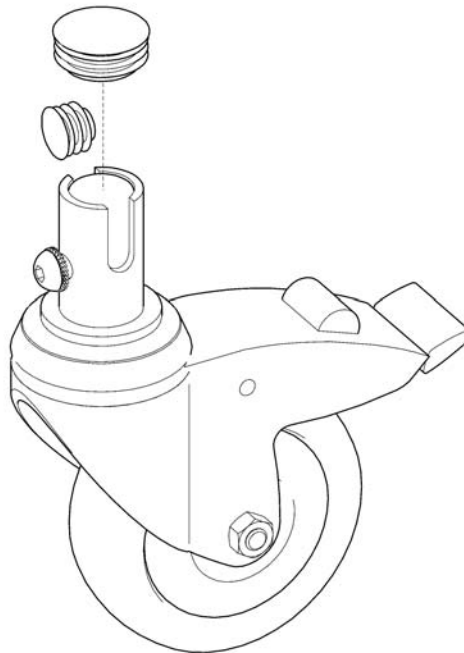


SP1441285

1	019398.D0	Hjulsats 80 mm kpl för eftermont från cb	1
2	SP1441285	(CASTOR 100MM, WITH BRAKE)	1
3	SP015111	Hjul 125mm SB750/SB755 sep lås komplett	1

(Central brake system for 85+90+105 cm width)


-	(Ø80 mm castors with central brake)	1
-	(Ø100 mm castors with central brake)	1
-	(Ø125 mm castors with central brake)	1
-	(Ø150 mm castors with central brake)	1

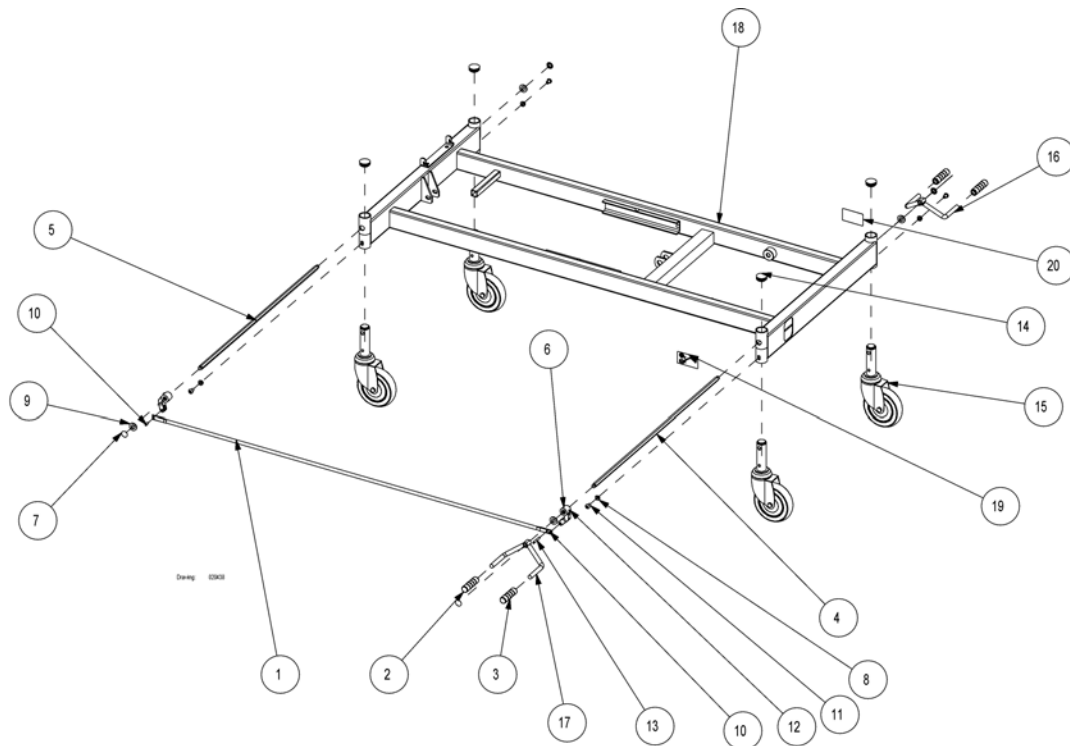(Ø80 mm castors with central brake)

Dwg. 0303


18	-	(Base for 85+90+105 cm width)	1
1	314284-1015	Central brake rod Ø10x15x1369 mm	1
2	414411	Pedalgummi röd c-lås ec kraton-g dia 11.5/20*70	2
3	414412	Pedalgummi grön c-lås ec kraton-g dia 11.5/20*70	2
4	414652.60	Axel nv 11x740	1
5	414653.60	Axel till centralbroms SB750	1
6	414654	Medbringare	2
7	803279	Starlock 10x19 Bedco låsbricka 10x19	4
8	803876	Fjäderbricka	4
9	804239	Nylonbricka 24x13x25 SB750	4
10	806644	Spiralspännstift 5x14	2
11	812875	Skruv mc6s m8x12 fzb hfc 9729	2
12	817243	Stoppskruv	2
13	821213	Spiralspännstift	1
14	1423911	Doppsko sfl32	4
15	1446312	Hjul 80 mm cb SB750	4
16	1435279-1015	Bromspedal SB750 höger	1
17	1435280-1015	Bromspedal SB750 vänster	1
19	1492781	Label for brake 40x70 left	1
20	1492782	Label for brake 40x70 right	1

(Ø100 mm castors with central brake)

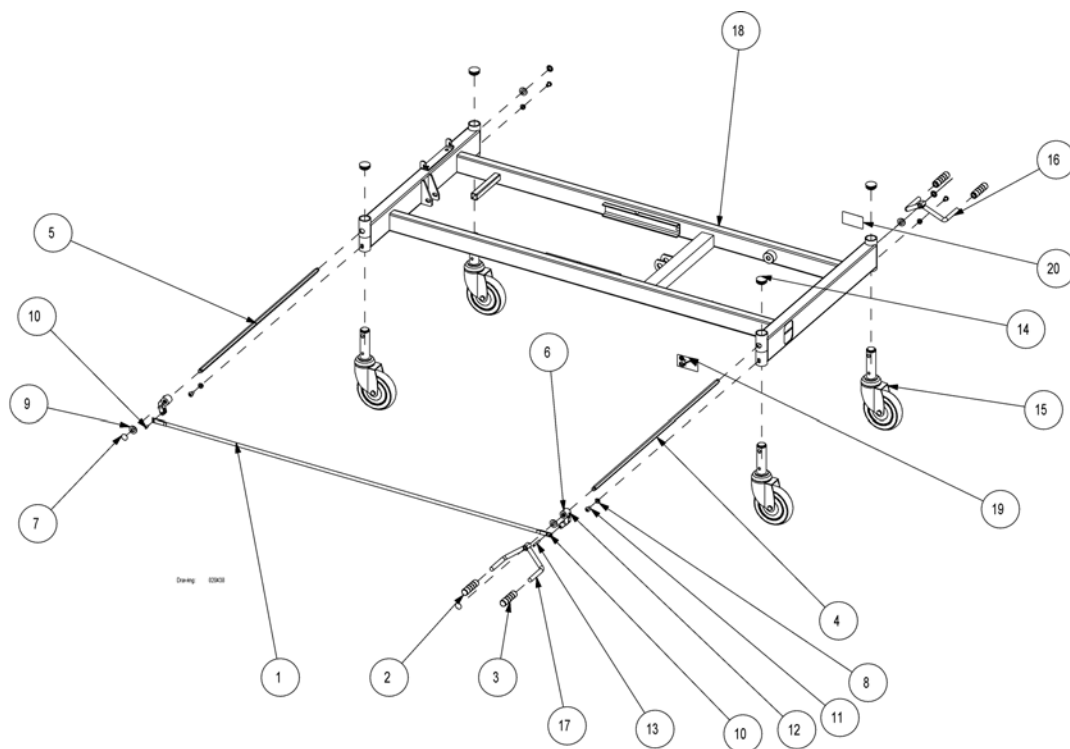

Dwg: 0303


18	-	(Base for 85+90+105 cm width)	1
1	314284-1015	Central brake rod Ø10x15x1369 mm	1
2	414411	Pedalgummi röd c-lås ec kraton-g dia 11.5/20*70	2
3	414412	Pedalgummi grön c-lås ec kraton-g dia 11.5/20*70	2
4	414652.60	Axel nv 11x740	1
5	414653.60	Axel till centralbroms SB750	1
6	414654	Medbringare	2
7	803279	Starlock 10x19 Bedco låsbricka 10x19	4
8	803876	Fjäderbricka	4
9	804239	Nylonbricka 24x13x25 SB750	4
10	806644	Spiralspännstift 5x14	2
11	812875	Skruv mc6s m8x12 fzb hfc 9729	2
12	817243	Stoppskruv	2
13	821213	Spiralspännstift	1
14	1423911	Doppsko sfl32	4
15	819460	Hjul 100 mm cb SB700/750 utan riktningspär	4
16	1435279-1015	Bromspedal SB750 höger	1
17	1435280-1015	Bromspedal SB750 vänster	1
19	1492781	Label for brake 40x70 left	1
20	1492782	Label for brake 40x70 right	1

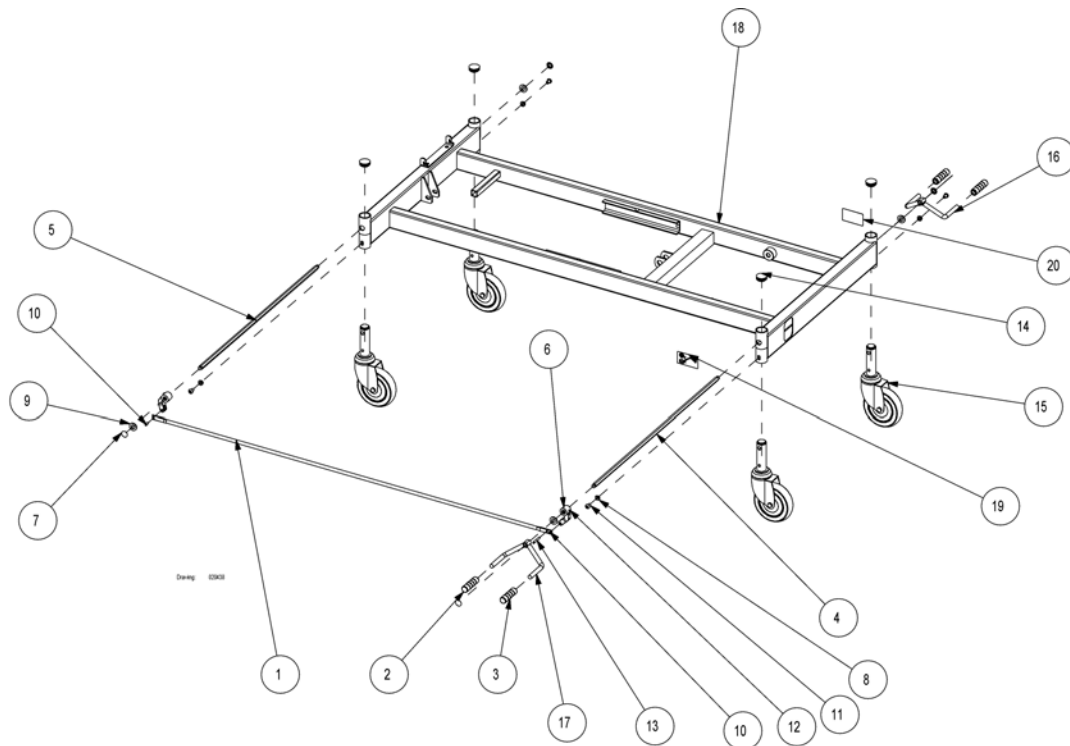(Ø125 mm castors with central brake)

Dwg. 0303

18	-	(Base for 85+90+105 cm width)	1
1	314284-1015	Central brake rod Ø10x15x1369 mm	1
2	414411	Pedalgummi röd c-lås ec kraton-g dia 11.5/20*70	2
3	414412	Pedalgummi grön c-lås ec kraton-g dia 11.5/20*70	2
4	414652.60	Axel nv 11x740	1
5	414653.60	Axel till centralbroms SB750	1
6	414654	Medbringare	2
7	803279	Starlock 10x19 Bedco låsbricka 10x19	4
8	803876	Fjäderbricka	4
9	804239	Nylonbricka 24x13x25 SB750	4
10	806644	Spiralspännstift 5x14	2
11	812875	Skruv mc6s m8x12 fzb hfc 9729	2
12	817243	Stoppskruv	2
13	821213	Spiralspännstift	1
14	1423911	Doppsko sfl32	4
15	1423937	Hjul 125mm cb SB750/400 utan riktningspär	4
16	1435279-1015	Bromspedal SB750 höger	1
17	1435280-1015	Bromspedal SB750 vänster	1
19	1492781	Label for brake 40x70 left	1
20	1492782	Label for brake 40x70 right	1

(Ø150 mm castors with central brake)

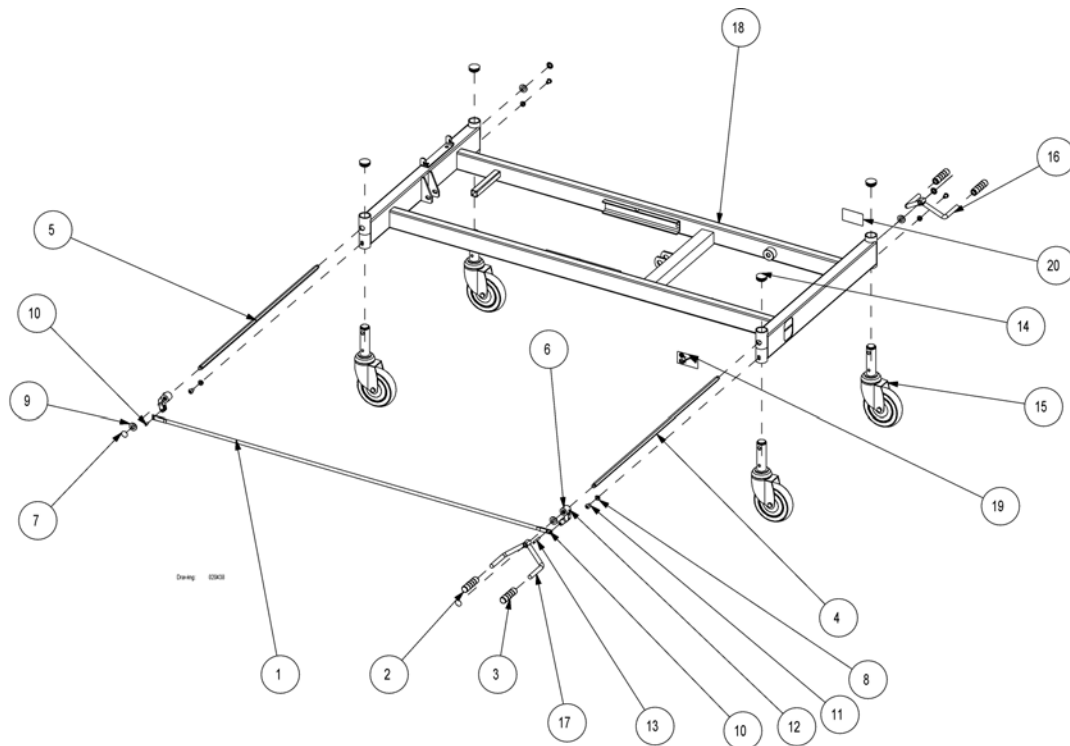


Drawing: 0303

18	-	(Base for 85+90+105 cm width)	1
1	314284-1015	Central brake rod Ø10x15x1369 mm	1
2	414411	Pedalgummi röd c-lås ec kraton-g dia 11.5/20*70	2
3	414412	Pedalgummi grön c-lås ec kraton-g dia 11.5/20*70	2
4	414652.60	Axel nv 11x740	1
5	414653.60	Axel till centralbroms SB750	1
6	414654	Medbringare	2
7	803279	Starlock 10x19 Bedco låsbricka 10x19	4
8	803876	Fjäderbricka	4
9	804239	Nylonbricka 24x13x25 SB750	4
10	806644	Spiralspännstift 5x14	2
11	812875	Skruv mc6s m8x12 fzb hfc 9729	4
12	817243	Stoppskruv	2
13	821213	Spiralspännstift	1
14	1423911	Doppsko sfl32	4
15	1423988	Hjul 150mm cb till SB750	4
16	1435279-1015	Bromspedal SB750 höger	1
17	1435280-1015	Bromspedal SB750 vänster	1
19	1492781	Label for brake 40x70 left	1
20	1492782	Label for brake 40x70 right	1


(Central brake system with dir. control for 85+90+105 cm width)

-	(Ø100 mm castors with central brake and dir. control)	1
-	(Ø125 mm castors with central brake and dir. control)	1
-	(Ø150 mm castors with central brake and dir. control)	1

(Ø100 mm castors with central brake and dir. control)

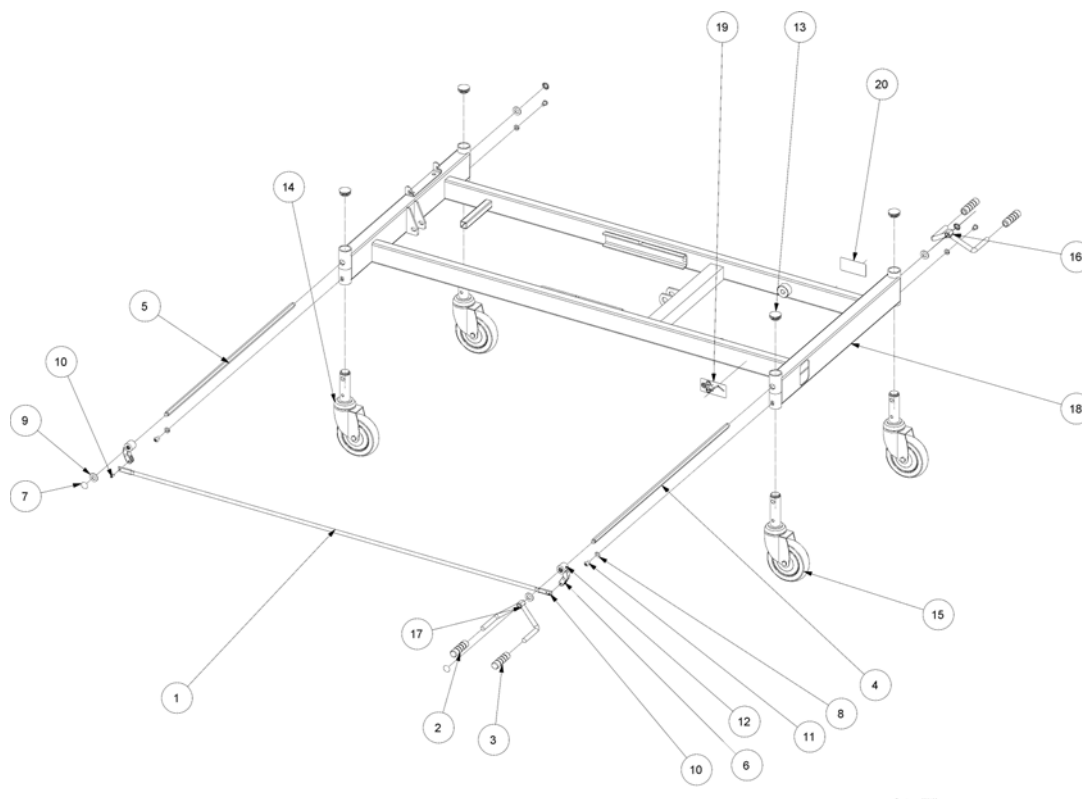


18	-	(Base for 85+90+105 cm width)	1
1	314284-1015	Central brake rod Ø10x15x1369 mm	1
2	414411	Pedalgummi röd c-lås ec kraton-g dia 11.5/20*70	2
3	414412	Pedalgummi grön c-lås ec kraton-g dia 11.5/20*70	2
4	414652.60	Axel nv 11x740	1
5	414653.60	Axel till centralbroms SB750	1
6	414654	Medbringare	2
7	803279	Starlock 10x19 Bedco låsbricka 10x19	4
8	803876	Fjäderbricka	4
9	804239	Nylonbricka 24x13x25 SB750	4
10	806644	Spiralspännstift 5x14	2
11	812875	Skruv mc6s m8x12 fzb hfc 9729	4
12	817243	Stoppskruv	2
13	1423911	Doppsko sfl32	4
14	819460	Hjul 100 mm cb SB700/750 utan riktningsspärr	3
15	819459	Hjul 100mm cb SB700/ Staccata m riktningsspärr	1
16	1435279-1015	Bromspedal SB750 höger	1
17	1435280-1015	Bromspedal SB750 vänster	1
19	1484312	Label for brake and direction control 40x80 le	1
20	1484313	Label for brake and direction control 40x80 ri	1

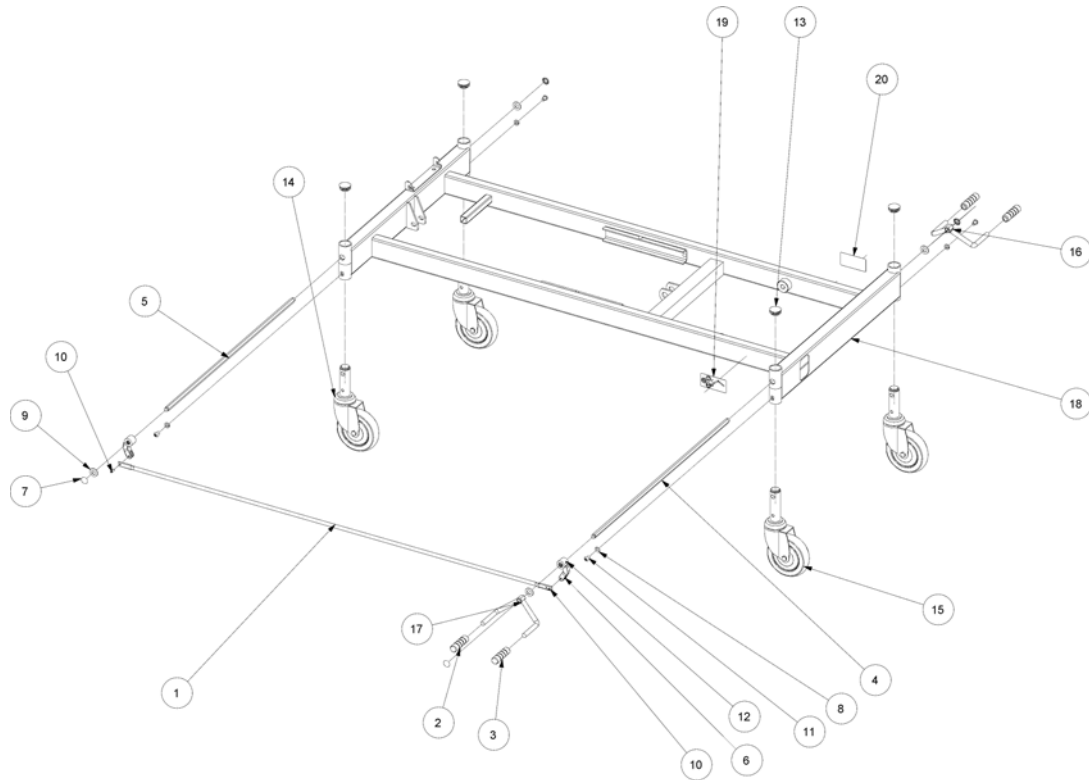(Ø125 mm castors with central brake and dir. control)

18	-	(Base for 85+90+105 cm width)	1
1	314284-1015	Central brake rod Ø10x15x1369 mm	1
2	414411	Pedalgummi röd c-lås ec kraton-g dia 11.5/20*70	2
3	414412	Pedalgummi grön c-lås ec kraton-g dia 11.5/20*70	2
4	414652.60	Axel nv 11x740	1
5	414653.60	Axel till centralbroms SB750	1
6	414654	Medbringare	2
7	803279	Starlock 10x19 Bedco låsbricka 10x19	4
8	803876	Fjäderbricka	4
9	804239	Nylonbricka 24x13x25 SB750	4
10	806644	Spiralspännstift 5x14	2
11	812875	Skruv mc6s m8x12 fzb hfc 9729	4
12	817243	Stoppskruv	2
13	1423911	Doppsko sfl32	4
14	1423937	Hjul 125mm cb SB750/400 utan riktningsspärr	3
15	1423938	Hjul till SB750/700 cb riktningsspärr 125mm	1
16	1435279-1015	Bromspedal SB750 höger	1
17	1435280-1015	Bromspedal SB750 vänster	1
19	1484312	Label for brake and direction control 40x80 le	1
20	1484313	Label for brake and direction control 40x80 ri	1

(Ø150 mm castors with central brake and dir. control)

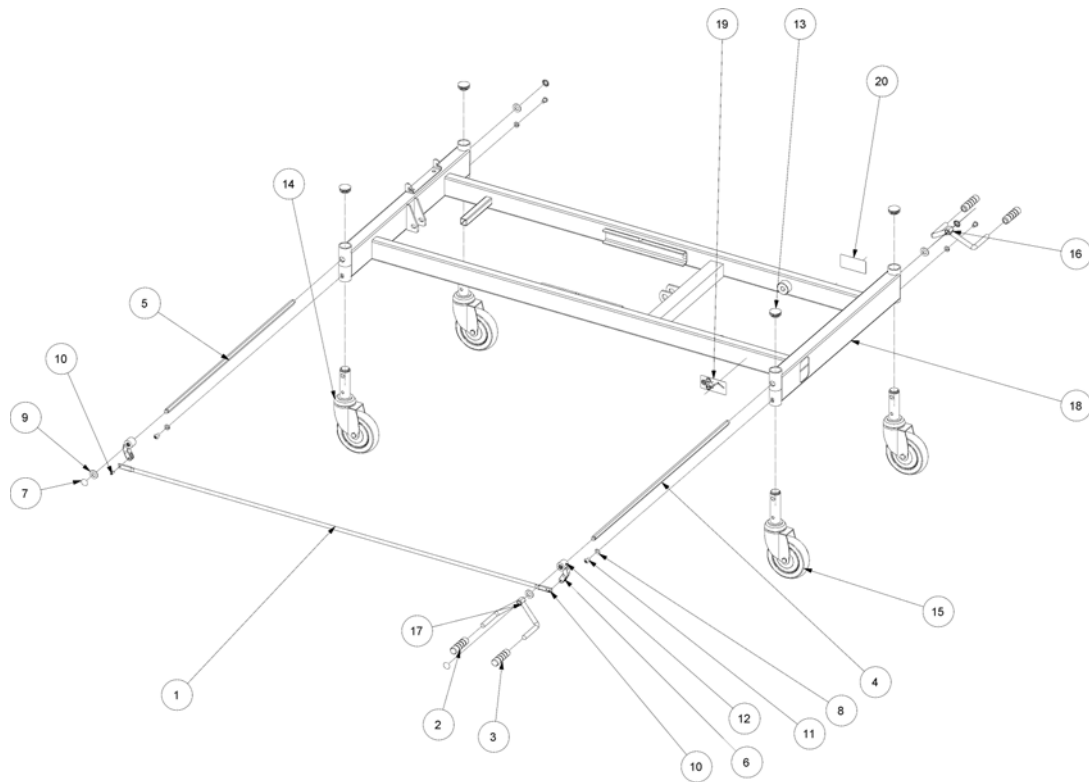

18	-	(Base for 85+90+105 cm width)	1
1	314284-1015	Central brake rod Ø10x15x1369 mm	1
2	414411	Pedalgummi röd c-lås ec kraton-g dia 11.5/20*70	2
3	414412	Pedalgummi grön c-lås ec kraton-g dia 11.5/20*70	2
4	414652.60	Axel nv 11x740	1
5	414653.60	Axel till centralbroms SB750	1
6	414654	Medbringare	2
7	803279	Starlock 10x19 Bedco låsbricka 10x19	4
8	803876	Fjäderbricka	4
9	804239	Nylonbricka 24x13x25 SB750	4
10	806644	Spiralspännstift 5x14	2
11	812875	Skruv mc6s m8x12 fzb hfc 9729	4
12	817243	Stoppskruv	2
13	1423911	Doppsko sfl32	4
14	1423988	Hjul 150mm cb till SB750	3
15	1423989	Castor ø150 tente with central brake and dir. c	1
16	1435279-1015	Bromspedal SB750 höger	1
17	1435280-1015	Bromspedal SB750 vänster	1
19	1484312	Label for brake and direction control 40x80 le	1
20	1484313	Label for brake and direction control 40x80 ri	1


(Central brake system with dir. control for 120 cm width)

-

(Ø125 mm castors with central brake and dir. control, 120 cm width)

1

(Ø125 mm castors with central brake and dir. control, 120 cm width)

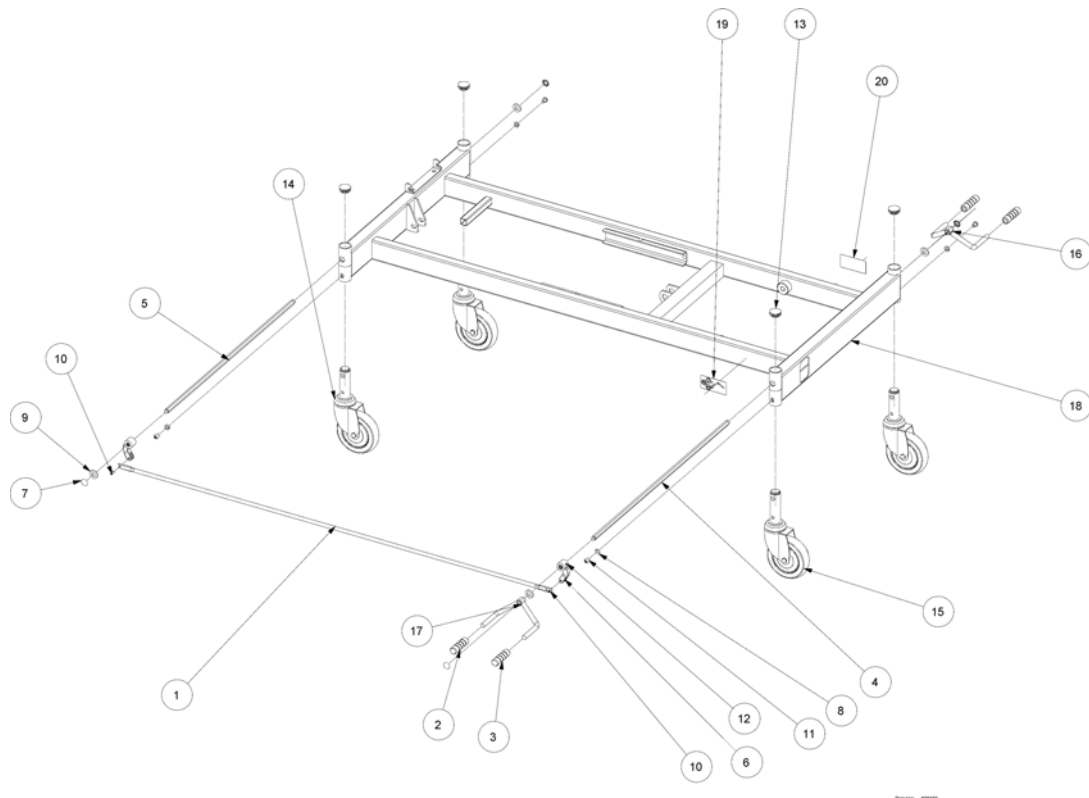

18	-	(Base for 120 cm width)	1
1	314284-1015	Central brake rod Ø10x15x1369 mm	1
2	414411	Pedalgummi röd c-lås ec kraton-g dia 11.5/20*70	2
3	414412	Pedalgummi grön c-lås ec kraton-g dia 11.5/20*70	2
4	421489.60	Axle nv 11x890 120 cm w idth	1
5	421488.60	Axle nv 11x850 120 cm w idth	1
6	414654	Medbringare	2
7	803279	Starlock 10x19 Bedco låsbricka 10x19	4
8	803876	Fjäderbricka	4
9	804239	Nylonbricka 24x13x25 SB750	4
10	806644	Spiralspännstift 5x14	2
11	812875	Skruv mc6s m8x12 fzb hfc 9729	4
12	817243	Stoppskruv	2
13	1423911	Doppsko sfl32	4
14	1423937	Hjul 125mm cb SB750/400 utan riktningsspärr	3
15	1423938	Hjul till SB750/700 cb riktningsspärr 125mm	1
16	1435279-1015	Bromspedal SB750 höger	1
17	1435280-1015	Bromspedal SB750 vänster	1
19	1484312	Label for brake and direction control 40x80 le	1
20	1484313	Label for brake and direction control 40x80 ri	1

(Lowering kit, 75mm castor)


-

(Lowering kit, Ø75 mm castor)

1

(Lowering kit, Ø75 mm castor)

1

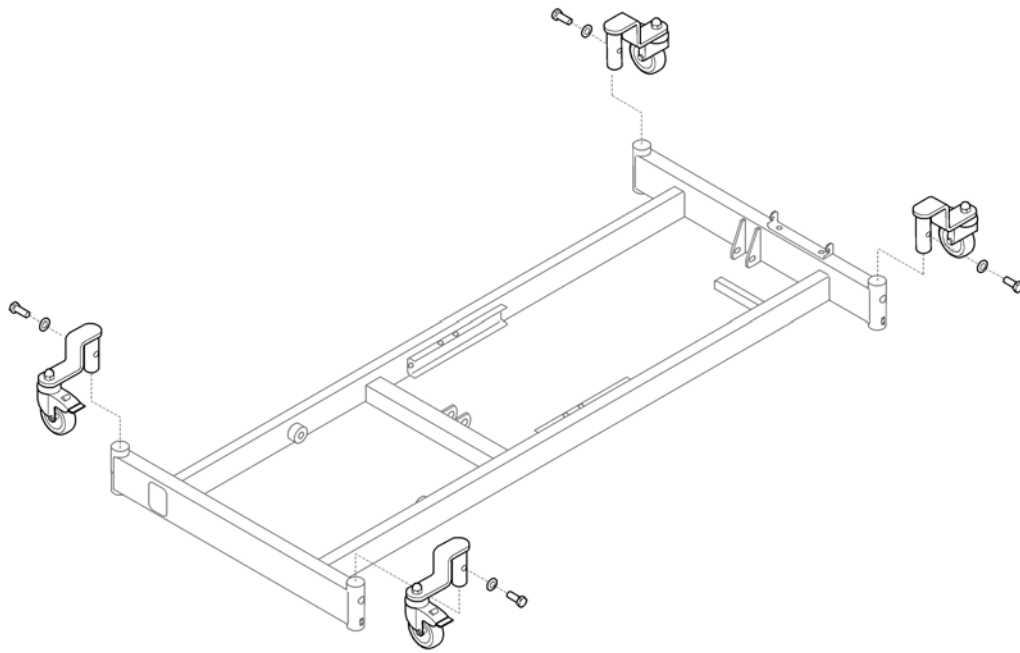

15P5021343

1 021343-0154


Lågbyggnadssats till säng SB755

1

(Invacare® SB750 =>2013.01.01) | (Bed ends and -plates)
(Bed end and -brackets, 85 cm width)

-	(Oda bed end plates, 85 cm width)	1
-	(Piggy bed end plates, 85 cm width)	1
-	(Karin bed end plates, 85 cm width)	1
-	(Sanne bed end plates, 85 cm width)	1
-	(Sanne bed end plates, 85 cm width, increased)	1
-	(Flex III bed end plates, 85 cm width)	1
-	(Oda and Piggy bed end brackets, 85 cm width)	1
-	(Sanne bed end brackets)	1
-	(Flex III bed end brackets)	1


(Oda bed end plates, 85 cm width)

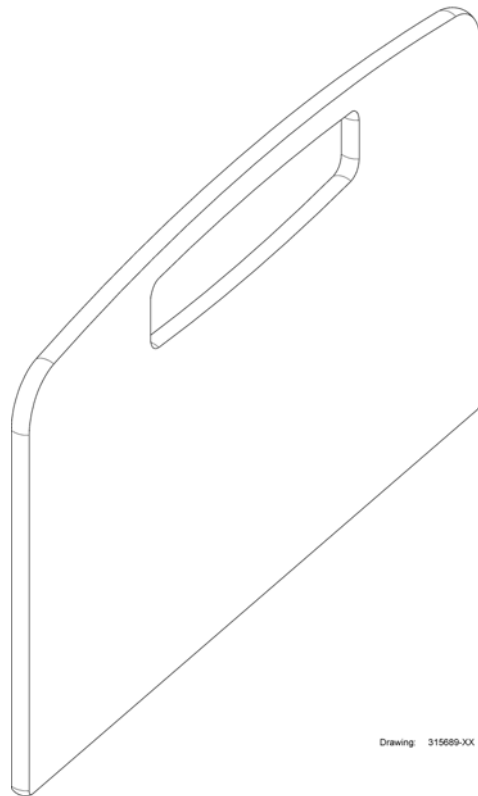

Drawing: 315669-XX


99	020430.01	Gavel Oda 85 cm par	1
99	020430.05	Oda bed end plate 85 cm width Flex III	1

(Invacare® SB750 =>2013.01.01) | (Bed ends and -plates)
(Piggy bed end plates, 85 cm width)

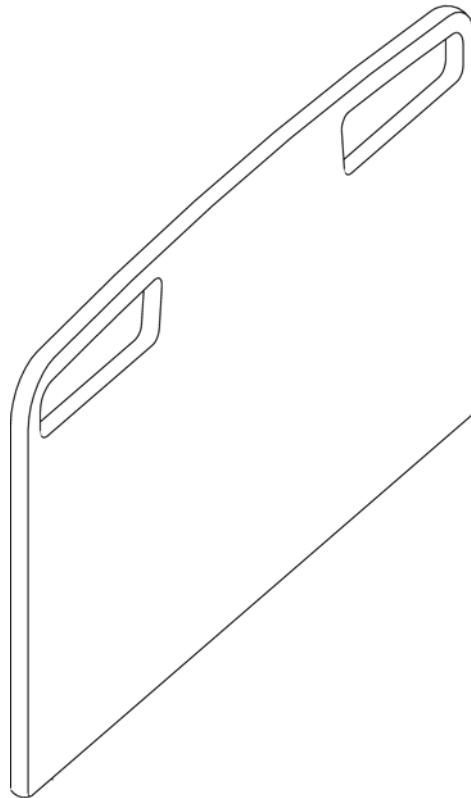


99	020431.01	Gavel Piggy 85cm till SB750 par	1
99	020431.05	Oda bed end plate 85 cm width complete	1

(Karin bed end plates, 85 cm width)

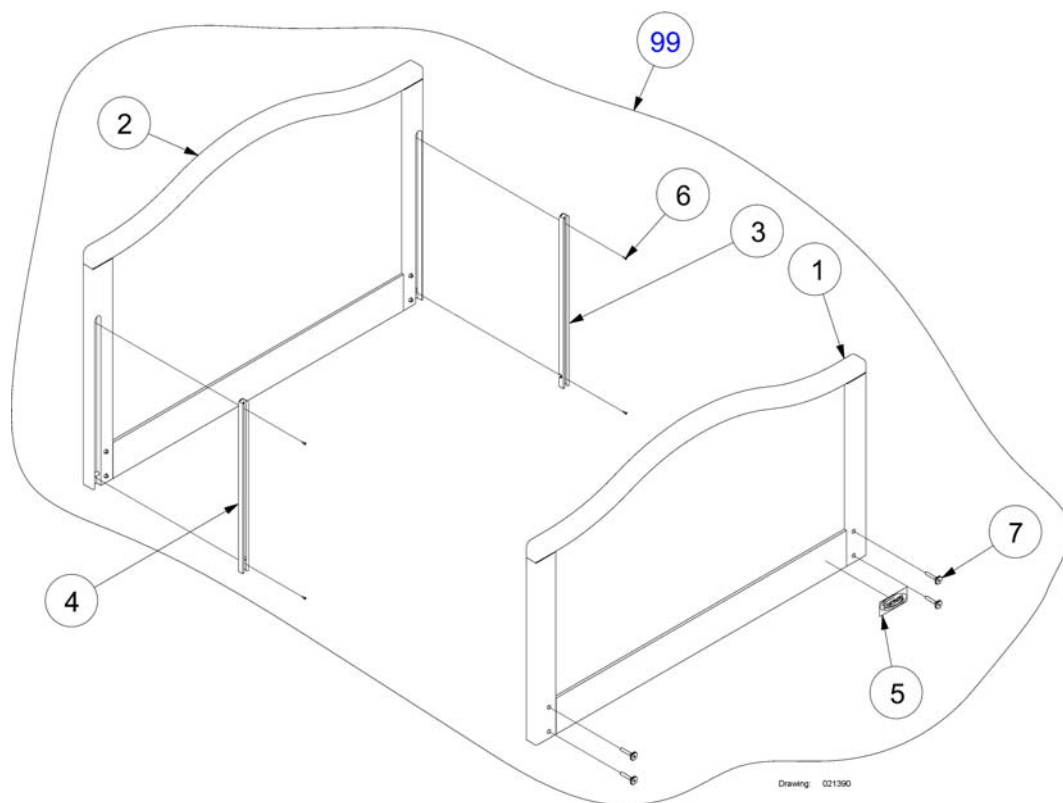


1	1505835	Not for separate sale - please contact Invacare	1
2	1505835	Not for separate sale - please contact Invacare	1
3	320504/L	Alu-skena till gavel Flex/Sanne vänster	2
4	320504/R	Alu-skena till gavel Flex/Sanne höger	2
5	417547	Label with Invacare logo	2
6	812847	Skruv till alu.skena Scanbedco 700	8
7	815720	Möbelskruv m6x40 till Flexgavel	8
99	021387.01	Sänggavel Karin hög/låg 85 cm bok natur	1

(Invacare® SB750 =>2013.01.01) | (Bed ends and -plates)
(Sanne bed end plates, 85 cm width)

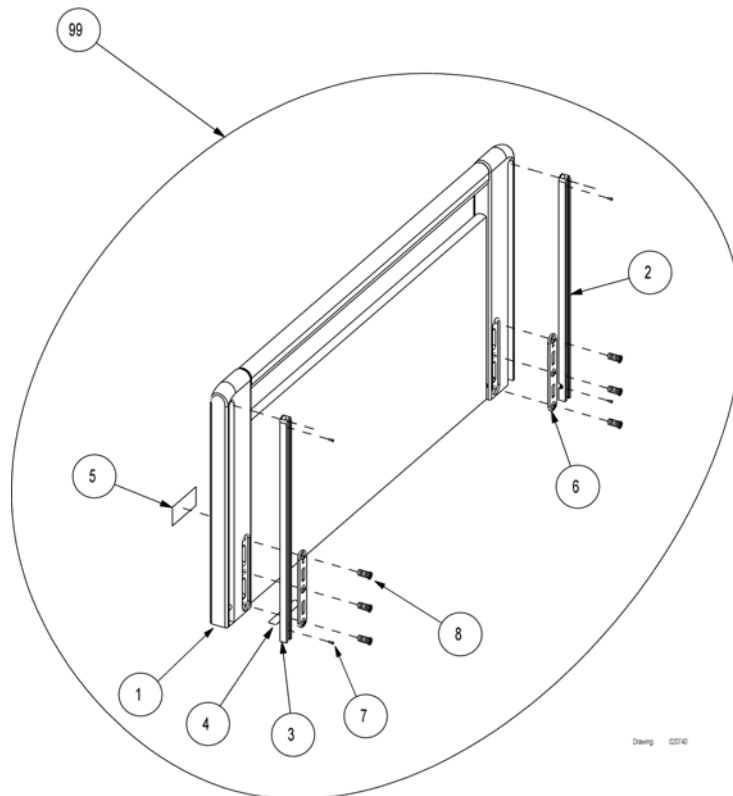


1	1505835	Not for separate sale - please contact Invacare	1
2	320504/L	Alu-skene till gavel Flex/Sanne vänster	4
3	320504/R	Alu-skene till gavel Flex/Sanne höger	4
4	1505835	Not for separate sale - please contact Invacare	1
5	417547	Label with Invacare logo	2
6	418441	Bracket with holes	4
7	812847	Skruv till alu.skene Scanbedco 700	8
8	818338	Skriv M8x24, special	12
99	020429.01	Sänggavel Sanne 85 cm bok natur	1

(Sanne bed end plates, 85 cm width, increased)

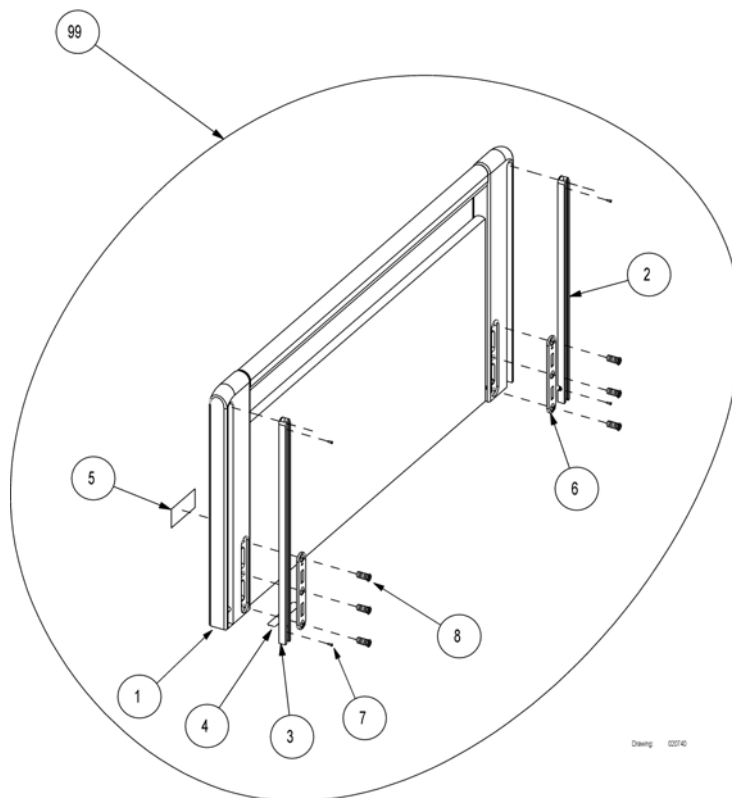


1	1505835	Not for separate sale - please contact Invacare	1
2	321519/L	Alu skena förhöjd till Flex/Sanne +5cm vänster	1
3	321519/R	Alu skena förhöjd till Flex/Sanne + 5cm höger	1
4	1505835	Not for separate sale - please contact Invacare	1
5	417547	Label with Invacare logo	1
6	418441	Bracket with holes	2
7	812847	Skruv till alu.skena Scanbedco 700	4
8	818338	Skrub M8x24, special	6
99	1451258-0101	Gavel Sanne III 400/750 hög 85 cm par	1

(Invacare® SB750 =>2013.01.01) | (Bed ends and -plates)
(Flex III bed end plates, 85 cm width)

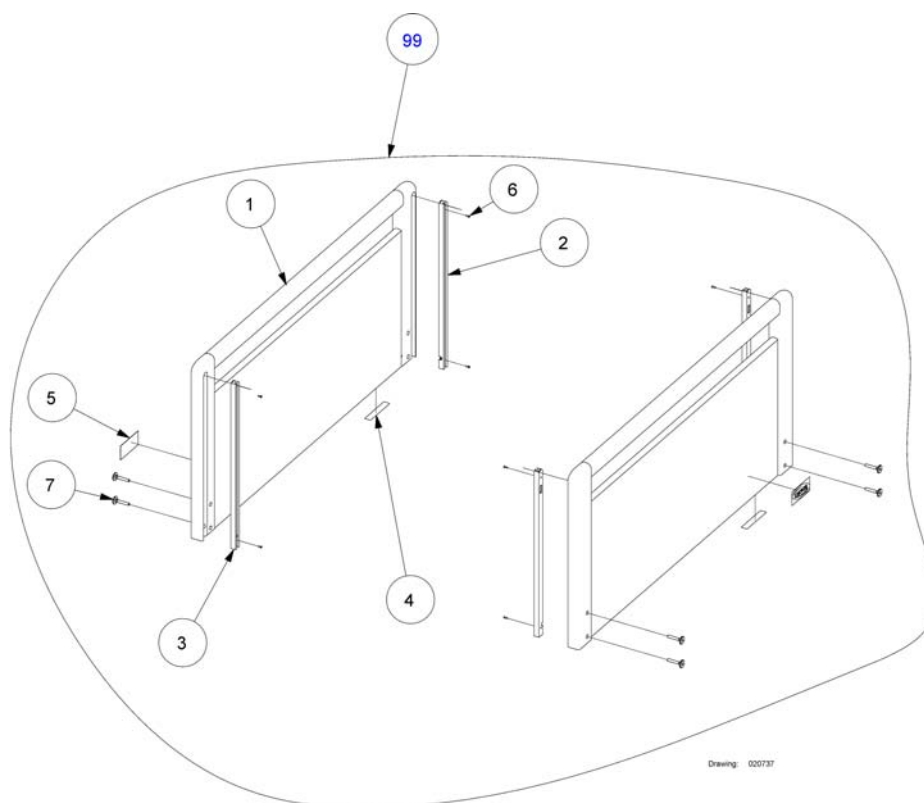


1	1505835	Not for separate sale - please contact Invacare	1
2	320504/L	Alu-skene till gavel Flex/Sanne vänster	2
3	320504/R	Alu-skene till gavel Flex/Sanne höger	2
4	1505835	Not for separate sale - please contact Invacare	1
5	417547	Label with Invacare logo	2
6	812847	Skruv till alu.skene Scanbedco 700	8
7	815720	Möbelskruv m6x40 till Flexgavel	8
99	020417.05	Sänggavel Flex 85 cm vitlaserad	1
99	020417.02	Gavel Flex III 85 cm körsbär par	1
99	020417.01	Sänggavel Flex 85 cm bok natur	1

(Oda and Piggy bed end brackets, 85 cm width)

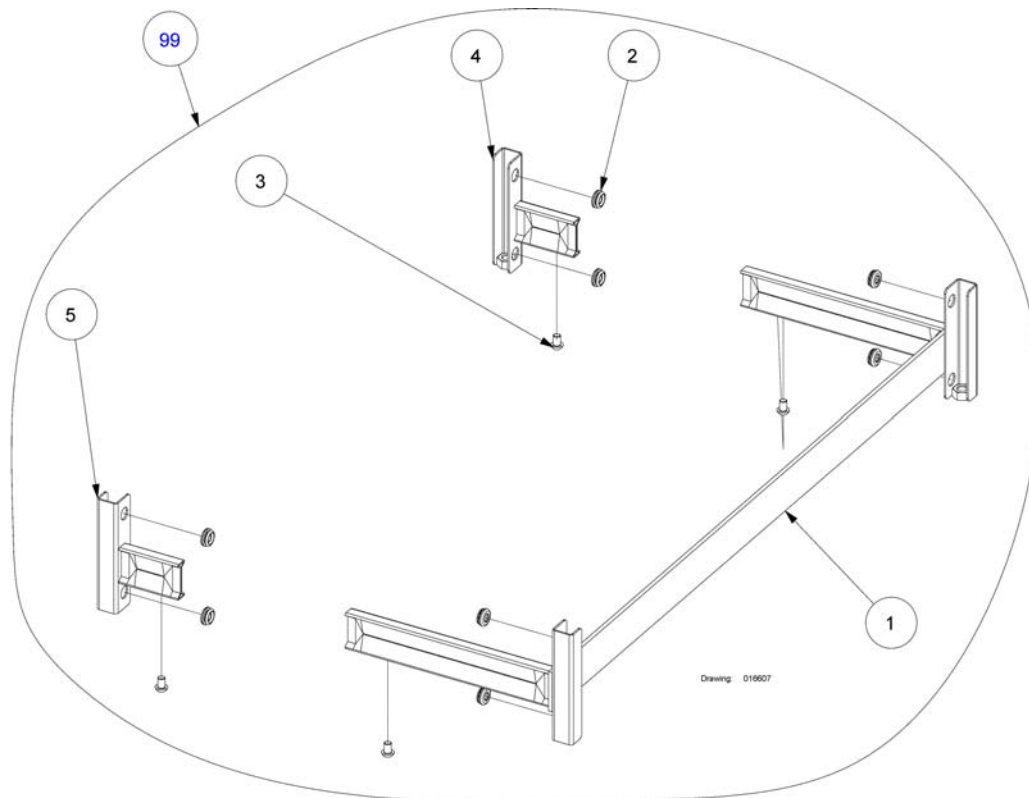

1	219569-1015	Bed end bracket with u-profile foot end	1
2	417008	Membrannippel dia 145x2	8
3	812875	Skruv mc6s m8x12 fzb hfc 9729	4
4	1492058-1015	Gavelbeslag m u-profil vä Piggy/Oda	1
5	1492059-1015	Gavelbeslag m u-profil hö Piggy/Oda	1
99	020421.D0	Gavelbeslag 85 cm med u-profil	1


(Sanne bed end brackets)

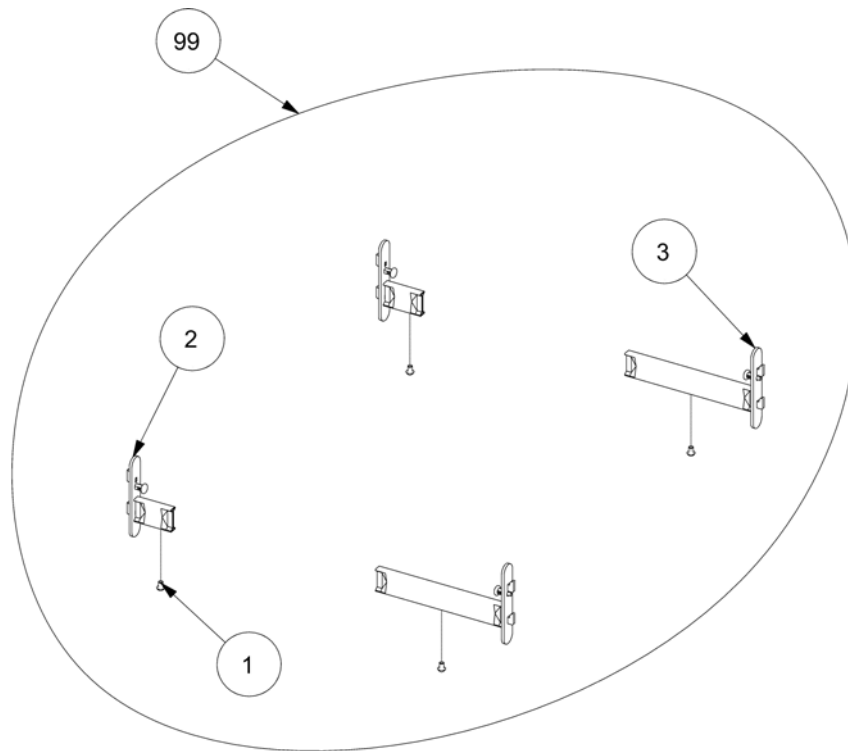

Drawing: 018742

1	812875	Skruv mc6s m8x12 fzb hfc 9729	4
2	1443879-0002	GAVELBESLAG HAKMODELL HUVUD (2 ST/SÄNG)	2
3	1443881-0002	Gavelbeslag Sanne Fot	2
99	018742.00	Gavelbeslag Sanne 85/90/105/120 cm SB750	1

(Flex III bed end brackets)

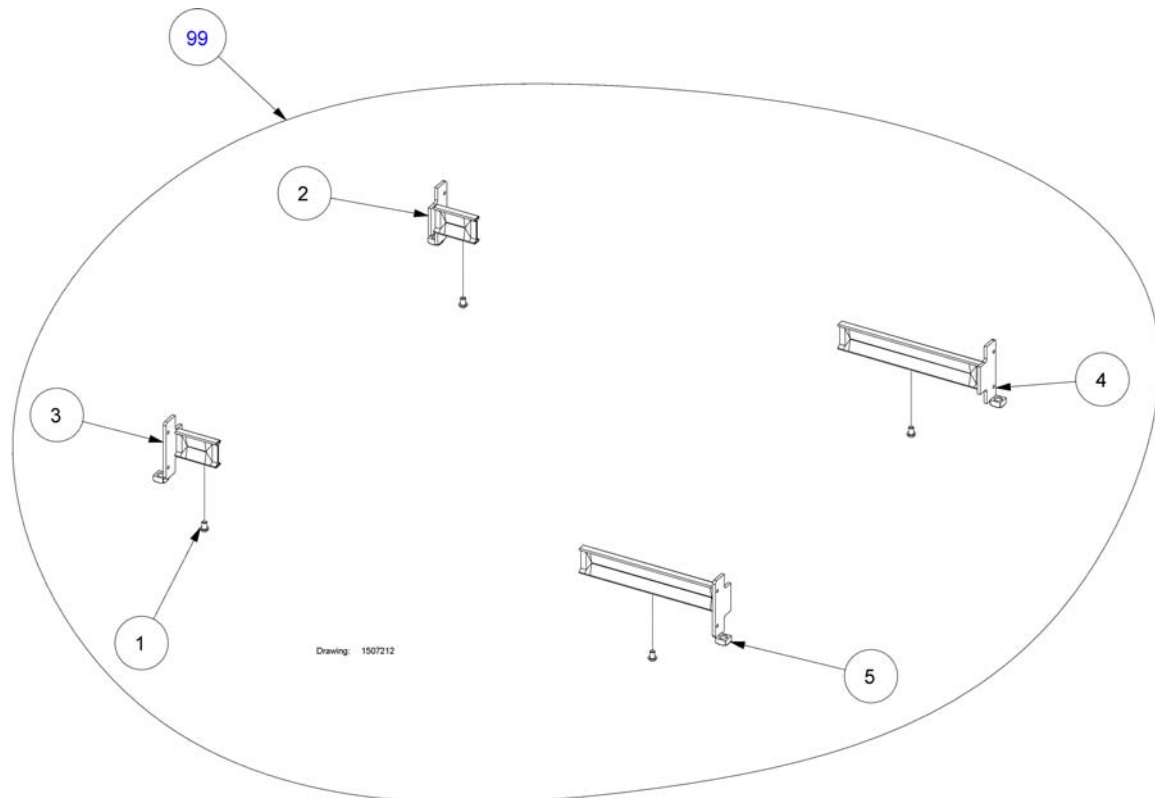


1	812875	Skruv mc6s m8x12 fzb hfc 9729	4
2	1488672-1015	Gavelbeslag Flex huvud vä SB750	1
3	1488673-1015	Gavelbeslag Flex huvud hö SB750	1
4	1507210-1015	Gavelbeslag SB750 fot vänster till Flex	1
5	1507211-1015	Gavelbeslag SB750 fot höger till Flex	1
99	1507212-1015	Gavelbeslag Flex 90cm huvud/fot SB750	1

(Invacare® SB750 =>2013.01.01) | (Bed ends and -plates)
(Bed end and -brackets, 90 cm width)


-	(Oda bed ends, 90 cm width)	1
-	(Piggy bed ends, 90 cm width)	1
-	(Karin bed end)	1
-	(Sanne bed end plates)	1
-	(Sanne bed end plates, 90 cm width, increased)	1
-	(Flex III bed end plates)	1
-	(Fittings with U-profile for Oda and Piggy)	1
-	(Sanne bed end brackets)	1
-	(Flex III bed end brackets)	1

(Oda bed ends, 90 cm width)

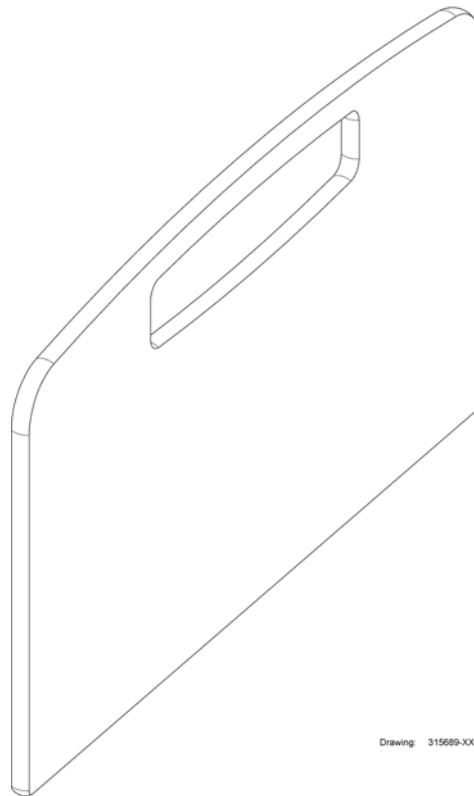


99	016609.01	Gavel Oda 85 cm till SB700 85cm alt SB750 90	1
99	016609.05	Oda bed end plate 90 cm width	1

(Piggy bed ends, 90 cm width)

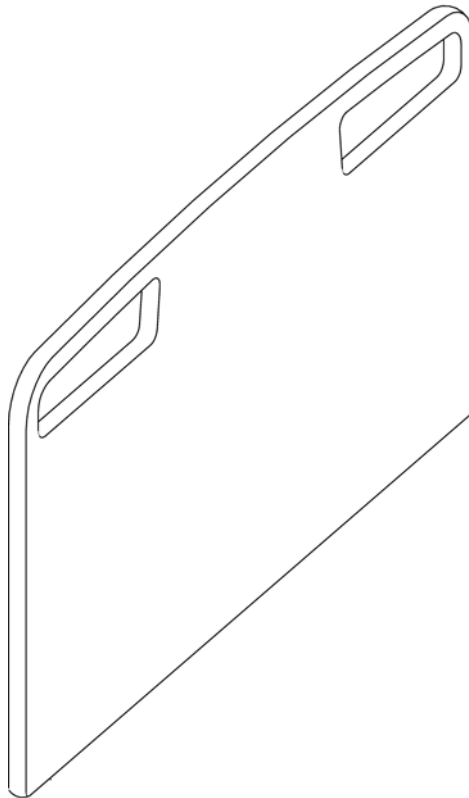


99	018940.01	Gavel Piggy 85 cm till Scanbedco par	1
99	018940.02	Piggy bed end plate 90 cm width	1
99	018940.05	Sänggavel Piggy 90 cm vitlaserad	1

(Karin bed end)

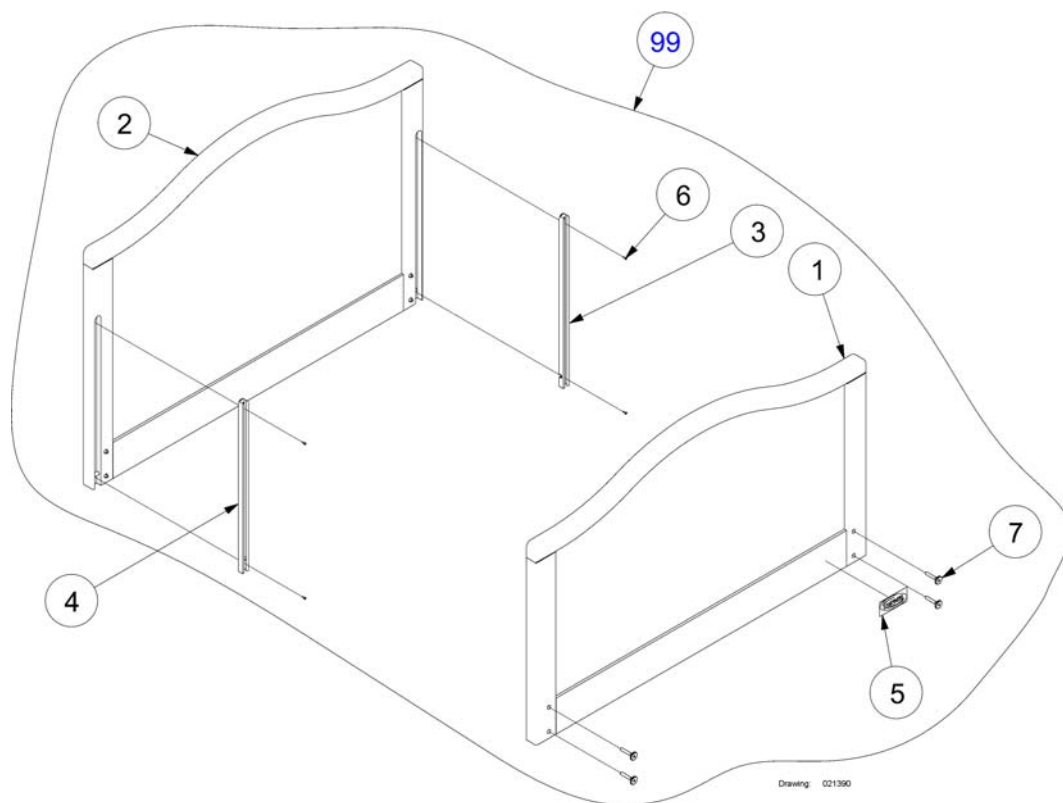


1	1505835	Not for separate sale - please contact Invacare	1
2	1505835	Not for separate sale - please contact Invacare	1
3	320504/L	Alu-skene till gavel Flex/Sanne vänster	2
4	320504/R	Alu-skene till gavel Flex/Sanne höger	2
5	417547	Label with Invacare logo	2
6	812847	Skruv till alu.skene Scanbedco 700	8
7	815720	Möbelskruv m6x40 till Flexgavel	8
99	021390.02	Karin hög/låg körsbär 90cm par	1
99	021390.01	Sänggavel Karin hög/låg 90 cm bok natur	1

(Sanne bed end plates)

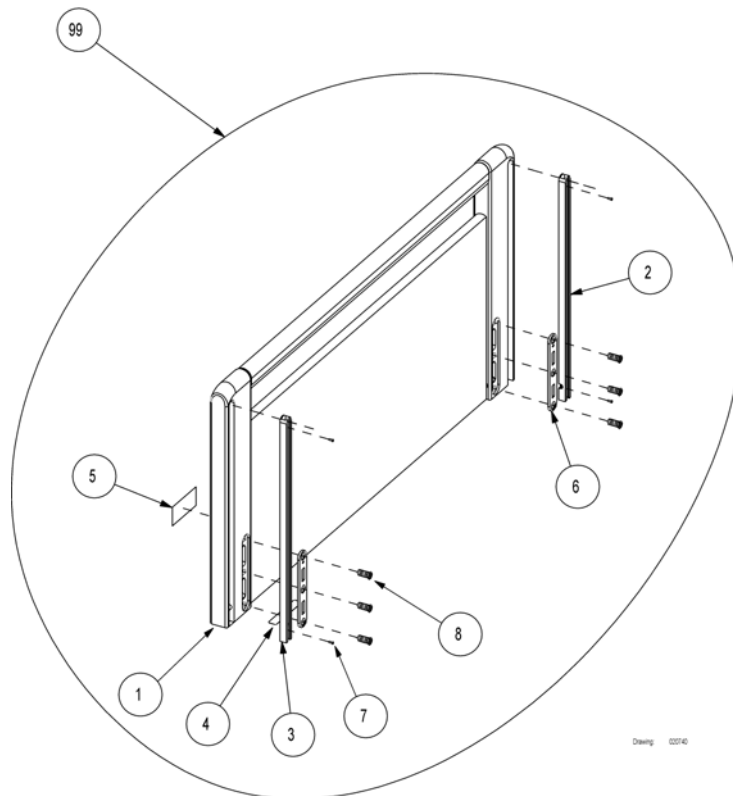


1	1505835	Not for separate sale - please contact Invacare	1
2	320504/L	Alu-skene till gavel Flex/Sanne vänster	2
3	320504/R	Alu-skene till gavel Flex/Sanne höger	2
4	1505835	Not for separate sale - please contact Invacare	1
5	417547	Label with Invacare logo	2
6	418441	Bracket with holes	4
7	812847	Skruv till alu.skene Scanbedco 700	8
8	818338	Skriv M8x24, special	12
99	020740.01	Sänggavel Sanne 90 cm bok natur	1

(Sanne bed end plates, 90 cm width, increased)

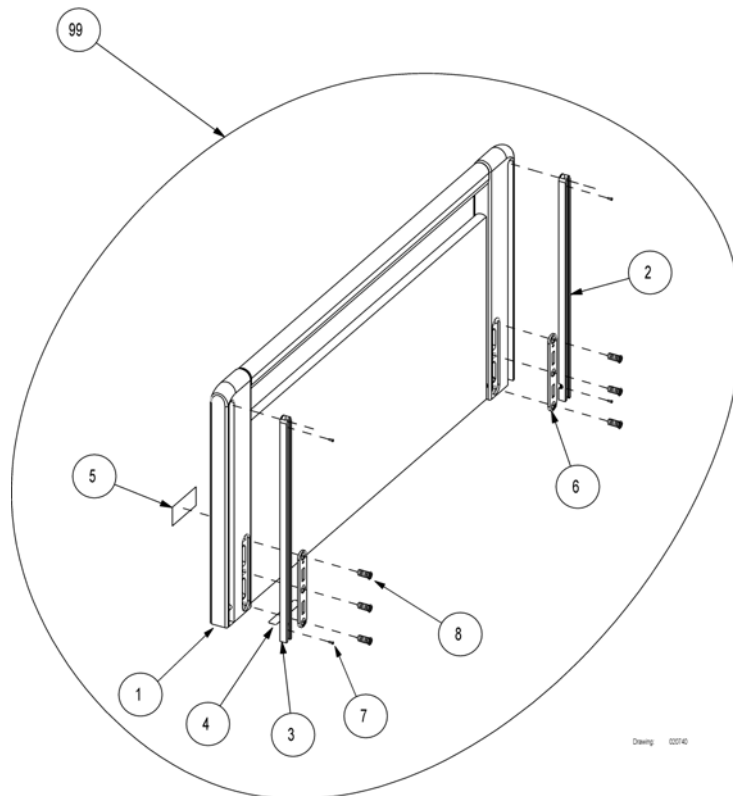


1	1505835	Not for separate sale - please contact Invacare	1
2	321519/L	Alu skena förhöjd till Flex/Sanne +5cm vänster	1
3	321519/R	Alu skena förhöjd till Flex/Sanne + 5cm höger	1
4	1505835	Not for separate sale - please contact Invacare	1
5	417547	Label with Invacare logo	1
6	418441	Bracket with holes	2
7	812847	Skruv till alu.skena Scanbedco 700	4
8	818338	Skrub M8x24, special	6
99	1451257-0101	Gavel Sanne III 400/750 hög 90 cm par	1

(Flex III bed end plates)

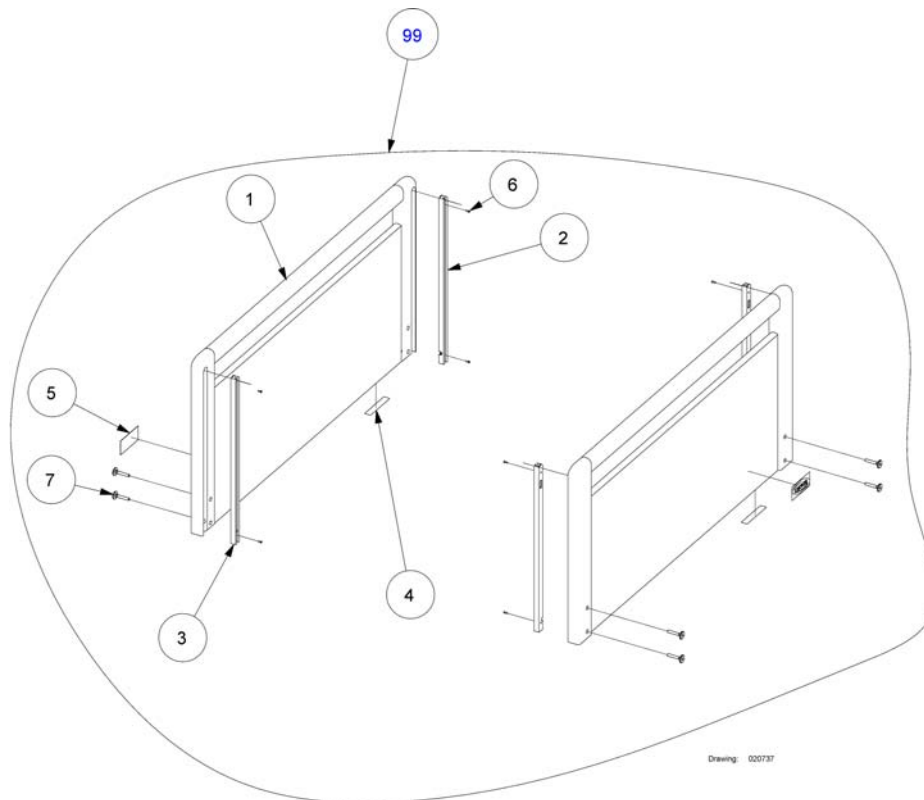

1	1505835	Not for separate sale - please contact Invacare	1
2	320504/L	Alu-skene till gavel Flex/Sanne vänster	2
3	320504/R	Alu-skene till gavel Flex/Sanne höger	2
4	1505835	Not for separate sale - please contact Invacare	2
5	417547	Label with Invacare logo	2
6	812847	Skruv till alu.skene Scanbedco 700	8
7	815720	Möbelskruv m6x40 till Flexgavel	8
99	020737.05	Sänggavel Flex 90 cm vitlaserad	1
99	020737.02	Gavel Flex III 90cm par körsbär	1
99	020737.01	Sänggavel Flex 90 cm bok natur	1

(Fittings with U-profile for Oda and Piggy)

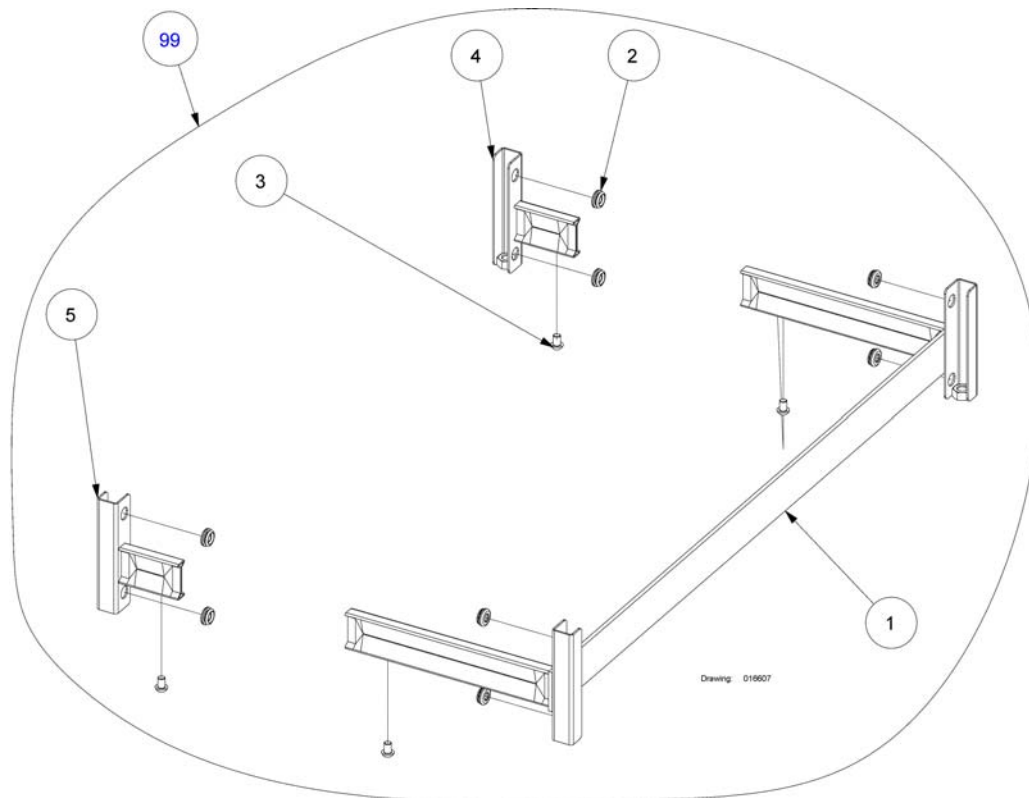


1	215961-1015	Bed end bracket u-profile 90 cm	1
2	417008	Membrannippel dia 145x2	8
3	812875	Skruv mc6s m8x12 fzb hfc 9729	4
4	1492058-1015	Gavelbeslag m u-profil vä Piggy/Oda	1
5	1492059-1015	Gavelbeslag m u-profil hö Piggy/Oda	1
99	016607.D0	Gavelbeslag med u-profil 85 cm Scanbedco par	1


(Invacare® SB750 =>2013.01.01) | (Bed ends and -plates)
(Bed end and -brackets, 105 cm width)

-	(Piggy bed end plates, 105 cm width)	1
-	(Sanne bed end plates, 105 cm width)	1
-	(Sanne bed end plates, 105 cm width, increased)	1
-	(Oda and Piggy bed end brackets, 105 cm width)	1
-	(Sanne bed end brackets)	1
-	(Flex III bed end brackets)	1

(Piggy bed end plates, 105 cm width)

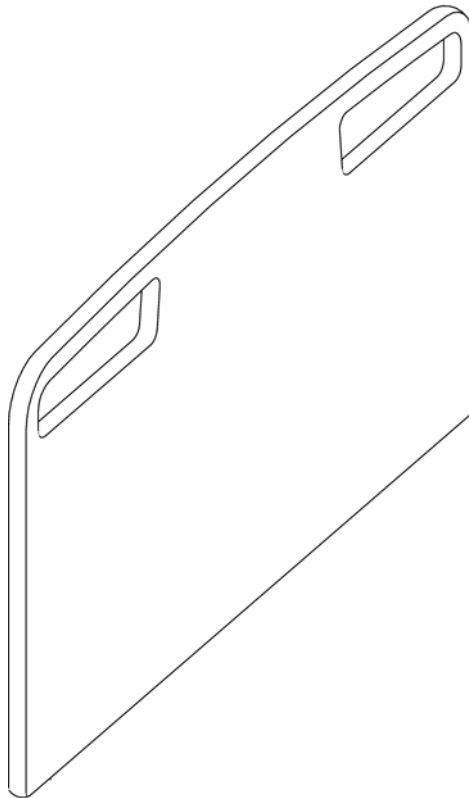


(Invacare® SB750 =>2013.01.01) | (Bed ends and -plates)
(Sanne bed end plates, 105 cm width)

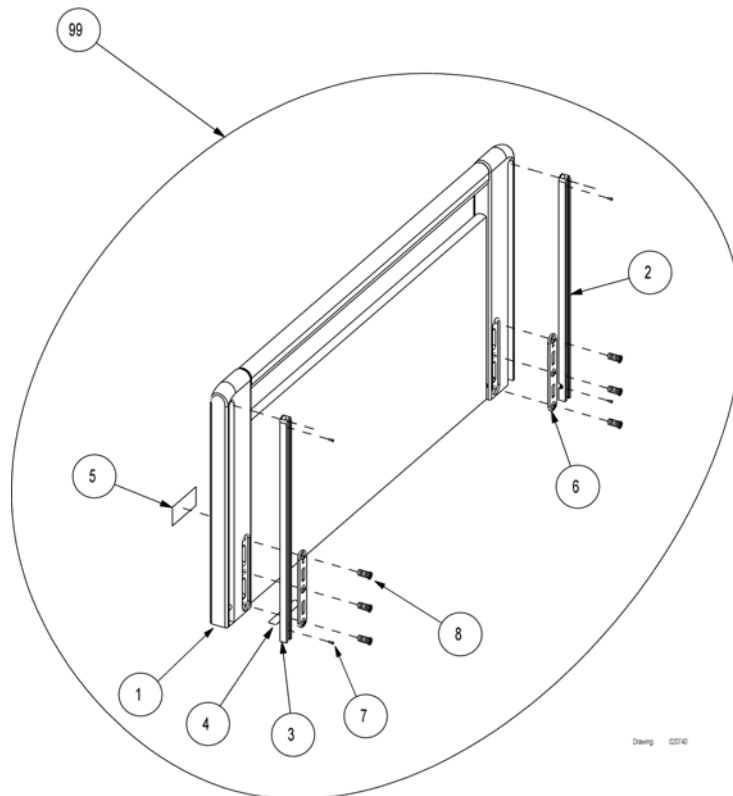


1	1505835	Not for separate sale - please contact Invacare	1
2	320504/L	Alu-skene till gavel Flex/Sanne vänster	2
3	320504/R	Alu-skene till gavel Flex/Sanne höger	2
4	1505835	Not for separate sale - please contact Invacare	1
5	417547	Label with Invacare logo	2
6	418441	Bracket with holes	4
7	812847	Skruv till alu.skene Scanbedco 700	8
8	818338	Screw M8x24, special	12
99	021953.07	Sänggavel Sanne 105 cm mix körsbär/bok natur	1
99	021953.02	Sänggavel Sanne 105 cm körsbär	1
99	021953.01	Sänggavel Sanne 105 cm bok natur	1

(Sanne bed end plates, 105 cm width, increased)

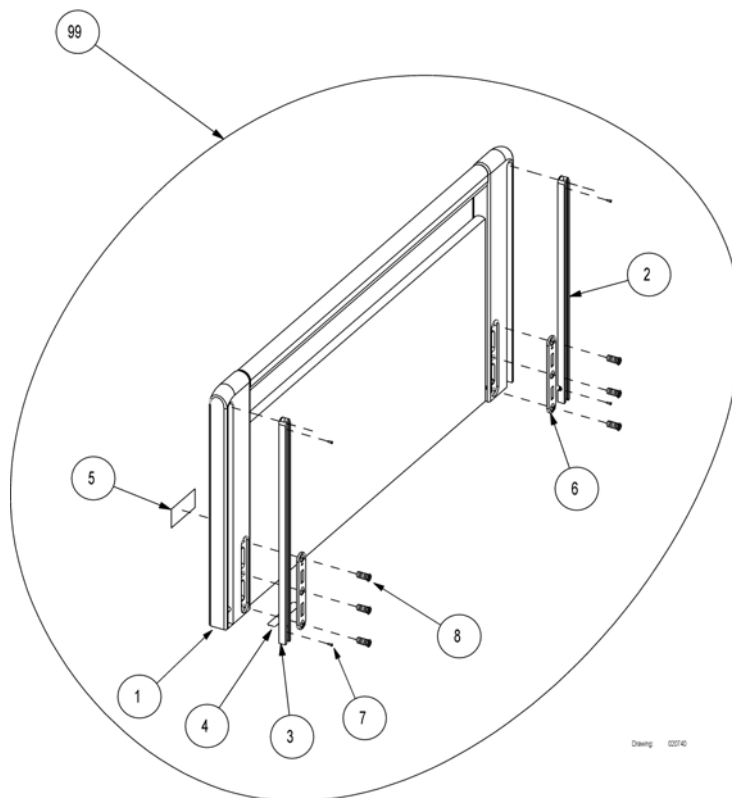

1	1505835	Not for separate sale - please contact Invacare	1
2	321519/L	Alu skena förhöjd till Flex/Sanne +5cm vänster	2
3	321519/R	Alu skena förhöjd till Flex/Sanne + 5cm höger	2
4	1505835	Not for separate sale - please contact Invacare	1
5	417547	Label with Invacare logo	2
6	418441	Bracket with holes	4
7	812847	Skruv till alu.skene Scanbedco 700	8
8	818338	Screw M8x24, special	12
99	1484403-0101	Gavel Sanne III 750/400 hög 105 cm par	1

(Oda and Piggy bed end brackets, 105 cm width)

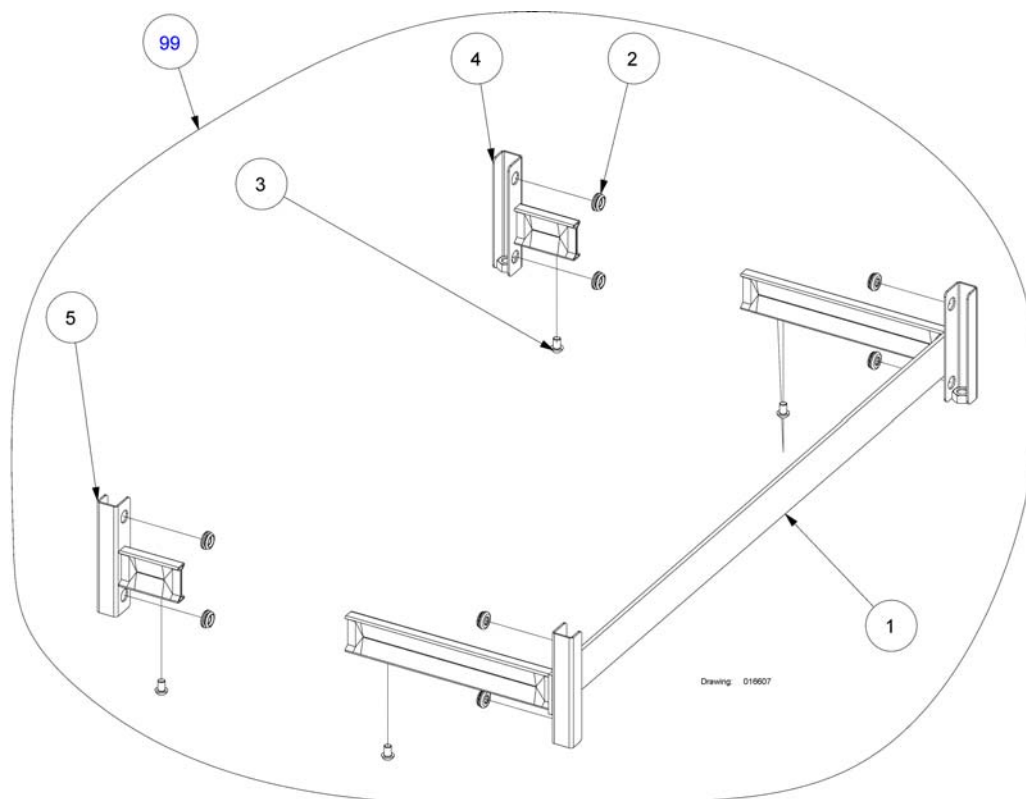


1	221147-1015	Bed end bracket u-profile 105 cm	1
2	417008	Membrannippel dia 145x2	8
3	812875	Skruv mc6s m8x12 fzb hfc 9729	4
4	1492058-1015	Gavelbeslag m u-profil vä Piggy/Oda	1
5	1492059-1015	Gavelbeslag m u-profil hö Piggy/Oda	1
99	021338.D0	Bed end bracket u-profile, 105 cm width	1

(Bed end and -brackets, 120 cm width)


-	(Sanne bed end plates, 120 cm width)	1
-	(Sanne bed end plates, 120 cm width, increased)	1
-	(Flex III bed end plates, 120 cm width)	1
-	(Sanne bed end brackets)	1
-	(Flex III bed end brackets)	1

(Invacare® SB750 =>2013.01.01) | (Bed ends and -plates)
(Sanne bed end plates, 120 cm width)

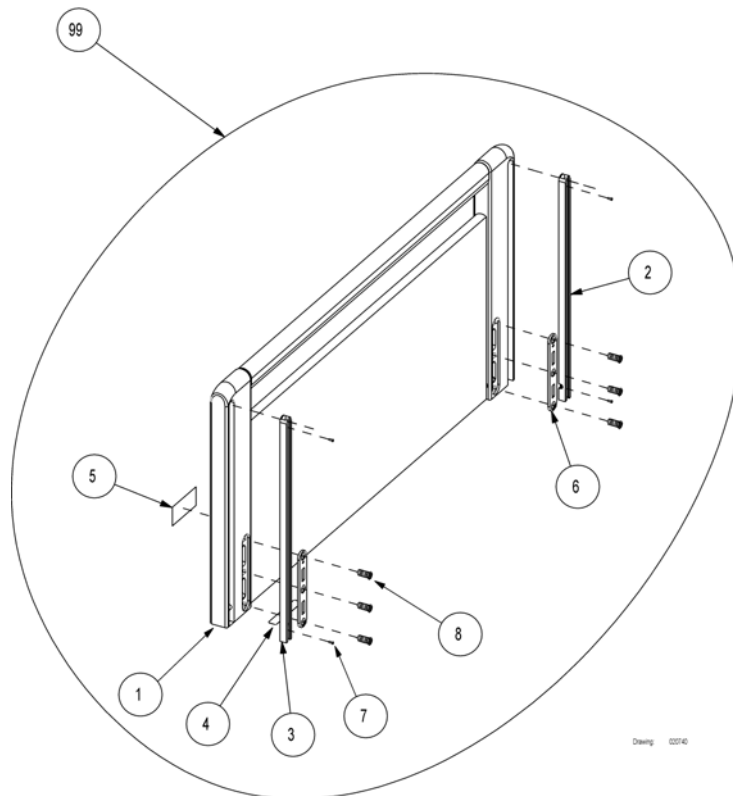


1	1505835	Not for separate sale - please contact Invacare	1
2	320504/L	Alu-skene till gavel Flex/Sanne vänster	2
3	320504/R	Alu-skene till gavel Flex/Sanne höger	2
4	1505835	Not for separate sale - please contact Invacare	1
5	417547	Label with Invacare logo	2
6	418441	Bracket with holes	4
7	812847	Skruv till alu.skene Scanbedco 700	8
8	818338	Screw M8x24, special	12
99	021954.07	Gavel Sanne mix 120cm	1
99	021954.02	Sänggavel Sanne 120 cm körsbär	1
99	021954.01	Sänggavel Sanne 120 cm bok natur	1
99	021954-0105	Bed end Sanne III, 120 cm width, 2 pcs.	1

(Sanne bed end plates, 120 cm width, increased)

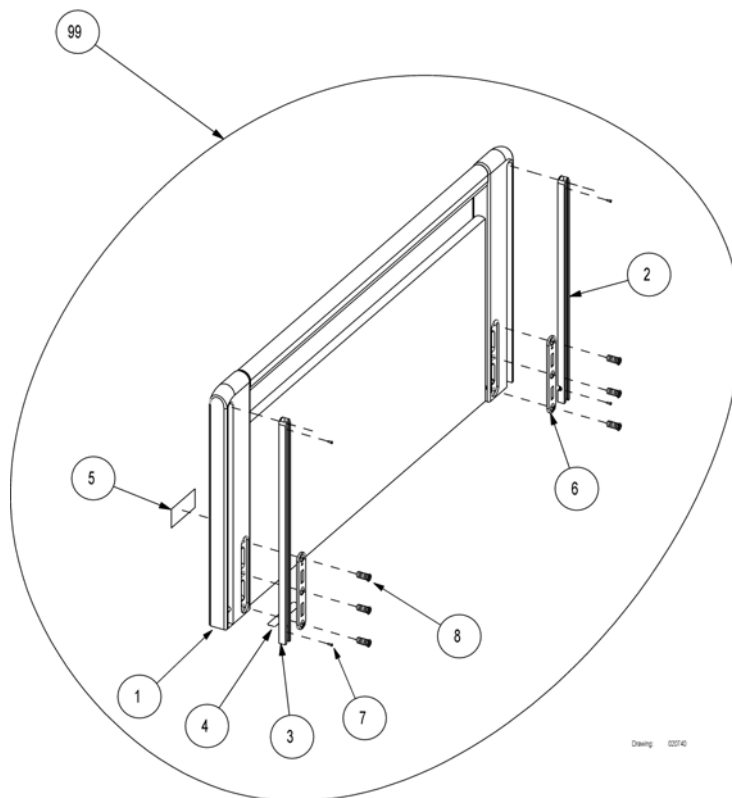

1	1505835	Not for separate sale - please contact Invacare	1
2	321519/L	Alu skena förhöjd till Flex/Sanne +5cm vänster	2
3	321519/R	Alu skena förhöjd till Flex/Sanne + 5cm höger	2
4	1505835	Not for separate sale - please contact Invacare	1
5	417547	Label with Invacare logo	2
6	418441	Bracket with holes	4
7	812847	Skruv till alu.skena Scanbedco 700	8
8	818338	Screw M8x24, special	12
99	1484404-0101	Gavel Sanne III 750/400 hög 120 cm par	1


(Invacare® SB750 =>2013.01.01) | (Bed ends and -plates)
(Flex III bed end plates, 120 cm width)

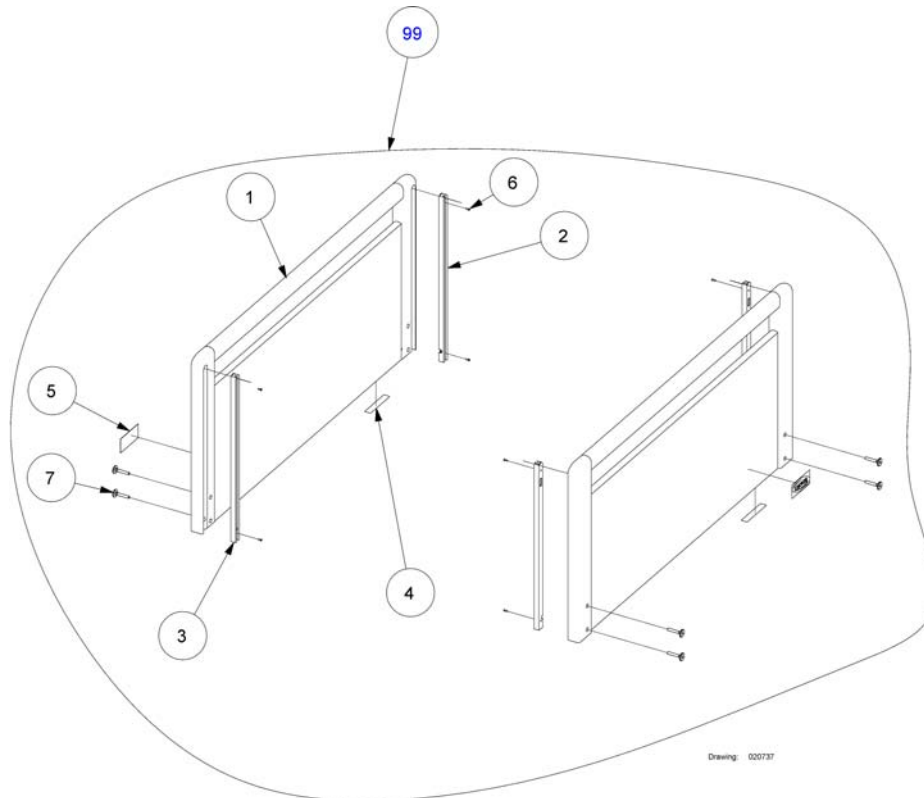

Drawing: 020737

1	1505835	Not for separate sale - please contact Invacare	1
2	320504/L	Alu-skene till gavel Flex/Sanne vänster	2
3	320504/R	Alu-skene till gavel Flex/Sanne höger	2
4	1505835	Not for separate sale - please contact Invacare	1
5	417547	Label with Invacare logo	2
6	812847	Skruv till alu.skene Scanbedco 700	8
7	815720	Möbelskruv m6x40 till Flexgavel	8
99	021917.01	Gavel Flex 120cm bred till SB750 par	1

(ACP box with accessories)

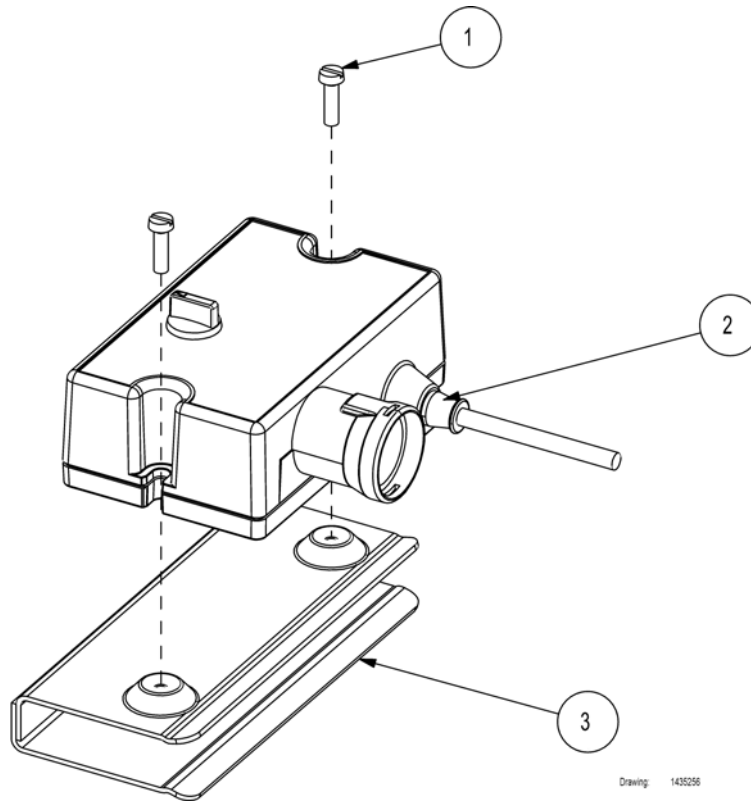


1	812358	Screw M4x14 ISO1207	2
2	1435255	ACP box, micro	1
3	1435305-1015	Bracket for ACP box	1

(Motor for back rest)


-	(Motor for backrest)	1
-	(Motor for backrest, with emergency lowering handle)	1
-	(Motor for backrest, V0)	1

(Motor for backrest)

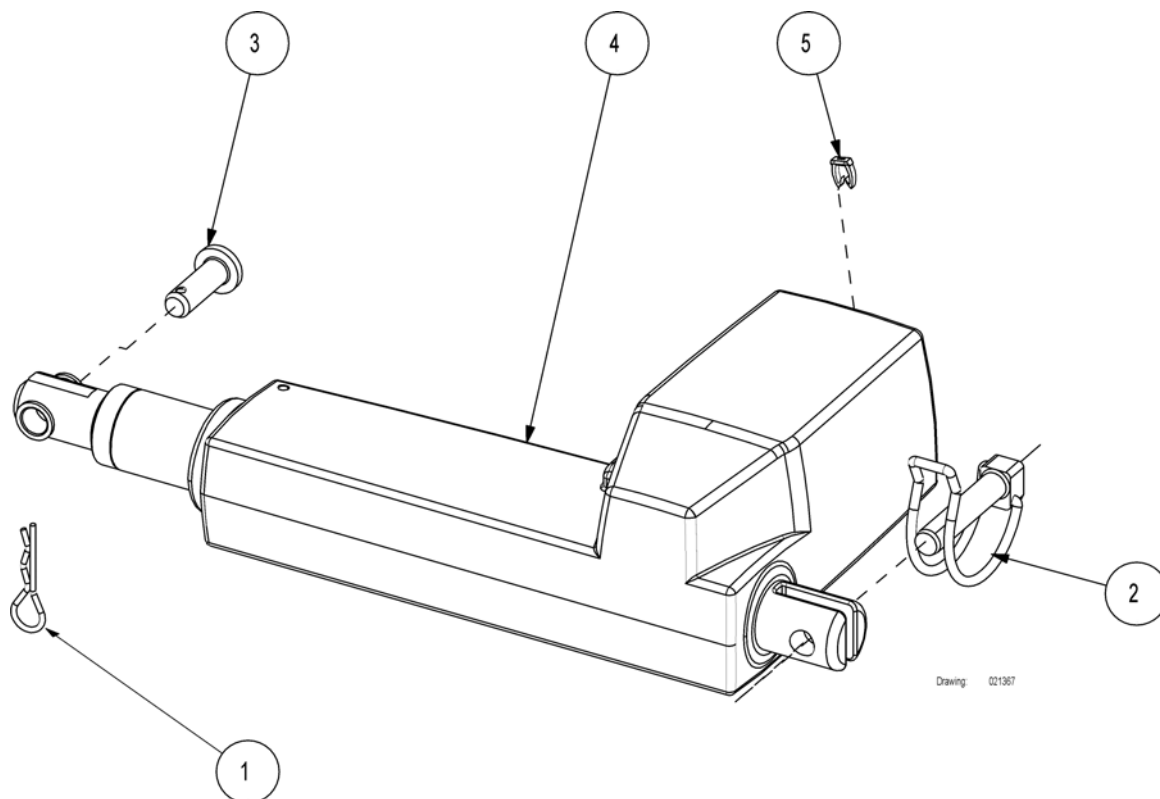


1	816427	Hårnålssprint SB750 t rygg/lår/ben-motor	1
2	819411	Traktorsprint ø10 till motor SB400/SB440/Etude	1
3	819468	Sprint 10x30 SB750 ryggdelsmotor	1
4	1423908	Motor rygg grå 24v till säng SB750	1
5	1444092	Cable marker 1 brown for backrest motor	1

(Motor for backrest, with emergency lowering handle)

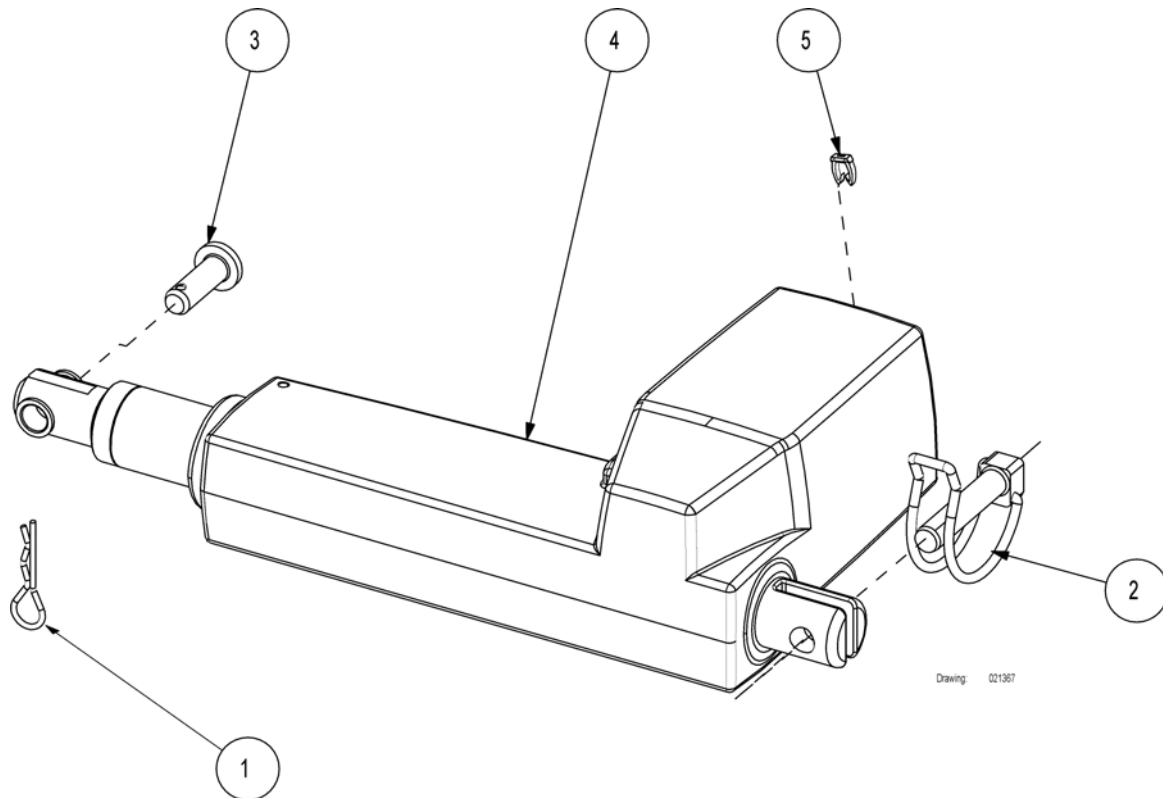

1	816427	Hårnålssprint SB750 t rygg/lår/ben-motor	1
2	819411	Traktorsprint ø10 till motor SB400/SB440/Etude	1
3	819468	Sprint 10x30 SB750 ryggdelsmotor	1
4	1441227	24v motor Ia31 ip66 ra I 7035	1
5	1444092	Cable marker 1 brown for backrest motor	1

(Motor for backrest, V0)

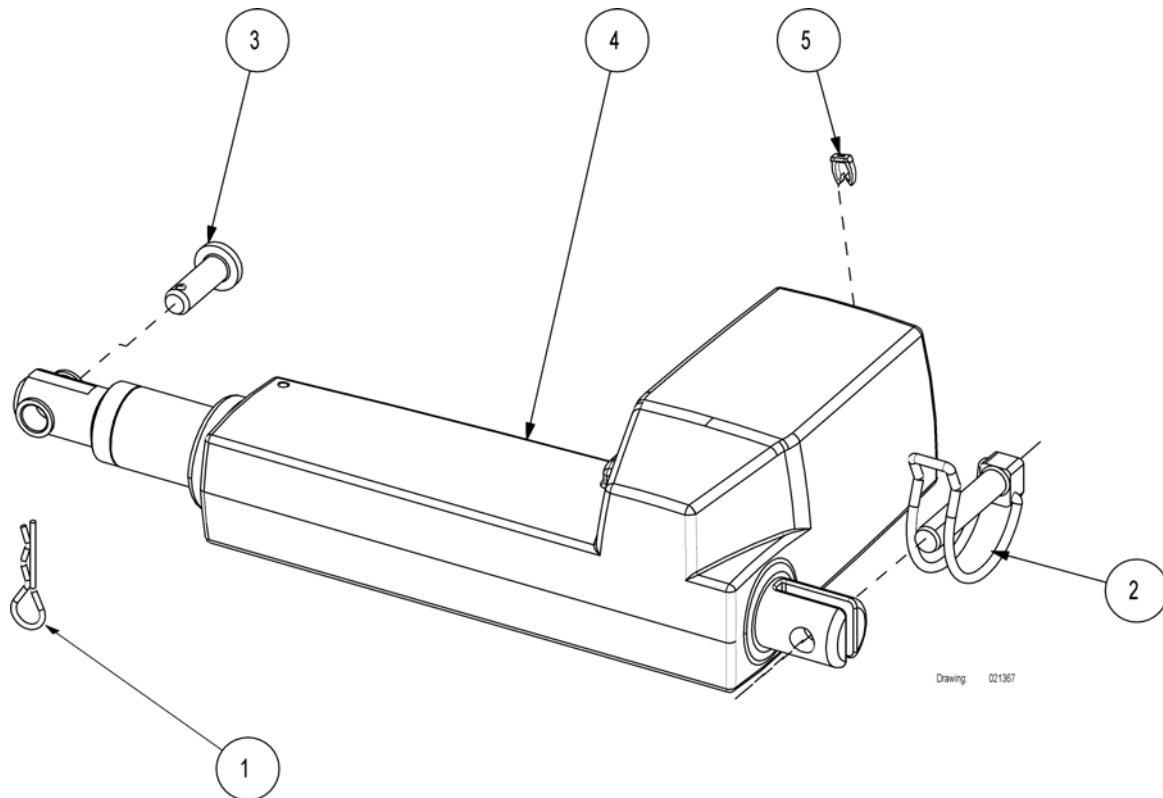


1	816427	Hårnålssprint SB750 t rygg/lår/ben-motor	1
2	819411	Traktorsprint ø10 till motor SB400/SB440/Etude	1
3	819468	Sprint 10x30 SB750 ryggdelsmotor	1
4	1492475-7035	24v motor LA31 IP66 v0 RAL 7035	1
5	1444092	Cable marker 1 brown for backrest motor	1

(Motor for thigh section)


-	(Motor for thigh section)	1
-	(Motor for thigh section, V0)	1

(Motor for thigh section)

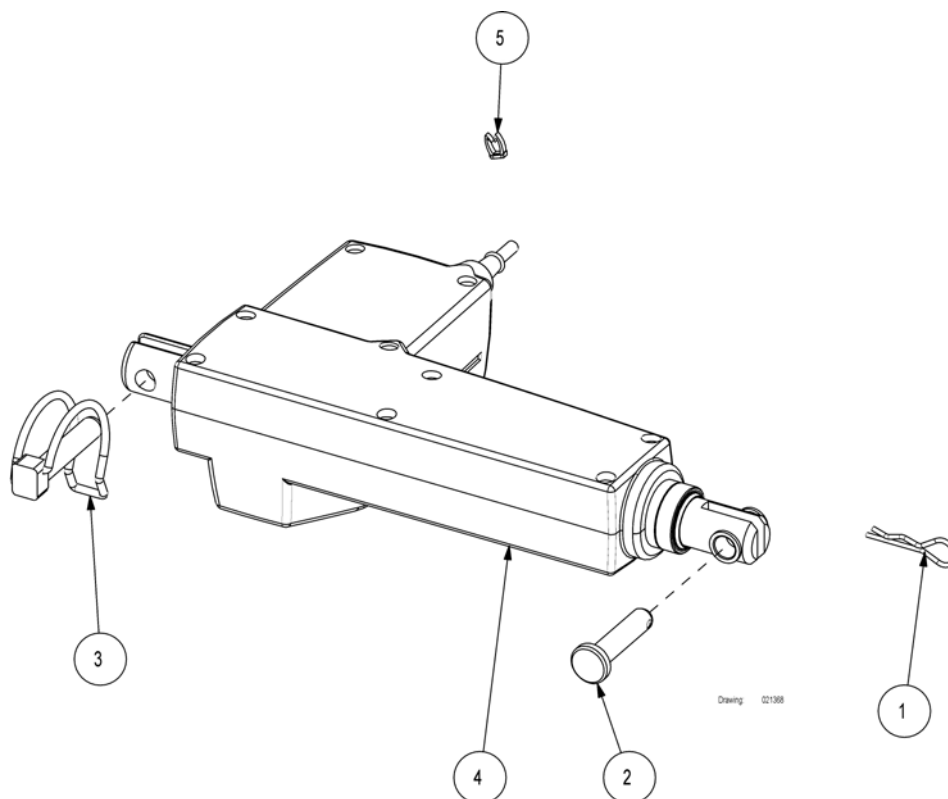

1	816427	Hårnålssprint SB750 t rygg/lår/ben-motor	1
2	818328	Splitbult 10 x 45	1
3	819411	Traktorsprint ø10 till motor SB400/SB440/Etude	1
4	1423903	Motor ben/lår grå ip66 SB750	1
5	1444094	Cable marker 3 orange for thigh motor	1

(Motor for thigh section, V0)

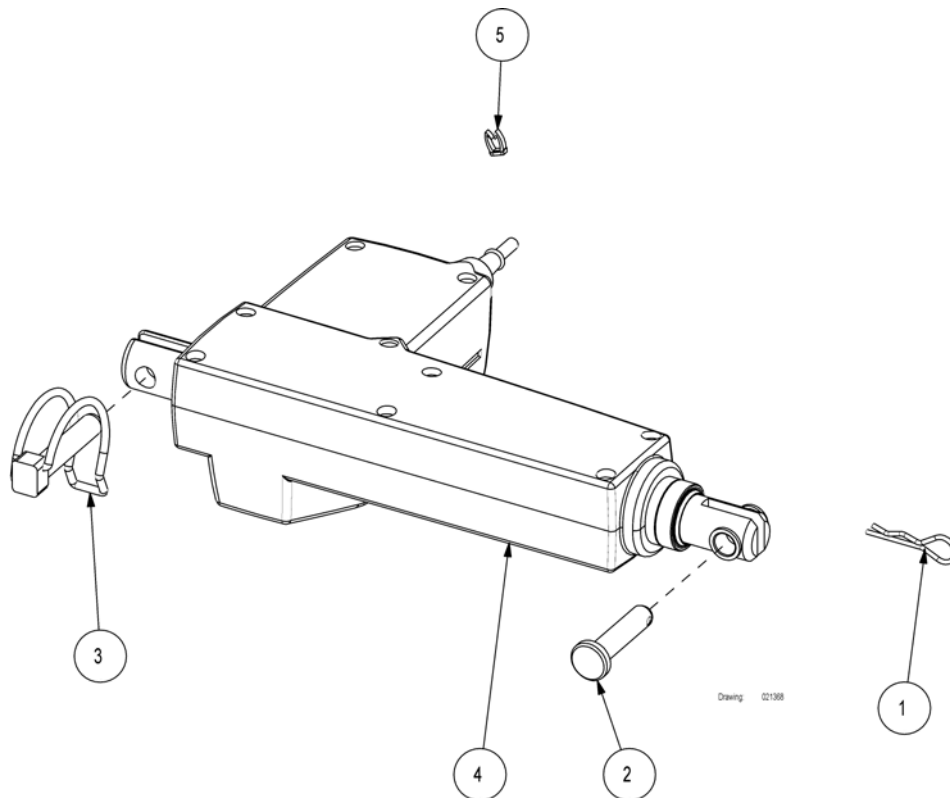


1	816427	Hårnålssprint SB750 t rygg/lår/ben-motor	1
2	818328	Splitbult 10 x 45	1
3	819411	Traktorsprint ø10 till motor SB400/SB440/Etude	1
4	1492476-7035	24v motor LA31 IP66 ra I 7035	1
5	1444094	Cable marker 3 orange for thigh motor	1

(Motor and gas spring for leg section)


-	(Motor for leg section)	1
-	(Motor for leg section, V0)	1
-	(Gas spring for 85 cm width)	1
-	(Gas spring for 90 cm width)	1

(Motor for leg section)

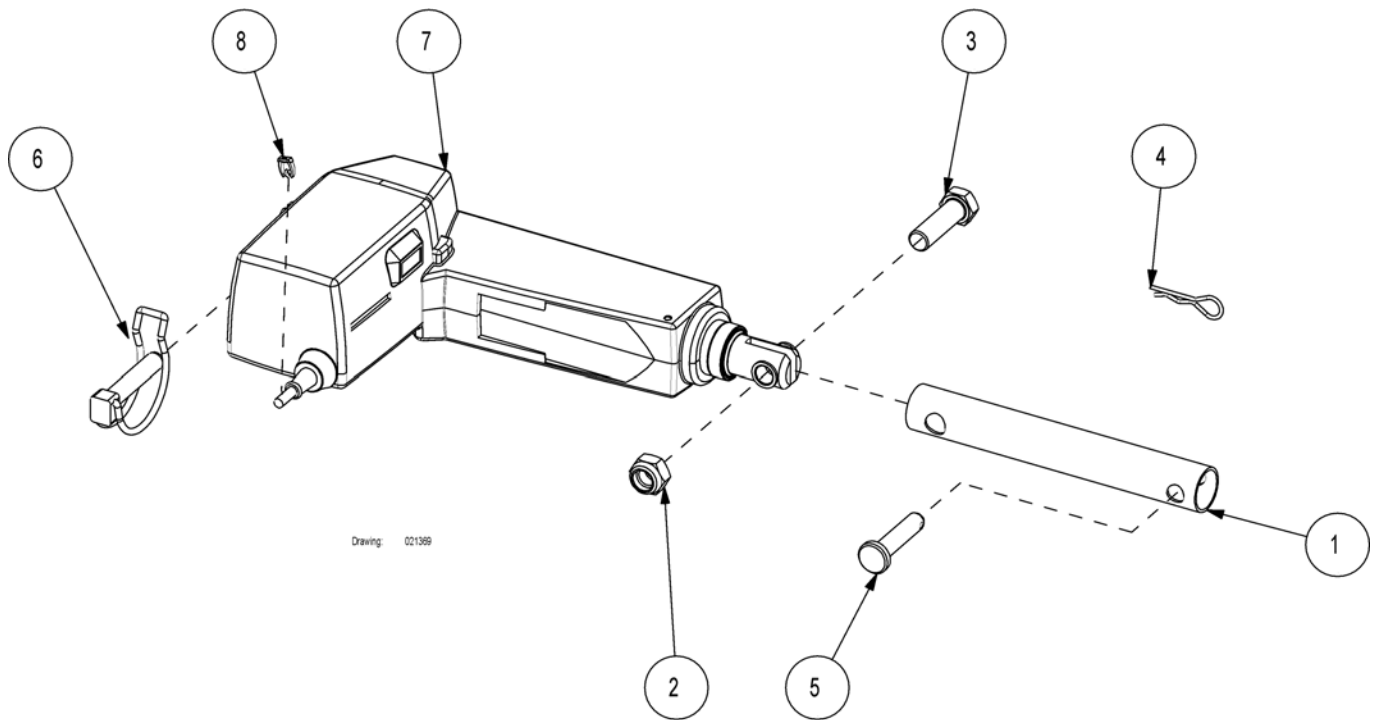

1	420172-1015	Tube Ø25x15x178	1
2	804481	Låsmutter m12 elzn	1
3	806762	Skruv till SB750	1
4	816427	Hårnålssprint SB750 t rygg/lår/ben-motor	1
5	818328	Splitbult 10 x 45	1
6	819411	Traktorsprint ø10 till motor SB400/SB440/Etude	1
7	1423903	Motor ben/lår grå ip66 SB750	1
8	1444095	Cable marker 4 yellow for leg motor	1


(Motor for leg section, V0)

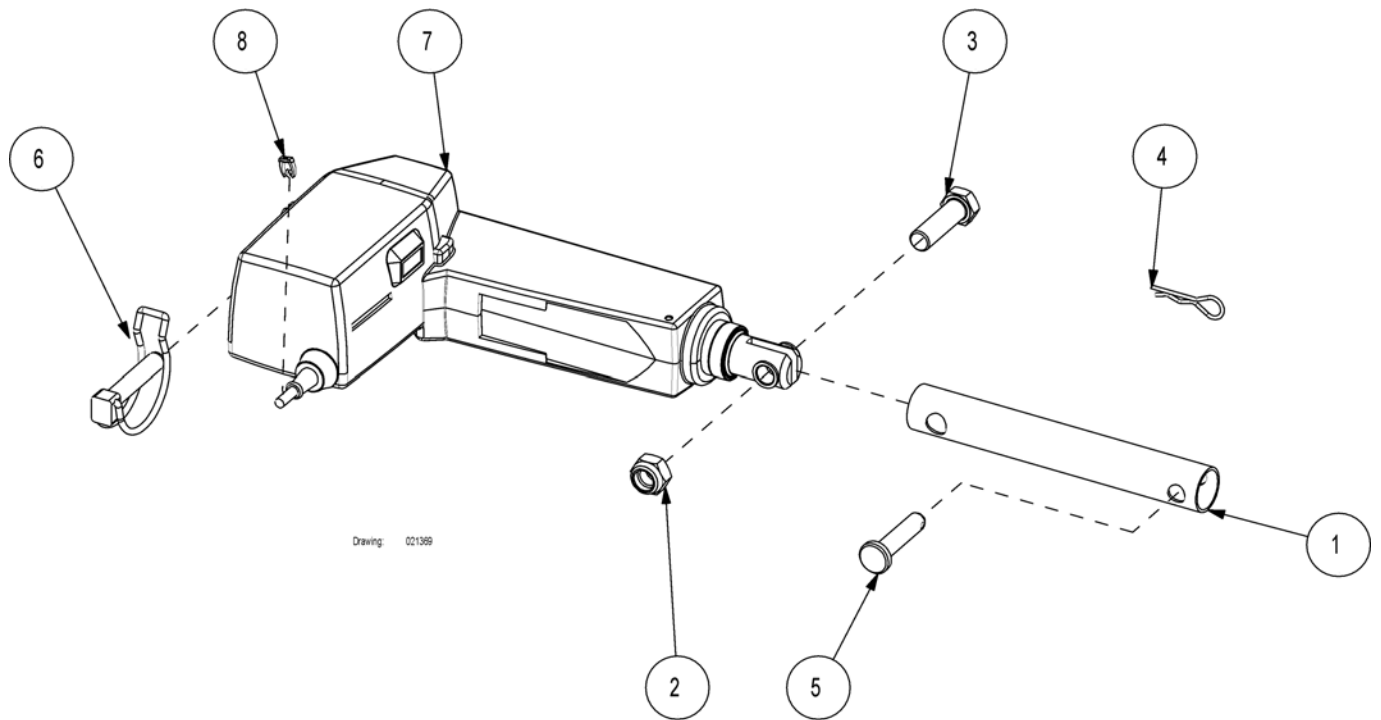

1	420172-1015	Tube Ø25x15x178	1
2	804481	Låsmutter m12 elzn	1
3	806762	Skruv till SB750	1
4	816427	Hårnålssprint SB750 t rygg/lår/ben-motor	1
5	818328	Splitbult 10 x 45	1
6	819411	Traktorsprint ø10 till motor SB400/SB440/Etude	1
7	1492476-7035	24v motor LA31 IP66 ra I 7035	1
8	1444095	Cable marker 4 yellow for leg motor	1

(Gas spring for 85 cm width)

99

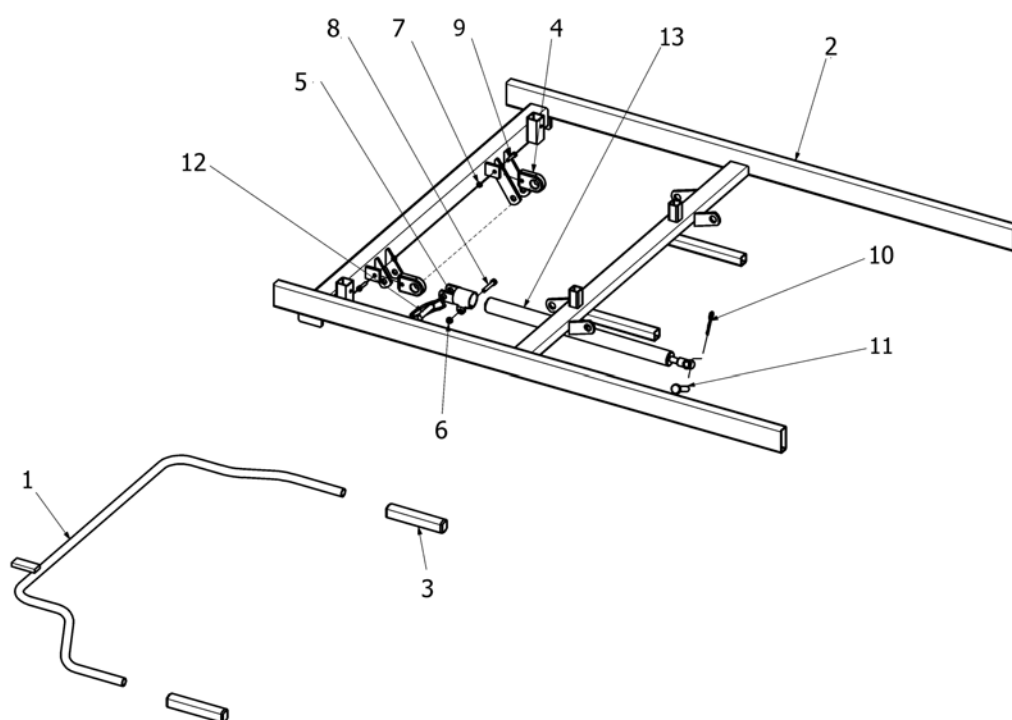


SP021371

99	SP021371-1015	(Gas spring for 85cm width)	0
1	219587-1015	Release lever for gas spring 85 cm width	1
2	221134-1015	Top frame foot end 85x183 cm	1
2	221154-1015	Top frame foot end 85x200 cm	1
3	314900-7035	Handtag till utl.spak SB700	2
4	404833	Hållare för utlösarbygel SB700/SB750	2
5	414911.60	Gasfjäder-fäste till bendel SB750	1
6	803992	Låsmutter m6 fzb	1
7	804458	Låsmutter m5 din 985-8	2
8	806785	Skruv mc6s m6x35 fzb hfc 681 din 912	1
9	806793	Skruv mcs m5x20 hfc 9033 din 84	2
10	816427	Hårnålssprint SB750 t rygg/lår/ben-motor	1
11	818328	Splitbult 10 x 45	1
12	819411	Traktorsprint ø10 till motor SB400/SB440/Etude	1
13	821248	Gaskolv bendel SB750 18-2-394-45 1000n	1

(Gas spring for 90 cm width)

99

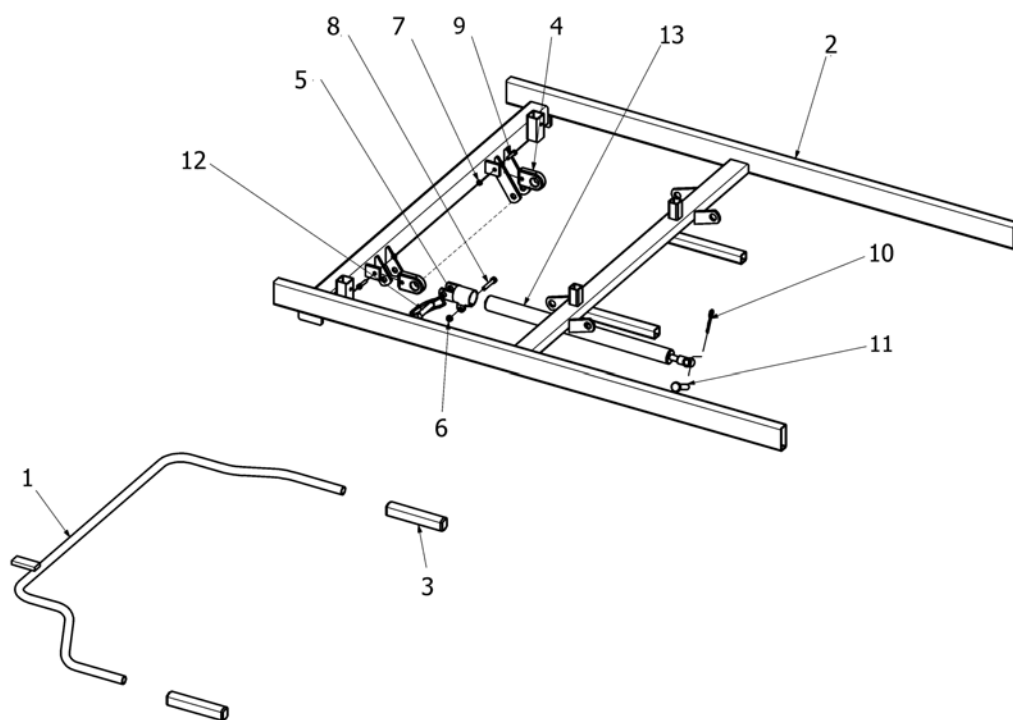


SP021371

99	SP021372-1015	(Gas spring for 90 cm width)	1
1	219598-1015	Release lever for gas spring 90 cm width	1
2	221162-1015	Top frame foot end 90x200 cm	1
2	221165-1015	Top frame foot end 90x183 cm	1
3	314900-7035	Handtag till utl.spak SB700	2
4	404833	Hållare för utlösarbygel SB700/SB750	2
5	414911.60	Gasfjäder-fäste till bendel SB750	1
6	803992	Låsmutter m6 fzb	1
7	804458	Låsmutter m5 din 985-8	2
8	806785	Skruv mc6s m6x35 fzb hfc 681 din 912	1
9	806793	Skruv mcs m5x20 hfc 9033 din 84	2
10	816427	Hårnålssprint SB750 t rygg/lår/ben-motor	1
11	818328	Splitbult 10 x 45	1
12	819411	Traktorsprint ø10 till motor SB400/SB440/Etude	1
13	821248	Gaskolv bendel SB750 18-2-394-45 1000n	1

(Motor for height adjustment)


-	(Motor for height adjustment)	1
-	(Motor for height adjustment, 120 cm width)	1
-	(Motor for height adjustment, 85+90+105 cm width, V0)	1
-	(Motor for height adjustment, 120 cm width, V0)	1

(Motor for height adjustment)

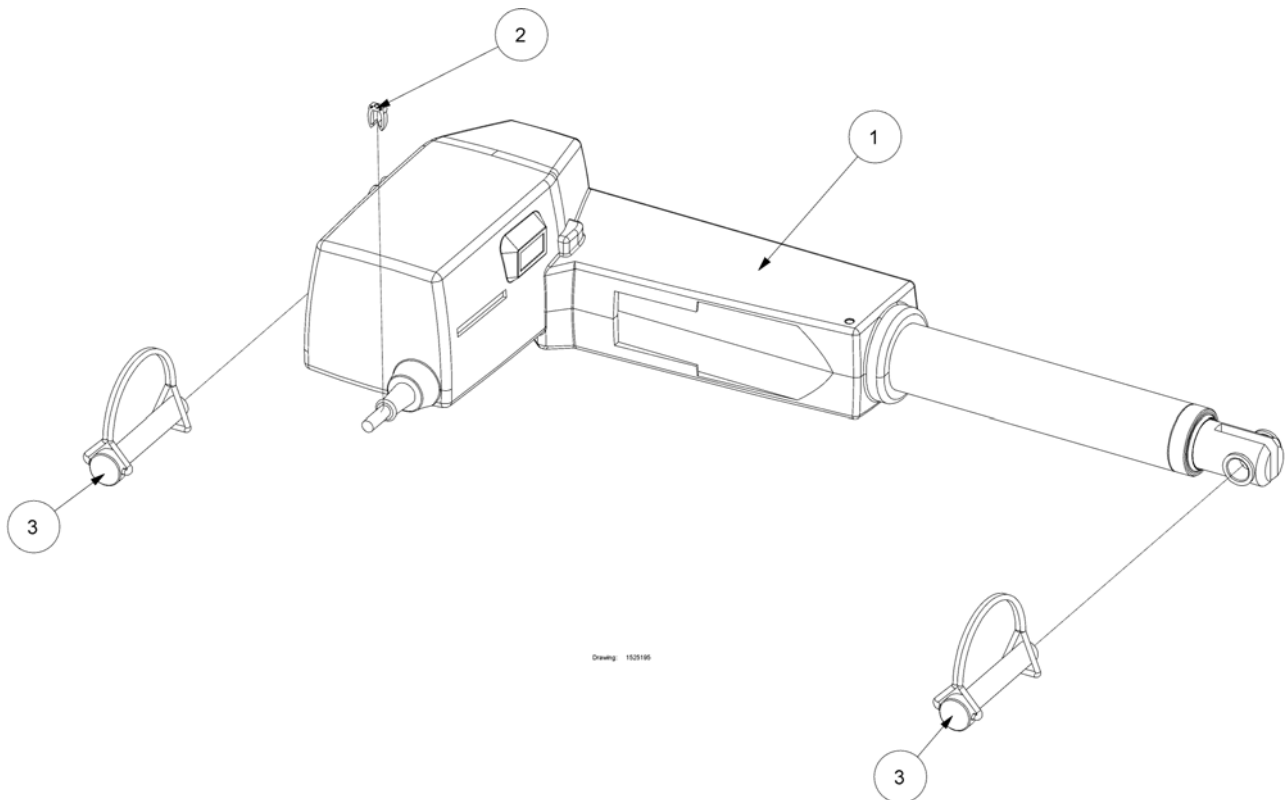

1	1423901	Motor lyft grå ip66 till säng SB750 bredd 85/90cm	1
2	1444093	Cable marker 2 red for height motor	1
3	1507121	Traktorsprint 10x50	2


(Motor for height adjustment, 120 cm width)

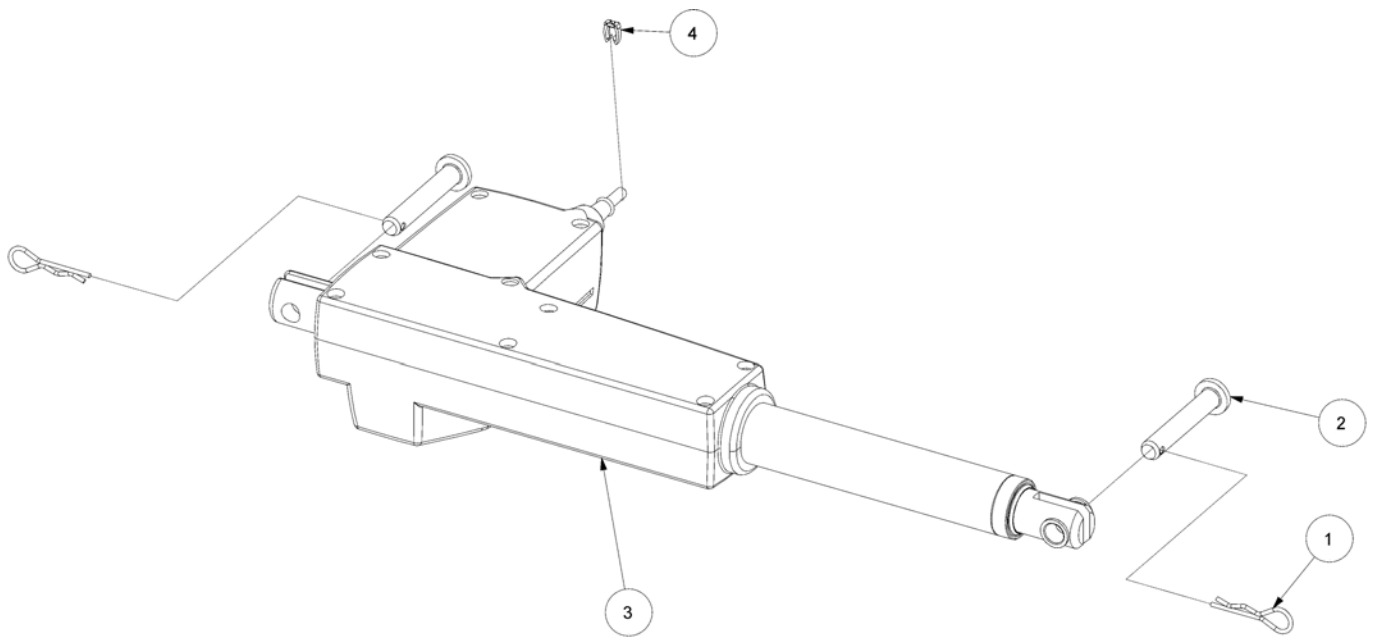

Drawing: 021370

1	816427	Hårnålssprint SB750 t rygg/lår/ben-motor	2
2	821204	Splitbult 10*60	2
3	1423909	Motor lyft grå till 105/120 cm bred SB750	1
4	1444093	Cable marker 2 red for height motor	1


(Motor for height adjustment, 85+90+105 cm width, V0)

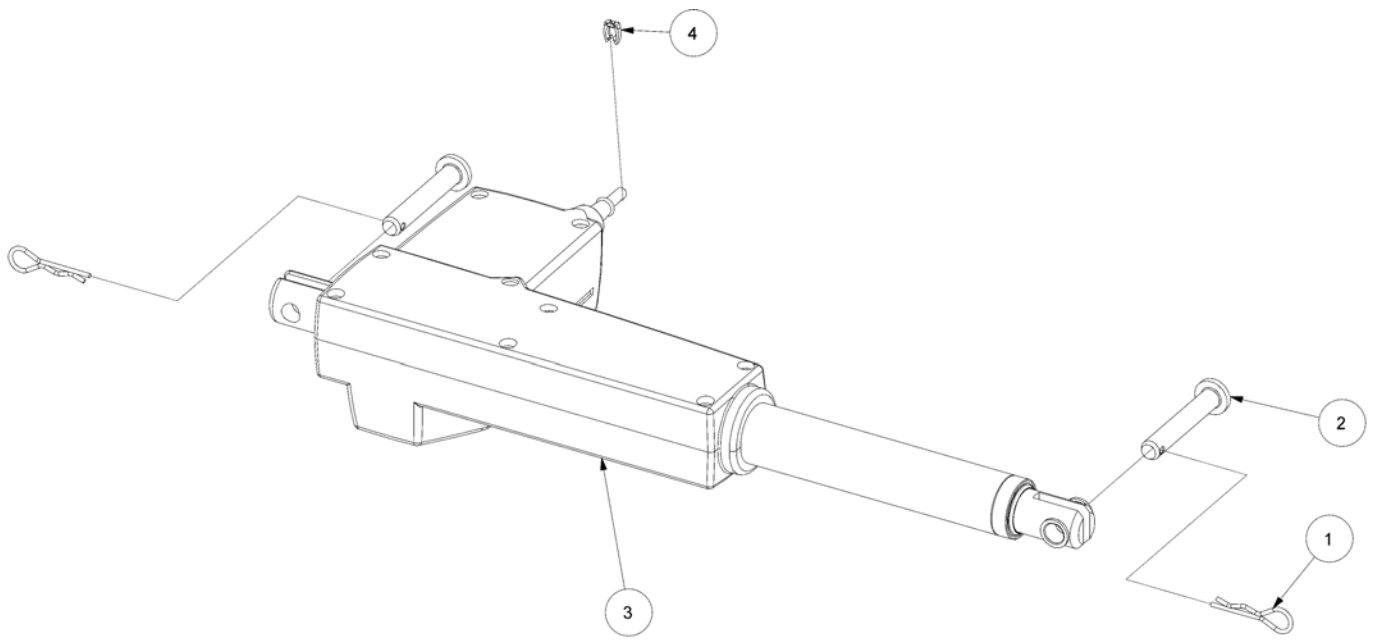

Drawing: 021370

1	816427	Hårnålssprint SB750 t rygg/lår/ben-motor	2
2	821204	Splitbult 10*60	2
3	1492477-7035	24v motor LA31 IP66 v0 RAL 7035	1
4	1444093	Cable marker 2 red for height motor	1

(Motor for height adjustment, 120 cm width, V0)

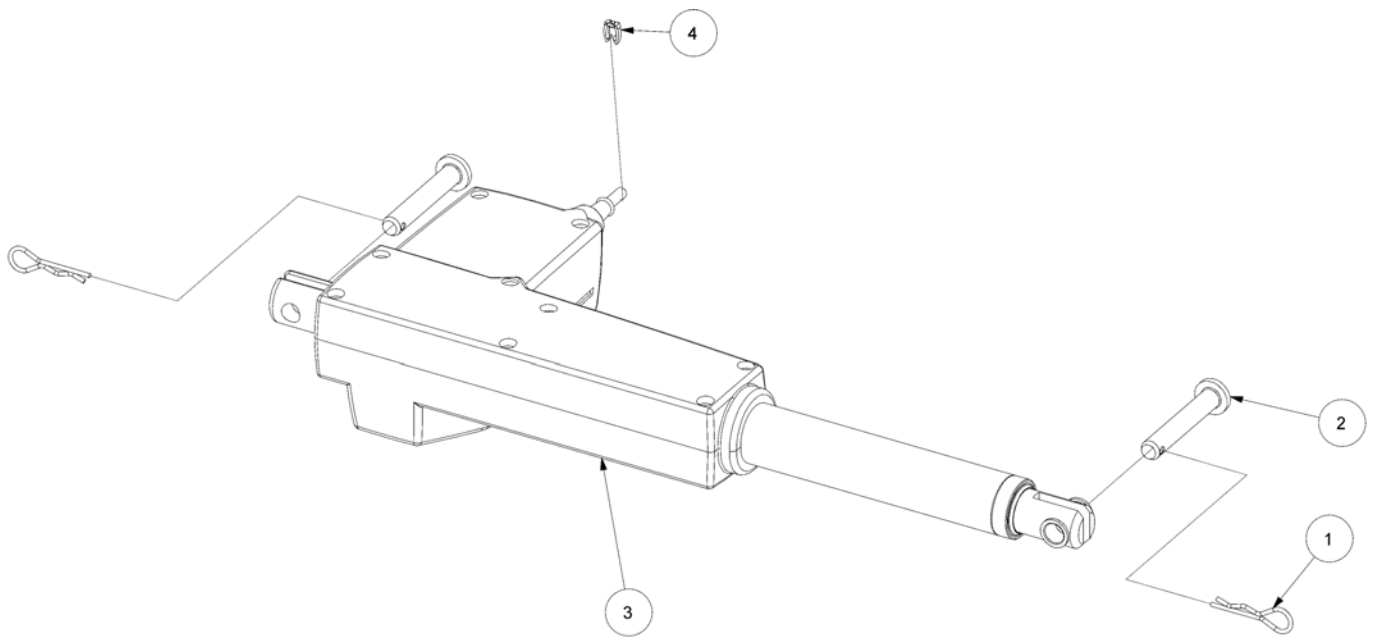


Drawing: 021370

1	816427	Hårnålssprint SB750 t rygg/lår/ben-motor	2
2	821204	Splitbult 10*60	2
3	1492479-7035	24v motor LA31 IP66 v0 RAL 7035	1
4	1444093	Cable marker 2 red for height motor	1

(Control unit for 85+90 cm width)


-	(Control unit without accumulator backup, 4 motors, EU)	1
-	(Control unit without accumulator backup, 3 motors, EU)	1
-	(Control unit with accumulator backup, 4 motors, EU)	1
-	(Control unit with accumulator backup, 3 motors, EU)	1
-	(Control unit without accumulator backup, 4 motors, UK)	1
-	(Control unit with accumulator backup, 4 motors, UK)	1
-	(Control unit without accumulator backup, 4 motors, Japan)	1
-	(Control unit with battery backup)	1
-	(Control unit without accumulator backup, 4 motors, V0, EU)	1

(Control unit without accumulator backup, 4 motors, EU)

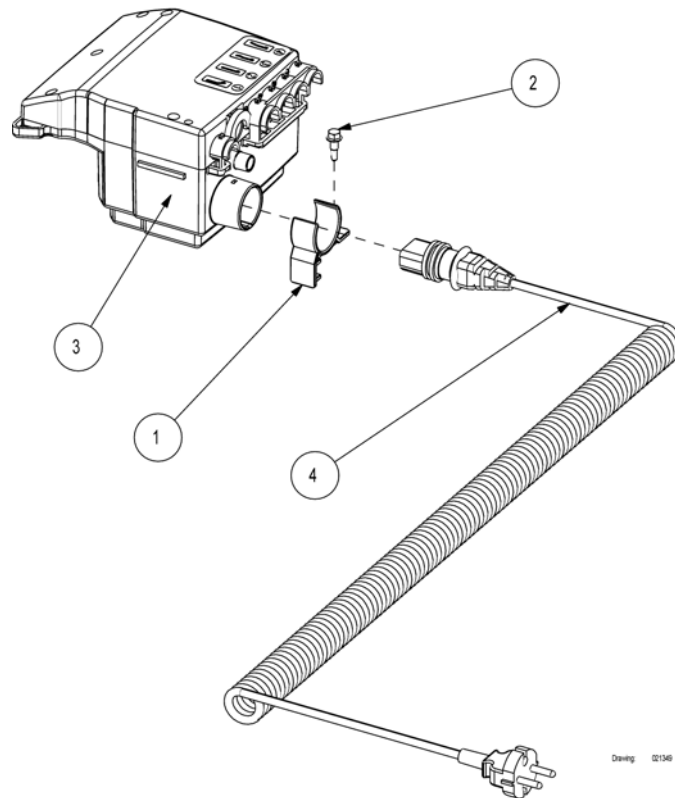


1	320522-7035	Hållare för elektronik SB750	1
2	819407	Skruv självgängande 5.5x19 SB750	1
3	1423906	Elektronik cb09 grå SB750	1
4	1424000	Nätkabel grå 755/750/400/vis NilcoIII/combi 3-d elekt	1

(Control unit without accumulator backup, 3 motors, EU)

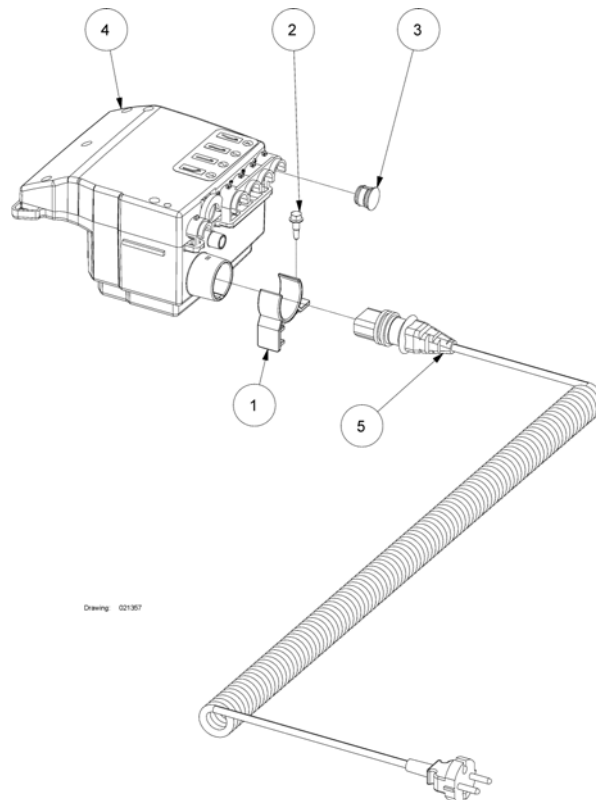


1	320522-7035	Hållare för elektronik SB750	1
2	819407	Skruv självgängande 5.5x19 SB750	1
3	819429-7035	Plug for control unit CB09	1
4	1423906	Elektronik cb09 grå SB750	1
5	1424000	Nätkabel grå 755/750/400/vis NilcoIII/combi 3-d elekt	1

(Control unit with accumulator backup, 4 motors, EU)

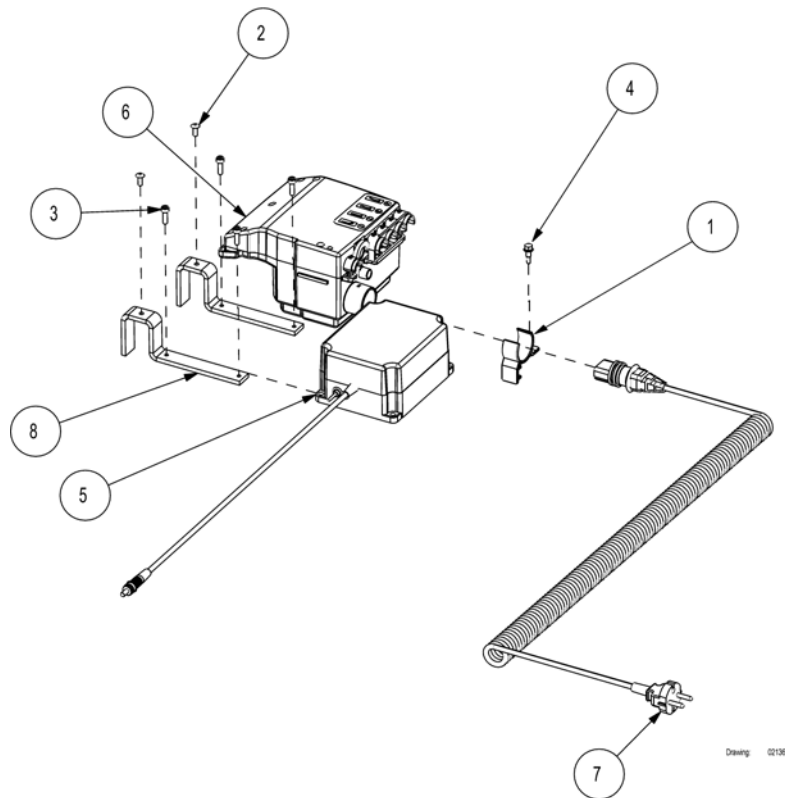

1	320522-7035	Hållare för elektronik SB750	0
2	803833	Popnit SB700/750	2
3	816480	Skruv till trafoplåt Vision	4
4	819407	Skruv självgängande 5.5x19 SB750	1
5	1423905	Accumulator backup unit for CB09	1
6	1423907	Elektronik f batteri- backup SB750 cb09 grå	1
7	1424000	Nätkabel grå 755/750/400/vis NilcoIII/combi 3-d elekt	1
8	1441342-1015	Bracket for accumulator backup	2


(Control unit with accumulator backup, 3 motors, EU)

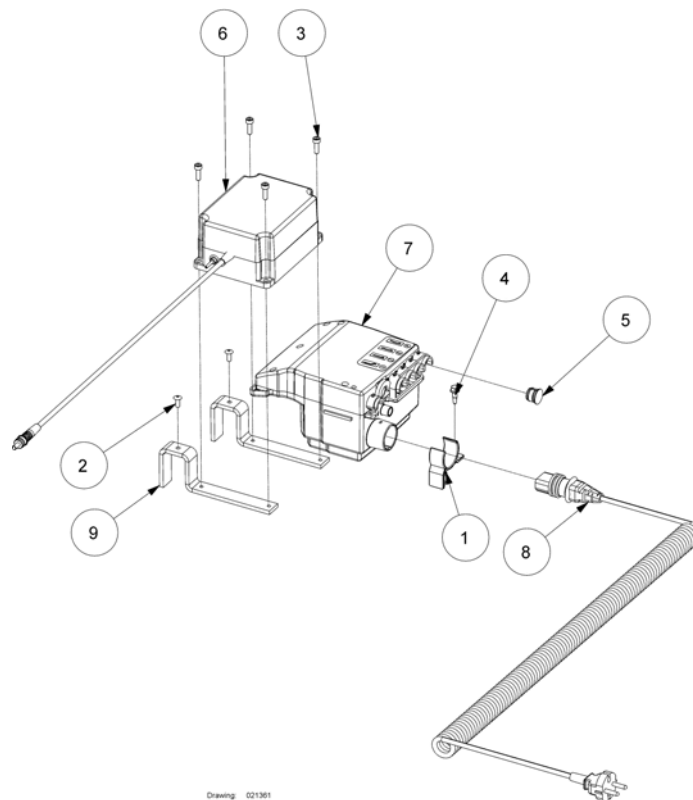

Drawing: 021361


1	320522-7035	Hållare för elektronik SB750	1
2	803833	Popnit SB700/750	2
3	816480	Skruv till trafoplåt Vision	4
4	819407	Skruv självgängande 5.5x19 SB750	1
5	819429-7035	Plug for control unit CB09	1
6	1423905	Accumulator backup unit for CB09	1
7	1423907	Elektronik f batteri- backup SB750 cb09 grå	1
8	1424000	Nätkabel grå 755/750/400/vis NilcoIII/combi 3-d elekt	1
9	1441342-1015	Bracket for accumulator backup	2

(Control unit without accumulator backup, 4 motors, UK)

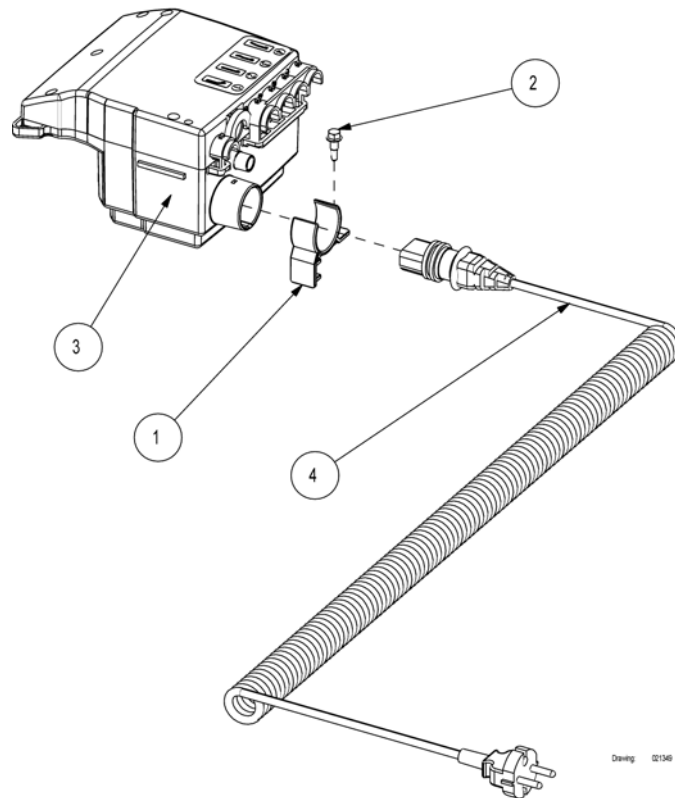


1	320522-7035	Hållare för elektronik SB750	1
2	819407	Skruv självgängande 5.5x19 SB750	1
3	1423906	Elektronik cb09 grå SB750	1
4	819430	Power cable UK	1

(Control unit with accumulator backup, 4 motors, UK)

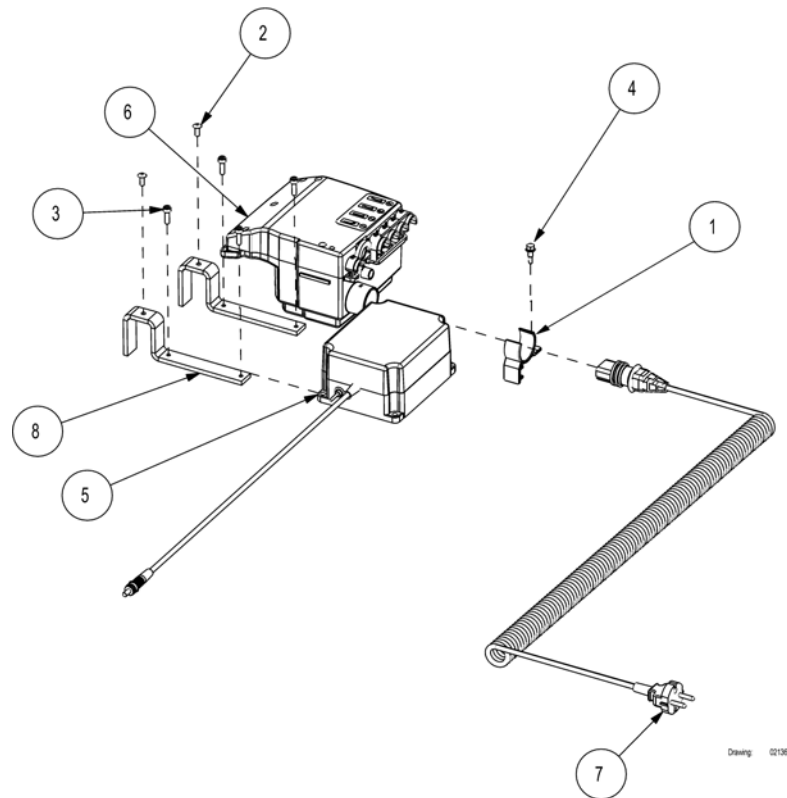


1	320522-7035	Hållare för elektronik SB750	1
2	803833	Popnit SB700/750	2
3	816480	Skruv till trafoplåt Vision	4
4	819407	Skruv självgängande 5.5x19 SB750	1
5	1423905	Accumulator backup unit for CB09	1
6	1423907	Elektronik f batteri- backup SB750 cb09 grå	1
7	819430	Power cable UK	1
8	1441342-1015	Bracket for accumulator backup	2

(Control unit without accumulator backup, 4 motors, Japan)

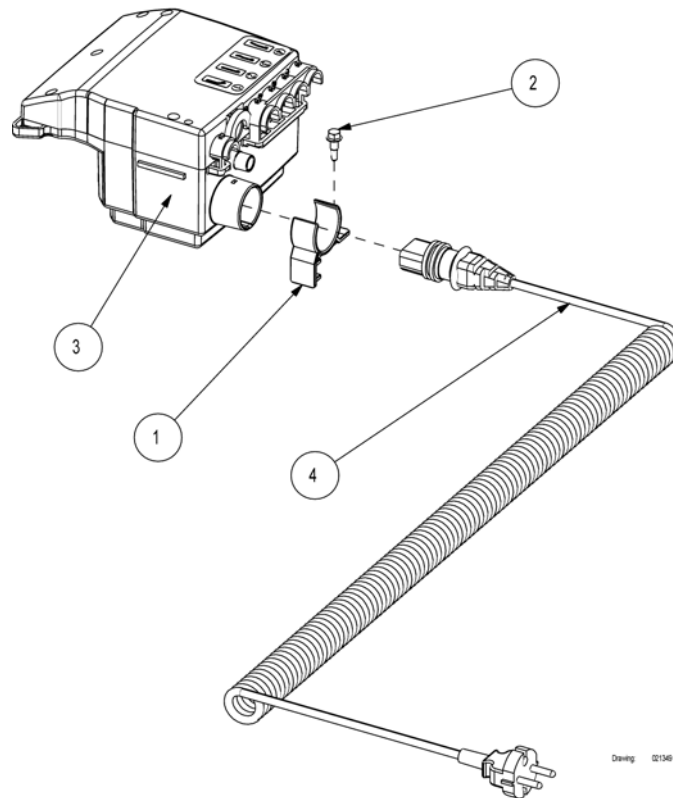


1	320522-7035	Hållare för elektronik SB750	1
2	819407	Skruv självgängande 5.5x19 SB750	1
4	1423892	Power cable Japan	1

(Control unit with battery backup)

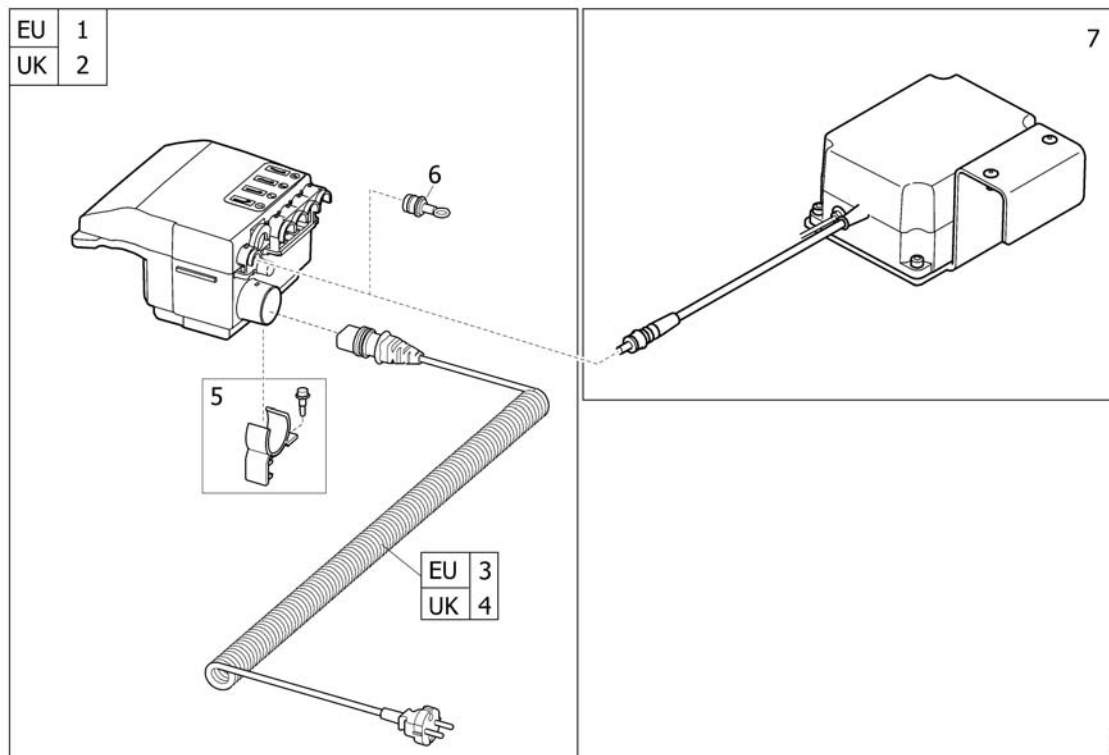

1	SP1441301	(Control unit CB09, EU, for battery backup)	1
2	SP1526967	(Control unit CB09, UK, for battery backup)	1
3	1424000	Nätkabel grå 755/750/400/vis NilcolIII/combi 3-d elekt	1
4	819430	Power cable UK	1
5	SP1580782	(Bracket for control unit CB09, cpl.)	1
6	1441299-7035	Plug for control unit CB09	1
7	1524336-0154	Batterienhet till batteribackup SB755	1

(Control unit without accumulator backup, 4 motors, V0, EU)

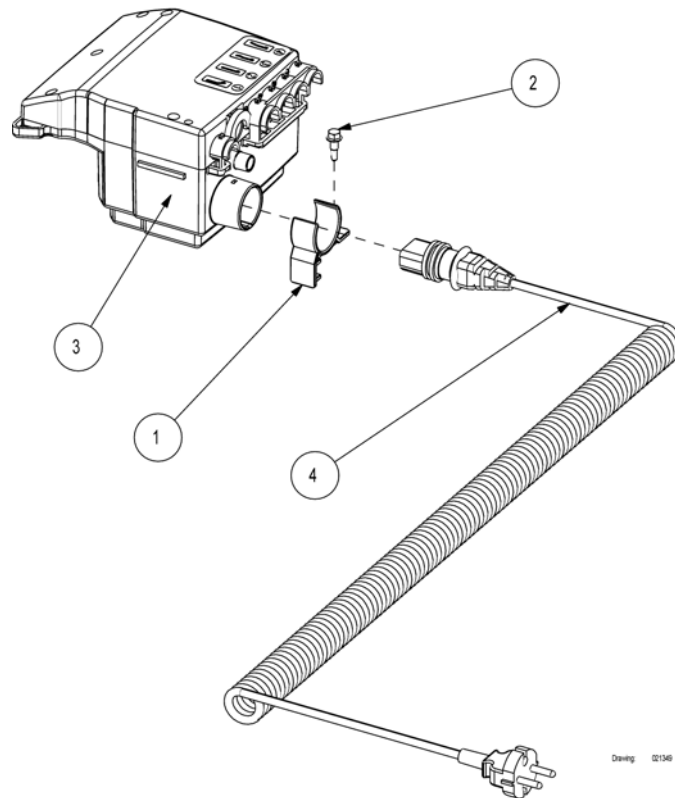


Drawing 02156

1	320522-7035	Hållare för elektronik SB750	1
2	819407	Skruv självgängande 5.5x19 SB750	1
3	1492474-7035	Control unit CB09 V0	1
4	1424000	Nätkabel grå 755/750/400/vis NilcoIII/combi 3-d elekt	1

(Control unit for 105+120 cm width)


-	(Control unit without accumulator backup, 4 motors, EU)	1
-	(Control unit without accumulator backup, 4 motors, V0, EU)	1

(Control unit without accumulator backup, 4 motors, EU)

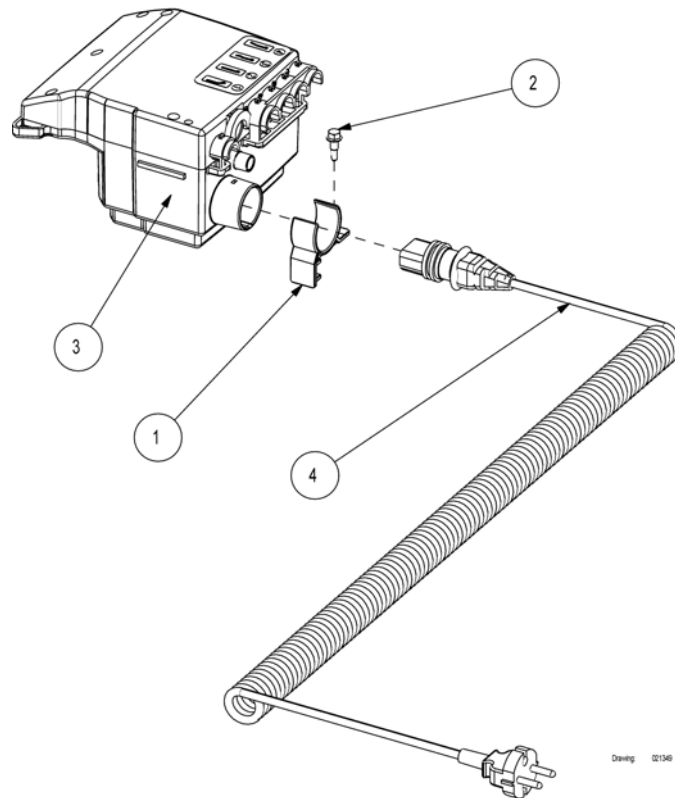


1	320522-7035	Hållare för elektronik SB750	1
2	819407	Skruv självgängande 5.5x19 SB750	1
3	1423978	Elektronik cb09 SB750 för 105/120cm bredd	1
4	1424000	Nätkabel grå 755/750/400/vis NilcoIII/combi 3-d elekt	1

(Control unit without accumulator backup, 4 motors, V0, EU)

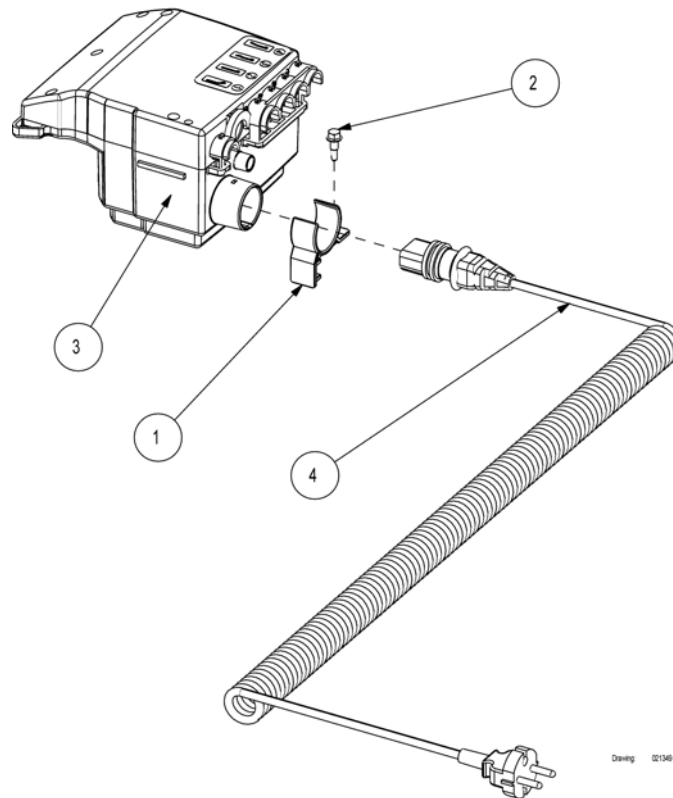


1	320522-7035	Hållare för elektronik SB750	1
2	819407	Skruv självgängande 5.5x19 SB750	1
3	1492478-7035	Control unit CB09 EAS V0	1
4	1424000	Nätkabel grå 755/750/400/vis NilcoIII/combi 3-d elekt	1

(Hand control)

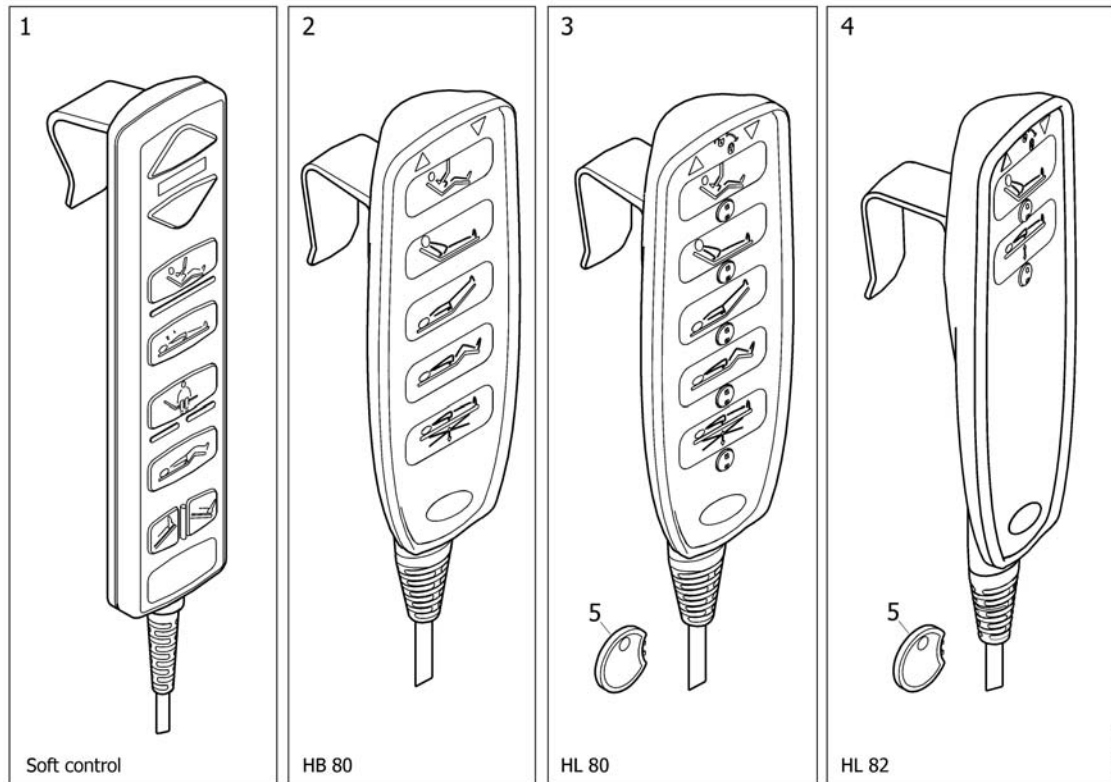


-	(Extension cable, handcontrol)	1
1	1423983 Manöverdosa softcontrol 4-motors SB750	1
2	1423986 Manöverdosa softcontrol 3-motors SB750/400	1
3	1492876 Manöverdosa 4mot grön knapp SB750 ers 1423902	1
4	1492972 Man.dosa 3mot grön knapp SB750/400 ers 1423904	1
5	319179-9005 Hand control 2 motors for CB09	1

(Extension cable, handcontrol)

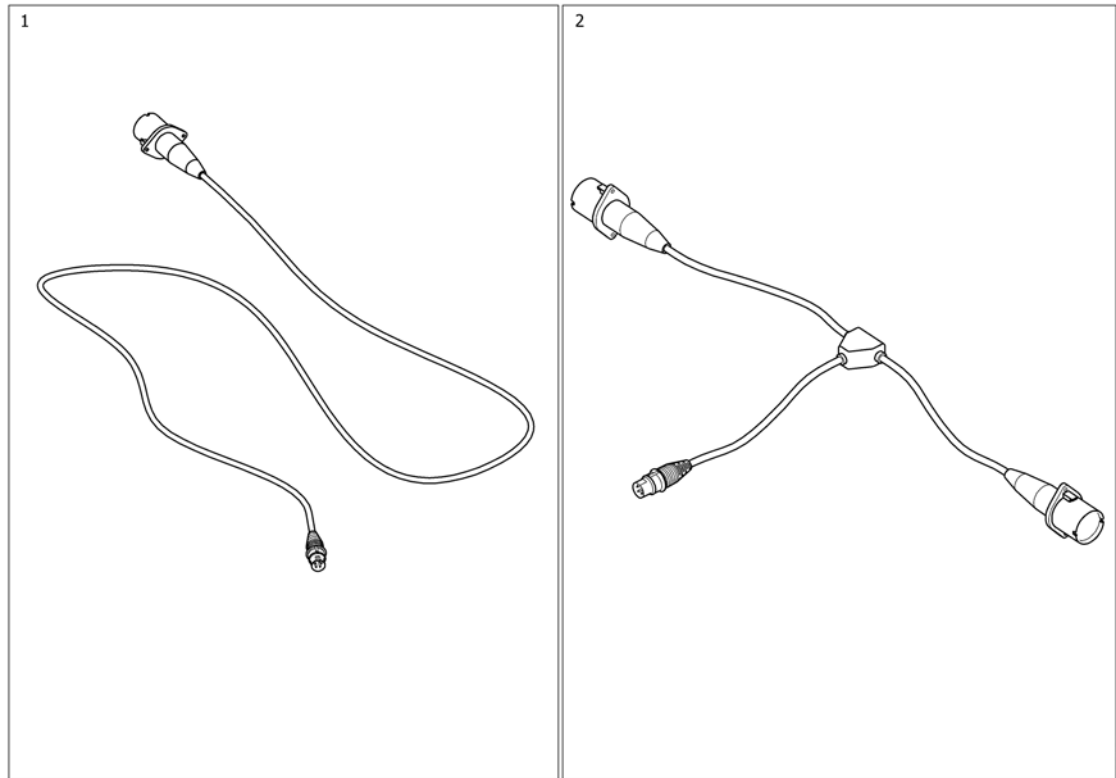


1	1532489-7035	Förlängningskabel 2,4 m	1
2	819465	Förgreningskabel	1

(Britt III side rail, for 183 cm length)

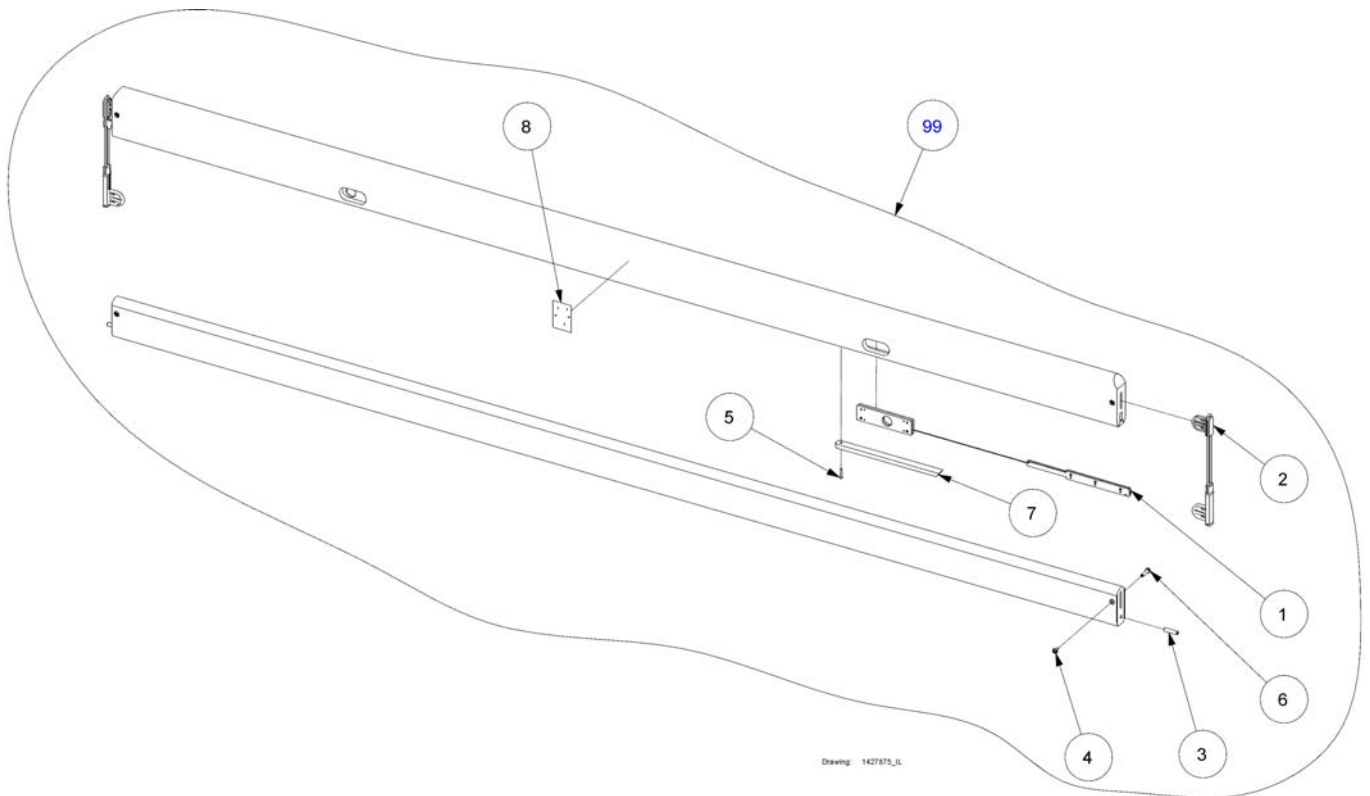


1	020024	Utlösare utbytesdelar Britt/Lilly sats/2 grind	1
2	319137	Glider t Brittgrind se 020023	2
3	413766	Sprint grind Britt	2
4	817265	Bokskruv hona SB750	4
5	814374	Wood screw $\varnothing 3 \times 20$ 0x20 hf c 324 for beech list	2
6	812367	Bokskruv hane SB750	4
7	1517167-0101	Wood strip for wood siderails	2
7	1517167-0102	Wood strip for wood siderails	2
8	1435328	Label for wooden side rail release	1
99	021342.01	Grind Britt 180cm till SB750 kort par	1

Grind Britt III

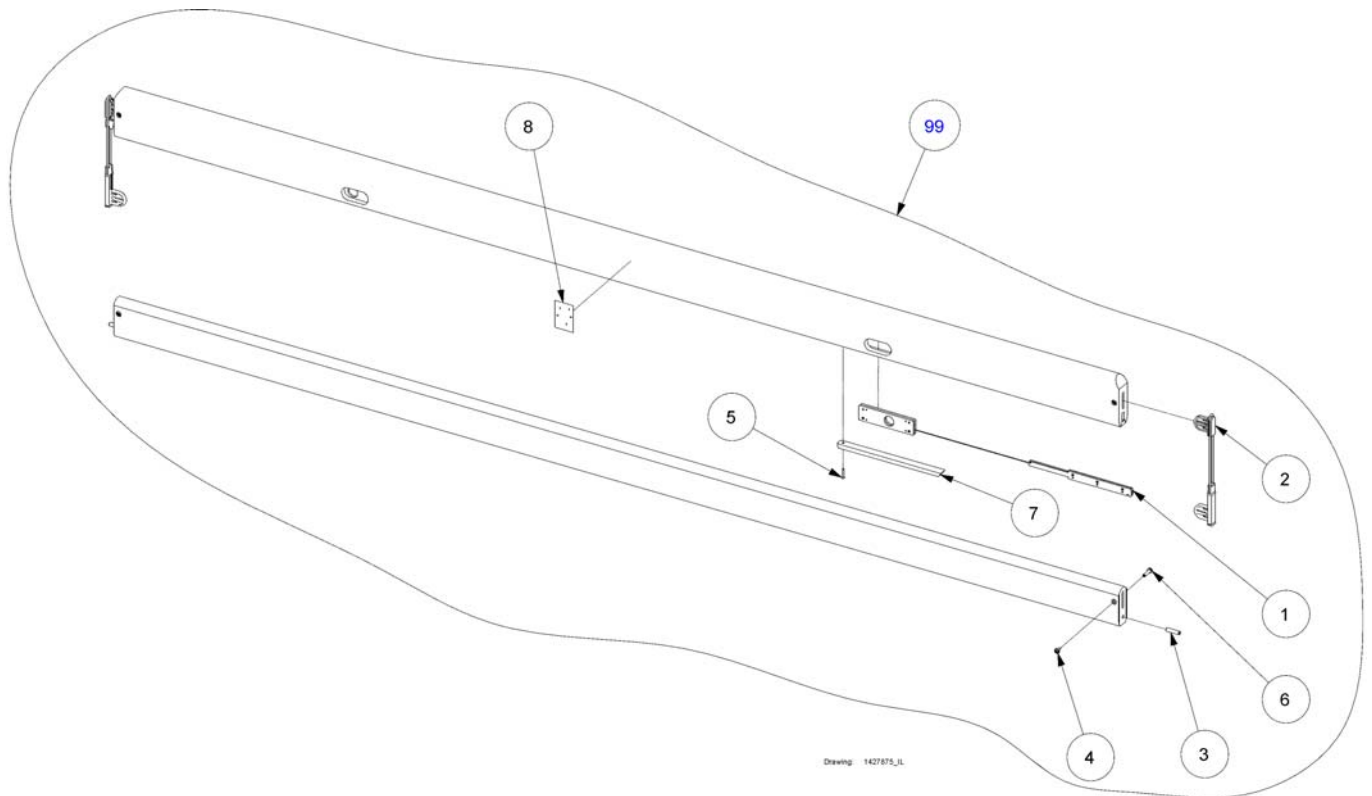


1	020024	Utlösare utbytesdelar Britt/Lilly sats/2 grind	1
2	319137	Glider t Brittgrind se 020023	2
3	413766	Sprint grind Britt	2
4	817265	Bokskruv hona SB750	4
5	814374	Wood screw $\varnothing 3 \times 20$ 0x20 hf c 324 for beech list	2
6	812367	Bokskruv hane SB750	4
7	1517167-0102	Wood strip for wood siderails	2
7	1517167-0101	Wood strip for wood siderails	2
8	1435328	Label for wooden side rail release	1
99	020434.05	Sänggrind Britt III vitlaserad	1
99	020434.02	Sänggrind Britt III körsbär	1
99	020434.01	Sänggrind Britt III bok natur	1

(Britt II side rail, extended)

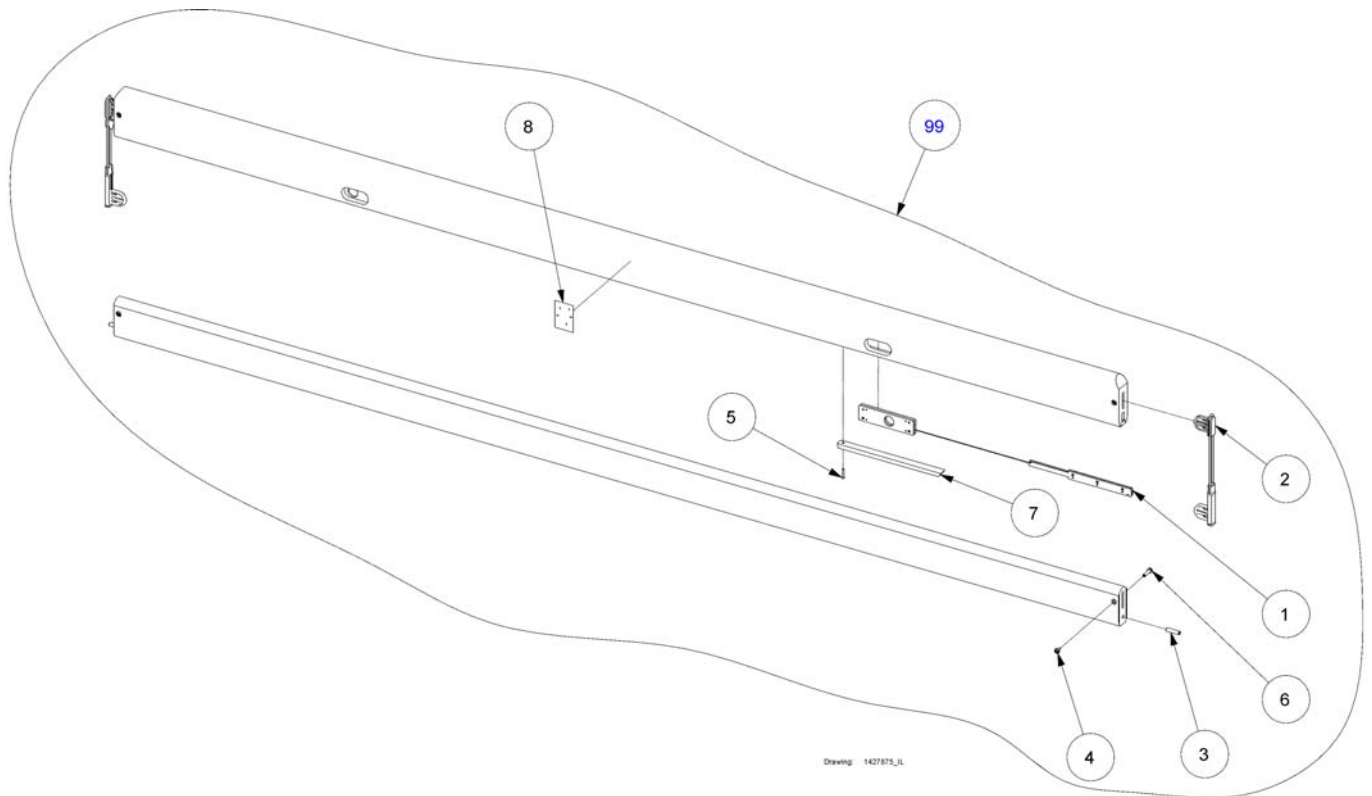


1	020024	Utlösare utbytesdelar Britt/Lilly sats/2 grind	1
2	319137	Glider t Brittgrind se 020023	2
3	413766	Sprint grind Britt	2
4	817265	Bokskruv hona SB750	4
5	814374	Wood screw $\varnothing 3 \times 20$ 0x20 hf c 324 for beech list	2
6	812367	Bokskruv hane SB750	4
7	1517167-0102	Wood strip for wood siderails	2
7	1517167-0101	Wood strip for wood siderails	2
8	1435328	Label for wooden side rail release	1
99	019372.05	Grind birtt III förl 15cm vitlaserad	1
99	019372.02	Sänggrind Britt III körsbär förlängd 15 cm	1
99	019372.01	Sänggrind Britt III bok natur förlängd 15 cm	1

(Britt IV side rail)

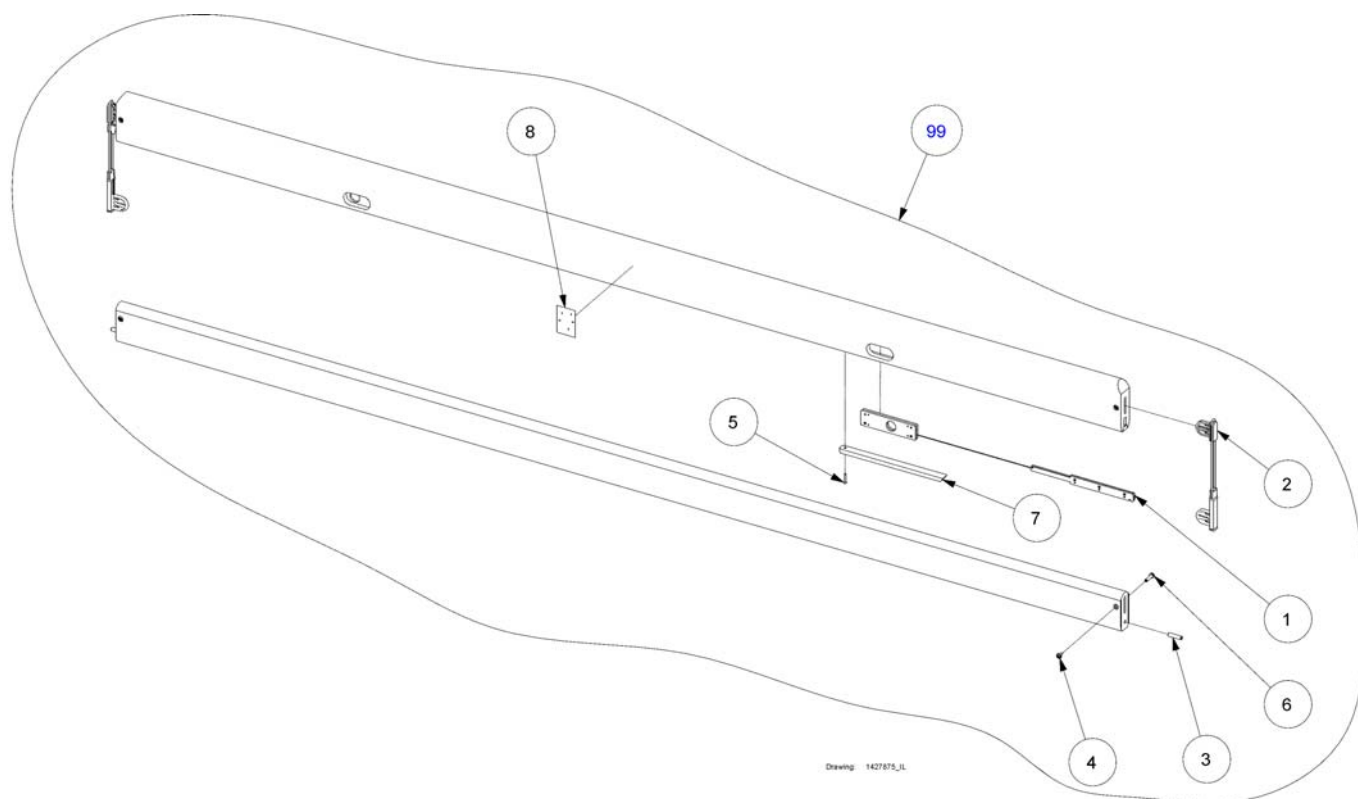


1	020024	Utlösare utbytesdelar Britt/Lilly sats/2 grind	1
2	319137	Glider t Brittgrind se 020023	2
3	413766	Sprint grind Britt	2
4	817265	Bokskruv hona SB750	4
5	814374	Wood screw $\varnothing 3 \times 20$ 0x20 hf c 324 for beech list	2
6	812367	Bokskruv hane SB750	4
7	1517167-0102	Wood strip for wood siderails	2
7	1517167-0101	Wood strip for wood siderails	2
8	1435328	Label for wooden side rail release	1
99	1427875-0105	Side rail Britt IV, 1 pair	1
99	1427875-0102	Side rail Britt IV, 1 pair	1
99	1427875-0101	Sänggrind Britt iv bok natur	1

(Lifting pole, Carebed, light)

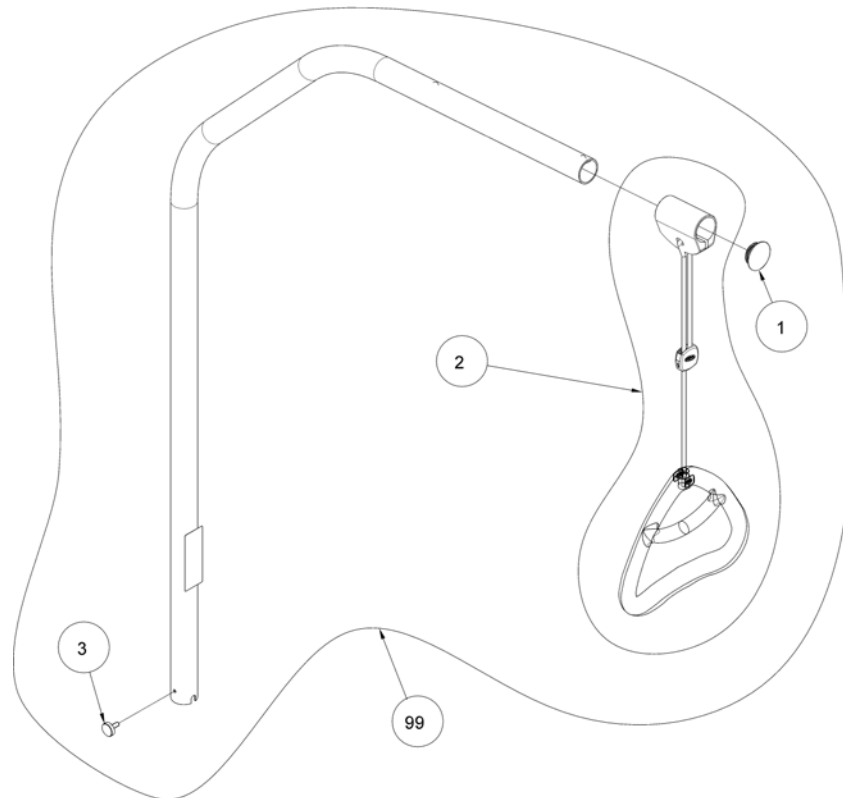


1	317836	Ändpropp	1
2	320597	Handtag kompl m snöre och fäste SB750/SB700	1
3	803979-9005	Fingerskruv ec t lyftbåge	1
99	1542093-1015	Lyftbåge beige SB750/400/440	1

(Support handles)

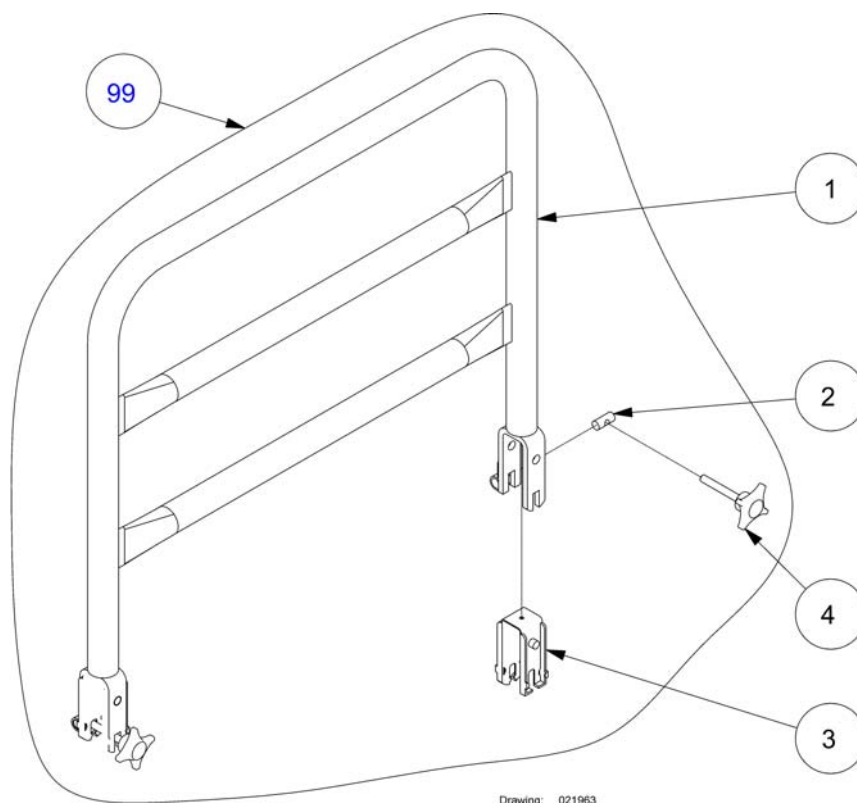

1	1505835	Not for separate sale - please contact Invacare	21
2	421468	Cylindermutter till s/v-handtag snabbblås	2
3	421843-9005	Plastinlägg U-fäste snabbblås s/v + Basic plus	2
3	421843-7035	Plastinlägg u-fäste snabbblås s/v + basic plus	2
4	821251-7035	Finger screw M6x45, RAL 7035	2
4	821251-9005	Fingerskruv M6x45 till Stöd/vändhandtag med snabbblås	2
99	1417511-0152	Support handle 40X30cm	1
99	1417510-7032	Support handle 25X80cm	1
99	021964.M0	Support handle 25X30cm	1
99	1417512-1015	Stöd/vändhandtag h 40 x b 95 snabbblås	1
99	1417512-0152	Support handle 40X95cm	1
99	1417512-7032	Support handle 40X95cm	1
99	1417510-1015	Stöd/vändhandtag h 25 x b 80 snabbblås	1
99	1417511-7032	Support handle 40X30cm	1
99	1417511-1015	Stöd/vändhandtag h 40 x b 30 snabbblås	1
99	1417510-0152	Support handle 25X80cm	1
99	021964.D0	Stöd/vändhandtag h 25 x b 30 snabbblås	1
99	021963.M0	Support handle 40X50cm	1
99	021963.D0	Stöd/vändhandtag h 40 x b 50 snabbblås	1


(Drip stand IV)

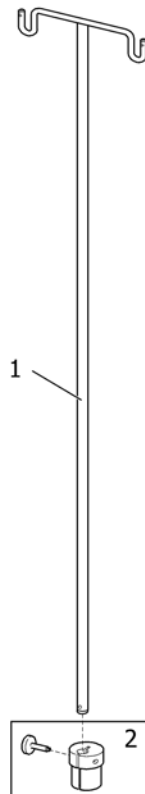

SF51496273

1	50.60030.B0	Droppställning Staccata vit	1
2	SP1570018	(Attachment cpl, dripstand)	1

(Support for hand control)

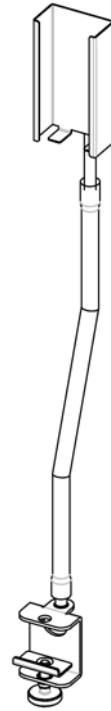


Drawing: 1451252_IL

1427870-1015	Hållare för manöverdosa hb70	1
1433835-1015	Hållare för manöverdosa soft control	1

(Sit-up handle)

1

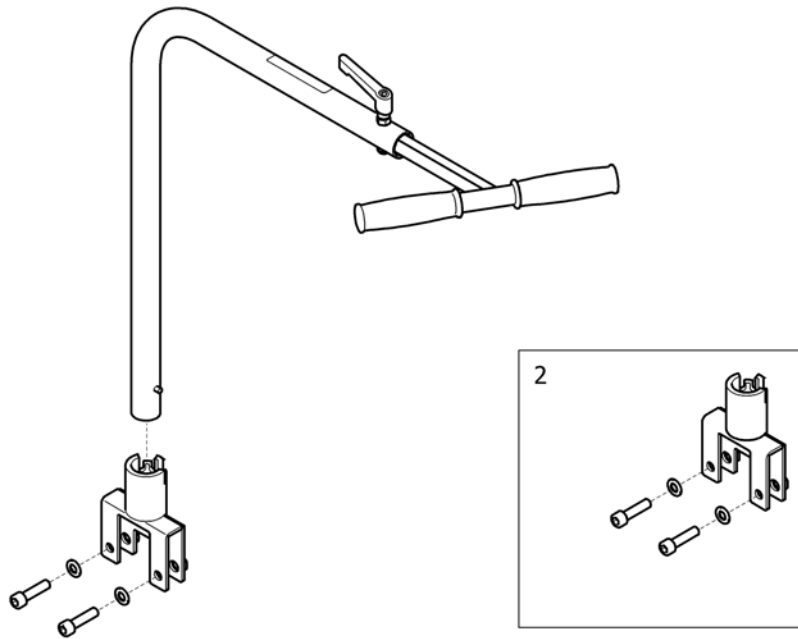

SPS1426899

1	1426899-1015	Sit-up stöd kpl SB750/ SB400/SB440/SB700	1
2	1555855-1015	(Bracket, Sit-up support cpl., SB750)	1



Yes, you can.®



Belgium & Luxemburg:

Invacare nv, Autobaan 22, B-8210 Loppem
Tel: (32) (0)50 83 10 10, Fax: (32) (0)50 83 10 11
belgium@invacare.com / www.invacare.be

Denmark:

Invacare A/S, Sdr. Ringvej 37, DK-2605 Brøndby
Tel: (45) (0)36 90 00 00, Fax: (45) (0)36 90 00 01
denmark@invacare.com / www.invacare.dk

Deutschland:

Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny
Tel: (49) (0)75 62 7 00 0, Fax: (49) (0)75 62 7 00 66
info@invacare-aquatec.com / www.invacare-aquatec.de

European Distributor Organisation:

Invacare, Kleiststraße 49, D-32457 Porta Westfalica
Tel: (49) (0)57 31 754 540, Fax: (49) (0)57 31 754 541
edo@invacare.com / www.invacare.eu.com

España:

Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona)
Tel: (34) (0)972 49 32 00, Fax: (34) (0)972 49 32 20
contactsp@invacare.com / www.invacare.es

France:

Invacare Poirier SAS, Route de St Roch, F-37230 Fondettes
Tel: (33) (0)2 47 62 64 66, Fax: (33) (0)2 47 42 12 24
contactfr@invacare.com / www.invacare.fr

Ireland:

Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland
Tel: (353) 1 810 7084, Fax: (353) 1 810 7085
ireland@invacare.com / www.invacare.ie

Italia:

Invacare Mecc San s.r.l., Via dei Pini 62, I-36016 Thiene (VI)
Tel: (39) 0445 38 00 59, Fax: (39) 0445 38 00 34
italia@invacare.com / www.invacare.it

Nederland:

Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede
Tel: (31) (0)318 695 757, Fax: (31) (0)318 695 758
nederland@invacare.com, csede@invacare.com / www.invacare.nl

Norge:

Invacare AS, Grensesvingen 9, Postboks 6230, Etterstad, N-0603 Oslo
Tel: (47) (0)22 57 95 00, Fax: (47) (0)22 57 95 01
norway@invacare.com, island@invacare.com / www.invacare.no

Österreich:

Invacare Austria GmbH, Herzog Odilostrasse 101, A-5310 Mondsee
Tel: (43) 6232 5535 0, Fax: (43) 6232 5535 4
info-austria@invacare.com / www.invacare.at

Portugal:

Invacare Lda, Rua Estrada Velha, 949, P-4465-784 Leça do Balio
Tel: (351) (0)225 1059 46/47, Fax: (351) (0)225 1057 39
portugal@invacare.com / www.invacare.pt

Sverige & Suomi:

Invacare AB, Fagerstagatan 9, S-163 91 Spånga
Tel: (46) (0)8 761 70 90, Fax: (46) (0)8 761 81 08
sweden@invacare.com, finland@invacare.com / www.invacare.se

Switzerland:

Invacare AG, Benkenstrasse 260, CH-4108 Witterswil
Tel: (41) (0)61 487 70 80, Fax: (41) (0)61 487 70 81
switzerland@invacare.com / www.invacare.ch

United Kingdom:

Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5AQ
Switchboard Tel: (44) (0)1656 776200, Fax: (44) (0)1656 776201
Customer services Tel: (44) (0)1656 776222, Fax: (44) (0)1656 776220
uk@invacare.com / www.invacare.co.uk