USER MANUAL

VICAIR®

LIBERTY BACK

relief by air

| | | |
|-----------|-------------------------|----|
| EN | USER MANUAL | 3 |
| NL | GEbruikersHANDLEIDING | 8 |
| DE | GEBRAUCHSANWEISUNG | 14 |
| FR | MANUEL DE L'UTILISATEUR | 20 |
| DA | BRUGERVEJLEDNING | 26 |
| SV | BRUKSANVISNING | 31 |
| NO | BRUKERMANUAL | 36 |
| FI | KÄYTTÖOHJE | 41 |
| ES | MANUAL DEL USUARIO | 46 |
| PT | MANUAL DE UTILIZADOR | 52 |
| IT | MANUALE UTENTE | 58 |
| GR | ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ | 64 |
| HR | KORISNIČKI PRIRUČNIK | 70 |
| SL | UPORABNIŠKI PRIROČNIK | 76 |
| PL | INSTRUKCJA OBSŁUGI | 82 |
| ZH | 用户手册 | 88 |
| JP | ユーザーマニュアル | 92 |

EN **VICAIR® LIBERTY BACK
USER MANUAL**

Dear customer,


Thank you for choosing a Vicair® product. We are sure you will appreciate the simplicity, reliability and fantastic effect. For quick installation you can use the quick installation guide provided with your cushion. In addition, please read and understand this user manual before use. Save both the Quick Installation Guide and this manual for future reference.

For more information or to (re)order manuals, contact your country's distributor. Go to vicair.com for distributor information.

INTRODUCTION

The Vicair Liberty back is a wheelchair back support cushion for users who require a high level of comfort in combination with a low to medium level of support and pressure ulcer prevention. It is designed for use in combination with the Vicair Strap back.

MAIN COMPONENTS

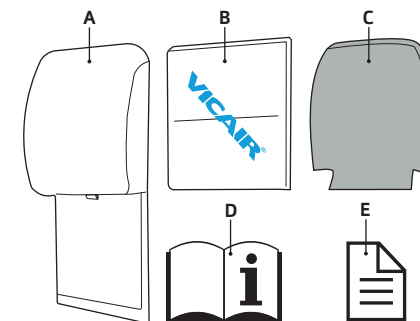- A** Cushion cover (Comfair Cover)
- B** Cushion
- C** Foam insert
- D** User manual
- E** Quick Installation Guide

GENERAL PRODUCT SPECIFICATIONS

| | |
|------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------|
| Product name: | Vicair Liberty back |
| Manufactured by: | Vicair B.V., The Netherlands |
| Warranty: | 24 months |
| Weight & sizes: | See information in the back of the manual |
| Available sizes: | See the grid in the back of the manual |
| Used materials: | All latex free |
| Cushion: | Polyvinylchloride |
| Filling: | Air-filled SmartCells™ and silicon oil |
| Foam insert: | Closed-cell polyethylene foam |
| Comfair Cover: | Front: knitted polyester fabric. Sides: knitted polyester 3D fabric. Back: nylon canvas fabric. Machine washable. Cover complies with EN1021-1, 1021-2. |

PRECAUTIONS AND SAFETY INSTRUCTIONS

A Vicair wheelchair back support cushion may only be used as a wheelchair back support cushion. The use of this product in any other way, is considered to be misuse. Vicair B.V. cannot be held responsible for damage caused by misuse, improper use or negligence. Make sure that you read and understand this manual completely.

Warning! Vicair B.V. recommends to install the product under direct supervision of a therapist or technical advisor.

Warning! Your back and spine should be supported by the air filled SmartCells within the cushion and should not touch the foam insert. Checking for “bottoming out” should be done during first use installation and after that on a regular basis (at least once a month).

Warning! Do not use as a shower or bath cushion.

Warning! Check your skin regularly for redness, especially during the first few days. If redness occurs and does not disappear after 15 minutes, consult your therapist or technical advisor.

Warning! No testing data is currently available about the safe use of Vicair back support cushions in motor vehicles. Therefore we do not encourage using the back support cushion in motor vehicles.

Warning! In case of soiling with body fluids, and when transferring the product to another user, the product must be disinfected to avoid cross contamination.

Warning! Every three months, check the product for wear and deflated cells. In case of serious damage, or a large number of deflated cells, do not continue to use the product. Consult your dealer.

Warning! Do not immerse this cushion in water or any other liquid and do not wash in a washing machine.

Warning! Using the Vicair Liberty back may affect the static and dynamic behaviour of the wheelchair. Check and test the balance and stability of the wheelchair with the aid of your therapist or technical advisor.

Warning! Only use the Vicair Liberty back in combination with the Vicair Strap back. Consult the Vicair Strap back user manual for information about installation, use, maintenance and cleaning of this product.

Caution! Do not use or leave the product close to a naked flame or a high temperature heat source.

Caution! Do not open the valve. Opening might cause silicon oil leakage.

Caution! Do not inflate the cushion.

Caution! Do not steam clean or autoclave the product. The product cannot withstand excessive heat or pressure. When the given cleaning instructions are not followed, it is very likely that the product and/or the product's cover will age more rapid.

Caution! Do not use aggressive or abrasive cleaners, or sharp objects to clean the cushion.

INSTALLATION

Warning! Vicair B.V. recommends to install the product under direct supervision of a therapist or technical advisor.

Warning! Using the Vicair Liberty back may affect the static and dynamic behaviour of the wheelchair. Check and test the balance and stability of the wheelchair with the aid of your therapist or technical advisor.

Warning! Only use the Vicair Liberty back in combination with the Vicair Strap back. Consult the Vicair Strap back user manual for information about installation, use, maintenance and cleaning of this product.

Caution! Do not open the valve. Opening might cause silicon oil leakage.

Caution! Do not inflate the cushion.

Notice! Always use cushion with cover.

Perform the following steps when installing the Vicair Liberty back

These steps correspond to the installation images 1 – 4 on the Quick Installation Guide:

- Check cushion size. The distance between the back posts of the wheelchair (X) determines the cushion size. X (min-max):
36-40 cm/14½-16" = LBC/1
40-44 cm/16-17½" = LBC/2
44-48 cm/17½-19" = LBC/3
- Check if the size of the cover matches the size of the cushion.
- Open the zipper on the topside of the cushion cover and check if cover contains the foam insert. The round side of the mushroom-shaped foam insert should be facing up.
- Insert cushion in cover. The print at the back side of the cushion indicates its back/front and upside. The side of the cushion cover with Velcro strips and a vanity flap attached to its outside is the back side of the cover. The foam insert should be between the back side of the cushion and the back side of the cushion cover.
- Pull the Velcro strip attached to the inside of the cushion cover over the top of the foam insert and attach it to the Velcro strip attached to the top-back side of the cushion.
- Close zipper. Make sure to not damage the cushion and/or foam insert.

If not yet installed: install your Vicair Strap back. Consult the Vicair Strap back user manual for the installation instructions of this item.

Install the Vicair Liberty back on your Vicair strap back.

These steps correspond to the installation images 5 – 9 on the Quick Installation Guide:

- Lift the vanity flap and place the cushion against the Vicair Strap back. The Velcro hooks on the cushion cover should hook onto the Velcro loops on the Vicair Strap back.

- Sit in the wheelchair for a few minutes, to allow the back support cushion to adapt to your body shape.
- Your therapist/technical advisor can now adjust the straps of the Vicair Strap back to the contours of your back.
- Fold the cushion's vanity flap over the Vicair Strap back.
- Fold the end of the vanity flap under your seat cushion. Optional: use the Velcro at the end of the vanity flap to attach it to the wheelchair seat.

Checking for “bottoming out”:

Image 10 - Quick Installation Guide

Warning! Your back and spine should be supported by the air filled SmartCells within the cushion and should not touch the foam insert. Checking for “bottoming out” should be done during first use installation and after that on a regular basis (at least once a month).

- Check for bottoming out, after sitting with your back against the back support cushion for approximately 5 minutes.
- Lean slightly forward, let your therapist or technical advisor slide a hand between your back and the cushion, then lean back again.
- Ask your therapist or technical advisor to check if your back is supported by at least 2,5 cm / 1" of SmartCells.
- Do not use the cushion if bottoming out occurs. In most cases, using a different cushion model or cushion size will solve the problem.

Stability

Warning! Using the Vicair Liberty back may affect the static and dynamic behaviour of the wheelchair. Check and test the balance and stability of the wheelchair with the aid of your therapist or technical advisor.

USE

Warning! Do not use as a shower or bath cushion.

Warning! Only use the Vicair Liberty back in combination with the Vicair Strap back. Consult the Vicair Strap back user manual for information about installation, use, maintenance and cleaning of this product.

Caution! Do not open the valve. Opening might cause silicon oil leakage.

Caution! Do not inflate the cushion.

Caution! Do not use or leave the product close to a naked flame or a high temperature heat source.

Checking for redness of the skin

Image 11 - Quick Installation Guide

Warning! Check your skin regularly for redness, especially during the first few days. If redness occurs and does not disappear after 15 minutes, consult your therapist or technical advisor.

Conditions for use and storage

- **User weight:**

Minimum: 15 kg / 33 lbs,
maximum: 150 kg / 330 lbs

- **Temperature:**

Minimum: -10 °C / 14 °F,
maximum: 50 °C / 122 °F

- **Use at high altitude**

Notice! Vicair B.V. recommends not to use the cushion above 2400 m (8000 ft) above sea level or below 50 m (164 ft) below sea level. Because each SmartCell is filled with air, the cushion may behave differently when used at high altitude or in an airplane.

- **Air humidity:**

Minimum: 0%, maximum 100%

Use of the Vicair back support cushions in motor vehicles

Warning! No testing data is currently available about the safe use of Vicair back support

cushions in motor vehicles. Therefore we do not encourage using the back support cushion in motor vehicles.

Re-use - Transferring the product to another user

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. The product has to be thoroughly cleaned and disinfected, before another person is allowed to use the it. See chapter 'Maintenance and cleaning' for instructions.

Replacing the cover

Under normal use, Vicair B.V. recommends to replace the cover every year. Replacement covers can be purchased separately.

Disposal

The disposal and recycling of used devices and packaging must comply with the applicable legal regulation. Ensure the product is cleaned prior to disposal to avoid any risk of contamination.

MAINTENANCE AND CLEANING

Warning! Every three months, check the product for wear and deflated cells. In case of serious damage, or a large number of deflated cells, do not continue to use the product. Consult your dealer.

Warning! Do not immerse this cushion in water or any other liquid and do not wash in a washing machine.

Warning! Only use the Vicair Liberty back in combination with the Vicair Strap back. Consult the Vicair Strap back user manual for information about installation, use, maintenance and cleaning of this product.

Caution! Do not open the valve. Opening might cause silicon oil leakage.

Caution! Do not inflate the cushion.

Caution! Do not steam clean or autoclave the product. The product cannot withstand excessive heat or pressure. When the given cleaning instructions are not followed, it is very likely that the product will age more rapid.

Caution! Do not use aggressive or abrasive cleaners, or sharp objects to clean the cushion.

Cleaning the Vicair Liberty back

- **Cleaning the cushion cover**

Remove the cushion and foam insert before cleaning. The cushion cover can be machine washed inside-out. Follow the washing instructions on the label attached to the inside of the cover. (60 °C= 140 °F)

- **Cleaning the cushion**

Wipe the cushion with a damp cloth using a mild detergent. Do not machine wash the cushion and do not immerse in water or any other liquid.

- **Cleaning the foam insert**

Clean the foam insert by using a disinfectant spray.

Disinfecting the Vicair Liberty back

Warning! In case of soiling with body fluids, and when transferring the product to another user, the product must be disinfected to avoid cross contamination. Clean the cushion, cover and foam insert as instructed above. Preferably use an antibacterial detergent. After cleaning spray the cushion and cover with a disinfectant.

WARRANTY

Please see www.vicair.com for complete information on warranty processes, terms and conditions.

Vicair B.V. offers a limited warranty against defects in workmanship and materials, provided the product has been used normally. Any damage arising from abnormal use, or caused by improper handling, cleaning, or negligence, is excluded from this warranty.

Warranty period

From original date of purchase: 24 months.

Should a defect in materials or workmanship occur within the above mentioned warranty period, Vicair B.V. will – at its own choice – repair or replace the product free of charge.

Remedies for breach of express warranties herein are limited to repair or replacement of the product.

In no event shall damages for breach of any warranty include any consequential damages or exceed the cost of non-conforming goods sold.

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